

# wombat

## living size 1+2

### Exploded views / Part lists

Rev. date (02.2020)

*On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.*



## Contents

Wombat living Size 1 and 2:

Size 1+2 - Lift on brace .....3

Gas pedal ..... 5

Power.....7

Power/tilt on brace .....9

Frames:

Pedal/Tilt on seat (120745) ..... 11

Power/Tilt on seat (121021)..... 13

Lift/Tilt on brace (121108)..... 16

Pedal/Tilt on brace (121412)..... 18

Lift on brace/Tilt on seat (121414)..... 20

Power/Tilt on brace (121568) ..... 22

Subassemblies


Back ..... 25

Seat ..... 27

Handbrace ..... 29

Footplate Size 1 and 2..... 31


Footplate Size 3 ..... 32

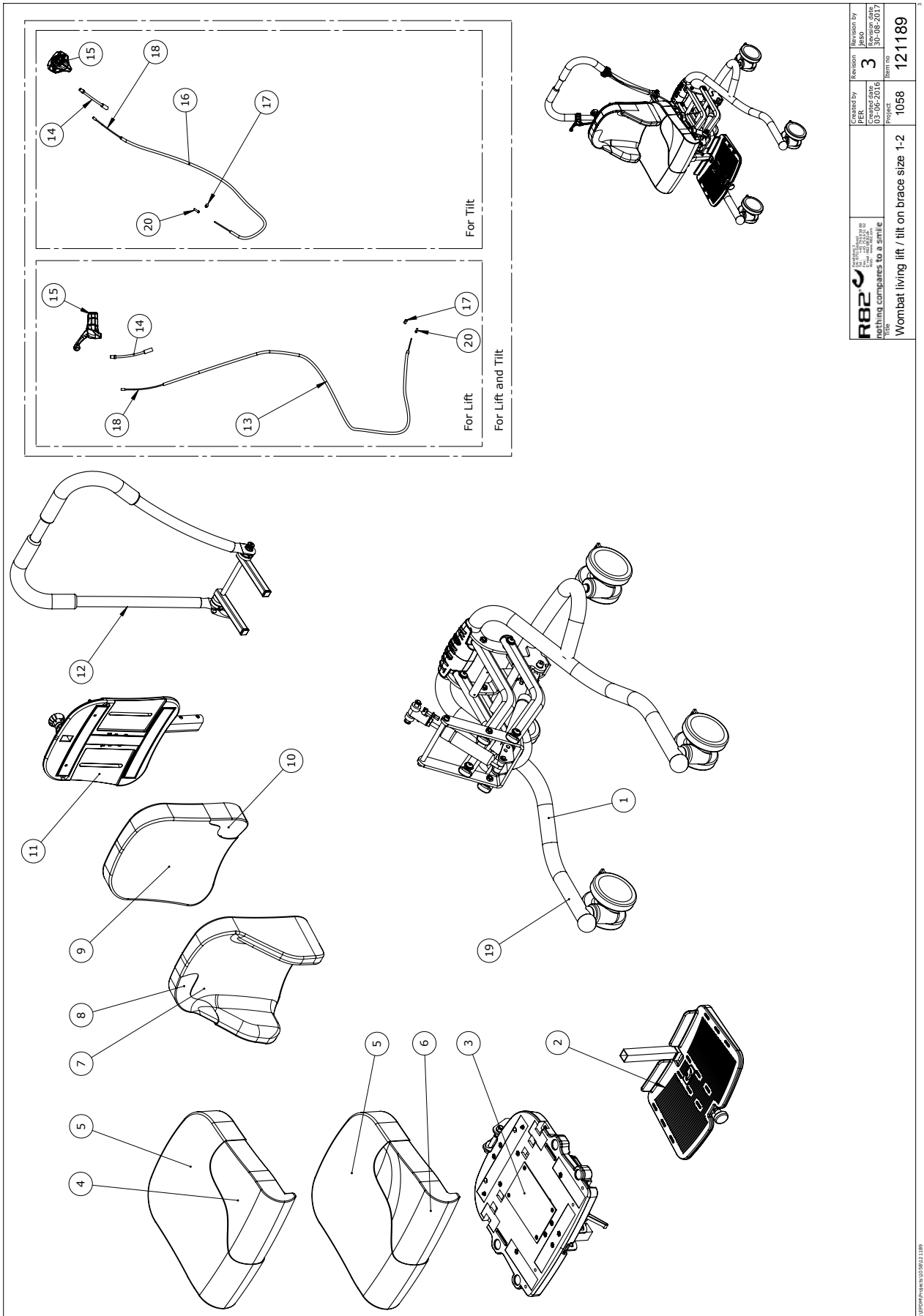


 Nothing compares to a smile Title:	Created by PER	Revision 3	Revision by Jaso
	Date: 03-06-2016	Item no. 1058	Date: 30-08-2017
Wombat living lift / tilt on brace size 1-2		Item no. 121189	

C:\EPM\Projects\1012013\1189


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	121108 See page 16	Frame Lift/Tilt on Brace			
1	121412 See page18	Frame Tilt on Brace			
1	121414 See page 20	Frame Lift on Brace			
2	121099-1	Footrest Wombat size 1	PCS	1	5707292622884
3	121093-1	Wombat seat size 1	PCS	1	5707292622938
4 to 10	See Pricelist	Cushions			
11	121092-1	Wombat back size 1	PCS	1	5707292622808
12	121058	Push handle complete str 1	PCS	1	5707292623331
13	5002750	Cable for brake, size 1+2	PCS	1	5707292413635
14	0380060	Flexible v-brake pipe.	PCS	1	5707292425195
14	0380060	Flexible v-brake pipe.	PCS	2	5707292425195
15	3503531	Gas pump lever with clamp	PCS	1	5707292413529
15	3503531	Gas pump lever with clamp	PCS	2	5707292413529
16	5002756	Cable for recline, outer 1-4	PCS	1	5707292413673
17	0388006	Screwnipple 5*7,5mm	PCS	1	5707292136114
17	0388006	Screwnipple 5*7,5mm	PCS	2	5707292136114
18	0371854-1	Soft inner cable for Stingray	PCS	1	5707292536945
18	0371854-1	Soft inner cable for Stingray	PCS	2	5707292536945
20	0387000	End nipple for outer casing	PCS	1	5707292136091
20	0387000	End nipple for outer casing	PCS	2	5707292136091

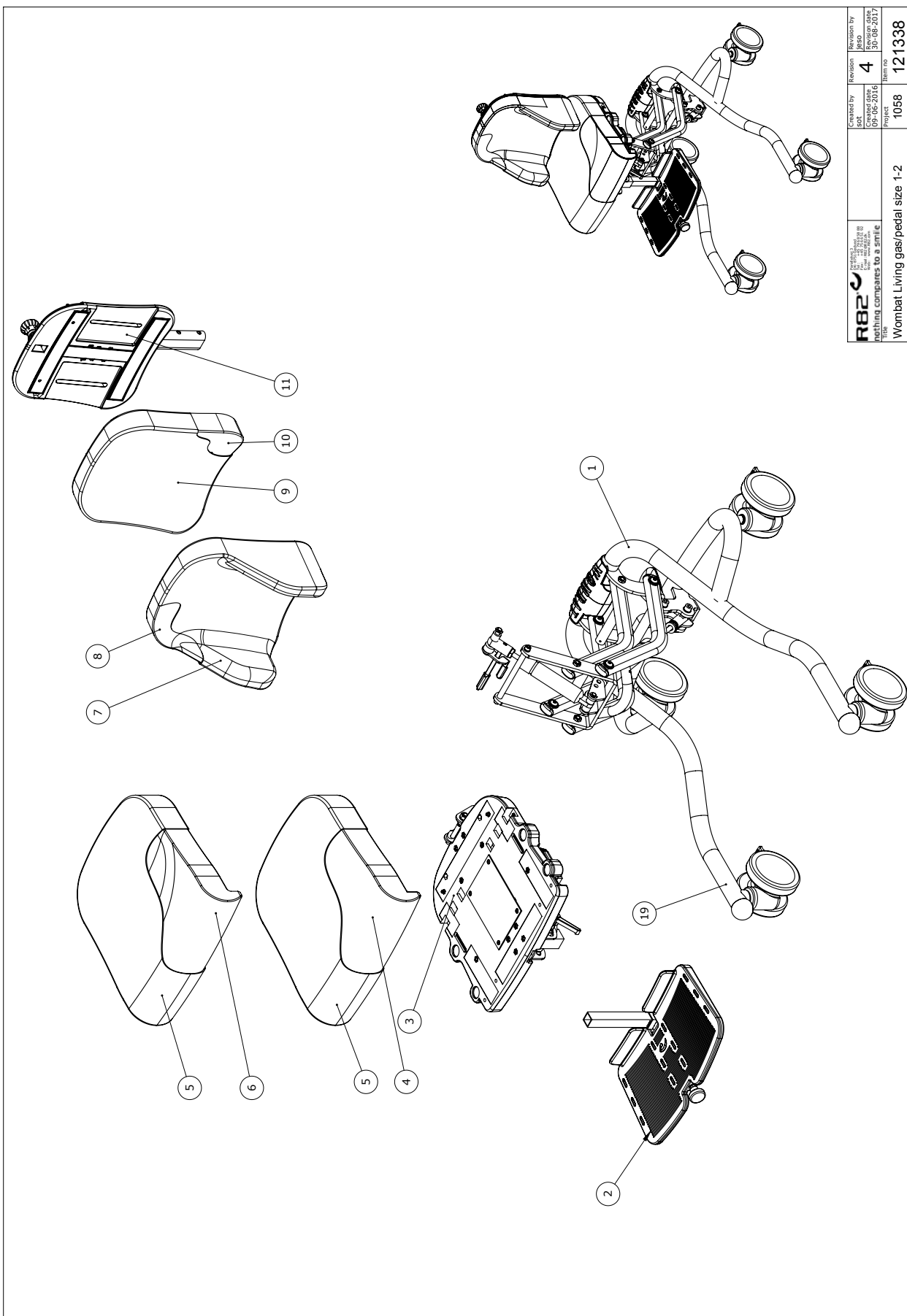

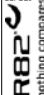 R82	Created by JYBA 09-06-2016	Revision by JYBA 30-08-2017
	Project Wombat Living gas/pedal size 1-2	Revision 4
Title Wombat Living gas/pedal size 1-2		Item no. 121338

C:\PDP\pdp\105831138

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	120745 (See page 11)	Frame w/pedal	STK	1	5707292622044
2	121099-1	Footrest Wombat size 1	STK	1	5707292622884
2	121099-2	Footrest Wombat size 2	STK	1	5707292623027
3	121093-1	Wombat seat size 1	STK	1	5707292622938
3	121093-2	Wombat seat size 2	STK	1	5707292622945
4 to 10	See Pricelist	Cushions			
11	121092-1	Wombat back size 1	STK	1	5707292622808
11	121092-2	Wombat back size 2	STK	1	
11	121092-21	Wombat back size 2	STK	1	5707292622822

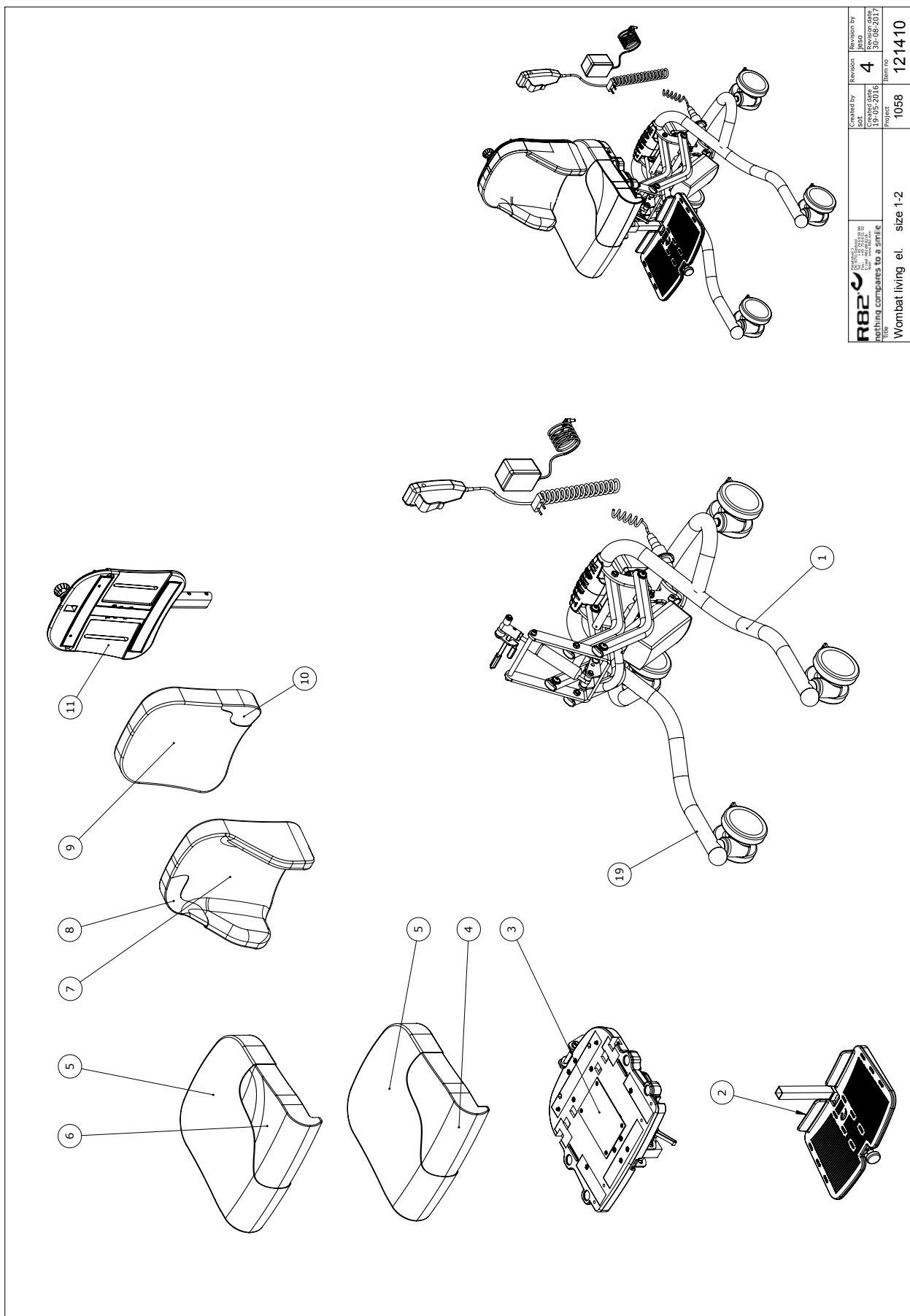



 nothing compares to a smile Title: Wombat living ei. size 1-2	Created by JYBA 19-05-2016	Revision 4	Revision by JYBA 30-08-2017
	Project 1068	Item no. 121410	

C:\pwworkspace\1058311410

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	121021 (See page 13)	Frame Wombat w/power	STK	1	5707292624321
2	121099-1	Footrest Wombat size 1	STK	1	5707292622884
2	121099-2	Footrest Wombat size 2	STK	1	5707292623027
3	121093-1	Wombat seat size 1	STK	1	5707292622938
3	121093-2	Wombat seat size 2	STK	1	5707292622945
4 to 10	See pricelist	Cushions			
11	121092-1	Wombat back size 1	STK	1	5707292622808
11	121092-2	Wombat back size 2	STK	1	
11	121092-21	Wombat back size 2	STK	1	5707292622822




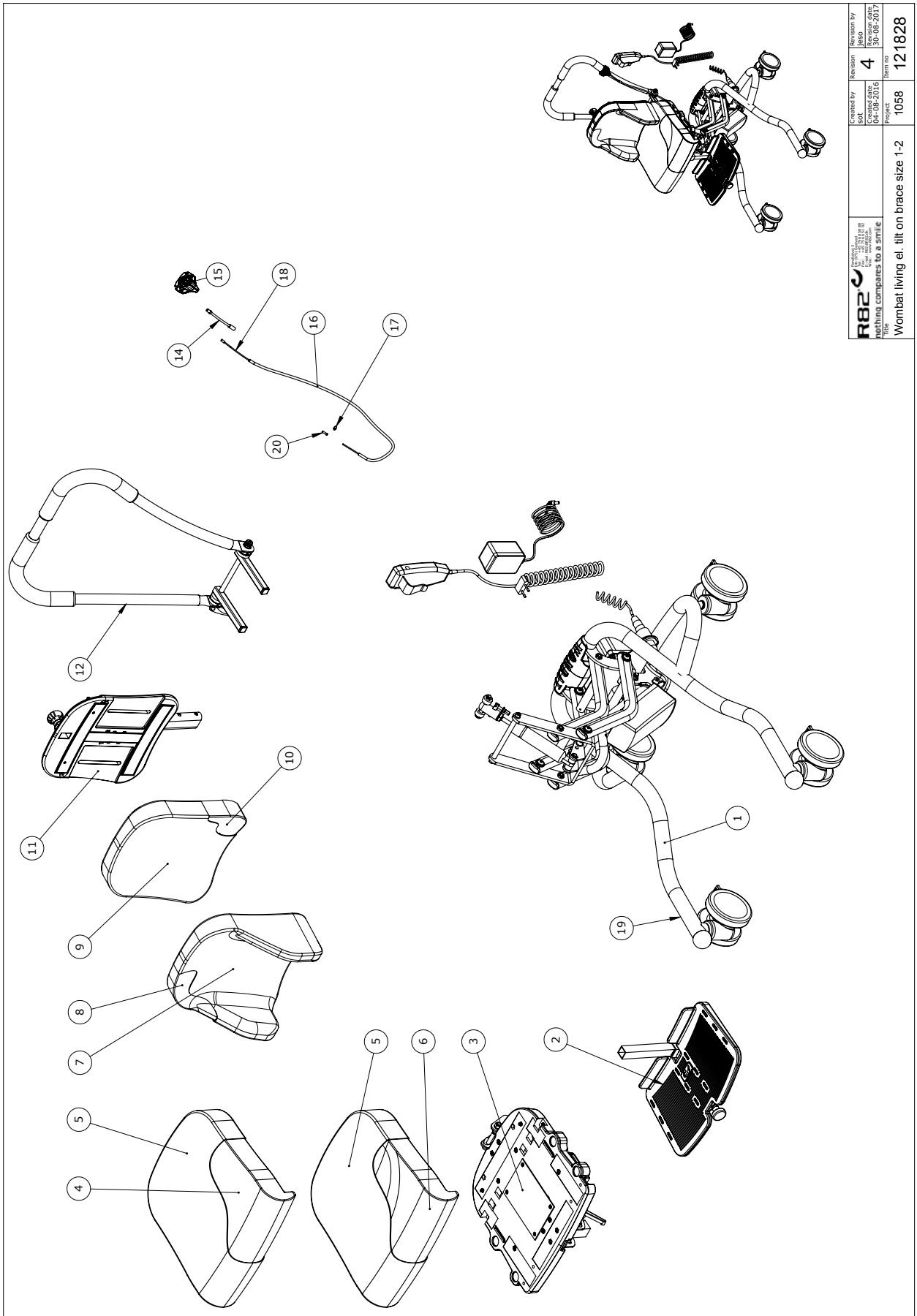

 R82	Created by JYBA 04-08-2016	Revision by JYBA 30-08-2017
	Project 1058	Item no. 121828
Title Wombat living ei. tilt on brace size 1-2		


C:\PDP\pdp\1058\312828


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	121568 (See page 23)	Frame Wombat w/power			
2	121099-2	Footrest Wombat size 2	PCS	1	5707292623027
2	121099-1	Footrest Wombat size 1	PCS	1	5707292622884
3	121093-2	Wombat seat size 2	PCS	1	5707292622945
3	121093-1	Wombat seat size 1	PCS	1	5707292622938
4 to 9	See Pricelist	Cushions			
11	121092-2	Wombat back size 2	PCS	1	
11	121092-1	Wombat back size 1	PCS	1	5707292622808
12	121058	Push handle complete str 1	PCS	1	5707292623331
14	0380060	Flexible v-brake pipe.	PCS	1	5707292425195
15	3503531	Gas pump lever with clamp	PCS	1	5707292413529
16	5002756	Cable for recline, outer 1-4	PCS	1	5707292413673
17	0388006	Screwnipple 5*7,5mm	PCS	1	5707292136114
18	0371854-1	Soft inner cable for Stingray	PCS	1	5707292536945
19	0494082	Etiket Wombat Living	PCS	1	5707292623294
20	0387000	End nipple for outer casing	PCS	1	5707292136091

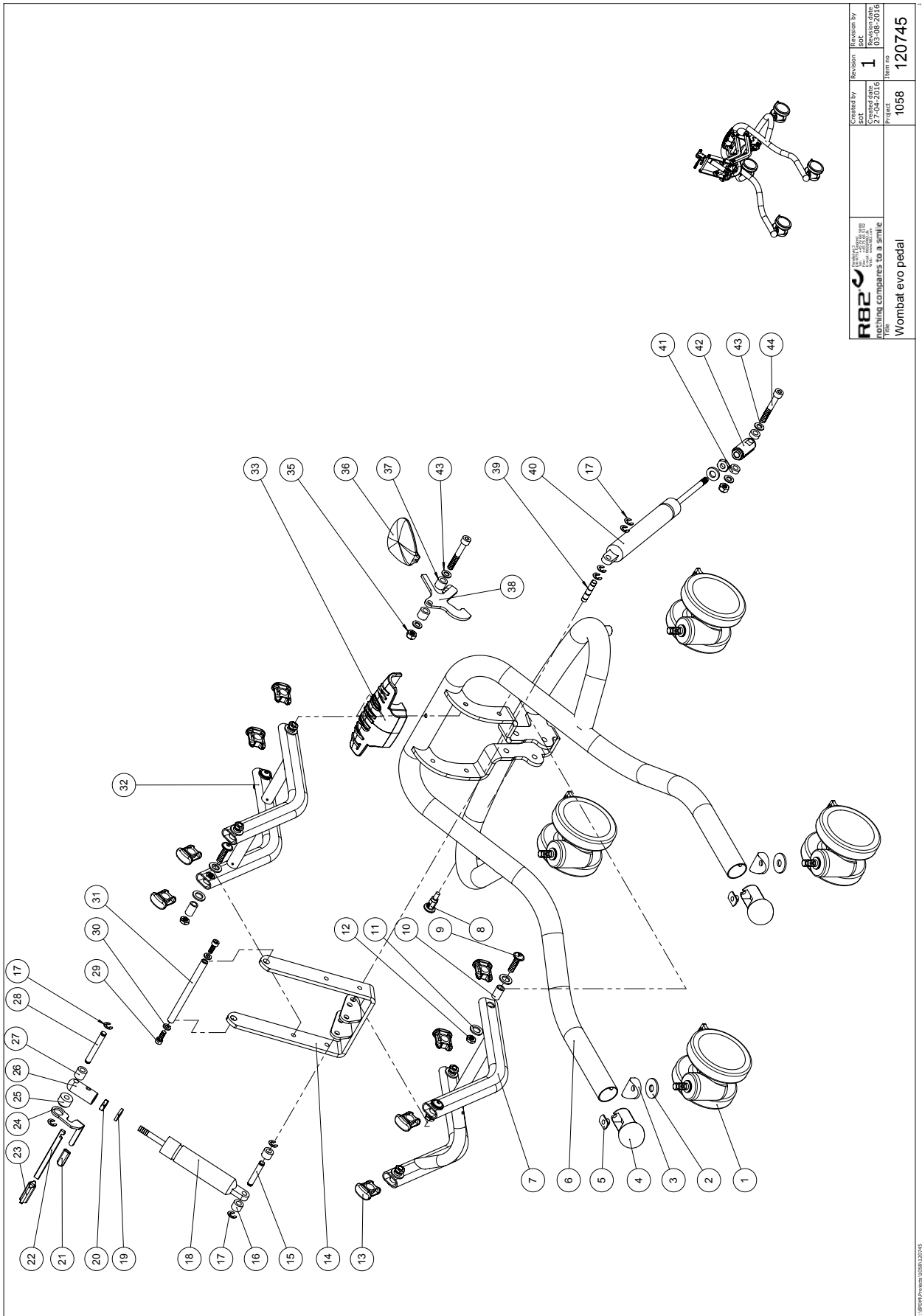

R82 A/S Parallelvej 3 8751 Gedved Denmark Tel: +45 79 68 58 88 Fax: +45 75 66 51 92 E-mail: R82@R82.dk Web: R82.org	Checked by set	Revision 1	Reviewed by set
	Create date 2016	Revision date 2016	Project no. 1058
Drawing companies to 3 part E			Item no. 120745
Wombat evo pedal			

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	3006825	75mm wheel w/brake, black	PCS	4	5707292371393
1	3006938	100mm double wheel with brake	PCS	4	5707292574930
2	1108050	Washer M10, din9021	PCS	2	5707292140401
3	82894	Saddle washer	PCS	2	5707292387578
4+5	5082829-60	Mount of ball and nut	PCS	2	5707292621962
6	82920-10	Frame wombat size 1+2	PCS	1	5707292622501
6	82920-35	Frame size 1+2 antrazit grey	PCS	1	5707292622525
7	82790-10	Carryarm white ral 9010	PCS	1	5707292622471
7	82790-35	Carryarm Antrazit grey	PCS	1	5707292622495
8	0380183	Spec. Lockhouse.M10*1, 32mm	PCS	1	5707292136046
9	0663031	Button head 8*30, Wombat	PCS	8	5707292139610
10	82933	Bearing bushing, Wombat	PCS	8	5707292125880
11	1310710	Nylon washer	PCS	16	5707292140807
12	1750800	Lock nut, stainless M8	PCS	8	5707292141484
13	82946	End piece 15*30, Wombat	PCS	8	5707292126016
14	82924-10	tilt brace	PCS	1	5707292621986
14	82924-35	tilt brace	PCS	1	5707292622020
15	80921	locking pin, Wombat	PCS	1	5707292111371
16	1350021	Distance piece 11*15*8,2mm	PCS	2	5707292140890
17	1145070	Lock washer 7mm	PCS	8	5707292140586
18	2203506	Bloc-o-lift gasspring 2745hu	PCS	1	5707292143297
19	1310712	Nylon washer, nature	PCS	2	5707292356345
20	1761000	Counter nut 10*1,0 fine thread	PCS	2	5707292141538
21	3501800	Black pvc cap 5,5*30mm	PCS	1	5707292146762
22	2200020	Release arm	PCS	1	5707292143037
23	2200030	Button for release arm	PCS	1	5707292143051
24	82913	Lever for Wombat	PCS	1	5707292125705
25	82930	Bush for striking plate	PCS	1	5707292125859
26	6118821	Horizontal release, Wombat	PCS	1	5707292186737
27	1350024	Distance piece 14*15*8,2mm	PCS	1	5707292140920
28	80922	Locking pin, Wombat	PCS	1	5707292111388
29	0602200	Insex screw 6*16	PCS	2	5707292138095
30	1103000	Fazet washer 6mm	PCS	2	5707292140289
31	82806	Axle	PCS	1	5707292125200
32	82921-10	Carrie arm, Wombat white	PCS	1	5707292621993
32	82921-35	Carryarm	PCS	1	5707292622037
33	82949-60	Bumper for wombat, black	PCS	1	5707292619815
35	1607000	Lock nut 8mm, large pack	PCS	2	5707292141378
36	3470335	Release button 4*8mm	PCS	1	5707292146632
37	1350023	Distance piece, black	PCS	2	5707292140913
38	82914	Pedal, Wombat	PCS	1	5707292125712
39	80924	Ø8*40 spec locking pin,wombat	PCS	1	5707292111401
40	2205300	Bloc-o-lift gasspring 5251hs	PCS	1	5707292143389
41	1350000	Distancepart black 6*15*8,2 mm	PCS	2	5707292140876
42	6118822	Horizontal release	PCS	1	5707292186744
43	1103200	Fazet washer 8mm	PCS	4	5707292140296
44	0603600	Insex screw 8*55	PCS	2	5707292138262

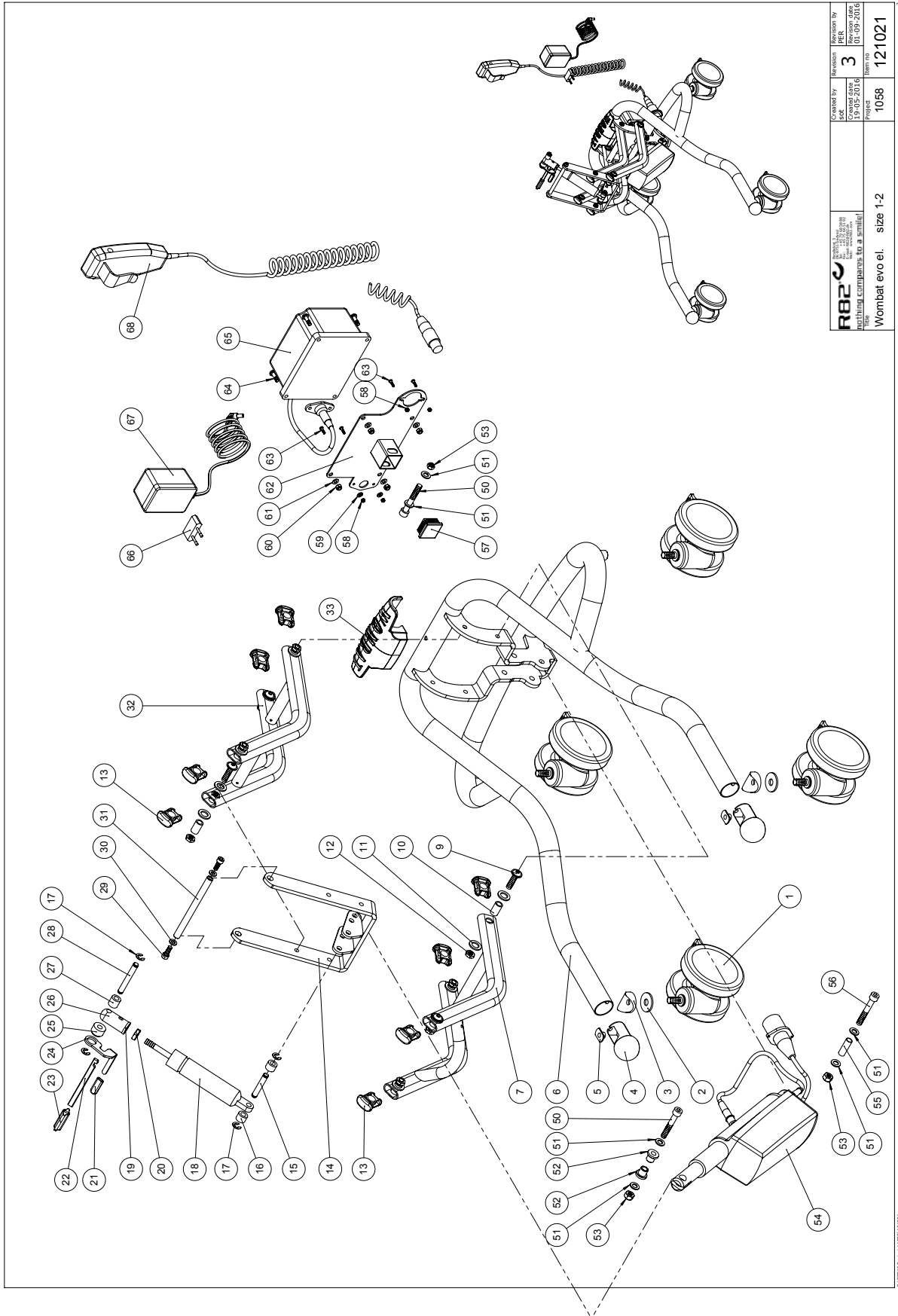


 R82 A/S Industrivej 11 8260 Viby Denmark Net: +45 79 68 58 88 Fax: +45 75 66 51 92 E-mail: R82@R82.dk Web: R82.org	Created by SJK 15.08.2016	Revision PER 3 01.09.2016 12.10.2016
	Project 1058	Item no. 121021

Wombat evo el. size 1-2


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	3006825	75mm wheel w/brake, black	PCS	4	5707292371393
1	3006938	100mm double wheel with brake	PCS	2	5707292574930
2	1108050	Washer M10, din9021	PCS	2	5707292140401
3	82894	Saddle washer	PCS	2	5707292387578
4+5	5082829-60	Mount of ball and nut	PCS	2	5707292621962
6	82920-10	Frame wombat size 1+2	PCS	1	5707292622501
6	82920-35	Frame size 1+2 antrazit grey	PCS	1	5707292622525
7	82790-10	Carryarm white ral 9010	PCS	1	5707292622471
7	82790-35	Carryarm Antrazit grey	PCS	1	5707292622495
9	0663031	Button head 8*30, Wombat	PCS	8	5707292139610
10	82933	Bearing bushing, Wombat	PCS	8	5707292125880
11	1310710	Nylon washer	PCS	16	5707292140807
12	1750800	Lock nut, stainless M8	PCS	8	5707292141484
13	82946	End piece 15*30, Wombat	PCS	8	5707292126016
14	82924-10	tilt brace	PCS	1	5707292621986
14	82924-35	tilt brace	PCS	1	5707292622020
15	80921	locking pin, Wombat	PCS	1	5707292111371
16	1350021	Distance piece 11*15*8,2mm	PCS	2	5707292140890
17	1145070	Lock washer 7mm	PCS	8	5707292140586
18	2203506	Bloc-o-lift gasspring 2745hu	PCS	1	5707292143297
19	1310712	Nylon washer, nature	PCS	2	5707292356345
20	1761000	Counter nut 10*1,0 fine thread	PCS	2	5707292141538
21	3501800	Black pvc cap 5,5*30mm	PCS	1	5707292146762
22	2200020	Release arm	PCS	1	5707292143037
23	2200030	Button for release arm	PCS	1	5707292143051
24	82913	Lever for Wombat	PCS	1	5707292125705
25	82930	Bush for striking plate	PCS	1	5707292125859
26	6118821	Horizontal release, Wombat	PCS	1	5707292186737
27	1350024	Distance piece 14*15*8,2mm	PCS	1	5707292140920
28	80922	Locking pin, Wombat	PCS	1	5707292111388
29	0602200	Insex screw 6*16	PCS	2	5707292138095
30	1103000	Fazet washer 6mm	PCS	2	5707292140289
31	82806	Axle	PCS	1	5707292125200
32	82921-10	Carrie arm, Wombat white	PCS	1	5707292621993
32	82921-35	Carryarm	PCS	1	5707292622037
34	82949-60	Bumper for wombat, black	PCS	1	5707292619815
50	0603550	Insex screw 8*50	PCS	2	5707292138255
51	1103200	Fazet washer 8mm	PCS	6	5707292140296
52	82848	Bush	PCS	2	5707292125484
53	1607000	Lock nut 8mm, large pack	PCS	3	5707292141378
54	2222133	Actuator la28 f/wombat	PCS	1	5707292492272
55	6108200	Pvc pipe pa11 Ø10/Ø8 mm	MTR	0,03	5707292185655
56	0603600	Insex screw 8*55	PCS	1	5707292138262
57	2505000	Ferrule e30*301,0-2,0 int.blac	PCS	1	5707292144140
58	1606250	Self-locking nut 3mm	PCS	4	5707292408396
59	1102800	Fazet washer 4mm	PCS	2	5707292140265
60	1606300	Lock nut 5 mm	PCS	4	5707292141354
61	1102900	Fazet washer 5 mm	PCS	4	5707292140272
62	77576	Battery console highlow	PCS	1	5707292490759
63	1902590	Crosshead screw M3*10, FZB	PCS	4	5707292408389

64	1902760	Cross-headed screw 5*16 mm	PCS	4	5707292142146
65	2222110	Batteryboks ba18027	PCS	1	5707292492258
66+67	2222161	Charger with plugs	PCS	1	5707292574046
68	2220600	Handset w/r82 print & charging	PCS	1	5707292492241

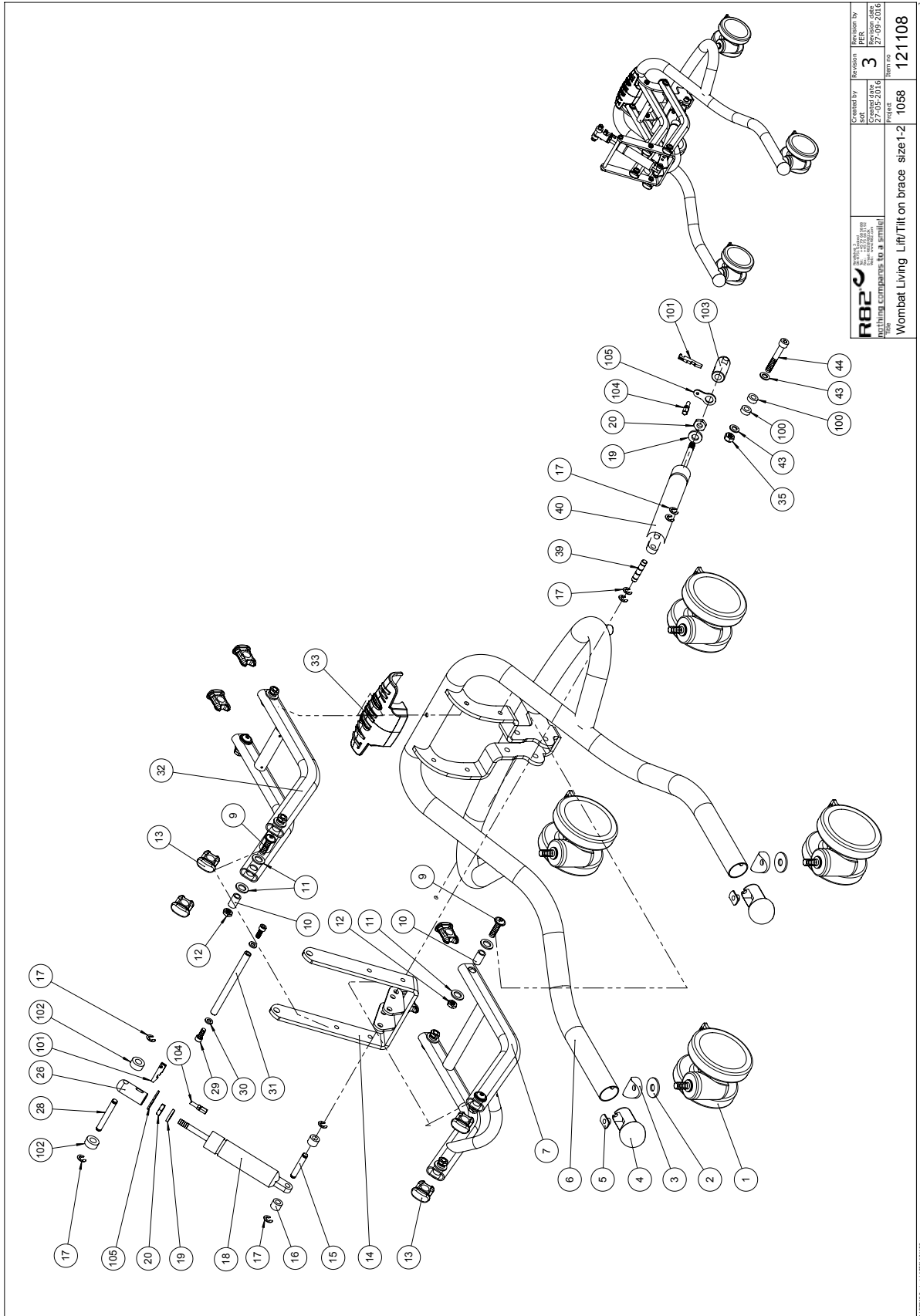




Position	No.	Description	Quantity	Unit of Measure	Ean code
1	3006825	75mm wheel w/brake, black	PCS	4	5707292371393
1	3006938	100mm double wheel with brake	PCS	4	5707292574930
2	1108050	Washer M10, din9021	PCS	2	5707292140401
3	82894	Saddle washer	PCS	2	5707292387578
4+5	5082829-60	Mount of ball and nut	PCS	2	5707292621962
4+5	5082829-60	Mount of ball and nut	PCS	2	5707292621962
6	82920-10	Frame wombat size 1+2	PCS	1	5707292622501
7	82790-10	Carryarm white ral 9010	PCS	1	5707292622471
9	0663031	Button head 8*30, Wombat	PCS	8	5707292139610
10	82933	Bearing bushing, Wombat	PCS	8	5707292125880
11	1310710	Nylon washer	PCS	16	5707292140807
12	1750800	Lock nut, stainless M8	PCS	8	5707292141484
13	82946	End piece 15*30, Wombat	PCS	8	5707292126016
14	82924-10	tilt brace	PCS	1	5707292621986
15	80921	locking pin, Wombat	PCS	1	5707292111371
16	1350021	Distance piece 11*15*8,2mm	PCS	2	5707292140890
17	1145070	Lock washer 7mm	PCS	8	5707292140586
18	2203506	Bloc-o-lift gasspring 2745hu	PCS	1	5707292143297
19	1310712	Nylon washer, nature	PCS	2	5707292356345
20	1761000	Counter nut 10*1,0 fine thread	PCS	2	5707292141538
26	6118821	Horizontal release, Wombat	PCS	1	5707292186737
28	80922	Locking pin, Wombat	PCS	1	5707292111388
29	0602200	Insex screw 6*16	PCS	2	5707292138095
30	1103000	Fazet washer 6mm	PCS	2	5707292140289
31	82806	Axle	PCS	1	5707292125200
32	82921-10	Carrie arm, Wombat white	PCS	1	5707292621993
33	82949-60	Bumper for wombat, black	PCS	1	5707292619815
35	1607000	Lock nut 8mm, large pack	PCS	2	5707292141378
39	80924	Ø8*40 spec locking pin,wombat	PCS	1	5707292111401
40	2205300	Bloc-o-lift gasspring 5251hs	PCS	1	5707292143389
43	1103200	Fazet washer 8mm	PCS	2	5707292140296
44	0603600	Insex screw 8*55	PCS	1	5707292138262
100	1350000	Distancepart black 6*15*8,2 mm	PCS	2	5707292140876
101	2200050	Small release arm	PCS	2	5707292143075
101	80922	Locking pin, Wombat	PCS	2	5707292111388
102	1350024	Distance piece 14*15*8,2mm	PCS	2	5707292140920
103	121699	Release head	PCS	1	5707292629746
104	0380000	Cable tighter M5*23	PCS	2	5707292135933
105	82268	Wire mounting by pedal	PCS	2	5707292123039
53	1607000	Lock nut 8mm, large pack	PCS	3	5707292141378
54	2222133	Actuator Ia28 f/wombat	PCS	1	5707292492272
55	6108200	Pvc pipe pa11 Ø10/Ø8 mm	MTR	0,03	5707292185655
56	0603600	Insex screw 8*55	PCS	1	5707292138262
57	2505000	Ferrule e30*301,0-2,0 int.blac	PCS	1	5707292144140
58	1606250	Self-locking nut 3mm	PCS	4	5707292408396
59	1102800	Fazet washer 4mm	PCS	2	5707292140265
60	1606300	Lock nut 5 mm	PCS	4	5707292141354
61	1102900	Fazet washer 5 mm	PCS	4	5707292140272
62	77576	Battery console highlow	PCS	1	5707292490759

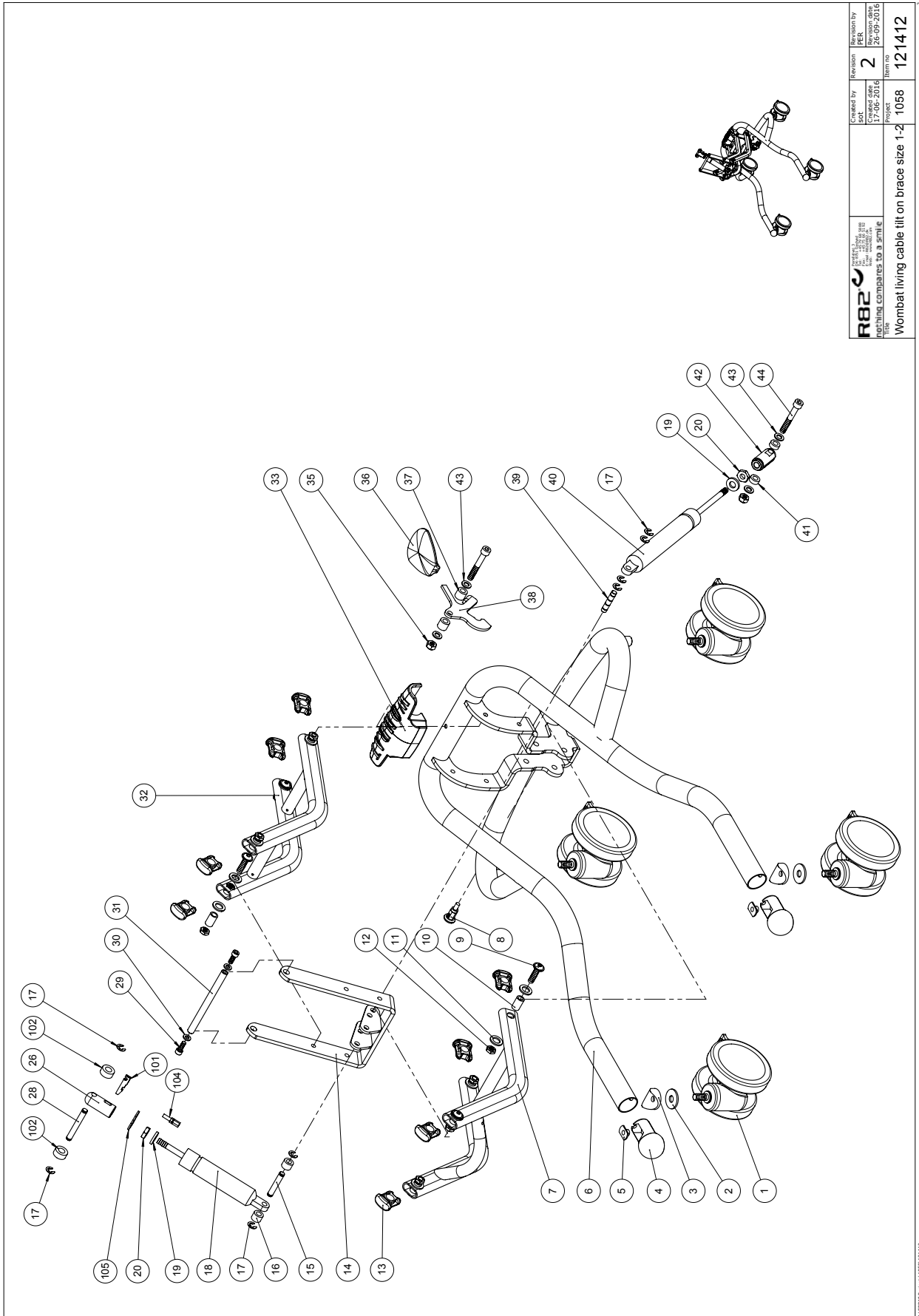


	Created by Project 17-06-2016 1058	Revision 2 26-09-2016	Revision by Revision date 26-09-2016
	Title Wombat living cable tilt on brace size 1-2 121412		


C:\Users\peter\Documents\121412


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	3006825	75mm wheel w/brake, black	PCS	4	5707292371393
1	3006938	100mm double wheel with brake	PCS	4	5707292574930
2	1108050	Washer M10, din9021	PCS	2	5707292140401
3	82894	Saddle washer	PCS	2	5707292387578
4+5	5082829-60	Mount of ball and nut	PCS	2	5707292621962
6	82920-10	Frame wombat size 1+2	PCS	1	5707292622501
7	82790-10	Carryarm white ral 9010	PCS	1	5707292622471
8	0380183	Spec. Lockhouse.M10*1, 32mm	PCS	1	5707292136046
9	0663031	Button head 8*30, Wombat	PCS	8	5707292139610
10	82933	Bearing bushing, Wombat	PCS	8	5707292125880
11	1310710	Nylon washer	PCS	16	5707292140807
12	1750800	Lock nut, stainless M8	PCS	8	5707292141484
13	82946	End piece 15*30, Wombat	PCS	8	5707292126016
14	82924-10	tilt brace	PCS	1	5707292621986
15	80921	locking pin, Wombat	PCS	1	5707292111371
16	1350021	Distance piece 11*15*8,2mm	PCS	2	5707292140890
17	1145070	Lock washer 7mm	PCS	8	5707292140586
18	2203506	Bloc-o-lift gasspring 2745hu	PCS	1	5707292143297
19	1310712	Nylon washer, nature	PCS	2	5707292356345
20	1761000	Counter nut 10*1,0 fine thread	PCS	2	5707292141538
26	6118821	Horizontal release, Wombat	PCS	1	5707292186737
28	80922	Locking pin, Wombat	PCS	1	5707292111388
29	0602200	Insex screw 6*16	PCS	2	5707292138095
30	1103000	Fazet washer 6mm	PCS	2	5707292140289
31	82806	Axle	PCS	1	5707292125200
32	82921-10	Carrie arm, Wombat white	PCS	1	5707292621993
33	82949-60	Bumper for wombat, black	PCS	1	5707292619815
35	1607000	Lock nut 8mm, large pack	PCS	2	5707292141378
36	3470335	Release button 4*8mm	PCS	1	5707292146632
37	1350023	Distance piece, black	PCS	2	5707292140913
38	82914	Pedal, Wombat	PCS	1	5707292125712
39	80924	Ø8*40 spec locking pin,wombat	PCS	1	5707292111401
40	2205300	Bloc-o-lift gasspring 5251hs	PCS	1	5707292143389
41	1350000	Distancepart black 6*15*8,2 mm	PCS	2	5707292140876
42	6118822	Horizontal release	PCS	1	5707292186744
43	1103200	Fazet washer 8mm	PCS	4	5707292140296
44	0603600	Insex screw 8*55	PCS	2	5707292138262
101	2200050	Small release arm	PCS	1	5707292143075
102	1350024	Distance piece 14*15*8,2mm	PCS	2	5707292140920
104	0380000	Cable tighter M5*23	PCS	1	5707292135933
105	82268	Wire mounting by pedal	PCS	1	5707292123039

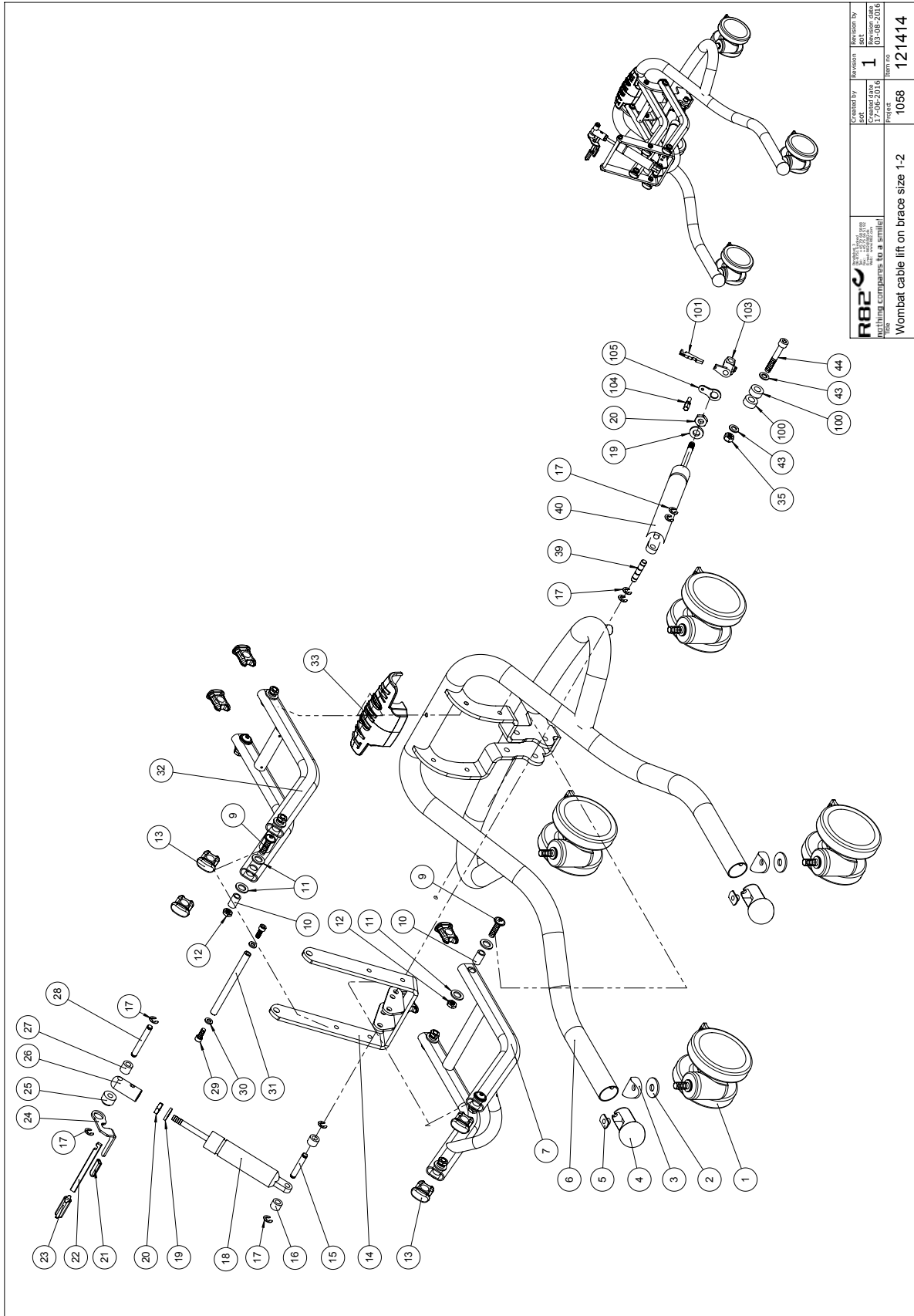

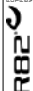 R82 A/S Parallelvej 3 8751 Gedved Denmark We're always ready to help you.	Created by S&K	Revision by S&K	Revision no. 1
	Project Wombat cable lift on brace size 1-2	Item no. 1058	Item no. 121414

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	3006825	75mm wheel w/brake, black	PCS	4	5707292371393
1	3006938	100mm double wheel with brake	PCS	4	5707292574930
2	1108050	Washer M10, din9021	PCS	2	5707292140401
3	82894	Saddle washer	PCS	2	5707292387578
4+5	5082829-60	Mount of ball and nut	PCS	2	5707292621962
6	82920-10	Frame wombat size 1+2	PCS	1	5707292622501
7	82790-10	Carryarm white ral 9010	PCS	1	5707292622471
9	0663031	Button head 8*30, Wombat	PCS	8	5707292139610
10	82933	Bearing bushing, Wombat	PCS	8	5707292125880
11	1310710	Nylon washer	PCS	16	5707292140807
12	1750800	Lock nut, stainless M8	PCS	8	5707292141484
13	82946	End piece 15*30, Wombat	PCS	8	5707292126016
14	82924-10	tilt brace	PCS	1	5707292621986
15	80921	locking pin, Wombat	PCS	1	5707292111371
16	1350021	Distance piece 11*15*8,2mm	PCS	2	5707292140890
17	1145070	Lock washer 7mm	PCS	8	5707292140586
18	2203506	Bloc-o-lift gasspring 2745hu	PCS	1	5707292143297
19	1310712	Nylon washer, nature	PCS	2	5707292356345
20	1761000	Counter nut 10*1,0 fine thread	PCS	2	5707292141538
21	3501800	Black pvc cap 5,5*30mm	PCS	1	5707292146762
22	2200020	Release arm	PCS	1	5707292143037
23	2200030	Button for release arm	PCS	1	5707292143051
24	82913	Lever for Wombat	PCS	1	5707292125705
25	82930	Bush for striking plate	PCS	1	5707292125859
26	6118821	Horizontal release, Wombat	PCS	1	5707292186737
27	1350024	Distance piece 14*15*8,2mm	PCS	1	5707292140920
28	80922	Locking pin, Wombat	PCS	1	5707292111388
29	0602200	Insex screw 6*16	PCS	2	5707292138095
30	1103000	Fazet washer 6mm	PCS	2	5707292140289
31	82806	Axle	PCS	1	5707292125200
32	82921-10	Carrie arm, Wombat white	PCS	1	5707292621993
33	82949-60	Bumper for wombat, black	PCS	1	5707292619815
35	1607000	Lock nut 8mm, large pack	PCS	2	5707292141378
39	80924	Ø8*40 spec locking pin,wombat	PCS	1	5707292111401
40	2205300	Bloc-o-lift gasspring 5251hs	PCS	1	5707292143389
43	1103200	Fazet washer 8mm	PCS	2	5707292140296
44	0603600	Insex screw 8*55	PCS	1	5707292138262
100	1350000	Distancepart black 6*15*8,2 mm	PCS	2	5707292140876
101	2200050	Small release arm	PCS	1	5707292143075
103	121699	Release head	PCS	1	5707292629746
104	0380000	Cable tighter M5*23	PCS	1	5707292135933
105	82268	Wire mounting by pedal	PCS	1	5707292123039

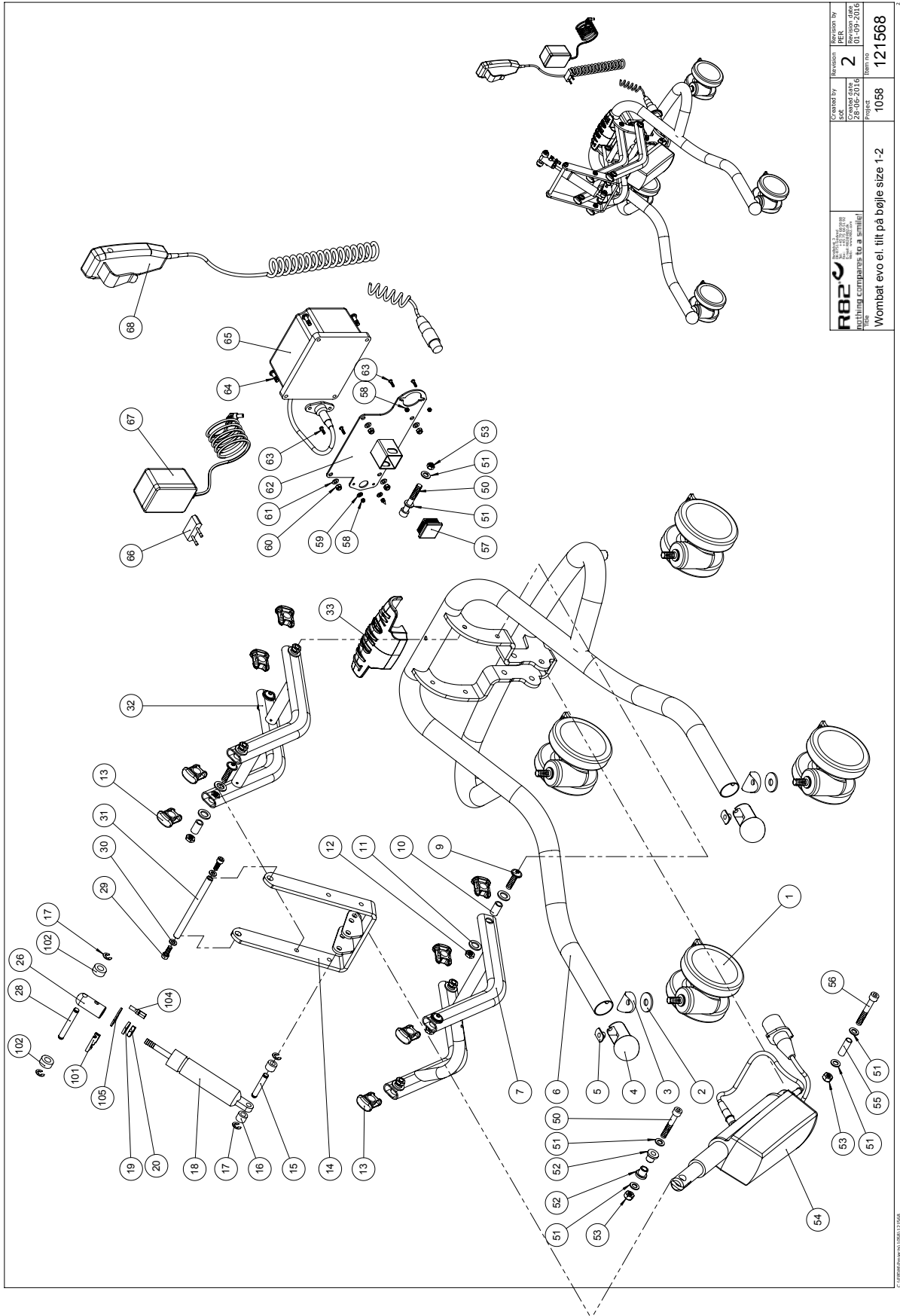


 R82 A/S Industrivej 11 2600 Lyngby Denmark We're helping companies to flourish	Created by SJK Date 24.09.2016	Revision PER 2 Date 14.09.2016	Item no. 121568
	Project 1058	Wombat evo el. tilt på bøjle size 1-2	Item no. 121568

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	3006825	75mm wheel w/brake, black	STK	4	5707292371393
1	3006938	100mm double wheel with brake	STK	2	5707292574930
2	1108050	Washer M10, din9021	STK	2	5707292140401
3	82894	Saddle washer	STK	2	5707292387578
4+5	5082829-60	Mount of ball and nut	STK	2	5707292621962
6	82920-10	Frame wombat size 1+2	STK	1	5707292622501
6	82920-35	Frame size 1+2 antrazit grey	STK	1	5707292622525
7	82790-10	Carryarm white ral 9010	STK	1	5707292622471
7	82790-35	Carryarm Antrazit grey	STK	1	5707292622495
9	0663031	Button head 8*30, Wombat	STK	8	5707292139610
10	82933	Bearing bushing, Wombat	STK	8	5707292125880
11	1310710	Nylon washer	STK	16	5707292140807
12	1750800	Lock nut, stainless M8	STK	8	5707292141484
13	82946	End piece 15*30, Wombat	STK	8	5707292126016
14	82924-10	tilt brace	STK	1	5707292621986
14	82924-35	tilt brace	STK	1	5707292622020
15	80921	locking pin, Wombat	STK	1	5707292111371
16	1350021	Distance piece 11*15*8,2mm	STK	2	5707292140890
17	1145070	Lock washer 7mm	STK	2	5707292140586
18	2203506	Bloc-o-lift gasspring 2745hu	STK	1	5707292143297
19	1310712	Nylon washer, nature	STK	2	5707292356345
20	1761000	Counter nut 10*1,0 fine thread	STK	2	5707292141538
26	6118821	Horizontal release, Wombat	STK	1	5707292186737
28	80922	Locking pin, Wombat	STK	1	5707292111388
29	0602200	Insex screw 6*16	STK	2	5707292138095
30	1103000	Fazet washer 6mm	STK	2	5707292140289
31	82806	Axle	STK	1	5707292125200
32	82921-10	Carrie arm, Wombat white	STK	1	5707292621993
32	82921-35	Carryarm	STK	1	5707292622037
34	82949-60	Bumper for wombat, black	STK	1	5707292619815
50	0603550	Insex screw 8*50	STK	2	5707292138255
51	1103200	Fazet washer 8mm	STK	6	5707292140296
52	82848	Bush	STK	2	5707292125484
53	1607000	Lock nut 8mm, large pack	STK	3	5707292141378
54	2222133	Actuator la28 f/wombat	STK	1	5707292492272
55	6108200	Pvc pipe pa11 Ø10/Ø8 mm	MTR	0,03	5707292185655
56	0603600	Insex screw 8*55	STK	1	5707292138262
57	2505000	Ferrule e30*301,0-2,0 int.blac	STK	1	5707292144140
58	1606250	Self-locking nut 3mm	STK	4	5707292408396
59	1102800	Fazet washer 4mm	STK	2	5707292140265
60	1606300	Lock nut 5 mm	STK	4	5707292141354
61	1102900	Fazet washer 5 mm	STK	4	5707292140272
62	77576	Battery console highlow	STK	1	5707292490759
63	1902590	Crosshead screw M3*10, FZB	STK	4	5707292408389
64	1902760	Cross-headed screw 5*16 mm	STK	4	5707292142146
65	2222110	Batteryboks ba18027	STK	1	5707292492258
66+67	2222161	Charger with plugs	STK	1	5707292574046
68	2220600	Handset w/r82 print & charging	STK	1	5707292492241
101	2200050	Small release arm	STK	1	5707292143075

102	1350024	Distance piece 14*15*8,2mm	STK	2	5707292140920
104	0380000	Cable tighter M5*23	STK	1	5707292135933
105	82268	Wire mounting by pedal	STK	1	5707292123039




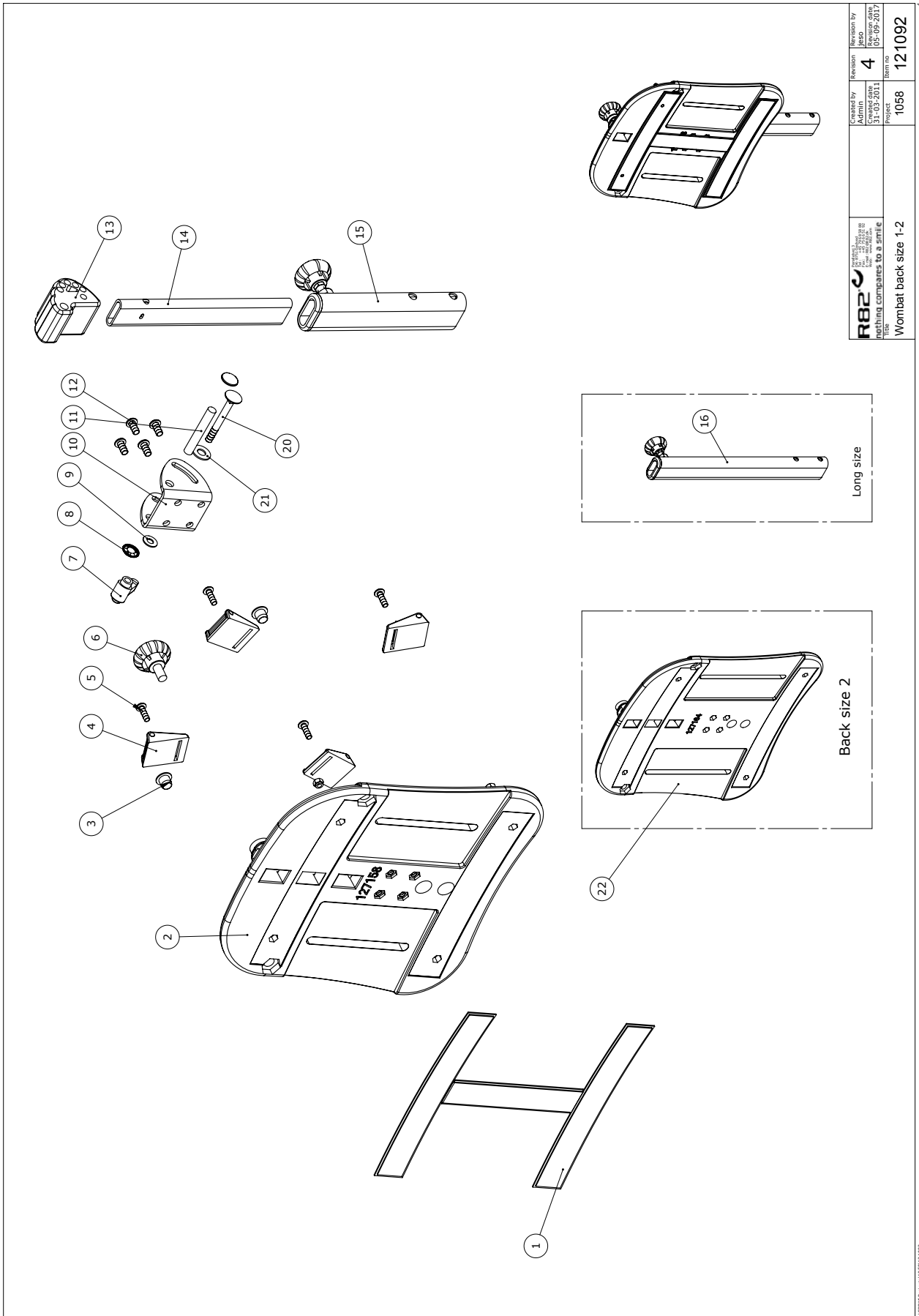


 nothing compares to a smile Title: Wombat back size 1-2	Created by: Admin: JYBA Date: 31-03-2011 Project: 1068	Revision by: JYBA Date: 05-09-2017 Revision: 4 Item no: 121092
---	---	--


C:\pwworkspace\1058311092


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	4913035	30mm black ps14 with glue hook		0,66	5707292469755
2	82943	Back size 1, Wombat	PCS	1	5707292125989
3	2501500	Ferrule d12*0,5-1,5 int.black	PCS	2	5707292143952
4	0445025	25mm press buckle, black	PCS	4	5707292465160
5	0658012	Button head M5*12 FZB	PCS	4	5707292139382
6	3503001	R82 handle M8x15mm screw	PCS	1	5707292413925
7	3502200	Tightening handle 42mm M6 bush	PCS	1	5707292146861
8	1109510	Starlock washer Ø8	PCS	2	5707292140456
9	1125618	Screenwasher fz6*18*3mm	PCS	1	5707292140494
10	82915	U-fitting for back	PCS	1	5707292125729
11	80920	Stick '8*54	PCS	1	5707292111364
12	0641200	Button head screw 6*12	PCS	4	5707292139252
13	82947-60	Back tilting device, black	PCS	1	5707292619822
14	82951	INNER BACK COLUMN	PCS	1	5707292530349
15	See Pricelist	Back column for Wombat (Short)	PCS	1	5707292532268
16	See Pricelist	Back column for Wombat (Long)			
20	1905545	Bolt M6*60, FZB	PCS	1	5707292142306
21	1105080	Stainless washer 8mm	PCS	1	5707292140364
22	127164	Wombat size 2 back	PCS	1	5707292789808
22	82945	BACK SIZE 2, WOMBAT	PCS	1	5707292126009

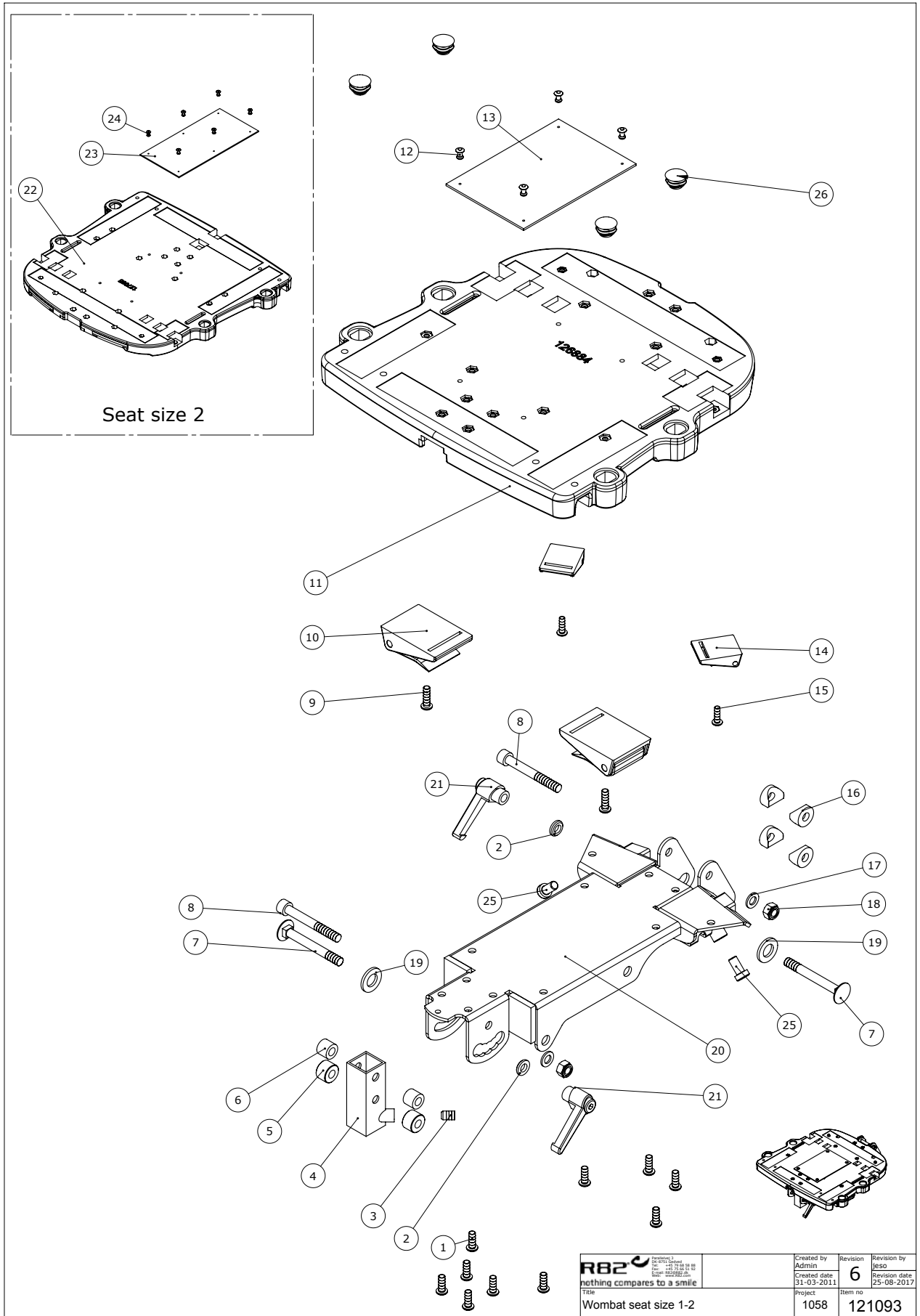

 nothing compares to a smile Title Wombat seat size 1-2	Created by Admin	Revision 6	Revision by Jeso
	Created date 31-03-2011	Revision date 25-08-2017	Item no 121093


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0641600	Button head screw 6*16	PCS	10	5707292139269
2	120701	Lock washer	PCS	2	5707292623416
3	0701000	Stop screw with crater 8*10	PCS	1	5707292139870
4	82926	Turning fitting	PCS	1	5707292125811
5	82934	Distance piece	PCS	2	5707292125897
6	1350022	Distance piece 12*15*8,2mm	PCS	2	5707292140906
7	1905710	Bolt M8*70 FZB wo/nut	PCS	2	5707292142368
8	0603665	Insex screw 8*65	PCS	2	5707292138286
9	0641620	Button head screw 6*20	PCS	2	5707292139276
10	0445025	25mm press buckle, black	PCS	2	5707292465160
11	82941	Seat size 1, Wombat	PCS	1	5707292125965
12	0506509	Rivet 4*10, stainless	PCS	4	5707292472441
13	4935300	100mm black velcro hook	MTR	0,15	5707292152107
14	0442070	FIXLOCK CLAMP 40 MM W/HOLE	PCS	2	5707292136398
15	0658010	Button head M5 x 10, stainless	PCS	2	5707292139375
15	0658020	Button head M5 x 20, stainless	PCS	2	5707292139405
16	82929	Distance piece	PCS	4	5707292125842
17	1103200	FAZET WASHER 8MM	PCS	4	5707292140296
18	1607000	Lock nut 8mm, large pack	PCS	2	5707292141378
19	1103500	Fazet washer 12mm	PCS	2	5707292140326
20	119445	Baseplate welded ny	PCS	1	5707292621405
21	3502300	Tightening handle 65mm M8 bush	PCS	2	5707292146885
22	82942	Seat size 2, Wombat	PCS	1	5707292125972
23	4935300	100mm black velcro hook	MTR	0,21	5707292152107
24	0506509	Rivet 4*10, stainless	PCS	6	5707292472441
25	0603530	Insex low head M8x16 stainless	PCS	2	5707292412492

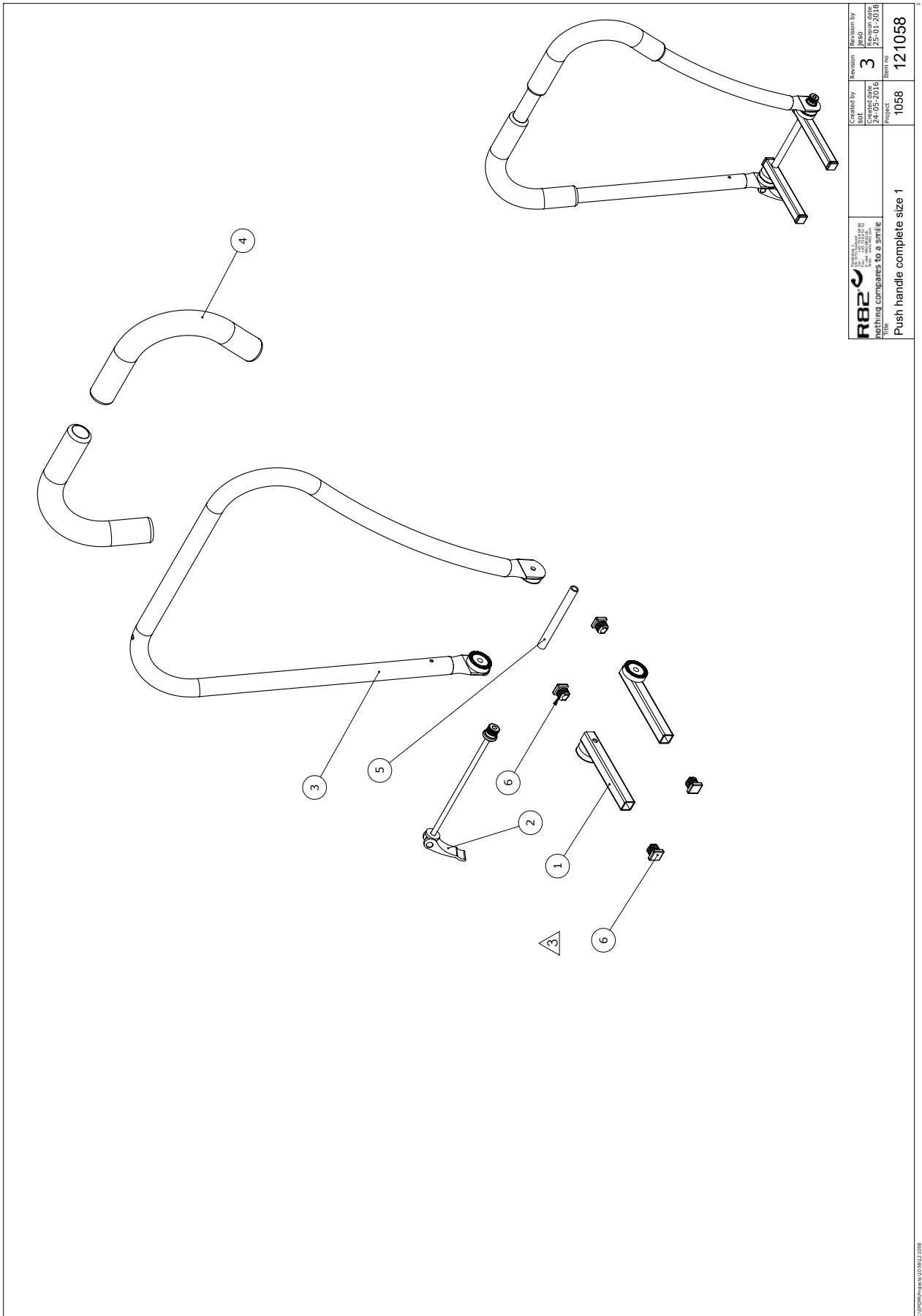

 nothing compares to a smile Title	Created by JYBA 24-05-2016	Revision by JYBA 25-01-2018
	Project 1068	Revision 3
Title Push handle complete size 1	Item no. 121058	


C:\pwworkspace\1015813108

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	120409	Bracket for handbrace size 1	PCS	2	5707292621641
2	121061	Handle size 1 angle adjustment	PCS	1	5707292622686
3	119439	Handbrace size 1 Wombat	PCS	1	5707292622112
4	2500520	Tpe cover Ø20mm	PCS	2	5707292401649
5	121080	Ø10 tube	PCS	1	5707292623317

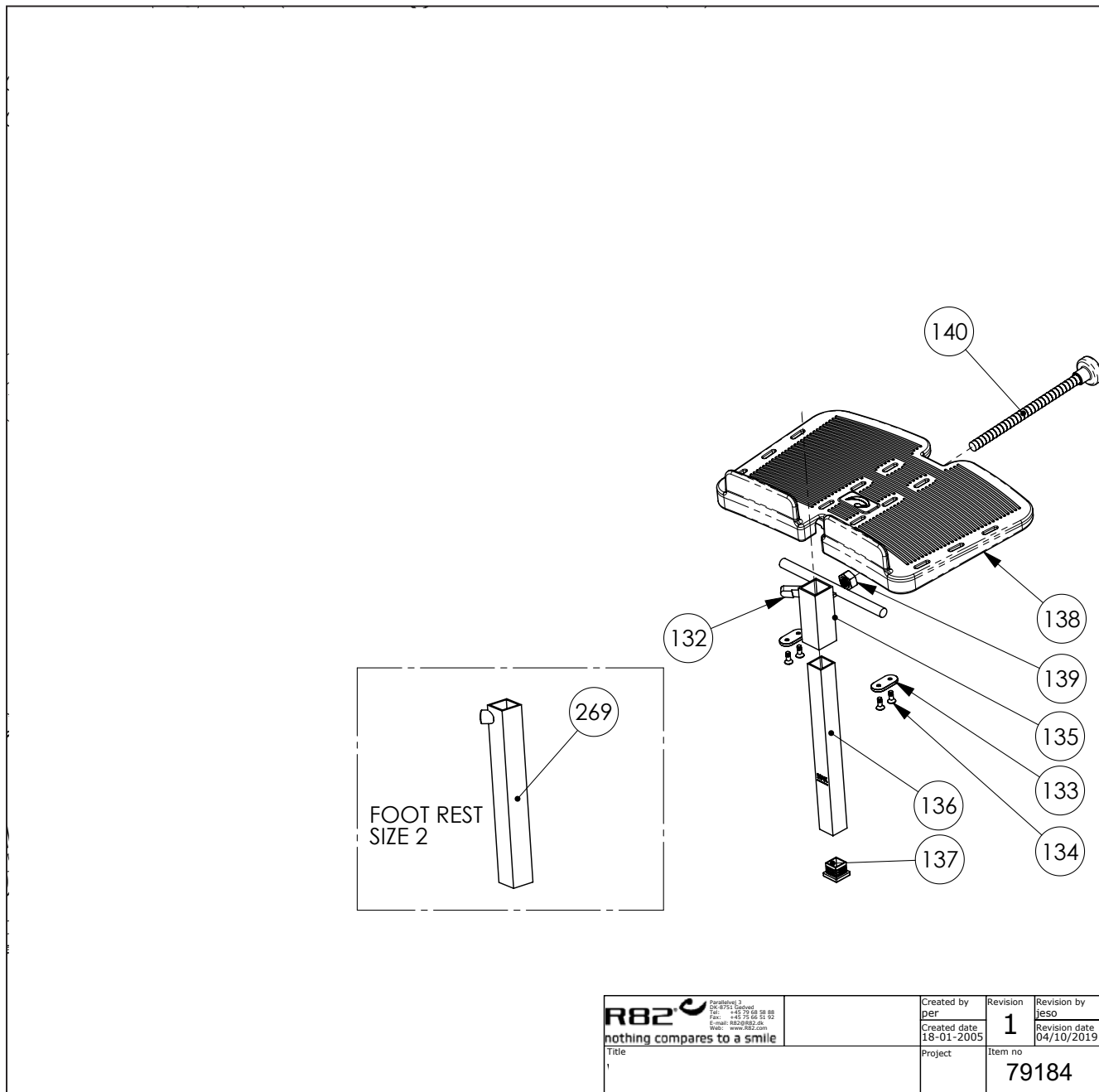

 nothing compares to a smile	Parallelvej 3 DK-8751 Gedved Tel: +45 79 68 58 88 Fax: +45 79 68 51 92 E-mail: R82@R82.dk Web: www.R82.org	Created by per Created date 18-01-2005	Revision <b>1</b>	Revision by jeSO Revision date 04/10/2019
	Title 1	Project	Item no <b>79184</b>	

Position	No.	Description	Quantity	Unit of Measure	Ean code
132	3502516	Tightening handle 45mm M8*16	PCS	1	5707292146915
133	82950	Hollow plate 16*32, Wombat	PCS	2	5707292126030
134	1940017	Sheet metal screw uhx4,2*13fzb	PCS	4	5707292142597
135	82925	Support fitting	PCS	1	5707292125804
136	82927	Pipe for footplate, Wombat	PCS	1	5707292125828
137	2504500	Ferrule e20*1,0-2,0 int.black	PCS	1	5707292144126
138	82940	Footplate size.2, Wombat	PCS	1	5707292125958
138	82944	Footplate size 1, Wombat	PCS	1	5707292125996
139	1604000	Steel nut 10 mm	PCS	1	5707292141293
140	3502790	Hand grip 28mm, M10*196mm	PCS	1	5707292363152
140	3502780	Handle 28mm,M10*160mm thread	PCS	1	5707292363145
269	82737	Pipe fitting long, Wombat 3	PCS	1	5707292399366

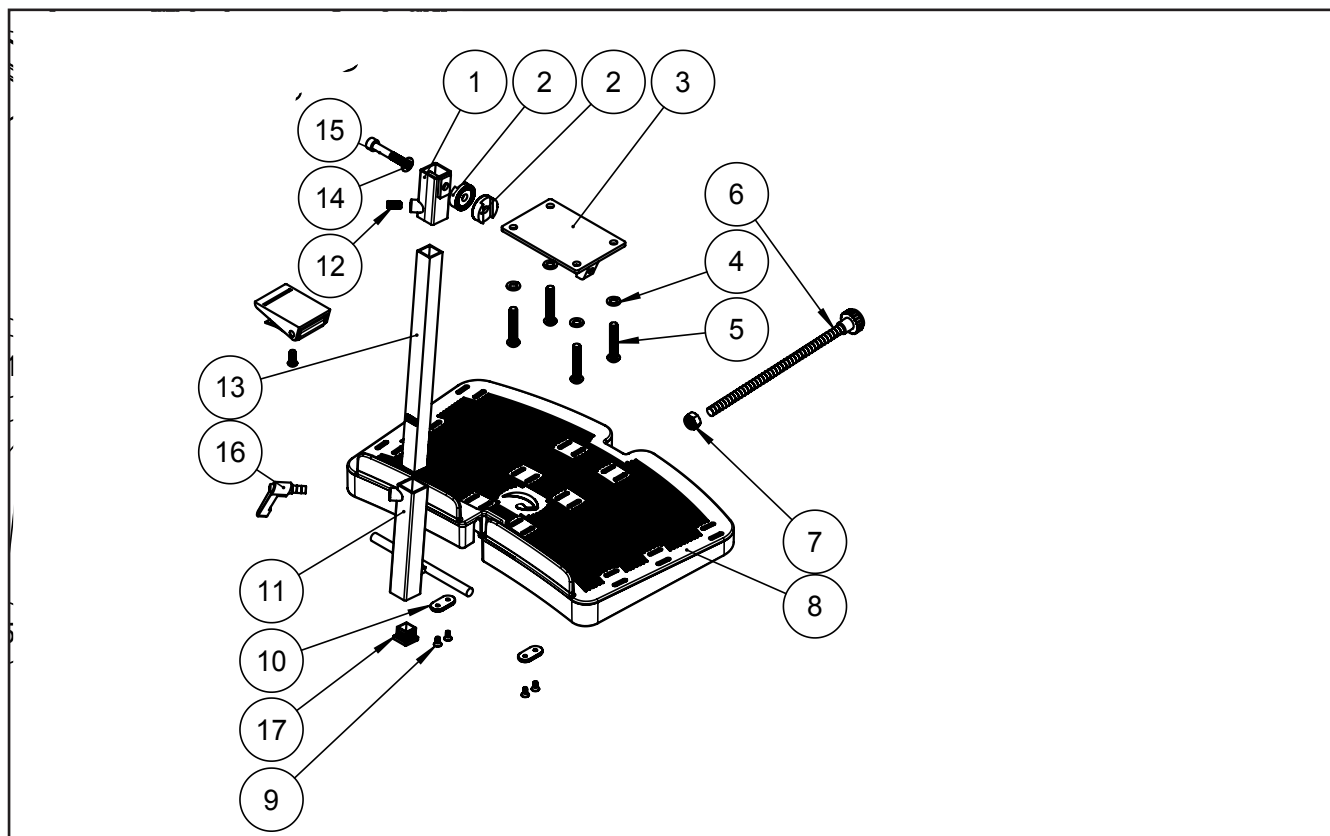

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	82735	Fix for footrest	PCS	1	5707292399335
1	82733	Profile	PCS	1	5707292399014
1	82736	Profile footrest	PCS	1	5707292399007
2	82726	Plate	PCS	1	5707292398994
2	80915	Weldbushing m8 for squire tube	PCS	1	5707292359285
2	74061	Tooth washer Ø30	PCS	2	5707292373816
3	80915	Weldbushing m8 for squire tube	PCS	1	5707292359285
3	82738	Shaft, footrest	PCS	1	5707292399373
3	82734	Plate complete	PCS	1	5707292398932
4	82727	Tube	PCS	1	5707292399342
4	118565	Spacing Piece	PCS	1	5707292616593
4	1101000	Center spring washer 8mm	PCS	4	5707292140173
5	0661145	Button head 8*45 FZB	PCS	4	5707292139535
6	3502795	Knob 28mm,M10*??mm thread	PCS	1	5707292402110
7	1604000	Steel nut 10 mm	PCS	1	5707292141293
8	82752	Footplate size 3, Wombat	PCS	1	5707292401373
9	1940017	Sheet metal screw uhx4,2*13fzb	PCS	4	5707292142597
10	82950	Hollow plate 16*32, Wombat	PCS	2	5707292126030
11	82737	Pipe fitting long, Wombat 3	PCS	1	5707292399366
12	0701000	Stop screw with crater 8*10	PCS	1	5707292139870
13	82787	Rod for Wombat footrest 280 mm	PCS	1	5707292401472
14	1103200	Fazet washer 8mm	PCS	1	5707292140296
15	0603660	Insex screw 8*60	PCS	1	5707292138279
16	3502516	Tightening handle 45mm M8*16	PCS	1	5707292146915
17	2504500	Ferrule e20*1,0-2,0 int.black	PCS	1	5707292144126