
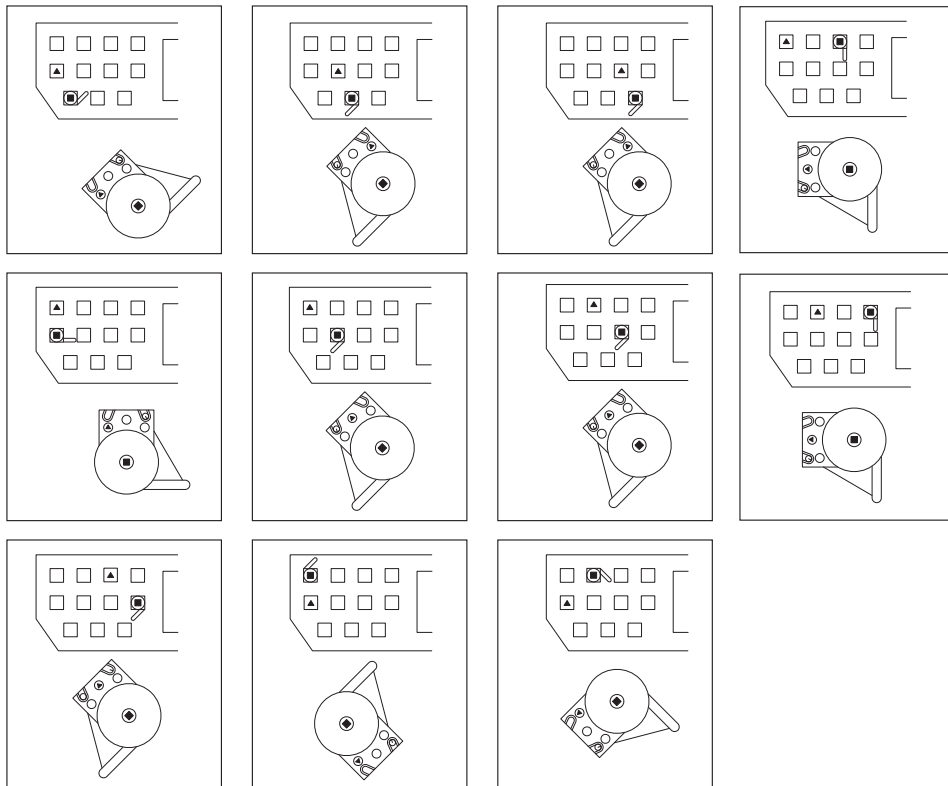
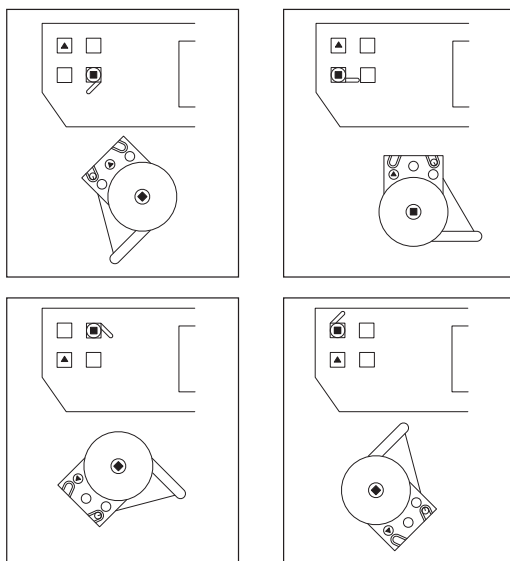
Drivhjulspositioner Cross/Cross XL / Rear wheel positions Cross/Cross XL / Antriebsradpositionen Cross/Cross XL


Drivhjulspositioner Twin / Rear wheel positions Twin / Antriebsradpositionen Twin

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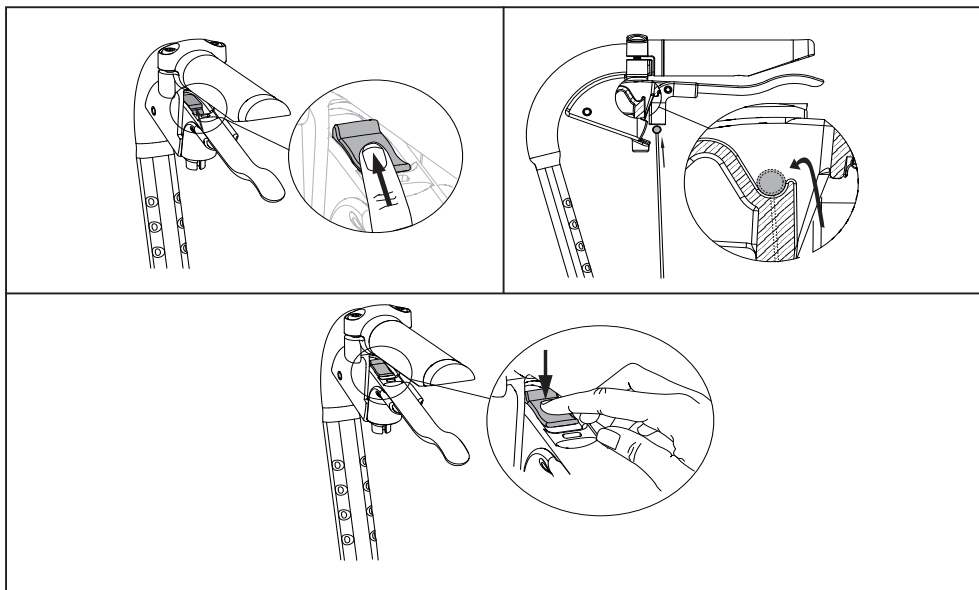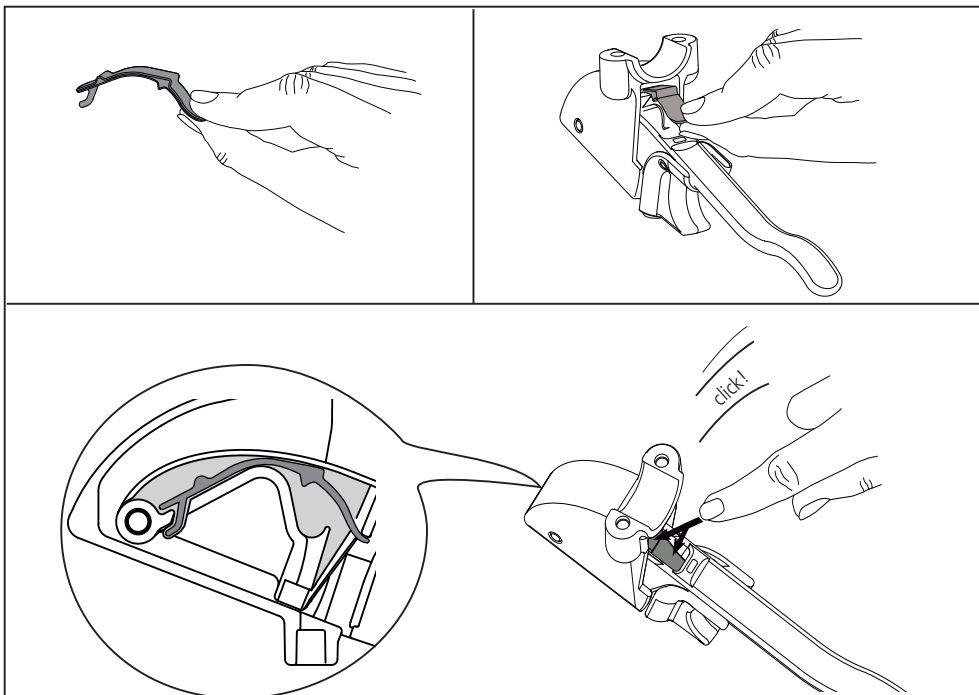Fitting Instructions

Removal

1. Remove the drive wheel with the QR axle.
2. Unscrew nut (C) (24 mm spanner) from in front of the hub sleeve (inside of frame), while holding nut (O) (24 mm spanner) behind the brake plate (outside of frame) to prevent it from turning.
3. Remove the locking pin (A) (M6 screw/nut).
4. Move the brake plate to the required drive wheel position as shown in the fitting instructions and refit the components in reverse order.

Fitting

1. Insert screw (A) through the hole in the brake plate.
2. Slide the cam (B) on to screw (A). Then fit the brake assembly in the required position. (See drawings on next page.)
3. Fit parts (C), (D) and (E). Fit washer (F) on screw (A). Screw on nut (G) and tighten securely.
4. Secure the end clamp (H) to the brake arm. Engage the nipple (I) in the brake plate.
5. Then fit the drive wheel (J).
6. Insert the cable through the sleeve.
7. Adjustment is made using nipple (I).
8. Remove brake (K) and replace screw (L).
9. Fit adapter (M) to screw (L). Then fit the brake.
10. Fit quick-release (N)

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