Monteringsvejledning (DK)

Montering af R82 5-punktssele

Monteringsanvisning (SV)

Montera R82 5-punktsbälte

Monteringsveiledning (N)

Montering av R82 5-punkt sele

Mounting instruction (GB)

How to mount the R82 5-point belt

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Montage des R82 5-Punkt-Gurts

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Monteren van de R82 5-puntsgordel

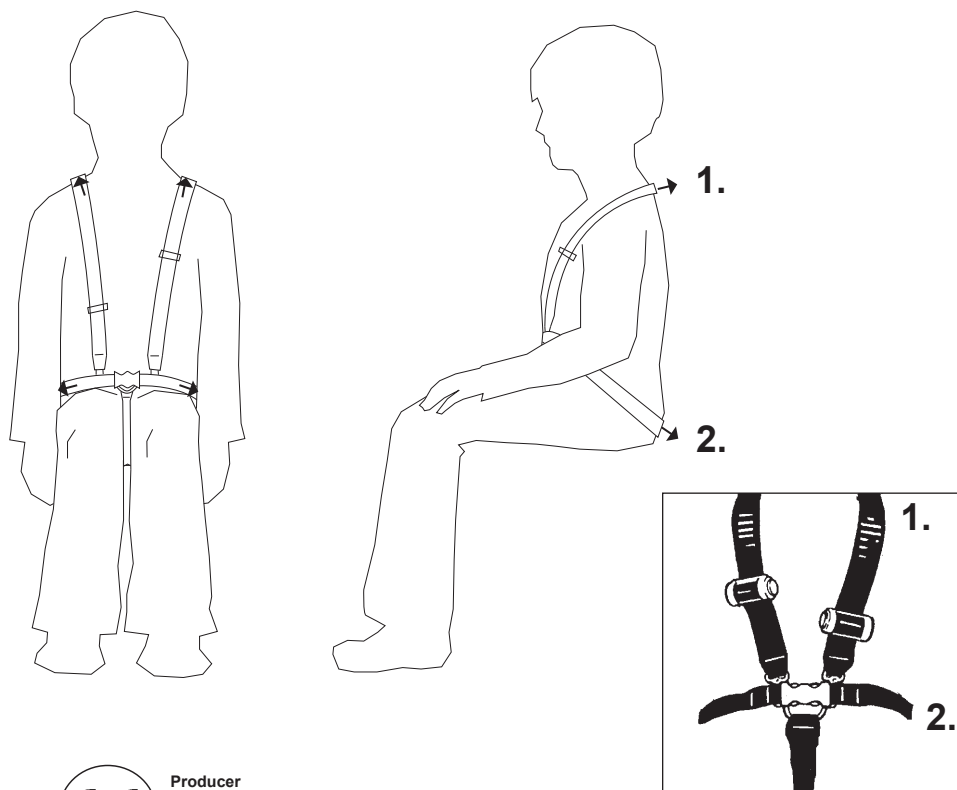
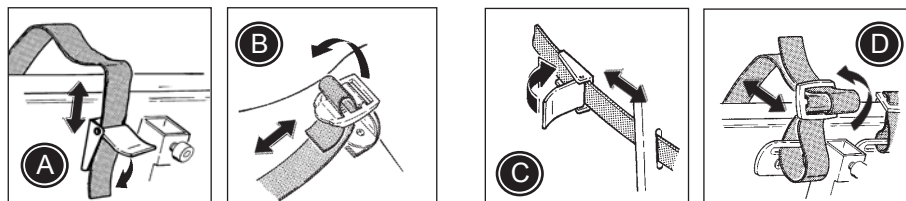
Instruction de montage (F)

Comment monter le harnais 5 points R82

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R82®

Montering af R82 5-punkt sele

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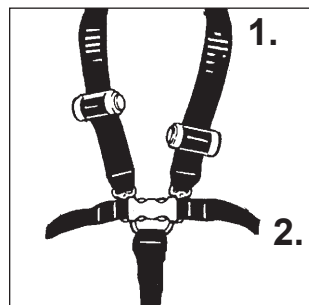
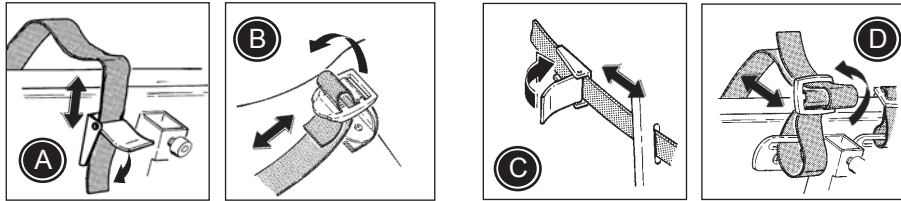
Mounting instruction

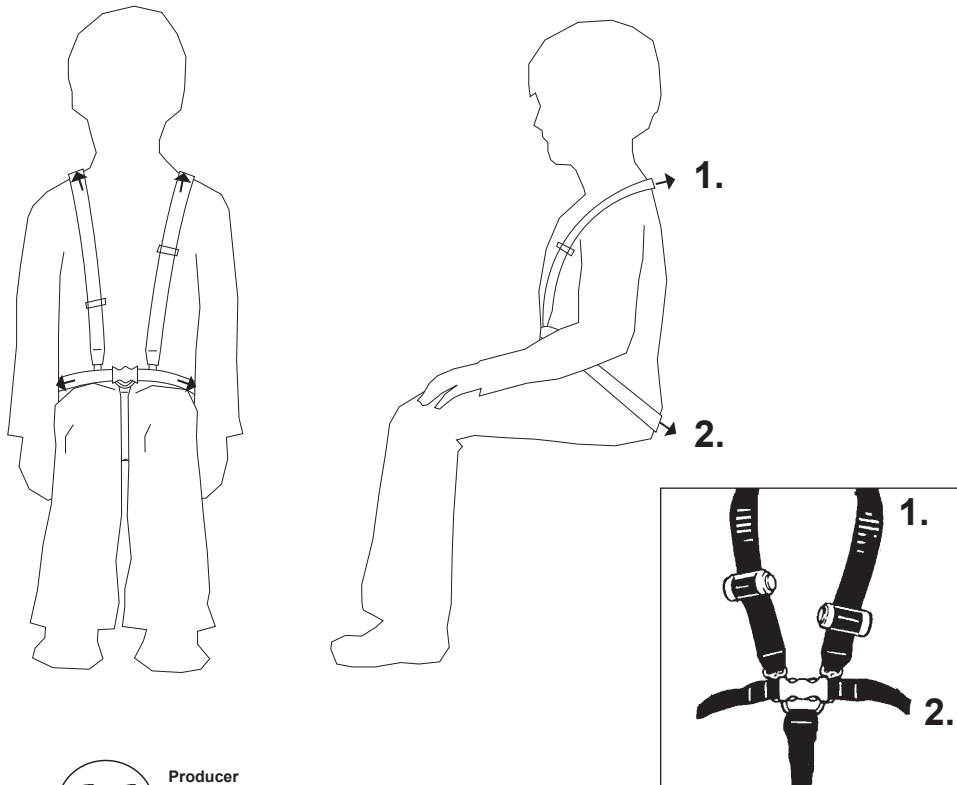
How to mount the R82 5-point belt

Use the fittings (A) and (B) on the chair to mount the 5-point belt. If no fittings are mounted, please use the enclosed fix locks and belt fittings to mount the 5-point belt as shown on picture (C) and (D). Any doubt on mounting, please contact a local therapist or authorized R82 dealer.

- * Always mount the shoulder straps (1) a little higher than the shoulder height of the user.
- * Mount the lower straps (2) at the back of the seat. Mount them in about 45° angle of the seat.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The 5-point belt is fitted correctly when two fingers can be put between the harness and the child's body. Check before every use.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.

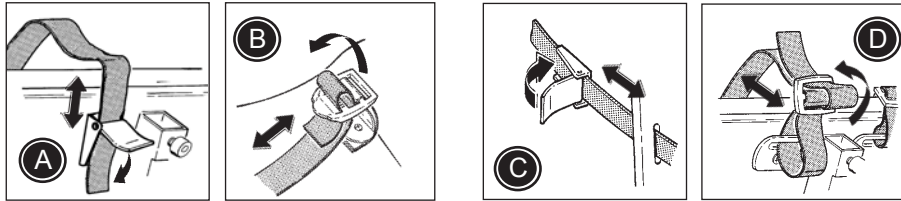
 Carefully read the care label on the harness to maintain and wash the 5-point belt correctly.

 The latest version of the mounting instruction is always available online: R82.org

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