
Recondition instruction **molift**[®]

Molift Transfer Pro by Etac

RE70199 - 2024-04-04 - Rev. 2.0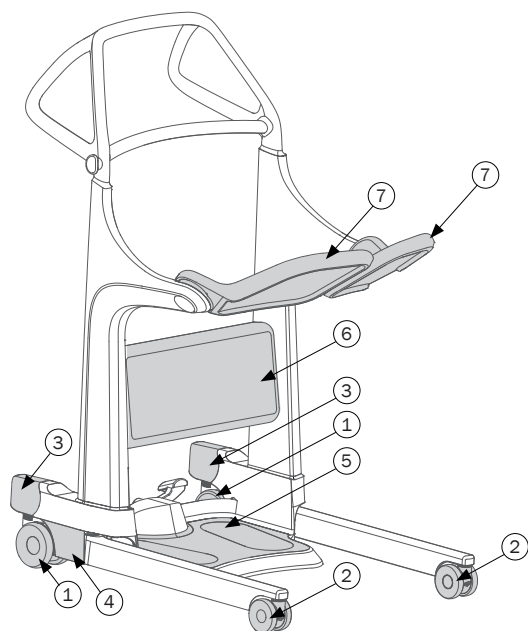

Reconditioning illustration Page


1. Rear castors	2
2. Front castors	2
3. Brake pedal	2
4. Leg adjustment covers - LH/RH	2
5. Footplate	3
6. Leg support	3
7. Seat pad	3

Reconditioning language Page


English	4
Svenska	5
Norsk	6
Dansk	7
Suomi	8
Deutsch	9
Nederlands	10
Français	11
Italiano	12
Español	13


1. Rear castors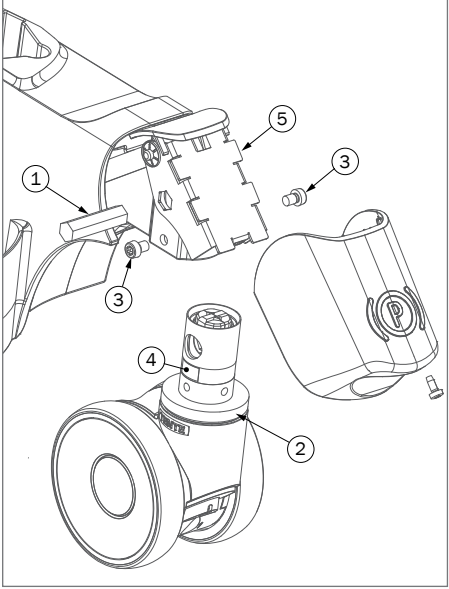


2. Front castors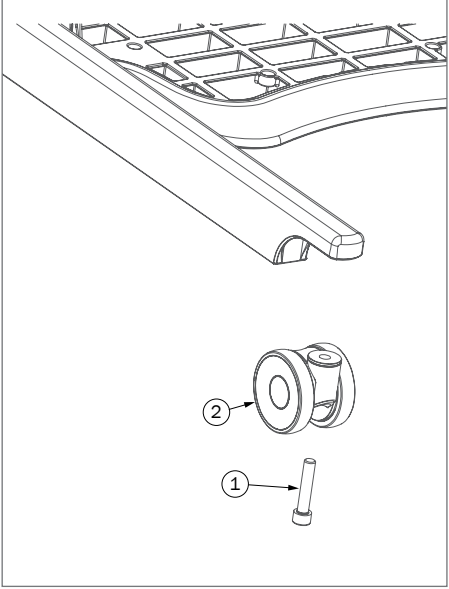

3. Brake pedal

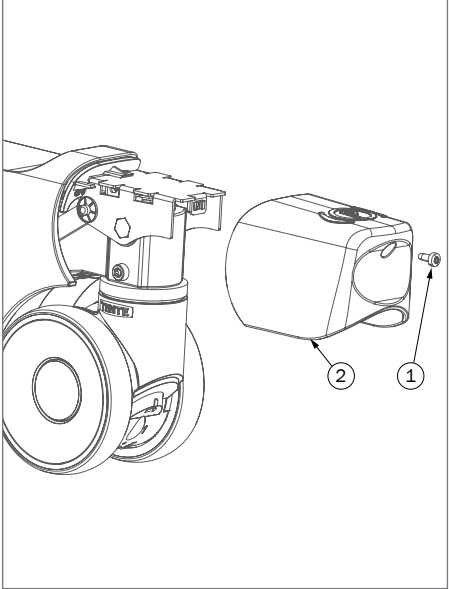
4. Leg adjustment covers – LH/RH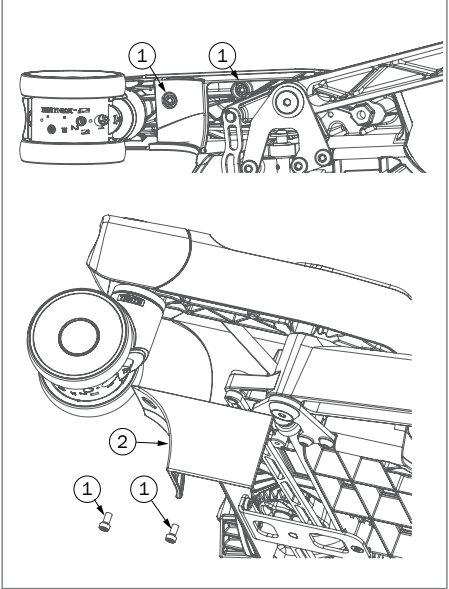


5.

Footplate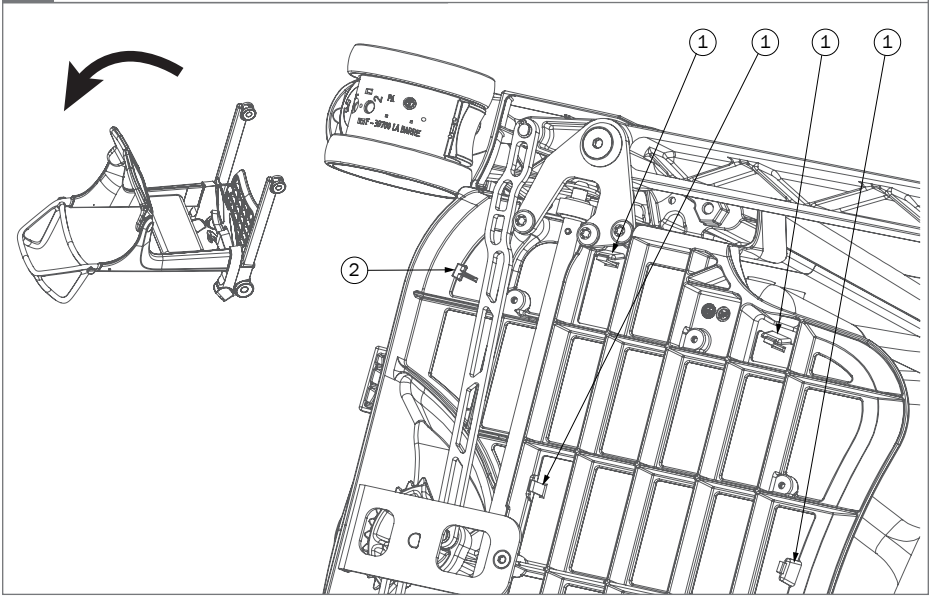


6.

Leg support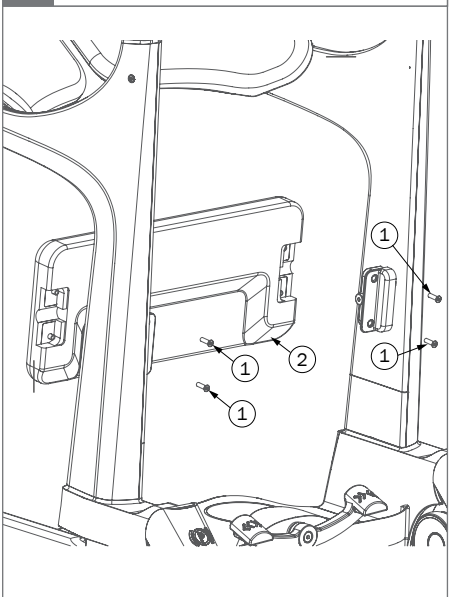

7.

Seat pad LH/RH

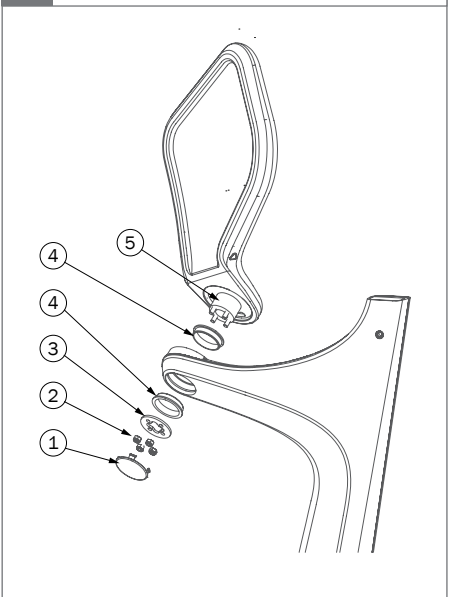1. Rear castors

- Place device on a platform with the rear castors over the edge so they are free to remove
- Remove brake pedal. See "Brake pedal"
- Push down to activate the brake pedal
- Remove the hexagon shaft (1)
- Support castor (2) (castor will fall out) and unscrew both bolts (3) (use TX30)
- Insert the hexagon shaft into the castor and activate the brake (use 11mm wrench)
- Insert new castor into the base. Yellow Label (4) on castor shaft must be on the left
- Install the two new bolts
- Install the hexagon shaft (brake must be activated when doing this)
- Install the brake pedal
- Check that the brake works. Must be able to be activated/deactivated from both brake pedals and verify that both castors lock

2. Front castors

- Place device on a platform with the front castors over the edge so they are free to remove
- Remove bolt (1) (use hex 8)
- Castor (2) is free to remove
- Install new castor w. new bolt

3. Brake pedal

- Unscrew screw (1) (use TX20)
- Pull hard back on brake pedal (2) to remove
- Slide new brake pedal into place
- Install new screw
- Check that the brakes works

4. Leg adjustment covers – LH/RH

- Turn castor as shown and move legs to outmost position to access bolts (1)
- Unscrew the 2 bolts
- Remove cover (2)
- Install new cover w. 2 new bolts

5. Footplate

- Activate brakes on device
- Pull the device back and lay it on the handle so that the bottom of the lifter is free
- Flip out all 8 front hooks on the footplate. Start from front to back. Footplate will be free to remove
- Install new/cleaned footplate by placing rear hooks in position and rotate footplate into position. Press footplate into position
- Check that the rear hooks are correctly placed (no gap under the footplate at the rear)

6. Leg support

- Unscrew the 4 bolts (1) (use TX20)
- Remove leg support (2)
- Install new leg support. Note: Cutout in leg support must face down as shown
- Install new bolts

7. Seat pad – Same procedure LH/RH

- Remove red cover (1) (use small screwdriver)
- Unscrew the 4 nuts (2) (use 10mm socket)
- Remove the washer under nuts (3)
- Pull out seat pad (5)
- Remove the two bushings (4)
- Install two new bushings into the frame
- Install seat pad into the bushings. Note: Bushings must stay in place during installing seat pad, and the slot in the seat pad must align with the stop pin
- Install new washer and 4 new nuts
- Check that the seat pad rotates freely from stop to stop
- Install new red cover

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Etac Ltd.
Etac A/S