

# Flamingo high-low

## Exploded views / Part lists


Rev. date (2023-11)


*On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.*



## Contents


Seat.....	3
Frame.....	5
Accessories .....	8
Armrest .....	11
Footrest.....	13
Side support .....	15

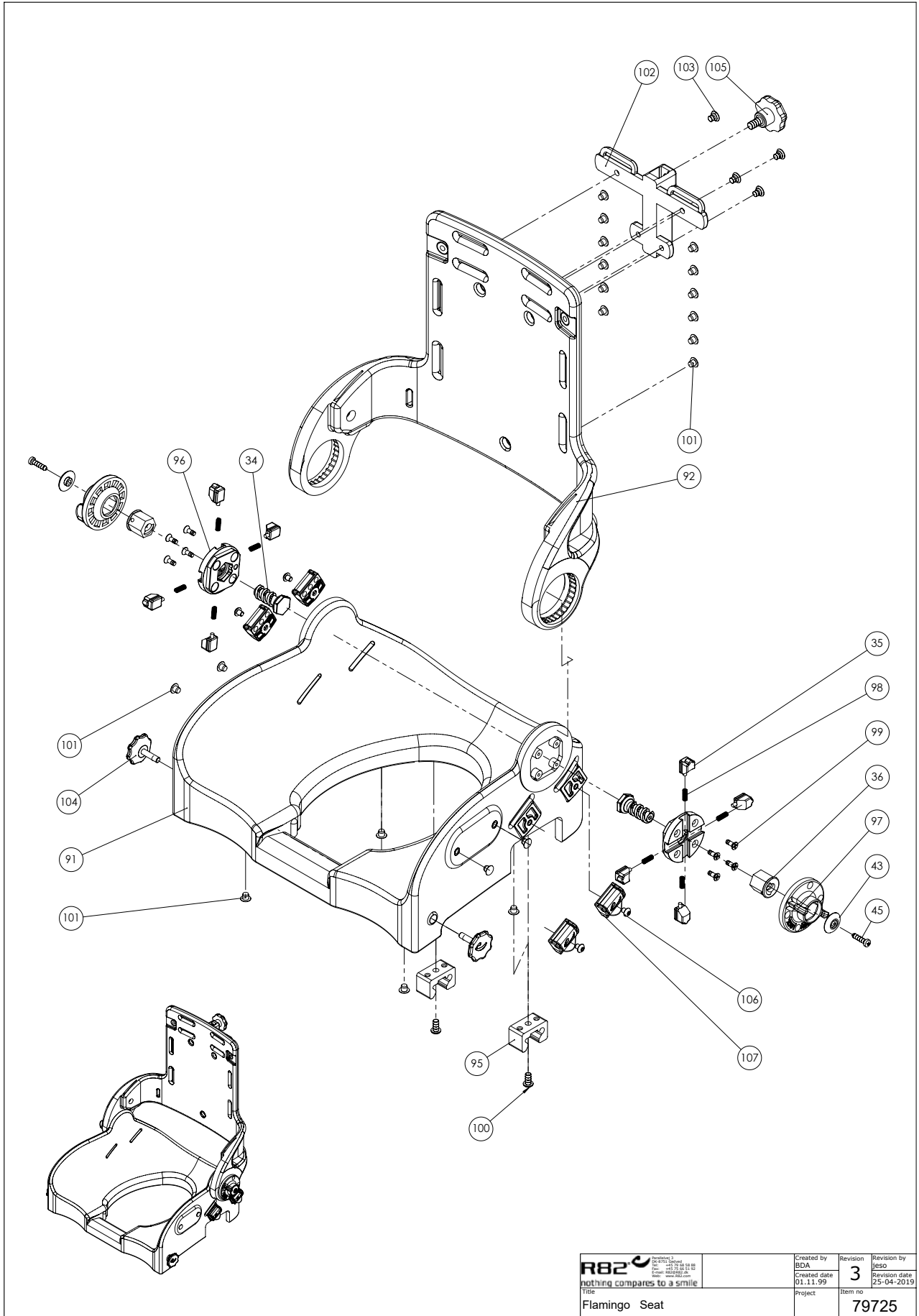

 R82 A/S nothing compares to a smile Tel: +45 79 68 58 88 Fax: +45 75 66 51 92 E-mail: R82@R82.dk Web: www.r82.com	Created by BDA	Revision <b>3</b>	Revision by jeso
	Created date 01.11.99	Revision date 25-04-2019	
Title Flamingo Seat	Project	Item no. <b>79725</b>	

C:\FROM\TECH\Explode\Flamingo\79725


Position	No.	Description	Quantity	Unit of Measure	Ean code
36+43 45+96 97+98 99	880153	Angle adj. kit Flamingo back	PCS	1	5707292429551
34	71136	Thread spindle, Flamingo	PCS	2	5707292104410
35	71133	T-flange, Flamingo	PCS	8	5707292104380
36	71128	Thread nut for angle adj.	PCS	2	5707292104359
43	71164	Knob Ø23, 5 for handle	PCS	2	5707292104540
45	1811112	Sheet metal screw 4,8*19 phx	PCS	2	5707292452184
91*	71302-1	Flamingo Seat shell, size 1	PCS	1	5707292629401
	71303-1	Flamingo Seat shell, size 2	PCS	1	5707292629418
	71304-1	Flamingo Seat shell, size 3	PCS	1	5707292629425
	71305-1	Flamingo Seat shell, size 4	PCS	1	5707292629432
92*	71312	Flamingo back shell, size 1	PCS	1	5707292105028
	71313	Flamingo back shell, size 2	PCS	1	5707292105035
	71314	FLAMINGO BACK SHELL, SIZE 3	PCS	1	5707292105042
	71315	Flamingo back shell, size 4	PCS	1	5707292105059
95	102429	Seatlock for Flamingo	PCS	2	5707292562036
97	71144	Hand wheel, Flamingo	PCS	2	5707292104427
99	1940038	ECO-SYN UH Ø4*14	PCS	8	5707292465597
100	0659200	Button head screw 6*12 stainl.	PCS	2	5707292139436
101	1310536	Covering button,white 13*5,0mm	PCS	16	5707292397911
102	81138-90	Mounting for neck support	PCS	1	5707292227157
103	0609727	Stainless insex 6x12 uh	PCS	4	5707292138484
104	112708	Knob for Seat m6x23	PCS	2	5707292604286
105	3501050	Star grip Ø40 M8*15mm white	PCS	1	5707292146717
106	0631953	Button head M5*6, stainless	PCS	4	5707292599988
107	110202	Snap Lock R82-25 With ø6	PCS	4	5707292575913

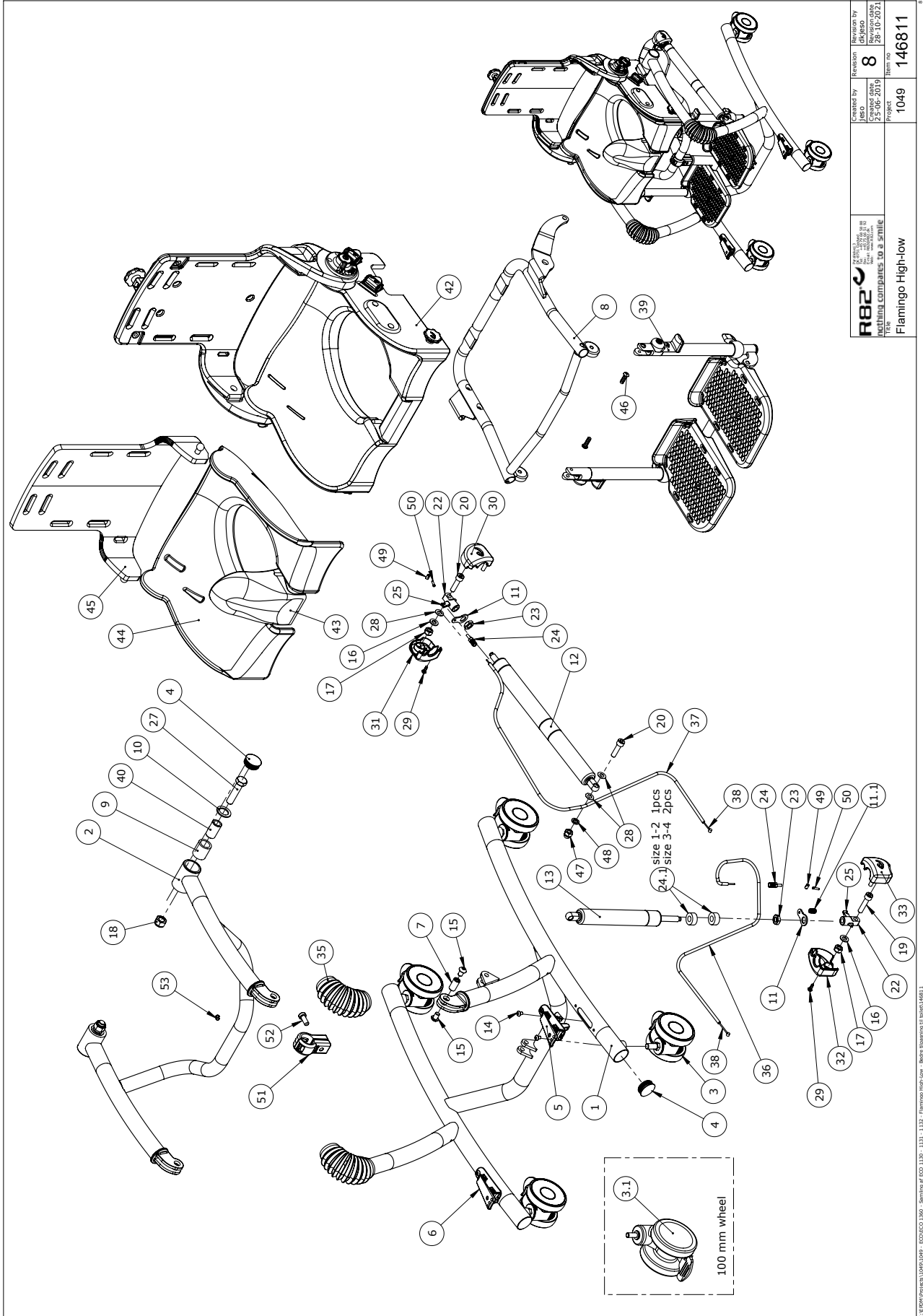

Position	No.	Description	Quantity	Unit of Measure	Ean code
1*	135994-10	Frame Bottom, size 3+4	PCS	1	5707292877178
	135990-10	Frame Bottom, size 1+2	PCS	1	5707292877161
2*	138539-10	Frame top, size 3+4	PCS	1	5707292893826
	138414-10	Frame top, size 1+2	PCS	1	5707292893802
3*	3006955	100MM DOUBLE WHEEL, WHITE	PCS	4	5707292601872
	3006830	75MM WHITE STEINCO WHEEL	PCS	4	5707292368560
	3006955	100mm double wheel, white	PCS	4	5707292601872
4	2507006	Ferrule for pipe, grey Ø32	PCS	6	5707292580917
5	111869-1	Release pedal complete f. tilt	PCS	1	5707292839596
6	114288-1	Release pedal komplet for lift	PCS	1	5707292839602
7	100236	Bush ø12 x M8 x 22,5	PCS	2	5707292559746
8*	112413-10	Seatframe complete size 4	PCS	1	5707292602817
	112412-10	Seatframe complete size 3	PCS	1	5707292602800
	112411-10	Seatframe complete size 2	PCS	1	5707292602794
	112410-10	Seatframe complete size 1	PCS	1	5710000000000
9	100234	Bearing ø25x18x28	PCS	2	5707292560001
10	100233	Disc ø27x18,2x2	PCS	2	5707292560025
11	82268	Wire mounting by pedal	PCS	2	5707292123039
11.1	0310100	Pressure spring 0,9*8*23*10,5v	PCS	1	5707292135513
12*	2206510	Lockable Gas traction spring	PCS	1	5707292603838
	2206505	Lockable Gas Traction spring	PCS	1	5707292630087
13*	2206160-2	Bloc-o-lift, 174268, 800N	PCS	1	5707292895851
	2206150-2	Bloc-o-lift, 174029, 600N	PCS	1	5707292895844
14	0631952	Button head M5*6, rf tufflok	PCS	4	5707292503725
15	0661153	Button head screw 8*10 stainl.	PCS	4	5707292616654
16	1105080	Stainless washer 8mm	PCS	2	5707292140364
17	1631080	Stainless lock nut 8mm	PCS	2	5707292141453
18	1631120	Stainless lock nut 12mm	PCS	2	5707292502049
19	0609835	Stainless insex screw 8*35	PCS	1	5707292138606
20	0609832	Stainless insex screw 8*30	PCS	2	5707292606037
22	138241	House for gasspring	PCS	2	5707292839923
23	1761005	Counter nut 10*0.1	PCS	2	5707292509925
24	0380001	Adj screw U-BA-BZ-E-SM5-VA	PCS	2	5707292829771
24.1	1310630	Distance piece, Ø10 (size 1 '1 PCS'; Size 2 '2 PCS')	PCS	2	5707292140753
25	2200050	Small release arm	PCS	2	5707292143075
26	2501600	Ferrule d15*1,0 black	PCS	1	5707292143969
27	0607999	Stainless Insex screw 12*50	PCS	2	5707292482013
28	1310645	Washer, nature Ø16*8*0,5mm	PCS	1	5707292446015
29	1810050	Plate screw 7981c st 2,9*13	PCS	2	5707292404671
30	113085	Cover, outside	PCS	1	5707292604279
31	113086	Cover, inside	PCS	1	5707292604262
32	113099	Cover, right	PCS	1	5707292604255
33	113098	Cover, left	PCS	1	5707292604248
35	112354	Bellow for frame/adjust. arm	PCS	2	5707292602039
36*	113783	Outher casing for lift, 3-4	PCS	1	5707292604347
	140634	outher casing f. lift size 1-2	PCS	1	5707290000000

\*Choose size/color/type

37*	109043	Cable seat tilt size 3-4	PCS	1	5707292575593
	140635	Outher casing f.tilt size 1-2	PCS	1	5707290000000
38	0371854-1	Soft inner cable for Stingray	PCS	2	5707292536945
39*	116969-4	Flamingo footrest kompl.size 4	PCS	1	5707292609588
	116969-3	Flamingo footsup,compl. s.3	PCS	1	5707292609571
	116969-2	Flamingo footsup,compl.s 2	PCS	1	5707290000000
	116969-1	Flamingo footsup,complete.s 1	PCS	1	5710000000000
40	100229	Bush	PCS	2	5707292560018
42*	71004	Back&seat, Flamingo size 4	PCS	1	5707292103956
	71003	Back&seat, Flamingo size 3	PCS	1	5707292103949
	71002	Back&seat, Flamingo size 2	PCS	1	5707290000000
	71001	Flamingo size&back sz 1, compl	PCS	1	5710000000000
43*	880192	Splashguard, high, size 2-4	PCS	1	5707292134547
	880191	Splashguard, high, size 1	PCS	1	5710000000000
44*	880164	Upholstery for Seat size 4	PCS	1	5707292134448
	880163	Upholstery Flamingo Seat sz.3	PCS	1	5707292134431
	880162	Upholstery Flamingo Seat sz.2	PCS	1	5707290000000
	880161	Upholstery Flamingo Seat sz.1	PCS	1	5710000000000
45*	880174	Upholstery sz.4,back,flamingo	PCS	1	5707292134486
	880173	Upholstery Flamingo back sz.3	PCS	1	5707292134479
	880172	Upholstery Flamingo back sz.2	PCS	1	5707290000000
	880171	Upholstery Flamingo back sz.1	PCS	1	5710000000000
46	0659555	Button head M6*20, A2 w/ ES-Lo	PCS	2	5707292612120
47	1781080	Stainless top nut 8mm	PCS	1	5707292141576
48	1100500	Spring washer 8mm, stainless	PCS	1	5707292140142
49	0388006	Screwnipple 5*7,5mm	PCS	2	5707292136114
50	0387000	End nipple for outer casing	PCS	1	5707292136091
51	140227	Lock for springstopper	SÆT	1	5707292881328
52	0603533	Insex M8x20, lav hoved RF	PCS	1	5707292575296
53	0506509	Rivet 4*10, stainless	PCS	1	5707292472441
54	1310645	Washer, nature Ø16*8*0,5 mm	PCS	2	5707292446015
*Choose size/color/type					

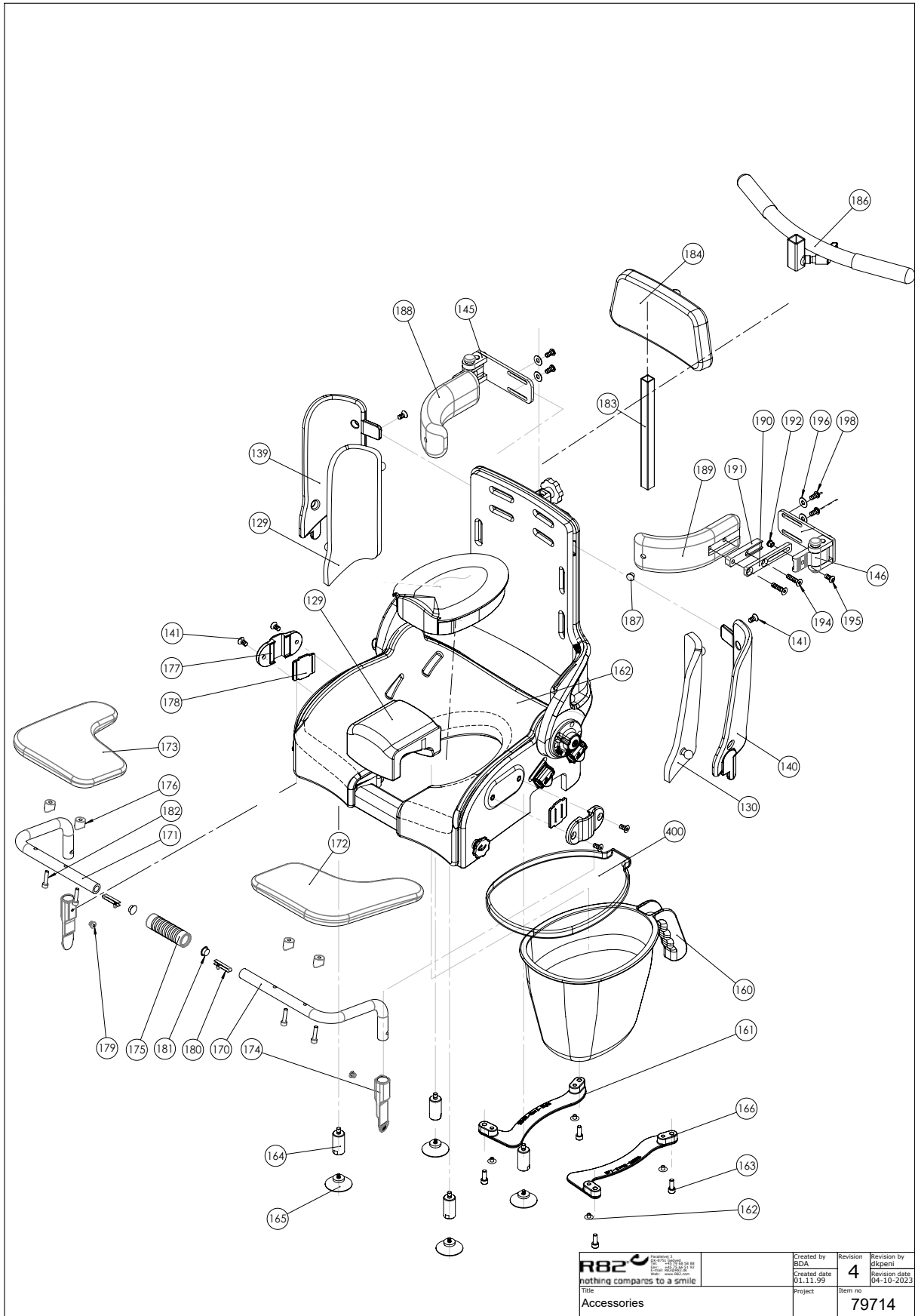

C:\EPDM\TECHN\Explosdi\Flamingo\79714

<p>R82<sup>®</sup> nothing compares to a smile</p>	<p>Created by BDA</p>	<p>Revision <b>4</b></p>	<p>Revision by dkkrent</p>
	<p>Created date 01.11.99</p>	<p>Revision date 04-10-2023</p>	<p>Item no. <b>79714</b></p>
<p>Title Accessories</p>		<p>Project</p>	




Position	No.	Description	Quantity	Unit of Measure	Ean code
129*	880181-H	Upholst. F/right side,flam 1	1	PCS	5707292254467
	880181-61H	Side upholstery, sz.1, right	1	PCS	5707292615893
	880182-H	Upholst. F/right side, flam. 2	1	PCS	5707292254481
	880182-61H	Side upholstery, sz.2, right	1	PCS	5707292615916
	880183-H	Upholst. F/right side,flam. 3	1	PCS	5707292254504
	880183-61H	Side upholstery, sz.3, right	1	PCS	5707292615930
	880184-H	Upholst. F/right side,flam. 4	1	PCS	5707292254528
130*	880184-61H	Side upholstery, sz.4, right	1	PCS	5707292615954
	880181-V	Upholst.f/left side flam. Sz.1	1	PCS	5707292254474
	880181-61V	Side upholstery, sz.1, left	1	PCS	5707292615909
	880182-V	Upholst. F/left side, flam. 2	1	PCS	5707292254498
	880182-61V	Side upholstery, sz.2, left	1	PCS	5707292615923
	880183-V	Upholst. F/left side, flam. 3	1	PCS	5707292254511
	880183-61V	Side upholstery, sz.3, left	1	PCS	5707292615947
139*	880184-V	Upholst f/left side,flam. 4	1	PCS	5707292254535
	880184-61V	Side upholstery, sz.4, left	1	PCS	5707292615961
	71322	Backside size 1,right-Flamingo	1	PCS	5707292105066
	71323	Backside size 2,right-Flamingo	1	PCS	5707292105073
140*	71324	Backside size 3,right-Flamingo	1	PCS	5707292105080
	71325	Backside size 4,right-Flamingo	1	PCS	5707292105097
	71332	Backside size 1, left-Flamingo	1	PCS	5707292105103
	71333	Backside size 2, left-Flamingo	1	PCS	5707292105110
141	71334	Backside size 3, left,flamingo	1	PCS	5707292105127
	71335	Backside size 4, left-Flamingo	1	PCS	5707292105134
145*	0609727	Stainless insex 6x12 uh	2	PCS	5707292138484
146*	5982560-R	Sidesupport 118*42,5mm, right	1	PCS	5707292396082
	5982580-R	Svingbar sidestøtte højre	1	PCS	5707292396105
160*	5982560-L	Sidesupport 118*42,5mm, left	1	PCS	5707292396020
	5982580-L	Svingbar sidestøtte venstre	1	PCS	5707292396044
161*	880128	Comode pan, Flamingo size 1+2	1	PCS	5707292134301
	880129	Comode pan, Flamingo size 3+4	1	PCS	5707292134318
162	135470	Pottebøjle str.2, højre	1	PCS	5707292893680
	136218	Pottebøjle str.1, højre	1	PCS	5707292893659
	135544	Pottebøjle str.3, højre	1	PCS	5707292893697
163	1310536	Covering button,white 13*5,0mm	4	PCS	5707292397911
164	0602206	Insex M6*16 rustfri m/ nylon	4	PCS	5707292896629
165	71134	Distancepart for suction cup	4	PCS	5707292104397
166*	2522015	Suction cup Ø80mm M6*12mm	4	PCS	5707292144461
170*	138575	Flaminco str. 2 - ABS	1	PCS	5707292893673
	136237	Pottebøjle str.1, venstre	1	PCS	5707292893666
	135538	Pottebøjle str.3, venstre	1	PCS	5707292873040
170*	71371	Supporting tube 1 left	1	PCS	5707292105394
	71373	Support rod for size 2	1	PCS	5707292105417
	71367	Supporting tube 3 left	1	PCS	5707292436771
	71369	Support rod size 4, left	1	PCS	5707292436757

\* choose size/color


Position	No.	Description	Quantity	Unit of Measure	Ean code
171*	71370	Supporting tube 1 right	1	PCS	5707292105387
	71372	Supporting tube 2 right	1	PCS	5707292105400
	71366	Supporting tube 3 right	1	PCS	5707292436764
	71368	Support rod size 4, right	1	PCS	5707292436740
172*	6175361	Armsupport tray small, left	1	PCS	5707292189707
	6075361	Arm rest table small, left PUR	1	PCS	5707292613103
	6175361	Armsupport tray small, left	1	PCS	5707292189707
	6175365	Armsupport tray large, left	1	PCS	5707292189721
	6075365	Arm rest left, big - Hell Blau	1	PCS	5707292613066
173*	6175359	Armsupport tray small,right	1	PCS	5707292189691
	6075359	Arm rest table small,right PUR	1	PCS	5707292613110
	6175363	Armsupport tray large, right	1	PCS	5707292189714
	6075363	Arm rest right, big - Hellblau	1	PCS	5707292613073
174	71130-90	Bushing, white	2	PCS	5707292223593
175	71132	Locking socket,white	1	PCS	5707292104373
176	104377	Saddle disc Ø15,5Ø5,3, h10,0	4	PCS	5707292568878
177	72134-90	Mounting for armrest, white	2	PCS	5707292223760
178	82408-90	Underlying plate armrest flam	2	PCS	5707292231482
179	0506519	Stainless blind rivet 4*12	2	PCS	5707292137388
180	6153700	Knob spring f.activity centre	2	PCS	5707292188601
181	1310552	Hedge plug k 5 natural	2	PCS	5707292581747
182	0603522	Insex ch M5*22 stainless	4	PCS	5707292490988
183	108407	Tube for back extension	1	PCS	5707292573452
184*	880118	Back extension, grey PU, sz. 1+2	N/A	N/A	5707292572936
	880119	Back extension, grey PU, sz. 3+4	N/A	N/A	5707292572943
186	880110	Push brace for flamingo/swan	1	PCS	5707292134219
187	2502519	Tube leg, grey, Ø16mm	2	PCS	5707292394415
188+189*	6075407	Side cushion, large, curved, lightblue	2	PCS	5707292613332
	6175407	Side cushion, large, curved	2	PCS	5707292389237
	6175409	Side cushion, small, curved	2	PCS	5707292389251
	6075408	Side cushion, medium, curved, lightblue	2	PCS	5707292613349
	6175408	Side cushion, medium, curved, grey	2	PCS	5707292389244
	6075409	Side cushion, small, curved, light blue	2	PCS	5707292613356
	6175403	Side cushion moon shaped	2	PCS	5707292381132
	6175402	Side cushion, square	2	PCS	5707292381125
6175401	Side cushion, long, straight	2	PCS	5707292381118	
190*	82569	Bar,straight,stainless, 9,5cm	2	PCS	5707292124869
	82648	Bar, straight,stainless,16cm	2	PCS	5707292389275
	81968	Bar, curved, 25 mm, l:12 cm	2	PCS	5707292120366
191	82570	Plastic spacer for Flamingo	2	PCS	5707292124876
192	82558	Nut 6x16	2	PCS	5707292124753
194*	0609730	Stainless insex 6*20 uh.	4	PCS	5707292138507
	0609727	Stainless insex 6x12 uh	4	PCS	5707292138484
195	0659200	Buttonhead screw M6*12 stainl.	2	PCS	5707292139436
196+198	5001009	Screws etc. For side supports	1	PCS	5707292154330
400*	137341	Lay for pot 1	1	PCS	5707292837240
	137345	Lay for pot 2	1	PCS	5707292837226

\* choose size/color

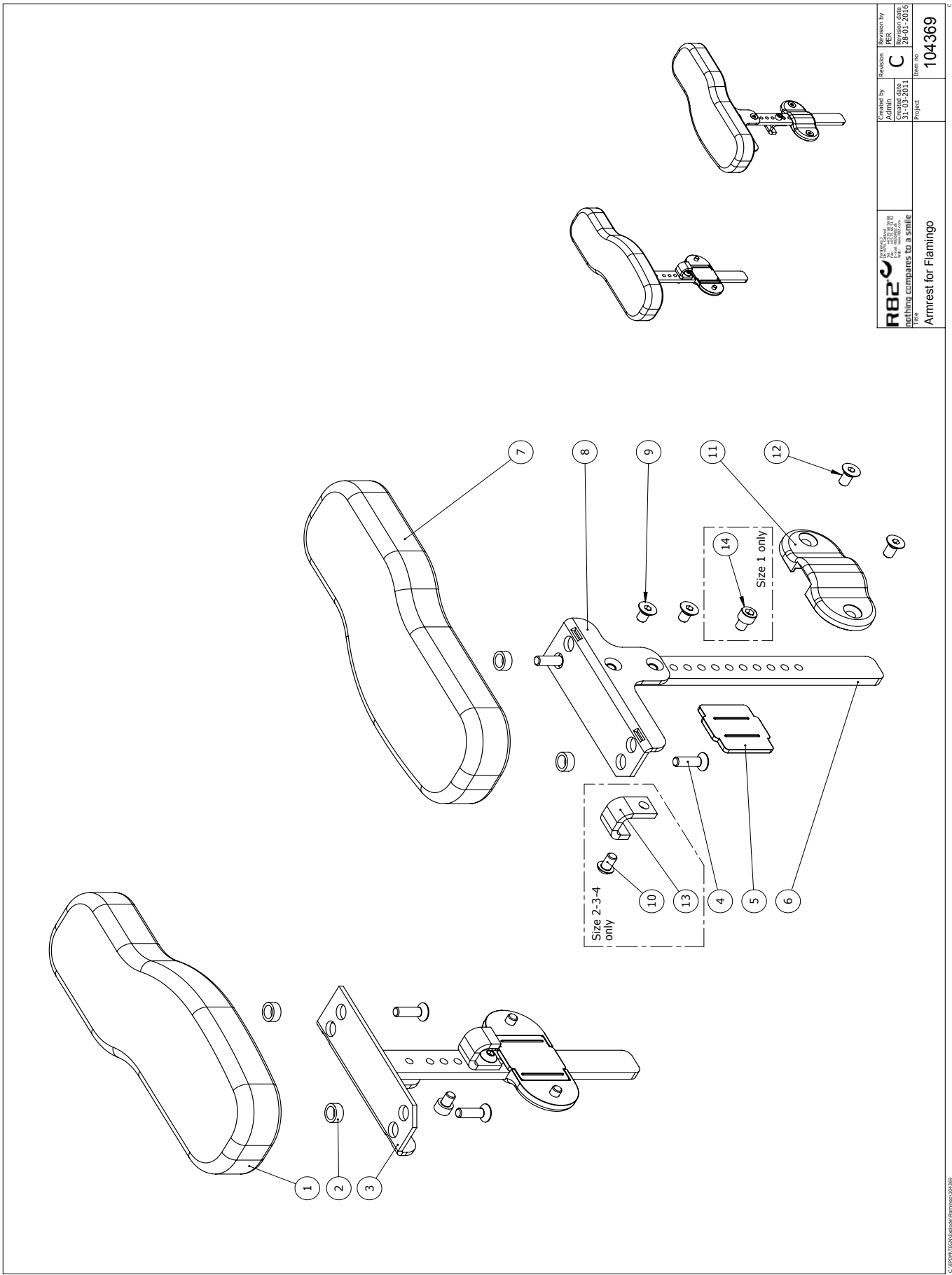

 R82	Checked by Admin	Revision by PER
	Created date 31-03-2011	Revision date 28-01-2016
Title Armrest for Flamingo		Item no. 104369

Position	Art. No.	Description	Unit of Measure	Quantity	Ean code
1	6175150	Armrest cushion, size 1, right	PCS	1	5707292189226
2	1310539	Distance piece 12/8,4*5mm	PCS	4	5707292423474
3	104360	Armrest bracket, right	PCS	1	5707292570703
4	0609730	Stainless insex 6*20 uh.	PCS	4	5707292138507
6	117054	Amrest bar	PCS	2	5707292611383
7	6175152	Armrest cushion, size 1, left	PCS	1	5707292189233
8	104359	Armrest bracket, left	PCS	1	5707292570697
9	0609760	Rustfri insex 6*10 uh. tuflok	PCS	4	5707292570796
10	0659005	Insex button head M6*10, stainl	PCS	2	5707292423740
13	117055	Brace	PCS	2	5707292611390
14	0602479	Insex M6*8, stainless	PCS	2	5707292465412
5+11+12	5001013	Plastic bag enclosing small	SET	1	5707292154378

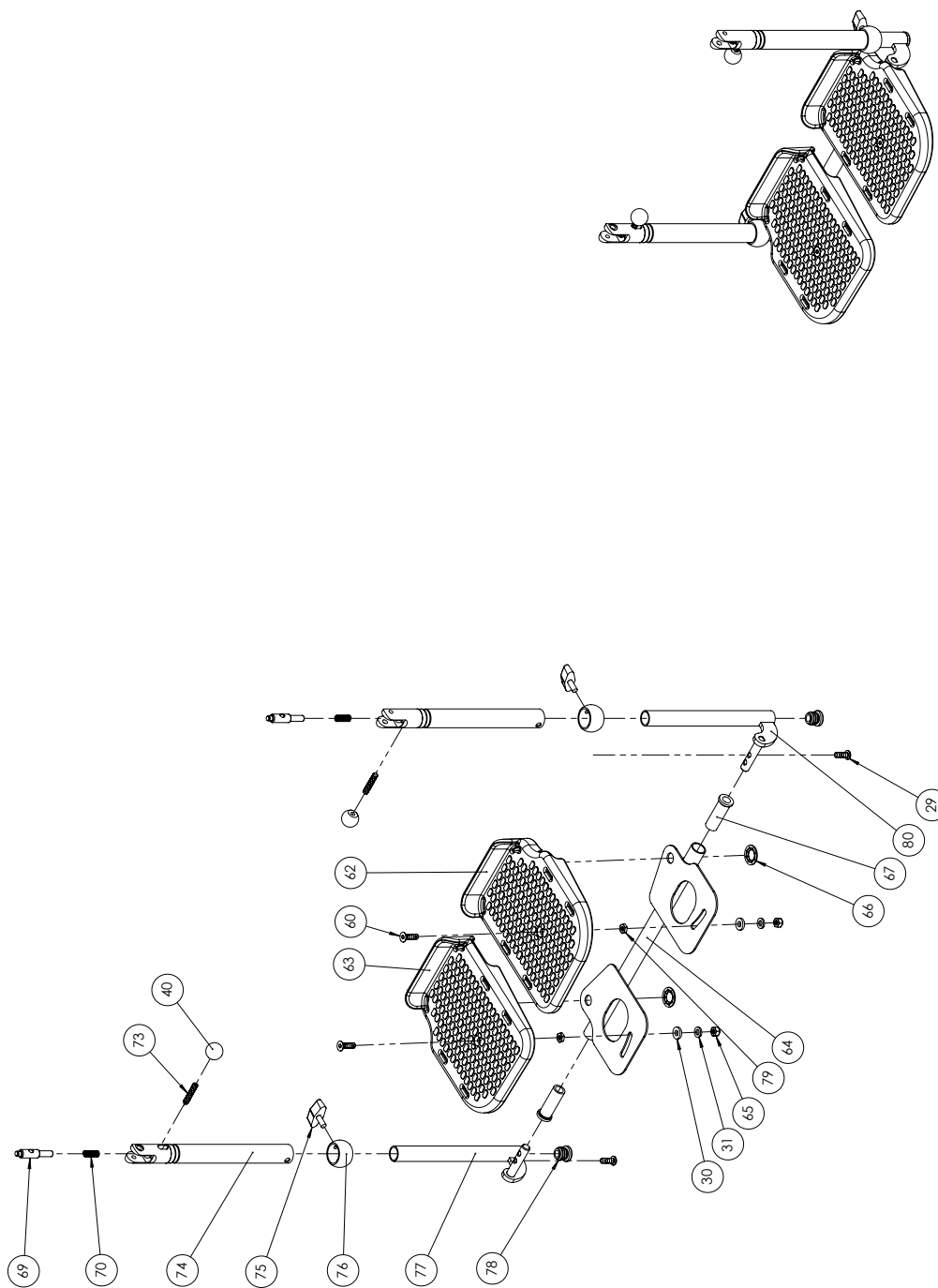


 R82	Approved by Address Created date Project	Revision for C Created date 31-03-2011 Project	Revision by for C Revision date 08-10-2014 Item no 79724
	Title Footrest for Flamingo		

C:\PDA\1004\1004\Footer\Flamingo\79724

Position	No.	Description	Quantity	Unit of Measure	Ean code
30	1310717	NYLON WASHER, NAT O17-8,4*2MM	2	PCS	5707292140821
40	3502008	KNOB HANDLE Ø20 M6	2	PCS	5707292146830
60	0609732	INSEX 6*22, STAINLESS	2	PCS	5707292409669
62*	71207	FOOTPLATE LEFT, FLAM. SZ 1-2	1	PCS	5707292104625
	71209	FOOTPLATE LEFT, FLAM. SZ 3-4	1	PCS	5707292104649
63*	71208	FOOTPLATE RIGHT, FLAM. SZ 1-2	1	PCS	5707292104632
	71210	FOOTPLATE RIGHT, FLAM. SZ 3-4	1	PCS	5707292104656
64*	71282	FOOTPLATE INLAY FLAM. SIZE 1	1	PCS	5707292104816
	71283	FOOTPLATE INLAY FLAM. SIZE 2	1	PCS	5707292104823
	71284	FOOTPLATE INLAY FLAM. SIZE 3	1	PCS	5707292104830
	71285	FOOTPLATE INLAY FLAM. SIZE 4	1	PCS	5707292104847
65	1631060	STAINLESS LOCK NUT 6MM	2	PCS	5707292141446
66	1109000	STARLOCK SKIVE M12, U/KAP, RF.	2	PCS	5707292140432
67	71119	FICTIONBUSH FOR FLAMINGO	2	PCS	5707292104267
69	71123	LOCK RATCHET	2	PCS	5707292104304
70	0310147	STAINLESS SPRING1*9,3*20MM*7,5	2	PCS	5707292135636
73	0806030	STAINLESS STOP SCREW 6*30MM	2	PCS	5707292140029
74*	71242	TELESCOPIC PIPE SZ. 1	2	PCS	5707292104687
	71243	TELESCOPIC PIPE SZ. 2-4	2	PCS	5707292104694
75	3503210	FINGER SCREW,WHITE M6X6	2	PCS	5707292147004
76	71118	BUSHING Ø22-18 FLAM.	2	PCS	5707292104250
77*	71272	TELESCOPE EXTENSION LEFT, SZ 1	1	PCS	5707292104755
	71273	TELESCOPE ARM LEFT 2	1	PCS	5707292104762
	71274	TELESCOPE EXT. LEFT, SZ 3+4	1	PCS	5707292104779
78	2502005	FERRULE D18*1,0-2,0 INT.WHITE	2	PCS	5707292144010
79	1750600	LOCK NUT, STAINLESS M6	2	PCS	5707292141477
80*	71277	TELESCOPE EXTENSION RIGHT	1	PCS	5707292104786
	71278	TELESCOPE EXTENSION RIGHT,SZ 2	1	PCS	5707292104793
	71279	TELESCOPE EXTENSION RIGHT,SZ3	1	PCS	5707292104809

\*Choose size/color/type

Pos no.	Qty.	Item no.	Description.
145	1	5982560-R	Svingbar sidestøtte højre
146	1	5982560-L	Svingbar sidestøtte venstre
187	2	2502519	DÆKHÆTTE Spec
189	2	6175408	Sidepelot mellem buet grå
190	2	155641	Bar, curved, 25 mm, l:10,5 cm
191	2	82570	Mellemlæg svb.sidest.flam.hvid
192	2	82558	M6 møtrik
194	4	0609730	INSEX UNDERS. M6x20
195	2	0659200	BUTTON HEAD M6x12
196	4	1105060	Rustfri skive 6,4*12*1,6 (a4)
198	4	0659200	Button head M6*12, rustfri

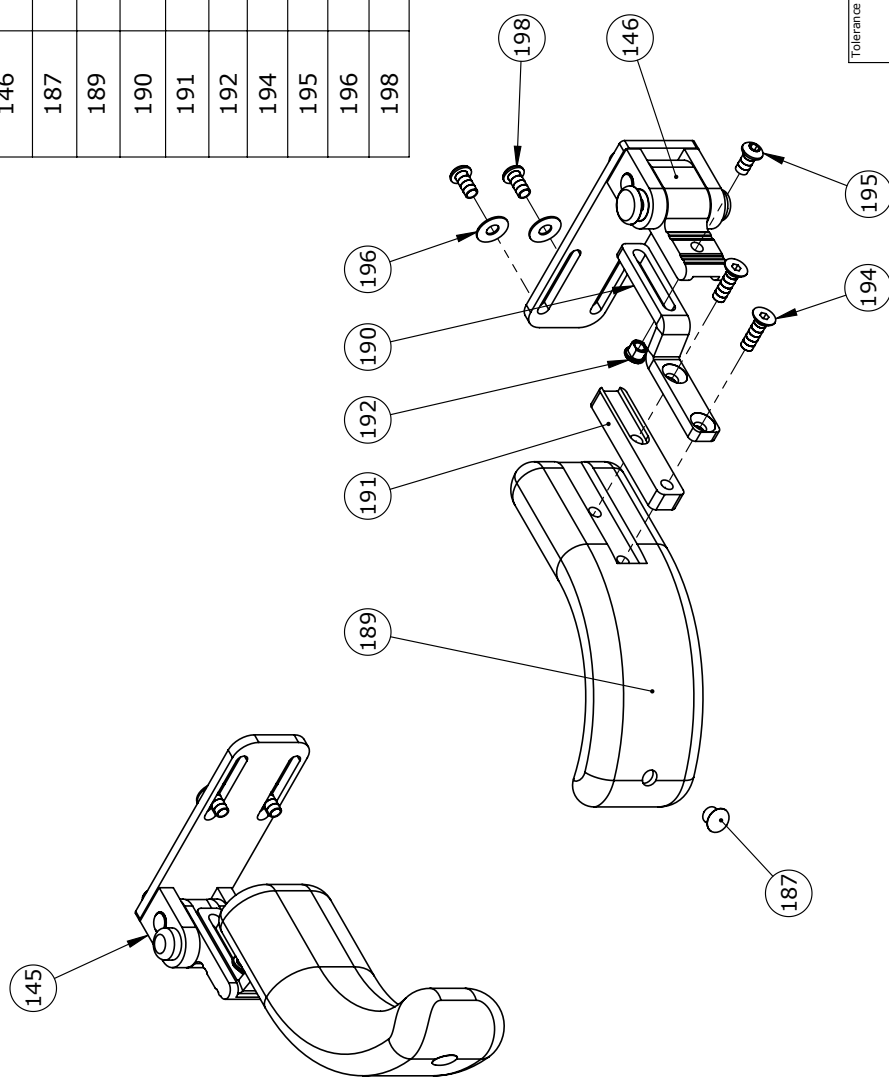

Tolerance	Project		R82 nothing compares to a smile	
Surface	117241	Created by	PER	Revision
Material		Created date	PER	Revision by
		11-02-2021	4	Revision date
		Scale	1:3	08-04-2021
		Weight	425.299g	
Title	Side Support Swing away Flamingo		Format	A4
			Item no	155650

C:\EPDM\TEGNET\explode\flamingo\155650