

High-low:xo

Exploded views / Parts list

Rev. date (2023.12)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.




Contents

Size 1-3, Indoor Gas spring	4
Item numbers for High-low:xo size 1-3, Indoor Gas spring.....	5
Size 1, Indoor Power	7
Item numbers for High-low:xo Size 1, Indoor Power.....	8
Size 1, Indoor Power- Central Brake	10
Item numbers for High-low:xo Size 1, Indoor Power- Central Brake.....	11
Size 2, Indoor Power	12
Item numbers for High-low:xo Size 2, Indoor Power.....	13
Size 2, Indoor Gas	15
Item numbers for High-low:xo Size 2, Indoor Gas	16
Size 2, Indoor Hydraulics.....	18
Item numbers for High-low:xo Size 2, Indoor Hydraulics	19
Size 3, Indoor Hydraulics.....	20
Item numbers for High-low:xo Size 3, Indoor Hydraulics	21
Size 3, Indoor Power	23
Item numbers for High-low:xo Size 3, Indoor Power	24
Size 1, Outdoor Gas Spring	26
Item numbers for High-low:xo size 1, Outdoor Gas Spring.....	27
Size 1, Outdoor Power	29
Item numbers for High-low:xo size 1, Outdoor Power	30
Size 2, Outdoor Gas	32
Item numbers for High-low:xo size 2, Outdoor Gas.....	33
Size 2, Outdoor Power	35
Item numbers for High-low:xo size 2, Outdoor Power	36
Foot rest.....	38
Item numbers for High-low:xo Foot rest.....	39
Outdoor Push Brace.....	41
Item numbers for High-low:xo Outdoor Push Brace.....	42
Anti Tip Indoor	43
Item numbers for High-low:xo Anti Tip Indoor.....	44
Anti Tip Indoor - for Hydraulic size 2 & 3	45
Item numbers for High-low:xo Anti Tip Indoor- for Hydraulic size 2 & 3.....	46

Anti Tip Outdoor.....	47
Item numbers for High-low:xo Anti Tip Outdoor.....	48
Threshold Assistance.....	49
Item numbers for High-low:xo Threshold Assistance	50
Lock of Lift Gas	51
Item numbers for High-low:xo Lock of Lift Gas.....	52
Transport Bracket.....	53
Item numbers for High-low:xo Transport Bracket.....	54
Support Springs	55
Item numbers for High-low:xo Support Springs.....	56
Communication Aid Assembly.....	57
Item numbers for High-low:xo Communication Aid Assembly	58
Bracket for Ekstra High Foot Support.....	59
Item numbers for High-low:xo Bracket for Ekstra High Foot Support	60
Foot operated central brake kit, size 2 - 3, gas and power (890012-2)	61
Item numbers for Foot operated central brake kit, size 2 - 3, gas and power (890012-2)	61
Foot operated central brake kit, size 3, hydraulic (890012-3).....	62
Item numbers for Foot operated central brake kit, size 3, hydraulic (890012-3)	62
Adaptor for x:panda shape size 3, kit 905090	63
Item numbers for Adaptor for x:panda shape size 3, kit 905090	63

Size 1-3, Indoor Gas spring

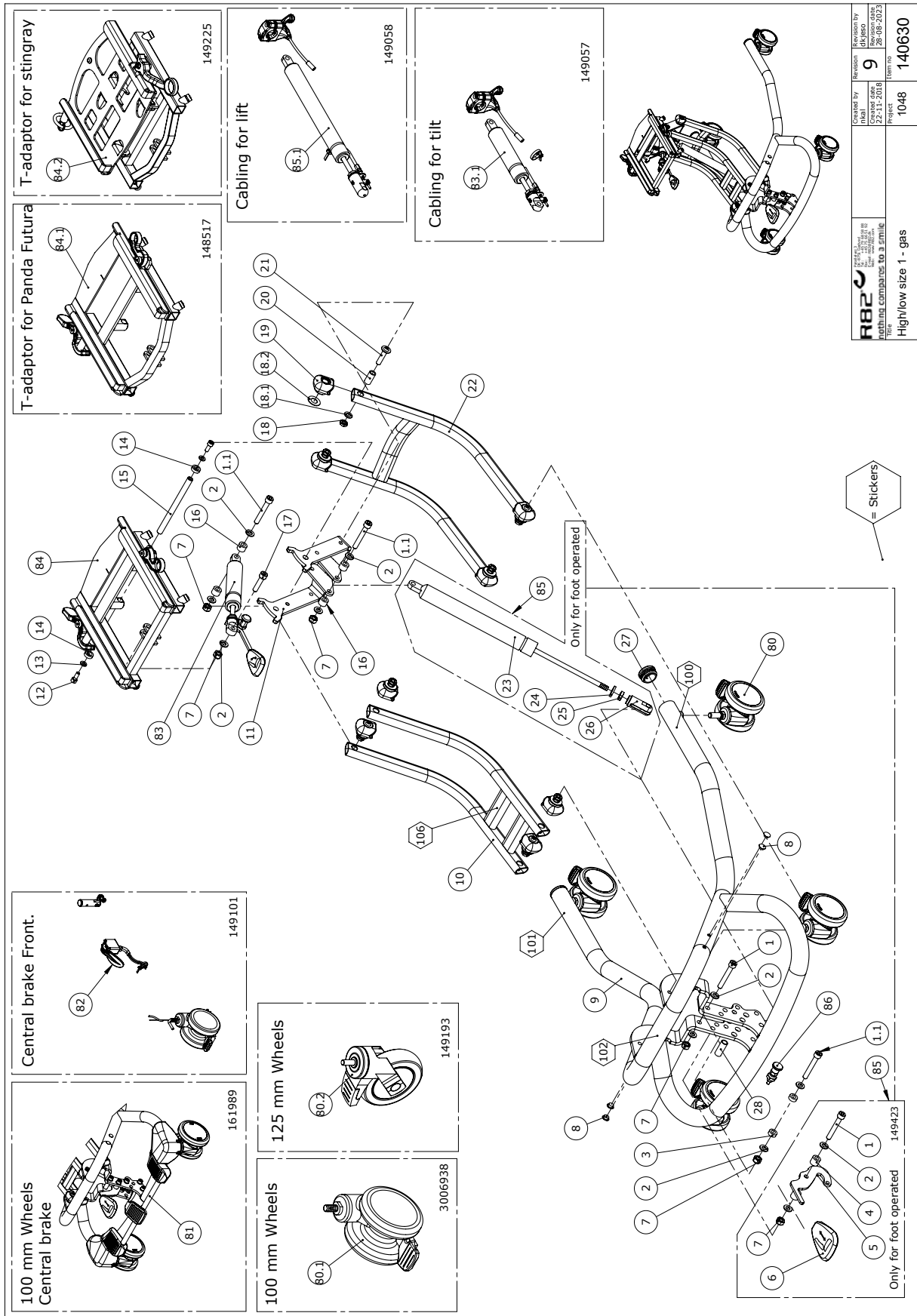

Created by	Revision	Revision by
22-11-2018	9	26-08-2023
Checked date	Item no	Project
Nothing compared to a similar	1048	140630
Title		
High/low size 1 - gas		


Item numbers for High-low:xo size 1-3, Indoor Gas spring

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0603600	Insex screw 8x55	1	PCS	5707292138262
1.1	0615011	Insex M8x55 fzb shortened	3	PCS	5707292928023
2	1103200	Fazet washer 8x16x1,5mm, FZB	9	PCS	5707292140296
3	1350000	Distancepart black 6x15xx8,2 mm	2	PCS	5707292140876
4	1350023	Distance piece, black	2	PCS	5707292140913
5	152830	Lever arm low	1	PCS	5707292924513
6	142199-1	Release Pedal	1	PCS	5707292910783
7	1607000	Lock nut 8mm, large pack	5	PCS	5707292141378
8	1310508	DR. fastener - afd. prop	4	PCS	5707292829795
9*	134813-04	Complete frame - size 1 (Black)	1	PCS	5707292895967
	134813-10	Complete frame - size 1 (White)	1	PCS	5707292908476
	134813-35	Complete frame - size 1 (Antrazit)	1	PCS	5707292908469
10*	135749-04	Upper arm - size 1/2 (Black)	1	PCS	5707292883742
	135749-10	Upper arm - size 1/2 (White)	1	PCS	5707292883704
	135749-35	Upper arm - size 1/2 (Antrazit)	1	PCS	5707292908452
11	140633-04	Tip brace High-low size 1-2 (Black)	1	PCS	5707292907479
12	0602200	Insex screw 6x16	2	PCS	5707292138095
13	1105060	SS washer M6 6,4x12x1,6	2	PCS	5707292140357
14	1350040	Distancepart black 15x5x10,5mm	2	PCS	5707292911162
15	140826	Shaft - adaptor	1	PCS	5707292907851
16	1350021	Distance piece 11x15x8,2mm	4	PCS	5707292140890
17	0615010	Insex M8x30 fzb, shortened	1	PCS	5707292367990
18	1750800	Lock nut, stainless M8	8	PCS	5707292141484
18.1	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
18.2	6108191	Plastic washer - black	8	PCS	5707292986368
19	136960	Endbush for arm	8	PCS	5707292907684
20	135399	Bushing	8	PCS	5707292913142
21	0663037	Pan Head black zink M8x35	8	PCS	5707292983367
22*	135738-04	Lower arm - size 1/2, Black	1	PCS	5707292883759
	135738-10	Lower arm - size 1/2 White	1	PCS	5707292906915
	135738-35	Lower arm - size 1/2, Antrazit	1	PCS	5707292906922
23	2205410-2	Bloc-o-Lift 1000 N	1	PCS	5707292927620
24	1310712	Nylon washer, nature	1	PCS	5707292356345
25	1761000	Counter nut 10x1,0 fine thread	1	PCS	5707292141538
26	6118822	Horizontal release	1	PCS	5707292186744
27	3501860	Dupsko DPD32	2	PCS	5707292840097
28	140344	Distance bushing	1	PCS	5707292911261
80	3006825	75mm wheel w/brake, black	4	PCS	5707292371393
80.1	3006938	100mm double wheel with brake	4	PCS	5707292574930
80.2	149193	125 mm Wheel with bolt+washer	4	PCS	5707292911193
81	161989	High/low size 1, central brake	1	PCS	5707292944191
82	149101-HL1	Central brake front wheels HL1	1	SET	5707292911667
83	140402	Gas spring size 1+2 Tilt	1	PCS	5707292911711
83.1	149057	Gas spring size 1-2 tilt cable	1	PCS	5707292911728
84	146853-04	x:panda adapter size 1-2	1	PCS	5707292907936
84.1	148517-04	Panda Futura adapter 1-4	1	PCS	5707292907929
84.2	149225-04	Adaptor for Stingray	1	PCS	5707292911766

* choose color

Position	No.	Description	Quantity	Unit of Measure	Ean code
85	149423	Foot operated Gascylinder HL1	1	PCS	5707292912824
85.1	149058	Gas spring HL size1 Lift cable	1	PCS	5707292911735
86	150266	Lock for lift gas	1	PCS	5707292915368
100	0494613	High-low:xo - silver	1	PCS	5707292921604
101	0494614	R82 by Etac logo - Silver	1	PCS	5707292921611
102	0494615	Size 1 - Silver	1	PCS	5707292921628
106	0493437	Label "exclamation mark"	1	PCS	5707292136909
* choose color					

Size 1, Indoor Power

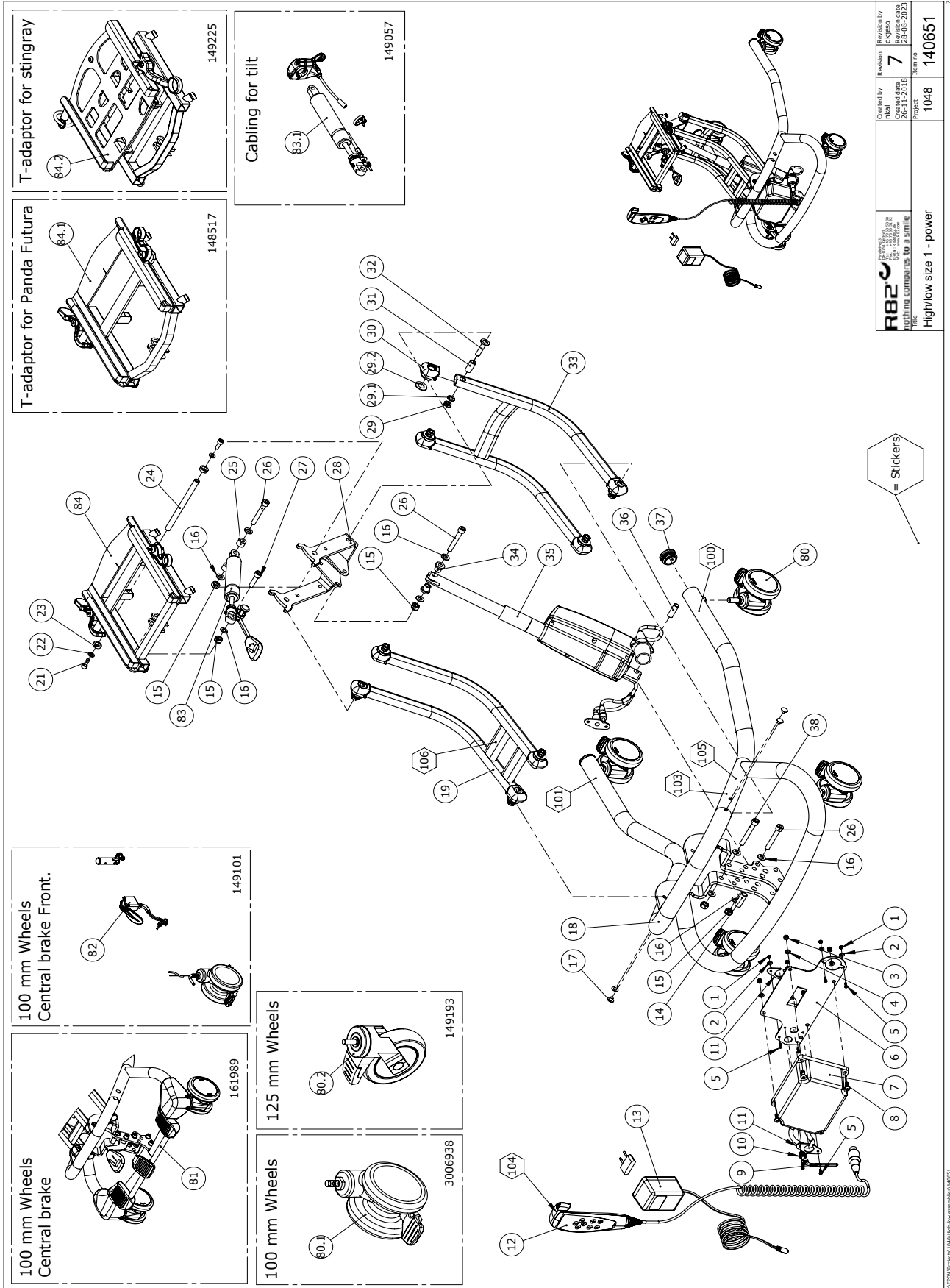


 R82 A/S Parallelvej 3 8751 Gedved Denmark	Created by Initial Date 26-11-2018	Revision dffler 7	Revision by Initial Date 28-08-2023
	Project 1048	Item no. 140651	High/low size 1 - power

Item numbers for High-low:xo Size 1, Indoor Power

Position	No.	Description	Quantity	Unit of measure	Ean code
1	1606250	Self-locking nut 3mm	6	PCS	5707292408396
2	1102800	Fazet washer 4mm	6	PCS	5707292140265
3	1606300	Lock nut 5 mm	4	PCS	5707292141354
4	1102902	Washer 5mm, A2	4	PCS	5707292411280
5	1902595	Crosshead screw M3x12, FZB	6	PCS	5707292565099
6	136851	Bracket batteri	1	PCS	5707292911216
7	2222110	Power unit ba18027	1	PCS	5707292492258
8	1902760	Cross-headed screw 5x16 mm	4	PCS	5707292142146
9	138882	Elastic band with clamp	1	PCS	5707292912879
10	2222112	Plug for battery box	1	PCS	5707292913241
11	138368	Reinforcement plate	2	PCS	5707292911223
12	2222089	Hand control HB30	1	PCS	5707292795489
13	2222161	Charger with plugs	1	PCS	5707292574046
14	140344	Distance bushing	1	PCS	5707292911261
15	1607000	Lock nut 8mm, large pack	5	PCS	5707292141378
16	1103200	Fazet washer 8x16x1,5mm, FZB	9	PCS	5707292140296
17	1310508	DR. fastener - afd. prop	4	PCS	5707292829795
18*	134813-04	Complete frame - size 1 (Black)	1	PCS	5707292895967
	134813-35	Complete frame - size 1 (Antrazit)	1	PCS	5707292908469
	134813-10	Complete frame - size 1 (White)	1	PCS	5707292908476
19*	135749-04	Upper arm - size 1/2 (Black)	1	PCS	5707292883742
	135749-35	Upper arm - size 1/2 (Antrazit)	1	PCS	5707292908452
	135749-10	Upper arm - size 1/2 (White)	1	PCS	5707292883704
21	0602200	Insex screw 6x16	2	PCS	5707292138095
22	1105060	SS washer M6 6,4x12x1,6	2	PCS	5707292140357
23	1350040	Distancepart black 15x5x10,5mm	2	PCS	5707292911162
24	140826	Shaft - adaptor	1	PCS	5707292907851
25	1350021	Distance piece 11x15x8,2mm	2	PCS	5707292140890
26	0615011	Insex M8x55 fzb shortened	3	PCS	5707292928023
27	0615010	Insex M8x30 fzb, shortened	1	PCS	5707292367990
28	140633-04	Tip brace High-low size 1-2 (Black)	1	PCS	5707292907479
29	1750800	Lock nut, stainless M8	8	PCS	5707292141484
29.1	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
29.2	6108191	Plastic washer - black	8	PCS	5707292986368
30	136960	Endbush for arm	8	PCS	5707292907684
31	135399	Bushing	8	PCS	5707292913142
32	0663037	Pan Head black zink M8x35	8	PCS	5707292983367
33*	135738-04	Lower arm - size 1/2, Black	1	PCS	5707292883759
	135738-35	Lower arm - size 1/2, Antrazit	1	PCS	5707292906922
	135738-10	Lower arm - size 1/2 White	1	PCS	5707292906915
34	82848	Bush	2	PCS	5707292125484
35	2222139	Actuator LA28 f/High&Low 1	1	PCS	5707292795472
36	P-6108200	Plast tube pa11 Ø10/Ø8mmx30mm	1	PCS	5707292585424
37	3501860	Dupsko DPD32	2	PCS	5707292840097
38	0603660	Insex screw 8x60	1	PCS	5707292138279
80	3006825	75mm wheel w/brake, black	4	PCS	5707292371393
80.1	3006938	100mm double wheel with brake	2	PCS	5707292574930
*choose color					

Position	No.	Description	Quantity	Unit of measure	Ean code
80.2	149193	125 mm Wheel with bolt+washer	4	PCS	5707292911193
81	161989	High/low size 1, central brake	1	SET	5707292944191
82	149101-HL1	Central brake front wheels HL1	1	SET	5707292911667
83	140402	Gas spring size 1+2 Tilt	1	PCS	5707292911711
83.1	149057	Gas spring size 1-2 tilt cable	1	PCS	5707292911728
84	146853-04	x:panda adapter size 1-2	1	PCS	5707292907936
84.1	148517-04	Panda Futura adapter 1-4	1	PCS	5707292907929
84.2	149225-04	Adaptor for Stingray	1	PCS	5707292911766
100	0494613	High-low:xo - silver	1	PCS	5707292921604
101	0494614	R82 by Etac logo - Silver	1	PCS	5707292921611
102	0494615	Size 1 - Silver	1	PCS	5707292921628
103	0494089	IP/BF sticker	1	PCS	5707292795809
104	0492119	Sticker handcontrol HLeI Womb	1	PCS	5707292552594
105	0494648	HLXO-WL-CA Conditions	1	PCS	5707292924490
106	0493437	Label "exclamation mark"	1	PCS	5707292136909
*choose color					

Size 1, Indoor Power- Central Brake

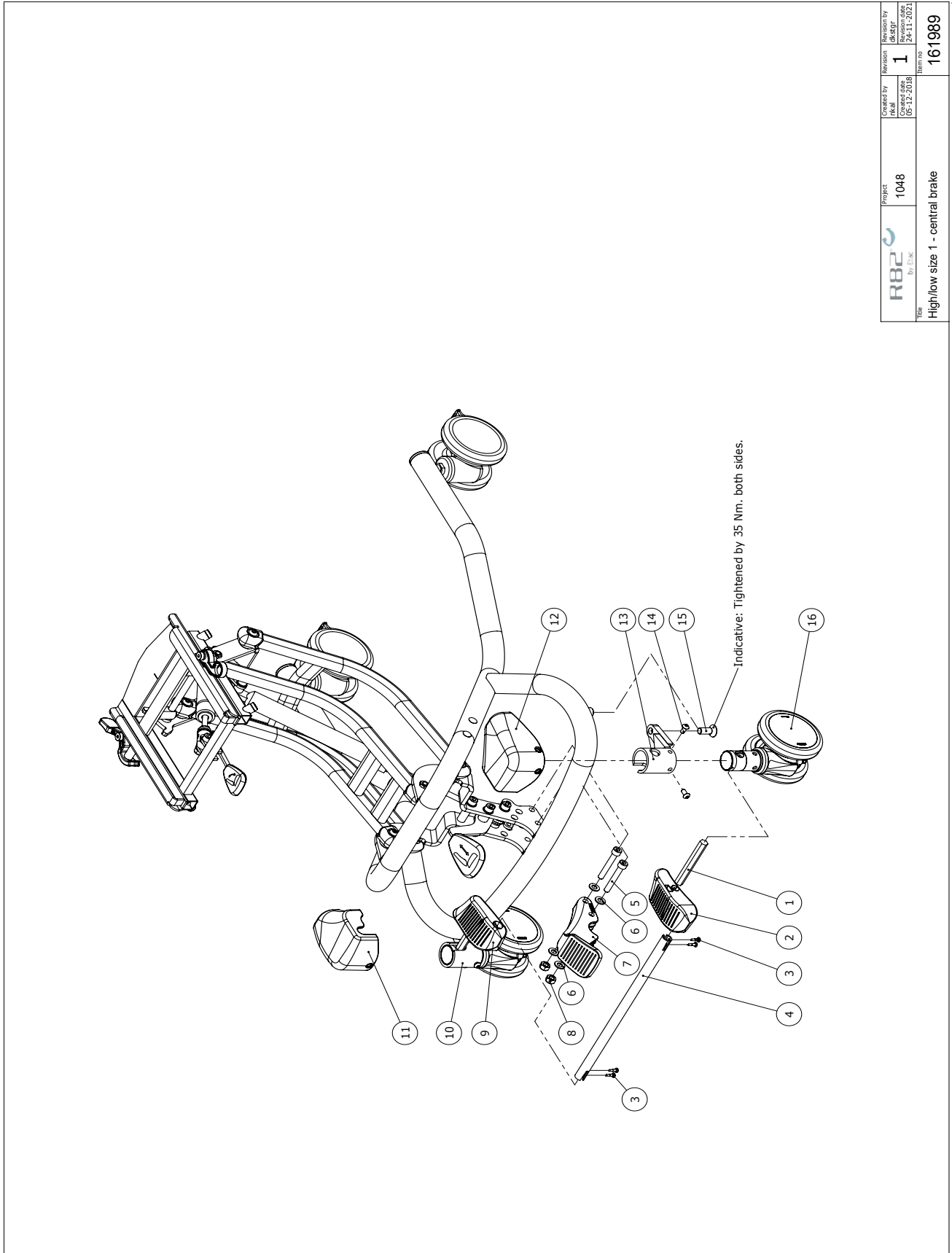


	Project	Revision by	Revision by
	1048	1	1
	Created date	Revision date	Item no
	05-12-2018	24-11-2021	161989
Title			
High/low size 1 - central brake			

Item numbers for High-low:xo Size 1, Indoor Power- Central Brake

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	138608	Hexagon bar CB size 1	2	PCS	5707292911308
2	139041	Pedal for central brake R	1	PCS	5707292910844
3	1811200	Sheet metal screw 3,5x13 phx	4	PCS	5707292141712
4	138840	Tube for central brake	1	PCS	5707292911322
5	0603600	Insex screw 8x55	2	PCS	5707292138262
6	1103200	Fazet washer 8x16x1,5mm, FZB	4	PCS	5707292140296
7	139326	Tip support	1	PCS	5707292910868
8	1607000	Lock nut 8mm, large pack	2	PCS	5707292141378
9	139067	Pedal for central brake L	1	PCS	5707292910851
10	161940	Bracket CB size 1 Left	1	PCS	5707292944177
11	139171	Str.1 Central brake cover L	1	PCS	5707292910813
12	139169	Str.1 central brake cover R	1	PCS	5707292910806
13	161787	Bracket CB size 1 right	1	PCS	5707292944184
14	0602193	Buttonhead - M6 x12, black	4	PCS	5707292900708
15	0612800	Insex screw counters. 10x25	2	PCS	5707292138774
16	3006970	100mm dob.wheel w/central br	2	PCS	5707292877468

Size 2, Indoor Power

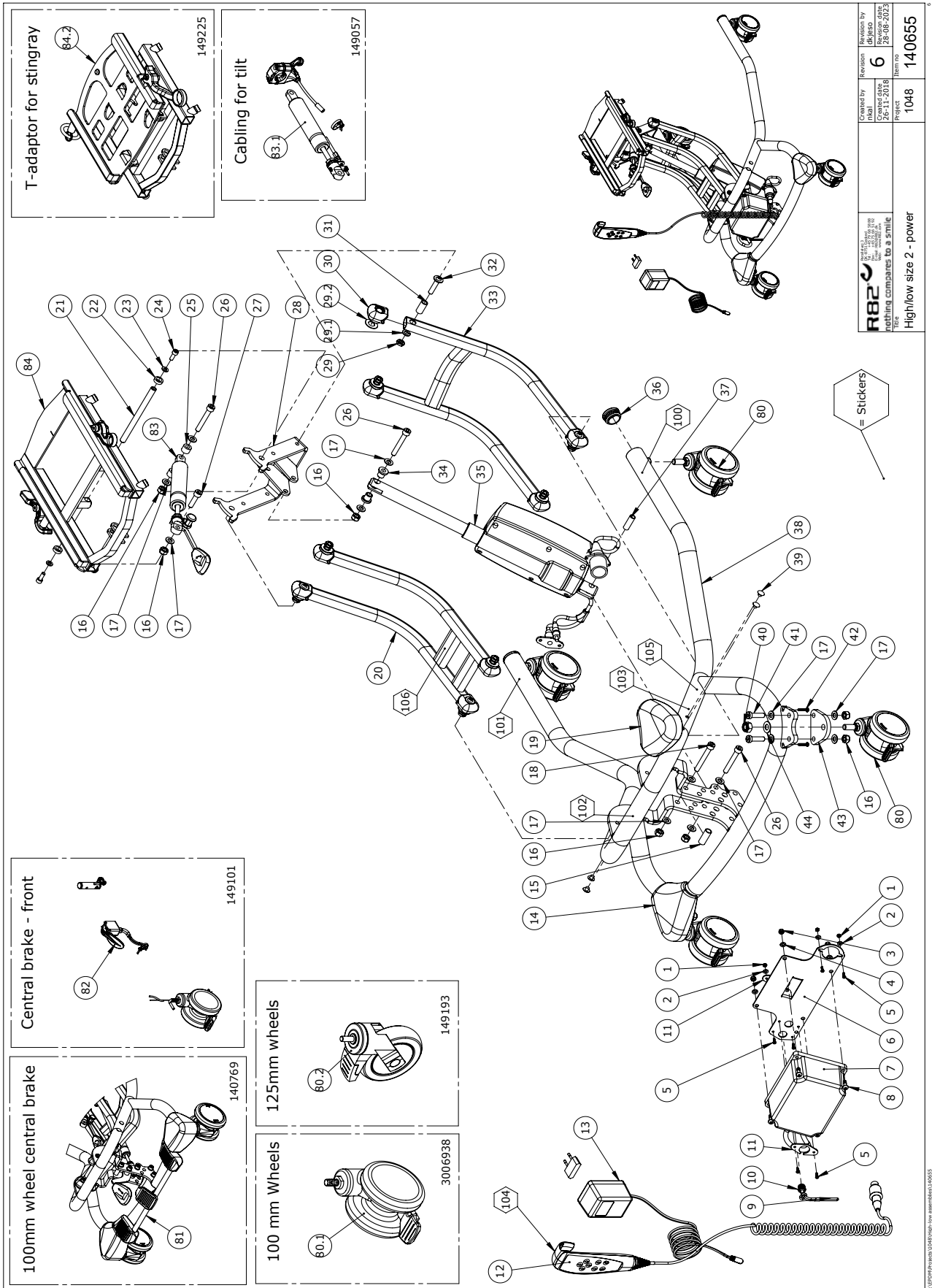

 nothing compares to a smile Title: High/low size 2 - power	Created by: JMS Date: 26-11-2018 Project: 1048	Revision by: JMS Date: 28-08-2023 Revision: 6 Item no: 140655
---	--	--

Item numbers for High-low:xo Size 2, Indoor Power


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1606250	Self-locking nut 3mm	6	PCS	5707292408396
2	1102800	Fazet washer 4mm	6	PCS	5707292140265
3	1606300	Lock nut 5 mm	4	PCS	5707292141354
4	1102902	Washer 5mm, A2	4	PCS	5707292411280
5	1902595	Crosshead screw M3*12, FZB	6	PCS	5707292565099
6	136851	Bracket batteri	1	PCS	5707292911216
7	2222110	Power unit ba18027	1	PCS	5707292492258
8	1902760	Cross-headed screw 5*16 mm	4	PCS	5707292142146
9	138882	Elastic band with clamp	1	PCS	5707292912879
10	2222112	Plug for battery box	1	PCS	5707292913241
11	138368	Reinforcement plate	2	PCS	5707292911223
12	2222089	Hand control HB30	1	PCS	5707292795489
13	2222161	Charger with plugs	1	PCS	5707292574046
14	139838	High/Low 2-3 standard cover L	1	PCS	5707292907677
15	140344	Distance bushing	1	PCS	5707292911261
16	1607000	Lock nut 8mm, large pack	9	PCS	5707292141378
17	1103200	Fazet washer 8*16*1,5mm, FZB	17	PCS	5707292140296
18	0603660	Insex screw 8*60	1	PCS	5707292138279
19	139826	High/Low 2-3 standard cover R	1	PCS	5707292907660
20	135749-04	Upper arm - size 1/2	1	PCS	5707292883742
21	140826	Shaft - adaptor	1	PCS	5707292907851
22	1350040	Distancepart black 15x5x10,5mm	2	PCS	5707292911162
23	1105060	SS washer M6 6,4*12*1,6	2	PCS	5707292140357
24	0602200	Insex screw 6*16	2	PCS	5707292138095
25	1350021	Distance piece 11*15*8,2mm	2	PCS	5707292140890
26	0615011	Insex M8*55 fzb shortened	3	PCS	5707292928023
27	0615010	Insex M8*30 fzb, shortened	1	PCS	5707292367990
28	140633-04	Tip brace High-low size 1-2	1	PCS	5707292907479
29	1750800	Lock nut, stainless M8	8	PCS	5707292141484
30	136960	Endbush for arm	8	PCS	5707292907684
31	135399	Bushing	8	PCS	5707292913142
32	0663037	Pan Head black zink M8x35	8	PCS	5707292983367
33	135738-04	Lower arm - size 1/2, Black	1	PCS	5707292883759
34	82848	Bush	2	PCS	5707292125484
34	82848	Bush	2	PCS	5707292125484
35	2222146	Actuator LA32 f/ High-Low 2	1	PCS	5707292903624
36	3501860	Dupsko DPD32	2	PCS	5707292840097
37	P-6108200	Plast tube pa11 Ø10/Ø8m- mx30mm	1	PCS	5707292585424
38	139825-04	Complete frame - size 2 and 3	1	PCS	5707292903419
39	1310508	DR. fastener - afd. prop	4	PCS	5707292829795
40	1607500	Lock nut 10 mm	2	PCS	5707292141385
41	0603536	Insex lov head M8x30	4	PCS	5707292575326
42	0610620	screw ø3,5x20 DELTA PT A2	4	PCS	5707292903778
43	139619	Adaptor for indoor wheels sz2	2	PCS	5707292911209
44	1102500	Planning washer 10 mm	2	PCS	5707292140234

* Choose size / color

Position	No.	Description	Quantity	Unit of Measure	Ean code
80	3006825	75mm wheel w/brake, black	4	PCS	5707292371393
80.1	3006938	100mm double wheel with brake	4	PCS	5707292574930
80.2	149193	125 mm Wheel with bolt+washer	4	PCS	5707292911193
81	140769	High/low size 2, central brake	1	PCS	5707292913050
82	149101-HL2+3	Central brake front wheels HL2	1	SET	5707292911674
83	140402	Gas spring size 1+2 Tilt	1	PCS	5707292911711
83.1	149057	Gas spring size 1-2 tilt cable	1	PCS	5707292911728
84	140907-04	x:panda adapter size 3	1	PCS	5707292907905
84.1	149225-04	Adaptor for Stingray	1	PCS	5707292911766
100*	0494613	High-low:xo - silver	1	PCS	5707292921604
	0494618	High-low:xo - black	1	PCS	5707292921987
101*	0494614	R82 by Etac logo - Silver	1	PCS	5707292921611
	0494619	R82 by Etac logo - Black	1	PCS	5707292921994
102*	0494616	Size 2 - Silver	1	PCS	5707292921635
	0494656	Str. mærke til high-low:xo	1	PCS	5707292930743
	0494616	Size 2 - Silver	1	PCS	5707292921635
103	0494089	IP/BF sticker	1	PCS	5707292795809
104	0492119	Sticker handcontrol HLeI Womb	1	PCS	5707292552594
105	0494648	HLXO-WL-CA Conditions	1	PCS	5707292924490
106	0493437	Label "exclamation mark"	1	PCS	5707292136909

* Choose size / color

Size 2, Indoor Gas

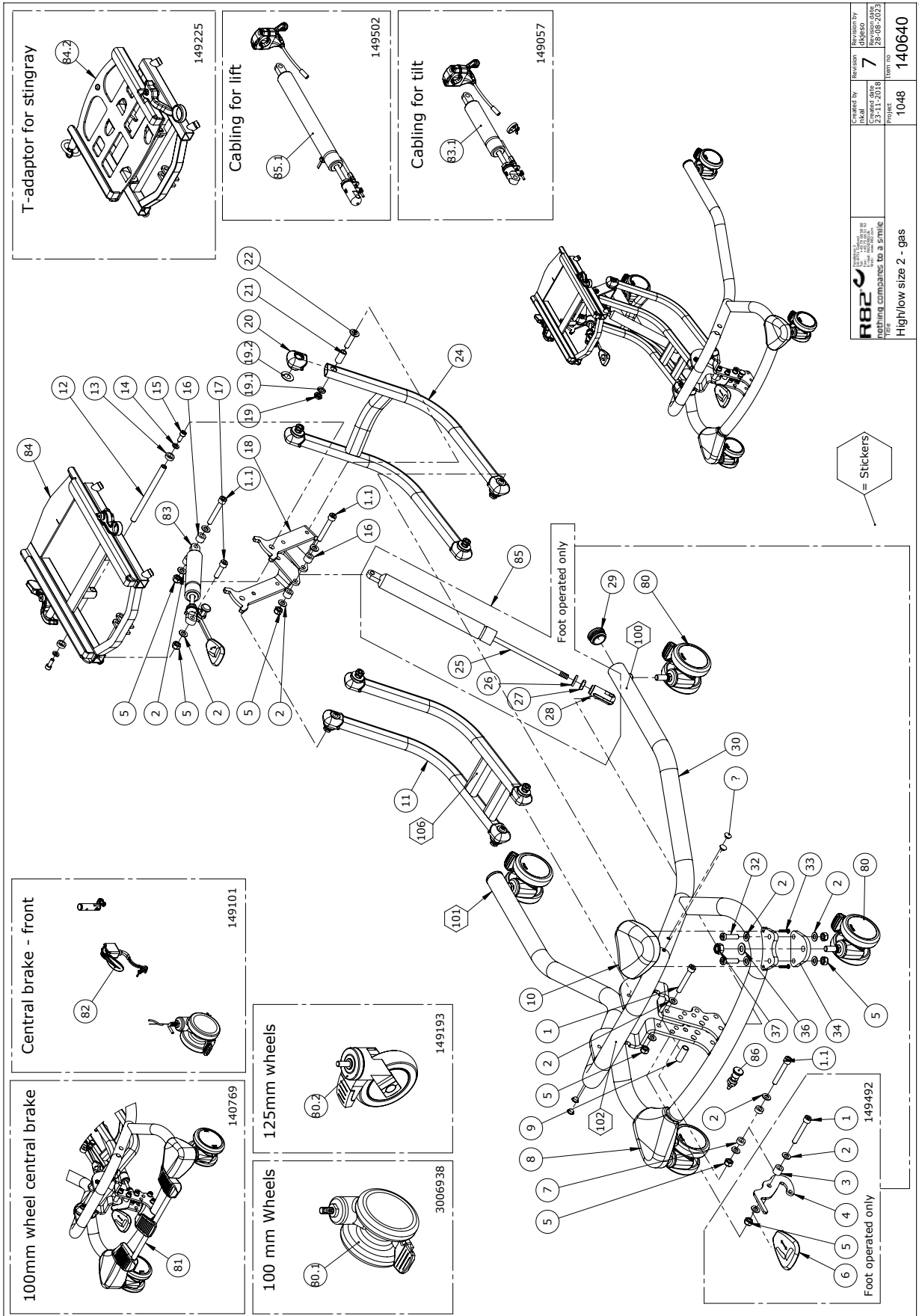


<p>R82 Elliptical Trainers Making companies to a smile Highlow size 2 - gas</p>	Created by	Revised by	Revised by
	Initial	Initial	dlj/aso
	Created date	Revised date	Revised date
	22-11-2018	28-08-2023	
	Page no.	Form no.	
	1048	1400640	

Item numbers for High-low:xo Size 2, Indoor Gas

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0603600	Insex screw 8x55	1	PCS	5707292138262
1.1	0603610	Insex M8x55, 12.9 flake zinc	3	PCS	5707292517630
2	1103200	Fazet washer 8x16x1,5mm, FZB	17	PCS	5707292140296
3	1350023	Distance piece, black	2	PCS	5707292140913
4	152830	Lever arm low	1	PCS	5707292924513
5	1607000	Lock nut 8mm, large pack	9	PCS	5707292141378
6	142199-1	Release Pedal	1	PCS	5707292910783
7	1350000	Distancepart black 6x15x8,2 mm	2	PCS	5707292140876
8	139838	High/Low 2-3 standard cover L	1	PCS	5707292907677
9	140344	Distance bushing	1	PCS	5707292911261
10	139826	High/Low 2-3 standard cover R	1	PCS	5707292907660
11	135749-04	Upper arm - size 1/2	1	PCS	5707292883742
12	140826	Shaft - adaptor	1	PCS	5707292907851
13	1350040	Distancepart black 15x5x10,5mm	2	PCS	5707292911162
14	1105060	SS washer M6 6,4x12x1,6	2	PCS	5707292140357
15	0602200	Insex screw 6x16	2	PCS	5707292138095
16	1350021	Distance piece 11x15x8,2mm	4	PCS	5707292140890
17	0615010	Insex M8x30 fzb, shortened	1	PCS	5707292367990
18	140633-04	Tip brace High-low size 1-2	1	PCS	5707292907479
19	1750800	Lock nut, stainless M8	8	PCS	5707292141484
19.1	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
19.2	6108191	Plastic washer - black	8	PCS	5707292986368
20	136960	Endbush for arm	8	PCS	5707292907684
21	135399	Bushing	8	PCS	5707292913142
22	0663037	Pan Head black zink M8x35	8	PCS	5707292983367
24	135738-04	Lower arm - size 1/2, Black	1	PCS	5707292883759
25	2205420	Bloc-o-Lift 1400 N	1	PCS	5707292912770
26	1310712	Nylon washer, nature	1	PCS	5707292356345
27	1761000	Counter nut 10x1,0 fine thread	1	PCS	5707292141538
28	6118822	Horizontal release	1	PCS	5707292186744
29	3501860	Dupsko DPD32	2	PCS	5707292840097
30	139825-04	Complete frame - size 2	1	PCS	5707292903419
31	1310508	DR. fastener - afd. prop	4	PCS	5707292829795
32	0603536	Insex lov head M8x30	4	PCS	5707292575326
33	0610620	Skrue ø3,5 x 20 Delta PT, A2	4	PCS	5707292903778
34	139619	Adaptor for indoor wheels sz2	2	PCS	5707292911209
36	1102500	Planning washer 10 mm	2	PCS	5707292140234
37	1607500	Lock nut 10 mm	2	PCS	5707292141385
80	3006825	75mm wheel w/brake, black	4	PCS	5707292371393
80.1	3006938	100mm double wheel with brake	4	PCS	5707292574930
80.2	149193	125 mm Wheel with bolt+washer	4	PCS	5707292911193
81	140769	High/low size 2, central brake	1	PCS	5707292913050
82	149101-HL2+3	Central brake front wheels HL2	1	SET	5707292911674
83	140402	Gas spring size 1+2 Tilt	1	PCS	5707292911711
83.1	149057	Gas spring size 1-2 tilt cable	1	PCS	5707292911728
84	140907-04	x:panda adapter size 3	1	PCS	5707292907905
85	149492	Foot operated Gascylinder HL2	1	PCS	5707292913920
86	150266	Lock for lift gas	1	PCS	5707292915368

100	0494613	High-low:xo - silver	1	PCS	5707292921604
101	0494614	R82 by Etac logo - Silver	1	PCS	5707292921611
102	0494616	Size 2 - Silver	1	PCS	5707292921635
106	0493437	Label "exclamation mark"	1	PCS	5707292136909

Size 2, Indoor Hydraulics

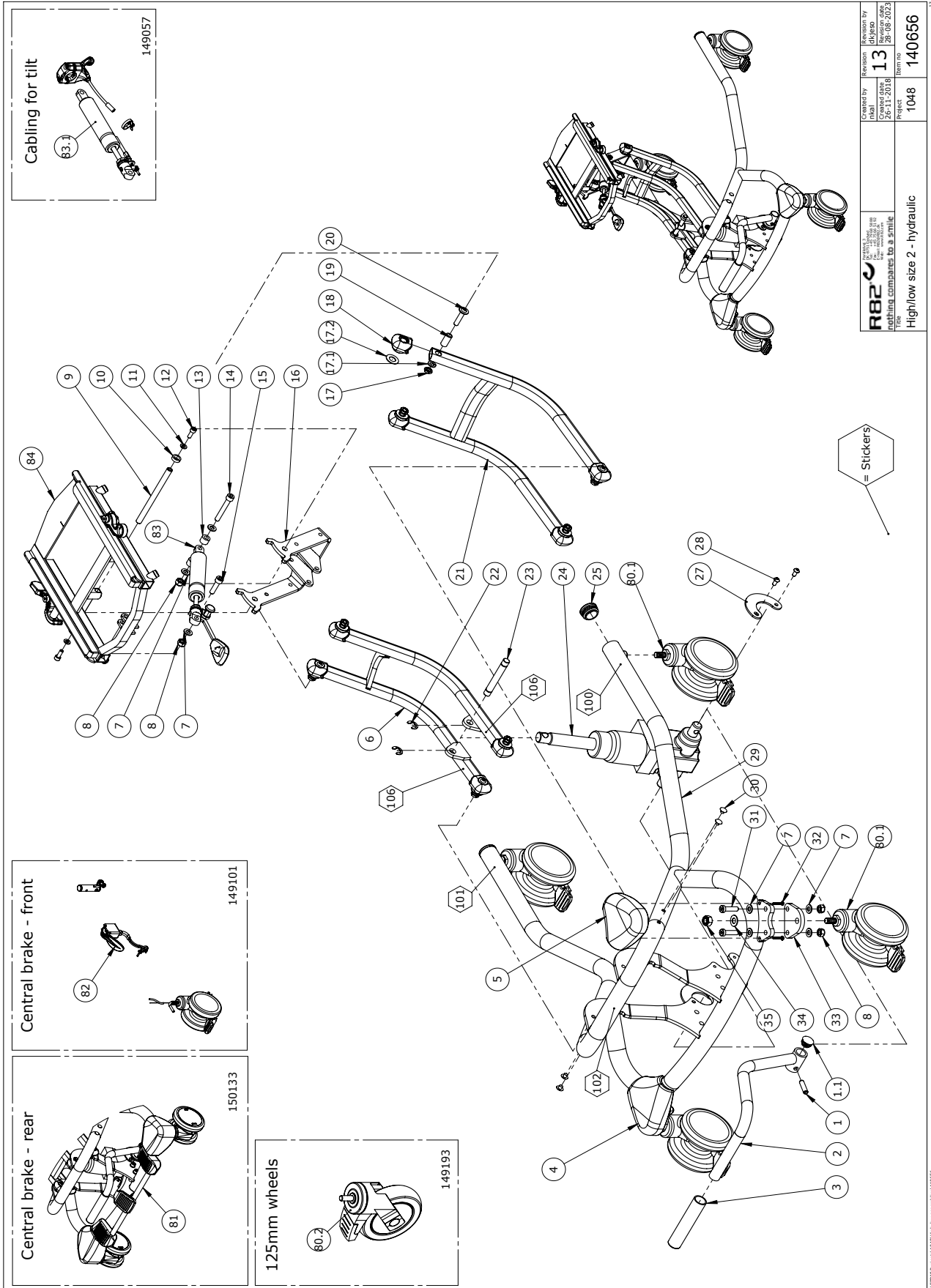

Cabling for tilt

149057

Central brake - front

149101

Central brake - rear

150133

125mm wheels


149193

<p>R82 Nothing compares to a smile</p>	Created by	Revision	Revision by
	Created date	13	13
	26-11-2018		26-11-2018
	Project	1048	Item no.
			140656
High/low size 2 - hydraulic			

Item numbers for High-low:xo Size 2, Indoor Hydraulics

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0584490	Stift din 1481 8x30 rf	1	PCS	5707292473356
1.1	2503010	Ferrule d22x1,8-3,5, black	1	PCS	5707292357632
2	139936-04	Pump lever	1	PCS	5707292908926
3	2500516	Grip 87.00 mm length	1	PCS	5707292356727
6	140421-04	Upper arm H/L sz1-2 Hydraulic	1	PCS	5707292903389
7	1103200	Fazet washer 8x16x1,5mm, FZB	3	PCS	5707292140296
8	1607000	Lock nut 8mm, large pack	2	PCS	5707292141378
9	140826	Shaft - adaptor	1	PCS	5707292907851
10	1350040	Distancepart black 15x5x10,5mm	2	PCS	5707292911162
11	1105060	SS washer M6 6,4x12x1,6	2	PCS	5707292140357
12	0602200	Insex screw 6x16	2	PCS	5707292138095
13	1350021	Distance piece 11x15x8,2mm	2	PCS	5707292140890
14	0615011	Insex M8x55 fzb shortened	1	PCS	5707292928023
15	0615010	Insex M8x30 fzb, shortened	1	PCS	5707292367990
16	140633-04	Tip brace High-low size 1-2	1	PCS	5707292907479
17	1750800	Lock nut, stainless M8	8	PCS	5707292141484
17.1	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
17.2	6108191	Plastic washer - black	8	PCS	5707292986368
18	136960	Endbush for arm	8	PCS	5707292907684
19	135399	Bushing	8	PCS	5707292913142
20	0663037	Pan Head black zink M8x35	8	PCS	5707292983367
21	135738-04	Lower arm - size 1/2, Black	1	PCS	5707292883759
22	1145095	lock ring for10mm,din 6799, rf	2	PCS	5707292473332
23	140387	Lock pin for hydraulics	1	PCS	5707292911285
24	2208000	Compact mk5 hydraulic pump	1	PCS	5707292461148
25	3501860	Dupsko DPD32	2	PCS	5707292840097
27	75615-04	Overlay	2	PCS	5707292914163
28	0641205	Button head screw 6x12	4	PCS	5707292597717
29	140417-04	Complete frame size2 hydraulic	1	PCS	5707292903433
30	1310508	DR. fastener - afd. prop	4	PCS	5707292829795
80.1	3006938	100mm double wheel with brake	4	PCS	5707292574930
80.2	149193	125 mm Wheel with bolt+washer	4	PCS	5707292911193
81	150133	High/low size 2-3 Hyd. CB	1	PCS	5707292914477
82	149101-HL2+3	Central brake front wheels HL2	1	SET	5707292911674
83.1	149057	Gas spring size 1-2 tilt cable	1	PCS	5707292911728
84	140907-04	x:panda adapter size 3	1	PCS	5707292907905
100	0494613	High-low:xo - silver	1	PCS	5707292921604
101	0494614	R82 by Etac logo - Silver	1	PCS	5707292921611
102	0494616	Size 2 - Silver	1	PCS	5707292921635
106	0493437	Label "exclamation mark"	2	PCS	5707292136909

Size 3, Indoor Hydraulics

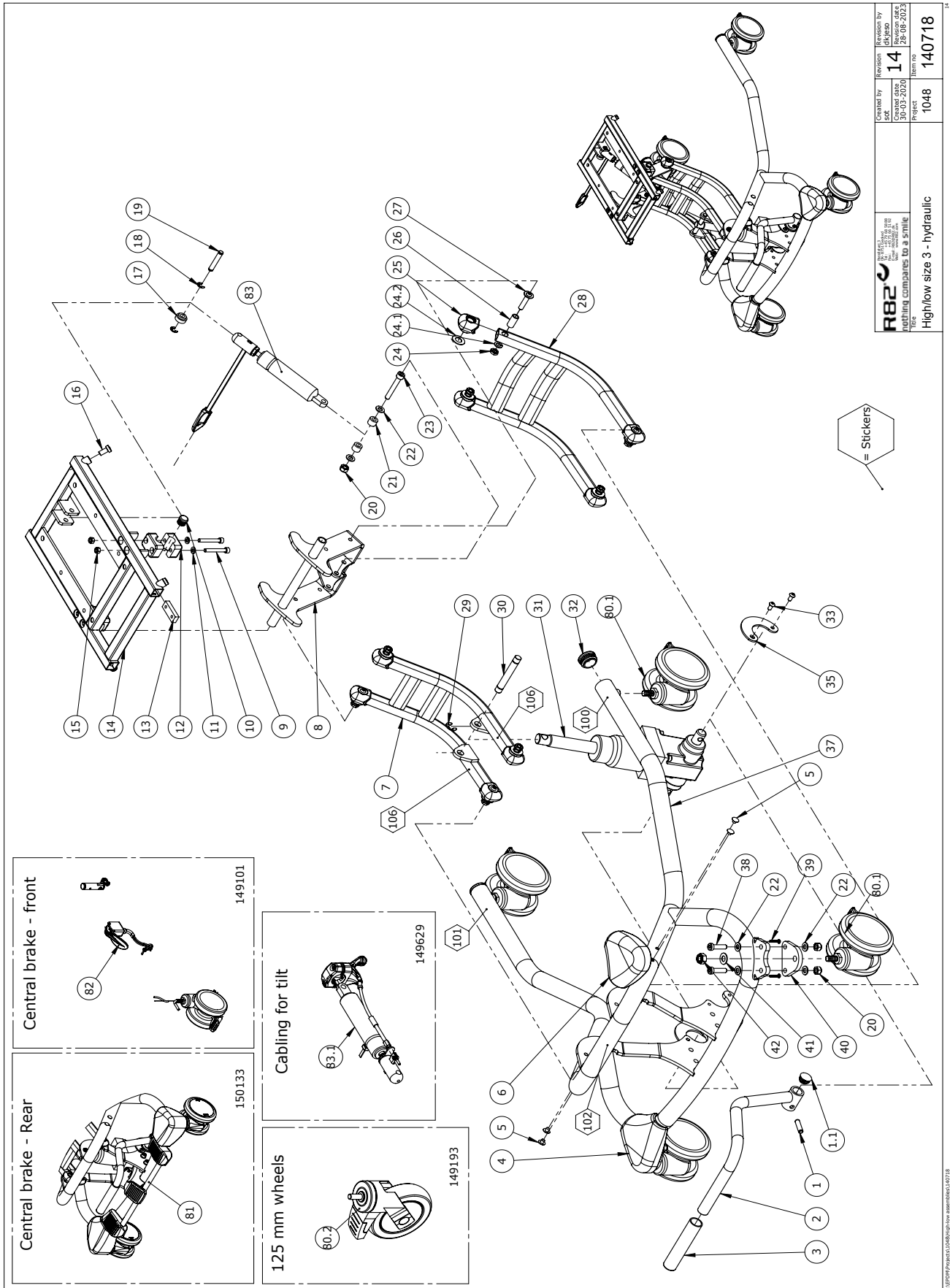


 R82 A/S Parallelvej 3 8751 Gedved Denmark	Revision by dby/psd 14	Revision 20-08-2025
	Create date 01-07-2021	Item no. 140718
Part no. 1048	High/low size 3 - hydraulic	

Item numbers for High-low:xo Size 3, Indoor Hydraulics

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0584490	Stift din 1481 8x30 rf	1	PCS	5707292473356
1.1	2503010	Ferrule d22x1,8-3,5, black	1	PCS	5707292357632
2	139936-04	Pump lever	1	PCS	5707292908926
3	2500516	Grip 87.00 mm length	1	PCS	5707292356727
4	139838	High/Low 2-3 standard cover L	1	PCS	5707292907677
5	1310508	DR. fastener - afd. prop	4	PCS	5707292829795
6	139826	High/Low 2-3 standard cover R	1	PCS	5707292907660
7	140424-04	Upper arm H/L sz3 Hydraulic	1	PCS	5707292903396
8	140667-04	Tip brace High-low size 3	1	PCS	5707292907486
9	0602445	Insex M6x45 FZB	4	PCS	5707292138156
10	2501600	Ferrule d15x1,0 black	2	PCS	5707292143969
11	1105060	SS washer M6 6,4x12x1,6	4	PCS	5707292140357
12	2521840	Pipe holder f/16mmsquare,black	4	PCS	5707292399236
13	77571	Fastening bracket	2	PCS	5707292489982
14	152248-04	Seat frame	1	PCS	5707292922588
15	1606500	Lock nut 6 mm	4	PCS	5707292141361
16	0603530	Insex low head M8x16 stainless	4	PCS	5707292412492
17	77574	Sleeve f. Strike	1	PCS	5707292490773
18	1145070	Lock washer 7mm	2	PCS	5707292140586
19	80921	locking pin, Wombat	1	PCS	5707292111371
20	1607000	Lock nut 8mm, large pack	5	PCS	5707292141378
21	1350021	Distance piece 11x15x8,2mm	2	PCS	5707292140890
22	1103200	Fazet washer 8x16x1,5mm, FZB	10	PCS	5707292140296
23	0615011	Insex M8x55 fzb shortened	1	PCS	5707292928023
24	1750800	Lock nut, stainless M8	8	PCS	5707292141484
24.1	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
24.2	6108191	Plastic washer - black	8	PCS	5707292986368
25	136960	Endbush for arm	8	PCS	5707292907684
26	135399	Bushing	8	PCS	5707292913142
27	0663037	Pan Head black zink M8x35	8	PCS	5707292983367
28	140126-04	Lower arm H/L sz3 Hydraulic	1	PCS	5707292903334
29	1145095	lock ring for10mm,din 6799, rf	2	PCS	5707292473332
30	140387	Lock pin for hydraulics	1	PCS	5707292911285
31	2208000	Compact mk5 hydraulic pump	1	PCS	5707292461148
32	3501860	Dupsko DPD32	2	PCS	5707292840097
33	0641205	Button head screw 6x12	4	PCS	5707292597717
35	75615-04	Overlay	2	PCS	5707292914163
37	140417-04	Complete frame size2 hydraulic	1	PCS	5707292903433
38	0603536	Insex lov head M8x30	4	PCS	5707292575326
39	0610620	screw ø3,5x20 DELTA PT A2	4	PCS	5707292903778
40	139619	Adaptor for indoor wheels sz2	2	PCS	5707292911209
41	1102500	Planning washer 10 mm	2	PCS	5707292140234
42	1607500	Lock nut 10 mm	2	PCS	5707292141385
80.1	3006938	100mm double wheel with brake	2	PCS	5707292574930
80.2	149193	125 mm Wheel with bolt+washer	4	PCS	5707292911193
81	150133	High/low size 2-3 Hyd. CB	1	PCS	5707292914477
82	149101-HL2+3	Central brake front wheels HL2	1	SET	5707292911674

83.1	149629	Gas spring size 3 tilt cable	1	PCS	5707292915016
100	0494613	High-low:xo - silver	1	PCS	5707292921604
101	0494614	R82 by Etac logo - Silver	1	PCS	5707292921611
102	0494617	Size 3 - Silver	1	PCS	5707292921642
106	0493437	Label "exclamation mark"	2	PCS	5707292136909

Size 3, Indoor Power

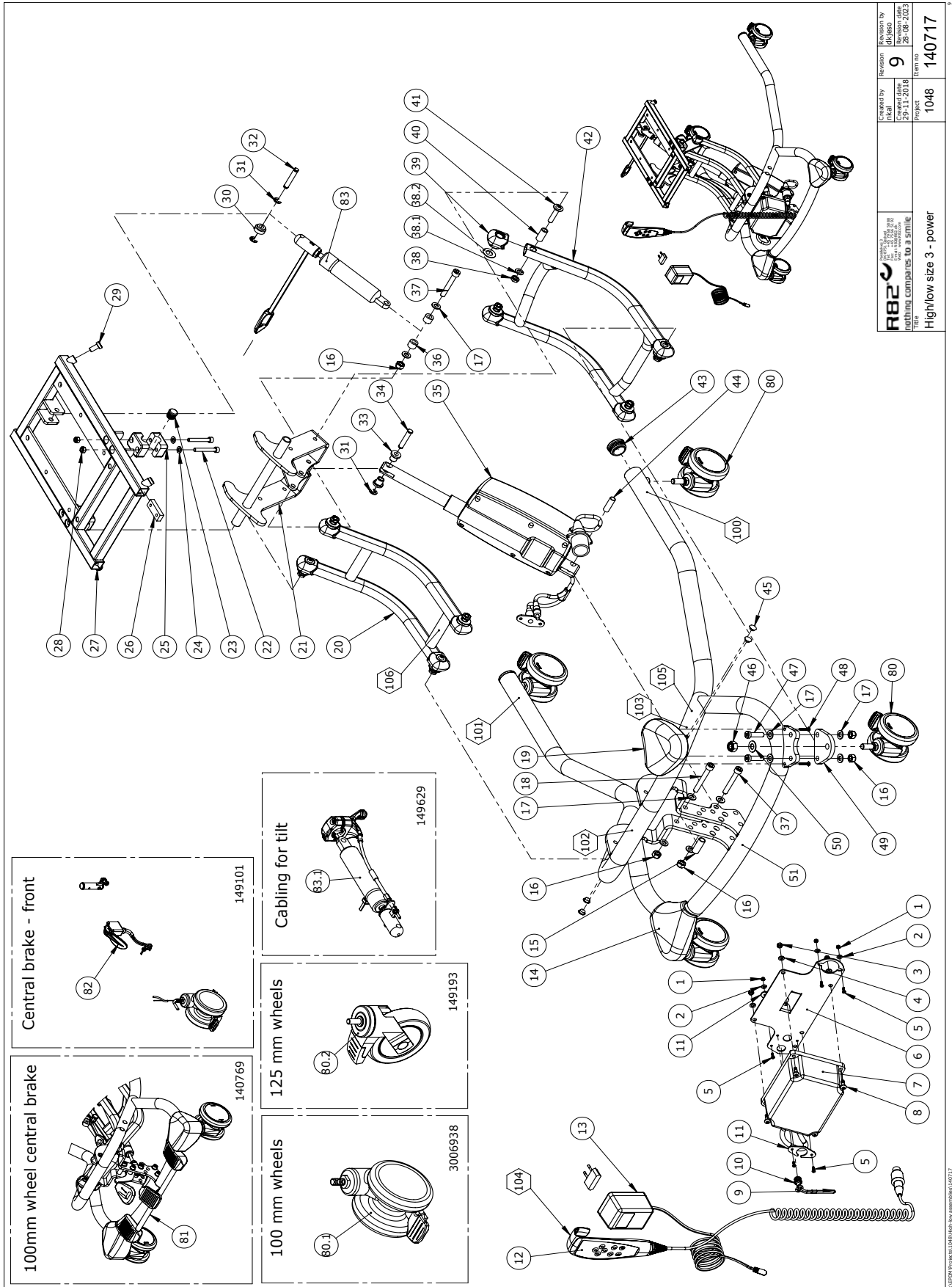


<p>R82 HILTI HILTI COMPANIES TO A SMILE HILTI High/Low size 3 - power</p>	Created by	Revision	Revision by
	149193	9	dk/psd
	2018-03-16	2018-03-16	2018-03-16
	Project	1048	Item no

Item numbers for High-low:xo Size 3, Indoor Power

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1606250	Self-locking nut 3mm	6	PCS	5707292408396
2	1102800	Fazet washer 4mm	6	PCS	5707292140265
3	1606300	Lock nut 5 mm	4	PCS	5707292141354
4	1102902	Washer 5mm, A2	4	PCS	5707292411280
5	1902595	Crosshead screw M3x12, FZB	6	PCS	5707292565099
6	136851	Bracket batteri	1	PCS	5707292911216
7	2222110	Power unit ba18027	1	PCS	5707292492258
8	1902760	Cross-headed screw 5x16 mm	4	PCS	5707292142146
9	138882	Elastic band with clamp	1	PCS	5707292912879
10	2222112	Plug for battery box	1	PCS	5707292913241
11	138368	Reinforcement plate	2	PCS	5707292911223
12	2222089	Hand control HB30	1	PCS	5707292795489
13	2222161	Charger with plugs	1	PCS	5707292574046
14	139838	High/Low 2-3 standard cover L	1	PCS	5707292907677
15	140344	Distance bushing	1	PCS	5707292911261
16	1607000	Lock nut 8mm, large pack	7	PCS	5707292141378
17	1103200	Fazet washer 8x16x1,5mm, FZB	14	PCS	5707292140296
18	0603660	Insex screw 8x60	1	PCS	5707292138279
19	139826	High/Low 2-3 standard cover R	1	PCS	5707292907660
20*	140189-04	Upper arm H/L sz3 Power, Black	1	PCS	5707292903372
	140189-10	Upper arm H/L sz3 Power, White	1	PCS	5707292931177
21	140667-04	Tip brace High-low size 3, Black	1	PCS	5707292907486
22	0602445	Insex M6x45 FZB	4	PCS	5707292138156
23	2501600	Ferrule d15x1,0 black	2	PCS	5707292143969
24	1105060	SS washer M6 6,4x12x1,6	4	PCS	5707292140357
25	2521840	Pipe holder f/16mmsquare,black	4	PCS	5707292399236
26	77571	Fastening bracket	2	PCS	5707292489982
27	152248-04	Seat frame	1	PCS	5707292922588
28	1606500	Lock nut 6 mm	4	PCS	5707292141361
29	0603530	Insex low head M8x16 stainless	4	PCS	5707292412492
30	77574	Sleeve f. Strike	1	PCS	5707292490773
31	1145070	Lock washer 7mm	3	PCS	5707292140586
32	80921	locking pin, Wombat	1	PCS	5707292111371
33	82848	Bush	2	PCS	5707292125484
34	140441	Lock pin	1	PCS	5707292911278
35	2222145	Actuator LA32 f/ HL3 + Wombat3	1	PCS	5707292903617
36	1350021	Distance piece 11x15x8,2mm	2	PCS	5707292140890
37	0615011	Insex M8x55 fzb shortened	2	PCS	5707292928023
38	1750800	Lock nut, stainless M8	8	PCS	5707292141484
38.1	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
38.2	6108191	Plastic washer - black	8	PCS	5707292986368
39	136960	Endbush for arm	8	PCS	5707292907684
40	135399	Bushing	8	PCS	5707292913142
41	0663037	Pan Head black zink M8x35	8	PCS	5707292983367
42*	140188-04	Lower arm sz3 Power, Black	1	PCS	5707292903358
	140188-10	Lower arm sz3 Power, White	1	PCS	5707292931184
43	3501860	Dupsko DPD32	2	PCS	5707292840097
* Choose color/size					

Position	No.	Description	Quantity	Unit of Measure	Ean code
44	P-6108200	Plast tube pa11 Ø10/Ø8mmx30mm	1	PCS	5707292585424
45	1310508	DR. fastener - afd. prop	4	PCS	5707292829795
46	1607500	Lock nut 10 mm	2	PCS	5707292141385
47	0603536	Insex lov head M8x30	4	PCS	5707292575326
48	0610620	screw ø3,5x20 DELTA PT A2	4	PCS	5707292903778
49	139619	Adaptor for indoor wheels sz2	2	PCS	5707292911209
50	1102500	Planning washer 10 mm	2	PCS	5707292140234
51*	139825-04	Complete frame - size 2 and 3, Black	1	PCS	5707292903419
	139825-10	Complete frame - size 2 and 3, White	1	PCS	5707292931009
80	3006825	75mm wheel w/brake, black	4	PCS	5707292371393
80.1	3006938	100mm double wheel with brake	4	PCS	5707292574930
80.2	149193	125 mm Wheel with bolt+washer	4	PCS	5707292911193
81	140769	High/low size 2, central brake	1	PCS	5707292913050
82	149101- HL2+3	Central brake front wheels HL2	1	SÆT	5707292911674
83	140413	Gas spring size 3 Tilt	1	PCS	5707292915238
83.1	149629	Gas spring size 3 tilt cable	1	PCS	5707292915016
100*	0494613	High-low:xo - silver	1	PCS	5707292921604
	0494618	High-low:xo - black	1	PCS	5707292921987
101*	0494614	R82 by Etac logo - Silver	1	PCS	5707292921611
	0494619	R82 by Etac logo - Black	1	PCS	5707292921994
102*	0494617	Size 3 - Silver	1	PCS	5707292921642
	0494657	Str. 3 mærke til high-low:xo - Black	1	PCS	5707292930750
103	0494089	IP/BF sticker	1	PCS	5707292795809
104	0492119	Sticker handcontrol HLeI Womb	1	PCS	5707292552594
105	0494648	HLXO-WL-CA Conditions	1	PCS	5707292924490
106	0493437	Label "exclamation mark"	1	PCS	5707292136909
* Choose color/size					

Size 1, Outdoor Gas Spring

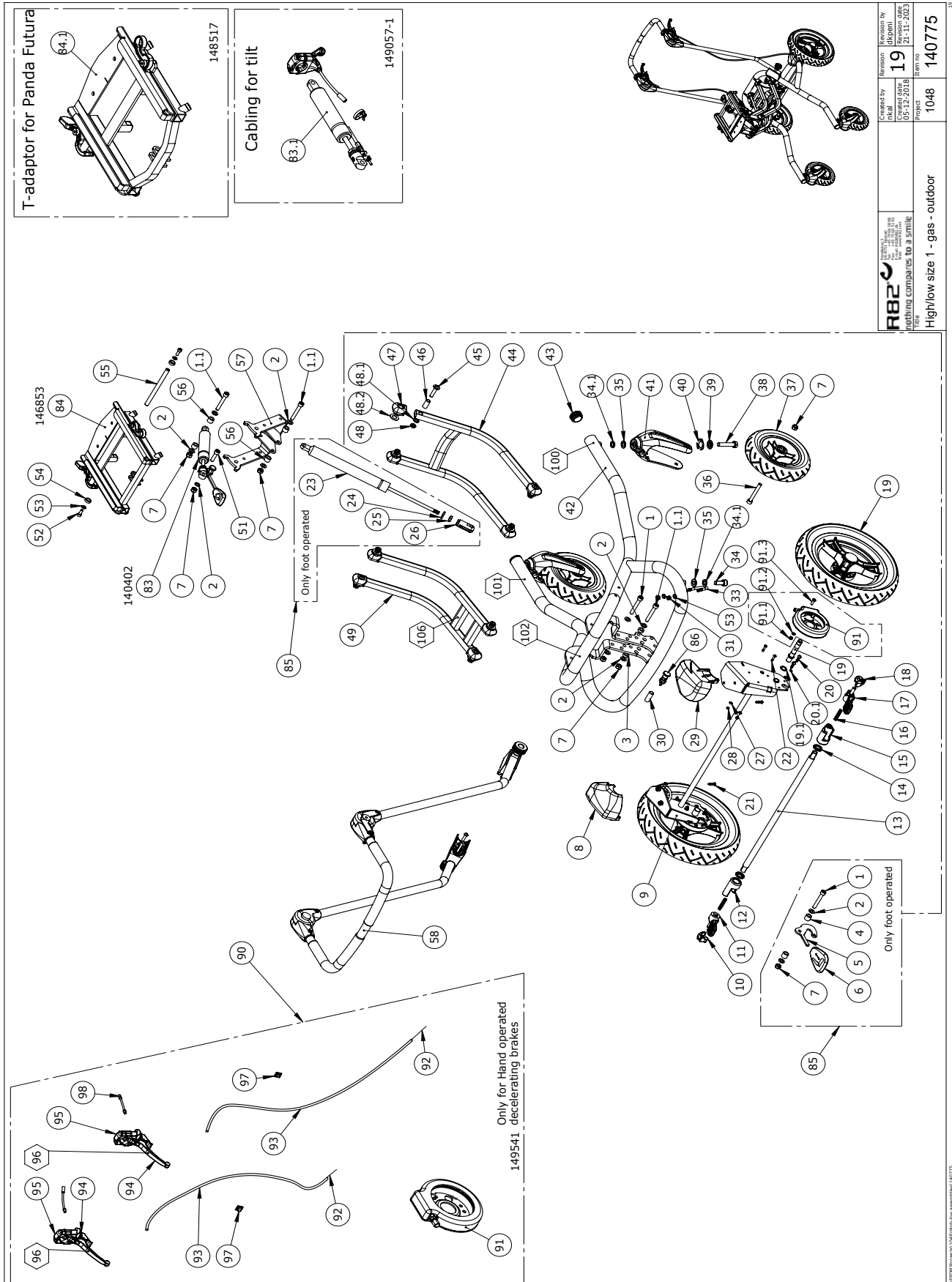

 R82 A/S Etac Group Etacvej 3 8751 Gedved Denmark Phone: +45 79 68 58 88 Fax: +45 75 66 51 92 E-mail: R82@etac.com Web: etac.com	Created by M.Hell	Revision 19 (M.Hell)	Date 05-12-2018
	Project 1048	Item no. 140775	Date 21-11-2023
High/low size 1 - gas - outdoor			Item no. 140775

© 2023 R82 A/S. All rights reserved. 140775

Item numbers for High-low:xo size 1, Outdoor Gas Spring


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0603600	Insex screw 8x55	1	PCS	5707292138262
1.1	0615011	Insex M8x55 fzb shortened	3	PCS	5707292928023
2	1103200	Fazet washer 8x16x1,5mm, FZB	9	PCS	5707292140296
3	1350023	Distance piece, black	2	PCS	5707292140913
4	1350023	Distance piece, black	2	PCS	5707292140913
5	154769	Lever arm low	1	PCS	5707292927453
6	142199-1	Release Pedal	1	PCS	5707292910783
7	1607000	Lock nut 8mm, large pack	7	PCS	5707292141378
8	139533	Str.1 Outdoor cover L	1	PCS	5707292910899
9	151913	Outdoor Bracket for size 1	1	PCS	5707292921871
10	76146	Bottom left, for brake	1	PCS	5707292491749
11	114304	Pedal left for brake	1	PCS	5707292605047
12	76148	House left, for brake	1	PCS	5707292491787
13	142356	Connecting bar outdoor	1	PCS	5707292911292
14	76145	Bush for connecting shaft	2	PCS	5707292491688
15	76143	House right, for brake	1	PCS	5707292491633
16	0308713	Stainless spring 1,0x6x40mmx17	2	PCS	5707292502032
17	114300	Pedal right for brake	1	PCS	5707292605030
18	76141	Bottom right, brake	1	PCS	5707292491589
19	3005050	Ø290,wheel AIR w/QR - PA6 GF15	2	PCS	5707292874559
19.1	0683900	Tight-washer 12x18x1	2	PCS	5707292139726
20	1940009	Eco syn 4x20 torx	4	PCS	5707292550897
20.1	1100484	Spring washer M4, stainless	4	PCS	5707292882349
21	0610620	screw ø3,5x20 DELTA PT A2	2	PCS	5707292903778
22	0522970	Screw M4x20	4	PCS	5707292908292
23	2205410-2	Bloc-o-Lift 1000 N	1	PCS	5707292927620
24	1310712	Nylon washer, nature	1	PCS	5707292356345
25	1761000	Counter nut 10x1,0 fine thread	1	PCS	5707292141538
26	6118822	Horizontal release	1	PCS	5707292186744
27	1102800	Fazet washer 4mm	4	PCS	5707292140265
28	1606275	Lock nut 4mm, large pack	4	PCS	5707292141347
29	139421	Str.1 Outdoor cover R	1	PCS	5707292910912
30	140344	Distance bushing	1	PCS	5707292911261
31	1606500	Lock nut 6 mm	4	PCS	5707292141361
33	136390	U-Bracket	2	PCS	5707292832078
34	0604500	Insex screw 10x25	2	PCS	5707292138378
34.1	1100510	Spring washer 10mm stainless	2	PCS	5707292140159
35	1103400	Fazet washer 10 mm	4	PCS	5707292140319
36	0603700	Insex screw 8x70	2	PCS	5707292138293
37	3009500	190x50 air racing	2	PCS	5707292483355
38	1902350	Steel set screw M10x50 FZB	2	PCS	5707292921512
39	151742	Bush for finger spring	2	PCS	5707292921390
40	6108153	Spring washer	2	PCS	5707292528414
41	2965160	Frontfork 200 mm	2	PCS	5707292928207
42*	134813-04	Complete frame - size 1 (Black)	1	PCS	5707292895967
	134813-35	Complete frame - size 1 (Antrazit)	1	PCS	5707292906922
	134813-10	Complete frame - size 1 (White)	1	PCS	5707292908476

* choose color

Position	No.	Description	Quantity	Unit of Measure	Ean code
43	3501860	Dupsko DPD32	2	PCS	5707292840097
44*	135738-04	Lower arm - size 1/2, Black	1	PCS	5707292883759
	135738-35	Lower arm - size 1/2, Antrazit	1	PCS	5707292906922
	135738-10	Lower arm - size 1/2 White	1	PCR	5707292906915
45	0663037	Pan Head black zink M8x35	8	PCS	5707292983367
46	135399	Bushing	8	PCS	5707292913142
47	136960	Endbush for arm	8	PCS	5707292907684
48	1750800	Lock nut, stainless M8	8	PCS	5707292141484
48.1	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
48.2	6108191	Plastic washer - black	8	PCS	5707292986368
49*	135749-04	Upper arm - size 1/2 (Black)	1	PCS	5707292883742
	135749-35	Upper arm - size 1/2 (Antrazit)	1	PCS	5707292908452
	135749-10	Upper arm - size 1/2 (White)	1	PCS	5707292883704
51	0615010	Insex M8x30 fzb, shortened	1	PCS	5707292367990
52	0602200	Insex screw 6x16	2	PCS	5707292138095
53	1105060	SS washer M6 6,4x12x1,6	6	PCS	5707292140357
54	1350040	Distancepart black 15x5x10,5mm	2	PCS	5707292911162
55	140826	Shaft - adaptor	1	PCS	5707292907851
56	1350021	Distance piece 11x15x8,2mm	4	PCS	5707292140890
57	140633-04	Tip brace High-low size 1-2 (Black)	1	PCS	5707292907479
58	140780-04	Outdoor Push brace (Black)	1	PCS	5707292909619
83	140402	Gas spring size 1+2 Tilt	1	PCS	5707292911711
83.1	149057-1	Gas spring size 1-2 tilt cable	1	PCS	5707292922113
84	146853-04	x:panda adapter size 1-2 (Black)	1	PCS	5707292907936
84.1	148517-04	Panda Futura adapter 1-4	1	PCS	5707292907929
85	155295	Foot operated Gascylinder HL1	1	PCS	5707292927460
86	150266	Lock for lift gas	1	PCS	5707292915368
90	149541	HL:xo dec. handbrake, Outdoor	1	SET	5707292913531
91	2983077	Band brake system	2	PCS	5707292971449
91.1	1631050	Prevailing torque lock nut m5	2	PCS	5707292510662
91.2	1102900	Fazet washer 5 mm	2	PCS	5707292140272
91.3	0658016	Button head M5x16 stainless	2	PCS	5707292139399
92	0371853	Strained wires 1,5mm 190 cm.	2	PCS	5707292135889
93	5002964	Outer casing f. Centralbreak	2	PCS	5707292353689
94	3503543	Brake driving lever,Black/Grey	2	PCS	5707292611512
95	3503536	Clamp w/screws	2	PCS	5707292427762
96	0494000	Label "brake" met. Film	2	PCS	5707292137159
97	2521854	Cable clamps 4mm, natural	2	PCS	5707292144416
98	0380060	Flexible v-brake pipe.	2	PCS	5707292425195
100	0494613	High-low:xo - silver	1	PCS	5707292921604
101	0494614	R82 by Etac logo - Silver	1	PCS	5707292921611
102	0494615	Size 1 - Silver	1	PCS	5707292921628
106	0493437	Label "exclamation mark"	1	PCS	5707292136909

* choose color

Size 1, Outdoor Power

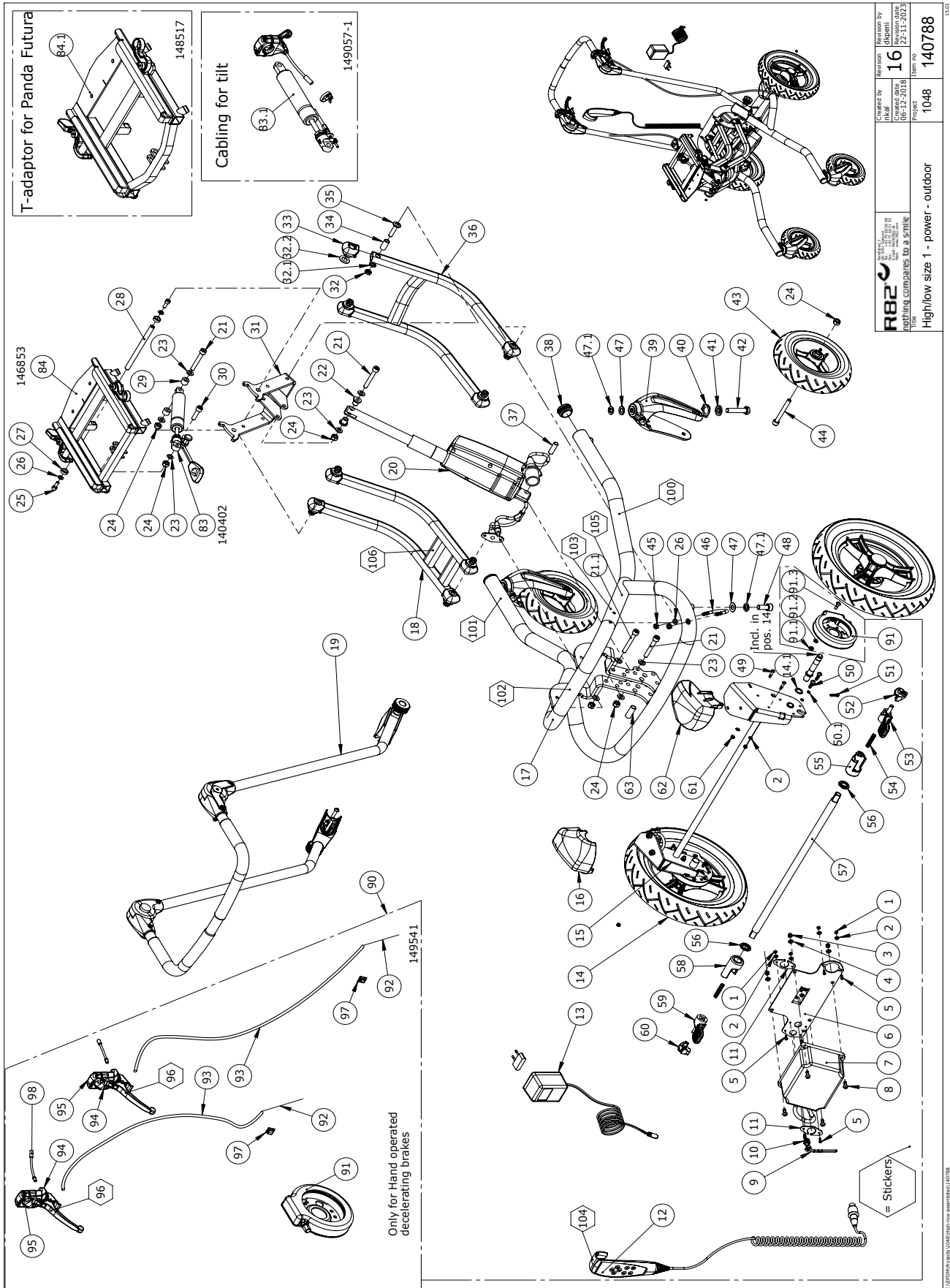


Item numbers for High-low:xo size 1, Outdoor Power

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1606250	Self-locking nut 3mm	6	PCS	5707292408396
2	1102800	Fazet washer 4mm	12	PCS	5707292140265
3	1606300	Lock nut 5 mm	4	PCS	5707292141354
4	1102902	Washer 5mm, A2	4	PCS	5707292411280
5	1902595	Crosshead screw M3x12, FZB	6	PCS	5707292565099
6	136851	Bracket batteri	1	PCS	5707292911216
7	2222110	Power unit ba18027	1	PCS	5707292492258
8	1902760	Cross-headed screw 5x16 mm	4	PCS	5707292142146
9	138882	Elastic band with clamp	1	PCS	5707292912879
10	2222112	Plug for battery box	1	PCS	5707292913241
11	138368	Reinforcement plate	2	PCS	5707292911223
12	2222089	Hand control HB30	1	PCS	5707292795489
13	2222161	Charger with plugs	1	PCS	5707292574046
14	3005050	Ø290,wheel AIR w/QR - PA6 GF15	2	PCS	5707292874559
14.1	0683900	Tight-washer 12x18x1	2	PCS	5707292139726
15	151913	Outdoor Bracket for size 1	1	PCS	5707292921871
16	139533	Str.1 Outdoor cover L	1	PCS	5707292910899
17	134813-04	Complete frame - size 1 (Black)	1	PCS	5707292895967
18	135749-04	Upper arm - size 1/2 (Black)	1	PCS	5707292883742
19	140780-04	Outdoor Push brace (Black)	1	PCS	5707292909619
20	2222139	Actuator LA28 f/High&Low 1	1	PCS	5707292795472
21	0615011	Insex M8x55 fzb shortened	3	PCS	5707292928023
21.1	0603660	Insex screw 8x60	1	PCS	5707292138279
22	82848	Bush	2	PCS	5707292125484
23	1103200	Fazet washer 8x16x1,5mm, FZB	9	PCS	5707292140296
24	1607000	Lock nut 8mm, large pack	7	PCS	5707292141378
25	0602200	Insex screw 6x16	2	PCS	5707292138095
26	1105060	SS washer M6 6,4x12x1,6	6	PCS	5707292140357
27	1350040	Distancepart black 15x5x10,5mm	2	PCS	5707292911162
28	140826	Shaft - adaptor	1	PCS	5707292907851
29	1350021	Distance piece 11x15x8,2mm	2	PCS	5707292140890
30	0615010	Insex M8x30 fzb, shortened	1	PCS	5707292367990
31	140633-04	Tip brace High-low size 1-2 (Black)	1	PCS	5707292907479
32	1750800	Lock nut, stainless M8	8	PCS	5707292141484
32.1	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
32.2	6108191	Plastic washer - black	8	PCS	5707292986368
33	136960	Endbush for arm	8	PCS	5707292907684
34	135399	Bushing	8	PCS	5707292913142
35	0663037	Pan Head black zink M8x35	8	PCS	5707292983367
36	135738-04	Lower arm - size 1/2, Black	1	PCS	5707292883759
37	P-6108200	Plast tube pal1 Ø10/Ø8mmx30mm	1	PCS	5707292585424
38	3501860	Dupsko DPD32	2	PCS	5707292840097
39	2965160	Frontfork 200 mm	2	PCS	5707292928207
40	6108153	Spring washer	2	PCS	5707292528414
41	151742	Bush for finger spring	2	PCS	5707292921390
42	1902350	Steel set screw M10x50 FZB	2	PCS	5707292921512
43	3009500	190x50 air racing	2	PCS	5707292483355

Position	No.	Description	Quantity	Unit of Measure	Ean code
44	0603700	Insex screw 8x70	2	PCS	5707292138293
45	1606500	Lock nut 6 mm	4	PCS	5707292141361
46	136390	U-Bracket	2	PCS	5707292832078
47	1103400	Fazet washer 10 mm	4	PCS	5707292140319
47.1	1100510	Spring washer 10mm stainless	2	PCS	5707292140159
48	0604500	Insex screw 10x25	2	PCS	5707292138378
49	0522970	Screw M4x20	4	PCS	5707292908292
50	1940009	Eco syn 4x20 torx	4	PCS	5707292550897
50.1	1100484	Spring washer M4, stainless	4	PCS	5707292882349
51	0610620	screw ø3,5x20 DELTA PT A2	2	PCS	5707292903778
52	76141	Bottom right, brake	1	PCS	5707292491589
53	114300	Pedal right for brake	1	PCS	5707292605030
54	0308713	Stainless spring 1,0x6x40mmx17	2	PCS	5707292502032
55	76143	House right, for brake	1	PCS	5707292491633
56	76145	Bush for connecting shaft	2	PCS	5707292491688
57	142356	Connecting bar outdoor	1	PCS	5707292911292
58	76148	House left, for brake	1	PCS	5707292491787
59	114304	Pedal left for brake	1	PCS	5707292605047
60	76146	Bottom left, for brake	1	PCS	5707292491749
61	1606275	Lock nut 4mm, large pack	4	PCS	5707292141347
62	139421	Str.1 Outdoor cover R	1	PCS	5707292910912
63	140344	Distance bushing	1	PCS	5707292911261
83	140402	Gas spring size 1+2 Tilt	1	PCS	5707292911711
83.1	149057-1	Gas spring size 1-2 tilt cable	1	PCS	5707292922113
84	146853-04	x:panda adapter size 1-2	1	PCS	5707292907936
84.1	148517-04	Panda Futura adapter 1-4	1	PCS	5707292907929
90	149541	HL:xo dec. handbrake, Outdoor	1	SET	5707292913531
91	2983077	Band brake system	2	PCS	5707292971449
91.1	1631050	Prevailing torque lock nut m5	2	PCS	5707292510662
91.2	1102900	Fazet washer 5 mm	2	PCS	5707292140272
91.3	0658016	Button head M5x16 stainless	2	PCS	5707292139399
92	0371853	Strained wires 1,5mm 190 cm.	2	PCS	5707292135889
93	5002964	Outer casing f. Centralbreak	2	PCS	5707292353689
94	3503543	Brake driving lever,Black/Grey	2	PCS	5707292611512
95	3503536	Clamp w/screws	2	PCS	5707292427762
96	0494000	Label "brake" met. Film	2	PCS	5707292137159
97	2521854	Cable clamps 4mm, natural	2	PCS	5707292144416
98	0380060	Flexible v-brake pipe.	2	PCS	5707292425195
100	0494613	High-low:xo - silver	1	PCS	5707292921604
101	0494614	R82 by Etac logo - Silver	1	PCS	5707292921611
102	0494615	Size 1 - Silver	1	PCS	5707292921628
103	0494089	IP/BF sticker	1	PCS	5707292795809
104	0492119	Sticker handcontrol HLel Womb	1	PCS	5707292552594
105	0494648	HLXO-WL-CA Conditions	1	PCS	5707292924490
106	0493437	Label "exclamation mark"	1	PCS	5707292136909

Size 2, Outdoor Gas

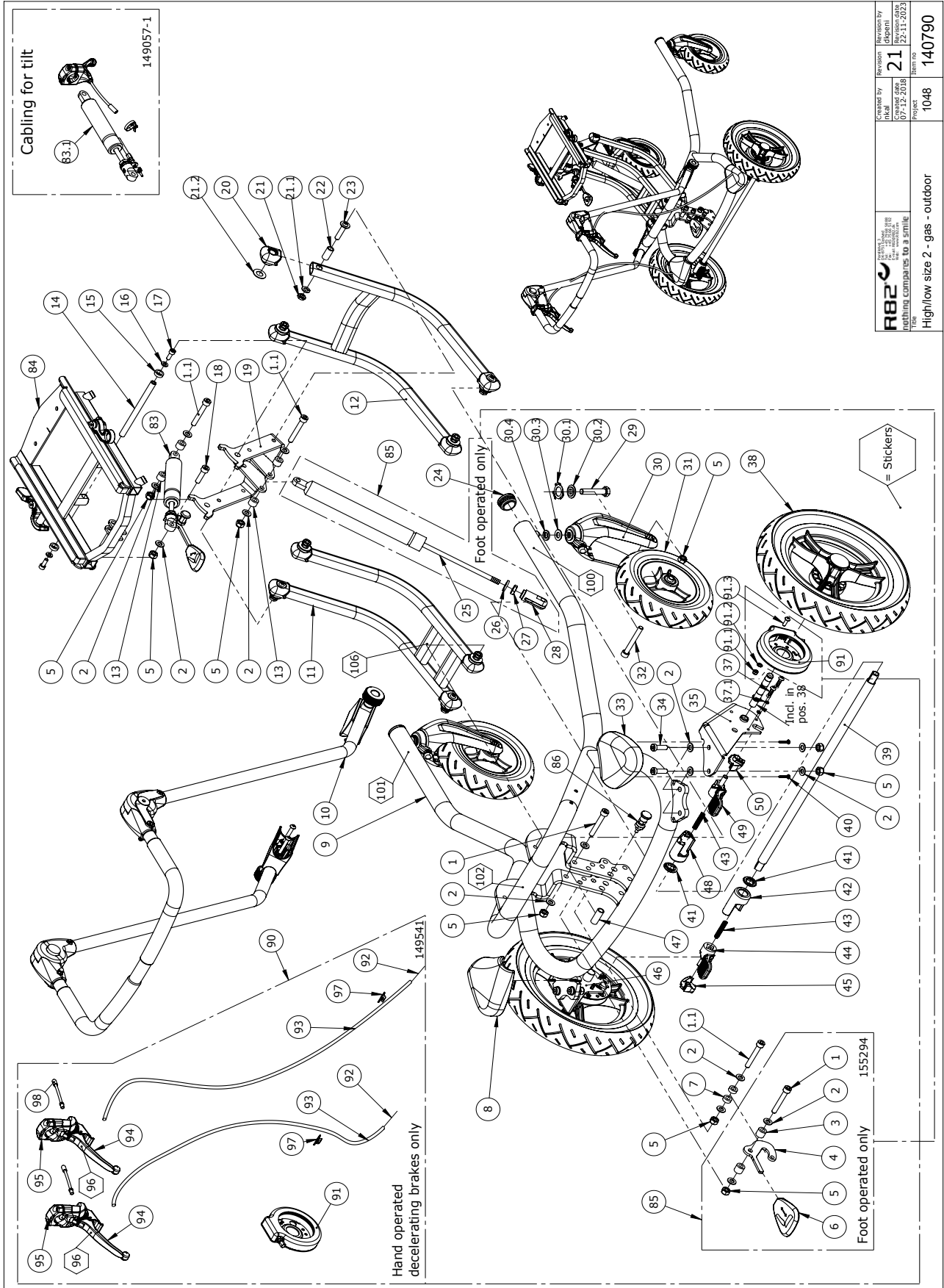

	Created by Mads 57-12-2018	Revision 21 18-11-2023
High/low size 2 - gas - outdoor	Project 1048	Item no 140790

C:\EPM\Projects\DKHighLow\assemblies\140790


Item numbers for High-low:xo size 2, Outdoor Gas

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0603600	Insex screw 8x55	1	PCS	5707292138262
1.1	0615011	Insex M8x55 fzb shortened	3	PCS	5707292928023
2	1103200	Fazet washer 8x16x1,5mm, FZB	17	PCS	5707292140296
3	1350023	Distance piece, black	2	PCS	5707292140913
4	154769	Lever arm low	1	PCS	5707292927453
5	1607000	Lock nut 8mm, large pack	1	PCS	5707292141378
6	142199-1	Release Pedal	1	PCS	5707292910783
7	1350000	Distancepart black 6x15x8,2 mm	2	PCS	5707292140876
8	139838	High/Low 2-3 standard cover L	1	PCS	5707292907677
9*	139825-04	Complete frame - size 2 and 3 (Black)	1	PCS	5707292903419
	139825-10	Complete frame - size 2 and 3 (White)	1	PCS	5707292931009
10	140780-04	Outdoor Push brace (Black)	1	PCS	5707292909619
11*	135749-04	Upper arm - size 1/2 (Black)	1	PCS	5707292883742
	135749-10	Upper arm - size 1/2 (White)	1	PCS	5707292883704
12*	135738-04	Lower arm - size 1/2, Black	1	PCS	5707292883759
	135738-10	Lower arm - size 1/2 White	1	PCS	5707292906915
13	1350021	Distance piece 11x15x8,2mm	4	PCS	5707292140890
14	140826	Shaft - adaptor	1	PCS	5707292907851
15	1350040	Distancepart black 15x5x10,5mm	2	PCS	5707292911162
16	1105060	SS washer M6 6,4x12x1,6	2	PCS	5707292140357
17	0602200	Insex screw 6x16	2	PCS	5707292138095
18	0615010	Insex M8x30 fzb, shortened	1	PCS	5707292367990
19	140633-04	Tip brace High-low size 1-2 (Black)	1	PCS	5707292907479
20	136960	Endbush for arm	8	PCS	5707292907684
21	1750800	Lock nut, stainless M8	8	PCS	5707292141484
21.1	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
21.2	6108191	Plastic washer - black	8	PCS	5707292986368
22	135399	Bushing	8	PCS	5707292913142
23	0663037	Pan Head black zink M8x35	8	PCS	5707292983367
24	3501860	Dupsko DPD32	2	PCS	5707292840097
25	2205420	Bloc-o-Lift 1400 N	1	PCS	5707292912770
26	1310712	Nylon washer, nature	1	PCS	5707292356345
27	1761000	Counter nut 10x1,0 fine thread	1	PCS	5707292141538
28	6118822	Horizontal release	1	PCS	5707292186744
29	1902350	Steel set screw M10x50 FZB	2	PCS	5707292921512
30	2965160	Frontfork 200 mm	2	PCS	5707292928207
30.1	6108153	Spring washer	2	PCS	5707292528414
30.2	151742	Bush for finger spring	2	PCS	5707292921390
30.3	1103400	Fazet washer 10 mm	2	PCS	5707292140319
30.4	1100510	Spring washer 10mm stainless	2	PCS	5707292140159
31	3009501	190x50 solid racing	2	PCS	5707292483362
32	0603700	Insex screw 8x70	2	PCS	5707292138293
33	139826	High/Low 2-3 standard cover R	1	PCS	5707292907660
34	0603536	Insex lov head M8x30	4	PCS	5707292575326
35	162749	Outdoor wheel bracket,R S2	1	PCS	5707292947697
37	1940009	Eco syn 4x20 torx	4	PCS	5707292550897
37.1	1100484	Spring washer M4, stainless	1	PCS	5707292882349

*choose color

Position	No.	Description	Quantity	Unit of Measure	Ean code
38	3005070	Ø290,wheel PUR w/Quick Release	2	PCS	5707292874580
39	162722	Connecting bar Outdoor - sz 2	1	PCS	5707292947901
40	0610620	screw ø3,5x20 DELTA PT A2	4	PCS	5707292903778
41	76145	Bush for connecting shaft	1	PCS	5707292491688
42	76148	House left, for brake	1	PCS	5707292491787
43	0308713	Stainless spring 1,0x6x40mmx17	2	PCS	5707292502032
44	114304	Pedal left for brake	1	PCS	5707292605047
45	76146	Bottom left, for brake	1	PCS	5707292491749
46	162750	Outdoor wheel bracket,L S2	1	PCS	5707292947680
47	140344	Distance bushing	1	PCS	5707292911261
48	76143	House right, for brake	1	PCS	5707292491633
49	114300	Pedal right for brake	1	PCS	5707292605030
50	76141	Bottom right, brake	1	PCS	5707292491589
83	140402	Gas spring size 1+2 Tilt	1	PCS	5707292911711
83.1	149057-1	Gas spring size 1-2 tilt cable	1	PCS	5707292922113
84	140907-04	x:panda adapter size 3	1	PCS	5707292907905
85	155294	Foot operated Gascylinder HL2	1	PCS	5707292927446
86	150266	Lock for lift gas	1	PCS	5707292915368
90	149541	HL:xo dec. handbrake, Outdoor	1	SET	5707292913531
91	2983077	Band brake system	2	PCS	5707292971449
91.1	1631050	Prevailing torque lock nut m5	2	PCS	5707292510662
91.2	1102900	Fazet washer 5 mm	2	PCS	5707292140272
91.3	0658016	Button head M5x16 stainless	2	PCS	5707292139399
92	0371853	Strained wires 1,5mm 190 cm.	2	PCS	5707292135889
93	5002964	Outer casing f. Centralbreak	2	PCS	5707292353689
94	3503543	Brake driving lever,Black/Grey	2	PCS	5707292611512
95	3503536	Clamp w/screws	2	PCS	5707292427762
96	0494000	Label "brake" met. Film	2	PCS	5707292137159
97	2521854	Cable clamps 4mm, natural	2	PCS	5707292144416
98	0380060	Flexible v-brake pipe.	2	PCS	5707292425195
100	0494613	High-low:xo - silver	1	PCS	5707292921604
101	0494614	R82 by Etac logo - Silver	1	PCS	5707292921611
102	0494616	Size 2 - Silver	1	PCS	5707292921635
106	0493437	Label "exclamation mark"	1	PCS	5707292136909
*choose color					

Size 2, Outdoor Power

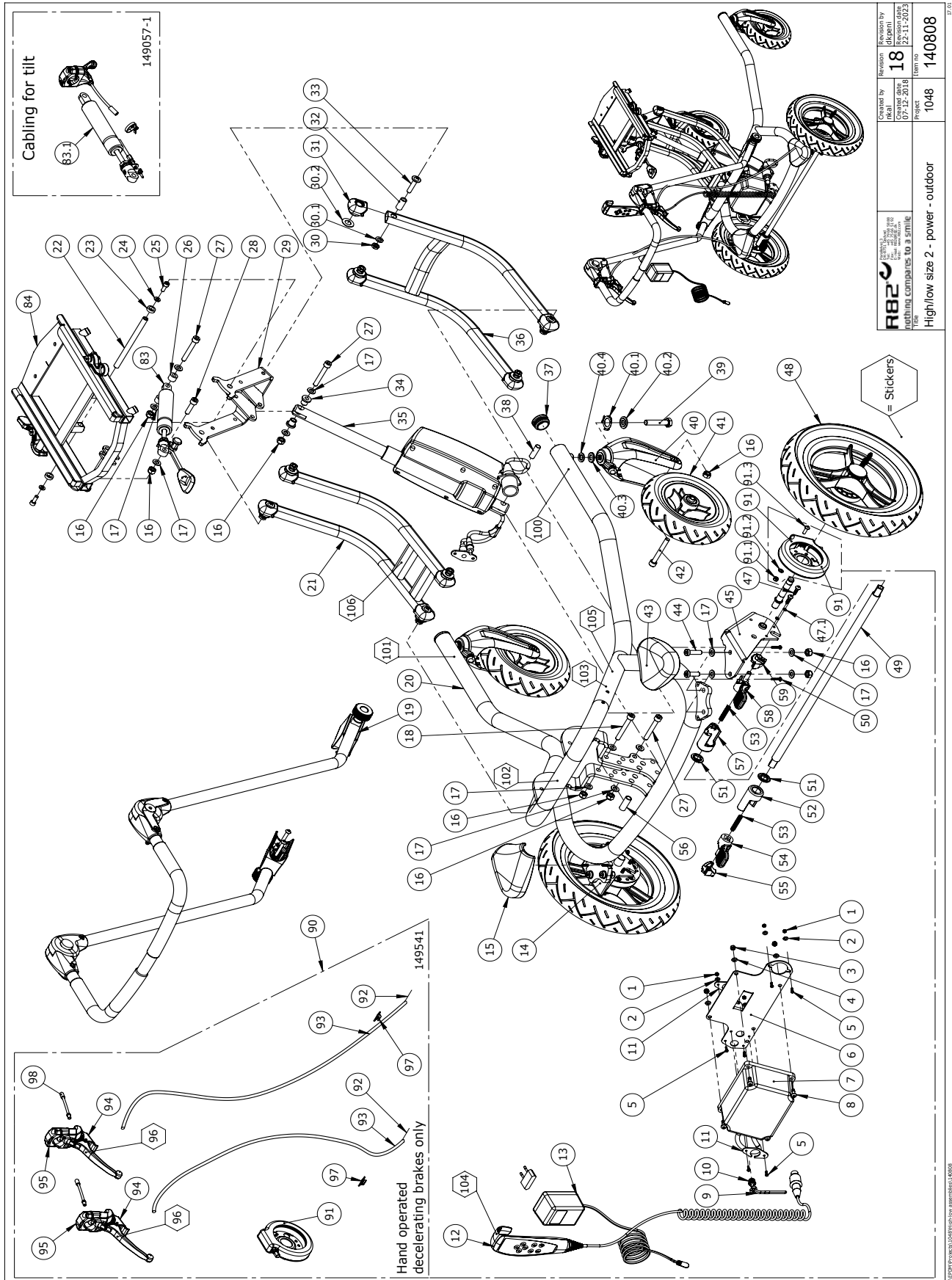

 R82 A/S Parallelvej 3 8751 Gedved Denmark	Created by nikal	Revision 18	Revision by dikpeni
	Date 07-10-2018	Date 07-11-2018	Date 07-11-2018
Project High/low size 2 - power - outdoor	Item no. 1048	Item no. 140808	


© JYBA Power Solutions High/low assembly 140808 37.0

Item numbers for High-low:xo size 2, Outdoor Power

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1606250	Self-locking nut 3mm	6	PCS	5707292408396
2	1102800	Fazet washer 4mm	6	PCS	5707292140265
3	1606300	Lock nut 5 mm	4	PCS	5707292141354
4	1102902	Washer 5mm, A2	4	PCS	5707292411280
5	1902595	Crosshead screw M3x12, FZB	6	PCS	5707292565099
6	136851	Bracket batteri	1	PCS	5707292911216
7	2222110	Power unit ba18027	1	PCS	5707292492258
8	1902760	Cross-headed screw 5x16 mm	4	PCS	5707292142146
9	138882	Elastic band with clamp	1	PCS	5707292912879
10	2222112	Plug for battery box	1	PCS	5707292913241
11	138368	Reinforcement plate	2	PCS	5707292911223
12	2222089	Hand control HB30	1	PCS	5707292795489
13	2222161	Charger with plugs	1	PCS	5707292574046
14	162750	Outdoor wheel bracket,L S2	1	PCS	5707292947888
15	139838	High/Low 2-3 standard cover L	1	PCS	5707292907677
16	1607000	Lock nut 8mm, large pack	11	PCS	5707292141378
17	1103200	Fazet washer 8x16x1,5mm, FZB	17	PCS	5707292140296
18	0603660	Insex screw 8xx60	1	PCS	5707292138279
19	140780-04	Outdoor Push brace (Black)	1	PCS	5707292909619
20*	139825-04	Complete frame - size 2 and 3 (Black)	1	PCS	5707292903419
	139825-10	Complete frame - size 2 and 3 (White)	1	PCS	5707292931009
21*	135749-04	Upper arm - size 1/2 (Black)	1	PCS	5707292883742
	135749-10	Upper arm - size 1/2 (White)	1	PCS	5707292883704
22	140826	Shaft - adaptor	1	PCS	5707292907851
23	1350040	Distancepart black 15x5x10,5mm	2	PCS	5707292911162
24	1105060	SS washer M6 6,4x12x1,6	2	PCS	5707292140357
25	0602200	Insex screw 6x16	2	PCS	5707292138095
26	1350021	Distance piece 11x15x8,2mm	2	PCS	5707292140890
27	0615011	Insex M8x55 fzb shortened	3	PCS	5707292928023
28	0615010	Insex M8x30 fzb, shortened	1	PCS	5707292367990
29	140633-04	Tip brace High-low size 1-2 (Black)	1	PCS	5707292907479
30	1750800	Lock nut, stainless M8	8	PCS	5707292141484
30.1	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
30.2	6108191	Plastic washer - black	8	PCS	5707292986368
31	136960	Endbush for arm	8	PCS	5707292907684
32	135399	Bushing	8	PCS	5707292913142
33	0663037	Pan Head black zink M8x35	8	PCS	5707292983367
34	82848	Bush	2	PCS	5707292125484
35	2222146	Actuator LA32 f/ High-Low 2	1	PCS	5707292903624
36*	135738-04	Lower arm - size 1/2, Black	1	PCS	5707292883759
	135738-10	Lower arm - size 1/2 White	1	PCS	5707292906915
37	3501860	Dupsko DPD32	2	PCS	5707292840097
38	P-6108200	Plast tube pa11 Ø10/Ø8mmx30mm	1	PCS	5707292585424
39	1902350	Steel set screw M10x50 FZB	2	PCS	5707292921512
40	2965160	Frontfork 200 mm	2	PCS	5707292928207
40.1	6108153	Spring washer	2	PCS	5707292528414
40.2	151742	Bush for finger spring	2	PCS	5707292921390
*choose color					

Position	No.	Description	Quantity	Unit of Measure	Ean code
40.3	1103400	Fazet washer 10 mm	2	PCS	5707292140319
40.4	1100510	Spring washer 10mm stainless	2	PCS	5707292140159
41	3009501	190x50 solid racing	2	PCS	5707292483362
42	0603700	Insex screw 8x70	2	PCS	5707292138293
43	139826	High/Low 2-3 standard cover R	1	PCS	5707292907660
44	0603536	Insex lov head M8x30	4	PCS	5707292575326
45	162749	Outdoor wheel bracket,R S2	1	PCS	5707292947697
47	1940009	Eco syn 4x20 torx	4	PCS	5707292550897
47.1	1100484	Spring washer M4, stainless	4	PCS	5707292882349
48	3005070	Ø290,wheel PUR w/Quick Release	2	PCS	5707292874580
49	162722	Connecting bar Outdoor - sz 2	1	PCS	5707292947901
50	0610620	screw ø3,5x20 DELTA PT A2	4	PCS	5707292903778
51	76145	Bush for connecting shaft	2	PCS	5707292491688
52	76148	House left, for brake	1	PCS	5707292491787
53	0308713	Stainless spring 1,0x6x40mmx17	2	PCS	5707292502032
54	114304	Pedal left for brake	1	PCS	5707292605047
55	76146	Bottom left, for brake	1	PCS	5707292491749
56	140344	Distance bushing	1	PCS	5707292911261
57	76143	House right, for brake	1	PCS	5707292491633
58	114300	Pedal right for brake	1	PCS	5707292605030
59	76141	Bottom right, brake	1	PCS	5707292491589
83	140402	Gas spring size 1+2 Tilt	1	PCS	5707292911711
83.1	149057-1	Gas spring size 1-2 tilt cable	1	PCS	5707292922113
84	140907-04	x:panda adapter size 3	1	PCS	5707292907905
90	149541	HL:xo dec. handbrake, Outdoor	1	SET	5707292913531
91	2983077	Band brake system	2	PCS	5707292971449
91.1	1631050	Prevailing torque lock nut m5	2	PCS	5707292510662
91.2	1102900	Fazet washer 5 mm	2	PCS	5707292140272
91.3	0658016	Button head M5x16 stainless	2	PCS	5707292139399
92	0371853	Strained wires 1,5mm 190 cm.	2	PCS	5707292135889
93	5002964	Outer casing f. Centralbreak	2	PCS	5707292353689
94	3503543	Brake driving lever,Black/Grey	2	PCS	5707292611512
95	3503536	Clamp w/screws	2	PCS	5707292427762
96	0494000	Label "brake" met. Film	2	PCS	5707292137159
97	2521854	Cable clamps 4mm, natural	2	PCS	5707292144416
98	0380060	Flexible v-brake pipe.	2	PCS	5707292425195
100	0494613	High-low:xo - silver	1	PCS	5707292921604
101	0494614	R82 by Etac logo - Silver	1	PCS	5707292921611
102	0494616	Size 2 - Silver	1	PCS	5707292921635
103	0494089	IP/BF sticker	1	PCS	5707292795809
104	0492119	Sticker handcontrol HLeI Womb	1	PCS	5707292552594
105	0494648	HLXO-WL-CA Conditions	1	PCS	5707292924490
106	0493437	Label "exclamation mark"	1	PCS	5707292136909
*choose color					

Foot rest

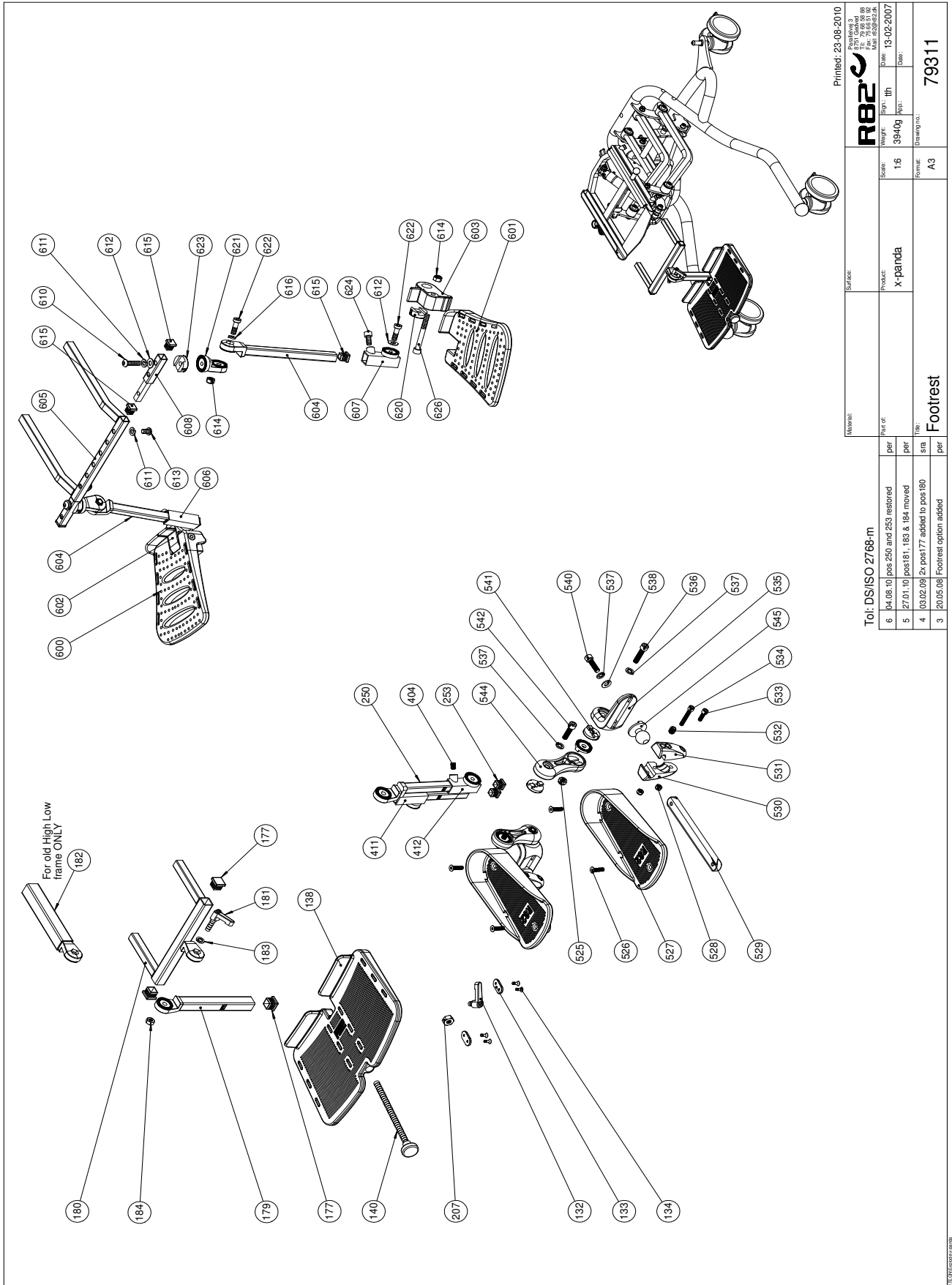

Printed: 23-06-2010

Material:		Surface:	
Part no:		Product:	
Weight:		Scale:	
Sign:		Date:	
tth		Date:	
3940g		16	
Form:		Form:	
A3		A3	
79311		79311	

Tot: DS/ISO 2768-m

6	04.08.10	pos 250 and 253 restored	per
5	27.01.10	pos 81, 183 & 184 moved	per
4	03.02.09	2x pos 177 added to pos 180	sra
3	20.05.08	Footrest option added	per


Item numbers for High-low:xo Foot rest

Position	No.	Description	Quantity	Unit of Measure	Ean code
132	3502516	TIGHTENING HANDLE 45MM M8x16	1	PCS	5707292146915
133	82950	HOLLOW PLATE 16x32, WOMBAT	2	PCS	5707292126030
134	1940017	SHEET METAL SCREW UHX4,2x13FZB	4	PCS	5707292142597
138	82944	FOOTPLATE SIZE 1, WOMBAT	1	PCS	5707292125996
138	82940	FODPLADE STR.2, WOMBAT	1	PCS	5707292125958
140	3502780	HANDLE 28MM,M10x160MM THREAD	1	PCS	5707292363145
140	3502790	HAND GRIP 28MM, M10x196MM	1	PCS	5707292363152
177	2504500	FERRULE E20x1,0-2,0 INT.BLACK	5	PCS	5707292144126
179*	75260	Rod size 0 for footrest	1	PCS	5707292458346
	75261	Rod size 1 for footrest	1	PCS	5707292455543
	75262	Rod size 2 for footrest	1	PCS	5707292458353
	75263	Rod size 3 for footrest	1	PCS	5707292458360
	75264	Rod size 4 for footrest	1	PCS	5707292455505
180	75270	HOLDER SIZE 1 FOR FOOTREST	1	PCS	5707292455598
180	75272	HOLDER SIZE 2 FOR FOOTREST	1	PCS	5707292458384
181	3504025	TIGHTENING HANDLE 45MM M8x25	1	PCS	5707292147103
182*	75266	HOLDER SIZE 1 FOR FOOTREST	1	PCS	5707292455611
	75267	HOLDER SIZE 2 FOR FOOTREST	1	PCS	5707292455604
183	1101000	CENTER SPRING WASHER 8MM	1	PCS	5707292140173
184	1603500	STEEL NUT 8MM, LARGE PACK	1	PCS	5707292141286
207	1607500	Lock nut 10 mm	1	PCS	5707292141385
250*	75285	Holder size 0, double footrest	1	PCS	5707292458445
	75286	Holder size 1, double footrest	1	PCS	5707292458452
	75287	HOLDER SIZE 2,DOUBBLE FOOTREST	1	PCS	5707292458469
	75288	HOLDER SIZE 3,DOUBBLE FOOTREST	1	PCS	5707292458476
	75289	Holder size , double footrest	1	PCS	5707292458483
253	2504000	FERRULE E15x1,0-2,0 INT.BLACK	2	PCS	5707292144102
404	0701000	Stop screw with crater 8x10	2	PCS	5707292139870
411	73734	Mounting for footrest,right	1	PCS	5707292433466
412	73735	Mounting for footrest,left	1	PCS	5707292433473
525	1607000	Lock nut 8mm, large pack	2	PCS	5707292141378
526	0612500	Insex screw counters. 6x25	4	PCS	5707292138712
527	82156-60	Footplate size 1,black,gazelle	2	PCS	5707292230492
527	82154-60	Footplate size 2,black,gazelle	2	PCS	5707292230478
527	82158-60	Footplate size 3 black,gazelle	2	PCS	5707292230508
528	1606500	Lock nut 6 mm	4	PCS	5707292141361
529	73648	Stick size 1, footrest	2	PCS	5707292445797
529	73649	Stick size 2, footrest	2	PCS	5707292445803
530	72724	Ball joining right	2	PCS	5707292421845
531	72723	Ball joining left	2	PCS	5707292421814
532	0701010	Set screw M10x10 w/flat end	2	PCS	5707292434098
533	0602300	Insex screw 6x20	2	PCS	5707292138101
534	0602440	Insex M6x40 FZB	2	PCS	5707292138149
535	72713	Bracked	2	PCS	5707292420763
536	0603000	Insex screw 8x35	2	PCS	5707292138217
537	1101000	Center spring washer 8mm	6	PCS	5707292140173
538	1102010	Face plate 8,4x18x2mm, FZB	2	PCS	5707292429766

*Choose size/left/right

Position	No.	Description	Quantity	Unit of Measure	Ean code
540	0602700	Insex screw 8x25	2	PCS	5707292138187
541	74062	Toothed disk 30x14	6	PCS	5707292377951
542	0602850	Insex M8x22, FZB	2	PCS	5707292435408
544	72712	Knee joint	2	PCS	5707292420756
545	72721	Ballhead	2	PCS	5707292428875
600	74052	Foot plate 1, right	1	PCS	5707292373748
601	74053	Foot plate 1, left	1	PCS	5707292373755
602	74044	Mounting for foot plate, right	1	PCS	5707292373687
603	74045	Mounting for foot plate, left	1	PCS	5707292373694
604*	75317	Rod for footrest size 0, 21 cm	2	PCS	5707292477774
	75319	Rod for footrest size 2, 34 cm	2	PCS	5707292477804
	75325	Rod for footrest size 3, 38 cm	2	PCS	5707292477811
605*	75327	Holder for footrest size 1	1	PCS	5707292477545
	75312	Holder for footrest size 2	1	PCS	5707292477538
606*	73734	Mounting for footrest,right	1	PCS	5707292433466
	73735	Mounting for footrest,left	1	PCS	5707292433473
607*	73735	Mounting for footrest,left	1	PCS	5707292433473
	73734	Mounting for footrest,right	1	PCS	5707292433466
610	0663000	Button head screw 8x30 stainl.	2	PCS	5707292139597
611	1100500	Spring washer 8mm, stainless	2	PCS	5707292140142
612	1102000	Planing washer 8mm	4	PCS	5707292140227
614	1607000	Lock nut 8mm, large pack	4	PCS	5707292141378
615	2504000	Ferrule e15x1,0-2,0 int.black	4	PCS	5707292144102
616	1100495	Spring washer M8	2	PCS	5707292458674
620	74061	Tooth washer Ø30	2	PCS	5707292373816
621	75315	Rotary joint	2	PCS	5707292477927
622	0603535	Insex low head M8x25	4	PCS	5707292575319
623	75329	Tooth disk Ø30x10	2	PCS	5707292482389
624	0603530	Insex low head M8x16 stainless	2	PCS	5707292412492
626	0603700	Insex screw 8x70	2	PCS	5707292138293
*Choose size/left/right					

Outdoor Push Brace

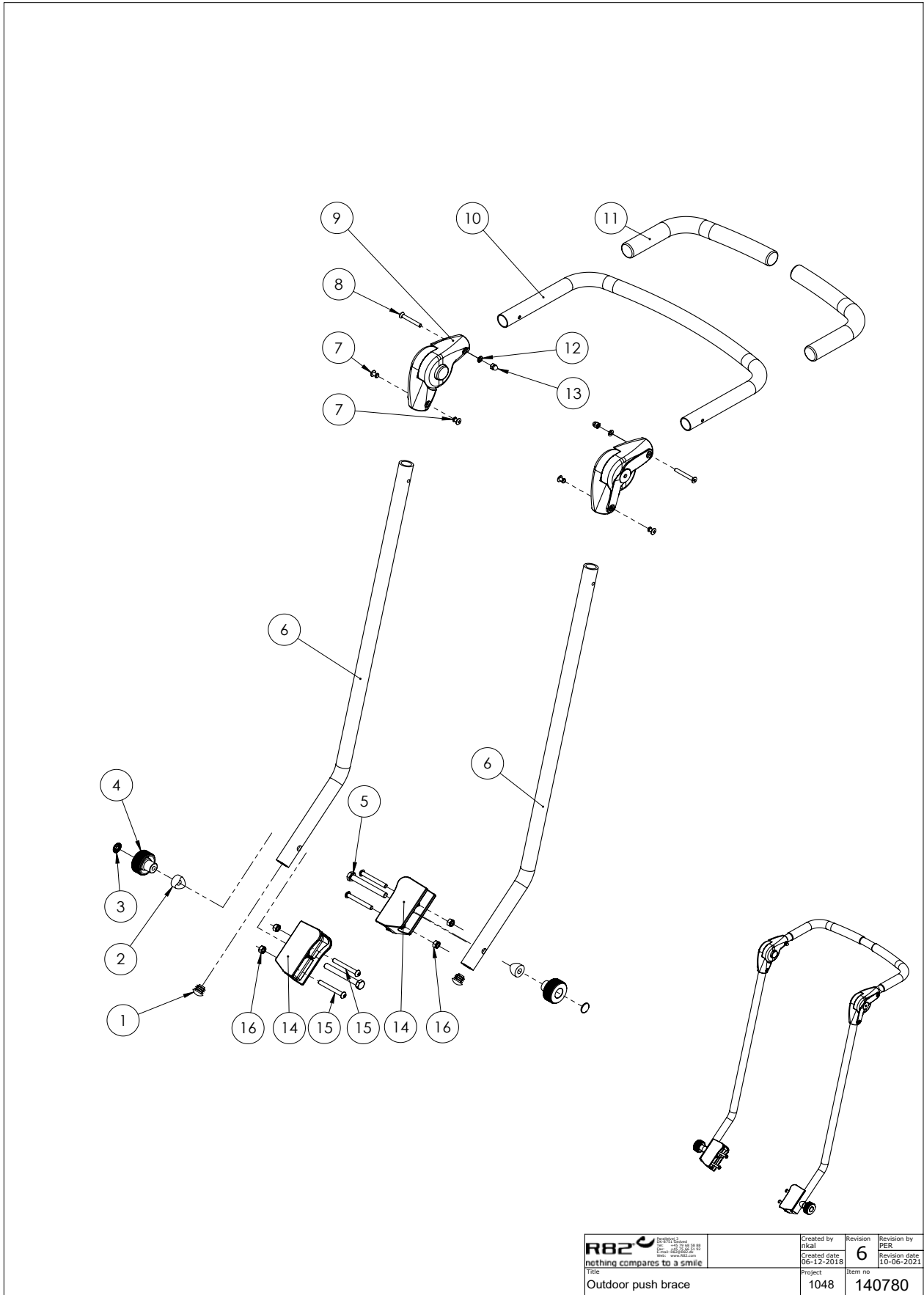


 <small>R82 A/S Parallelvej 3 8751 Gedved Denmark</small>	Created by nkal	Revision 6	Revision by PER
	Created date 06-12-2018	Item no 140780	Revision date 10-06-2021
Title Outdoor push brace		Project 1048	

Item numbers for High-low:xo Outdoor Push Brace

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	2502002	Ferrule d18x0,8-2,5 black	2	PCS	5707292433947
2	1357500	Distance pcs 20x8,4x4mm	2	PCS	5707292141026
3	1109500	Starlock washer with cape 6	2	PCS	5707292140449
4	3470400	Hand grip without cover	2	PCS	5707292146663
5	1902320	Steel set screw 8x70mm	2	PCS	5707292142023
6	138802-04	Tube for pushbrace	2	PCS	5707292911339
7	0506508	Rivet 5x14, stainless	4	PCS	5707292359476
8	0609724	Insex 5x40 uh. Fzb	2	PCS	5707292434999
9	0859090	Locking joint Ø22mm	2	PCS	5707292440242
10	138949-04	Tube for pushbrace	1	PCS	5707292910356
11	2500520	Tpe cover Ø20mm	2	PCS	5707292401649
12	1102902	Washer 5mm, A2	2	PCS	5707292411280
13	1780501	Top nut, locking ring	2	PCS	5707292931115
14	155184	Mounting bracket High/low	2	PCS	5707292928139
15	0659771	Button head M6x50 FZB	4	PCS	5707292923721
16	1606500	Lock nut 6 mm	4	PCS	5707292141361

Anti Tip Indoor

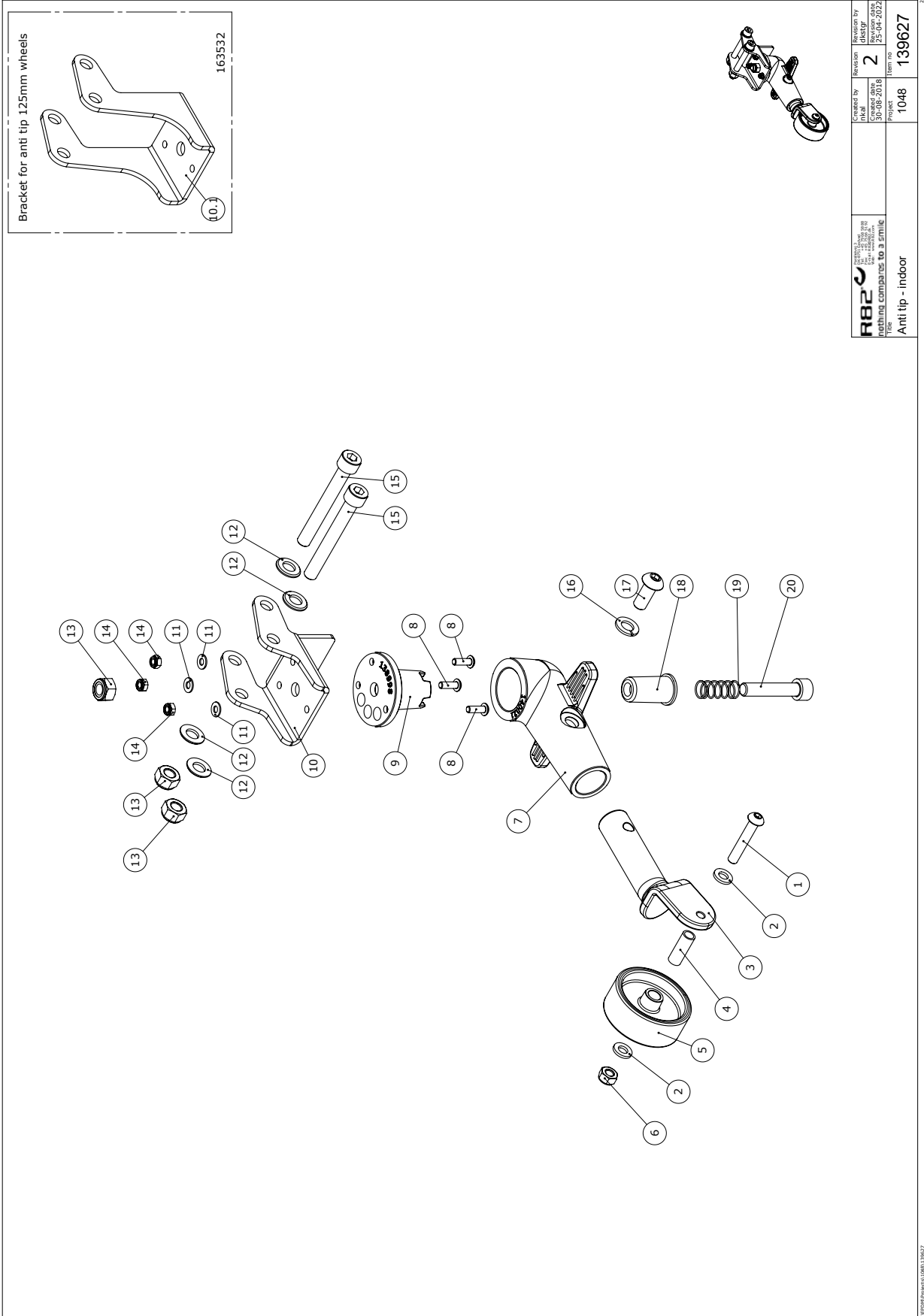


<p>R82 Fastening companies to a smile</p>	Created by nikel	Revision 2	Revision by dilestip
	30-08-2018	30-08-2018	22-Jul-2022
File Anti tip - indoor	Project 1048	Item no 139627	

C:\BPP\PowerLine\139627

Item numbers for High-low:xo Anti Tip Indoor

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0659585	Button head M6x35, stainless	1	PSC	5707292139498
2	1105060	SS washer M6 6,4x12x1,6	2	PSC	5707292140357
3	141344	Wheel mount anti tilt right	1	PSC	5707292882103
4	111855	Bush for anti tilt device	1	PSC	5707292592989
5	303610	Wheel 50x19 Ø8 PU	1	PSC	5707292612748
6	1631060	Stainless lock nut 6mm	1	PSC	5707292141446
7	140021	Base for tip brace	1	PSC	5707292910882
8	0601900	Button head M4x12 FZB	3	PSC	5707292908285
9	138998	Anti tip base	1	PSC	5707292910875
10	138873	Bracket for anti tip	1	PSC	5707292911247
10.1	163532	Bracket for anti tip -indoor	1	PSC	5707292949141
11	1102800	Fazet washer 4mm	3	PSC	5707292140265
12	1103200	Fazet washer 8x16x1,5mm, FZB	4	PSC	5707292140296
13	1607000	Lock nut 8mm, large pack	3	PSC	5707292141378
14	1606275	Lock nut 4mm, large pack	3	PSC	5707292141347
15	0603665	Insex screw 8x65	2	PSC	5707292138286
16	1101050	Spring washer M8, wavy FZB	1	PSC	5707292458667
17	0662000	Button head screw 8x20 stainl.	1	PSC	5707292139573
18	103128	Spring retainer anti tilt	1	PSC	5707292568069
19	0310129	pressure spring 0,9x11x26mmx7v	1	PSC	5707292438782
20	0603455	Insex M8x40, fzb, st 12.9	1	PSC	5707292482853

Anti Tip Indoor - for Hydraulic size 2 & 3

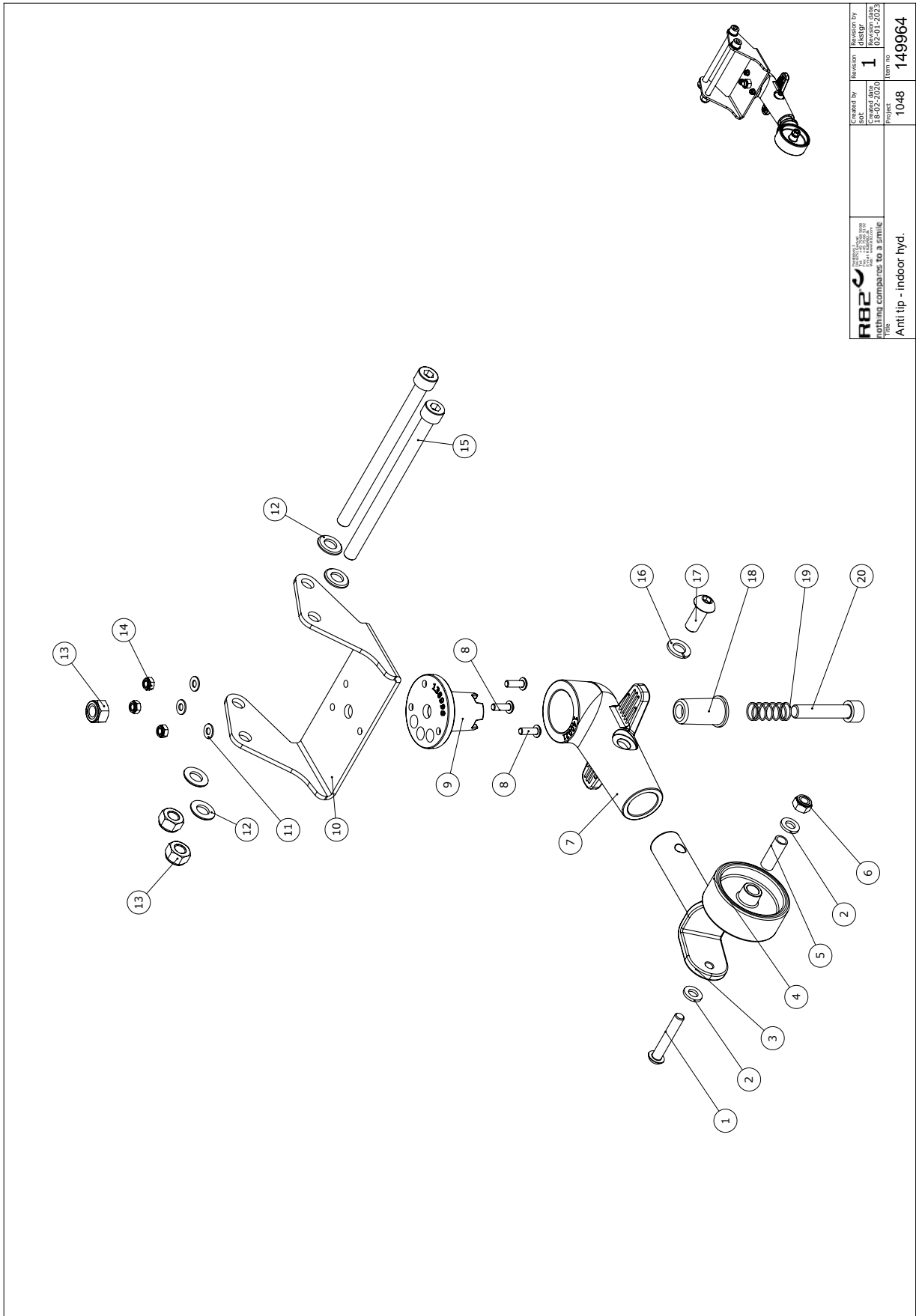


 R82 A/S Parallelvej 3 8751 Gedved Denmark	Created by 18-02-2020	Revision 1	Revision by dlestrp
	Project 1048	Item no 149964	Date 02-01-2023
Title Anti tip - indoor hyd.			

C:\BPP\Produktion\149964

Item numbers for High-low:xo Anti Tip Indoor- for Hydraulic size 2 & 3

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0659585	Button head M6x35, stainless	1	PSC	5707292139498
2	1105060	SS washer M6 6,4x12x1,6	2	PSC	5707292140357
3	141344	Wheel mount anti tilt right	1	PSC	5707292882103
4	303610	Wheel 50x19 Ø8 PU	1	PSC	5707292612748
5	111855	Bush for anti tilt device	1	PSC	5707292592989
6	1631060	Stainless lock nut 6mm	1	PSC	5707292141446
7	140021	Base for tip brace	1	PSC	5707292910882
8	0601900	Button head M4x12 FZB	3	PSC	5707292908285
9	138998	Anti tip base	1	PSC	5707292910875
10	149962	Bracket for anti hyd.	1	PSC	5707292972279
11	1102800	Fazet washer 4mm	3	PSC	5707292140265
12	1103200	Fazet washer 8x16x1,5mm, FZB	4	PSC	5707292140296
13	1607000	Lock nut 8mm, large pack	3	PSC	5707292141378
14	1606275	Lock nut 4mm, large pack	3	PSC	5707292141347
15	0603120	Cyl. Head screw M8x120 zinck	2	PSC	5707292972415
16	1101050	Spring washer M8, wavy FZB	1	PSC	5707292458667
17	0662000	Button head screw 8x20 stainl.	1	PSC	5707292139573
18	103128	Spring retainer anti tilt	1	PSC	5707292568069
19	0310129	pressure spring 0,9x11x26mmx7v	1	PSC	5707292438782
20	0603455	Insex M8x40, fzb, st 12.9	1	PSC	5707292482853

Anti Tip Outdoor

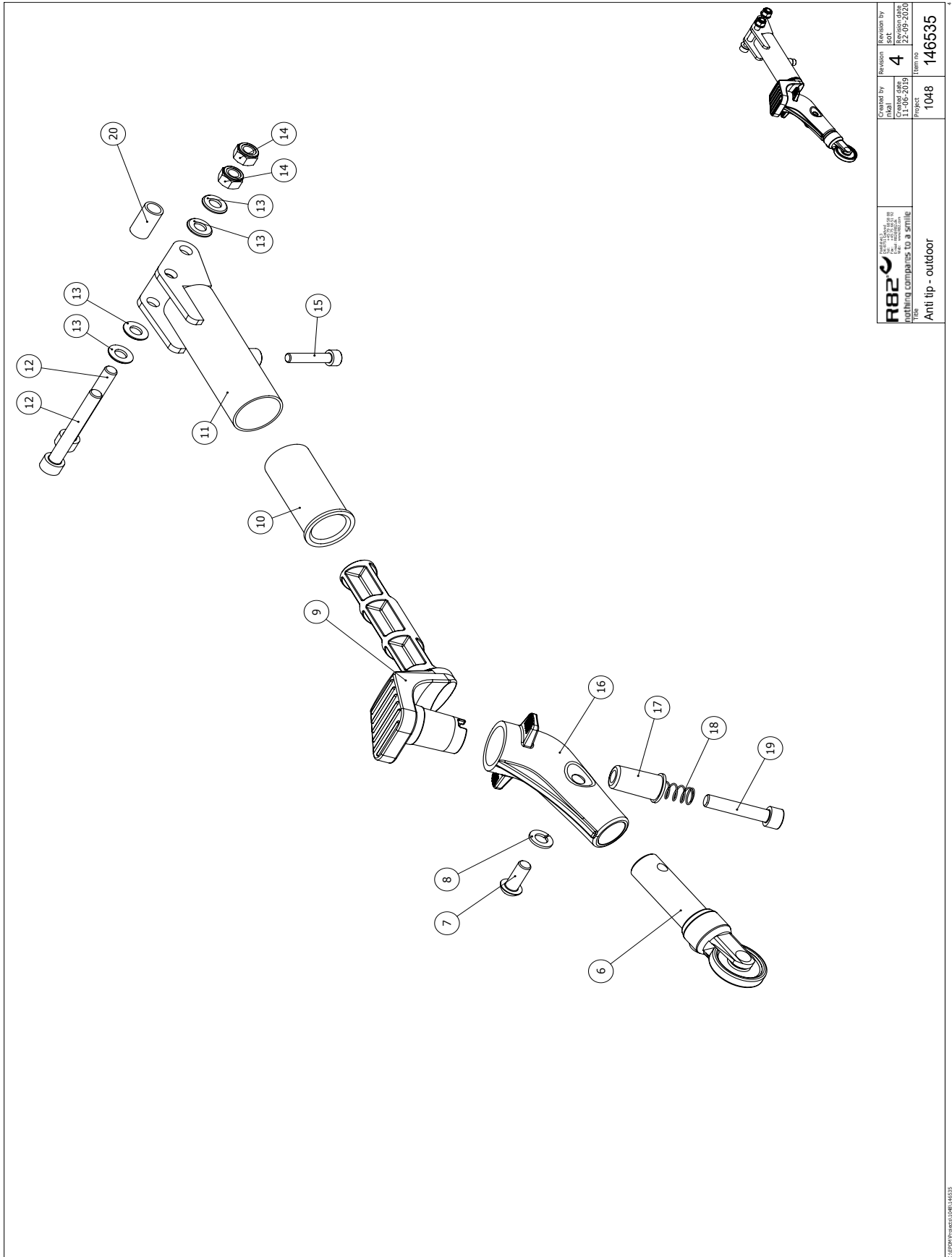


 R82 A/S Parallelvej 3 8751 Gedved Denmark	Created by mkel	Revision 4	Revision by SOT
	Created date 12-09-2015	Part no. 1048	Revision date 22-09-2020
INCLUDING COMPANIES TO A/S FINITE		Item no. 146535	
Anti tip - outdoor			

C:\APP\PROJECT\0408\146535

Item numbers for High-low:xo Anti Tip Outdoor

Position	No.	Description	Quantity	Unit of Measure	Ean code
6	5097004	Extentionbar with wheel,short	1	PCS	5707292416216
7	0662000	Button head screw 8x20 stainl.	1	PCS	5707292139573
8	1101050	Spring washer M8, wavy FZB	1	PCS	5707292458667
9	141345	Main body anti tilt device	1	PCS	5707292882110
10	100848	Plastic bush for anti tilt	1	PCS	5707292568052
11	139145	Bracket for anti tip (outdoor)	1	PCS	5707292910042
12	0603600	Insex screw 8x55	2	PCS	5707292138262
13	1103200	Fazet washer 8x16x1,5mm, FZB	4	PCS	5707292140296
14	1607000	Lock nut 8mm, large pack	2	PCS	5707292141378
15	0602401	Insex screw 6x30	1	PCS	5707292138125
16	74771	Lower part antitip	1	PCS	5707292411624
17	103128	Spring retainer anti tilt	1	PCS	5707292568069
18	0310129	Pressure spring 0,9 x 11 x 26mm x 7v	1	PCS	5707292438782
19	0603455	Insex M8x40, fzb, st 12.9	1	PCS	5707292482853
20	151459	Bush for anti tip outdoor	1	PCS	5707292920171

Threshold Assistance

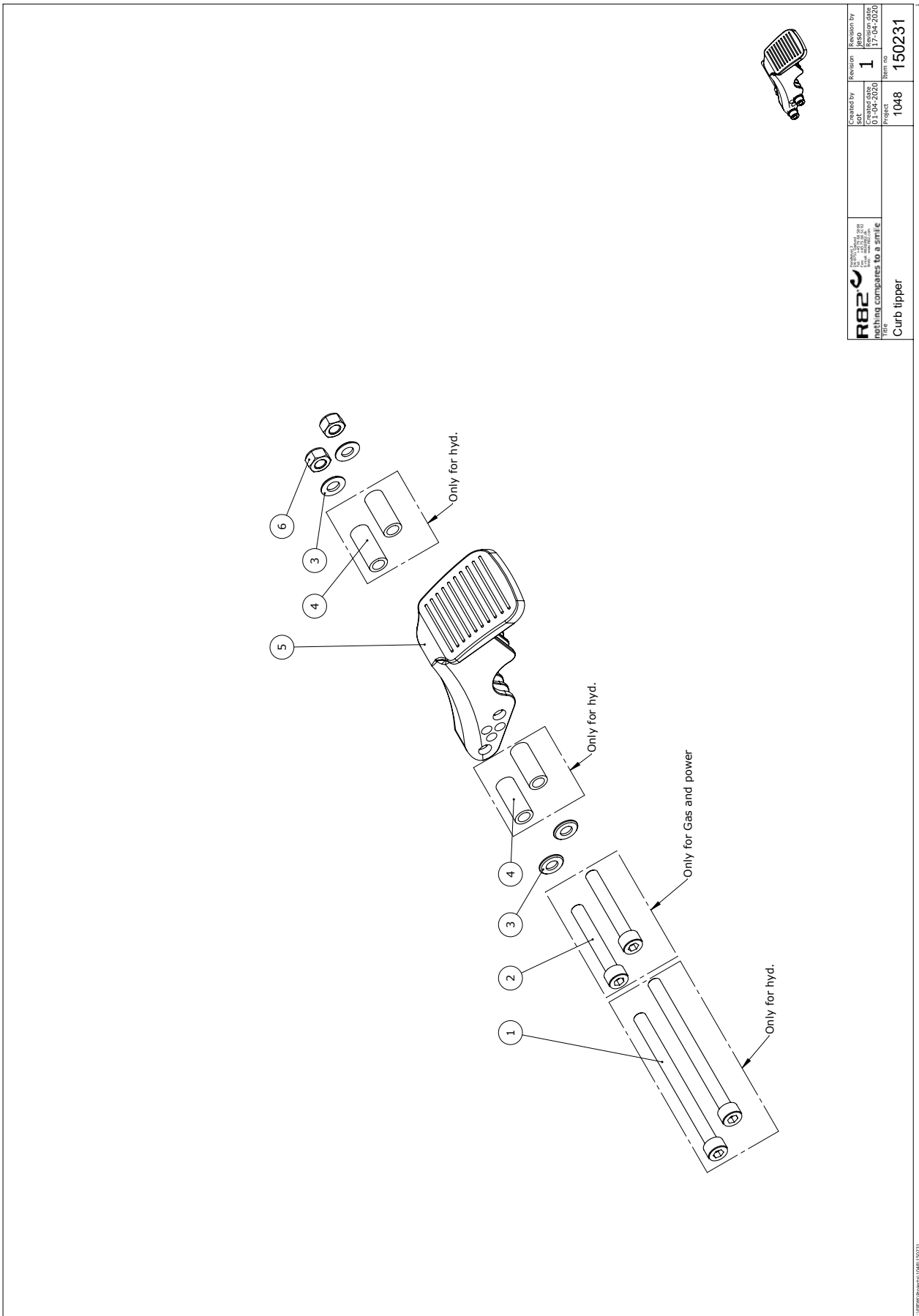

 R82 A/S Nothing compares to a smile Title: Curb tipper	Created by 01-04-2020	Revision 1	Revision by 17-04-2020
	Project 1048	Item no 150231	

C:\Users\p\Documents\150231

Item numbers for High-low:xo Threshold Assistance

Position	No.	Description	Quantity	Unit of Measure	Ean code
1 (only for hydraulic)	0603975	Insex screw 8x110	2	PCS	5707292138330
2 (only for gas & power)	0603600	Insex screw 8x55	2	PCS	5707292138262
3	1103200	Fazet washer 8x16x1,5mm, FZB	4	PCS	5707292140296
4 (only for hydraulic)	150136	Distance bush for hyd. cb.	4	PCS	5707292914491
5	139326	Tip support	1	PCS	5707292910868
6	1607000	Lock nut 8mm, large pack	2	PCS	5707292141378


Lock of Lift Gas

	Created by	Revision	Revision by
	Created date	2	Revision date
Title		Project	Item no
Lock for lift gas		1048	150266

Item numbers for High-low:xo Lock of Lift Gas

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0380175	Positioneringsbolt	1	PCS	5707292926951
2	1761000	Counter nut 10x1,0 fine thread	1	PCS	5707292141538

Transport Bracket

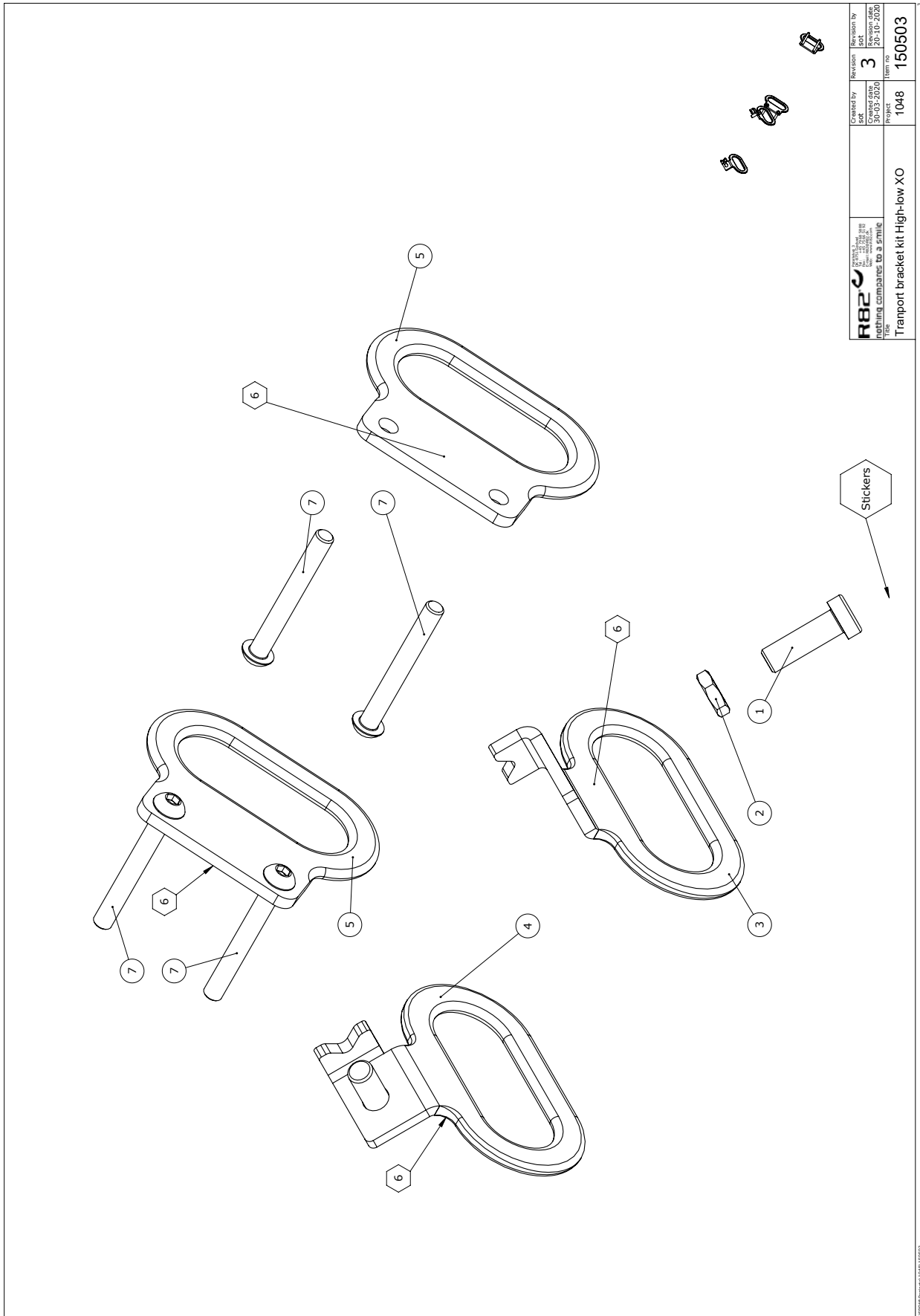

 R82 A/S Parallelvej 3 8751 Gedved Denmark nothing compares to a smile TRUK	Created by Set 30/03/2020	Revision Set 3	Revision by Set 20/10/2020
	Project 1048	Item no. 150503	Project 1048

Item numbers for High-low:xo Transport Bracket

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0603535	Insex low head M8x25	2	PCS	5707292575319
2	1750800	Lock nut, stainless M8	2	PCS	5707292141484
3	140992	Transport bracket left	1	PCS	5707292910943
4	140842	Transport bracket right	1	PCS	5707292910936
5	142368	Transport bracket rear	2	PCS	5707292911230
6	0493438	Hooksymbol yellow/black	4	PCS	5707292136916
7	0659771	Button head M6x25 FZB	4	PCS	5707292923721

Support Springs

 R82 A/S Parallelvej 3 8751 Gedved Denmark	Created by JYBA DKKK	Revision 1	Revision by JYBA DKKK
	Project 1048	Item no. 150672	Revision date 2010-07-28


CUSTOM PROJECT: 150672

Item numbers for High-low:xo Support Springs

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0603750	Insex M8x75, FZB	1	PCS	5707292138309
2	1103200	Fazet washer 8x16x1,5mm, FZB	4	PCS	5707292140296
3* (for above 25 kg)	2202815	Lift-o-Mat 150N	2	PCS	5707292912787
(for above 40 kg)	2202830	Lift-o-Mat 300N	2	PCS	5707292912794
(for above 50 kg)	2202850	Lift-o-Mat 500N	2	PCS	5707292912800
4	1607000	Lock nut 8mm, large pack	2	PCS	5707292141378
5	0603700	Insex screw 8x70	1	PCS	5707292138293
*choose size					

Communication Aid Assembly

Pos no.	Qty.	Item no.	Description.
1	1	T-Frame X-panda indoor	
2	1	139126	Communication aid bracket(size2 footrest)
3	2	0602200	M6*16-CY-FZB-A-8.8
4	1	138974	Communication Aid Bracket

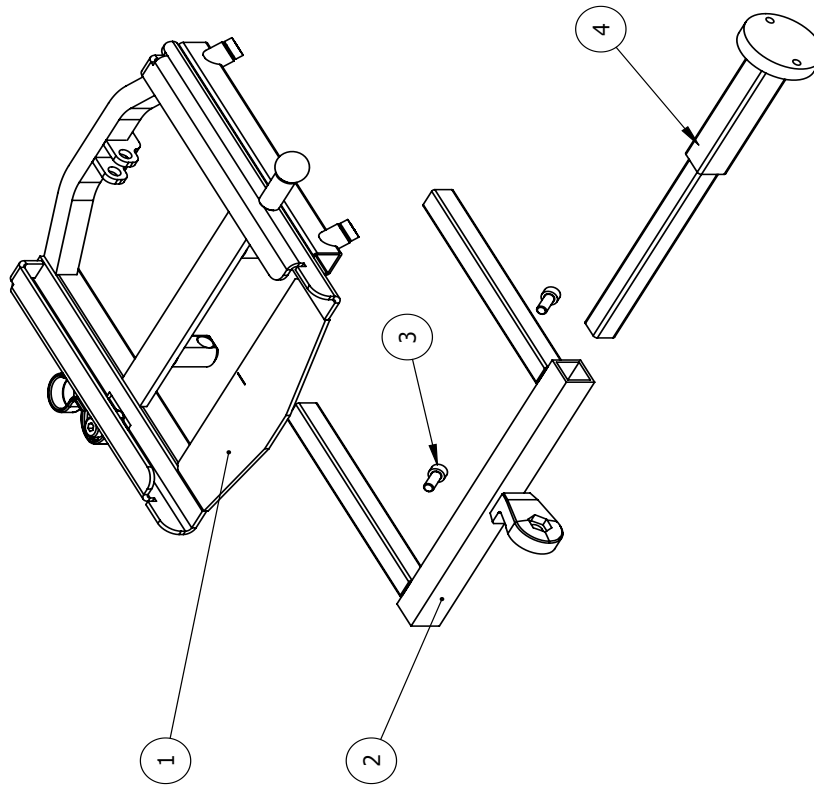



Tolerance	Project	R82 nothing compares to a smile
Surface	Tool	Created by PER
Material	Revision	PER
	Created date	30-08-2018
	Revision date	04-09-2018
	Scale	1:4
	Weight	2560g
Title	Item no	139849
Communication Aid Assembly		Format A4

Item numbers for High-low:xo Communication Aid Assembly

Position	No.	Description	Quantity	Unit of Measure	Ean code
2	139126	Communication aid bracket, size 2 footrest	1	PCS	5707292868701
3	0603530	Insex low head M8x16 stainless	2	PCS	5707292412492
4	138974	Communication Aid bracket	1	PCS	5707292845528

Bracket for Ekstra High Foot Support

Pos no.	Qty.	Item no.	Description.
1	1	0602205	M6*16-CY-SS-A-A2
3	1	82415	Rest-mounting
3	1	2506500	FRS25x15-B
4	1	2504500	FRS20x20-B

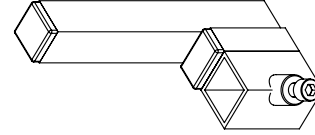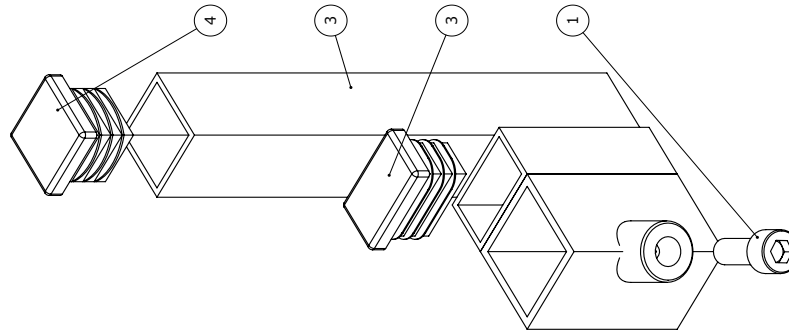


	Project	Created by	Revision	Revision by
		Jens	2	df/seo
		09/05/2020		04/10/2021
Title		Item no		
Bracket for ekstra high footsupport		85135		

Item numbers for High-low:xo Bracket for Ekstra High Foot Support

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0602900	Insex screw 8x16	1	PCS	5707292138200
2	2506500	Ferrule fr25x15x1,0-2,0 int.b.	2	PCS	5707292144201
3	82415	Rest bracket	1	PCS	5707292359933


Foot operated central brake kit, size 2 - 3, gas and power (890012-2)

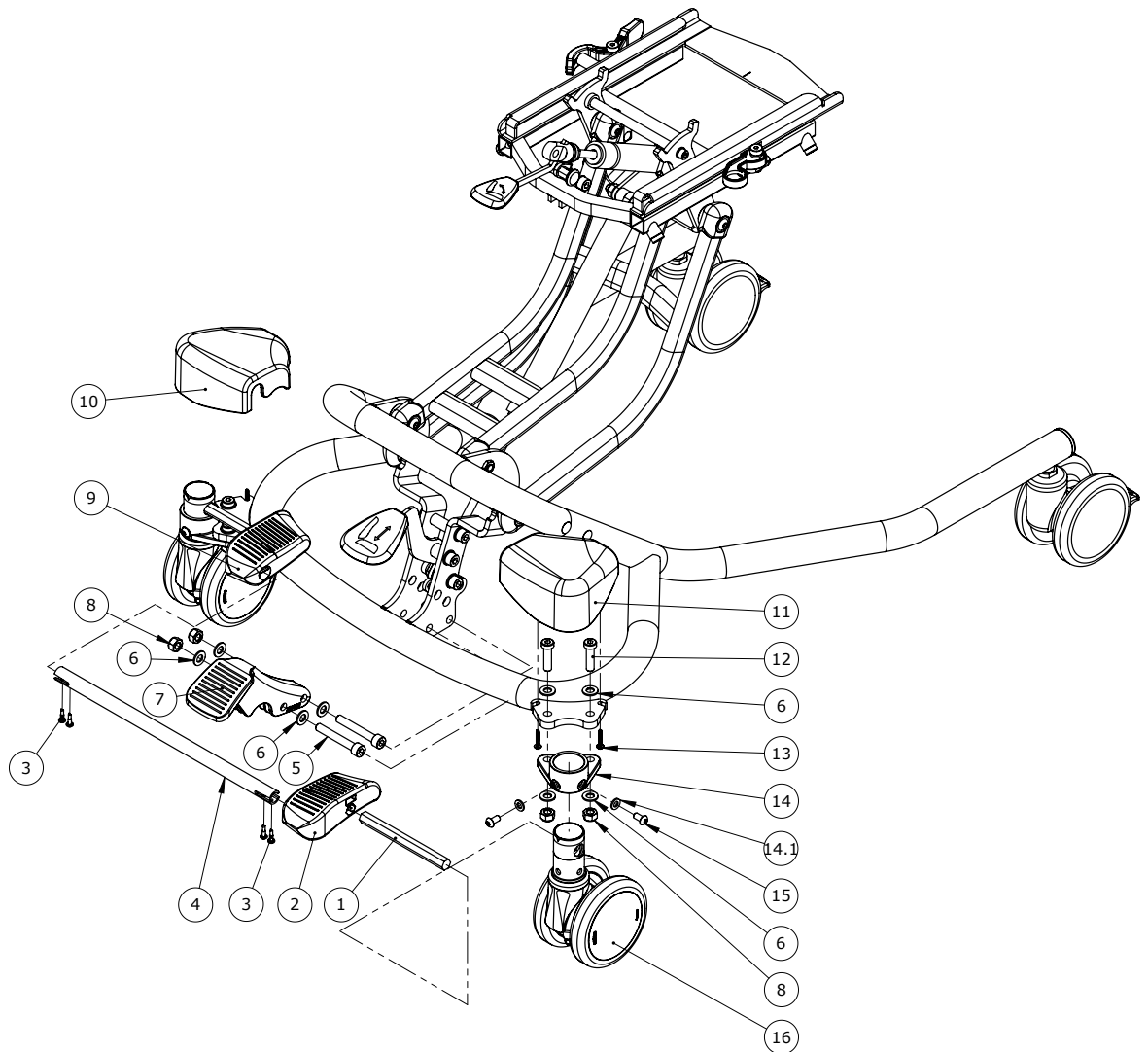

Item numbers for Foot operated central brake kit, size 2 - 3, gas and power (890012-2)

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	139809	Hexagon bar CB size 2	2	PCS	5707292911315
2	139041	Pedal for central brake R	1	PCS	5707292910844
3	1811200	Sheet metal screw 3,5x13 phx	4	PCS	5707292141712
4	138840	Tube for central brake	1	PCS	5707292911322
5	0603600	Insex screw 8x55	2	PCS	5707292138262
6	1103200	Fazet washer 8x16x1,5mm, FZB	12	PCS	5707292140296
7	139326	Tip support	1	PCS	5707292910868
8	1607000	Lock nut 8mm, large pack	6	PCS	5707292141378
9	139067	Pedal for central brake L	1	PCS	5707292910851
10	139871	Str.2/3 Central Brake Cover L	1	PCS	5707292910837
11	139814	Str.2/3 Central Brake Cover R	1	PCS	5707292910820
12	0603536	Insex lov head M8x30	4	PCS	5707292575326
13	0610620	screw ø3,5x20 DELTA PT A2	4	PCS	5707292903778
14	139818	Central brake adapter size 2	2	PCS	5707292910929
14.1	1105060	SS washer M6 6,4x12x1,6	4	PCS	5707292140357
15	0602193	Buttonhead - M6 x12, black	4	PCS	5707292900708
16	3006970	100mm dob.wheel w/central br	2	PCS	5707292877468


Foot operated central brake kit, size 3, hydraulic (890012-3)

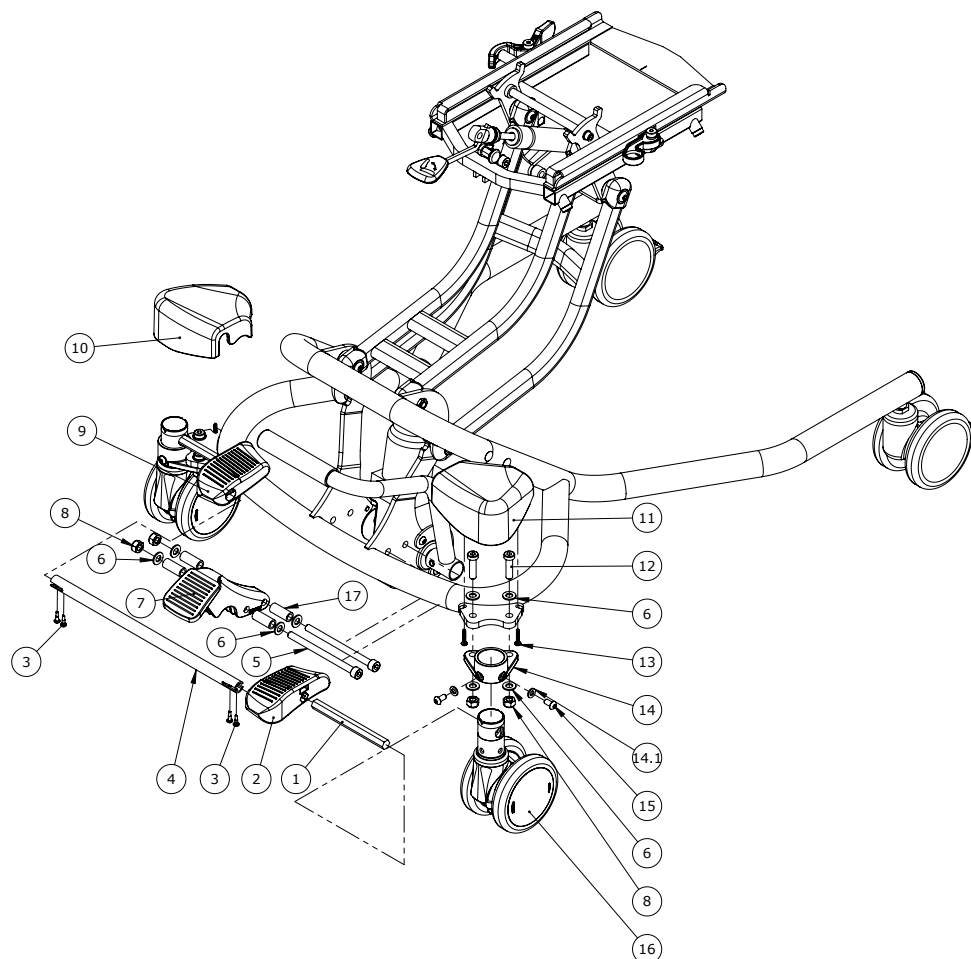

Item numbers for Foot operated central brake kit, size 3, hydraulic (890012-3)

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	139809	Hexagon bar CB size 2	2	PCS	5707292911315
2	139041	Pedal for central brake R	1	PCS	5707292910844
3	1811200	Sheet metal screw 3,5x13 phx	4	PCS	5707292141712
4	138840	Tube for central brake	1	PCS	5707292911322
5	0603975	Insex screw 8x110	2	PCS	5707292138330
6	1103200	Fazet washer 8x16x1,5mm, FZB	12	PCS	5707292140296
7	139326	Tip support	1	PCS	5707292910868
8	1607000	Lock nut 8mm, large pack	6	PCS	5707292141378
9	139067	Pedal for central brake L	1	PCS	5707292910851
10	139871	Str.2/3 Central Brake Cover L	1	PCS	5707292910837
11	139814	Str.2/3 Central Brake Cover R	1	PCS	5707292910820
12	0603536	Insex lov head M8x30	4	PCS	5707292575326
13	0610620	screw ø3,5x20 DELTA PT A2	4	PCS	5707292903778
14	139818	Central brake adapter size 2	2	PCS	5707292910929
14.1	1105060	SS washer M6 6,4x12x1,6	4	PCS	5707292140357
15	0602193	Buttonhead - M6 x12, black	4	PCS	5707292900708
16	3006970	100mm dob.wheel w/central br	2	PCS	5707292877468
17	150136	Distance bush for hyd. cb.	4	PCS	5707292914491

Adaptor for x:panda shape size 3, kit 905090

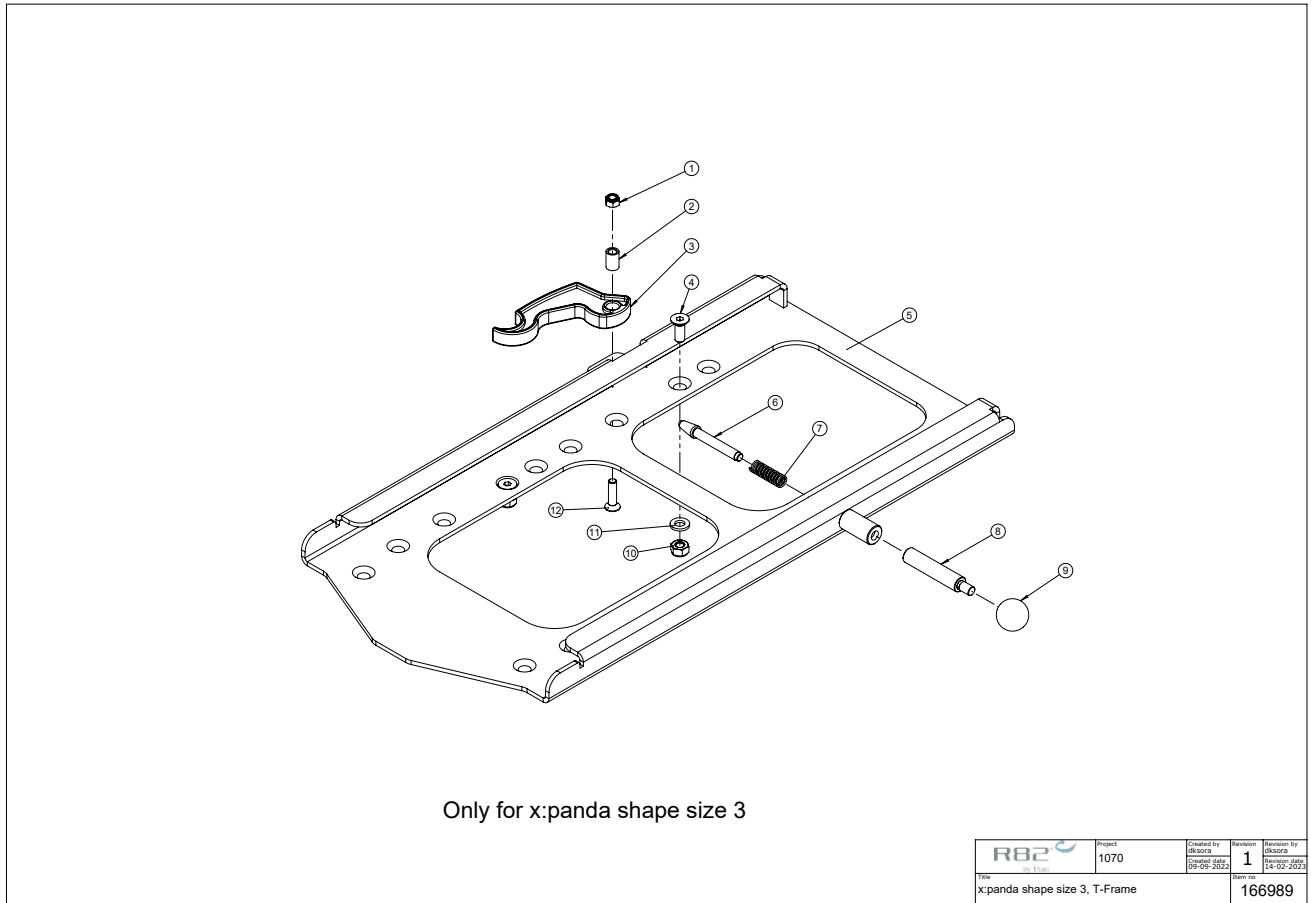

Item numbers for Adaptor for x:panda shape size 3, kit 905090

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1606300	Lock nut 5 mm	1	PCS	5707292141354
2	75641	Bush 5-8x11,8	1	PCS	5707292466501
3	75905	Clamping grip	1	PCS	5707292534590
4	0621600	Insex screw counters. 6x16	4	PCS	5707292138965
5	166990	T-Frame x:panda shape size 3	1	PCS	5707292976130
6	6113000	Spindle, t-part/frame	1	PCS	5707292186362
7	0310100	Pressure spring 0,9x8x23x10,5v	1	PCS	5707292135513
8	6118750	Handle x:panda	1	PCS	5707292530509
9	3502008	Knob handle Ø20 M6	1	PCS	5707292146830
10	1606500	Lock nut 6 mm	4	PCS	5707292141361
11	1105060	SS washer M6 6,4x12x1,6	4	PCS	5707292140357
12	0615020	Insex screw counters. 5x20	1	PCS	5707292138866