

Kudu[®]

Exploded views / Part lists

Rev. date (2023.10)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.



Contents

Kudu with 20"/22"/24" rear wheels (101182).....	4
Item numbers for Kudu with 20"/22"/24" rear wheels (101182).....	5
Kudu with 20"/22"/24" rear wheels DB/HB (111503).....	7
Item numbers for Kudu with 20"/22"/24" rear wheels DB/HB (111503).....	8
Kudu with 12½" drum brake rear wheels (111504).....	11
Item numbers for Kudu with 12½" drum brake rear wheels (111504).....	12
Kudu with 16" drum brake rear wheels (111505).....	14
Item numbers for Kudu with 16" drum brake rear wheels (111505).....	15
Frame assembly 22"/24" (113933).....	17
Item numbers for Frame assembly 22"/24" (113933).....	18
Frame assembly 12½"/16" (111718).....	19
Item numbers for Frame assembly 12½"/16" (111718).....	20
Frame assembly 22"/24" Straight (118197-x).....	21
Item numbers for Frame assembly 22"/24" Straight (118197-x).....	22
Seat assembly size 1 (115022-1), size 2 (115022-2).....	23
Item numbers for Seat assembly size 1 (115022-1), size 2 (115022-2).....	23
Seat assembly size 3 (115023-3), size 4 (115023-4).....	24
Item numbers for Seat assembly size 3 (115023-3), size 4 (115023-4).....	24
Back assembly (138658).....	25
Item numbers for Back assembly (138658).....	26
Foot plate, centrally mounted, size 1 (95152-160), size 2 (95152-260).....	27
Item numbers for Foot plate, centrally mounted, size 1 (95152-160), size 2 (95152-260).....	27
Foot plate, centrally mounted, size 3 (95152-360), size 4 (95152-460).....	28
Item numbers for Foot plate, centrally mounted, size 3 (95152-360), size 4 (95152-460).....	28
Foot rests, parted, size 2 (95151-260).....	29
Item numbers for Foot rests, parted, size 2 (95151-260).....	29
Foot rests, parted, size 3 (95151-360) size 4 (95151-460).....	30
Item numbers for Foot rests, parted, size 3 (95151-360) size 4 (95151-460).....	30
Push brace, angle adjustable (111629).....	31
Item numbers for Push brace, angle adjustable (111629).....	31
Height adjustable kit for angle adjustable push brace (95630-10).....	32
Item numbers Height adjustable kit for angle adjustable push brace (95630-10).....	32

Push handles, set (95370-10)	33
Item numbers Push handles, set (95370-10).....	33
Wheel brake, right, size 1-2 (111859)	34
Item numbers for Wheel brake, right, size 1-2 (111859).....	34
Wheel brake, left, size 1-2 (111860)	35
Item numbers for Wheel brake, left, size 1-2 (111860).....	35
Wheel brake, right, size 3-4 (121228)	36
Item numbers for Wheel brake, right, size 3-4 (121228).....	36
Wheel brake, left, size 3-4 (121229)	37
Item numbers for Wheel brake, left, size 3-4 (121229).....	37
Wheel brake, right, 12" (135654)	38
Item numbers for Wheel brake, right, 12" (135654)	38
Wheel brake, left, 12" (135652)	39
Item numbers for Wheel brake, left, 12" (135652)	39
Wheel brake, right, 16" (169538)	40
Item numbers for Wheel brake, right, 16" (169538).....	40
Wheel brake, left, 16" (169537).....	41
Item numbers for Wheel brake, left, 16" (169537).....	41

Item numbers for Kudu with 20"/22"/24" rear wheels (101182)


Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	0609729	Stainless insex 6x16	4	PCS	5707292435330
2*	113933-XXX	Frame assembly	-	-	-
	118197-XXX	Frame assembly	-	-	-
3.1	121228	Wheel brake, right side	1	STK	5707292623898
3*	121228	Wheel brake size 3-4, right	1	PCS	5707292583086
	111859	Wheel brake size 1-2, right	1	PCS	5707292623898
4	2985065	Compl. Stainless quick-release	2	PCS	5707292145345
5.1	121229	Wheel brake, left side	1	STK	5707292623904
5	121229	Wheel brake size 3-4 left	1	PCS	5707292583154
6*	950958-1	Anti-tip Kudu, long, reinforced	1	SET	5707292867551
	950953-1	Anti-tip device Kudu, reinforced	1	SET	5707292867544
7*	3202210	22" wheel no brake, air, black	2	PCS	5707292584090
	3202410	24" wheel, no brake, air, black	2	PCS	5707292584137
	3102015	20" wheel no brake flexel	2	PCS	5707292584052
	3102415	24" wheel, no brake, mas, black	2	PCS	5707292837653
	3102215	22" hjul no brake flexel	2	PCS	5707292837646
	3202010	20" wheel, no brake, air, black	2	PCS	5707292584045
8*	P-95151-460	Footsupport complete parted str.4	1	SET	5707292588333
	P-95151-360	Footsupport complete parted str.3	1	SET	5707292588326
	P-95151-260	Footsupport complete parted str.2	1	SET	5707292588258
9-12	See Pricelist	-	-	-	-
13*	95744-260	Cushion covering size 2	1	PCS	5707292586834
	95744-460	Cushion covering size 4	1	PCS	5707292586858
	95744-360	Cushion covering sz. 3	1	PCS	5707292586841
	95744-160	Cushion covering size 1	1	PCS	5707292586827
	95744-060	Cushion covering size 0	1	PCS	5707292621610
14	115023-X	Seat assemblies	-	-	-
15	0661500	Button head screw 8x16 stainl.	1	PCS	5707292139559
16	1105080	Stainless washer 8mm	5	PCS	5707292140364
17	1631080	Stainless lock nut 8mm	4	PCS	5707292141453
18*	P-95152-360	Footrest size 3	1	PCS	5707292588371
	P-95152-460	Footrest size 4	1	PCS	5707292588388
	P-95151-460	Footrest complete parted str.4	1	SET	5707292588333
	P-95152-260	Footrest size 2	1	PCS	5707292588364
	P-95152-160	Footrest size 1	1	PCS	5707292588340
19	1350015	Dist.piece, black 7xØ15xØ8,2mm	2	PCS	5707292467225
20	0609835	Stainless insex screw 8x35	1	PCS	5707292138606
21*	2207080-3	Springstopper adjuster s. 3+4	1	PCS	5707292632845
	2207070-3	Springstopper adjuster size 2	1	PCS	5707292632838
	2207060-3	Springstopper adjuster size 1	1	PCS	5707292632821
21.1*	136357	Fogim spring stopper size 1	1	PCS	5707292829443
	136355	Fogim spring stopper size 2	1	PCS	5707292829436
	136089	Fogim spring stopper size 3-4	1	PCS	5707292825353
22	0661145	Button head 8x45 FZB	2	PCS	5707292139535
23	1310711	Nylon washer 15x8,4x1,5mm	2	PCS	5707292429933
24	138658	Back assemblies size 1-4 See page 23	1	PCS	-
25	111629	Push brace size 1-4 See page 29	1	PCS	-

*Choose size, color or type

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
26*	95370-10	Push handles	1	SET	5707292586568
	95629-2	Push brace size 2	1	PCS	5707292586537
27	3503531	Gas pump lever with clamp	1	PCS	5707292413529
28	0371853	Strained wires 1,5mm 190 cm.	1	PCS	5707292135889
29*	5002998	Cable for new rabbit, 39 cm	1	PCS	5707292436016
	5002756	Cable for recline, outer 1-4	1	PCS	5707292413673
30	0388006	Screw nipple 5x7,5mm	3	PCS	5707292136114
31	136474-X	Angle adjust. Push brace See page 31	-	-	-
33*	5002756	Cable for recline, outer 1-4	2	PCS	5707292413673
	5001600	Cable for back new Rabbit	2	PCS	5707292154491
34	0371854-1	Soft inner cable for Stingray	2	PCS	5707292536945
35	116344	Fixed back recline size 3-4	1	PCS	5707292618214
36	1310528	Covering stud, black 13mm	2	PCS	5707292470348
37	5002751	Cable for brake, size 3+4	1	PCS	5707292413642
38	2985000	Pressureplug black soft rubber	2	PCS	5707292145314
39	1608000	Lock nut 12mm, large pack	2	PCS	5707292141392
40	6108150	Spring screw 29,57x17,48	2	PCS	5707292378200
41	111853	Stop washer \varnothing 12,2x \varnothing 20x3,7	2	PCS	5707292587374
42	2985210	Ball bearing 02 6001 2rs	4	PCS	5707292379412
43	100643	Distance Bush	2	PCS	5707292552105
44*	1604500	Steel nut 12mm, large pack	2	PCS	5707292141309
	1760012	Kontranut M12,fzb	2	PCS	5707292519078
45	109378	Wheel protector	4	PCS	5707292581303
46	1907150	Steel bolt 8x75mm	2	PCS	5707292142474
47	77980	FRONT FORK 20 - 22"	2	PCS	5707292534132
48	1902372	Steel set screw M12x55 FZB	2	PCS	5707292519436
49	1105080	Stainless washer 8mm	4	PCS	5707292140364
50	1631080	Stainless lock nut 8mm	2	PCS	5707292141453
51*	3007901	Wheel Kudu, 7" - tyre black	2	PCS	5707292587343
	3007736-1	Wheel 140x40mm with rubber	2	PCS	5707292558213
52*	3007901	Wheel Kudu, 7" - tyre black	2	PCS	5707292587343
	3007902	Wheel, 7" Kudu , tyre black	2	PCS	5707292587350
75*	2983510	22" spokes protect, Kudu	2	PCS	5707292575975
	2983520	24" spokes protect, Kudu	2	PCS	5707292575982
	2983500	20" spokes protect, Kudu	2	PCS	5707292575906
76	2982401	Clips for spoke guards, black	6	PCS	5707292427670
77	0494524	Refleks \varnothing 22mm grey	8	PCS	5707292623911
78	0493437	Label "exclamation mark"	6	PCS	5707292136909
79	1103500	Fazet washer 12mm	2	PCS	5707292140326
80*	118151	Antitip Mounting stud +/- 6 g	2	PCS	5707292614315
	139560	Extended Antitip Mount.stud	2	PCS	5707292867520

*Choose size, color or type

Kudu with 20"/22"/24" rear wheels DB/HB (111503)

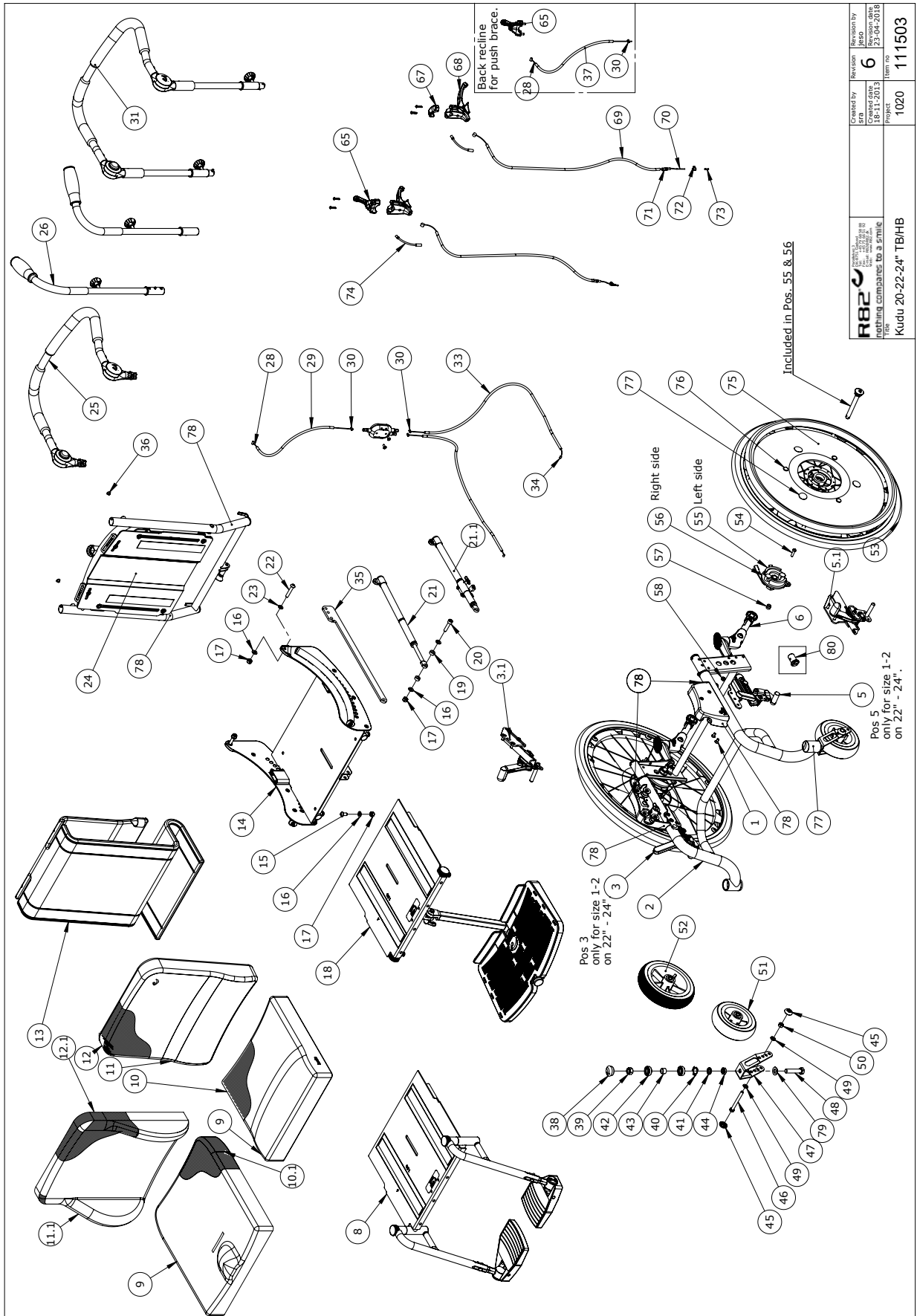

<p>nothing compares to a smile</p>	Created by	Revision	Revision by
	18-11-2012	6	23-04-2018
Project	1020	Item no.	111503
Title		Kudu 20-22-24" TB/HB	

Item numbers for Kudu with 20"/22"/24" rear wheels DB/HB (111503)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	0609729	Stainless insex 6x16	4	STK	5707292435330
2	118197-XXX	Frame assemblies	-	-	-
3.1	121228	Wheel brake, right side	1	STK	5707292623898
4	2985065	Compl. Stainless quick-release	2	STK	5707292145345
5	111860	Wheel brake size 1-2, left	1	PCS	5707292587671
5.1	121229	Wheel brake, left side	1	STK	5707292623904
6*	950958-1	Anti-tip Kudu, long, reinforced	1	SÆT	5707292867551
	950953-1	Anti-tip device Kudu, reinforced.	1	SÆT	5707292867544
8*	P-95151-460	Footsupport complete parted str.4	1	SET	5707292588333
	P-95151-360	Footsupport complete parted str.3	1	SET	5707292588326
	P-95151-260	Footsupport complete parted str.2	1	SET	5707292588258
9-12	See Pricelist	-	-	-	-
13*	95744-460	Cushion covering size 4	1	PCS	5707292586858
	95744-360	Cushion covering sz. 3	1	PCS	5707292586841
	95744-260	Cushion covering size 2	1	PCS	5707292586834
	95744-160	Cushion covering size 1	1	PCS	5707292586827
	95744-060	Cushion covering size 0	1	PCS	5707292621610
14	115023-4	Seat assemblies	-	-	-
15	0661500	Button head screw 8x16 stainl.	1	STK	5707292139559
16	1105080	Stainless washer 8mm	5	STK	5707292140364
17	1631080	Stainless lock nut 8mm	4	STK	5707292141453
18*	P-95152-460	Footrest size 4	1	PCS	5707292588388
	P-95152-360	Footrest size 3	1	PCS	5707292588371
	P-95152-260	Footrest size 2	1	PCS	5707292588364
	P-95152-160	Footrest size 1	1	PCS	5707292588340
19	1350015	Dist.piece, black 7xØ15xØ8,2mm	2	STK	5707292467225
20	0609835	Stainless insex screw 8x35	1	STK	5707292138606
21*	2207080-3	Springstopper adjuster s. 3+4	1	PCS	5707292632845
	2207070-3	Springstopper adjuster size 2	1	PCS	5707292632838
	2207060-3	Springstopper adjuster size 1	1	PCS	5707292632821
21.1*	136089	Fogim spring stopper size 3-4	1	PCS	5707292825353
	136355	Fogim spring stopper size 2	1	PCS	5707292829436
	136357	Fogim spring stopper size 1	1	PCS	5707292829443
22	0661145	Button head 8x45 FZB	2	STK	5707292139535
23	1310711	Nylon washer 15x8,4x1,5mm	2	STK	5707292429933
24	138658	Back assemblies size 1-4 See page 23	1	PCS	-
25	111629	Push brace size 1-4 See page 29	1	PCS	-
26	95370-10	Push handles	1	SÆT	5710000000000
28	0371853	Strained wires 1,5mm 190 cm.	2	STK	5707292135889
29*	5002756	Cable for recline, outer 1-4	1	STK	5707292413673
	5002998	Cable for new rabbit, 39 cm	1	STK	5707292436016
30	0388006	Screwnipple 5x7,5mm	3	STK	5707292136114
31	136474-X	Angle adjust. Push brace See page 31	-	-	-
33	5002756	Cable for recline, outer 1-4	2	STK	5707292413673
34	0371854-1	Soft inner cable for Stingray	2	STK	5707292536945
35	116344	Fixed back recline size 3-4	1	STK	5710000000000
36	1310528	Covering stud, black 13mm	2	STK	5707292470348

*Choose size, color or type


Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
37*	5002747	Outer casing for break	1	STK	5707292415899
	5002751	Cable for brake, size 3+4	1	STK	5710000000000
38	2985000	Pressureplug black soft rubber	2	STK	5707292145314
39	1608000	Lock nut 12mm, large pack	2	STK	5707292141392
40	6108150	Spring screw 29,57x17,48	2	STK	5707292378200
41	111853	Stop washer \varnothing 12,2x \varnothing 20x3,7	2	STK	5707292587374
42	2985210	Ball bearing 02 6001 2rs	4	STK	5707292379412
43	100643	Distance Bush	2	STK	5707292552105
44*	1760012	Kontranut M12,fzb	2	STK	5707292519078
	1604500	Steel nut 12mm, large pack	2	STK	5707292141309
45	109378	Wheel protector	4	STK	5707292581303
46	1907150	Steel bolt 8x75mm	2	STK	5707292142474
47	77980	FRONT FORK 20 - 22"	2	STK	5707292534132
48	1902372	Steel set screw M12x55 FZB	2	STK	5707292519436
49	1105080	Stainless washer 8mm	4	STK	5707292140364
50	1631080	Stainless lock nut 8mm	2	STK	5707292141453
51	3007736-1	Wheel 140x40mm with rubber	2	STK	5707290000000
52*	3007901	Wheel Kudu, 7" - tyre black	2	STK	5707292587343
	3007736-1	Wheel 140x40mm with rubber	2	STK	5710000000000
	3007902	Wheel, 7" Kudu , tyre black	2	STK	5710000000000
53*	3102425	24" wheel, with brake,black	2	STK	5707292837622
	3102225	22" wheel W/db massive	2	STK	5707292837615
	3202220	22" wheel, db, air, black	2	STK	5707290000000
	3102425	24" wheel, with brake,black	2	STK	5710000000000
	3202420	24" wheel, db, air, black	2	STK	5710000000000
	3202020	20" wheel w/db air	2	STK	5710000000000
	3102025	20" wheel w/db flexel	2	STK	5710000000000
54	0659570	Button head screw 6x25 stainl.	2	STK	5707292139474
55	2983082	Brake anchor w/QR left	1	STK	5707292599452
56	2983083	Brake anchor w/QR right	1	STK	5707292599469
57	104387	Spacer	4	STK	5707292582553
58	1631060	Stainless lock nut 6mm	2	STK	5707292141446
65+66	3503532	Gas pump lever without clamp	2	STK	5707292427755
67	3503536	Clamp w/screws	1	STK	5707292427762
68*	3503535	Brake parking lever wo/clamp	2	STK	5707292413536
	3503543	Brake driving lever,Black/Grey	2	STK	5707292611512
69*	5002751	Cable for brake, size 3+4	2	STK	5707292413642
	5002750	Cable for brake, size 1+2	2	STK	5707292413635
70	0371853	Strained wires 1,5mm 190 cm.	2	STK	5707292135889
71	0380000	Cable tighter M5x23	2	STK	5707292135933
72	0388010	Screwnippel 7x7mm	2	STK	5707292136121
73	0387000	End nipple for outer casing	2	STK	5707292136091
74	0380060	Flexible v-brake pipe.	2	STK	5707292425195
75*	2983520	24" spokes protect, Kudu	2	STK	5707292575982
	2983510	22" spokes protect, Kudu	2	STK	5707292575975
	2983520	24" spokes protect, Kudu	2	STK	5710000000000
	2983500	20" spokes protect, Kudu	2	STK	5710000000000
76	2982401	Clips for spoke guards, black	6	STK	5707292427670
77	0494524	Refleks \varnothing 22mm grey	8	STK	5707292623911

*Choose size, color or type

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
78	0493437	Label "exclamation mark"	6	STK	5707292136909
80	139560	Extended Antitip Mount.stud	2	STK	5707292867520

*Choose size, color or type

Kudu with 12½" drum brake rear wheels (111504)

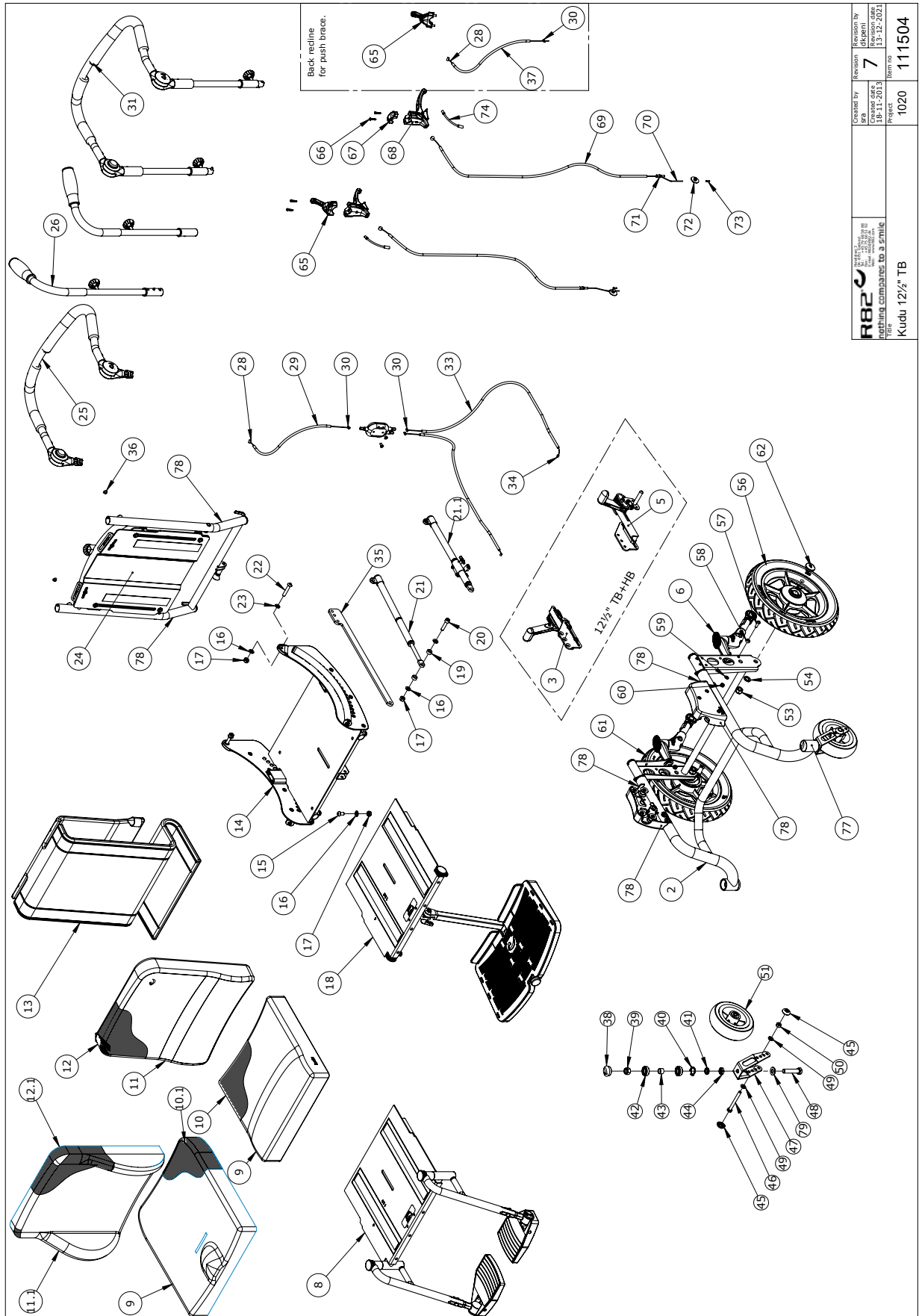

 R82 A/S Kudu 12½" TB	Created by SKK	Revision 7	Drawn by dkppn
	Created date 18-11-2013	Revision date 13-12-2021	Part no. 111504
Part name Kudu 12½" TB		Part no. 1020	Part no. 111504

© 2021 R82 A/S. All rights reserved.

Item numbers for Kudu with 12½" drum brake rear wheels (111504)


Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
2	111718-110	Frame assembly size 1, white	1	PCS	5707292585615
3	P-135654	Wheel brake 12,5" right side	1	PCS	5707292904324
5	P-135652	Wheel brake 12,5" left side	1	PCS	5707292904317
6*	950958-1	Anti-tip Kudu, long, reinforced	1	SET	5707292867551
	950953-1	Anti-tip device Kudu, reinforced	1	SET	5707292867544
8*	P-95151-460	Footsupport complete parted str.4	1	SET	5707292588333
	P-95151-360	Footsupport complete parted str.3	1	SET	5707292588326
	P-95151-260	Footsupport complete parted str.2	1	SET	5707292588258
9-12	See Pricelist	-	-	-	-
13*	95744-460	Cushion covering size 4	1	PCS	5707292586858
	95744-360	Cushion covering sz. 3	1	PCS	5707292586841
	95744-260	Cushion covering size 2	1	PCS	5707292586834
	95744-160	Cushion covering size 1	1	PCS	5707292586827
	95744-060	Cushion covering size 0	1	PCS	5707292621610
14	115023-X	Seat assemblies	-	-	-
15	0661500	Button head screw 8x16 stainl.	1	PCS	5707292139559
16	1105080	Stainless washer 8mm	5	PCS	5707292140364
17	1631080	Stainless lock nut 8mm	4	PCS	5707292141453
18*	P-95152-460	Footrest size 4	1	PCS	5707292588388
	P-95152-360	Footrest size 3	1	PCS	5707292588371
	P-95152-260	Footrest size 2	1	PCS	5707292588364
	P-95152-160	Footrest size 1	1	PCS	5707292588340
19	1350015	Dist.piece, black 7xØ15xØ8,2mm	2	PCS	5707292467225
20	0609835	Stainless insex screw 8x35	1	PCS	5707292138606
21*	2207080-3	Springstopper adjuster s. 3+4	1	PCS	5707292632845
	2207070-3	Springstopper adjuster size 2	1	PCS	5707292632838
	2207060-3	Springstopper adjuster size 1	1	PCS	5707292632821
21.1*	136089	Fogim spring stopper size 3-4	1	PCS	5707292825353
	136355	Fogim spring stopper size 2	1	PCS	5707292829436
	136357	Fogim spring stopper size 1	1	PCS	5707292829443
22	0661145	Button head 8x45 FZB	2	PCS	5707292139535
23	1310711	Nylon washer 15x8,4x1,5mm	2	PCS	5707292429933
24	138658	Back assemblies size 1-4 See page 23	1	PCS	-
25	111629	Push brace size 1-4 See page 29	1	PCS	-
26*	95370-10	Push handles	1	SET	5707292586568
	95629-2	Push brace size 2	1	PCS	5707292586537
28	0371853	Strained wires 1,5mm 190 cm.	1	PCS	5707292135889
29	5002998	Cable for new rabbit, 39 cm	1	PCS	5707292436016
30	0388006	Screwnipple 5x7,5mm	3	PCS	5707292136114
31	136474-X	Angle adjust. Push brace See page 31	-	-	-
33	5001600	Cable for back new Rabbit	2	PCS	5707292154491
34	0371854-1	Soft inner cable for Stingray	2	PCS	5707292536945
35	116344	Fixed back recline size 3-4	1	PCS	5707292618214
36	1310528	Covering stud, black 13mm	2	PCS	5707292470348
37	5002751	Cable for brake, size 3+4	1	PCS	5707292413642
38	2985000	Pressureplug black soft rubber	2	PCS	5707292145314
39	1608000	Lock nut 12mm, large pack	2	PCS	5707292141392
40	6108150	Spring screw 29,57x17,48	2	PCS	5707292378200

*Choose size, color or type

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
41	111853	Stop washer \varnothing 12,2x \varnothing 20x3,7	2	PCS	5707292587374
42	2985210	Ball bearing 02 6001 2rs	4	PCS	5707292379412
43	100643	Distance Bush	2	PCS	5707292552105
44	1760012	Kontranut M12,fzb	2	PCS	5707292519078
45	109378	Wheel protector	4	PCS	5707292581303
46	1907150	Steel bolt 8x75mm	2	PCS	5707292142474
47	77980	FRONT FORK 20 - 22"	2	PCS	5707292534132
48	1902372	Steel set screw M12x55 FZB	2	PCS	5707292519436
49	1105080	Stainless washer 8mm	4	PCS	5707292140364
50	1631080	Stainless lock nut 8mm	2	PCS	5707292141453
51	3007736-1	Wheel 140x40mm with rubber	2	PCS	5707292558213
53	1608000	Lock nut 12mm, large pack	2	PCS	5707292141392
54	1101600	Center spring washer 12mm	2	PCS	5707292140197
56+71+72	3201610	PU, Wheel 16" black left	1	PCS	5707292587268
57	0641620	Button head screw 6x20	2	PCS	5707292139276
58	104387	Spacer	2	PCS	5707292582553
59	1105060	Stainless washer 6mm	2	PCS	5707292140357
60	1631060	Stainless lock nut 6mm	2	PCS	5707292141446
61+71+72	3201600	PU-Wheel, black 16" right	1	PCS	5707292587237
62	2704300	Black hub cap for rolko wheel	2	PCS	5707292144515
65+66	3503532	Gas pump lever without clamp	1	PCS	5707292427755
67	3503536	Clamp w/screws	1	PCS	5707292427762
68	3503543	Brake driving lever,Black/Grey	2	PCS	5707292611512
69	5002751	Cable for brake, size 3+4	2	PCS	5707292413642
69	5002750	Cable for brake, size 1+2	2	PCS	5707292413635
70	0371853	Strained wires 1,5mm 190 cm.	2	PCS	5707292135889
73	0387000	End nipple for outer casing	2	PCS	5707292136091
74	0380060	Flexible v-brake pipe.	2	PCS	5707292425195
77	0494524	Refleks \varnothing 22mm grey	2	PCS	5707292623911
78	0493437	Label "exclamation mark"	6	PCS	5707292136909

*Choose size, color or type

Kudu with 16" drum brake rear wheels (111505)

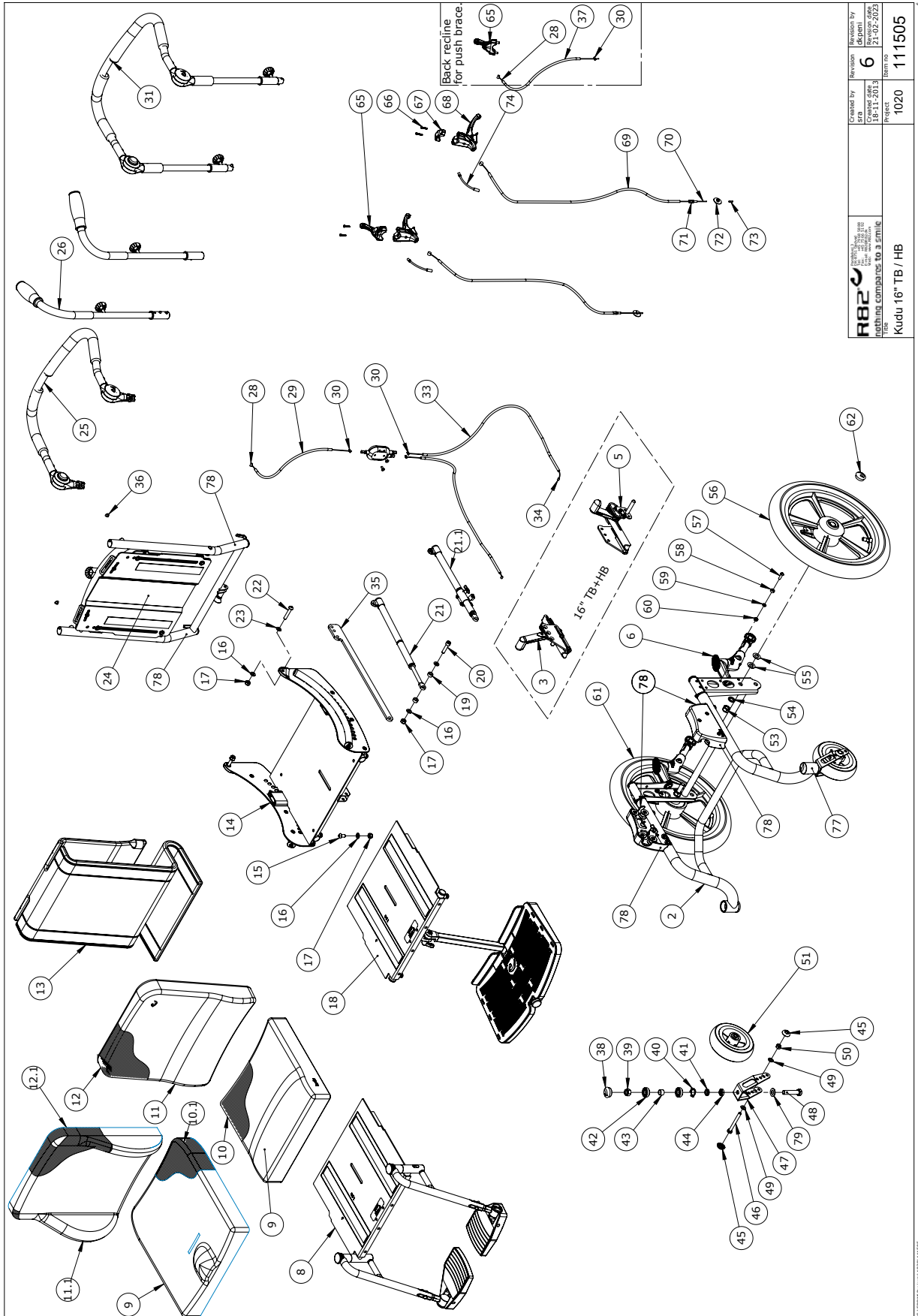

 R82 A/S Parallelvej 3 8751 Gedved Denmark	Created by srb	Checked by JDS	Revision 6
	Project 1020	Part no. 111505	Drawn by JDS

Item numbers for Kudu with 16" drum brake rear wheels (111505)


Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
2	111718-XXX	Frame assemblies	-	-	-
3	P-169538	Wheel brake 16" right	1	PCS	5707292977359
5	P-169537	Wheel brake 16" left	1	PCS	5707292977342
6*	950958-1	Anti-tip Kudu, long, reinforced	1	SET	5707292867551
	950953-1	Anti-tip device Kudu, reinforced.	1	SET	5707292867544
8*	P-95151-460	Footsupport complete parted str.4	1	SET	5707292588333
	P-95151-360	Footsupport complete parted str.3	1	SET	5707292588326
	P-95151-260	Footsupport complete parted str.2	1	SET	5707292588258
9-12	See Pricelist	-	-	-	-
13*	95744-460	Cushion covering size 4	1	PCS	5707292586858
	95744-360	Cushion covering sz. 3	1	PCS	5707292586841
	95744-260	Cushion covering size 2	1	PCS	5707292586834
	95744-160	Cushion covering size 1	1	PCS	5707292586827
	95744-060	Cushion covering size 0	1	PCS	5707292621610
14	115023-X	Seat assemblies	-	-	-
15	0661500	Button head screw 8x16 stainl.	1	PCS	5707292139559
16	1105080	Stainless washer 8mm	5	PCS	5707292140364
17	1631080	Stainless lock nut 8mm	4	PCS	5707292141453
18*	P-95152-260	Footrest size 2	1	PCS	5707292588364
	P-95152-160	Footrest size 1	1	PCS	5707292588340
19	1350015	Dist.piece,black 7xØ15xØ8,2mm	2	PCS	5707292467225
20	0609835	Stainless inset screw 8x35	1	PCS	5707292138606
21*	2207070-3	Springstopper adjuster size 2	1	PCS	5707292632838
	2207060-3	Springstopper adjuster size 1	1	PCS	5707292632821
21.1*	136355	Fogim spring stopper size 2	1	PCS	5707292829436
	136357	Fogim spring stopper size 1	1	PCS	5707292829443
22	0661145	Button head 8x45 FZB	2	PCS	5707292139535
23	1310711	Nylon washer 15x8,4x1,5mm	2	PCS	5707292429933
24	138658	Back assemblies size 1-4 See page 23	1	PCS	-
25	111629	Push brace size 1-4 See page 29	1	PCS	-
26*	95370-10	Push handles	1	SET	5707292586568
	95629-2	Push brace size 2	1	PCS	5710000000000
28	0371853	Strained wires 1,5mm 190 cm.	1	PCS	5707292135889
29*	5002756	Cable for recline, outer 1-4	1	PCS	5707292413673
	5002998	Cable for new rabbit, 39 cm	1	PCS	5707292436016
30	0388006	Screwnipple 5x7,5mm	3	PCS	5707292136114
33	5001600	Cable for back new Rabbit	2	PCS	5707292154491
33	5002756	Cable for recline, outer 1-4	2	PCS	5710000000000
34	0371854-1	Soft inner cable for Stingray	2	PCS	5707292536945
36	1310528	Covering stud, black 13mm	2	PCS	5707292470348
38	2985000	Pressureplug black soft rubber	2	PCS	5707292145314
39	1608000	Lock nut 12mm, large pack	2	PCS	5707292141392
40	6108150	Spring screw 29,57x17,48	2	PCS	5707292378200
41	111853	Stop washer Ø12,2xØ20x3,7	2	PCS	5707292587374
42	2985210	Ball bearing 02 6001 2rs	4	PCS	5707292379412
43	100643	Distance Bush	2	PCS	5707292552105
44	1760012	Kontranut M12,fzb	2	PCS	5707292519078
45	109378	Wheel protector	4	PCS	5707292581303

*Choose size, color or type

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
46	1907150	Steel bolt 8x75mm	2	PCS	5707292142474
47	77980	FRONT FORK 20 - 22"	2	PCS	5707292534132
48	1902372	Steel set screw M12x55 FZB	2	PCS	5707292519436
49	1105080	Stainless washer 8mm	4	PCS	5707292140364
50	1631080	Stainless lock nut 8mm	2	PCS	5707292141453
51	3007736-1	Wheel 140x40mm with rubber	2	PCS	5707292558213
53	1608000	Lock nut 12mm, large pack	2	PCS	5707292141392
54	1101600	Center spring washer 12mm	2	PCS	5707292140197
55	1103500	Fazet washer 12mm	4	PCS	5707292140326
57*	0641620	Button head screw 6x20	2	PCS	5707292139276
	659570	Button head screw 6x25 stainl.	2	PCS	5707292139474
58	104387	Spacer	2	PCS	5707292582553
59	1105060	Stainless washer 6mm	2	PCS	5707292140357
60	1631060	Stainless lock nut 6mm	2	PCS	5707292141446
62	2704300	Black hub cap for rolko wheel	2	PCS	5707292144515
67	3503536	Clamp w/screws	1	PCS	5707292427762
68*	3503543	Brake driving lever,Black/Grey	2	PCS	5707292611512
	3503535	Brake parking lever wo/clamp	2	PCS	5707292413536
	5002751	Cable for brake, size 3+4	2	PCS	5707292413642
69	5002750	Cable for brake, size 1+2	2	PCS	5707292413635
70	0371853	Strained wires 1,5mm 190 cm.	2	PCS	5707292135889
73	0387000	End nipple for outer casing	2	PCS	5707292136091
74	0380060	Flexible v-brake pipe.	2	PCS	5707292425195
77	0494524	Refleks Ø22mm grey	2	PCS	5707292623911
78	0493437	Label "exclamation mark"	6	PCS	5707292136909
79	1103500	Fazet washer 12mm	2	PCS	5707292140326
56+71+72*	3201610	PU, Wheel 16" black left	1	PCS	5707292587268
	3201600	PU-Wheel, black 16" right	1	PCS	5710000000000
	3201630	Wheel, Air 16" - black, left	1	PCS	5710000000000
61+71+72	3201620	Wheel, Air black 16" right	1	PCS	5710000000000
65+66	3503532	Gas pump lever without clamp	1	PCS	5707292427755

*Choose size, color or type

Frame assembly 22"/24" (113933)

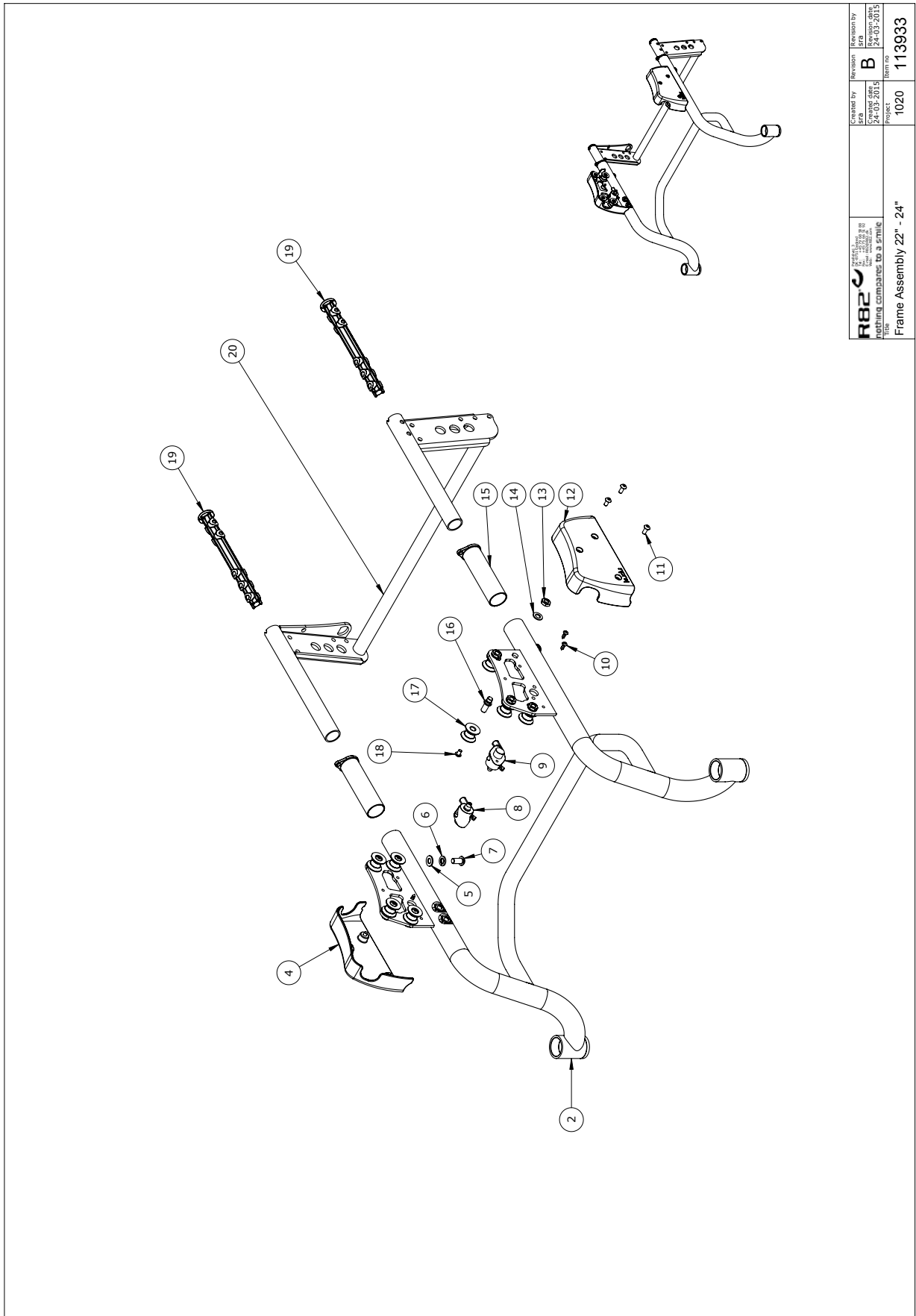

 R82 A/S Parallelvej 3 8751 Gedved Denmark	Drawn by LAF	Created date 24-03-2015	Revision by B Revision date 24-03-2015	Project 1020	Part no. 113933
	Nothing compare to a style			Project 1020	


C:\pdp\pdp\pdp\113933

Item numbers for Frame assembly 22"/24" (113933)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
2*	108367-10	Steel Assembly, frame size 1 white	1	PCS	5707292581655
	108367-31	Steel Assembly, frame size 1 red	1	PCS	5707292584182
	108367-35	Steel Assembly, frame size 1 gray	1	PCS	5707292584212
	108366-10	Steel Assembly, frame size 2 white	1	PCS	5707292581648
	108366-31	Steel Assembly, frame size 2 red	1	PCS	5707292584243
	108366-35	Frame size 2, painted	1	PCS	5707292584236
	107917-10	Steel, frame size 3, painted	1	PCS	5707292581426
	107917-31	Steel Assembly, frame size 3 red	1	PCS	5707292584267
	107917-35	Frame size 3, painted grey	1	PCS	5707292584274
	103395-31	Steel Assembly, frame size 4	1	PCS	5707292584250
	103395-10	Assembly, frame size 4 painted	1	PCS	5707292581280
	103395-35	Steel Assembly, frame size 4 grey	1	PCS	5707292584281
4*	110207	Cover, right size 1+2	1	PCS	5707292582522
	108364	Cover, right	1	PCS	5707292581310
5	1105080	Stainless washer 8mm	2	PCS	5707292140364
6	1100500	Spring washer 8mm, stainless	2	PCS	5707292140142
7	0662002	Button head M8x20, A2 w/glue	2	PCS	5707292600554
8	107228	Lock mechanism right	1	PCS	5707292581334
9	104636	Lock mechanism left	1	PCS	5707292581341
10	1940029	Pan head screws eco-syn 5x12	4	PCS	5707292597625
11	0658015	Button head M5x12 FZB w/ES-Loc	6	PCS	5707292614445
12*	110206	Cover, left size 1+2	1	PCS	5707292582539
	108365	Cover, left	1	PCS	5707292581327
13	1750800	Lock nut, stainless M8	8	PCS	5707292141484
14	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
15	103258	Guide bush	2	PCS	5707292576071
16	103117	Roller rod	8	PCS	5707292567031
17	103151	Roller Kudu	8	PCS	5707292567048
18	0631952	Button head M5x6, rf tuflok	8	PCS	5707292503725
19	111114	Rear frame Insert	2	PCS	5707292581389
20*	113113-35	Rear Wheel frame size 1	1	PCS	5707292598776
	113114-35	Rear Wheel frame size 2	1	PCS	5707292598783
	113115-35	Rear Wheel frame size 3	1	PCS	5707292598790
	113116-35	Rear Wheel frame size 4	1	PCS	5707292598806

*Choose size, color or type

Frame assembly 12½"/16" (111718)

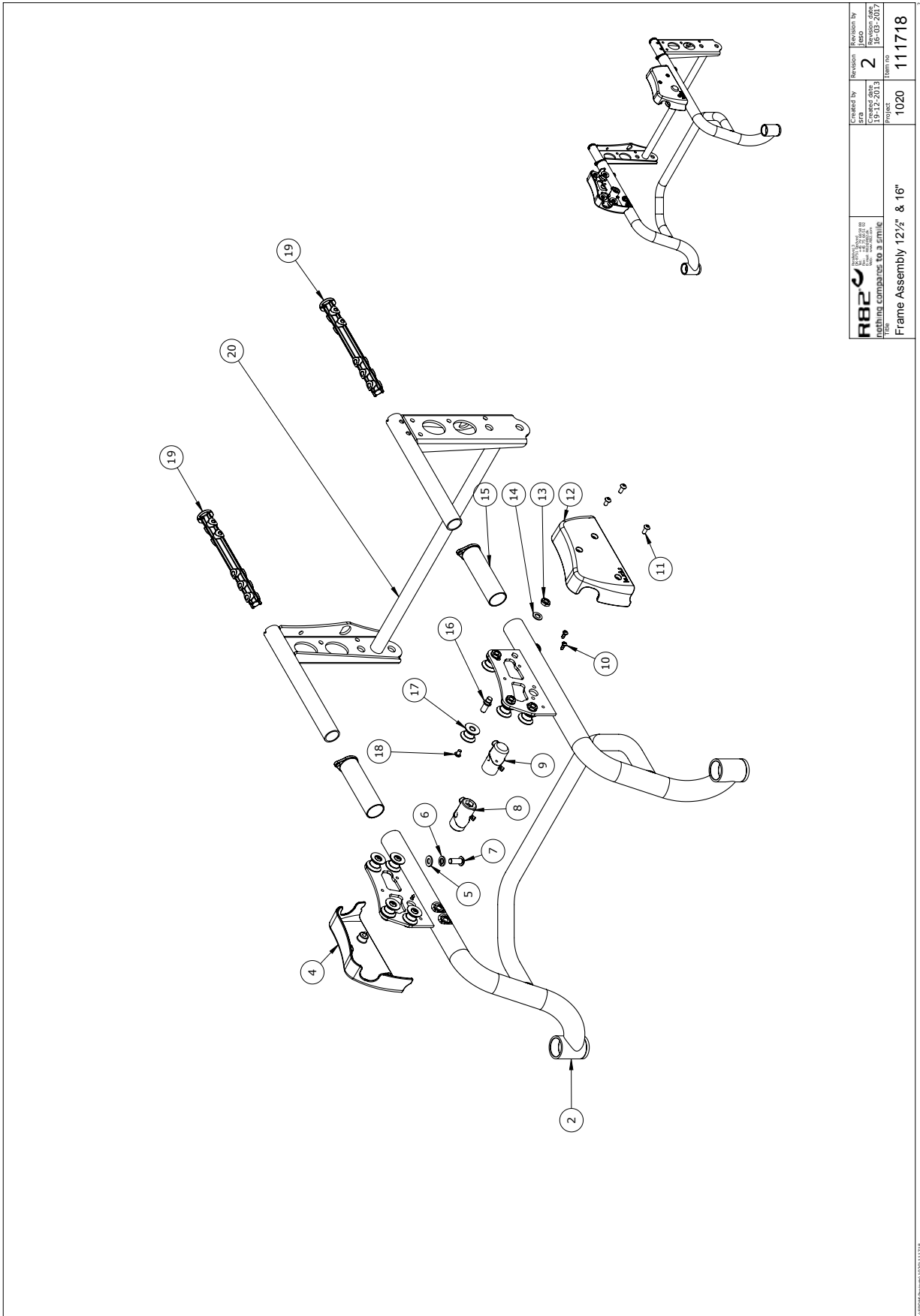

 R82 A/S Parallelvej 3 8751 Gedved Denmark	Created by ERA 15/09/2013	Revision by JYBA 15/09/2013	Revision by JYBA 15/09/2013
	Project 1020	Team no. 111718	Job no. 111718

R82 A/S
 Parallelvej 3
 8751 Gedved
 Denmark
 Frame Assembly 12½" & 16"


C:\pmp\pmp\kudu\1020\111718

Item numbers for Frame assembly 12½"/16" (111718)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
2*	108367-10	Steel Assembly, frame size 1 white	1	PCS	5707292581655
	108367-31	Steel Assembly, frame size 1 red	1	PCS	5707292584182
	108367-35	Steel Assembly, frame size 1 gray	1	PCS	5707292584212
	108366-10	Steel Assembly, frame size 2 white	1	PCS	5707292581648
	108366-31	Steel Assembly, frame size 2 red	1	PCS	5707292584243
	108366-35	Frame size 2, painted	1	PCS	5707292584236
	107917-10	Steel, frame size 3, painted	1	PCS	5707292581426
	107917-31	Steel Assembly, frame size 3 red	1	PCS	5707292584267
	107917-35	Frame size 3, painted grey	1	PCS	5707292584274
	103395-10	Assembly, frame size 4 painted	1	PCS	5707292581280
	103395-31	Steel Assembly, frame size 4	1	PCS	5707292584250
	103395-35	Steel Assembly, frame size 4 grey	1	PCS	5707292584281
4*	110207	Cover, right size 1+2	1	PCS	5707292582522
	108364	Cover, right	1	PCS	5707292581310
5	1105080	Stainless washer 8mm	2	PCS	5707292140364
6	1100500	Spring washer 8mm, stainless	2	PCS	5707292140142
7	0662002	Button head M8x20, A2 w/glue	2	PCS	5707292600554
8	107228	Lock mechanism right	1	PCS	5707292581334
9	104636	Lock mechanism left	1	PCS	5707292581341
10	1940029	Pan head screws eco-syn 5x12	4	PCS	5707292597625
11	0658015	Button head M5x12 FZB w/ES-Loc	6	PCS	5707292614445
12*	110206	Cover, left size 1+2	1	PCS	5707292582539
	108365	Cover, left	1	PCS	5707292581327
13	1750800	Lock nut, stainless M8	8	PCS	5707292141484
14	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
15	103258	Guide bush	2	PCS	5707292576071
16	103117	Roller rod	8	PCS	5707292567031
17	103151	Roller Kudu	8	PCS	5707292567048
18	0631952	Button head M5x6, rf tuflok	8	PCS	5707292503725
19	111114	Rear frame Insert	2	PCS	5707292581389
20*	110776-35	12½" Wheel frame size 1	1	PCS	5707292582560
	110783-35	12½"+16" Wheel frame size 2	1	PCS	5707292582621
	110790-35	12½" Wheel frame size 3	1	PCS	5707292582652
	109297-35	12½" Wheel frame size 4	1	PCS	5707292582683

*Choose size, color or type

Frame assembly 22"/24" Straight (118197-x)

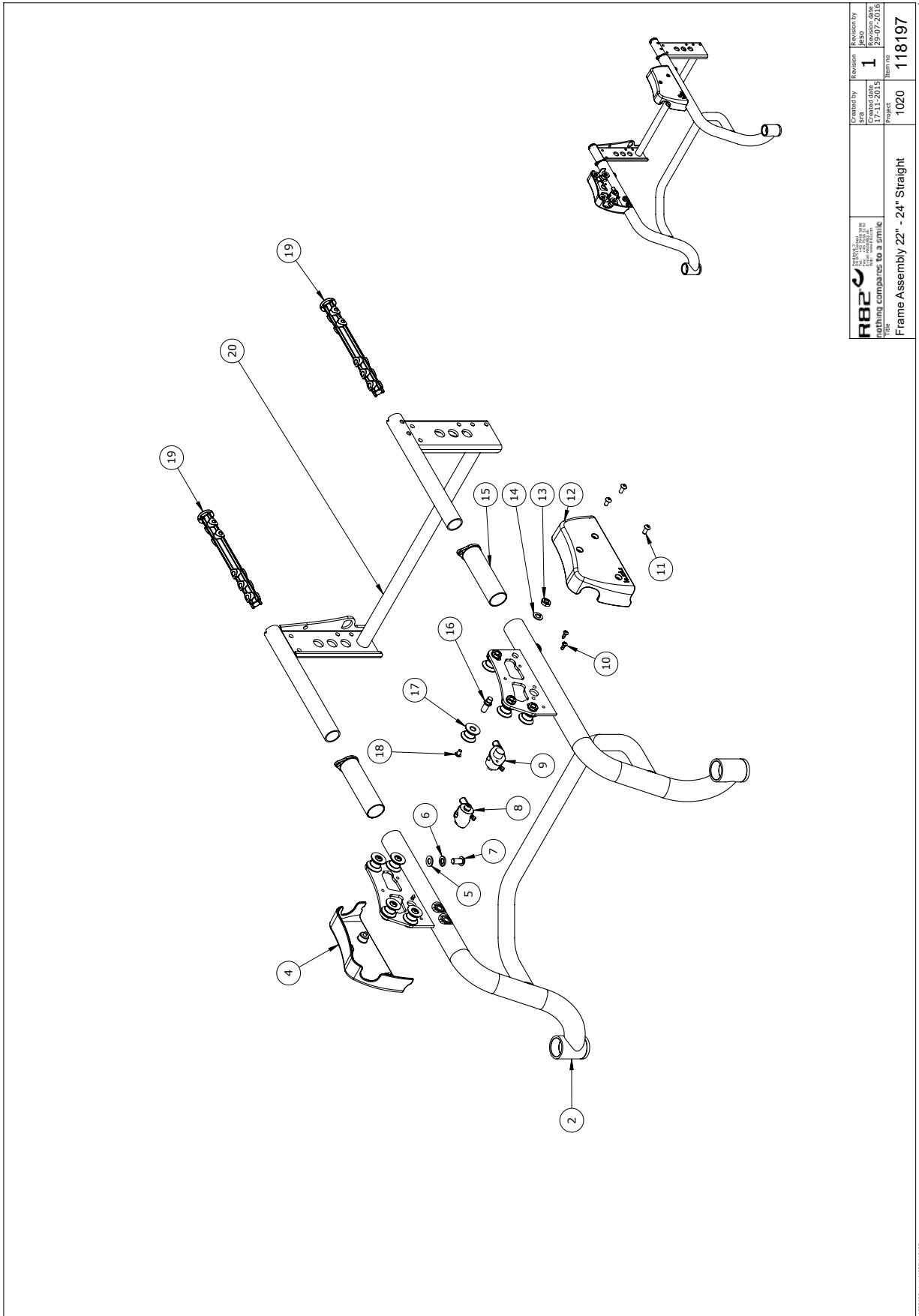

R82 R82 A/S Parallelvej 3 8751 Gedved Denmark Tel: +45 79 68 58 88 Fax: +45 75 66 51 92 E-mail: R82@etac.com Web: etac.com	Created by SFB Created date 29-07-2010	Checked by JMSO Checked date 29-07-2010	Revision by JMSO Revision date 29-07-2010
	Project 1020	Part no. 118197	Item no. 118197


C:\PMP\pmp\118197


Item numbers for Frame assembly 22"/24" Straight (118197-x)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
2*	108367-10	Steel Assembly, frame size 1 white	1	PCS	5707292581655
	108367-31	Steel Assembly, frame size 1 red	1	PCS	5707292584182
	108367-35	Steel Assembly, frame size 1 gray	1	PCS	5707292584212
	108366-10	Steel Assembly, frame size 2 white	1	PCS	5707292581648
	108366-31	Steel Assembly, frame size 2 red	1	PCS	5707292584243
	108366-35	Frame size 2, painted	1	PCS	5707292584236
	107917-10	Steel, frame size 3, painted	1	PCS	5707292581426
	107917-31	Steel Assembly, frame size 3 red	1	PCS	5707292584267
	107917-35	Frame size 3, painted grey	1	PCS	5707292584274
	103395-10	Assembly, frame size 4 painted	1	PCS	5707292581280
	103395-31	Steel Assembly, frame size 4	1	PCS	5707292584250
	103395-35	Steel Assembly, frame size 4 grey	1	PCS	5707292584281
4*	110207	Cover, right size 1+2	1	PCS	5707292582522
	108364	Cover, right	1	PCS	5707292581310
5	1105080	Stainless washer 8mm	2	PCS	5707292140364
6	1100500	Spring washer 8mm, stainless	2	PCS	5707292140142
7	0662002	Button head M8x20, A2 w/glue	2	PCS	5707292600554
8	107228	Lock mechanism right	1	PCS	5707292581334
9	104636	Lock mechanism left	1	PCS	5707292581341
10	1940029	Pan head screws eco-syn 5x12	4	PCS	5707292597625
11	0658015	Button head M5x12 FZB w/ES-Loc	6	PCS	5707292614445
12*	110206	Cover, left size 1+2	1	PCS	5707292582539
	108365	Cover, left	1	PCS	5707292581327
13	1750800	Lock nut, stainless M8	8	PCS	5707292141484
14	1101050	Spring washer M8, wavy FZB	8	PCS	5707292458667
15	103258	Guide bush	2	PCS	5707292576071
16	103117	Roller rod	8	PCS	5707292567031
17	103151	Roller Kudu	8	PCS	5707292567048
18	0631952	Button head M5x6, rf tuflok	8	PCS	5707292503725
19	111114	Rear frame Insert	2	PCS	5707292581389
20*	118099-35	Rear Wheel frame size2 straigh	1	PCS	5707292613561
	118100-35	Rear Wheel frame size2 straigh	1	PCS	5707292613615
	118101-35	Rear Wheel frame size3 straigh	1	PCS	5707292613622
	118102-35	Rear Wheel frame size4 straigh	1	PCS	5707292613639

*Choose size, color or type

Seat assembly size 1 (115022-1), size 2 (115022-2)

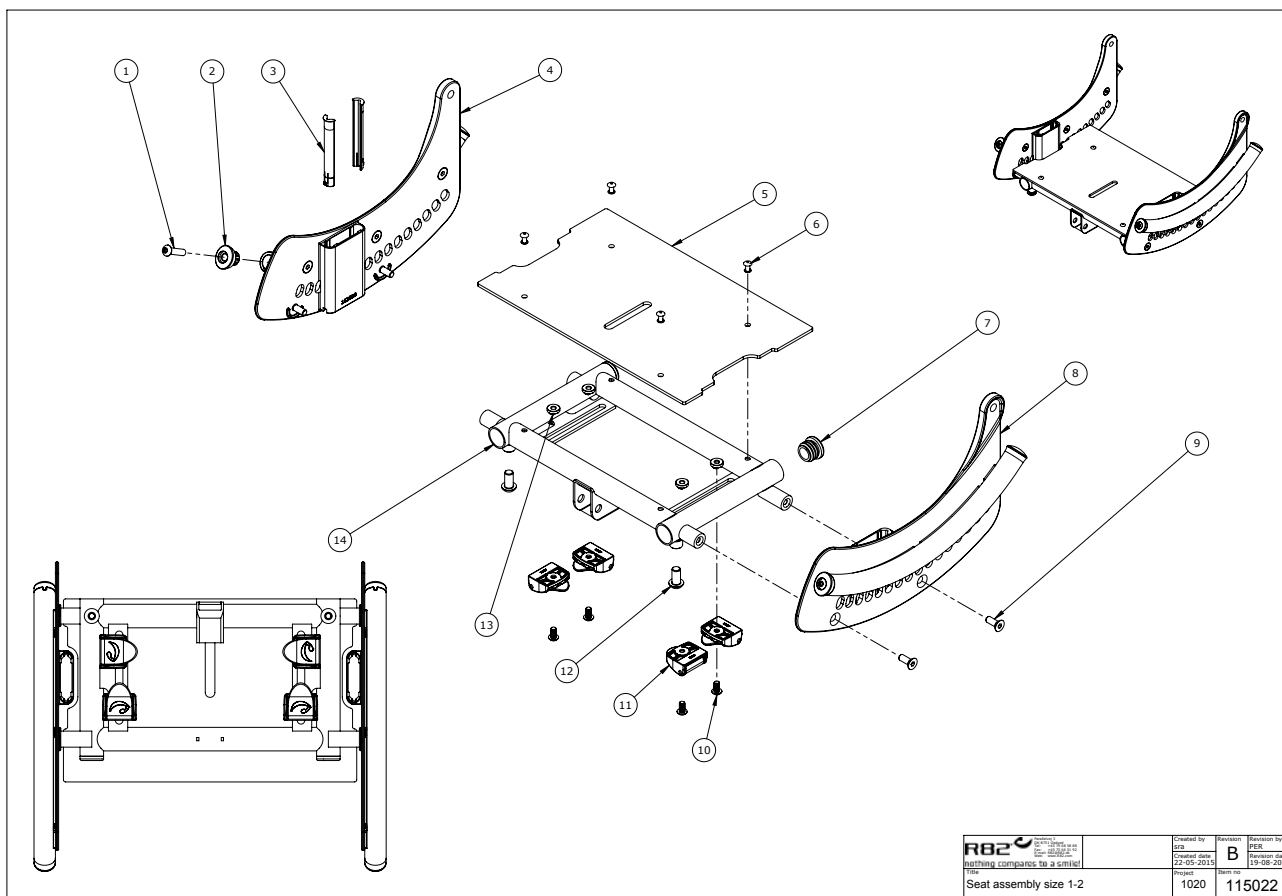


 <small>nothing compares to a rivet!</small>	Created by	Revision
	Order date	Revision date
Part	1020	115022

Item numbers for Seat assembly size 1 (115022-1), size 2 (115022-2)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	0659570	Button head screw 6x25 stainl.	PCS	2	5707292139474
2	103951	Stop	PCS	2	5707292582317
3	113718	Arm support guide	PCS	4	5707292604460
4	112887	Seat frame Right Size 1-2	PCS	1	5707292600455
5*	108910	Seat plate size 1	PCS	1	5707292582980
	108908	Seat plate size 2	PCS	1	5707292582874
6	0506513	Stainless blind rivet Ø4, 8x12	PCS	4	5707292137326
7	2503010	Ferrule d22x1,8-3,5, black	PCS	2	5707292357632
8	112886	Seat frame Left Size 1-2	PCS	1	5707292600448
9	0609729	Stainless insex 6x16	PCS	4	5707292435330
10	0631952	Button head M5x6, rf tuflok	PCS	4	5707292503725
11	75306	Snap lock R82 25	PCS	4	5707292437624
12	0661500	Button head screw 8x16 stainl.	PCS	2	5707292139559
13	78150	Groove nut M5	PCS	4	5707292576118
14*	108729	Seat frame size 1	PCS	1	5707292582997
	108722	Seat frame size 2, black	PCS	1	5707292582799

*Choose size, color or type

Back assembly (138658)

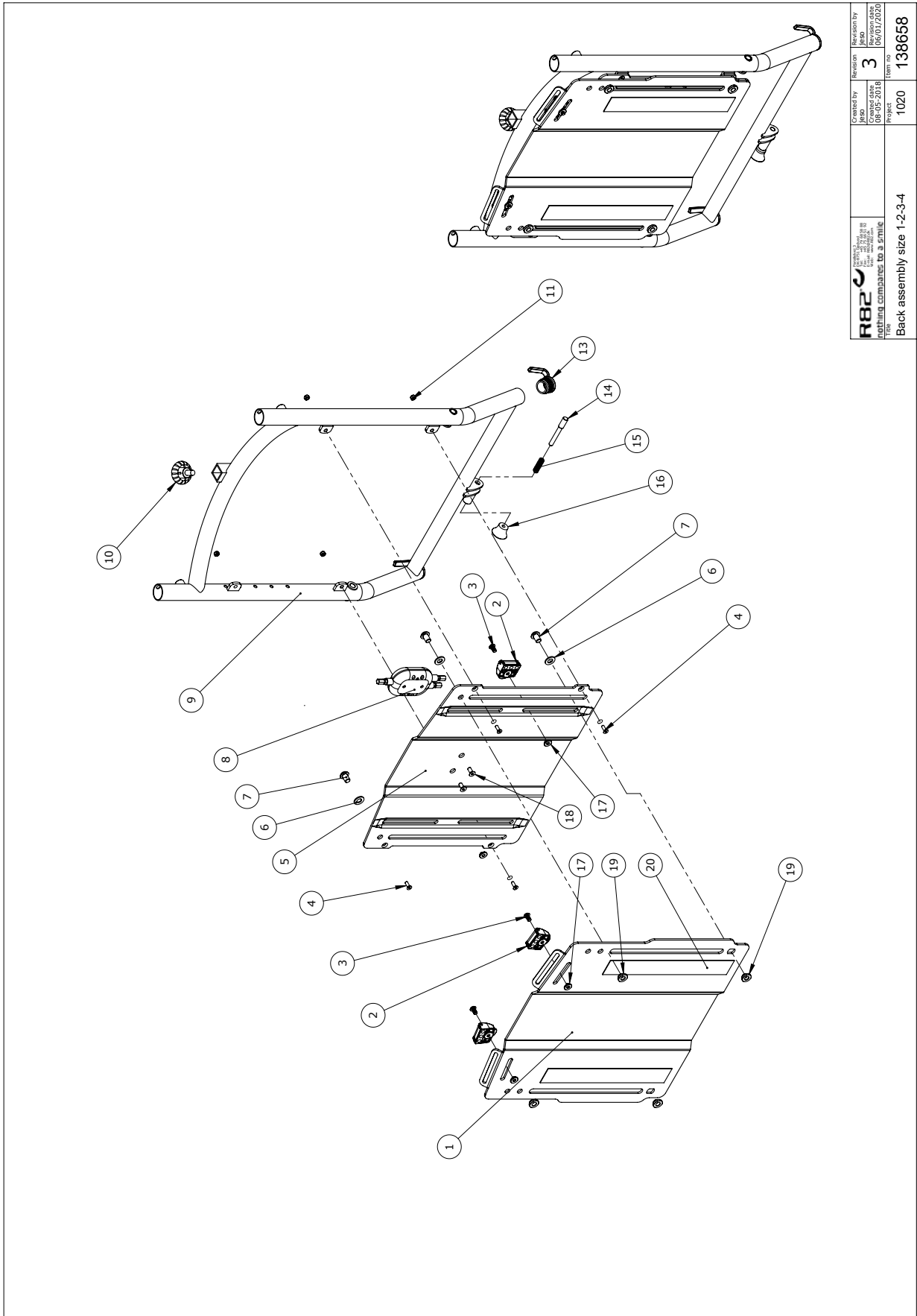

 nothing compares to a smile	Created by 1020	Revision 3	Revision by 1020
	Created date 08-05-2018	Created date 08-05-2018	Revision date 06/01/2020
Project 1020		Item no 138658	


Back assembly size 1-2-3-4

C:\ERP\Kudu\Kudu\138658\138658_001.dwg - 001.dwg - Rev. 001 (P. 001) 138658

Item numbers for Back assembly (138658)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1*	137465	Back plate front size 4	1	PCS	5707292870193
	137464	Back plate front size 3	1	PCS	5707292870186
	137463	Back plate front size 2	1	PCS	5707292870179
	137462	Back plate front size 1	1	PCS	5707292870162
2	75306	Snap lock R82 25	4	PCS	5707292437624
3	0631952	Button head M5x6, rf tuflok	4	PCS	5707292503725
4	0614012	Insex base M4x12, FZB	4	PCS	5707292138811
5*	137471	Back plate rear size 4	1	PCS	5707292870230
	137470	Back plate rear size 3	1	PCS	5707292870223
	137468	Back plate rear size 2	1	PCS	5707292870216
	137466	Back plate rear size 1	1	PCS	5707292870209
6	1105080	Stainless washer 8mm	4	PCS	5707292140364
7	0661150	Button head screw 8x10 stainl.	4	PCS	5707292579041
8	3503542	Box - 2-1 braking cables	1	PCS	5707292600608
9*	137368-10	Back frame size 4, white	1	PCS	5707292835758
	137367-10	Back frame size 3, white	1	PCS	5707292835710
	137366-10	Back frame size 2, white	1	PCS	5707292835673
	137365-10	Back frame size 1, White	1	PCS	5707292835635
10	3503001	R82 handle M8x15mm screw	1	PCS	5707292413925
11	1606275	Lock nut 4mm, large pack	4	PCS	5707292141347
13	114295	Cab for cable	2	PCS	5707292618795
14	112034	Pin for gas spring back	1	PCS	5707292589163
15	0310101	Pressure spring stainles	1	PCS	5707292471994
16	3706256	Knob 25, M6 thread	1	PCS	5707292147219
17	78150	Groove nut M5	4	PCS	5707292576118
18	0631862	Insex uh. 5x12, FZB	2	PCS	5707292352293
19	6118920	M6 Nut	4	PCS	5707292433237
20	4913040	30mm adhesive hook, black	0,6	MTR	5707292786517
*Choose size, color or type					

Foot plate, centrally mounted, size 1 (95152-160), size 2 (95152-260)

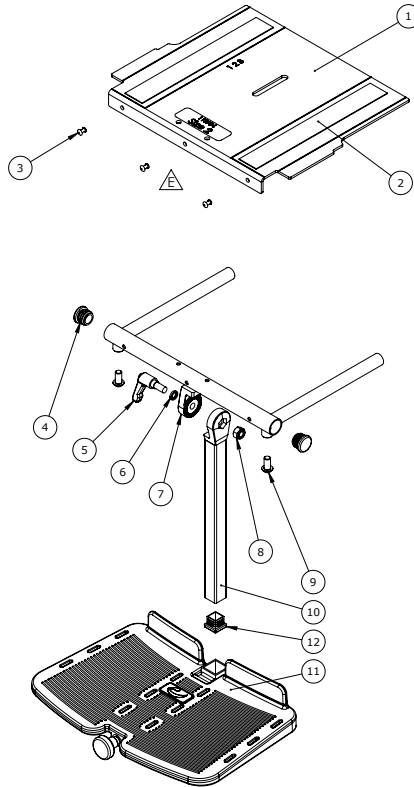

Pos no.	Qty.	Item no.	Description.
1	1	113904	Seat extension plate size 2
2	2	30mm velcro	
3	3	0506509	RT4x10-7.5-SS
4	2	2501100	FRR22-W
5	1	3502516	Handle M8-45x15
6	1	1100495	W8*12*2-SP-SS
7	1	108743	Seat extension size 2
8	1	1607000	NUT8*8-FZB-LC
9	2	0661500	M8*16-BH-SS-A-A2
10	1	75261	Rod size 1 for footrest
11	1		Footsupport size 2
12	1	2504500	FRS20x20-B


R82	Created by E	Revised by E
Approved by E	Created on 19-12-2013	Revised on 29-11-2015
File Foot support model Wombat size 1-2	Project 1020	Draw no. 111561

Item numbers for Foot plate, centrally mounted, size 1 (95152-160), size 2 (95152-260)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1*	113904	Seat extension plate size 2	1	PCS	5707292602855
	113905	Seat extension plate size 1	1	PCS	5707292602848
2*	4913040	30mm adhesive hook, black	0,5	MTR	5707292786517
	4913040	30mm adhesive hook, black	0,36	MTR	5707292786517
3*	0506513	Stainless blind rivet Ø4, 8*12	3	PCS	5707292137326
4	2503010	Ferrule d22x1,8-3,5, black	2	PCS	5707292357632
5	3504025	Tightening handle 45mm M8x25	1	PCS	5707292147103
6	1100495	Spring washer M8	1	PCS	5707292458674
7*	108743	Seat extension size 2	1	PCS	5707292582843
	108753	Seat extension size 1	1	PCS	5707292583048
8	1631080	Stainless lock nut M8	1	PCS	5707292141453
9	0661500	Button head screw 8x16 stainl.	2	PCS	5707292139559
10	75261	Rod size 1 for footrest	1	PCS	5707292455543
11*	95145-1	Footplate model Wombat, size 1	1	PCS	5707292331113
	95145-0	Foot plate, model Wombat, 0	1	PCS	5707292585363
12	2504500	Ferrule e20x1,0-2,0 int.black	1	PCS	5707292144126

*Choose size, color or type

Foot plate, centrally mounted, size 3 (95152-360), size 4 (95152-460)

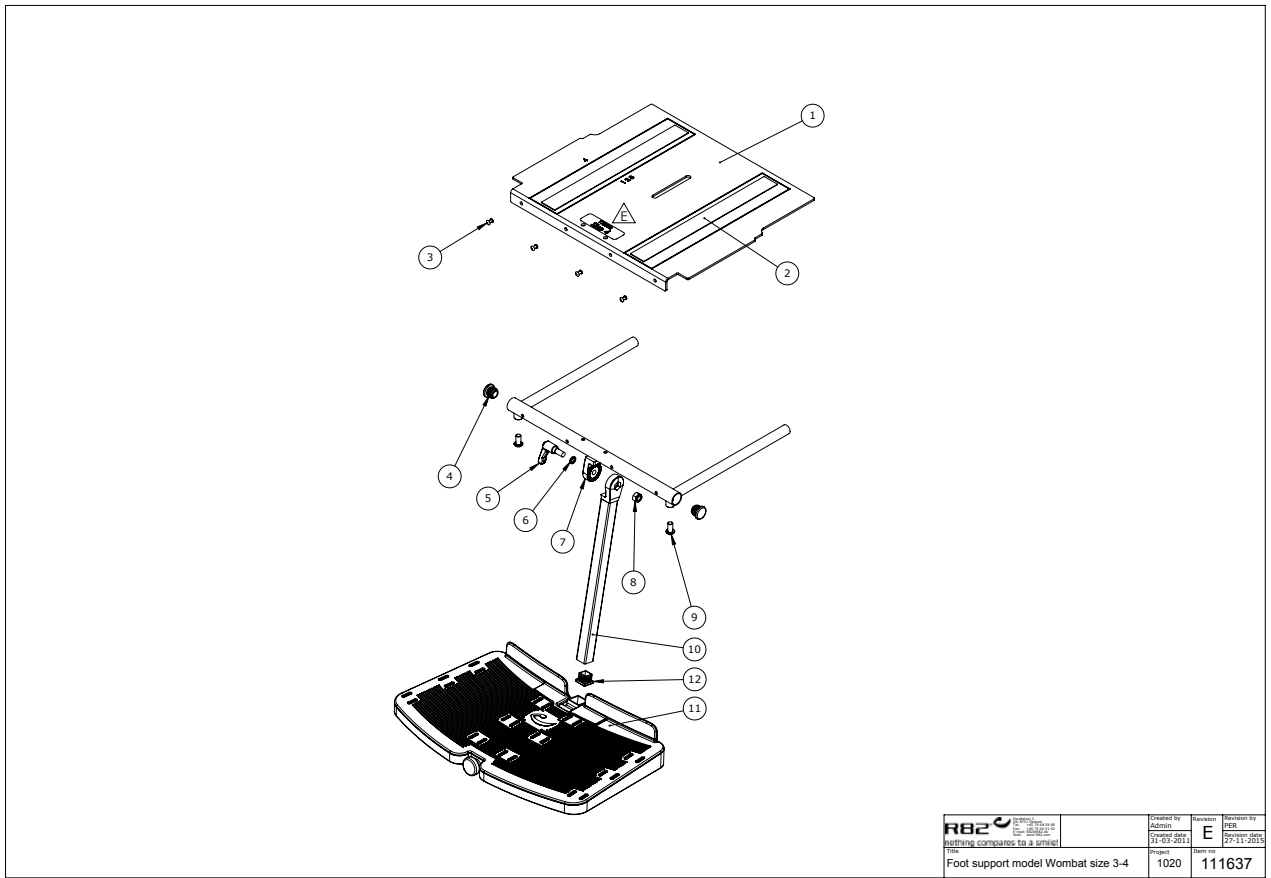


Item numbers for Foot plate, centrally mounted, size 3 (95152-360), size 4 (95152-460)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1*	113903	Seat extension plate size 3	1	PCS	5707292602862
	113902	Seat extension plate size 4	1	PCS	5707292602879
2	4913040	30mm adhesive hook, black	0,65	MTR	5707292786517
3	0506513	Stainless blind rivet Ø4, 8x12	4	PCS	5707292137326
4	2503010	Ferrule d22x1,8-3,5, black	2	PCS	5707292357632
5	3504025	Tightening handle 45mm M8x25	1	PCS	5707292147103
6	1100495	Spring washer M8	1	PCS	5707292458674
7*	107907	Seat extension size 3	1	PCS	5707292582775
	103183	Seat extension size 4	1	PCS	5707292581853
8	1631080	Stainless lock nut 8mm	1	PCS	5707292141453
9	0661500	Button head screw 8x16 stainl.	2	PCS	5707292139559
10	75261	Rod size 1 for footrest	1	PCS	5707292455543
11*	95145-2	Footplate model Wombat	1	PCS	5707292331120
	95145-3	Footplate model Wombat	1	PCS	5707292398833
12	2504500	Ferrule e20x1,0-2,0 int.black	1	PCS	5707292144126

*Choose size, color or type

Foot rests, parted, size 2 (95151-260)

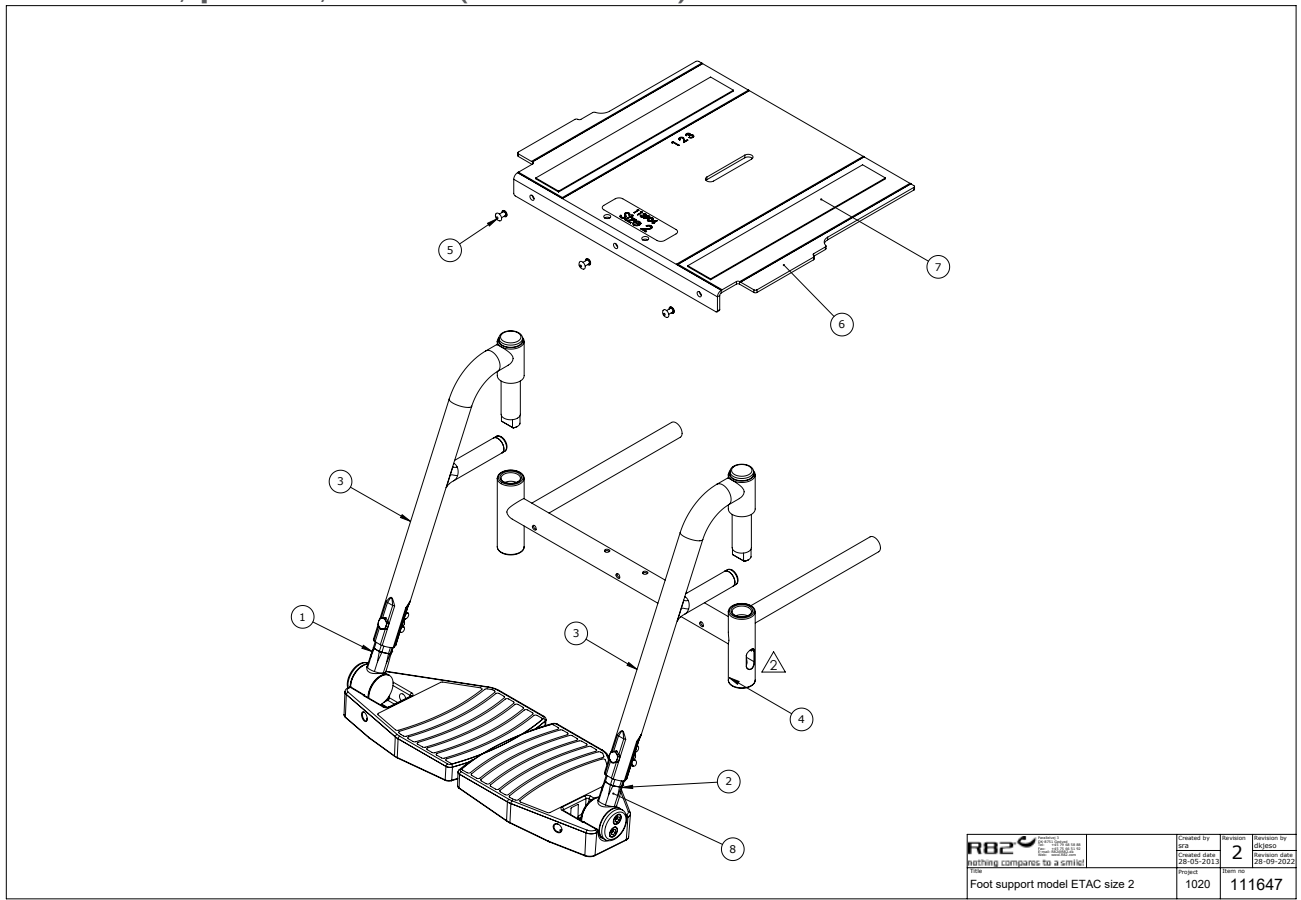


Item numbers for Foot rests, parted, size 2 (95151-260)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1*	5025653-1	lower footsupport parted Right	1	PCS	5707292587916
	5025642-1	lower foot support, parted,	1	PCS	5707292587930
	5025644-1	lower footsupport parted Right	1	PCS	5707292587954
2*	5025653-2	lower footsupport parted Left	1	PCS	5707292587923
	5025642-2	lower footsupport parted left	1	PCS	5707292587947
	5025644-2	lower footsupport parted Left	1	PCS	5707292587961
3*	5025572-01	toppart foot support, parted,	2	PCS	5707292588012
	5025573-01	toppart foot support, parted,	2	PCS	5707292588029
4	109220	Footrest system size 2	1	PCS	5707292583253
5	0506513	Stainless blind rivet Ø4, 8x12	4	PCS	5707292137326
6	113904	Seat extension plate size 2	1	PCS	5707292602855
7	4913040	30mm adhesive hook, black	0,5	MTR	5707292786517
8	0493437	Label "exclamation mark"	2	PCS	5707292136909

*Choose size, color or type

Foot rests, parted, size 3 (95151-360) size 4 (95151-460)

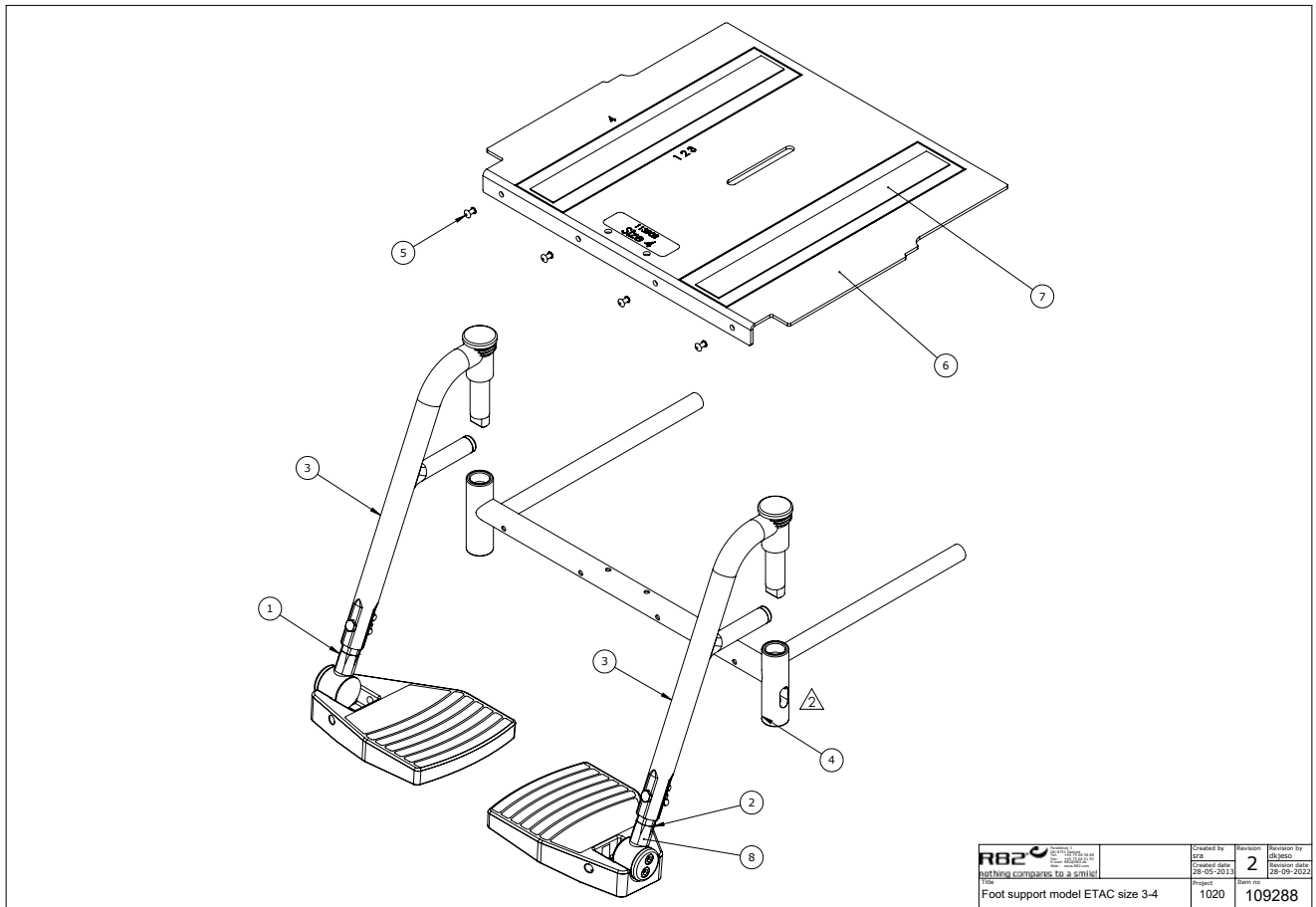


Item numbers for Foot rests, parted, size 3 (95151-360) size 4 (95151-460)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1*	5025653-1	lower footsupport parted Right	1	PCS	5707292587916
	5025642-1	lower foot support, parted,	1	PCS	5707292587930
	5025644-1	lower footsupport parted Right	1	PCS	5707292587954
2*	5025653-2	lower footsupport parted Left	1	PCS	5707292587923
	5025642-2	lower footsupport parted left	1	PCS	5707292587947
	5025644-2	lower footsupport parted Left	1	PCS	5707292587961
3*	5025572-01	toppart foot support, parted,	2	PCS	5707292588012
	5025573-01	toppart foot support, parted,	2	PCS	5707292588029
4*	109225	Footrest system size 3	1	PCS	5707292583277
	109230	Footrest system size 4	1	PCS	5707292583291
5	0506513	Stainless blind rivet Ø4, 8x12	4	PCS	5707292137326
6*	113903	Seat extension plate size 3	1	PCS	5707292602862
	113902	Seat extension plate size 4	1	PCS	5707292602879
7	4913040	30mm adhesive hook, black	0,5	MTR	5707292786517
8	0493437	Label "exclamation mark"	2	PCS	5707292136909

*Choose size, color or type


Push brace, angle adjustable (111629)

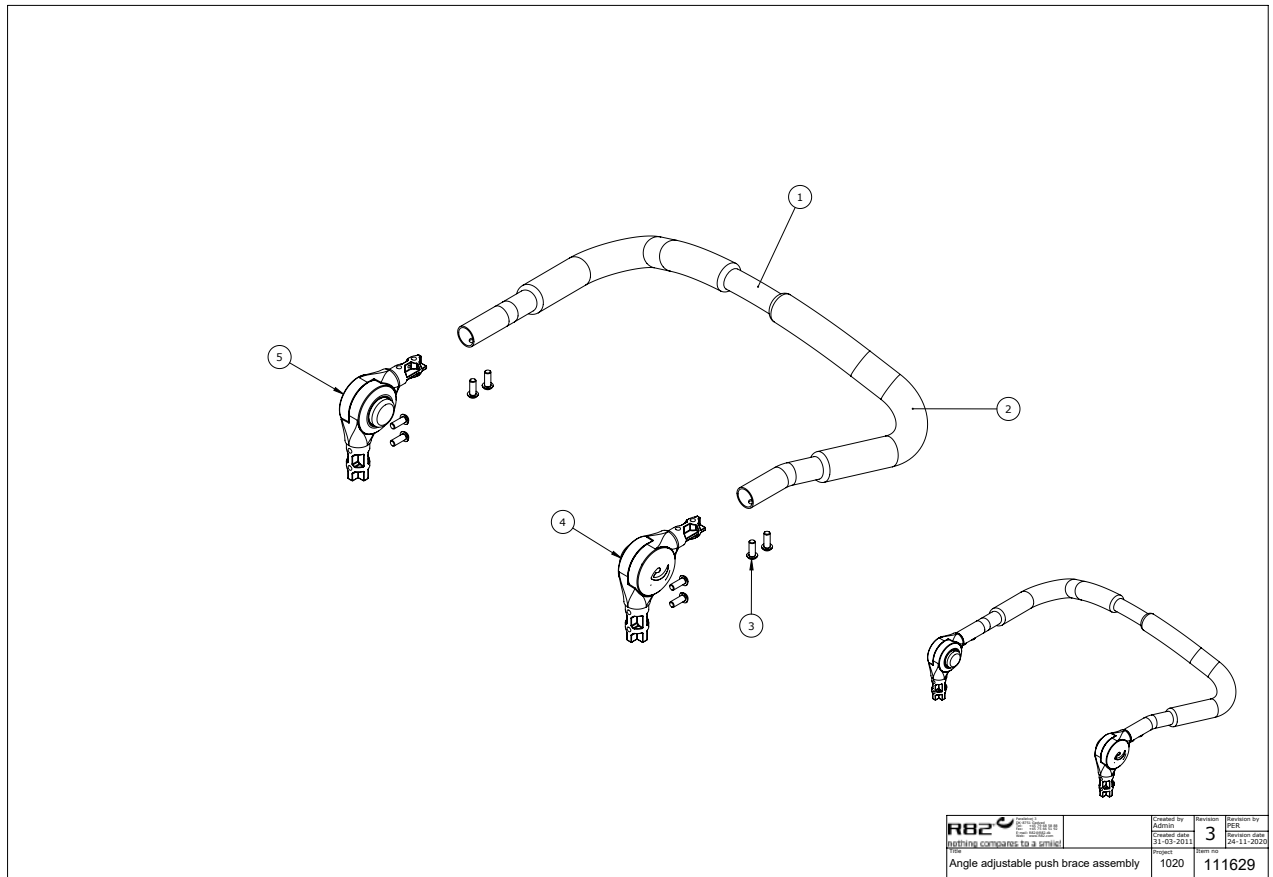

Item numbers for Push brace, angle adjustable (111629)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1*	108869	Handbrace size 1	1	PCS	5707292581136
	77915	Handbrace size 2	1	PCS	5707292581044
	77985	Handbrace size 3	1	PCS	5707292580856
	78127	Handbrace size 4	1	PCS	5707292580719
2	2500520	Tpe cover Ø20mm	2	PCS	5707292401649
3	0663120	Button Head M6x16 stainless st	8	PCS	5707292581532
4	121339	Angel joint left ø19xø22.3	1	PCS	5707292631701
5	121367	Angel joint right ø19x ø 22.3	1	PCS	5707292631695
*Choose size, color or type					

Height adjustable kit for angle adjustable push brace (95630-10)

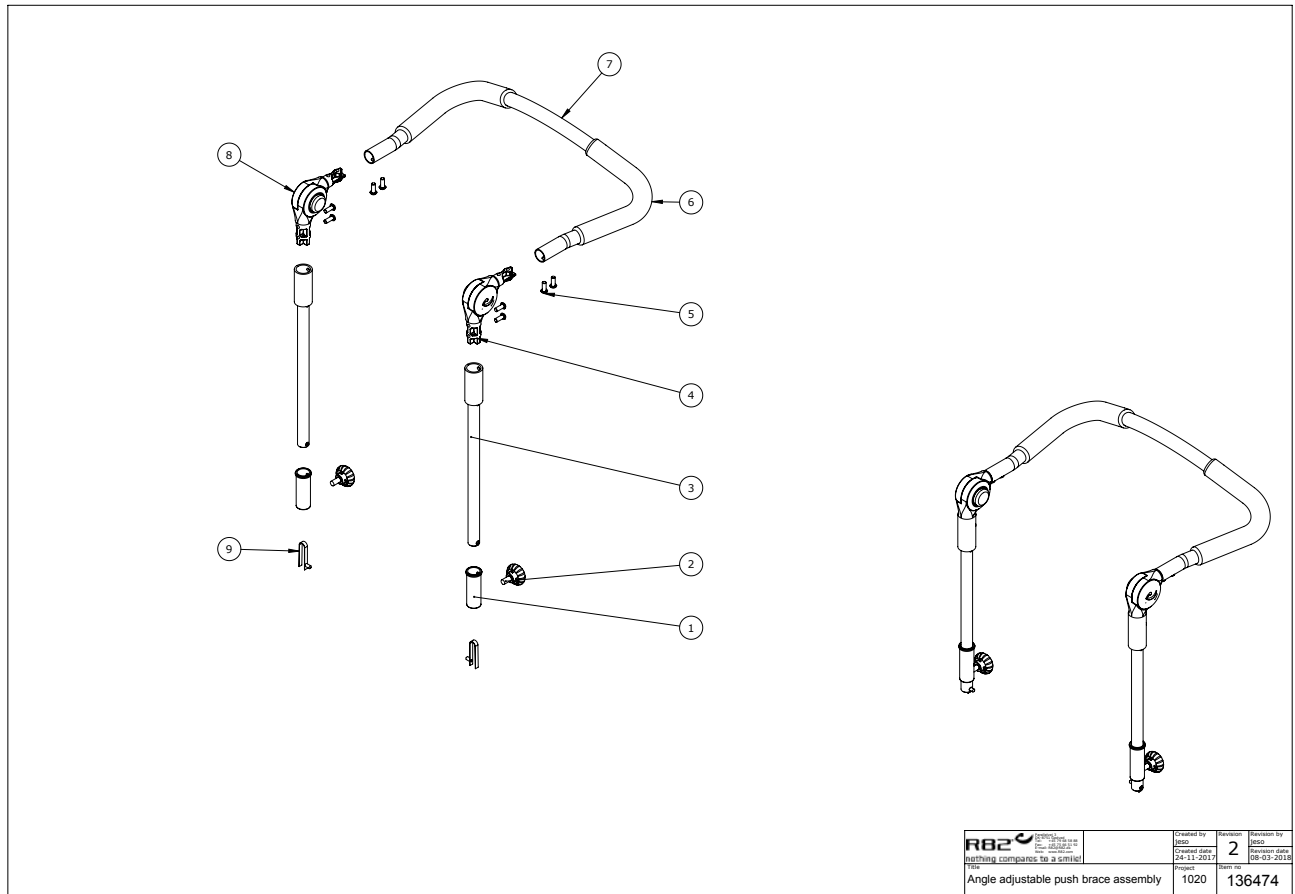


Item numbers Height adjustable kit for angle adjustable push brace (95630-10)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	108640	Guide bush for handle	2	PCS	5707292576101
2	3501201	R82 handle M8x25mm screw	2	PCS	5707292413932
3	136476	Welded pipe for Adjustable	2	PCS	5707292835802
4	121339	Angel joint left Ø19xØ22.3	1	PCS	5707292631701
5	0663120	Button Head M6x16 stainless st	8	PCS	5707292581532
6	2500520	Tpe cover Ø20mm	2	PCS	5707292401649
7*	108869	Handbrace size 1	1	PCS	5707292581136
	77915	Handbrace size 2	1	PCS	5707292581044
	77985	Handbrace size 3	1	PCS	5707292580856
	78127	Handbrace size 4	1	PCS	5707292580719
8	121367	Angel joint right Ø19x Ø 22.3	1	PCS	5707292631695
9	6153510	knob spring Ø5x10 mm	2	PCS	5707292590305

*Choose size, color or type


Push handles, set (95370-10)

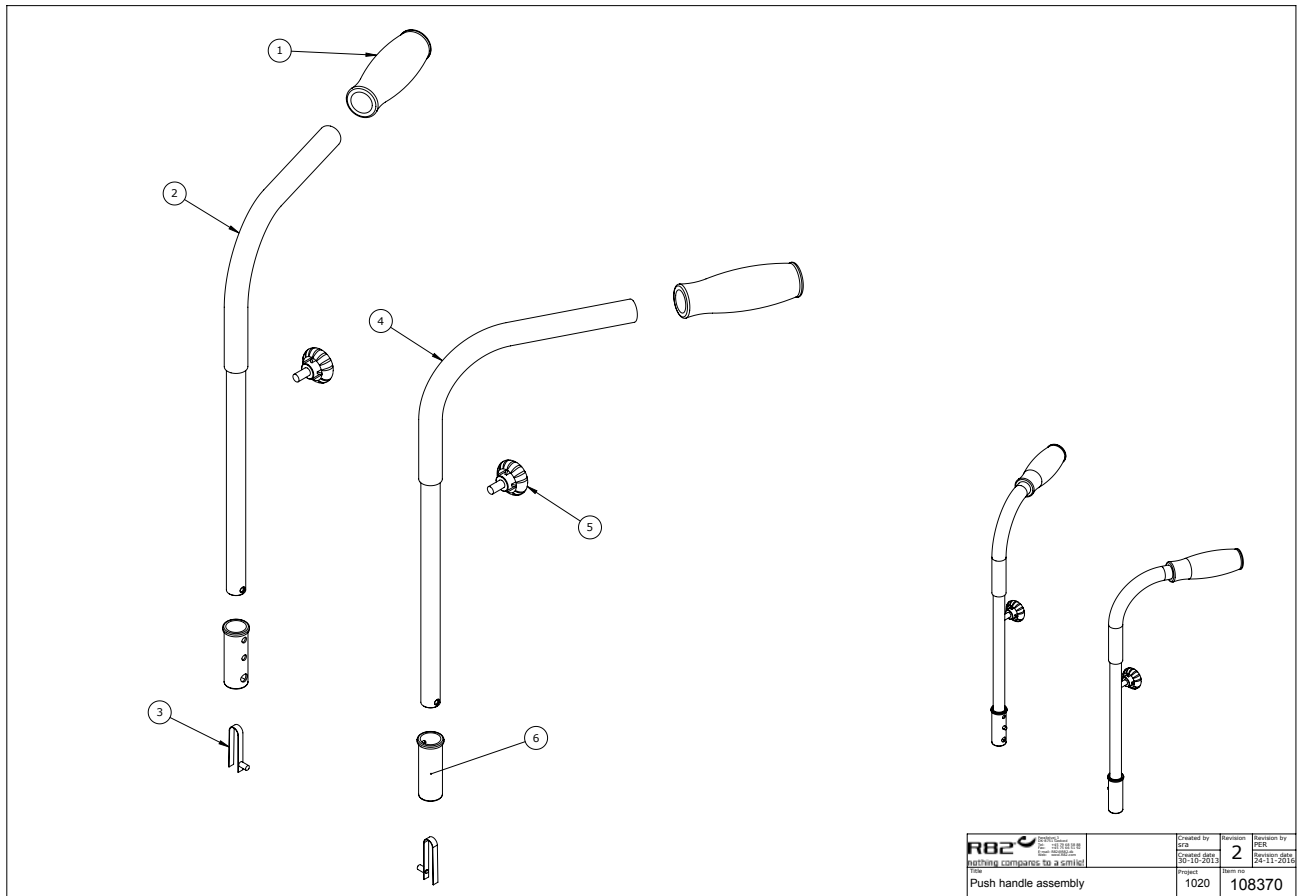

Item numbers Push handles, set (95370-10)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	108371	Rubber grip for handle	2	PCS	5707292583857
2	109836	Hand brace, right	1	PCS	5707292583185
3	6153510	Spring Ø5x10 mm	2	PCS	5707292590305
4	109837	Hand brace, left	1	PCS	5707292583178
5	3501201	R82 handle M8x25mm screw	2	PCS	5707292413932


Wheel brake, right, size 1-2 (111859)

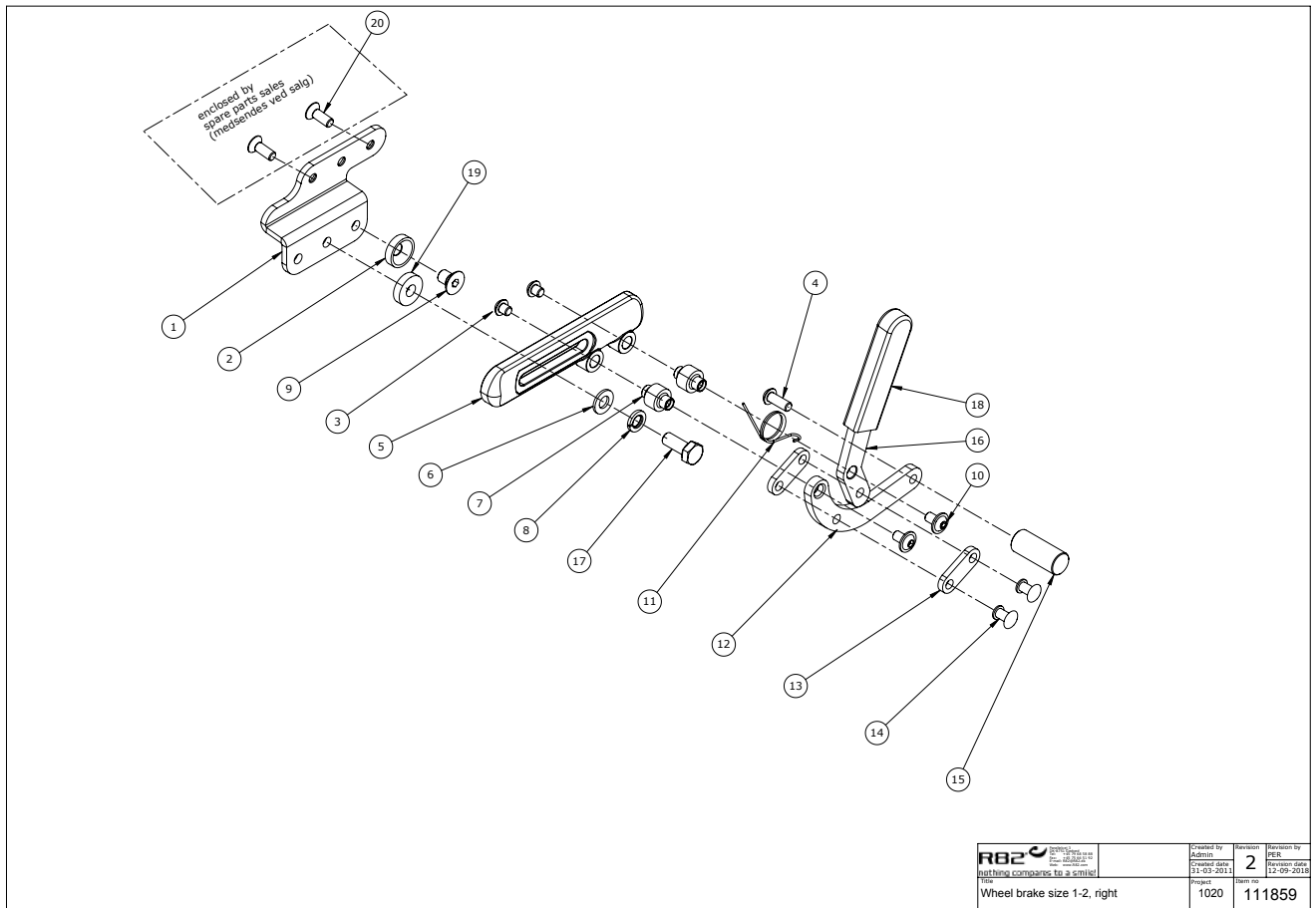

Item numbers for Wheel brake, right, size 1-2 (111859)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	108881	Brake fix plate left side	1	PCS	5707292583093
2	108889	Spacer disc	1	PCS	5707292583109
3	0624802	Insex countersunk M8x12	1	PCS	5707292589309
4	0631961	Buttonhead M6x6,zn/fe&black	2	PCS	5707292577764
5	100983	Brake housing left	1	PCS	5707292574503
6	1105080	Stainless washer 8mm	1	PCS	5707292140364
7	6109200	M6 thread bush, braking system	2	PCS	5707292185952
8	1100500	Spring washer 8mm, stainless	1	PCS	5707292140142
9	0602540	Hexagon Head Screw M8X20, A2	1	PCS	5707292602459
10	0641600	Button head screw 6x16	1	PCS	5707292139269
11	74139	Spring for brake, right	1	PCS	5707292377517
12	111863	Brake lever size 1-2	1	PCS	5707292587725
13	72338	Mounting 12x4x34,5	2	PCS	5707292107794
14	0503175	Black case, Ø6,35x17,5mm	2	PCS	5707292137258
15	72337	Brake ratchet	1	PCS	5707292107787
16	111861	Wheelbrake handle, R size 1+2	1	PCS	5707292587701
17	0641100	B.h. W/flange M6x10	2	PCS	5707292139245
18	2500512	Rubber cap for Panther brake	1	PCS	5707292143730
19	111858	Spacer disc Ø8	1	PCS	5707292587695


Wheel brake, left, size 1-2 (111860)

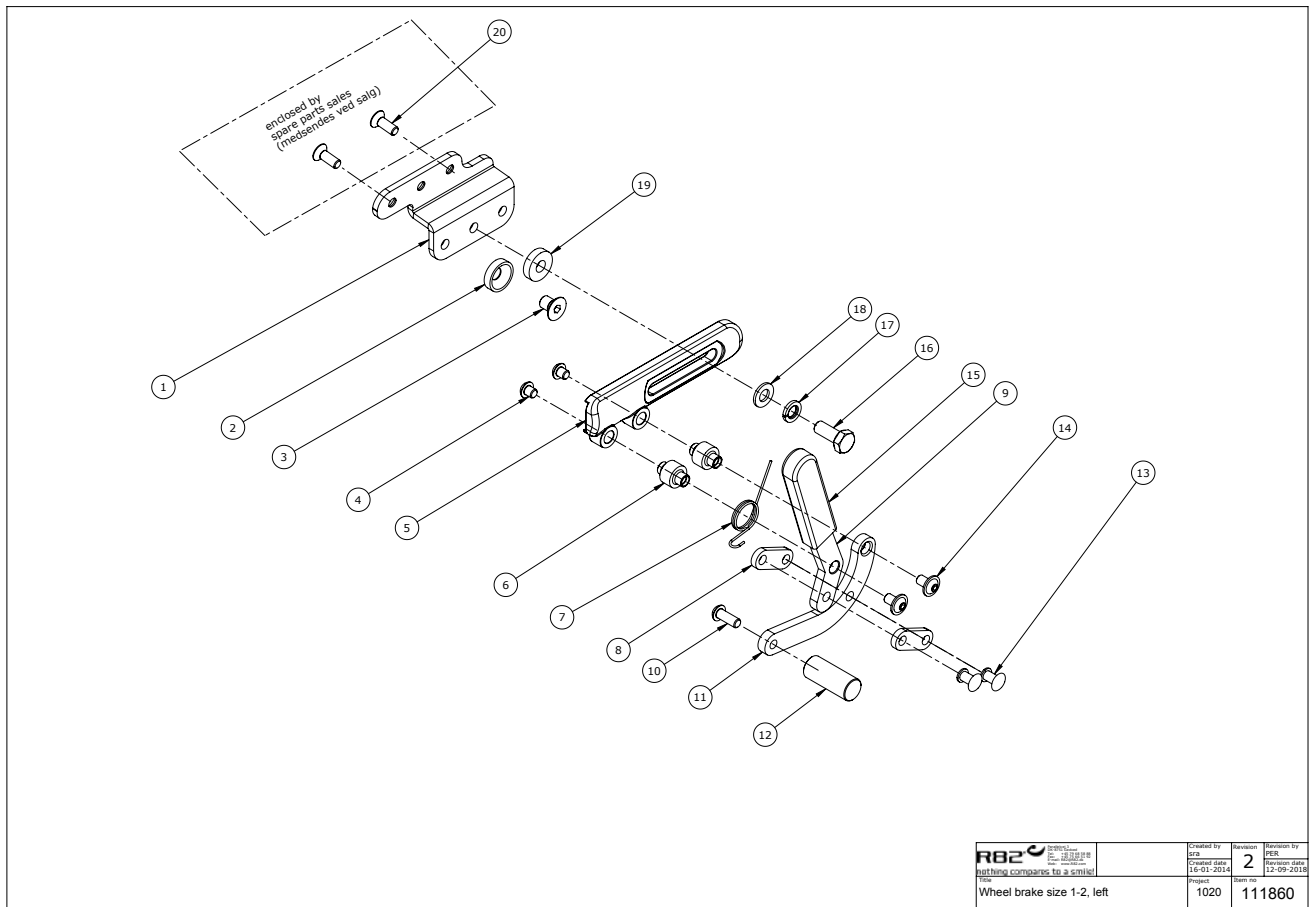

Item numbers for Wheel brake, left, size 1-2 (111860)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	104307	Brake fix plate right side	1	PCS	5707292583161
2	108889	Spacer disc	1	PCS	5707292583109
3	0624802	Insex countersunk M8x12	1	PCS	5707292589309
4	0631961	Buttonhead M6x6,zn/fe&black	2	PCS	5707292577764
5	100997	Brake housing right	1	PCS	5707292574510
6	6109200	M6 thread bush, braking system	2	PCS	5707292185952
7	74138	Spring for brake, left	1	PCS	5707292377500
8	72338	Mounting 12x4x34,5	2	PCS	5707292107794
9	111862	Wheelbrake handle, L size 1+2	1	PCS	5707292587718
10	0641600	Button head screw 6x16	1	PCS	5707292139269
11	111863	Brake lever size 1-2	1	PCS	5707292587725
12	72337	Brake ratchet	1	PCS	5707292107787
13	0503175	Black case, Ø6,35x17,5mm	2	PCS	5707292137258
14	0641100	B.h. W/flange M6x10	2	PCS	5707292139245
15	2500512	Rubber cap for Panther brake	1	PCS	5707292143730
16	0602540	Hexagon Head Screw M8X20, A2	1	PCS	5707292602459
17	1100500	Spring washer 8mm, stainless	1	PCS	5707292140142
18	1105080	Stainless washer 8mm	1	PCS	5707292140364
19	111858	Spacer disc Ø8	1	PCS	5707292587695


Wheel brake, right, size 3-4 (121228)

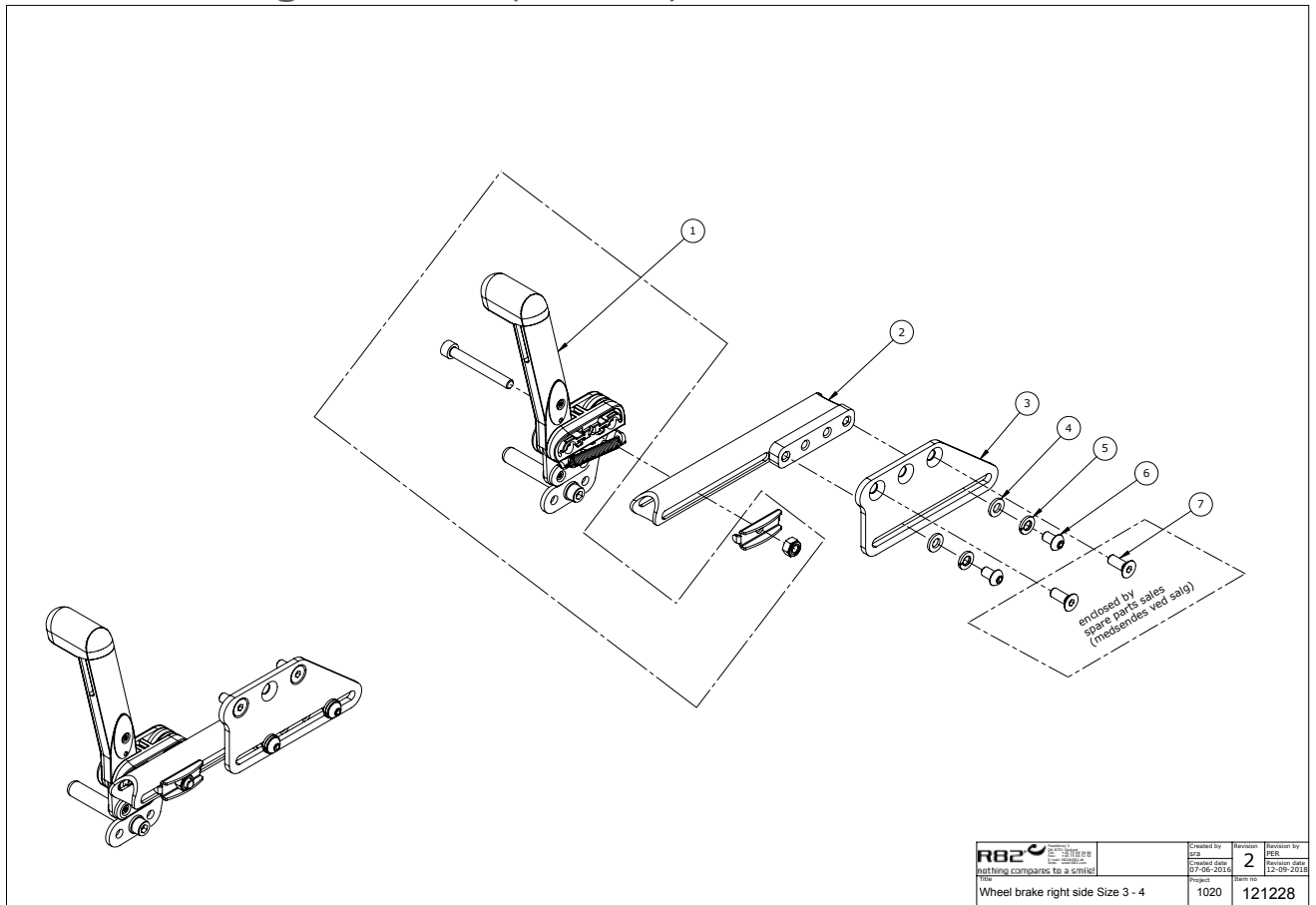

Item numbers for Wheel brake, right, size 3-4 (121228)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	2983101	Brake for KUDU, Right	1	PCS	5707292631367
2	116328	Bracket for brake	1	PCS	5707292623676
3	116325	Plate for brake left	1	PCS	5707292624505
4	1103000	Fazet washer 6mm	2	PCS	5707292140289
5	1100800	Center spring washer 6mm	2	PCS	5707292140166
6	0641200	Button head screw 6x12	2	PCS	5707292139252
7	0609729	Stainless insex 6x16	2	PCS	5707292435330


Wheel brake, left, size 3-4 (121229)

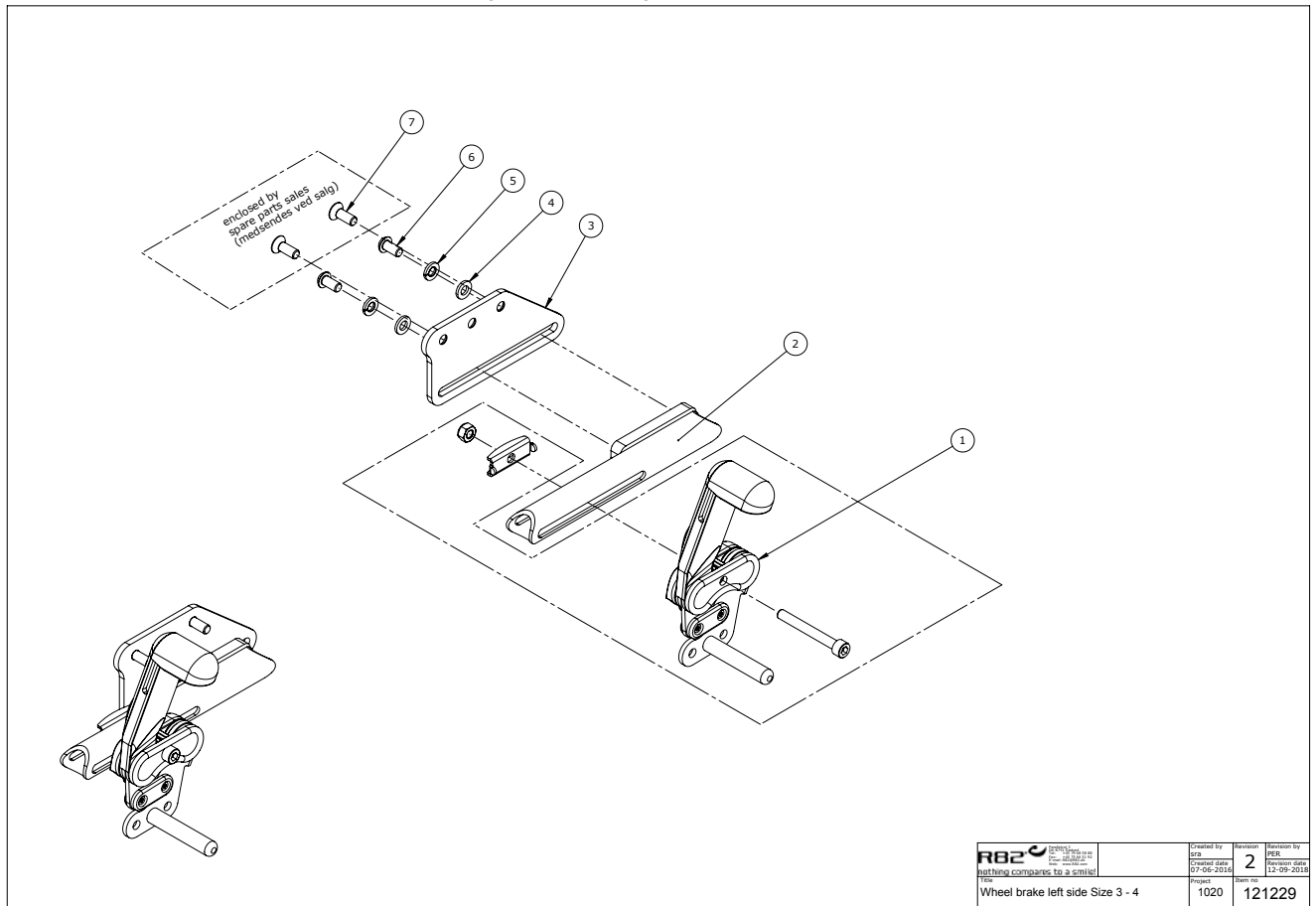

Item numbers for Wheel brake, left, size 3-4 (121229)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	2983100	Brake for KUDU - Left.	1	PCS	5707292631329
2	116328	Bracket for brake	1	PCS	5707292623676
3	116326	Plate for brake right	1	PCS	5707292624512
4	1103000	Fazet washer 6mm	2	PCS	5707292140289
5	1100800	Center spring washer 6mm	2	PCS	5707292140166
6	0641200	Button head screw 6x12	2	PCS	5707292139252
7	0609729	Stainless insex 6x16	2	PCS	5707292435330


Wheel brake, left, 12" (135652)

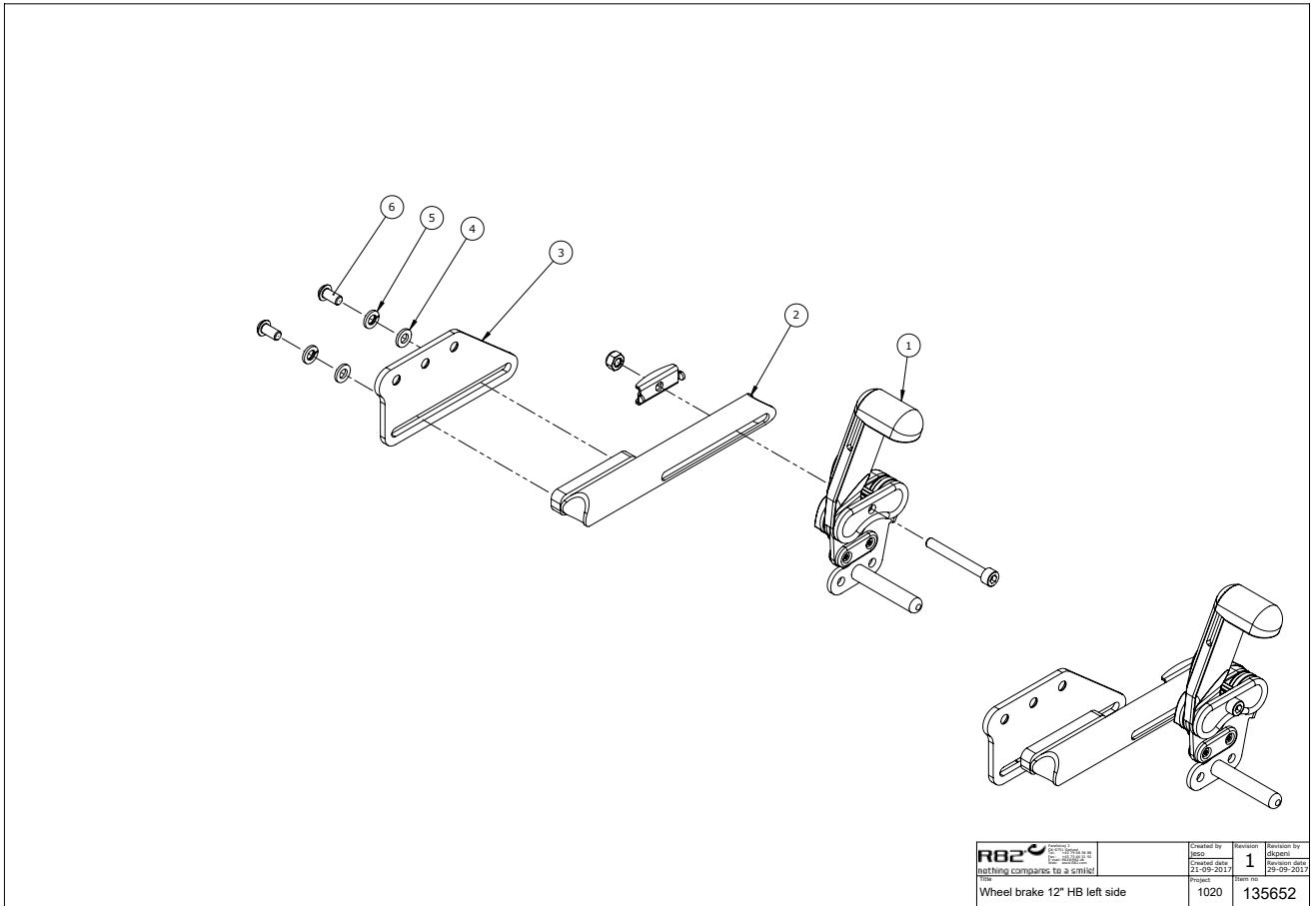

Item numbers for Wheel brake, left, 12" (135652)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	2983101	Brake for KUDU, Left	1	PCS	5707292631367
2	116328	Bracket for brake	1	PCS	5707292623676
3	116326	Plate for brake right/left	1	PCS	5707292624512
4	1103000	Fazer washer 6mm	2	PCS	5707292140289
5	1100800	Center spring washer 6mm	2	PCS	5707292140166
6	0641200	Button head screw 6x12	2	PCS	5707292139252

Wheel brake, right, 16" (169538)

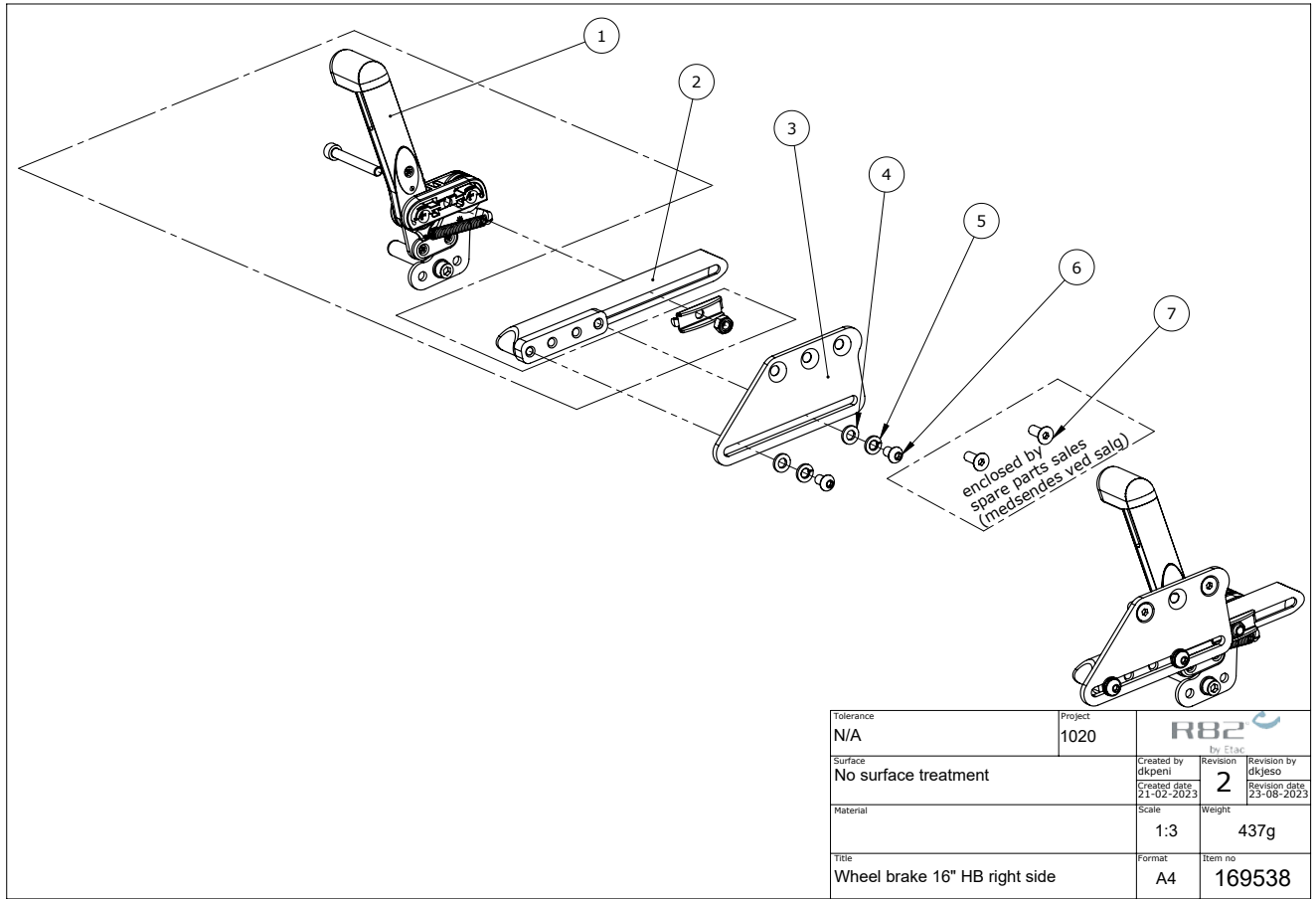


C:\EPDM\Projects\1020\Kudu simpel model\169538

Item numbers for Wheel brake, right, 16" (169538)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	2983100	Brake for KUDU - right	1	PCS	5707292631329
2	116328	bracket for brake	1	PCS	5707292623676
3	116326	Plate for brake right/left	1	PCS	5707292624512
4	1103000	W6x12, fazet washer 6mm	2	PCS	5707292140289
5	1100800	M6x11 spring washer 6 mm	2	PCS	5707292140166
6	0641200	M6x12 FZB button head	2	PCS	5707292139252
7	0609729	Stainless insex M6x16	2	PCS	5707292435330

Wheel brake, left, 16" (169537)

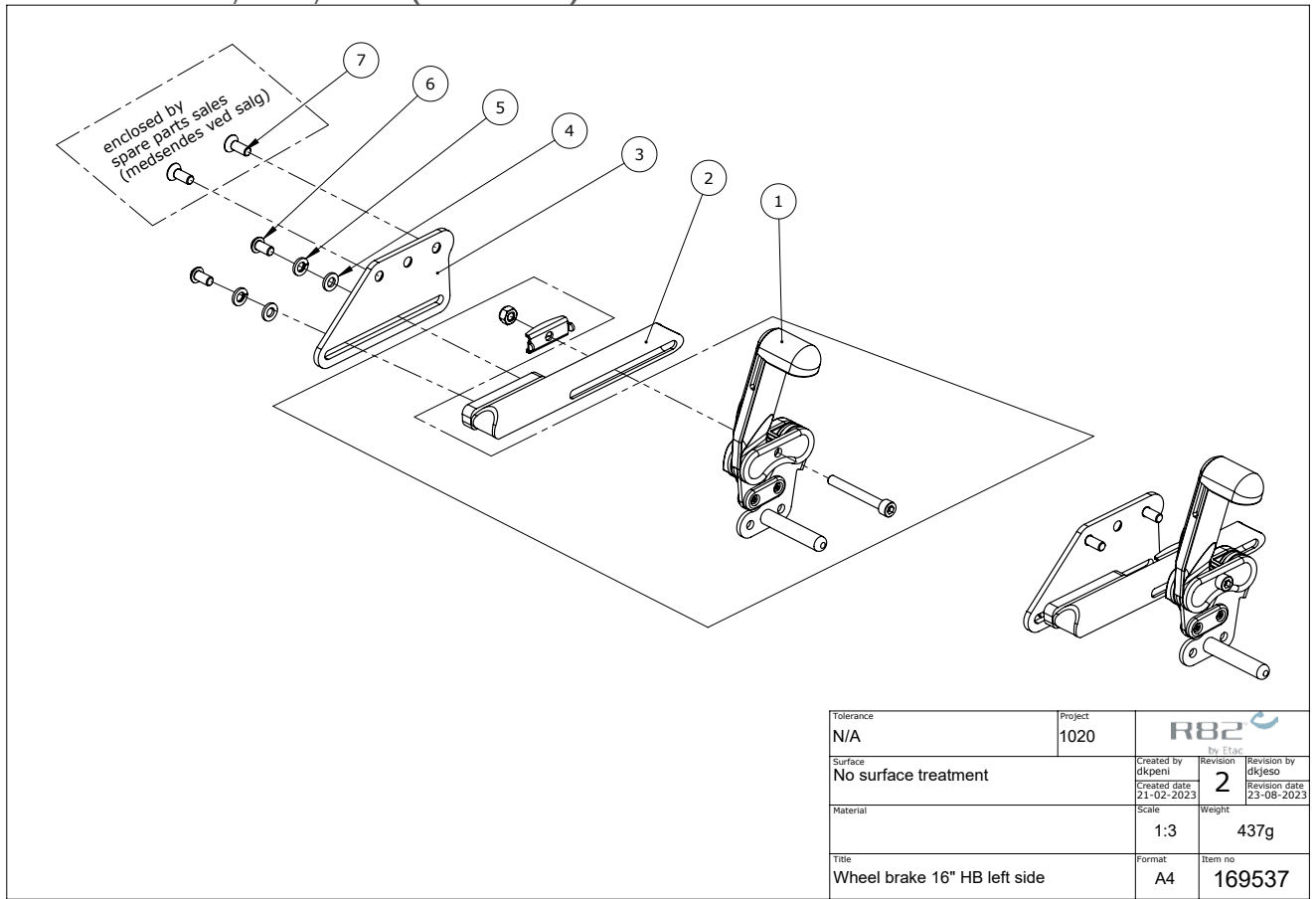

C:\EPCM\Projects\1020\Kudu simpel model\169537

Item numbers for Wheel brake, left, 16" (169537)

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	2983101	Brake for Kudu, left	1	PCS	5707292631367
2	116328	bracket for brake	1	PCS	5707292623676
3	116325	Plate for brake left/right	1	PCS	5707292624505
4	1103000	W6x12, fazet washer 6mm	2	PCS	5707292140289
5	1100800	M6x11 spring washer 6 mm	2	PCS	5707292140166
6	0641200	M6x12 FZB button head	2	PCS	5707292139252
7	0609729	Stainless insex M6x16	2	PCS	5707292435330