

Molift Extension Arm is made for safe and easy mounting of Molift Nomad to the trolley in a rail system. Molift Extension Arm are delivered in two parts: Hook with securing ring (1) and handle (2)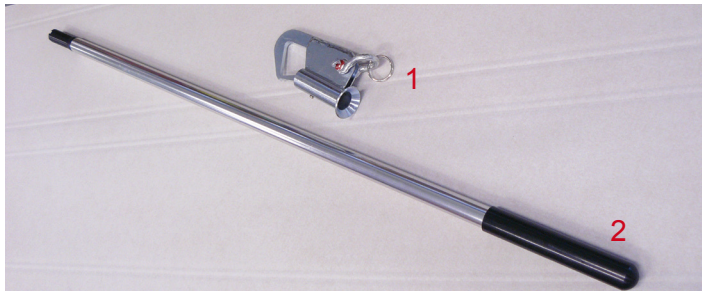

The extension arm can be attached to the lifter in two possible ways. The Molift Nomad can be attached directly on the extension arm hook with the carabine hook (3). The lifter will lose some lifting height.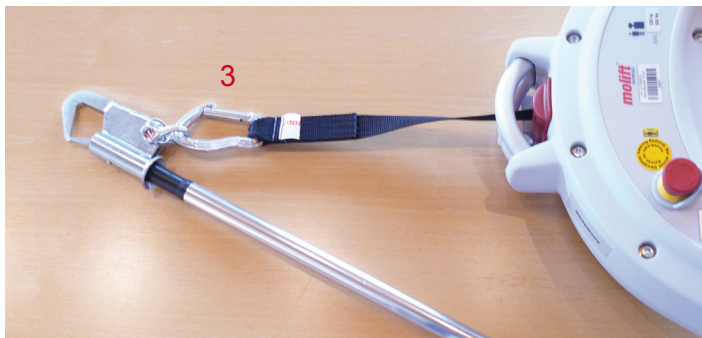

NOTE! The securing ring must be fixed in the hole in the screw head, to secure the screw(7). It is recommended to remove the carabine hook from the belt (4) and attach the lifting belt directly on the attachment hook. In this way the lifter will not lose lifting height.


Remove the lower hook on the extension hook (5).

Thread the lower hook on the lifting belt and place the securing ring (6) on the hook, replace the screw.

The screw is secured by pushing the securing ring (7) through the hole in the screw head.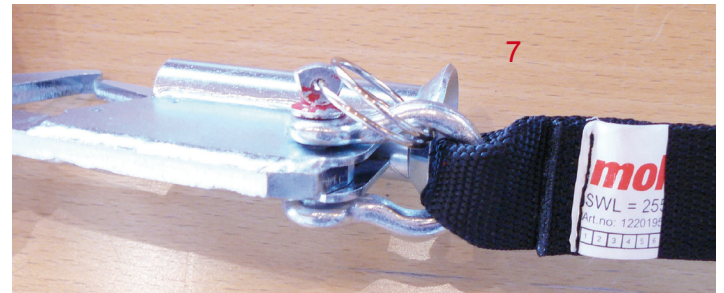

When the screw is secured, the lifter and extension arm are ready for use.

Attach the extension arm handle on the hook and the lifter is now ready to be attached to the rail trolley. Use the handle to open the attachment hook. When the lifter is attached to the trolley, the handle can be removed and stored on a suitable place (i.e. on Molift Cart).

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