660 mm (26")


100 kg (220.4 lb)

Multi Frame:x User Guide

EN 12183

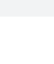2021-01 rev.9

| | | | |
|---|---------|---|----------|
|  | 3 |  | 20 - 21 |
|  | 4 - 5 |  | 22 |
|  | 6 |  | 22 |
|  | 7 |  | 23 - 26 |
|  | 8 | mm (inch) | 27 |
|  | 9 |  | 28 |
|  | 10 - 11 | ENGLISH | 29 - 32 |
|  | 12 | DEUTSCH | 33 - 36 |
|  | 13 | NEDERLANDS | 37 - 40 |
|  | 14 | DANSK | 41 - 44 |
|  | 15 | NORSK | 45 - 48 |
|  | 16 - 17 | SVENSKA | 49 - 52 |
|  | 18 | SUOMI | 53 - 56 |
|  | 19 | ESPAÑOL | 57 - 60 |
| | | FRANÇAIS | 61 - 64 |
| | | ITALIANO | 65 - 68 |
| | | PORTUGUÊS | 69 - 72 |
| | | PORTUGUÊS DO BRASIL | 73 - 76 |
| | | РОССИЯ | 77 - 80 |
| | | POLSKI | 81 - 84 |
| | | ČESKY | 85 - 88 |
| | | 中國 | 89 - 92 |
| | | 日本 | 93 - 96 |
| | | ΕΛΛΗΝΙΚΗ | 97 - 100 |
| | | | |
| | | | |

Multi Frame:x 3


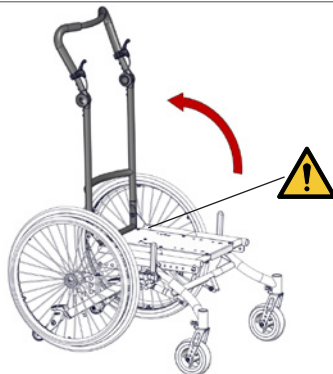
1.


2.

05 

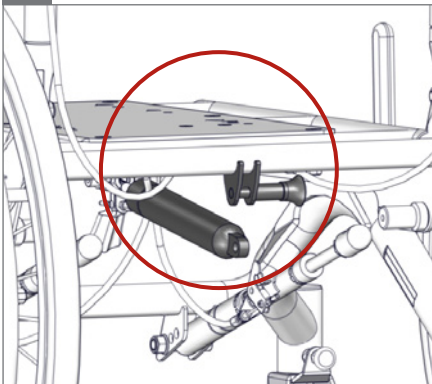
3.

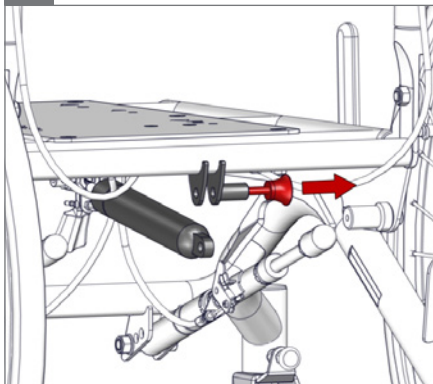
4.

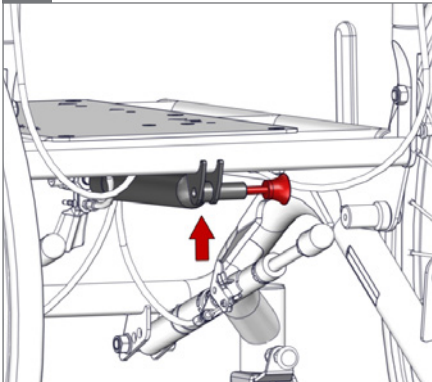5.

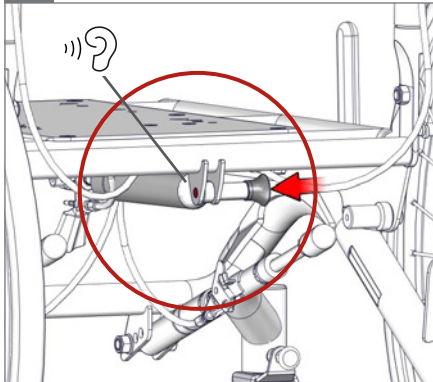6.


7.

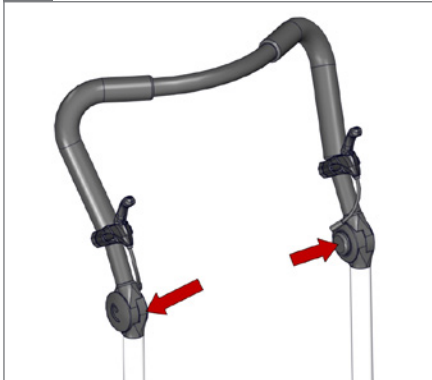
8.

9.

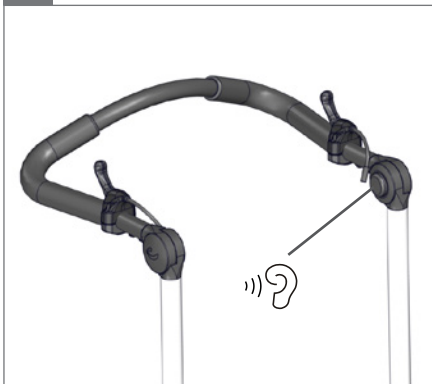10.



11.




05

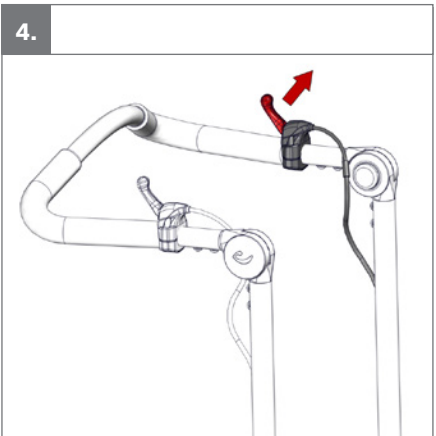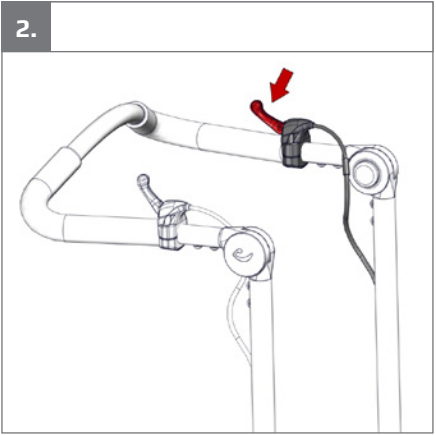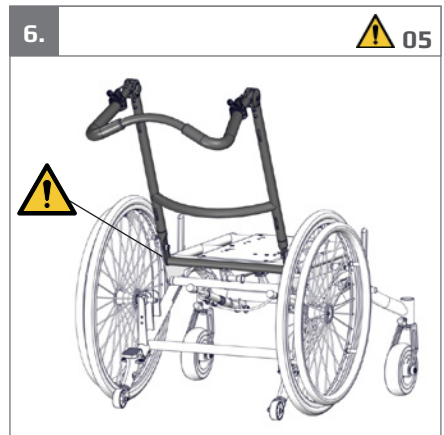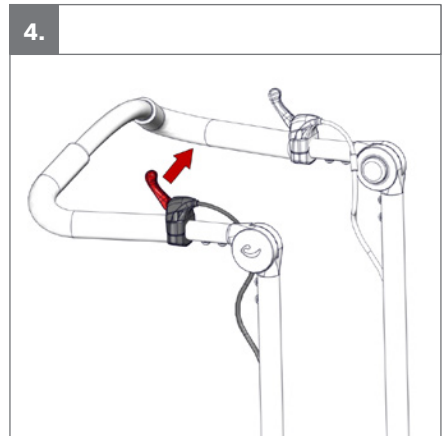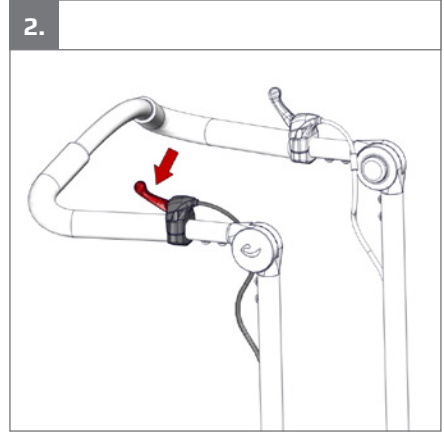12.



1.

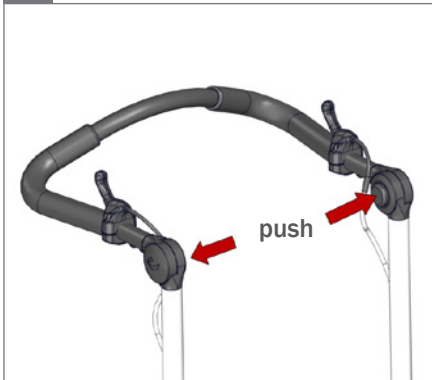2.

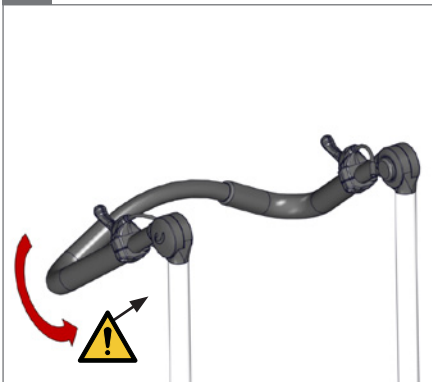
3.

05 

4.

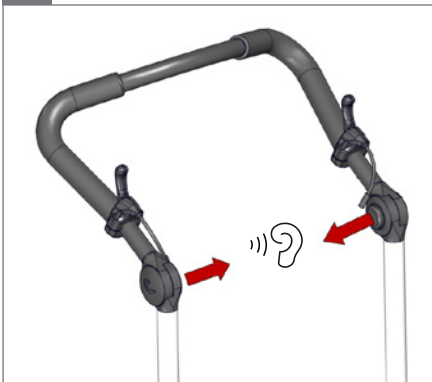5.


6.

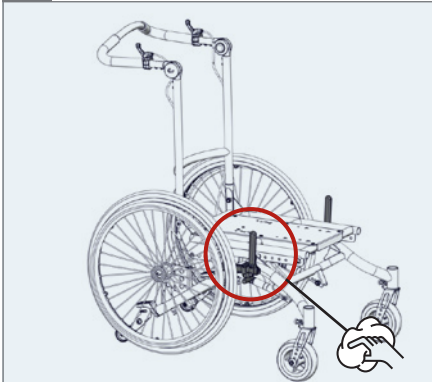
1.


2.

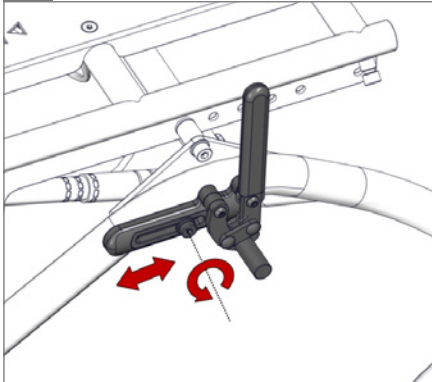
5 mm - 2,5 Nm

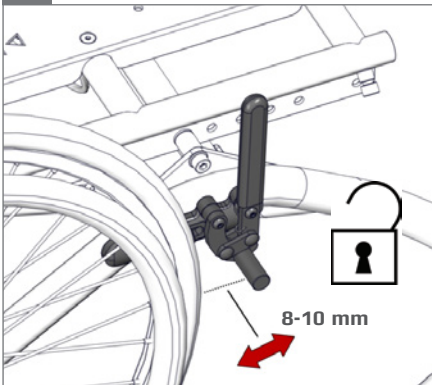3.

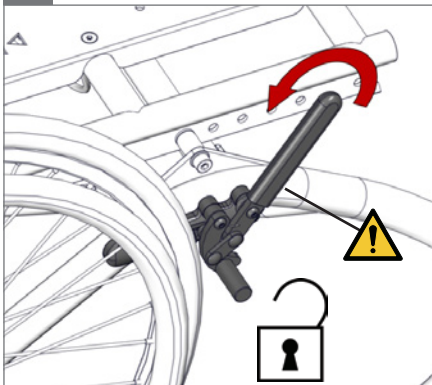
4.


5.

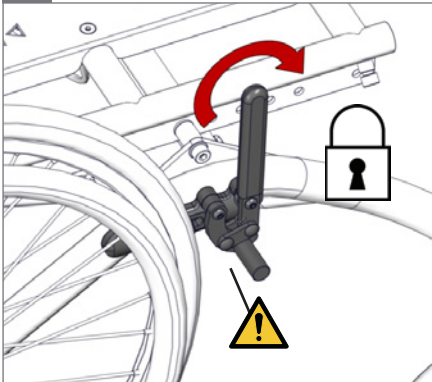05 !

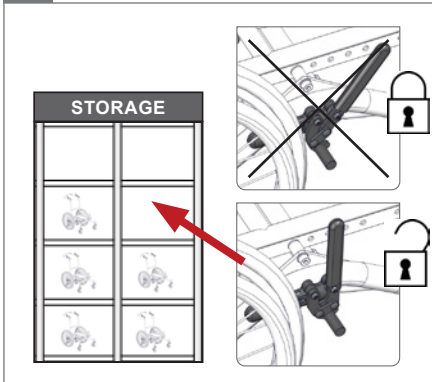6.



1.

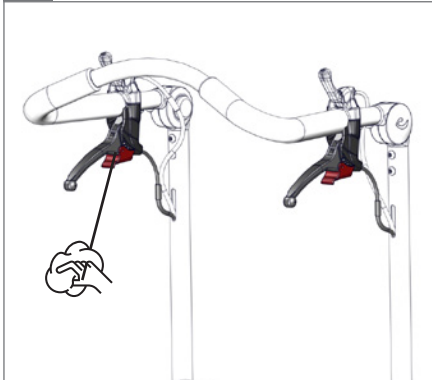
2.

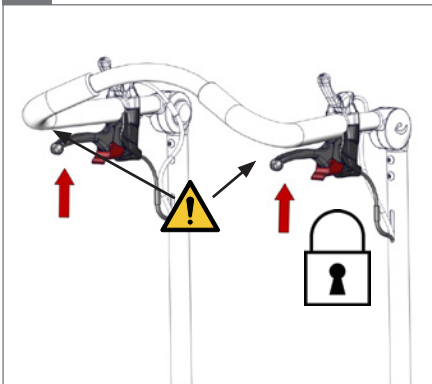
3.

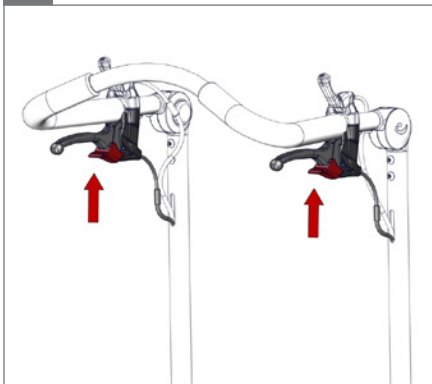
4.

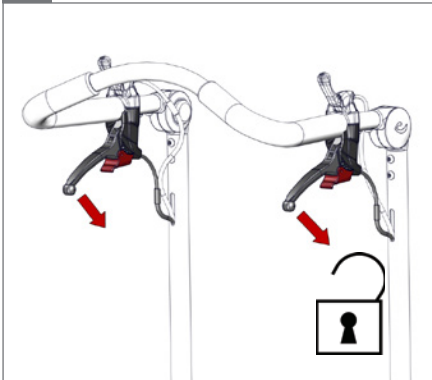5.

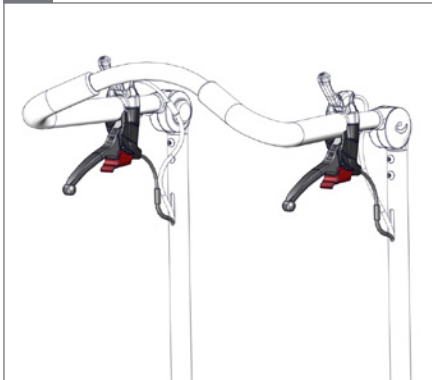6.


7.


8.

 8 mm

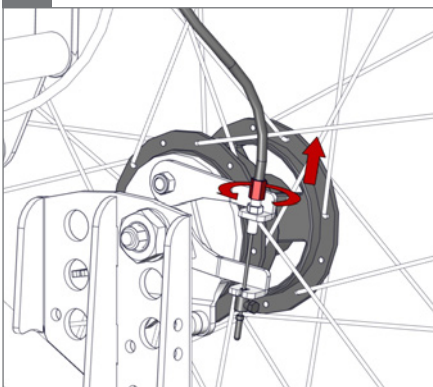
9.

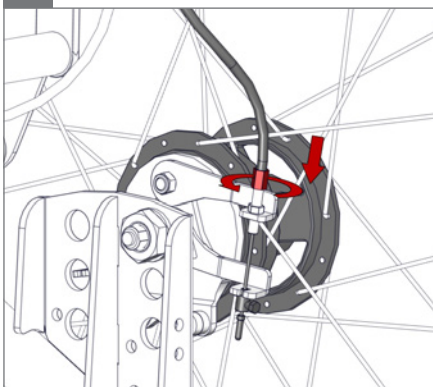
10.

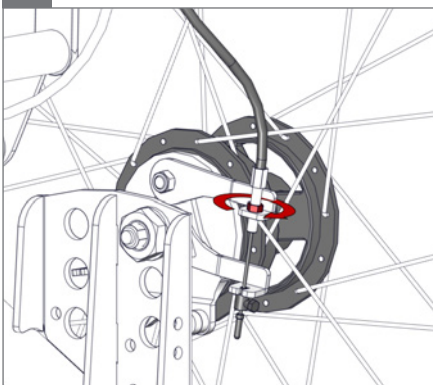11.

 8 mm

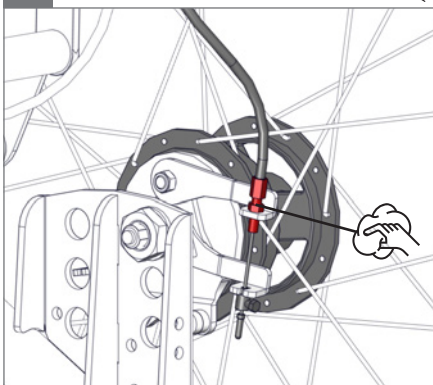12.



1.

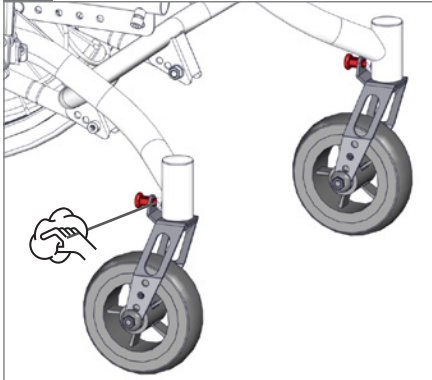
2.

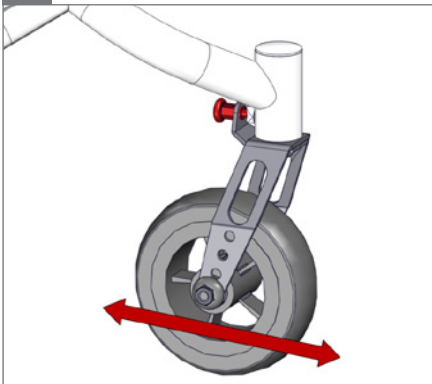
3.

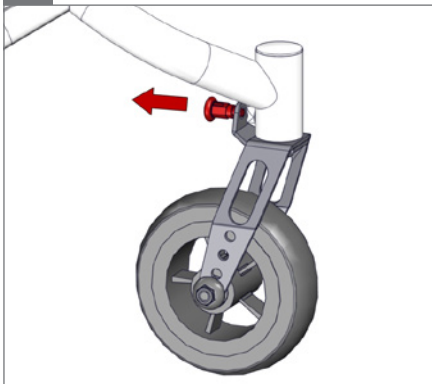
4.

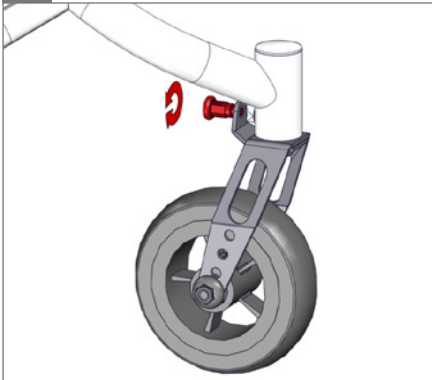5.

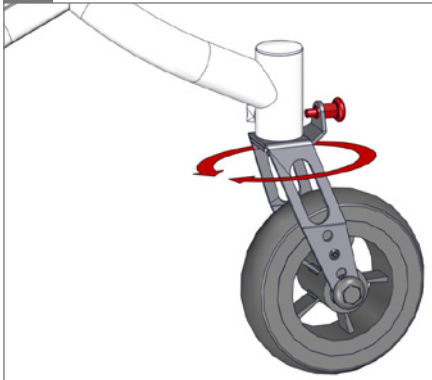6.



1.

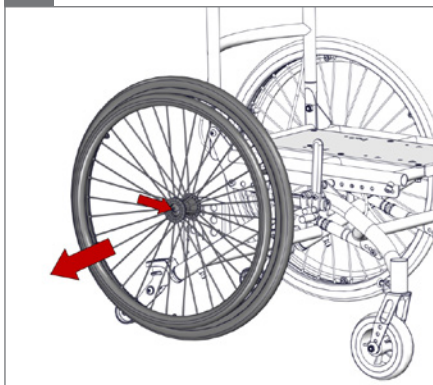
2.

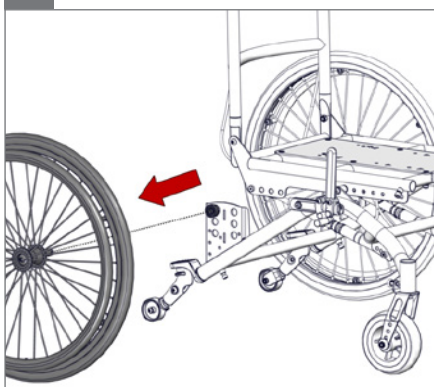
3.

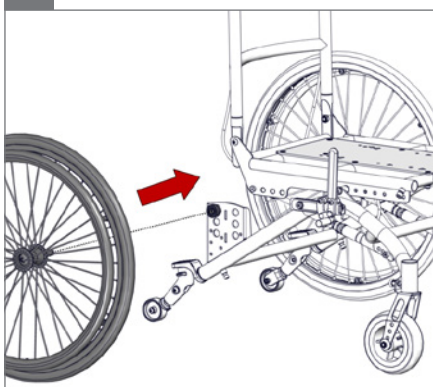
4.

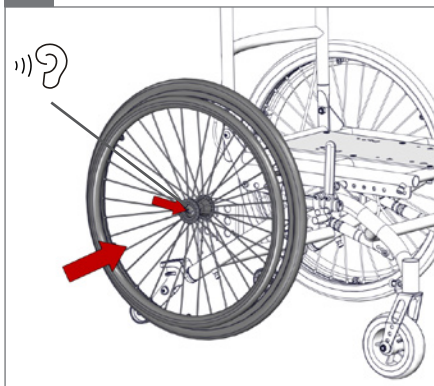5.

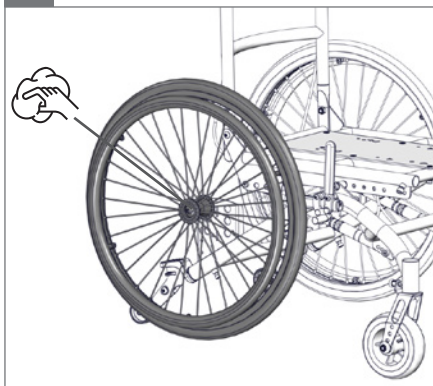
6.


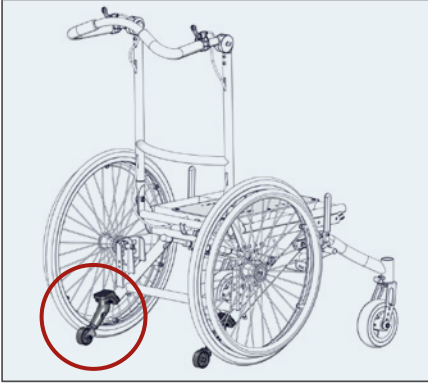
1.

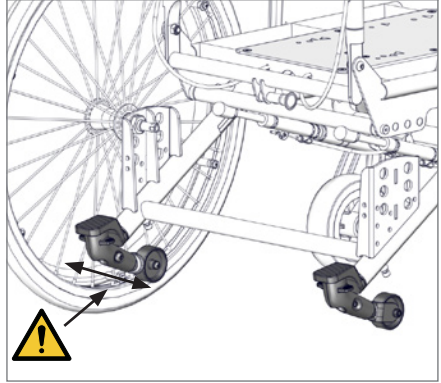
2.

 05

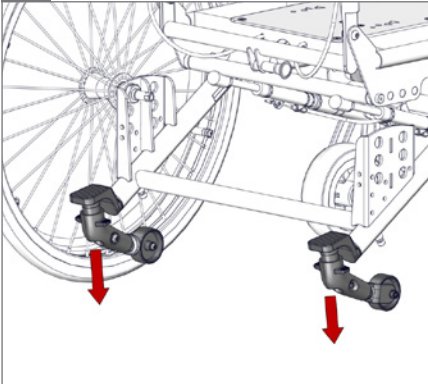
3.

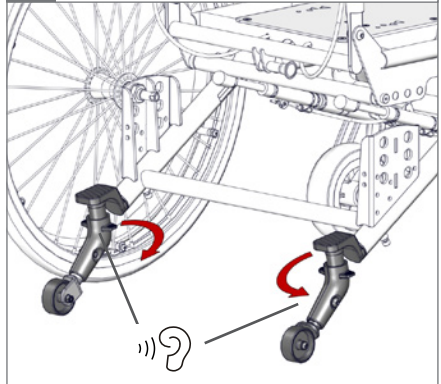
4.

 02

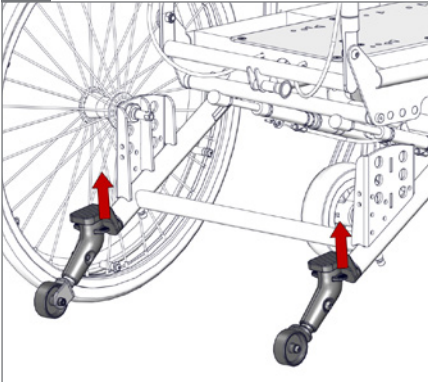5.

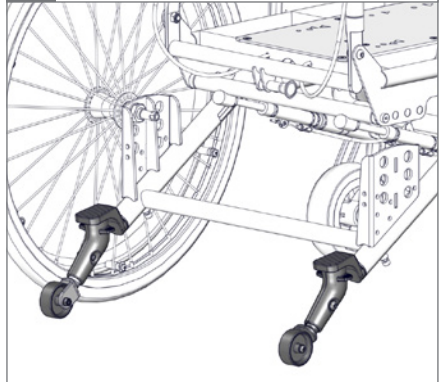6.


1.

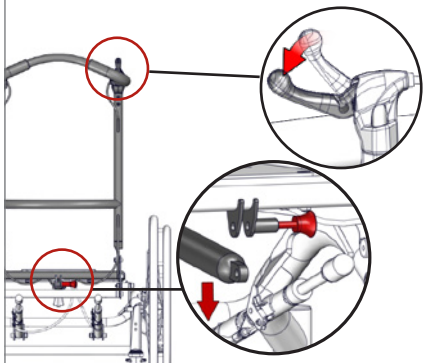
2.


3.

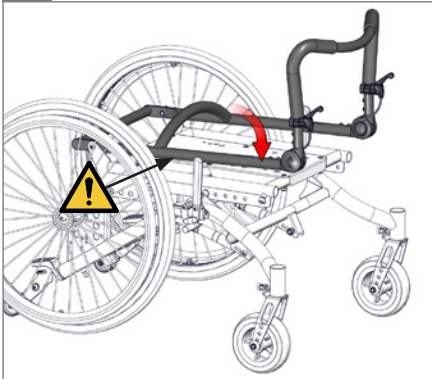
05

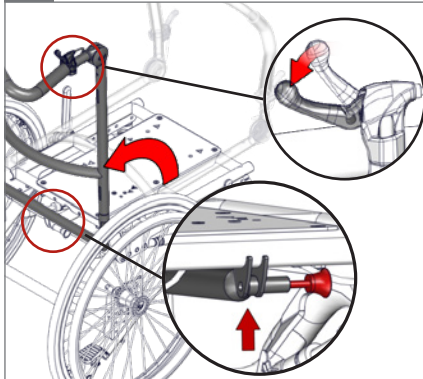4.


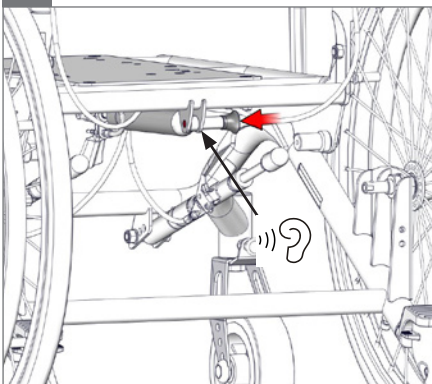
5.

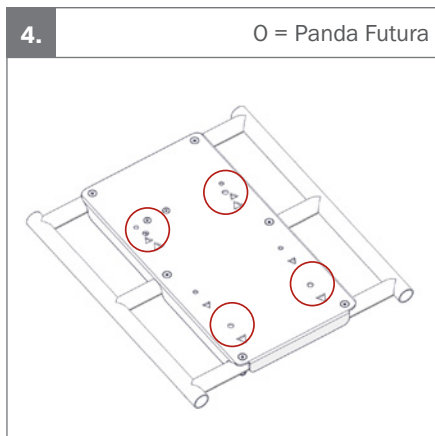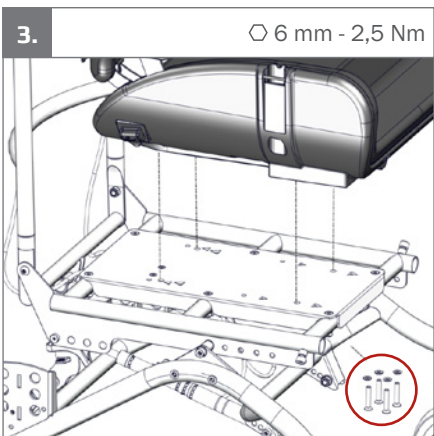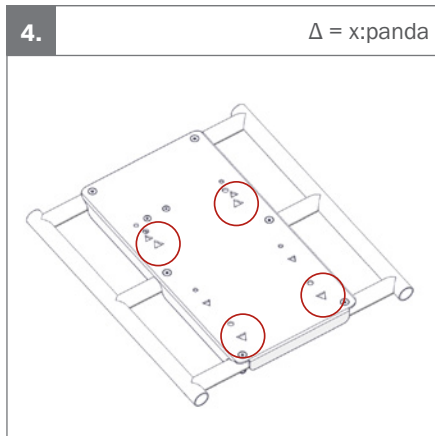
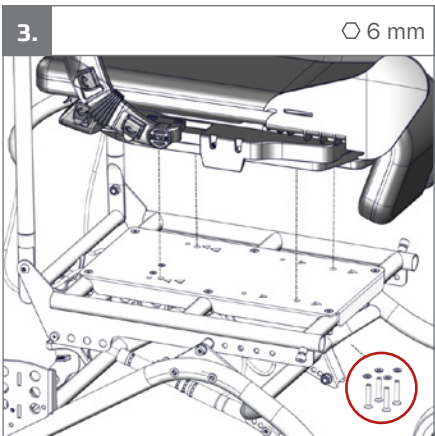
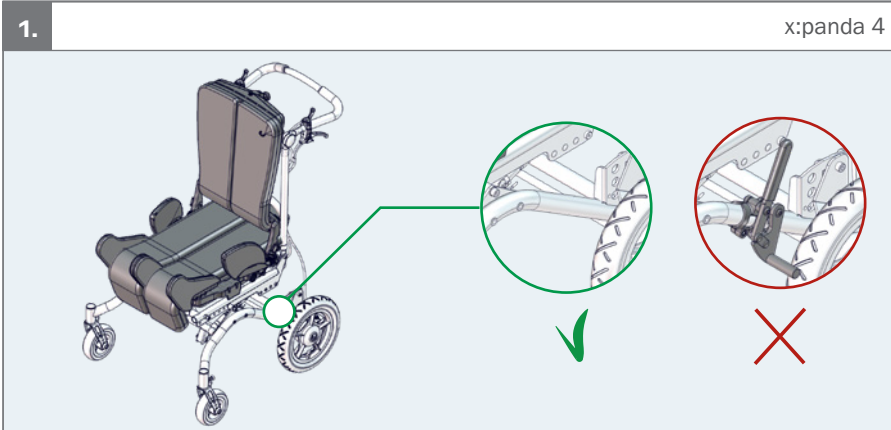
6.

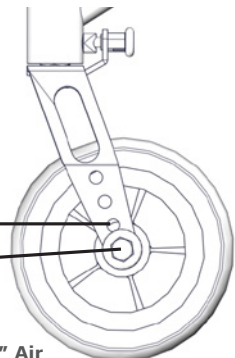
| | | 12½" | 20" | 22" | 24" |
|----------------|--|------|-----|-----|-----|
| | | | | | |
| Panda Futura 5 | | √ | √ | √ | √ |



| | | 12½" | 20" | 22" | 24" |
|-----------|--|------|-----|-----|-----|
| | | ⊗ | ⊗ | ⊗ | ⊗ |
| x:panda 4 | | √ | - | - | - |


|  |  |  |  |  |
|---|---|---|---|---|
| | 12 1/2" | 20" | 22" | 24" |
| 6" | √ | √ | √ | √ |
| 7" | √ | √ | √ | √ |
| 6" | 1. | 1. | 1. | 1. |
| 7" | 2. | 2. | 2. | 2. |
| MIN PSI/bar | - | 85/5.9 | 85/5.9 | 85/5.9 |
| MAX PSI/bar | 26/1.8 | 145/10 | 145/10 | 145/10 |
| Handrim diameter | - | 445 mm (17 1/2") | 480 mm (19") | 530 mm (20 3/4") |








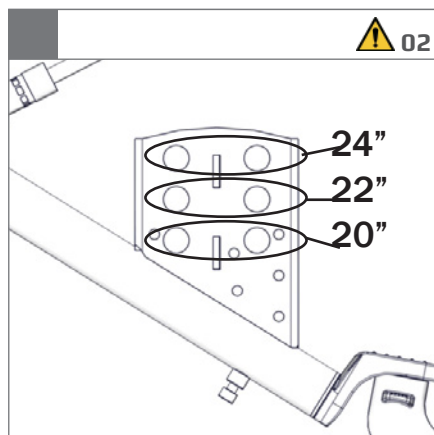
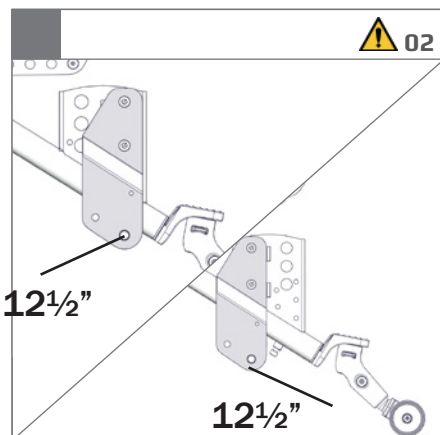
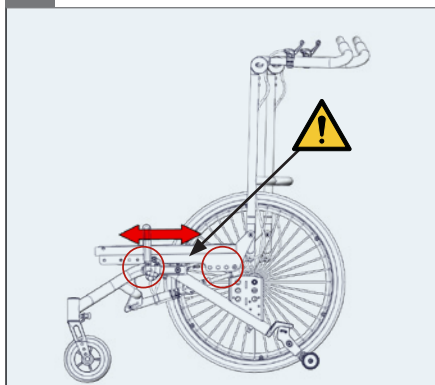
7" Air
MAX 36PSI/2.5bar



1.  02

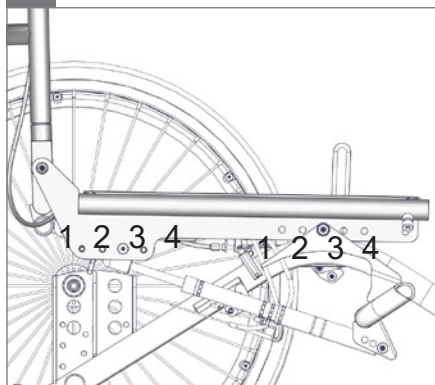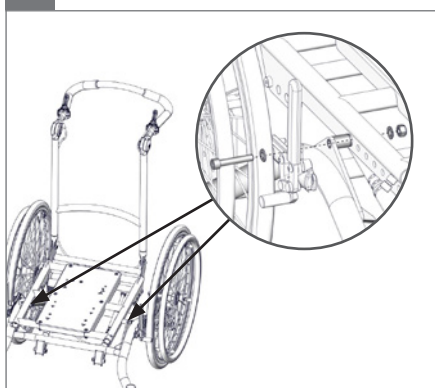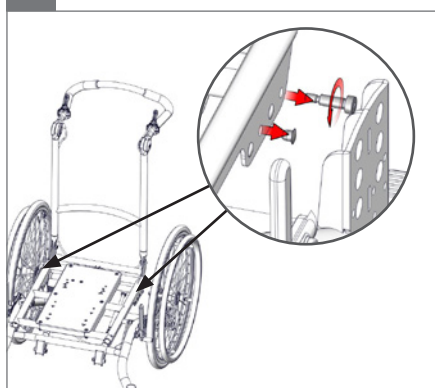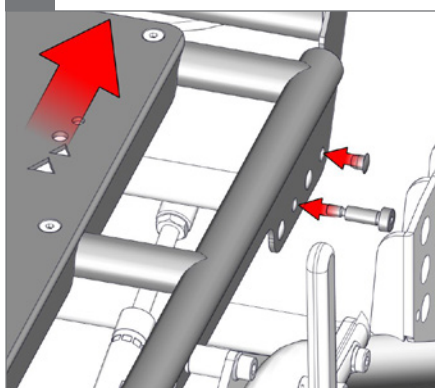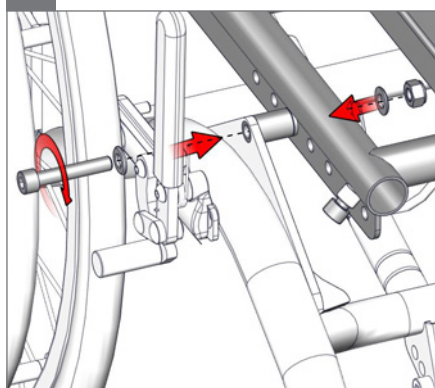2.

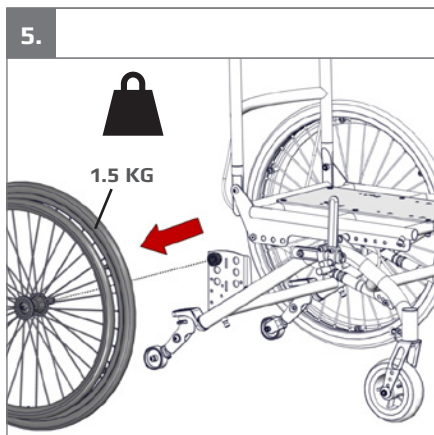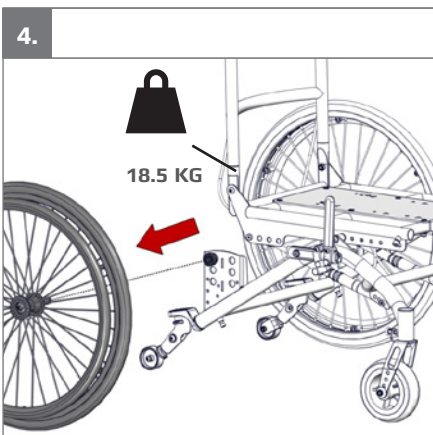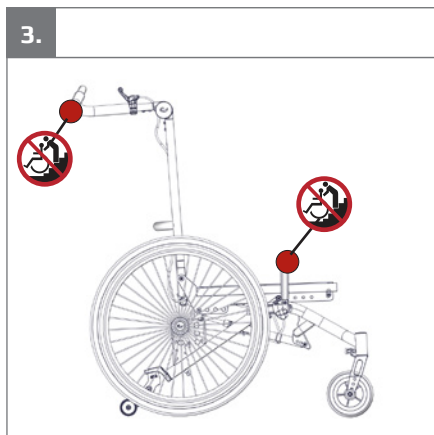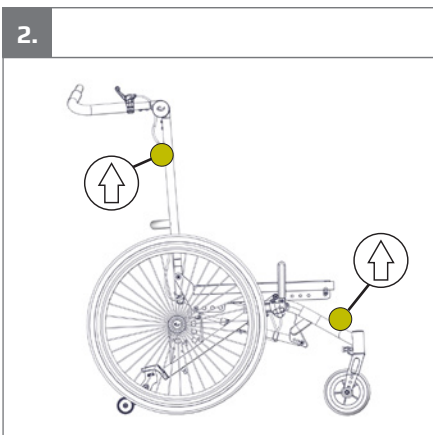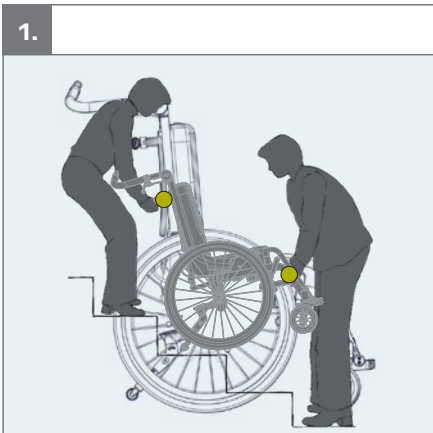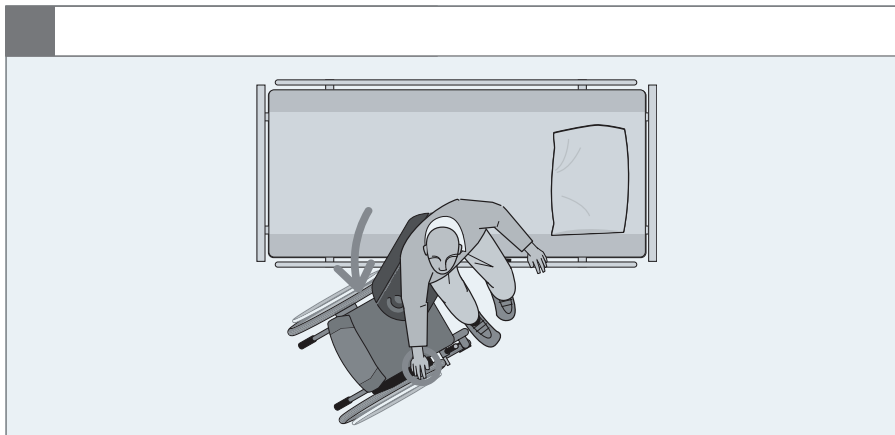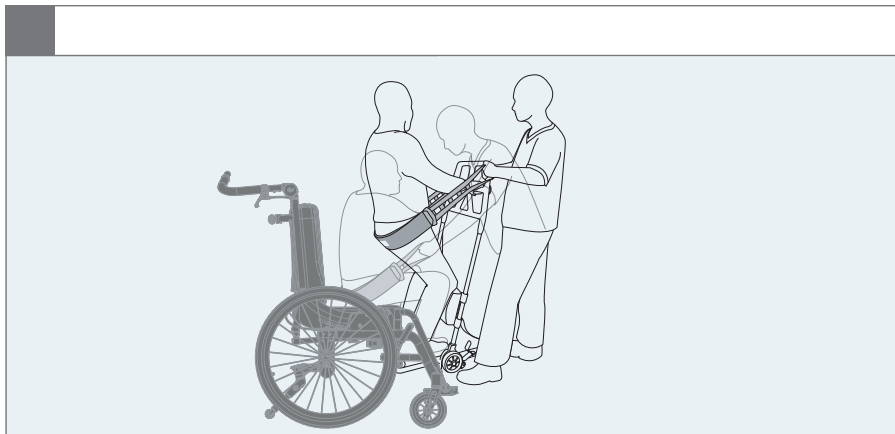3.  6 mm &  13 mm4.  4 mm5.  4 mm6.  6 mm &  13 mm

EN: For indoor and outdoor use

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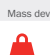
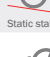
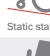
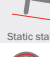
ES: Uso incorrecto


BR: Uso incorreto

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| mm (inch) |  3 |
|--|---|
|  Overall width | 660 mm (26") |
|  Width between wheels | 535 mm (21") |
|  Length | 840 mm (33") |
| Folded length | 840 mm (33") |
|  Overall height | 980 mm (38½") |
|  Back height | 570 mm (22½") |
|  Back angle | 0–30° |
|  Seat plane angle | -5–30° |
|  Seat surface height at front edge | 410 / 435 / 460 mm (16¼" / 17¼" / 18") |
|  Base plate (width x depth) | 215 x 405 mm (8½" x 16") |
| Footrest to adaptor plate | 260–430 mm (10¼–17") |
|  Max load | 100 kg (220.4 lb) |
|  Max load at transportation | 100 kg (220.4 lb) |
|  Mass device | 20 kg (44 lb) |
|  Max rated load | 122 kg (268.9 lb) |
|  Static stability downhill | >10° |
|  Static stability uphill | >10° |
|  Static stability sideways | >10° |
|  Handrim diameter | 480 (19") |

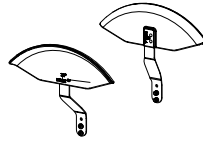
9996097118


9996097098

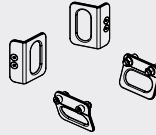
9996097755

9996097099

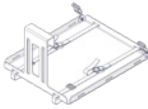
9996097009

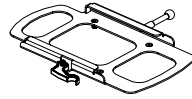9996097265

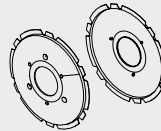
9996097714


9996097137

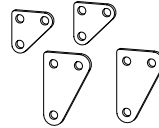9996097167

9996097096

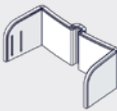9996097738

N/A

9996097739

N/A

9996097097

N/A

ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and adults with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The device is a frame for seating systems for children and adults with disabilities.

Indications

The device is used together with a seating system (i.e. a modular seating system like R82 x:panda or R82 Panda Futura). It can be used by children and adults with disabilities, who lack the ability to maintain a seated position and move around (Within the room or between rooms / outdoors) without an assistive device. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age.

The user is the person sitting in the seat used with the frame. The caregiver is the person operating the frame. Never leave the user unattended in this product. Ensure permanent supervision by a caregiver. The use of the device will be discontinued more times during the day where the user will be lying down, standing or sitting in another assistive device or a conventional product.

For transportation the device is suitable for children and adults in the weight range of 43 kg to specified max user weight for transportation.

Contra indications

There are no known contra indications

Precautions

There are no known precautions

Application areas

The product is for indoor and outdoor use

Enclosed tools:

- 4, 5 and 6 mm Allen key
- 13 mm spanner

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12183

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Alignment of user weight/mass of the combined seat and base, approval for transportation in vehicles of both products (ISO16840-4 for seat and ISO7176-19 for base)
- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the etac homepage/download.

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded. The device may be reissued within its lifetime.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

Presales information

- www.etac.com

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Be aware of any MAX mark indications before adjustment
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Always use the product in a moderate pace. Do not run with the product
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/accessories.
- Information about transportation can be found in the document "Transportation in Motor Vehicles" supplied with the product
- Information about roadability can be found in the document "Roadability" supplied with the product
- Information regarding frame and seat combinations including max. load in combination can be found at: etac.com

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Removable parts without use of tool: wheels and foot support. The removal will have adverse effect on the device
- Activate the brakes before placing the user into the product
- Activate the anti-tips when the product is being used on inclines
- Activate the anti-tips before angling the seat and back
- Deactivate the anti-tips when forcing kerbs higher than 100 mm
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Ensure the wheels are fully-functional and secure before use
- The correct tyre pressure is printed on the tyre casing and listed in the section for tyre technical data.
- In case of a tyre puncture do not use the product until remedied
- Ease of access to and maneuverability in, motor vehicles can be significantly affected by size of device and turning radius, and the smaller device and/or device with a shorter turning radius will generally provide greater ease of vehicle access and maneuverability to a forward facing position. In accordance with ISO 7176-19 Annex D, the product is evaluated regarding the ability to accommodate vehicle-anchored pelvic and shoulder belts. The device receives an overall score of "Good"
- Unoccupied device is suitable for land and air transport
- Attaching additional loads can adversely affect stability. Position of points intended to carry additional loads are specified in the relevant Mounting Guide referred to under section for accessories
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift
- To achieve the required static stability the Multi Frame:x with the Panda Future 5 seat must be set in the most forwards position. In other positions the static stability is not sufficient. When setting the Multi Frame:x with other seats or changing position, please ensure that the seat and wheels are in a stable position.
- Inspect the product and all of its accessories and replace any worn out parts before use

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product.

Contact local dealer for assistance, if needed, in setting up, using or maintenance of the device.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment

Weekly

- Check that all wheels are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use
- Disinfect the product. please check the disinfection section in this User Guide for more information
- Tire pressure

Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

Washing machine

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

Before washing remove the following items and clean them separately:

- Seat
- Accessories

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users), every time the product is reissued for use and after long term storage (more than 4 months). The inspection should be carried out by a technically competent person who has been trained in the use of the product.

Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- If a seat adapter is mounted, check that the red handle for locking works and that the safety lock "clicks into position when placing a seat into the adapter
- Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required
- Quick release wheels are removable/replaceable
- Anti-tips can be fixed in working and resting position
- Tire pressure
- Before storage: Always deactivate wheel locks/brakes
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

Safety product

Due to variant and settings the wheelchair may become larger than the recommended dimensions.

Compliance with recommended overall width and overall length ensures unhindered use of emergency escape routes. (See table 1)

Table 1

| Recommended dimensions | Ready for use (mm) |
|----------------------------|--------------------|
| Overall length - maximum | 1200 |
| Overall width - maximum | 700 |
| Overall height - maximum | 1200 |
| Turning diameter - maximum | 1000 |
| Pivot width - maximum | 1300 |
| Ground clearance - minimum | 30 |

Product safety

Due to variant and settings the wheelchair may not meet the limits for wheelchairs transportable by train. Measures can be taken to verify whether the specific product in question meets the limits. (See table 2)

Table 2

| Wheelchairs transportable by train - limits | |
|--|---|
| Overall length - maximum | 1200 mm + 50 mm for feet |
| Overall width - maximum | 700 mm + 50 mm on each side for hands |
| Overall height - maximum | 1375 mm (including 95th percentile male occupant) |
| Turning diameter - maximum | 1500 mm |
| Smallest wheels | Accommodate a gap of 75 mm horizontal and 50 mm vertical |
| Weight - maximum | 200 kg for manual wheelchair and occupant (including baggage) |
| Obstacle height that can be overcome - maximum | 50 mm |
| Ground clearance - minimum | 60 mm under foot rest with upward slope angle of 10° on top for going forward |
| Safe slope on which the wheelchair remain stable - maximum | 6° dynamic stability in all directions 9° static stability in all directions, including with brake applied |

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R82 A/S