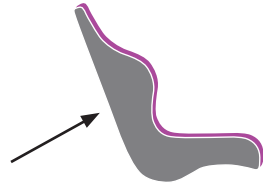
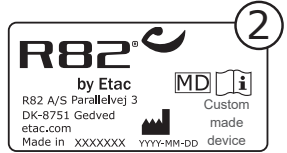
Turtle User Guide


Custom made device

DANSK	3-17
ENGLISH	8-12
NORSK	13-17
РОССИЯ	18-22


EN	DK	NO	RU
Labels identification	Identifikation af etiketter	Etikettidentifikasjon	Идентификация этикеток
1: Serial number	1: Serienummer	1: Serienummer	1: серийный номер
2: Manufacturer	2: Producent	2: Produsenten	2: режиссер

<p>For molded units Typically located on the back of the device.</p> <p>For R82 frames Check the accompanying user guide</p>	<p>For formstøbte enheder Typisk placeret på ryggen af enheden.</p> <p>For R82 stel Se placering af labels i User Guide der følger med stellet</p>	<p>For støpte enhetene Typisk ligger på baksiden av enheten.</p> <p>For R82 chassis Se plassering av etiketter i brukerhåndboken som følger med rammen</p>	<p>Для получения формованных блоков Как правило, расположены на задней стороне устройства.</p> <p>Для R82 шасси См размещения меток в руководстве пользователя, который поставляется с рамой</p>
--	--	--	--

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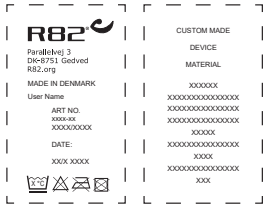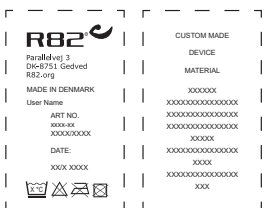Labels in inlays
Labels i indlæg
Etiketter i innleggene
Этикетки в постах

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and adults with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

Turtle is form-casted plastic and foam units. This means that no Turtle products are alike. Turtle is shaped using an imprint of a user, this results in exactly the support required for a user.

Turtle is classified as, custom-made device and only a technically competent professional, trained in the use of these products should prepare the products for a user.

The product is for indoor and outdoor use except Turtle products intended for Toilet and bath, these are intended for indoor use. R82 assumes no liability for damage or personal injury if the product is not used for the intended purpose.

When combining Turtle with other products. The combination must not change the intended purpose of either products or modify the products in such a way that compliance with the applicable requirements may be affected.

The combiner of the products must ensure, that applicable requirements are fulfilled.

Applicable requirements examples: alignment of user weight/mass of the combined seat and base, approval for transportation in vehicles of both products (ISO16840-4 for seat and ISO7176-19 for base), material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the etac homepage/download.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

Consult instructions

The latest version of the instructions is always available on the etac website and can be printed in larger sizes

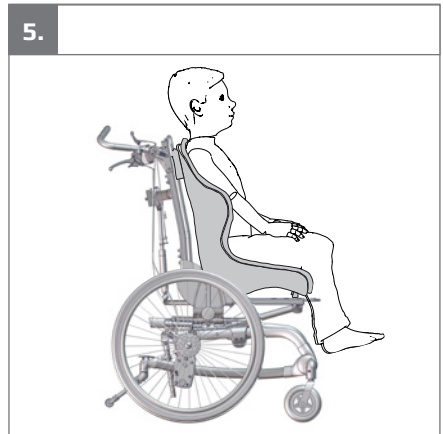
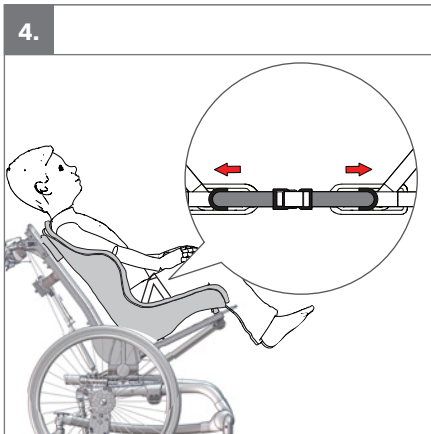
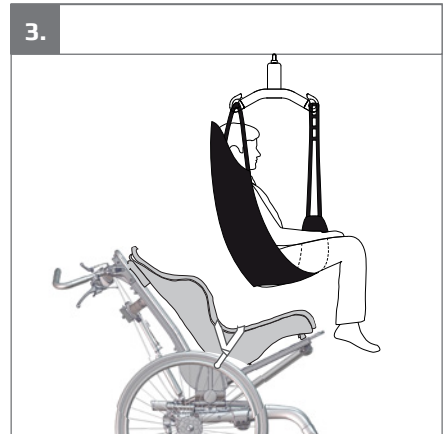
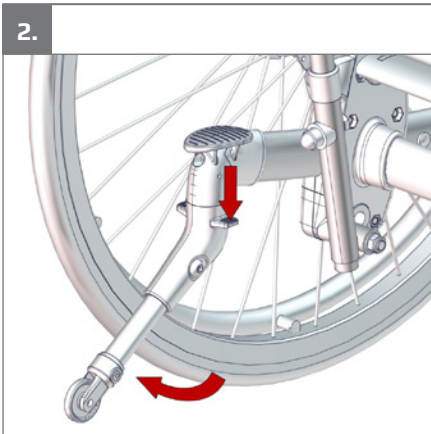
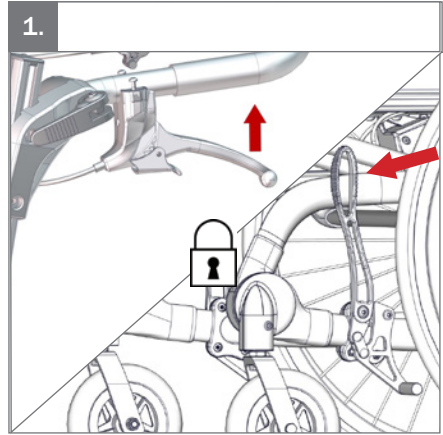
Adjustment

This product has been selected and set up in cooperation with a qualified professional. Adjustment of the product helps to maintain optimal support and is important in relation to the user's growth and development. The caretaker takes responsibility for the safety and effectiveness of the properly adjusted position. These settings should not be changed without the permission of the qualified professional.

Location of a user

In molded units on a R82 frame see illustration 1-5

Always check that the user is correctly positioned and secured using vest and belts

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions. In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Always use the product in a moderate pace. Do not run with the product

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- The product is molded for one user only. Multiple users is not allowed
- The brake and anti-tip must be activated before placing a user
- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children

- The brake and anti-tip must be activated before placing a user
- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the back
- Ensure the castors are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- Be careful not to squeeze fingers when angling a seat and when adjusting accessories
- When a user is seated in the product, any vest and belt must be secured properly
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1 and EN 1021-2
- This product might lose its flame resistant characteristics if using other than original R82 cushions

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

Disposal - 

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- The instructions in the accompanying User Guide for the frame where the product is mounted should also be read carefully before use and stored for future reference
- Store the enclosed tools on/at the product

Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment
- Check the cover for the foam unit. If necessary, wash the cover according to the instructions on the label in the cover

Cleaning

It is important that the product is kept clean, both for the well-being of the user and for the life of the product.

It is a good idea to take the inserts out of the plastic frame / plastic shell at night. In this way, Condensation dry and odors are minimized and the life of the product is extended.

- Incontinence cover is marked with washing instructions, can be washed at 90° C and air dried
- Dirt on the plastic foam frame / plastic shell is removable, using a sponge or soft brush. A mild detergent or disinfectant may be used if necessary
- Username is given on the wash instructions for foam and plastic insertion

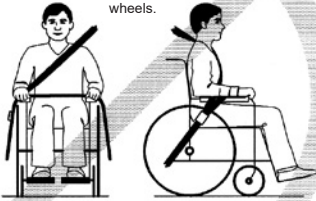
Foam units and covers

- The foam unit does not tolerate washing and must not be dried on a heater, in a drying cabinet or a dryer. As an option, a foam coating can be ordered, which can then be wiped with a damp cloth.
- The wash instructions for the cover are located at the top of the back part and at the back of the seat part
- The cover for molded Turtle units is removable, machine washable at 60 ° C and can be air dried. Always check with the washing instructions on the cover before washing
- To make sure the cushions are properly placed, you may want to put a piece of plastic over the velcro. After placing the cushions, this can easily be pulled out
- The fabric is biocompatibility tested according to ISO 10993-5: 2009

Plastic units

- The washing instructions are placed at the top of the back.
- Hand wash is preferable due to wear, but the inserts can withstand machine wash at 40 ° C. Always check with the washing instructions on the insert before washing.
- The inserts are marked with washing instructions as well as the number 1, 2 or 3. Inlay 1 is closest to the user.
- The inserts must dry at room temperature. Do not place the inserts directly against a heat source and do not dry them in the drying room or dry room as this may cause the inserts to lose their shape.
- The inserts must dry at room temperature. Do not place the inserts directly against a heat source and do not dry them in a drying room or a dryer as this may cause the inserts to lose their shape.
- The inserts are biocompatibility tested according to ISO 10993-5: 2009

Belt restraints should not be held away from the body by wheelchair components or parts, such as the wheelchair armrests or wheels.

Transportation in motor vehicles

- The molded unit is approved for transport to users in motor vehicles when mounted on an approved frame and facing the direction of travel. The following must be followed for the safety to be maintained.
- Approval is voided by changes in the unit that have not been made by a qualified professional
- The user must, if possible, be moved to one of the vehicle's fixed seats and use the vehicle seat belt
- The molded device is surrogate tested and approved according to ISO 16840-4: 2009
- The lowest maximum weight for the frame or foam / plastic unit must always be observed
- Foam units are approved for transport with a maximum load of 136 kg. Also see max weight for the frame
- Plastic units are approved for transport with a maximum load of 75 kg. Also see max weight for the frame
- If weighing below 22 kg, foam or plastic units cannot be used. An approved car seat is recommended
- The molded unit must be mounted on a frame that has been dynamically tested and approved according to ISO 7176-19: 2008

The frame with the molded device is fastened to the bottom of the motor vehicle. Follow the manufacturer's instructions for transportation in the User Guide for the relevant frame.

Transportation

- Head-support must be used
- If the backrest cannot be set to 0-10 ° relative to the vertical or the seat cannot be set at 0-30 ° relative to the horizontal, the approval for transport in a motor vehicle will be canceled in accordance with ISO 16840-4: 2009.

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R82 A/S