


Page 1: Standard Assembly


SIZE	KIT NAME	ITEM NO.	DESCRIPTION	QTY2	KIT
18	8" Urethane Wheel & Bearing Assy 7/16"	27	8" Urethane Wheel & Bearing Assy 7/16"	1	992647
12	Footplate Assembly SMALL - LH	29	CX-10 Footrest Assembly, LH	1	992658
12	Footplate Assembly SMALL - RH	30	CX-10 Footrest Assembly, RH	1	992659
14	Footplate Assembly STD - LH	29	Footrest Assembly - Standard LHS	1	992660
14	Footplate Assembly STD - RH	30	Footrest Assembly - Standard RHS	1	992661
16, 18	Footplate Assembly Adult - LH	29	Footrest Assy, Adult, LH	1	992662
16, 18	Footplate Assembly Adult - RH	30	Footrest Assy, Adult, RH	1	992663
12, 14, 16	11.5" x 2.5" Rear Solid Tire on Grey Rim (Ea)	27	Rear Wheel 290mm x 60mm (Rolko)	1	994746

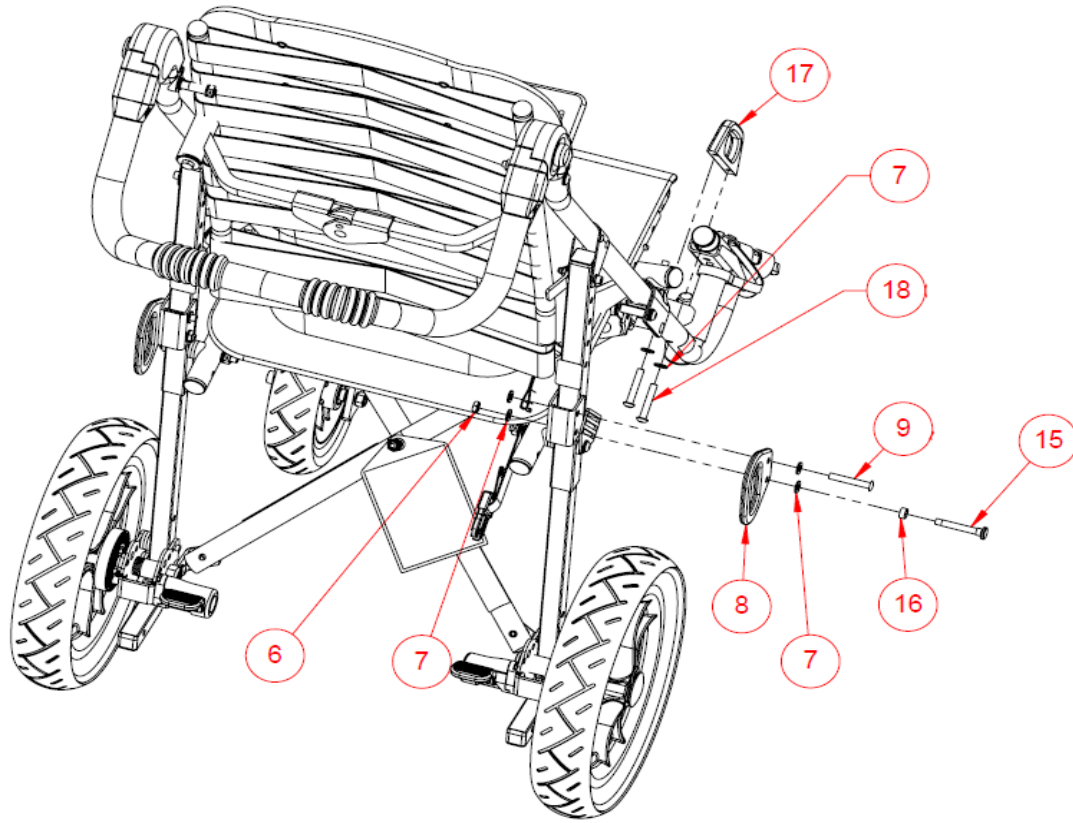


Page 2: Rider Bus Kit

SIZE	KIT NAME	ITEM NO.	DESCRIPTION	QTY2	KIT	
12	EZ12T Transit Conversion Kit	6	Locknut 1/4"-28 Jam Flexlock Black Zinc	2	990461	
12		7	Washer 1/4" AN-960-416	10		
12		8	Transport Anchor	2		
12		9	Bolt 1/4"-28 x 3" Socket Button Head Alloy S	2		
12		13	Caution Card WC/19	1		
12		14	Anchor Logo	4		
12		15	Transit Anchor Lock Pin 1-3/8"	2		
12		16	Transit Anchor Lock Pin Sleeve	2		
12		17	Inline Transit Anchor	2		
12		18	BOLT 1/4"-20 X 1-1/2" Socket Button Head	4		
14		6	Locknut 1/4"-28 Jam Flexlock Black Zinc	2		
14		7	Washer 1/4" AN-960-416	12		

14	EZ14T Transit Conversion Kit	8	Transport Anchor	2	990561
14		9	Rivet 1/4" X 1-5/8" Stainless Steel	2	
14		13	Caution Card WC/19	1	
14		14	Anchor Logo	4	
14		15	Transit Anchor Lock Pin 1-5/8"	2	
14		16	Transit Anchor Lock Pin Sleeve	2	
14		17	Inline Transit Anchor	2	
14		18	BOLT 1/4"-20 X 1-1/2" Socket Button Head	4	
14		19	Removable Thread Locker (Blue)-Medium S	AR	
16	EZ16T Transit Conversion Kit	6	Locknut 1/4"-28 Jam Flexlock Black Zinc	2	990661
16		7	Washer 1/4" AN-960-416	10	
16		8	Transport Anchor	2	
16		9	Rivet 1/4" X 1-5/8" Stainless Steel	2	
16		13	Caution Card WC/19	1	
16		14	Anchor Logo	2	
16		15	Transit Anchor Lock Pin 1-3/4"	2	
16		16	Transit Anchor Lock Pin Sleeve	2	
16		-	Rubber Dampener	2	
16	17	Inline Transit Anchor	2		
16	18	BOLT 1/4"-20 X 1-1/2" Socket Button Head	4		
18	EZ18T Transit Conversion Kit	6	Locknut 1/4"-28 Jam Flexlock Black Zinc	2	990761
18		7	Washer 1/4" AN-960-416	10	
18		8	Transport Anchor	2	
18		9	Rivet 1/4" X 1-5/8" Stainless Steel	2	
18		13	Caution Card WC/19	1	
18		14	Anchor Logo	4	
18		7	Transit Anchor Lock Pin 1-3/4"	2	
18		16	Transit Anchor Lock Pin Sleeve	2	
18		17	Inline Transit Anchor	2	
18	18	BOLT 1/4"-20 X 1-1/2" Socket Button Head	4		


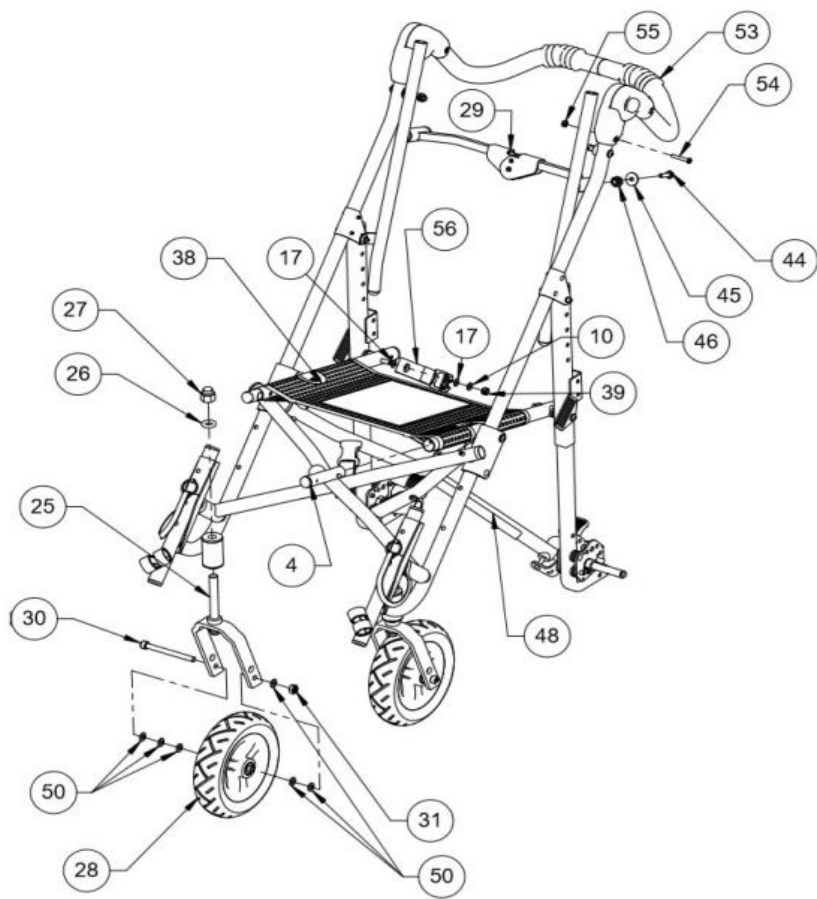


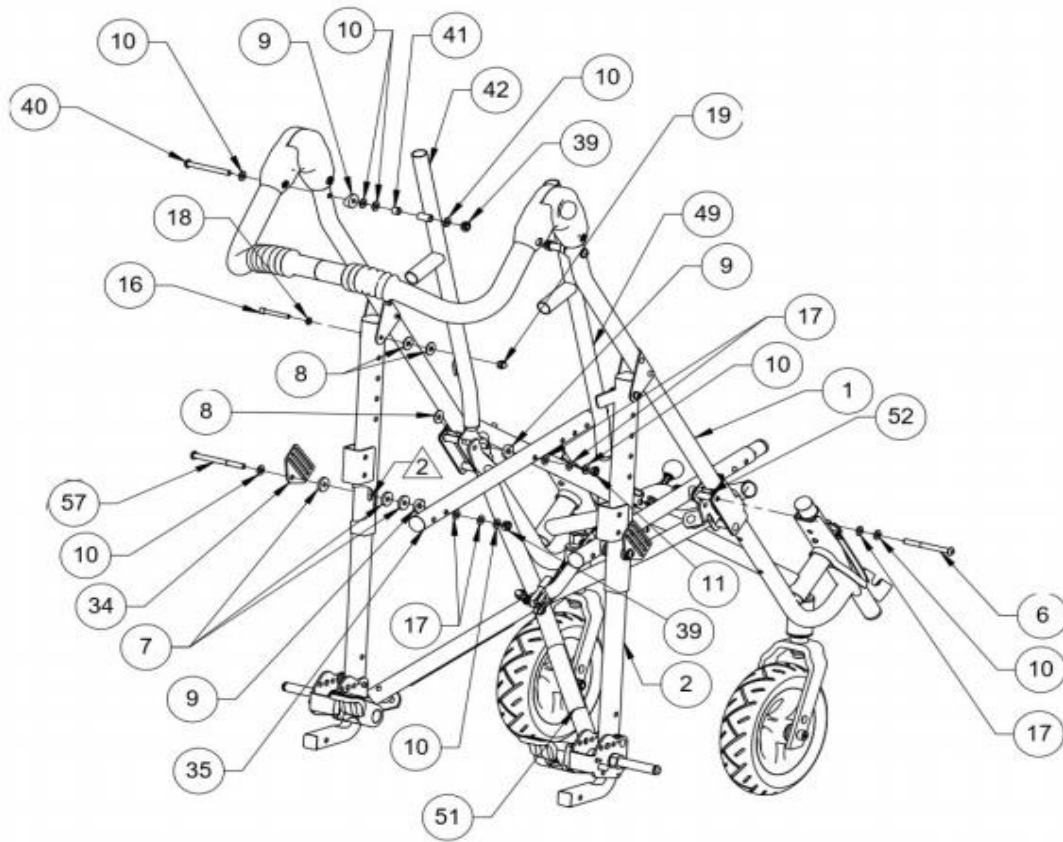

Page 3: Rear Frame Assembly, Size
12-16 Only. For Size 18, please
scroll to Page 4


SIZE	KIT NAME	ITEM NO.	DESCRIPTION	QTY2	KIT
12	EZ12 Front Assembly - Std	1	EZ-12 Front Frame Assembly	1	990401
12	EZ12 Rear Lock Brace Assembly	29	EZ-12 Rear Lock Assembly	1	990410
12	EZ12 Seat Tubes Assembly	35	EZ-12 Seat Tube	2	990450
12	EZ12 Seat Back Tubes - pair	42	EZ-12 Fabric Tube Assembly	2	990469
12	EZ12 Seat Support w/Velcro	38	EZ-12 Seat Support w/Velcro	1	990496
14	EZ14 Front Assembly - Std	1	EZ-14 Front frame assy	1	990501
14	EZ14 Rear Assembly - Std	2	EZ-14 Rear Frame Assembly	1	990503
14		12	EZ-14 Rear Upper Leg Welded	1	
14	EZ14 Rear Lock Brace Assembly	29	EZ-14 Rear Lock Assembly	1	990510
14	EZ14 Seat Tube Assembly	35	EZ-14 Seat Tube	2	990550
14	EZ14 Fabric Tube Outer Welded - LH	42	EZ-14 Back Tube Welded LH	1	990565
14	EZ14 Fabric Tube Outer Welded - RH	49	EZ-14 Back Tube Welded RH	1	990566
14	EZ14 One Piece Push Handle	53	EZ-14 Single Pc Handle Assembly	1	990592
14		54	Screw #10-24 X 1-1/4" Phil Pan	2	

14		55	Locknut #10-24 Jam Flexlock Black Zinc	2	
14	EZ14 Seat Support w/Velcro	38	EZ-14 Seat Support w/Velcro	1	990596
16	EZ16 Front Assembly - Std	1	EZ-16 Front Frame Assy	1	990601
16	EZ16 Rear Assembly - Std	2	EZ-16 Rear Frame Assembly	1	990603
16	EZ16 Rear Lock Brace Assembly	29	EZ-16 Rear Lock Assembly	0	990610
16	EZ16 Upper Rear Leg Assy - Std LH	12	EZ-16 Rear Leg Upper Welded	1	990651
16		39	Locknut 1/4"-28 Flexlock Black Zinc	1	
16	EZ16 Fabric Tube Outer Welded - LH	42	EZ-16 Fabric Tube Outer Welded LH	1	990665
16	EZ16 Fabric Tube Outer Welded - RH	49	EZ-16 Fabric Tube Outer Welded RH	1	990666
16	EZ16 One Piece Push Handle	53	EZ-16 Single Pc Handle Assembly	1	990692
16		54	Screw #10-24 X 1-1/4" Phil Pan	2	
16		55	Locknut #10-24 Jam Flexlock Black Zinc	2	
16	EZ16 Seat Support w/Velcro	38	EZ-16 Seat Support w/Velcro	1	990696
12, 16	VV Seat Extension Tube - pair	4	VV-12 Seat Tube Extension Assy	1	994023
12, 14, 16	7.5" x 2" Front Solid Tire on Grey Rim (Ea)	28	Front Wheel w/3-Spokes 182.5x51mm Rolk	1	994749
12, 14, 16		30	Bolt M8-1.25mm x 90mm Socket Cap Scre	1	
12, 14, 16		31	Locknut M8-1.25 Nylon Insert	1	
12, 14, 16		50	Washer 5/16" X 9/16"	6	
12, 14, 16	Caster Fork Assembly 7.5" X 2" (ROLKO)	25	Balloon Caster Fork Assembly	1	996009
12, 14, 17		26	"0" Ring Buna-Nitrile	1	
14	EZ Closure Strap Replacement	11	Locknut 1/4"-28 Jam Flexlock Black Zinc	1	997315
14, 16		56	Footplate Strap	1	

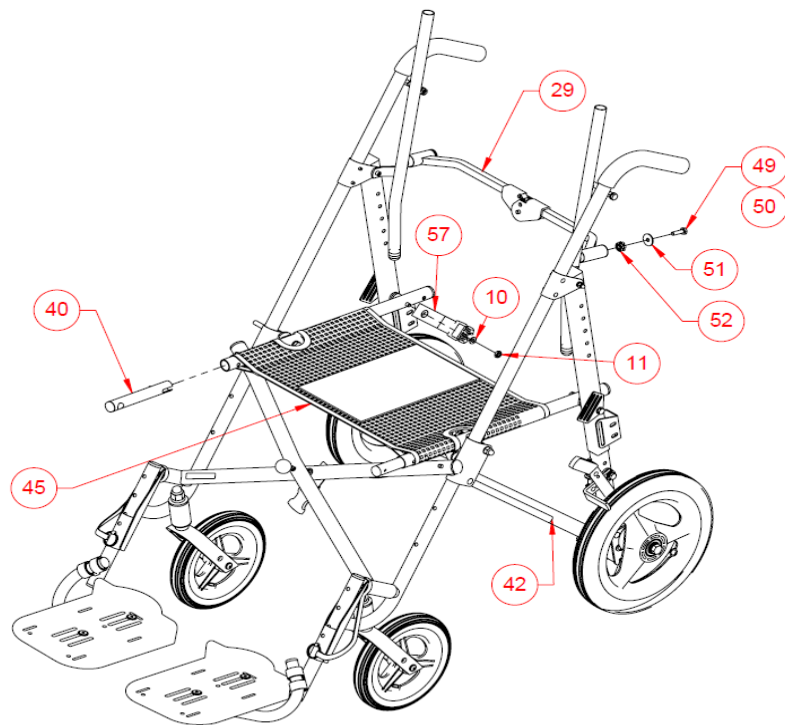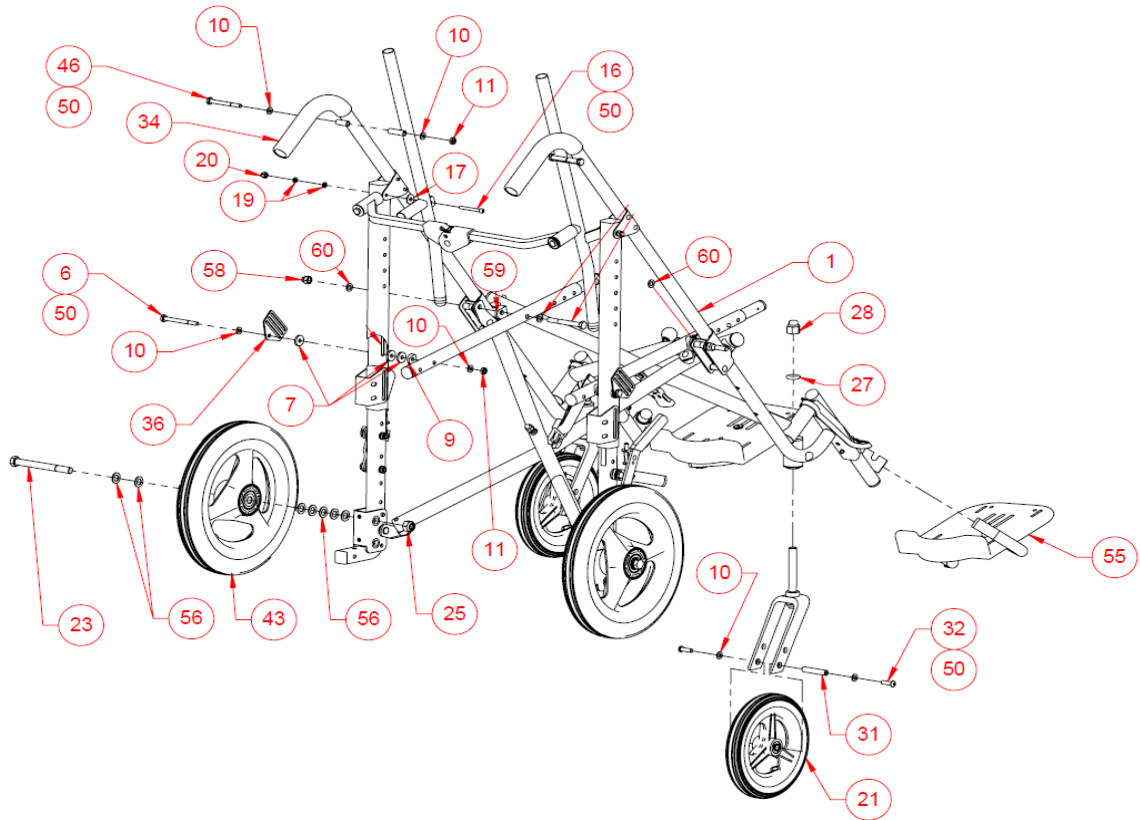

Page 4: Rear Frame Assembly, Size
18 Only

SIZE	KIT NAME	ITEM NO.	DESCRIPTION	QTY2	KIT
18	EZ18 Front Assembly - Std	1	EZ-18 Front Frame Assembly	1	990701
18		33	Handle Grip- 3/4"ID X 7"	2	
18		34	3M Rubber & Gasket Adhesive 5oz (847)	1	
18	EZ18 Rear Assembly - Std	2	EZ-18 Rear Frame Assy.	1	990703
18	EZ18 Rear Lock Brace Assembly	29	EZ-18 Rear Lock Brace Assy	1	990710
18	EZ18 Seat Tubes Assembly	37	EZ-18 Seat Tube	1	990750
18	EZ18 Fabric Tube Outer Welded - LH	47	EZ-18 Back Tube Welded LH	1	990765
18	EZ18 Fabric Tube Outer Welded - RH	48	EZ-18 Back Tube Welded RH	1	990766
18	EZ18 Seat Support w/Velcro	45	EZ-18 Seat Support w/Velcro	1	990796
18	Wheel Assembly with 3/8" Bearing - 8" Transit (Ea)	21	8" Urethane Wheel & Bearing Assy 3/8"	1	992646
18		31	Threaded Barrel	1	
18		32	Bolt 1/4"-28 x 7/8" Socket Button Head - blk z	2	
18		N/A	Thread Locker-Medium-Vibra-tite	1	
18	Footplate Assembly Adult - LH	54	Footrest Assy, Adult, LH	1	992662
18	Footplate Assembly Adult - RH	55	Footrest Assy, Adult, RH	1	992663
18	CM18 Upper Rear Leg Assembly LH	12	EZ-18 Rear Leg Upper Welded	1	994233
18	8" Rear Wheel Assy w/3/8" Bearing	21	8" Urethane Wheel & Bearing Assy 3/8"	1	994745
18	CX10, CX/CC/CL/CS12, EZ/CV18 Seat Ext Assembly	40	CX-10/12 Seat Extension Tube Assy Short	1	994777
18	Caster Fork Assembly 8" x 2"	26	Caster Stem Assembly	1	996309
18		27	"0" Ring Buna-Nitrile	1	
18		28	Locknut 1/2"-20 Capped Nylon - blk zinc 098	1	
18	EZ Closure Strap Replacement	11	Locknut 1/4"-28 Jam Flexlock Black Zinc	1	997315
18		57	Footplate Strap	1	

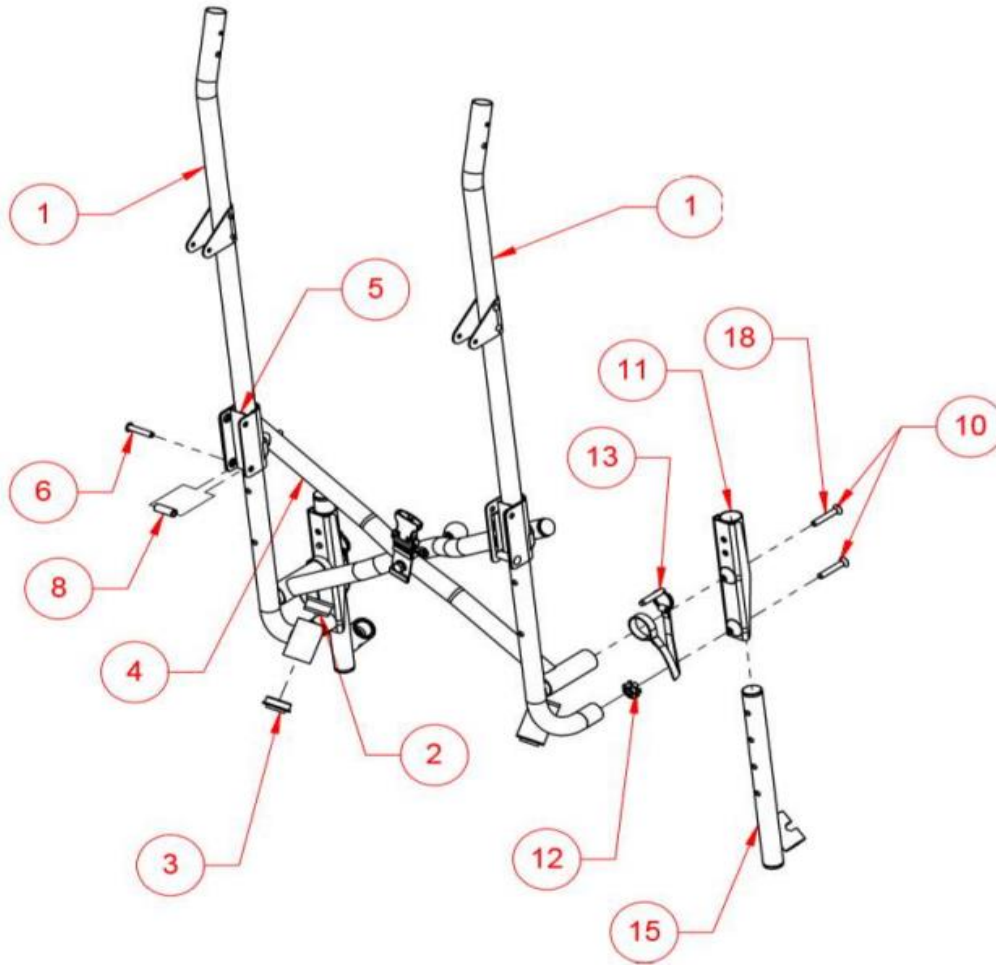


Page 5: Front Frame Assembly

SIZE	KIT NAME	ITEM NO.	DESCRIPTION	QTY2	KIT
12	EZ12 Front X Brace Assembly	4	EZ-12 Front "X" Brace Assembly	1	990408
14	EZ14 Front X Brace Assembly	4	EZ-14 Front X brace assembly	1	990508
16	EZ16 Front X Brace Assembly	4	EZ-16 Front X-Brace Assembly	1	990608
18	EZ18 Front X Brace Assembly	4	EZ-18 Front X Brace Assembly	1	990708
18	EZ18 Push Handle Tube Assembly - Std	1	EZ-18 Handle Tube Assy, LH	1	990711
18		1	EZ-18 Handle Tube Assy, RH	1	
18		2	Caster Stem Bushing	2	
18		3	Flanged Bearing 1/2" shielded Greased	2	
All Sizes		11	Tube Connector 7/8" Heavy Duty	Varies per Kit	
18		-	Bolt 1/4"-28 x 1-1/2" Socket Button Head- b	2	
18		-	Locknut 1/4"-28 Capped Nylon 098	2	
12		EZ/CV12 Front Leg Tube Assy (1pc	1	EZ-12 Front Leg Assy for 1-Pc Handle, LH	
12	1		EZ-12 Front Leg Assy for 1-Pc Handle, RH	1	

12	Handle) - Std	2	Caster Stem Bushing	2	
12		3	Flanged Bearing 1/2" shielded Greased	2	
14	EZ/CV14 Front Leg Tube Assy (1pc Handle) - Std	1	EZ-14 Front Leg Assy for 1-Pc Handle, LH	1	992370
14		1	EZ-14 Front Leg Assy for 1-Pc Handle, RH	1	
14		2	Caster Stem Bushing	2	
14		3	Flanged Bearing 1/2" shielded Greased	2	
16	EZ/CV16 Front Leg Tube Assy (1pc Handle) - Std	1	EZ-16 Front Leg Welded Assy for 1-Pc Hand, LH	1	992470
16		1	EZ-16 Front Leg Welded Assy for 1-Pc Hand, RH	1	
16		2	Caster Stem Bushing	2	
16		3	Flanged Bearing 1/2" shielded Greased	2	
16		-	Pivot Bracket 7/8"-7/8"	2	
16		-	Rivet 3/16" X 1-1/8" S/T	4	
All Sizes	Detent Pin W/ Webbing Assembly	12	Detent Pin With Webbing Assembly	1	995605
All Sizes		11	Tube Connector 7/8" Heavy Duty	1	
All Sizes	Standard Footplate Assembly Kit LH	13	Footplate Adjustment Tube LH	1	996110
All Sizes	Standard Footplate Assembly Kit RH	14	Footplate Adjustment Tube RH	1	996111
12	Included in 992270 and 990408 Kits	-	Bolt 1/4"-28 x 1-1/2" Socket Button Head- b	2	Included in 992270 and
12		-	Locknut 1/4"-28 Capped Nylon 098	2	
All Sizes	Included in 992270, 996110, and 996111 Kits	11	Tube Connector 7/8" Heavy Duty	Varies per Kit	Included in 992270, 996110, and 996111 Kits
14	Included in 992370 and 990508 Kits	N/A	Bolt 1/4"-28 x 1-1/2" Socket Button Head- b	2	Included in 992370 and
14		N/A	Locknut 1/4"-28 Capped Nylon 098	2	
All Sizes	Included in 992370, 996110, and 996111 Kits	11	Tube Connector 7/8" Heavy Duty	Varies per Kit	Included in 992370, 996110, and 996111 Kits
16	Included in 992470 and 990608 Kits	-	Bolt 1/4"-28 x 1-1/2" Socket Button Head- b	2	Included in 992470 and
16		-	Locknut 1/4"-28 Capped Nylon 098	2	
All Sizes	Included in 992470, 996110, and 996111 Kits	11	Tube Connector 7/8" Heavy Duty	Varies per Kit	Included in 992470, 996110, and 996111 Kits
All Sizes	Included in 996110 and 996111	10	Footrest Extens. Housing Assy.	1	Included in

All Sizes	Kits	12	Detent Pin With Webbing Assembly	1	996110 and
-----------	------	----	----------------------------------	---	------------

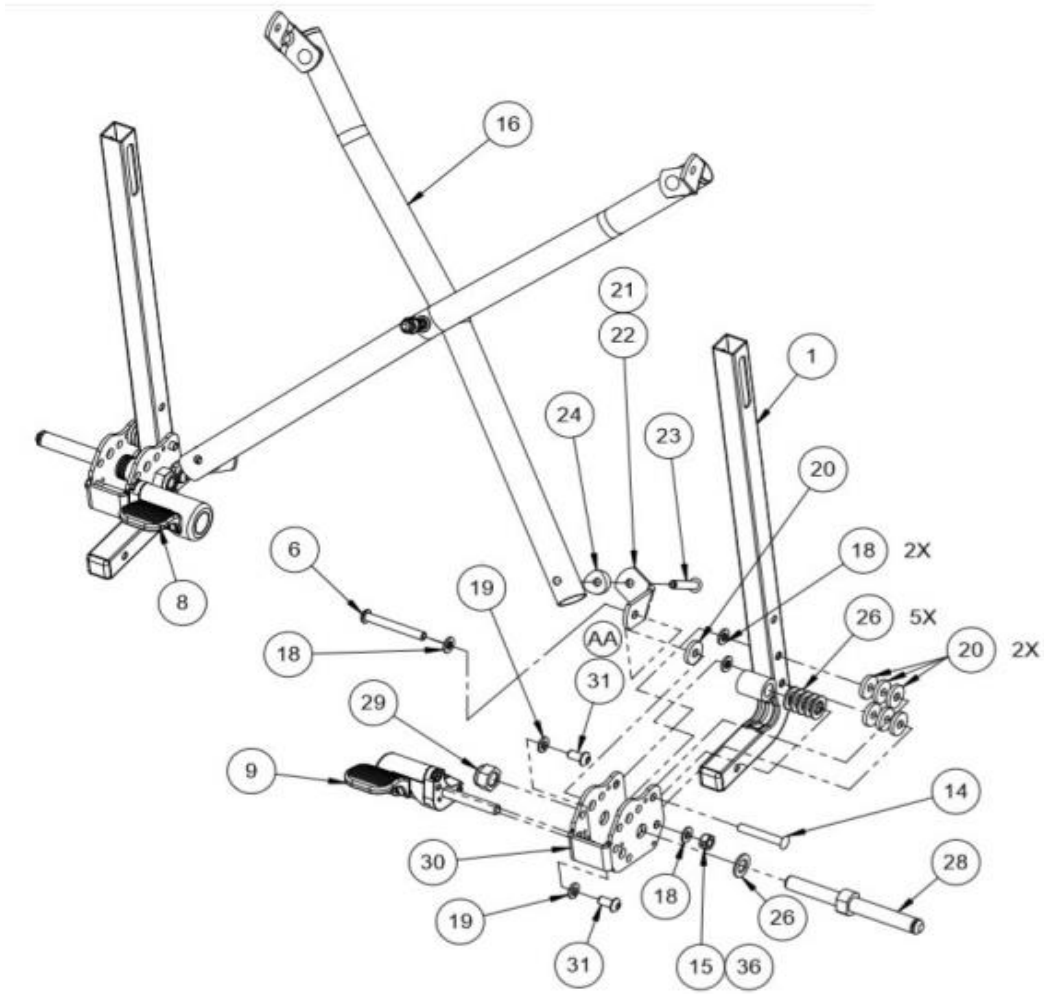

Page 6: Rear Frame Assembly

SIZE	KIT NAME	ITEM NO.	DESCRIPTION	QTY2	KIT
12	EZ12 Rear X Brace Assembly	1	EZ-12 Rear "X" Brace Assembly	1	990409
14	EZ14 Rear X Brace Assembly	1	EZ-14 Rear X Brace Assembly	1	990509
16	EZ16 Rear X Brace Assembly	1	EZ-16 Rear X-Brace Assembly	1	990609
18	EZ18 Rear X Brace Assembly	1	EZ-18 Rear X Brace Assembly	1	990709
18		9	Washer 1/4" AN-960-416	2	
12, 14, 16		6	R82 Brake Assembly LH	1	
12, 14, 16		7	R82 Brake Assembly RH	1	
12, 14, 16		9	Washer 1/4" AN-960-416	4	
12, 14, 16		19	Locknut 1/4"-28 Flexlock Black Zinc	4	
12, 14, 16		23	Coved Washer - 3/4"	4	
12, 14, 16					

12, 14, 16	R82 Brake Replacement Kit (CX10-12 ME12)	24	Washer 7/16" AN-960-716	12	997310
12, 14, 16		27	Rear Axle	2	
12, 14, 16		28	Locknut 7/16"-20 Nylon Insert Black Zinc	2	
12, 14, 16		29	R82 Brake Bracket	2	
12, 14, 16		30	Bolt 1/4"-20 X 1/2" Socket Button Head blak	4	
12, 14, 16		37	Split Lock Washer 1/4" ID	4	