

gazelle ps

Exploded views / Part lists


Rev. date (11.2017)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.



Contents

Gazelle PS size 1+2+3.....	3
Basis plate and back support.....	6
Accessories	9

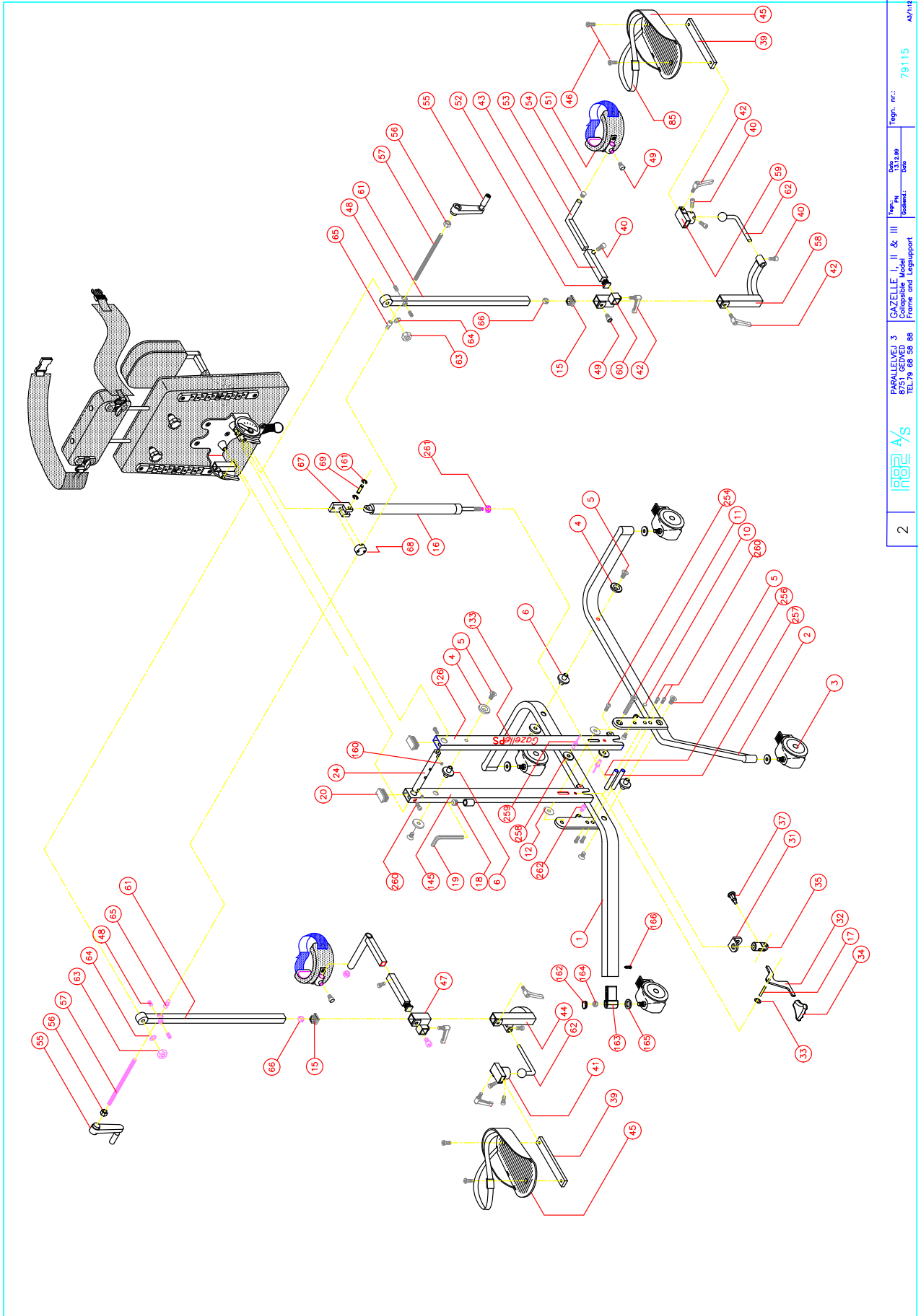


2		PARALLELVEJ 3 8751 GEDVED TEL: 79 68 58 88	GAZELLE I, II & III Collapsible Model Frame and Legsupport	Part No. 79115	Engl. nr.: 79115	Rev. No. Date
---	---	--	--	-------------------	---------------------	------------------

Position	No.	Description	Unit of Measure	Quantity	Ean code
1*	82246	FRAME RIGHT GAZELLE I	PIECES	1	5707292122834
	82247	FRAME RIGHT, GAZELLE II	PIECES	1	5707292122841
	82248	FRAME RIGHT GAZELLE III	PIECES	1	5707292122858
2*	82243	FRAME LEFT GAZELLE I	PIECES	1	5707292122803
	82244	FRAME LEFT, GAZELLE II	PIECES	1	5707292122810
	82245	FRAME LEFT, GAZELLE III	PIECES	1	5707292122827
3	3006825	75MM WHEEL W/BRAKE, BLACK	PIECES	4	5707292371393
4*	61150-38	COUNTER SCREW, GAZELLE PS 1	PIECES	4	5707292220646
	61150-36	COUNTER SCREW, GAZELLE PS 2	PIECES	4	5707292220622
	61150-37	COUNTER SCREW, GAZELLE PS 3	PIECES	4	5707292220639
5	0612800	INSEX SCREW COUNTERS. 10*25	PIECES	6	5707292138774
6*	61182-38	COLOURED WASHER, GAZELLE PS 1	PIECES	3	5707292220707
	61182-36	COLOURED WASHER, GAZELLE PS 2	PIECES	3	5707292220684
	61182-37	COLOURED WASHER, GAZELLE PS 3	PIECES	3	5707292220691
10	1310550	COVERING STUD, BLACK, GAZELLE	PIECES	1	5707292140722
11	0631807	COUNTER SCREW M10*65, FZB	PIECES	1	5707292139160
12	6108198	PL. WASHER WHITE, 9*18*1,2MM	PIECES	2	5707292185648
13	82175	BALL AND JOCKET JOINT, FOOTREST	PIECES	2	5707292122247
16*	2205000	BLOC-O-LIFT 3728IF 350N	PIECES	1	5707292143334
	2204250	BLOC-O-LIFT 5323HT 400N SOLID	PIECES	1	5707292143327
17	0529060	CYLINDER PIN Ø5,0X32MM, DIN6325	PIECES	1	5707292137517
18	2501710	FERRULE D16 M/6.2, HOLE	PIECES	1	5707292143983
19	0589600	INSEX SCREW KEY N6	PIECES	1	5707292137876
20	2507510	FERRULE FR40, W/CUT	PIECES	2	5707292144270
24	6117400	AXLE Ø 20X150, GAZELLE	PIECES	1	5707292186492
31	82283	ANGLE MOUNTING, SPRINGLOCK	PIECES	1	5707292123138
32	82281	RELEASE ARM, GAZELLE PS 1	PIECES	1	5707292123114
33	82627	BUSH FOR RELEASE ARM	PIECES	1	5707292369246
34	2200035	LARGE BUTTON FOR RELEASE ARM	PIECES	1	5707292143068
35	6118860	RELEASE FOR GASSPRING, GAZELLE	PIECES	1	5707292186751
37	0380181	HOUSE FOR LOCK MECHANISM M10X1	PIECES	1	5707292136039
39	82230	ROD F. "DUCKFOOT", GAZELLE 1	PIECES	2	5707292122742
40	0602900	INSEX SCREW 8*16	PIECES	6	5707292138200
41	82139-36	JOINT FOR FOOTREST, GAZ. PS 1	PIECES	1	5707292230386
42	3502516	TIGHTENING HANDLE 45MM M8*16	PIECES	4	5707292146915
43	82223	EXTENSION ROD FOR KNEESUPPORT	PIECES	2	5707292122674
44	82164	HOLDER FOR 'DUCK'FOOT 1, RIGHT	PIECES	1	5707292122131
45	82156-60	FOOTPLATE SIZE 1, BLACK, GAZELLE	PIECES	2	5707292230492
46	0612500	INSEX SCREW COUNTERS. 6*25	PIECES	4	5707292138712
47	82216-36	HOLDER FOR KNEESUPP. 1, RIGHT	PIECES	1	5707292230645
48	0723060	STOP SCREW WITH STUD 6*10	PIECES	4	5707292139955
49	0602500	INSEX SCREW 8*12	PIECES	4	5707292138163
51	5039060	PUR KNEESUPPORT SIZE 1, W/STRAP	PIECES	2	5707292159946
52*	2504000	FERRULE E15*1,0-2,0 INT. BLACK	PIECES	3	5707292144102
	2504500	FERRULE E20*1,0-2,0 INT. BLACK	PIECES	2	5707292144126
53	82218	ADJUSTABLE FOR GAZELLE KNEESUP	PIECES	2	5707292122629
54	2501600	FERRULE D15*1,0 BLACK	PIECES	4	5707292143969
55	3515000	SWING HANDLE M10 W. TILT HANDE	PIECES	2	5707292147134
56	1604000	STEEL NUT 10 MM	PIECES	2	5707292141293

57*	1990100	STUD BOLT M10*145 W/STOP, FZB	PIECES	2	5707292358332
	1990200	STUD BOLT M10*165 W/STOP, FZB	PIECES	2	5707292358363
	1990300	STUD BOLT M10*188 W/STOP, FZB	PIECES	2	5707292358370
58	82100	HOLDER FOR 'DUCK'FOOT 1, LEFT	PIECES	1	5707292121561
59	82140-36	TURNING LINK,FOOTSUP. GAZ. 1	PIECES	1	5707292230447
60	82217-36	HOLDER FOR KNEESUPP. 1, LEFT	PIECES	1	5707292230676
61*	82121	BAR F. FOOT SUPPORT, GAZELLE 1	PIECES	2	5707292121769
	82122	ROD FOR FOOT SUPPORT,GAZELLE 2	PIECES	2	5707292121776
	82123	ROD FOR FOOT SUPPORT, GAZELLE 3	PIECES	2	5707292121783
62	82179	ARM FOR JOINT	PIECES	2	5707292122285
63	1607500	LOCK NUT 10 MM	PIECES	2	5707292141385
64	1102500	PLANNING WASHER 10 MM	PIECES	2	5707292140234
65	6117800	BALL TAP, GAZELLE PS	PIECES	2	5707292186539
66	82125	M10 NUT F. FOOT SUPP., GAZELLE	PIECES	2	5707292121806
67	I-6117200	FITTING, GAS SPRING GAZELLE	PIECES	1	5707292214959
68	6117700	FITTING F. FOOT BRACE, GAZEL.	PIECES	1	5707292186522
69	80927	8*30, LOCKING PIN FOR GAZELLE	PIECES	1	5707292376985
85	5515150	GIRTH FOOT-STRAP W.FIXL.F.TO.C	PIECES	2	5707292177810
126*	82238-36	PILLAR, LEFT, GAZELLE PS 1	PIECES	1	5707292230744
	82240-37	PILLAR, LEFT, GAZELLE PS 2	PIECES	1	5707292230782
	82289-38	PILLAR, LEFT, GAZELLE PS 3	PIECES	1	5707292231147
133	0493010	Gazelle logo 10*129mm, silver	PIECES	2	5707292396662
145*	82237-36	PILLAR, RIGHT, GAZELLE PS 1	PIECES	1	5707292230720
	82239-37	PILLAR, RIGHT, GAZELLE PS 2	PIECES	1	5707292230768
	82284-38	PILLAR, RIGHT, GAZELLE PS 3	PIECES	1	5707292231093
160	0723050	STOP SCREW WITH POINT 6*5	PIECES	2	5707292139948
161	1145070	LOCK WASHER 7MM	PIECES	2	5707292140586
162	82356	PLUG FOR HOUSING, GAZELLE	PIECES	4	5707292359209
163	82355	HOUSING FOR WHEEL, GAZELLE	PIECES	4	5707292363664
164	1607500	LOCK NUT 10 MM	PIECES	4	5707292141385
165	1103400	FAZET WASHER 10 MM	PIECES	4	5707292140319
166	1940030	PLASTITE 45 Z PHX 5*20	PIECES	4	5707292142610
254	0662500	BUTTON HEAD SCREW 8*25 STAINL.	PIECES	1	5707292139580
256	6109365	STEER SLEEVE FOR GAZELLE Ø10	PIECES	1	5707292186102
257	6109366	STYRE BØSNING F/GAZELLE Ø12	PIECES	1	5707292186119
258	6109367	WASHER FOR GAZELLE	PIECES	1	5707292186126
259	0506508	RIVET 5*14, STAINLESS	PIECES	3	5707292359476
260	0641600	BUTTON HEAD SCREW 6*16	PIECES	6	5707292139269
261	1761000	COUNTER NUT 10*1,0 FINE THREAD	PIECES	1	5707292141538
262	0602188	INSEX M6*8, FZB	PIECES	2	5707292138071

*		CHOOSE SIZE
---	--	-------------

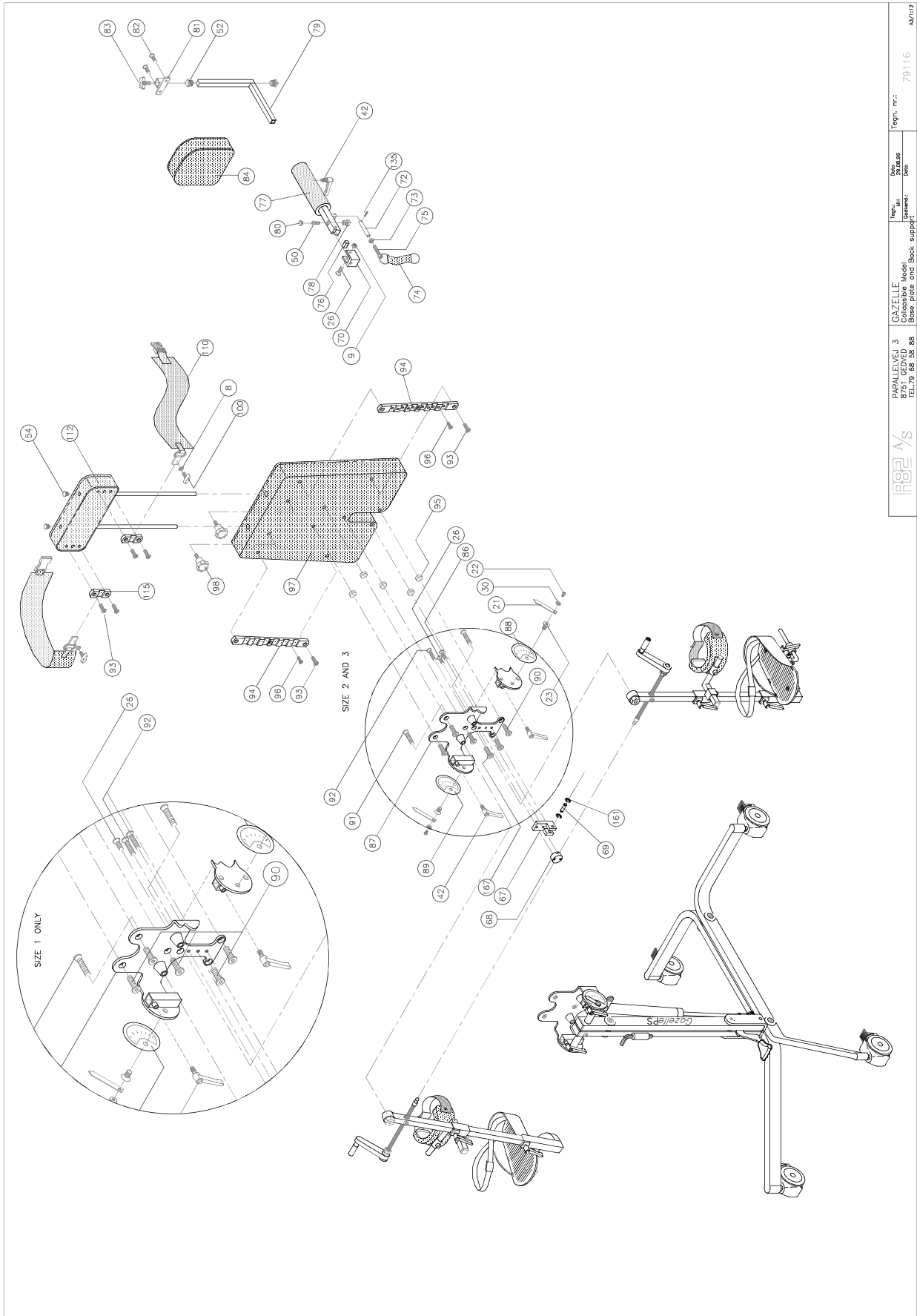


	PÅRALLELVEJ 3 8751 GEDVED TEL: 79 68 58 88	GAZELLE Collapsible Model Base Plate and Back support	Type: JYBA Date: 08.05.05 Issue: 01	Egen. nr.: 79116 04/01/03
---	--	---	---	------------------------------

Position	No.	Description	Quantity	Unit of Measure	Ean code
8	1103000	FAZET WASHER 6MM	2	PCS	5707292140289
9	1606500	LOCK NUT 6 MM	2	PCS	5707292141361
21	6153200	BLACK ARROW F. GAZELLE	2	PCS	5707292188564
22	1902600	CROSS-HEADED SCREW 4*10MM	2	PCS	5707292142108
23	0617000	SPEC. INSEX SCREW COUNTERS.	2	PCS	5707292138910
26	0623200	INSEX SCREW COUNTERS. 6*30	1	PCS	5707292138996
30	1102800	FAZET WASHER 4MM	2	PCS	5707292140265
42	3502516	TIGHTENING HANDLE 45MM M8*16	2	PCS	5707292146915
50	0645630	BUTTON HEAD SCREW 6*30 BLACK	1	PCS	5707292139320
52	2504000	FERRULE E15*1,0-2,0 INT.BLACK	3	PCS	5707292144102
54	2501600	FERRULE D15*1,0 BLACK	2	PCS	5707292143969
67	I-6117200	FITTING, GAS SPRING GAZELLE	1	PCS	5707292214959
68	6117700	FITTING F. FOOT BRACE, GAZEL.	1	PCS	5707292186522
69	80927	8*30, LOCKING PIN FOR GAZELLE	1	PCS	5707292376985
70*	6117100	FITTING F. ABDUCTION,GAZELLE 1	1	PCS	5707292186478
	6117300	FITTING F. ABDUCTION,GAZ 2+3	1	PCS	5707292186485
71	6117500	TILT JOINT. F. LOCK BAR, GAZE.	1	PCS	5707292186508
72	6117600	AXLE Ø10*61, GAZELLE	1	PCS	5707292186515
73	1603000	STEEL NUT 6 MM	1	PCS	5707292141279
74	5001051	GIRTHBAND FOR BUTTOM RELEASE	1	PCS	5707292154422
75	0618000	SPEC. INSEX SCREW 6*30	1	PCS	5707292138927
76	I-82127	STOP FOR TILT	1	PCS	5707292216885
77	82145	SEAT ROD WITH PUR	1	PCS	5707292121998
78	2521000	PIPE HOLDER Ø 10 MM, WHITE	1	PCS	5707292144362
79*	82143	HOLDER F. "BUTTOM-SUP."GAZEL.1	1	PCS	5707292121974
	82144	HOLDER F."BOTTOM-SUP." GAZ.2+3	1	PCS	5707292121981
	82144	HOLDER F."BOTTOM-SUP." GAZ.2+3	1	PCS	5707292121981
80	2502518	TUBE LEG BLACK 16MM	1	PCS	5707292144034
81	82163	"BUTTOM-MOUNTING" GAZELLE	1	PCS	5707292122124
82	0621600	INSEX SCREW COUNTERS. 6*16	2	PCS	5707292138965
83	3655000	WING SCREW 32*17 M8*15MM BLACK	1	PCS	5707292147189
84*	6175031	BACKSUPPORT PUR GAZELLE SZ. 1	1	PCS	5707292188960
	6175032	BACK SUPPORT SIZE 2&3	1	PCS	5707292188977
	6175032	BACK SUPPORT SIZE 2&3	1	PCS	5707292188977
86	0623500	INSEX SCREW COUNTERS. 6*35	1	PCS	5707292139009
87	82118	TILT FITTING, GAZELLE	1	PCS	5707292121738
88	82149	DEGREE-INDICATOR LEFT GAZELLE	1	PCS	5707292122032
89	82150	DEGREE-INDICATOR RIGHT GAZELLE	1	PCS	5707292122049
90	0624825	INSEX SCREW COUNTERS. 8*25	5	PCS	5707292139078
91	0614500	INSEX SCREW COUNTERS. 10*45	2	PCS	5707292138828
92*	0623500	INSEX SCREW COUNTERS. 6*35	2	PCS	5707292139009
	0624500	INSEW SCREW COUNTERS. 6*45	1	PCS	5707292139023
93	0611000	INSEX SCREW COUNTERS. 6*20	4	PCS	5707292138682
94	82152	RAIL F. BASE PLATE GAZELLE	2	PCS	5707292122063
95	1350010	DISTANCEPART BLACK 8*15*8,2MM	5	PCS	5707292140883
96	1825000	PL.SCREW COUNTERS. 4*20MM	8	PCS	5707292141866
97*	6175011	MAIN CHEST SUPPORT SIZE 1	1	PCS	5707292188915
	6175012	MAIN CHEST SUPPORT SIZE 2	1	PCS	5707292188922
	6175013	MAIN CHEST SUPPORT SIZE 3	1	PCS	5707292188939

98	3501301	R82 handle M6*25mm Screw	2	PCS	5707292413956
100	3652501	R82 HANDLE M6*16MM SCREW	2	PCS	5707292413963
110*	5540010	CHESTGIRTH F. GAZELLE SIZE 1	1	SET	5707292178145
	5540020	CHESTGIRTH F.GAZELLE SIZE 2	1	SET	5707292178152
	5540030	CHESTGIRTH F. GAZELLE SIZE 3	1	SET	5707292178169
112*	6175051	PUR CHESTPLATE SIZE 1,GAZELLE	1	PCS	5707292189011
	6175052	PUR CHESTPLATE SIZE 2,GAZELLE	1	PCS	5707292189028
	6175053	PUR CHESTPLATE SIZE 3,GAZELLE	1	PCS	5707292189035
115	82153	RAIL F.DISTANCE-&CHINSUP.GAZEL	2	PCS	5707292122070
135	0541600	TIGHTENING SPRIG 4*16	1	PCS	5707292137616
161	1145070	LOCK WASHER 7MM	2	PCS	5707292140586
167	0621600	INSEX SCREW COUNTERS. 6*16	2	PCS	5707292138965

*		CHOOSE SIZE			
---	--	-------------	--	--	--

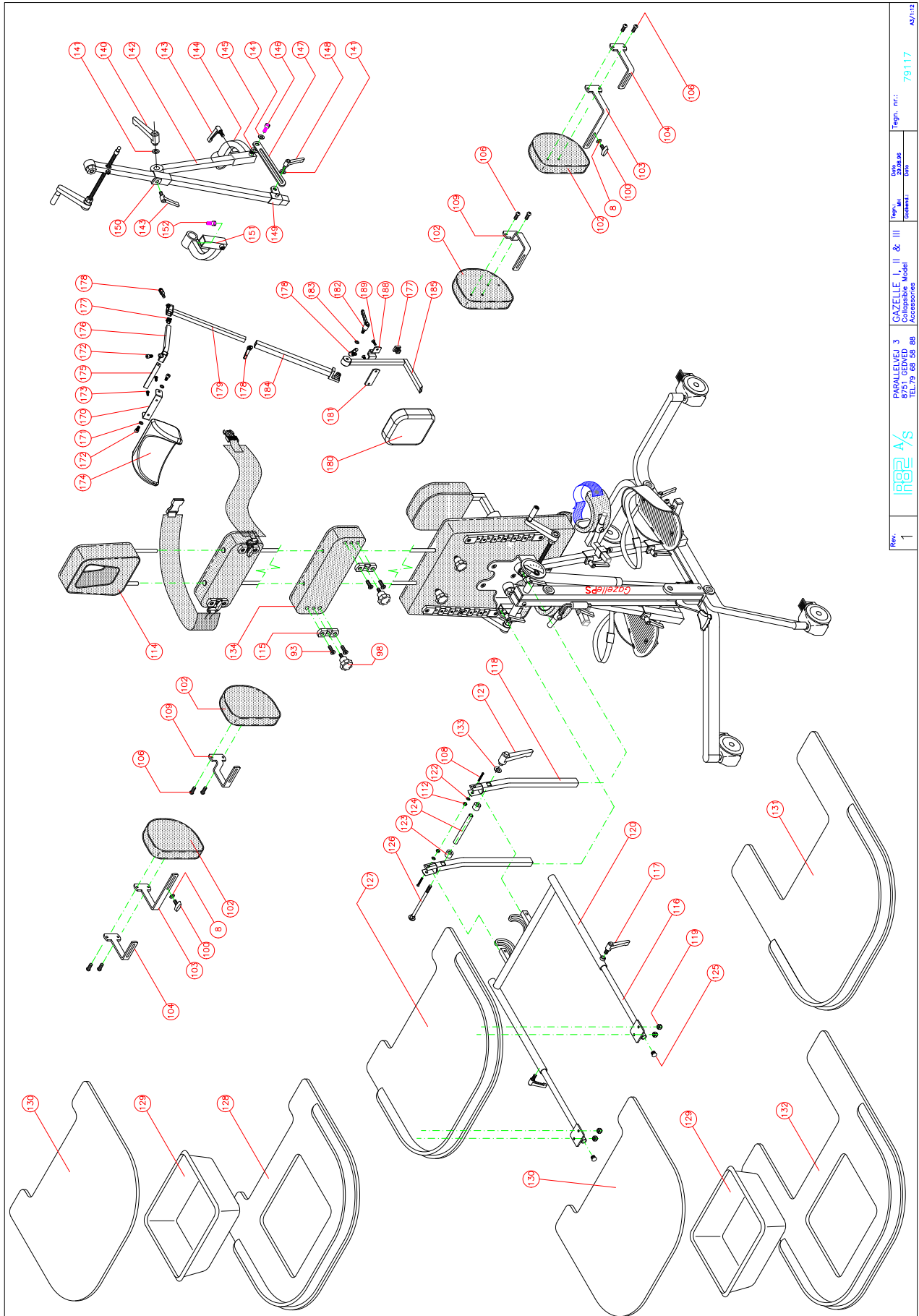

Rev.	1	Part No.	GAZELLE I, II & III Collapsible Model Accessories	Fig. nr.	79117
Part No.	PARALLELVEJ 3 8751 GEDVED Model TEL:79 85 58 88	Part No.		Fig. nr.	

Position	No.	Description	Quantity	Unit of Measure	Ean code
8	1106000	WASHER 6MM	2	PCS	5707292140371
93	0611000	INSEX SCREW COUNTERS. 6*20	4	PCS	5707292138682
98	3501401	R82 handle M6*20mm Screw	2	PCS	5707292413901
100	3652501	R82 HANDLE M6*16MM SCREW	2	PCS	5707292413963
102*	6175541	Pur side supports size 1	2	PCS	5707292494207
	6175542	Pur side supports size 2	1	PCS	5707292494214
103	82147	EXTERNAL SIDESUP.MOUNT.GAZELLE	2	PCS	5707292122018
104	82151	STD.SIDESUPP.MOUNTING, GAZELLE	2	PCS	5707292122056
106	0641600	BUTTON HEAD SCREW 6*16	4	PCS	5707292139269
108	0640930	BUTTON HEAD M5*30 FZB	2	PCS	5707292139221
109	82148	PELOTMOUNTING FOR EXTRA DEPTH	2	PCS	5707292122025
112	1606300	LOCK NUT 5 MM	2	PCS	5707292141354
114*	6175561	PUR HEADSUPPORT 1, GAZELLE	1	PCS	5707292494269
	6175562	PUR HEADSUPPORT 2, GAZELLE	1	PCS	5707292494276
	6175563	PUR HEADSUPPORT 3, GAZELLE	1	PCS	5707292494283
115	82153	RAIL F.DISTANCE-&CHINSUP.GAZEL	2	PCS	5707292122070
116	82099	MOUNTING FOR TRAY, GAZELLE	2	PCS	5707292121554
117	3502516	TIGHTENING HANDLE 45MM M8*16	2	PCS	5707292146915
118*	82168	ROD FOR TABLE GAZELLE SIZE 1	2	PCS	5707292122179
	82169	ROD FOR TABLE GAZELLE SIZE 2	2	PCS	5707292122186
	82170	ROD FOR TABLE GAZELLE SIZE 3	2	PCS	5707292122193
119	1780600	TOP NUT 6MM	4	PCS	5707292141552
120	82308	FRAME FOR TRAY, GAZELLE	1	PCS	5707292123282
121	3502300	TIGHTENING HANDLE 65MM M8 BUSH	1	PCS	5707292146885
122	1102900	FAZET WASHER 5 MM	4	PCS	5707292140272
123	82311	DISTANCE PIECE, GAZELLE TRAY	1	PCS	5707292123312
124	82309	BUSH FOR TRAY	2	PCS	5707292123299
125	2501600	FERRULE D15*1,0 BLACK	4	PCS	5707292143969
126	1905740	BOLT M8*150 FZB	1	PCS	5707292142375
127*	81468	TRAY WITHOUT HOLE, GAZELLE 1	1	PCS	5707292115621
	81511	TRAY WITHOUT HOLE, GAZELLE 2	1	PCS	5707292116055
	81512	TRAY WITHOUT HOLE, GAZELLE 3	1	PCS	5707292116062
128*	81575	TRAY FOR GAZELLE SIZE 1,W/HOLE	1	PCS	5707292116673
	81576	TRAY FOR GAZELLE SIZE 2,W/HOLE	1	PCS	5707292116680
	81577	TRAY FOR GAZELLE SIZE 3,W/HOLE	1	PCS	5707292116697
129	81397	BOWL FOR GAZELLE TRAY 1-3	1	PCS	5707292115003
130*	8137670	PLATE FOR TRAY W/HOLE SIZE 1	1	PCS	5707292194299
	8137680	PLATE FOR TRAY W/HOLE SIZE 2	1	PCS	5707292194305
	8137690	PLATE FOR TRAY W/HOLE SIZE 3	1	PCS	5707292194312
131*	81469	TABLEPLATE EXT +20CM GAZELLE 1	1	PCS	5707292115638
	81514	TABLEPLATE EXT +20CM GAZELLE 2	1	PCS	5707292116086
	81515	TABLEPLATE EXT +20CM GAZELLE 3	1	PCS	5707292116093
132*	81563	TRAY W/SQUARE HOLE F. BOWL 1	1	PCS	5707292116574
	81564	TRAY W/SQUARE HOLE F. BOWL 2	1	PCS	5707292116581
	81565	TRAY W/SQUARE HOLE F. BOWL 3	1	PCS	5707292116598
133	1102010	FACE PLATE 8,4*18*2MM, FZB	1	PCS	5707292429766
134*	6175571	PUR CHEST SUPPORT SIZE 1,GAZ	1	PCS	5707292494290
	6175572	PUR CHEST SUPPORT SIZE 2,GAZ	1	PCS	5707292494306
	6175573	PUR CHEST SUPPORT SIZE 3,GAZ	1	PCS	5707292494313

140	3504000	TIGHTENING HANDLE 42MM M8*20	2	PCS	5707292147097
141	1102000	PLANING WASHER 8MM	8	PCS	5707292140227
142	81426	CONE ON ROD F.TABLE-&FOOTPLATE	2	PCS	5707292115294
143	3502516	TIGHTENING HANDLE 45MM M8*16	4	PCS	5707292146915
144*	82207	ADJUSTABLE HOLDER FOR FOOT, 1	1	PCS	5707292122537
	82209	ADJUSTABLE HOLDER FOR FOOT, 2	1	PCS	5707292122551
	82214	ADJUSTABLE HOLDER FOR FOOT, 3	1	PCS	5707292122605
146	0602900	INSEX SCREW 8*16	2	PCS	5707292138200
147	82206	LOCK SLIDE	2	PCS	5707292122520
148*	3502600	TIGHTENING HANDLE 65MM M8*16	2	PCS	5707292146922
	3504000	TIGHTENING HANDLE 42MM M8*20	2	PCS	5707292147097
149*	82205-36	MOUNTING FOR SLIDE, FOOTREST 1	2	PCS	5707292230614
	82205-37	MOUNTING FOR SLIDE, FOOTREST 2	2	PCS	5707292230621
	82205-38	MOUNTING FOR SLIDE, FOOTREST 3	2	PCS	5707292230638
150*	82204-36	CONUS MOUNTING FOR FOOTREST 1	2	PCS	5707292230584
	82204-37	CONUS MOUNTING FOR FOOTREST 2	2	PCS	5707292230591
	82204-38	CONUS MOUNTING FOR FOOTREST 3	2	PCS	5707292230607
151*	82208	ADJUSTALBE HOLDER FOR FOOT 1	1	PCS	5707292122544
	82213	ADJUSTALBE HOLDER FOR FOOT 2	1	PCS	5707292122599
	82215	ADJUSTALBE HOLDER FOR FOOT 3	1	PCS	5707292122612
152	0602500	INSEX SCREW 8*12	2	PCS	5707292138163
170	81410	U-MOUNTING FOR NECK-SUPPORT	1	PCS	5707292115133
171	1101000	CENTER SPRING WASHER 8MM	2	PCS	5707292140173
172	0602900	INSEX SCREW 8*16	3	PCS	5707292138200
173	0641600	BUTTON HEAD SCREW 6*16	2	PCS	5707292139269
174*	6171550	HEADREST SIZE 1,BLACK PUR-FOAM	1	PCS	5707292188786
	6172550	HEADREST SIZE 2,BLACK PU-FOAM	1	PCS	5707292188823
	6173550	HEADREST SIZE 3,BLACK PUR-FOAM	1	PCS	5707292188861
	6174550	HEADREST 4, BLACK PU-FOAM	1	PCS	5707292188908
175	81411	Distance-pipe f. neck-mounting	1	PCS	5707292115140
176	81773	Holder for Swan-neck	1	PCS	5707292118561
177	2504000	FERRULE E15*1,0-2,0 INT.BLACK	1	PCS	5707292144102
178	3502516	TIGHTENING HANDLE 45MM M8*16	3	PCS	5707292146915
179*	82561	ADJUSTEMNET BAR 1	1	PCS	5707292124784
	82571	ADJUSTEMNET BAR 2+3	1	PCS	5707292124883
180*	5034400	UPHOL.BACKSUPPORT SIZE 1 GAZEL	1	PCS	5707292158437
	5034450	UPHOL.BACKSUP.SIZE 2&3 GAZELLE	1	PCS	5707292158444
181	74932	UNDERLAYER FOR NECKREST MOUNTI	1	PCS	5707292453174
182	3502600	TIGHTENING HANDLE 65MM M8*16	1	PCS	5707292146922
183	1102000	PLANING WASHER 8MM	1	PCS	5707292140227
184*	82563	ARM WITH CONUS 1	1	PCS	5707292124807
	82573	ARM WITH CONUS 2+3	1	PCS	5707292124906
185	82576	ARM WITH INTERNAL CONUS 2+3	1	PCS	5707292124920
	82566	ARM WITH INTERNAL CONUS 1	1	PCS	5707292124838
188	74930	MOUNTING FOR NECKREST, MOULDED	1	PCS	5707292453150
189	0621600	INSEX SCREW COUNTERS. 6*16	2	PCS	5707292138965

*		CHOOSE SIZE			
---	--	-------------	--	--	--