

# manatee

## Exploded views / Part lists


Rev. date (07.2020)


*On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.*



## Contents


Manatee size 0 .....	3
Manatee size 1+2.....	5
Bath frame.....	7

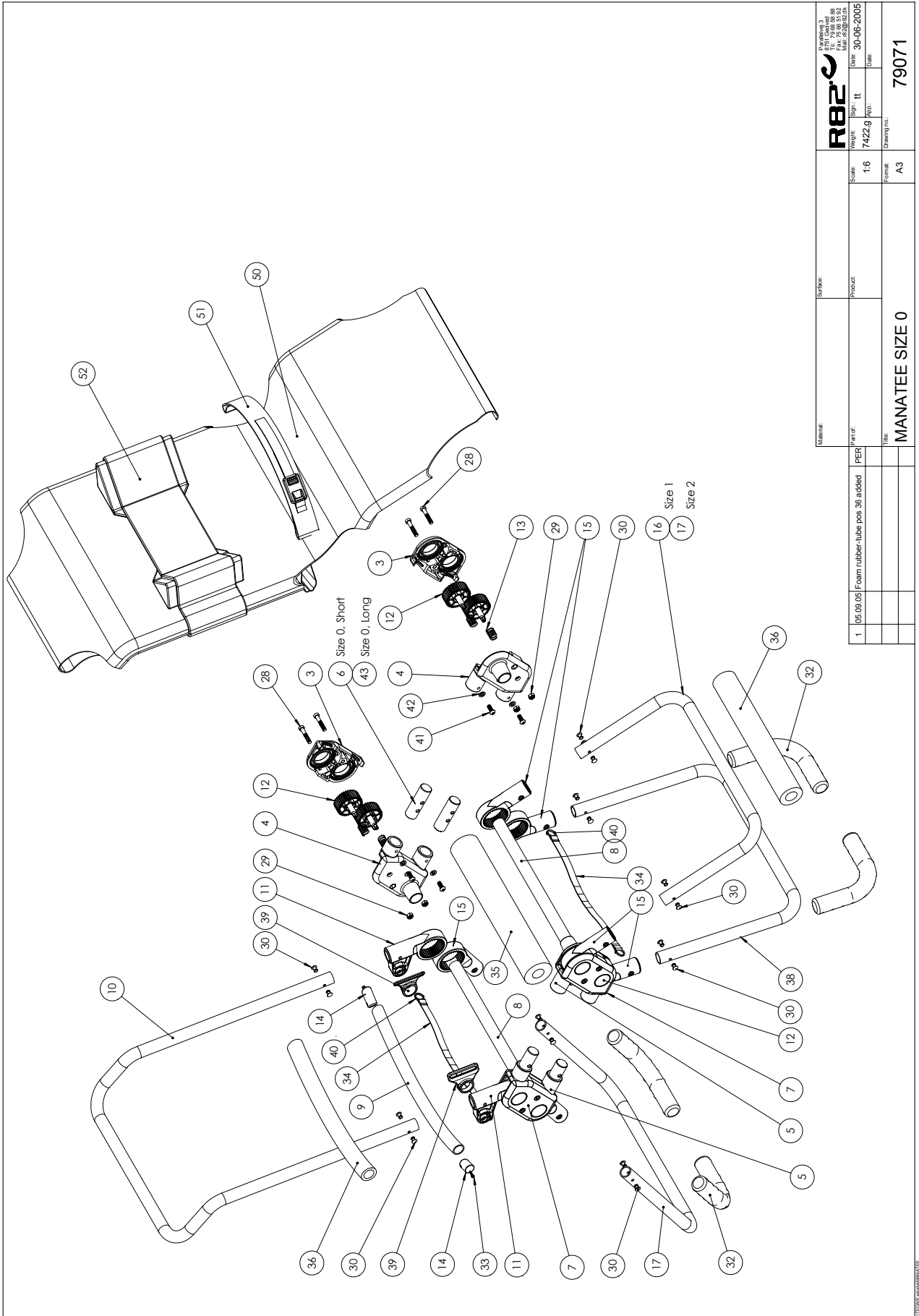

Material:	Number:	 R82 A/S Parallelvej 3 8751 Gedved Denmark Tel: +45 75 66 51 92 Fax: +45 75 66 51 92 Mail: r82@r82.dk	
Product:	Scale:	Sign:	Date:
	1:6	tt	30-06-2005
		Page:	
		Drawing no.:	79071
Title:		MANATEE SIZE 0	

1	05.09.05	Foam rubber-tube pos 36 added	PER

Position	No.	Description	Quantity	Unit of Measure	Ean code
3	73542	Cover for ass.fit.left	2	Pieces	5707292388353
4	73545	Assembly fitting front,left	2	Pieces	5707292388384
5	73544	Assembly fitting front,right	2	Pieces	5707292388377
6	73538	Tube for Manatee frame sz. 0	4	Pieces	5707292414021
7	73543	Cover for ass.fit.right	2	Pieces	5707292388360
8	73531	Tube for frame size 1 and 2	2	Pieces	5707292387813
9	73532	Backbrace Manatee size 1 and 2	1	Pieces	5707292387820
10	73526	Backbrace long,size 1	1	Pieces	5707292387769
11	73547	Turnable element f. Back	2	Pieces	5707292388407
12	73548	Toothed rim, fo 19003,manatee	8	Pieces	5707292388414
13	0310152	Stainless spring. 1,2*12*30m	8	Pieces	5707292388872
14	73550	Plug for tube, back	2	Pieces	5707292388438
15	73546	Turnable element f. Back	6	Pieces	5707292388391
16	73528	Brace frame size 1, Manatee	1	Pieces	5707292387783
17	73527	Long brace size 1+2,manatee	1	Pieces	5707292387776
28	0609837	Stainless insex ch M6*37	8	Pieces	5707292572080
29	1631060	Stainless lock nut 6mm	8	Pieces	5707292141446
30	0506513	Stainless blind rivet Ø4, 8*12	18	Pieces	5707292137326
32	2500530	Tpe Ø22*30*200, Manatee	4	Pieces	5707292381781
33	1940040	Eco-syn z M4*25 stainless	2	Pieces	5707292385727
34	5550112	Releasestrap for Manatee	3	Pieces	5707292120328
35	2500434	Sponge o22*50*340mm	1	Pieces	5707292388551
36	2500430	Sponge o22*35*300mm	2	Pieces	5707292388544
38	73533	Front brace with solid core	1	Pieces	5707292389510
39	73549	Mounting for safety straps	2	Pieces	5707292388421
40	0216150	S-hook,30mm, stainless	6	Pieces	5707292390943
41	0659550	Button head M6*20, stainless	8	Pieces	5707292139467
42	1100490	Spring washer M6 stainless	8	Pieces	5707292411297
43	73539	Tube for Manatee frame sz. 0	4	Pieces	5707292414014
50	882205	Manatee cover size 0, blue	1	Pieces	5707292413789
51	883035	Manatee hip belt, blue	1	Pieces	5707292387974
52	883015	Pads f/headrest/sidesu.,blue	1	Pieces	5707292387899

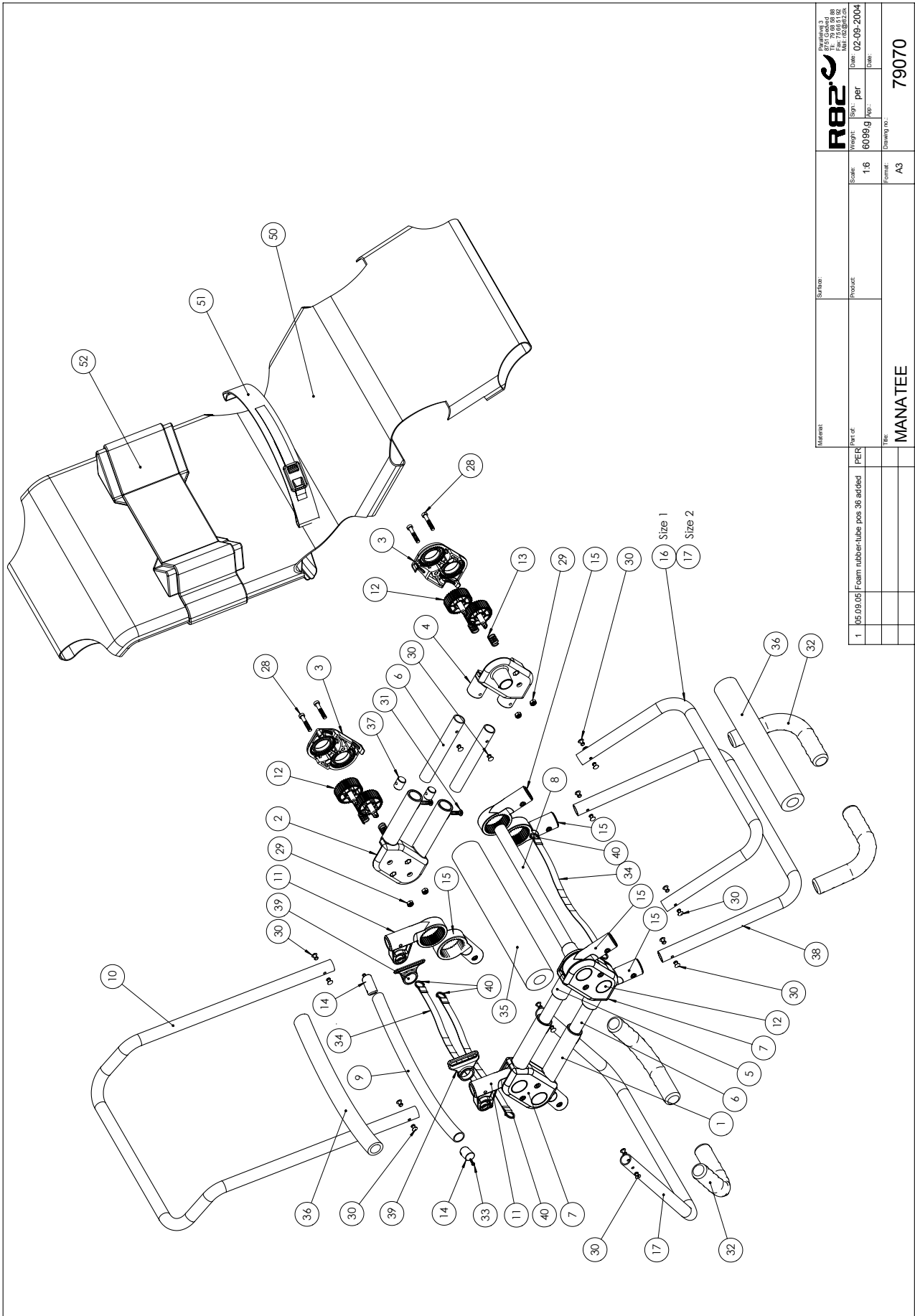


Material:		Scale:		Part of:	
Product:		1:6		PER	
Drawing no.:		Format:		Title:	
MANATEE		A3		MANATEE	

1	05.09.05	Foam rubber tube pos 38 adfied	PER

Material:		Scale:		Part of:	
Product:		1:6		PER	
Drawing no.:		Format:		Title:	
MANATEE		A3		MANATEE	

Material:		Scale:		Part of:	
Product:		1:6		PER	
Drawing no.:		Format:		Title:	
MANATEE		A3		MANATEE	

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	73541	Assembly fitting, rear, right	1	Pieces	5707292388346
2	73540	Assembly fitting, rear, left	1	Pieces	5707292388339
3	73542	Cover for ass. fit. left	2	Pieces	5707292388353
4	73545	Assembly fitting front, left	1	Pieces	5707292388384
5	73544	Assembly fitting front, right	1	Pieces	5707292388377
6	73529	Tube for frame size 1	4	Pieces	5707292387790
6	73530	Tube for frame size 2	4	Pieces	5707292387806
7	73543	Cover for ass. fit. right	2	Pieces	5707292388360
8	73531	Tube for frame size 1 and 2	1	Pieces	5707292387813
8	73534	Tube f. Frame size 3	1	Pieces	5707292505156
9	73532	Backbrace Manatee size 1 and 2	1	Pieces	5707292387820
9	73535	Brace curved, size 3	1	Pieces	5707292505163
10	73526	Backbrace long, size 1	1	Pieces	5707292387769
10	73525	Backbrace long, size 2	1	Pieces	5707292386403
10	73524	Backbrace long, size 3	1	Pieces	5707292505170
11	73547	Turnable element f. Back	2	Pieces	5707292388407
12	73548	Toothed rim, fo 19003, manatee	8	Pieces	5707292388414
13	0310152	Stainless spring. 1,2*12*30m	8	Pieces	5707292388872
14	73550	Plug for tube, back	2	Pieces	5707292388438
15	73546	Turnable element f. Back	6	Pieces	5707292388391
16	73528	Brace frame size 1, Manatee	1	Pieces	5707292387783
17	73527	Long brace size 1+2, manatee	1	Pieces	5707292387776
17	73523	Brace size 3	1	Pieces	5707292505187
28	0609837	Stainless insex ch M6*37	8	Pieces	5707292572080
29	1631060	Stainless lock nut 6mm	8	Pieces	5707292141446
30	0506513	Stainless blind rivet Ø4, 8*12	22	Pieces	5707292137326
31	0659550	Button head M6*20, stainless	4	Pieces	5707292139467
32	2500530	Tpe Ø22*30*200, Manatee	4	Pieces	5707292381781
33	1940040	Eco-syn z M4*25 stainless	2	Pieces	5707292385727
34	5550112	Releasestrap for Manatee	3	Pieces	5707292120328
35	2500434	Sponge o22*50*340mm	1	Pieces	5707292388551
35	2500435	Sponge o22*50*410mm	1	Pieces	5707292516626
36	2500430	Sponge o22*35*300mm	2	Pieces	5707292388544
37	73551	M6-thread bush	4	Pieces	5707292389886
38	73533	Front brace with solid core	1	Pieces	5707292389510
38	73537	Front brace size 3, solid core	2	Pieces	5707292505149
39	73549	Mounting for safety straps	2	Pieces	5707292388421
40	0216150	S-hook, 30mm, stainless	6	Pieces	5707292390943
50	882214	Manatte cover size 1, red	1	Pieces	5707292388087
50	882215	Manatee cover size 1, blue	1	Pieces	5707292388094
50	882224	Manatee cover size 2, red	1	Pieces	5707292386984
50	882225	Manatee cover size 2, blue	1	Pieces	5707292386991
50	882234	Manatte cover size 3, red	1	Pieces	5707292505200
50	882235	Manatte cover size 3, blue	1	Pieces	5707292505224
51	883034	Manatee hip belt, red	1	Pieces	5707292387967
51	883035	Manatee hip belt, blue	1	Pieces	5707292387974
51	883084	Manatee hip belt, red, size 3	1	Pieces	5707292514547
51	883085	Manatee hip belt, blue, size 3	1	Pieces	5707292514554
52	883014	Pads f/headrest/sidesu., red	1	Pieces	5707292387882
52	883015	Pads f/headrest/sidesu., blue	1	Pieces	5707292387899
52	883064	Pads f/headrest/sidesu., red	1	Pieces	5707292514509
52	883065	Pads f/headrest/sidesu., blue	1	Pieces	5707292514516

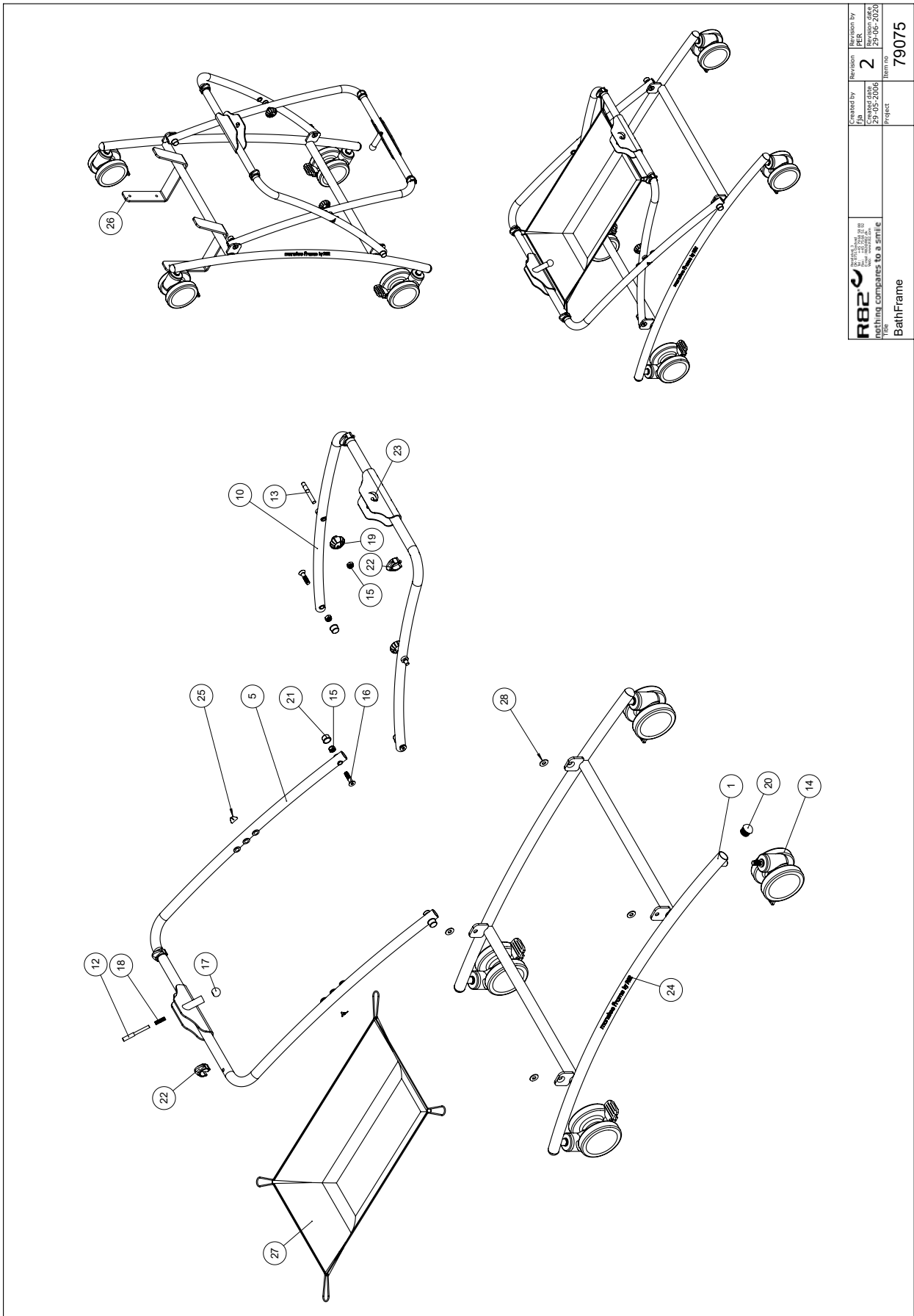


 nothing compares to a smile Title: BathFrame	Created by JBA	Revision by JBA	Revision 2
	Created date 29-05-2006	Revision date 29-06-2020	Item no. 79075
Project		Project	

© INFORMATION DESIGN BATHFRAMES 79075

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	73801	Underframe, bath frame	1	Pieces	5707292441263
5	73802	Upperframe, outer part	1	Pieces	5707292441270
10	73803	Upperframe, inner part	1	Pieces	5707292441287
12	73816	Lock shaft, frame	2	Pieces	5707292441386
13	73817	Lock shaft, frame	2	Pieces	5707292441393
14	3006830	75mm white steinco wheel	4	Pieces	5707292368560
15	1631080	Stainless lock nut 8mm	6	Pieces	5707292141453
16	0625050	Insex screw counters. 8*40,	4	Pieces	5707292470690
17	3502000	Ball grap Ø20 M6, special	2	Pieces	5707292146823
18	0310100	Pressurespring 0,9*8*23*10,5v	2	Pieces	5707292135513
19	3470393	R82 handle M8*14mm nut w.cover	2	Pieces	5707292427458
20	2503520	Plastic plug d25*1,0-2,0 black	4	Pieces	5707292439369
21	2501406	Nut protection, black, f/M8	4	Pieces	5707292375919
22	73819	Safety pin	4	Pieces	5707292441447
23	0493120	Label R82 logo	2	Pieces	5707292466297
24	0493110	Manatee logo	2	Pieces	5707292466280
25	0493437	Label "exclamation mark"	2	Pieces	5707292136909
26	882110	Storage bracket Manatee frame	2	Pieces	5707292441416
27	882104	Basket for Manatee frame, red	1	Pieces	5707292468338
27	882105	Basket for Manatee frame, blue	1	Pieces	5707292468345
28	1105080	Stainless washer 8 mm	4	Pieces	5707292140364



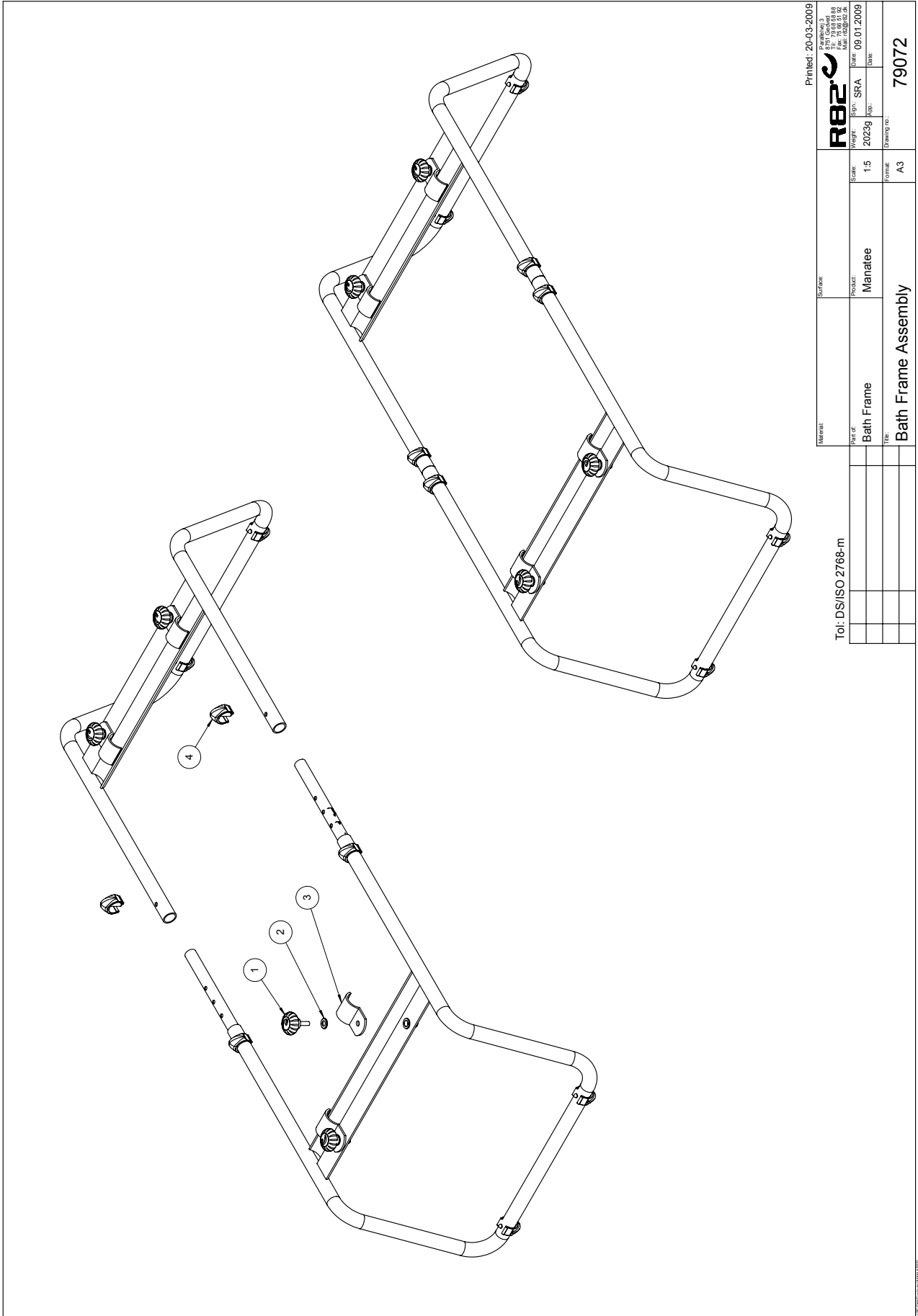

Printed: 20-03-2009

Material:		Surface:	
Part of:		Product:	
Bath Frame		Manatee	
Title:		Bath Frame Assembly	
Weight:	2023g	Scale:	1:5
Sign:	SRA	Form:	A3
Date:	09.01.2009	Drawing no.:	79072

Tel.: DS/ISO 2768-m

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	35021201	R82 HANDLE M8 X 25 MM SCREW	4	PCS	5707292413932
2	1103200	FAZET WASHER 8 MM	4	PCS	5707292140296
3	75997	BRACKET, MANATEE BATH FRAME	4	PCS	570729502384
4	73819	SAFETY PIN	4	PCS	570729441447