

Multi Frame

Exploded views / Parts list

Rev. date (2023.06)


On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.



Contents

Multi Frame with 12½" wheels	3
Item numbers for Multi Frame with 12½" wheels	4
Multi Frame with 22" wheels.....	7
Item numbers for Multi Frame with 22" wheels.....	8
Adaptor for x:panda shape size 3, kit 905090	11
Item numbers for Adaptor for x:panda shape size 3, kit 905090	11

Multi Frame with 12½" wheels

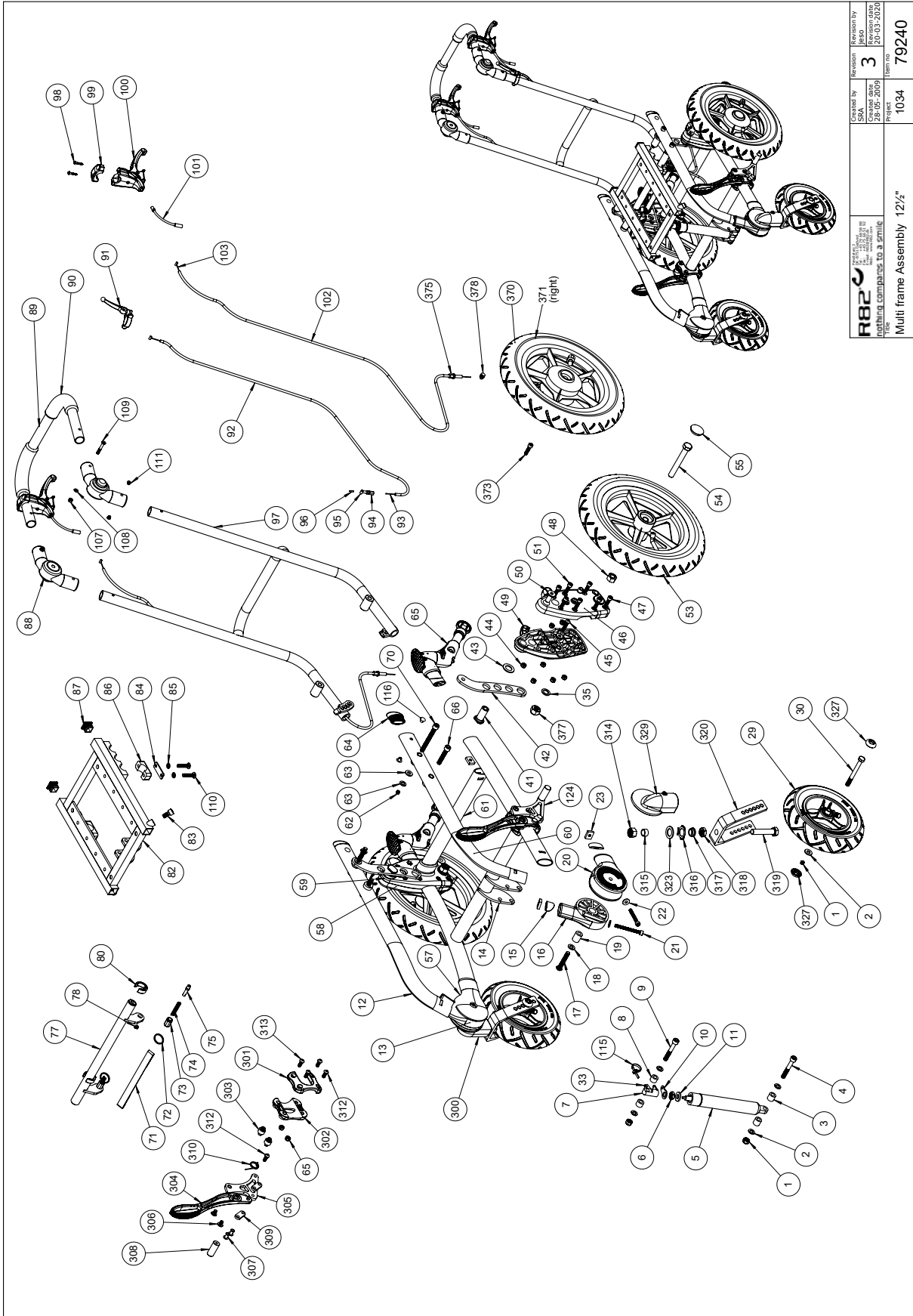

 R82 A/S Parallelvej 3 8751 Gedved Denmark	Created by SAK	Revision by SAK	Revision 3
	Date 28-05-2009	Date 20-03-2020	Item no. 79240
Title Multi frame Assembly 12½"		Project 1034	Item no. 79240


Item numbers for Multi Frame with 12½" wheels

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1631080	Stainless lock nut 8mm	PCS	4	5707292141453
2	1103200	Fazet washer 8mm	PCS	6	5707292140296
3	1350024	Distance piece 14x15x8,2mm	PCS	2	5707292140920
4	603660	Insex screw 8x60	PCS	1	5707292138279
8	1350021	Distance piece 11x15x8,2mm	PCS	2	5707292140890
9	603600	Insex screw 8x55	PCS	1	5707292138262
12	75812-54	Upper frame welded (right)	PCS	1	5707292552167
13	74704	Holder front wheel upper,right	PCS	1	5707292411693
14	75682-54	Nedre stel sverst, str. 1,sølv	PCS	1	5707292504661
14	75683-54	Nedre stel sverst, str. 2,sølv	PCS	1	5710000000000
14	75684-54	Bottom frame weld., s.3, silv	PCS	1	5710000000000
15	74683	Lock, moving part, short	PCS	4	5707292425904
16	74705	Holder front wheel upper,left	PCS	1	5707292411648
17	663045	Button head M8x45, stainless	PCS	2	5707292139634
18	1105080	Stainless washer 8mm	PCS	2	5707292140364
19	74654	Bush Ø16/Ø8, 1x21, Cougar	PCS	2	5707292411662
20	74703	For wheel lower, left	PCS	1	5707292411631
21	602460	Insex M6x60, stainless	PCS	4	5707292400727
22	1106025	Washer M6 o16/o6,4x1,6	PCS	6	5707292404664
23	74090	Mounting for frontfork	PCS	6	5707292374066
29	3009500	190x50 air racing	PCS	2	5707292483355
29	3009501	190x50 solid racing	PCS	2	5707290000000
30	1073661	Steel bolt M8x80 rf part.threa	PCS	2	5707292556585
35	1103500	Fazet washer 12mm	PCS	4	5707292140326
41	75696	Bush for wheel support	PCS	2	5707292503862
42	75706-1	Height adjustment bracket	PCS	2	5707292532688
42	75756	Height adjustment bracket	PCS	2	5710000000000
43	1102750	Washer 16mm	PCS	2	5707292140258
44	1631060	Stainless lock nut 6mm	PCS	14	5707292141446
45	659580	Button head screw 6x30 mm	PCS	2	5707292139481
46	74710-1	Wheel suspension outer left	PCS	1	5707292509567
47	609827	Stainless insex M6x20	PCS	2	5707292375049
48	1604500	Steel nut 12mm, large pack	PCS	2	5707292141309
49	74708-1	Wheel suspension inner left	PCS	1	5707292509543
50	612700	Insex screw counters. 10x20	PCS	2	5707292138767
51	602425	Insex M6x25, stainless	PCS	12	5707292404596
53	3009515	12½x2 1/4" air racing	PCS	2	5707292507020
54	1920100	Steel bolt M12x100, FZB	PCS	2	5707292518118
55	2704300	Black hub cap for rolko wheel	PCS	2	5707292144515
57	74702	Front wheel lower, right	PCS	1	5707292411686
58	74711-1	Wheel suspension outer right	PCS	1	5707292509574
59	74709-1	Wheel suspension inner right	PCS	1	5707292509550
60	75813-54	Upper frame welded (Left)	PCS	1	5707292552174
61	74758	Cross tube, size 1 lower frame	PCS	1	5707292412317
61	74607	Cross tube 40x20, size 2	PCS	1	5710000000000
61	74614	Cross tube 40x20, size 3	PCS	1	5710000000000
62	1607001	Self-locking nut 8mm	PCS	2	5707292401281
63	1310717	Nylon washer, nat o17-8,4x2mm	PCS	4	5707292140821

64	74767	Plug for ellipse 20 x 40	PCS	2	5707292412669
65	970957	Anti-tip devic,Multi Frame all	SET	1	5707292562166
66	603500	Insex screw 8x45	PCS	2	5707292138231
70	603750	Insex M8x75, FZB	PCS	2	5707292138309
71	5550116	Pp adjustable strap, black	PCS	1	5707292374714
71	5550117	Pp adjustable strap, black.	PCS	1	5710000000000
77	75814-54	Cross tube painted (size 1)	PCS	1	5707292552099
77	100826-54	Cross tube painted (size 2)	PCS	1	5710000000000
77	100827-54	Title Cross tube welded size3	PCS	1	5710000000000
78	506519	Stainless blind rivet 4x12	PCS	2	5707292137388
80	75714	Fix ring	PCS	2	5707292504562
82	75720-01	Seat frame -black	PCS	1	5707292545503
83	609816	Stainless insex screw 8x16	PCS	2	5707292138569
84	75752	Pipeholder support plate	PCS	2	5707292513779
85	1100490	Spring washer M6 stainless	PCS	4	5707292411297
86	75713	Pipe holder	PCS	2	5707292504555
87	2504500	Ferrule e20x1,0-2,0 int.black	PCS	2	5707292144126
91	3503531	Gas pump lever with clamp	PCS	1	5707292413529
92	5002987	Outer casing 103cm	PCS	1	5707292520104
92	5002747	Outer casing for break	PCS	1	5710000000000
93	371853	Strained wires 1,5mm 190 cm.	PCS	1	5707292135889
95	388006	Screw nipple 5x7,5mm	PCS	1	5707292136114
96	387000	End nipple for outer casing	PCS	1	5707292136091
97	101341-54	Back frame (size 1) - D,silver	PCS	1	5707292558794
97	101377-54	Back frame (size 1) - extended	PCS	1	5707292559289
97	101349-54	Back frame (size 2) - D,silver	PCS	1	5710000000000
97	101378-54	Back frame (size 1) - extended	PCS	1	5710000000000
97	101357-54	Back frame size 3, wide & D	PCS	1	5710000000000
97	101379-54	Back frame (size 3) - extended	PCS	1	5710000000000
100	3503535	Brake parking lever wo/clamp	PCS	2	5710000000000
101	380060	Flexible v-brake pipe.	PCS	2	5710000000000
102	5002747	Outer casing for break	PCS	2	5710000000000
103	371853	Strained wires 1,5mm 190 cm.	PCS	2	5710000000000
110	659570	Button head screw 6x25 stainl.	PCS	4	5707292139474
111	506508	Rivet 5x14, stainless	PCS	4	5707292359476
115	235075	Cable tape, black	PCS	1	5707292135308
116	1310529	Covering stud, black 6,5mm	PCS	4	5707292445445
327	109378	Wheel protector	PCS	2	5707292581303
370	3009517	12½x2 1/4"right,t/b air racing	PCS	1	5710000000000
370	3009519	12½x2 1/4"db,right,fast racing	PCS	1	5710000000000
371	3009518	12½x2 1/4" left,t/b air racing	PCS	1	5710000000000
371	3009520	12½x2 1/4" db,left fast racing	PCS	1	5710000000000
373	609827	Stainless insex M6x20	PCS	2	5710000000000
375	380000	Cable tighter M5x23	PCS	2	5710000000000
377	1608000	Lock nut 12mm, large pack	PCS	2	5707292141392
378	380040	Cable bench screw M5	PCS	2	5710000000000
53+54	3009516	12½x2 1/4" solid racing	PCS	2	5710000000000
98+99	3503536	Clamp w/screws	PCS	2	5710000000000

5+6 7+10 11+33 94	5040056	Gas spring 200n, Multiframe 1	PCS	1	5707292509444
314-323 300	141091	Front wheel assembly right	PCS	1	5707292880963
314-323 329	140901	Front wheel assembly Left	PCS	1	5707292879059
65+301- 313	970939-LM	Parking brake on frame, left	PCS	1	5707292788320
65+301- 313	970939-RM	Parking brake on frame, right	PCS	1	5707292788337
72-75	5097137	Snap lock with ring	PCS	2	5707292562081
88+89 90+107 108+109	5097125	Hand brace wheelchair frame s1	PCS	1	5707292509468
	5001037	Bag with screws for adaptor	PCS	1	5710000000000
5+6 7+10 11+33 94	5040059	Gas springkit 250N, size 2	PCS	1	5710000000000
88+89 90+107 108+109	5097126	Handbrace wheelchair frame s2	PCS	1	5710000000000
35+301- 313	970939-L	Parking brake on frame, left	PCS	1	5710000000000
65+301- 313	970939-R	Parking brake (on frame),right	PCS	1	5710000000000
5+6 7+10 11+33 94	5040061	Gas spring 350n, size3	PCS	1	5710000000000
88+89 90+107 108+109	5097128	Handbrace wheelchair frame	PCS	1	5710000000000

Multi Frame with 22" wheels

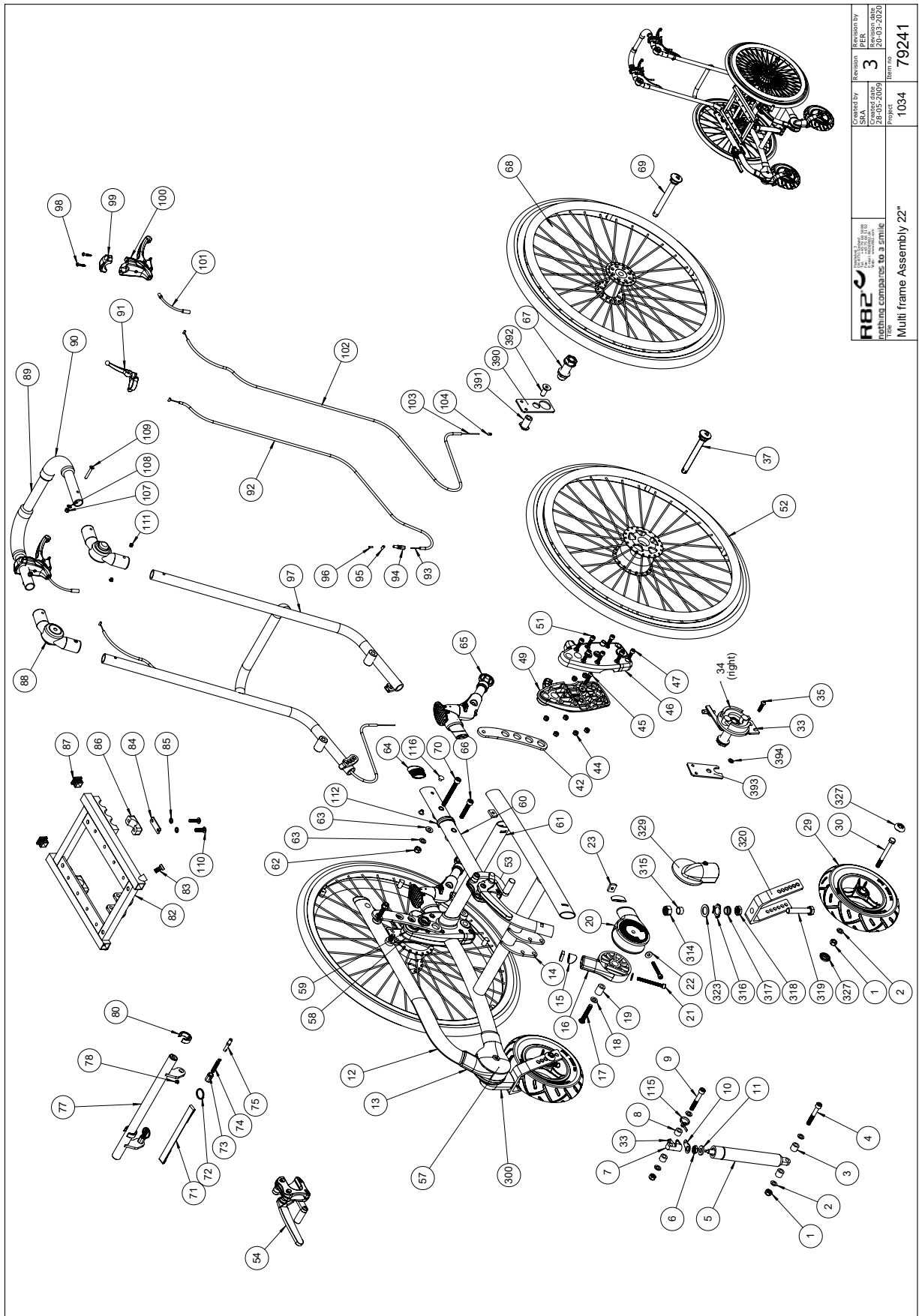

 R82 A/S Parallelvej 3 8751 Gedved Denmark	Checked by SKA 28-05-2009	Revision PER 3	Revision date 20-03-2020
	Program 1034	Part no. 79241	
Multi frame Assembly 22"			

C:\EPSON\TCOA\Kaslova\Kaslova\79241.dwg


Item numbers for Multi Frame with 22" wheels

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1631080	Stainless lock nut 8mm	PCS	4	5707292141453
2	1103200	Fazet washer 8mm	PCS	6	5707292140296
3	1350024	Distance piece 14x15x8,2mm	PCS	2	5707292140920
4	603660	Insex screw 8x60	PCS	1	5707292138279
8	1350021	Distance piece 11x15x8,2mm	PCS	2	5707292140890
9	603600	Insex screw 8x55	PCS	1	5707292138262
12	75812-54	Upper frame welded (right)	PCS	1	5707292552167
13	74704	Holder front wheel upper,right	PCS	1	5707292411693
14	75682-54	Nedre stel sverst, str. 1,sølv	PCS	1	5707292504661
14	75683-54	Nedre stel sverst, str. 2,sølv	PCS	1	5710000000000
15	74683	Lock, moving part, short	PCS	4	5707292425904
16	74705	Holder front wheel upper,left	PCS	1	5707292411648
17	663045	Button head M8x45, stainless	PCS	2	5707292139634
18	1105080	Stainless washer 8mm	PCS	2	5707292140364
19	74654	Bush Ø16/Ø8, 1x21, Cougar	PCS	2	5707292411662
20	74703	For wheel lower, left	PCS	1	5707292411631
21	602460	Insex M6x60, stainless	PCS	4	5707292400727
22	1106025	Washer M6 o16/o6,4x1,6	PCS	4	5707292404664
23	74090	Mounting for frontfork	PCS	6	5707292374066
29	3007910	180x45 black/pneum. Wheel	PCS	2	5707292146045
29	3007875	175x40 wheel, black, solid	PCS	2	5707290000000
29	3009501	190x50 solid racing	PCS	2	5710000000000
30	1073661	Steel bolt M8x80 rf part.threa	PCS	2	5707292556585
35	602205	Stainless insex M6x16	PCS	2	5707292404732
42	75706-1	Height adjustment bracket	PCS	2	5707292532688
44	1631060	Stainless lock nut 6mm	PCS	14	5707292141446
45	659580	Button head screw 6x30 mm	PCS	2	5707292139481
46	74710-1	Wheel suspension outer left	PCS	1	5707292509567
47	609827	Stainless insex M6x20	PCS	2	5707292375049
49	74708-1	Wheel suspension inner left	PCS	1	5707292509543
51	602425	Insex M6x25, stainless	PCS	12	5707292404596
52	3202220	22"wheel w/db, air, black	PCS	2	5707292584113
52	3102225	22" wheel with drumbr.,massive	PCS	2	5707292837615
53	75736	Wheel brake,left	PCS	1	5707292474803
54	75737	Wheel brake, right	PCS	1	5707292484765
57	74702	Front wheel lower, right	PCS	1	5707292411686
58	74711-1	Wheel suspension outer right	PCS	1	5707292509574
59	74709-1	Wheel suspension inner right	PCS	1	5707292509550
60	75813-54	Upper frame welded (Left)	PCS	1	5707292552174
61	74758	Cross tube, size 1 lower frame	PCS	1	5707292412317
61	74607	Cross tube 40x20, size 2	PCS	1	5710000000000
62	1607001	Self-locking nut 8mm	PCS	2	5707292401281
63	1310717	Nylon washer, nat o17-8,4x2mm	PCS	4	5707292140821
64	74767	Plug for ellipse 20 x 40	PCS	2	5707292412669
65	970957	Anti-tip devic,Multi Frame all	SET	1	5707292562166
66	603500	Insex screw 8x45	PCS	2	5707292138231
68	3102215	22" wheel, no break, flexel	PCS	2	5707292837646
68	3202210	22" wheel, no brake, air black	PCS	2	5707292584090

70	603750	Insex M8x75, FZB	PCS	2	5707292138309
71	5550116	Pp adjustable strap, black	PCS	1	5707292374714
77	75814-54	Cross tube painted (size 1)	PCS	1	5707292552099
77	100826-54	Cross tube painted (size 2)	PCS	1	5710000000000
78	506519	Stainless blind rivet 4x12	PCS	2	5707292137388
80	75714	Fix ring	PCS	2	5707292504562
82	75720-01	Seat frame -black	PCS	1	5707292545503
83	609816	Stainless insex screw 8x16	PCS	2	5707292138569
84	75752	Pipeholder support plate	PCS	2	5707292513779
85	1100490	Spring washer M6 stainless	PCS	4	5707292411297
86	75713	Pipe holder	PCS	2	5707292504555
87	2504500	Ferrule e20x1,0-2,0 int.black	PCS	2	5707292144126
91	3503531	Gas pump lever with clamp	PCS	1	5707292413529
92	5002987	Outer casing 103cm	PCS	1	5707292520104
92	5002747	Outer casing for break	PCS	1	5710000000000
93	371853	Strained wires 1,5mm 190 cm.	PCS	1	5707292135889
95	388006	Screwnipple 5x7,5mm	PCS	1	5707292136114
96	387000	End nipple for outer casing	PCS	1	5707292136091
97	101341-54	Back frame (size 1) - D,silver	PCS	1	5707292558794
97	101377-54	Back frame (size 1) - extended	PCS	1	5707292559289
97	101349-54	Back frame (size 2) - D,silver	PCS	1	5710000000000
97	101378-54	Back frame (size 1) - extended	PCS	1	5710000000000
100	3503535	Brake parking lever wo/clamp	PCS	2	5707292413536
101	380060	Flexible v-brake pipe.	PCS	2	5707292425195
102	5002751	Cable for brake, size 3+4	PCS	2	5707292413642
103	371853	Strained wires 1,5mm 190 cm.	PCS	2	5707292135889
104	380100	Clip for cable	PCS	2	5707292136015
110	659570	Button head screw 6x25 stainl.	PCS	4	5707292139474
111	506508	Rivet 5x14, stainless	PCS	4	5707292359476
112	76156	Cable retainer	PCS	2	5707292510280
115	235075	Cable tape, black	PCS	1	5707292135308
116	1310529	Covering stud, black 6,5mm	PCS	4	5707292445445
327	109378	Wheel protector	PCS	2	5707292581303
390	111978	Lower arm for mud guard	PCS	2	5707292588883
391	107280	Bush for mudguard	PCS	2	5707292571564
392	612700	Insex screw counters. 10x20	PCS	2	5707292138767
393	111979	Lower arm for mud guard DB	PCS	2	5707292588890
394	1105060	Stainless washer 6mm	PCS	2	5707292140357
33+37	2983061	Brake plate complet. A-side.	PCS	1	5707292505040
34+37	2983062	Brake plate complet. B-side	PCS	1	5707292505057
67+69	2983063	Axle set 1x qr 012x98mm Cougar	PCS	2	5707292505064
98+99	3503536	Clamp w/screws	PCS	2	5707292427762
72-75	5097137	Snap lock with ring	PCS	2	5707292562081
314-323 300	141091	Front wheel assembly right	PCS	1	5707292880963
314-323 329	140901	Front wheel assembly Left	PCS	1	5707292879059
88+89 90+107 108+109	5097125	Hand brace wheelchair frame s1	PCS	1	5707292509468

5+6 7+10 11+33 94	5040056	Gas spring 200n, Multiframe 1	PCS	1	5707292509444
88+89 90+107 108+109	5097126	Handbrace wheelchair frame s2	PCS	1	5710000000000
5+6 7+10 11+33 94	5040059	Gas springkit 250N, size 2	PCS	1	5710000000000
	5001037	Bag with screws for adaptor	PCS	1	5707292497505

Adaptor for x:panda shape size 3, kit 905090

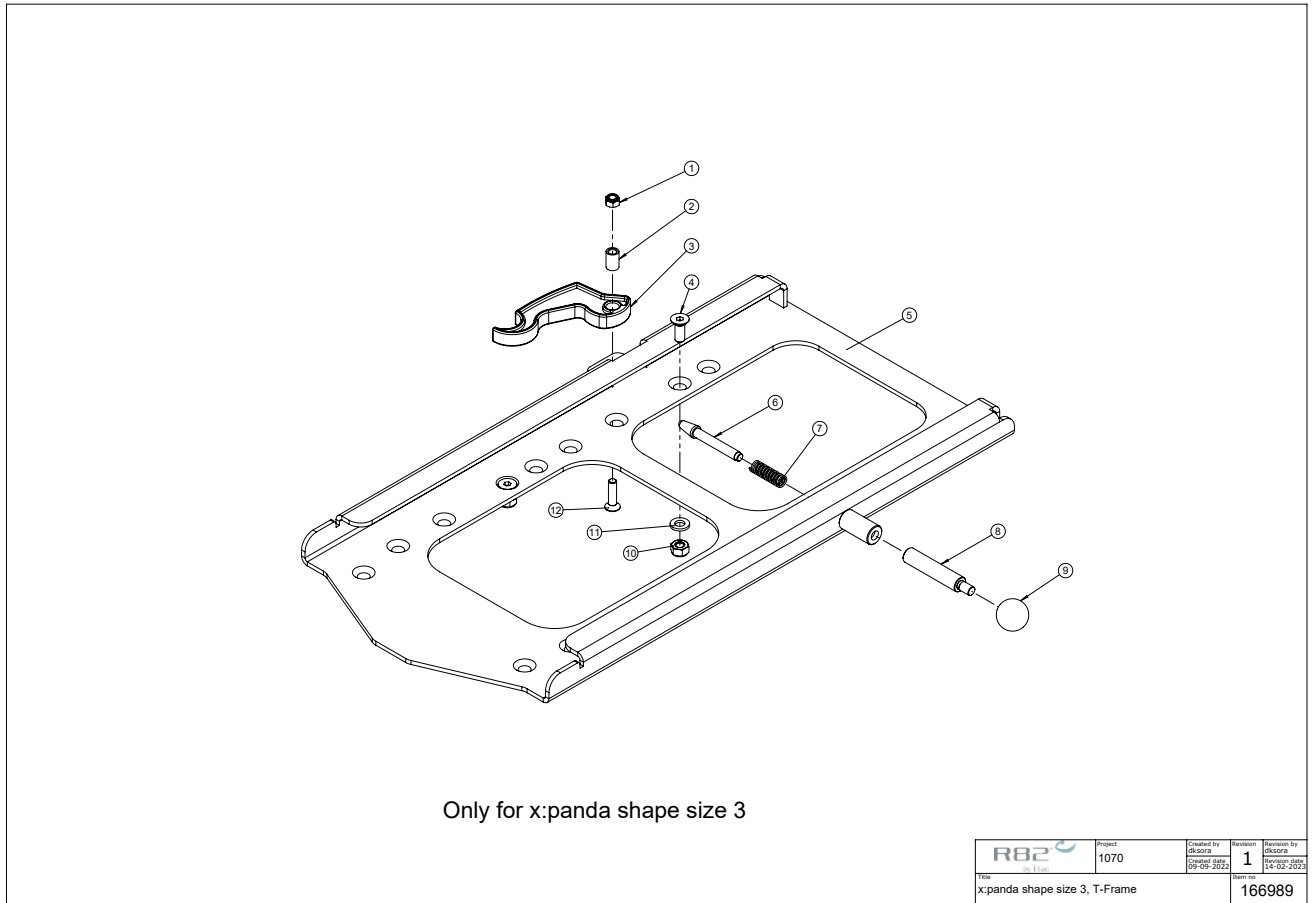

Item numbers for Adaptor for x:panda shape size 3, kit 905090

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1606300	Lock nut 5 mm	1	PCS	5707292141354
2	75641	Bush 5-8x11,8	1	PCS	5707292466501
3	75905	Clamping grip	1	PCS	5707292534590
4	0621600	Insex screw counters. 6x16	4	PCS	5707292138965
5	166990	T-Frame x:panda shape size 3	1	PCS	5707292976130
6	6113000	Spindle, t-part/frame	1	PCS	5707292186362
7	0310100	Pressure spring 0,9x8x23x10,5v	1	PCS	5707292135513
8	6118750	Handle x:panda	1	PCS	5707292530509
9	3502008	Knob handle Ø20 M6	1	PCS	5707292146830
10	1606500	Lock nut 6 mm	4	PCS	5707292141361
11	1105060	SS washer M6 6,4x12x1,6	4	PCS	5707292140357
12	0615020	Insex screw counters. 5x20	1	PCS	5707292138866