

# panda futura

## Exploded views / Part lists


Rev. date (11.2018)

*On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.*




## Contents

Standard size 1.....	3
Standard size 2, 2½, 3 and 4.....	5
Active size 2, 2½, 3 and 4.....	9

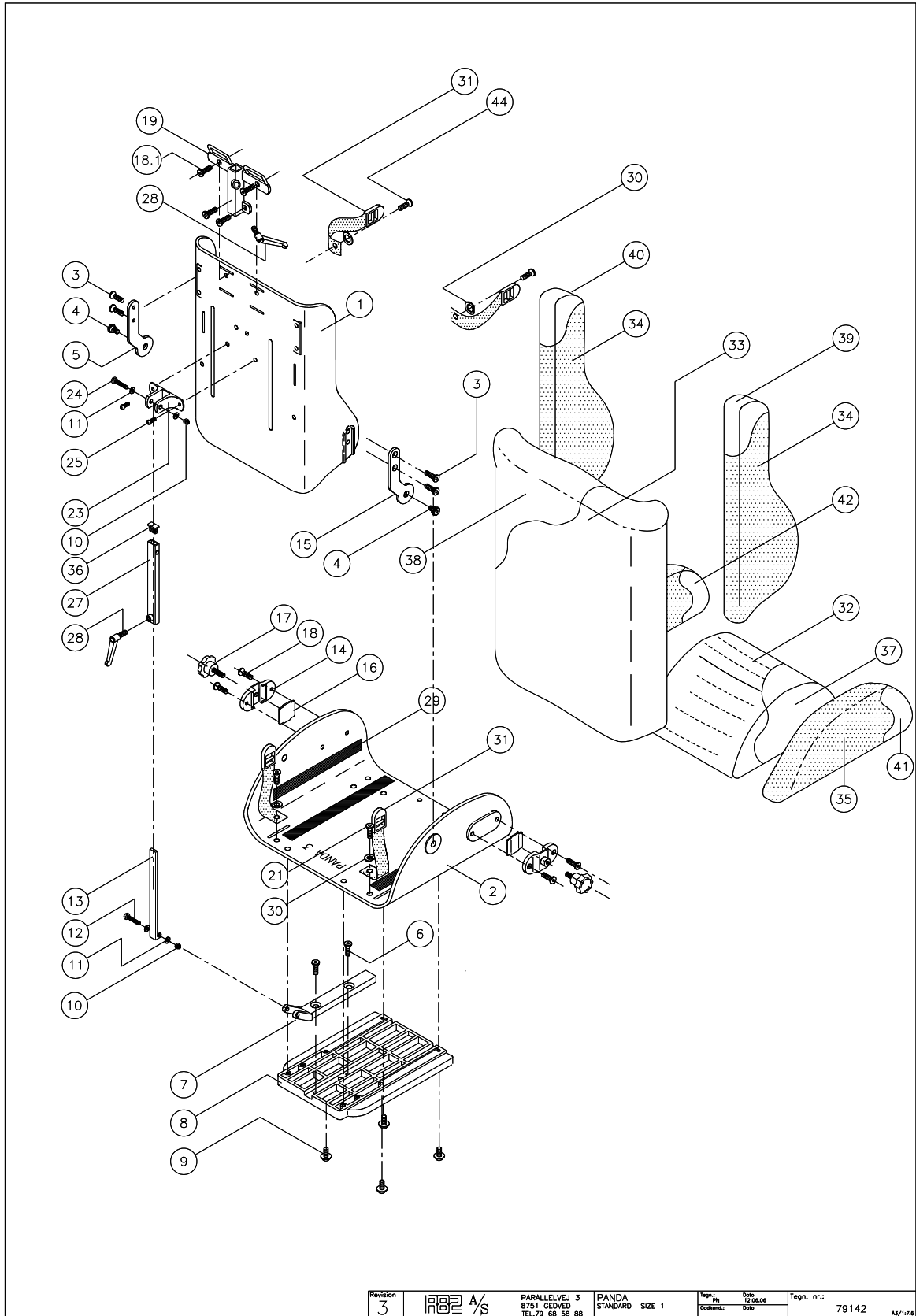

Panda Standard Size 1					
No.	Position	Description	Quantity	Unit of Measure	Ean code
1	82260-60	Plast back Panda 1, black	1	PCS	5707292230836
1	82260-66	Plast back Panda 1, blue	1	PCS	5707292230850
2	82265-60	Plast Seat Panda 1, black	1	PCS	5707292230874
2	82265-66	Plast Seat Panda 1, blue	1	PCS	5707292230898
3	0621600	Insex screw counters. 6*16	4	PCS	5707292138965
4	6109002	Screw spec. M8, Panda 5	2	PCS	5707292466754
5	82327	Side-mounting, left, Panda 2	1	PCS	5707292123459
6	0621600	Insex screw counters. 6*16	2	PCS	5707292138965
7	82329	Tip-mounting Panda 1+2,	1	PCS	5707292123473
8	82301-62	Panda pvc adaptor black	1	PCS	5707292231239
9	0659571	Button head w/flange M6*25 FZB	4	PCS	5707292355843
10	1606500	Lock nut 6 mm	1	PCS	5707292141361
11	1103000	Fazet washer 6mm	2	PCS	5707292140289
12	1901500	Steel set screw 6*35mm	1	PCS	5707292141965
13	81711	Internal rod f.back tip, pa1,2	1	PCS	5707292118004
14	82403	Pvc mounting f/armrest, Panda	2	PCS	5707292123992
15	82326	Side-mounting, right, Panda 2	1	PCS	5707292123442
16	82408	Underlying plate, pvc armrest	2	PCS	5707292124043
17	3652501	R82 handle M6*16mm screw	2	PCS	5707292413963
18	0611200	Insex screw counters. 6*12	4	PCS	5707292138699
18.1	0611600	Insex screw counters. 6*12	4	PCS	5707292138705
19	81152	Mounting for headrest Panda 1	1	PCS	5707292112750
21	0611200	Insex screw counters. 6*12	2	PCS	5707292138699
23	81712	Back mounting for Panda 1-4	1	PCS	5707292118011
24	1901800	Steel set screw 6*40mm	1	PCS	5707292141972
25	0659300	Button head M6*12 fzb, black	2	PCS	5707292139443
27	81710	External pipe for back tip,pan	1	PCS	5707292117991
28	3502516	Tightening handle 45mm M8*16	2	PCS	5707292146915
29	4913035	30mm black ps14 with glue hook	1	PCS	5707292469755
30	1125000	Screenwasher FZB black 5/16*30	2	PCS	5707292140470
31	5550260	Sewn strap holder.Pan.futura	2	PCS	5707292178459
32	896111-65	Cover f.seat,futura sz.1, red	1	PCS	5707292495938
32	896111-66	Cover f.seat,futura sz.1,blue	1	PCS	5707292498229
32	896111-70	Cover f/seatcushion,fut.sz. 1	1	PCS	5707292294340
32	896111-71	Cover f.seat,futura sz.1,b.abc	1	PCS	5707292294364
32	896111-72	Cover f.seat,futura sz.1,b.hop	1	PCS	5707292294388
32	896111-73	Cover Seat futura size 1	1	PCS	5707292355638
32	896111-78	Cover f.seat,futura sz.1,g.abc	1	PCS	5707292294401
32	896111-79	Cover f.seat,futura sz.1,g.hop	1	PCS	5707292294425
32	89638-191	Cover,seat panda futura sz. 1	1	PCS	5707292302397
32	89638-193	Cover, seat panda futura sz. 1	1	PCS	5707292302403
32	89638-195	Cover for seat, Panda 1,red	1	PCS	5707292493231
32	89638-196	Cover for seat, Panda 1,blue	1	PCS	5707292493248
32	89638-198	Cover,seat cushion 1,green abc	1	PCS	5707292302410
33	895211-65	Cover,backr.futura std.1,red	1	PCS	5707292495839
33	895211-66	Cover,backr.futura std.1,blue	1	PCS	5707292498212
33	895211-70	Cover for backcush.futura,sz 1	1	PCS	5707292291882
33	895211-71	Cover,backr.futura std.1,b.abc	1	PCS	5707292291905
33	895211-72	Cover,backr.futura std.1,b.hop	1	PCS	5707292291929
33	895211-73	Cover f/backrest futura sz.1	1	PCS	5707292355614

33	895211-78	Cover,backr.futura std.1,g.abc	1	PCS	5707292291943
33	895211-79	Cover,backr.futura std.1,g.hop	1	PCS	5707292291967
33	895212-65	Cover,back s.1 std,+5, red	1	PCS	5707292496676
33	895212-66	Cover,back s.1 std,+5, blue	1	PCS	5707292519320
33	895212-71	Cover,back s.1 std,+5,blue abc	1	PCS	5707292291981
33	895212-72	Cover,back s.1 std,+5,blue hop	1	PCS	5707292291998
33	895212-73	Cover, back cushion +5cm,sz. 1	1	PCS	5707292367068
33	89639-191	Cover,back Panda futura sz.1	1	PCS	5707292303141
33	89639-193	Cover, back size 1, red abc	1	PCS	5707292303158
33	89639-195	Cover/back Panda 1,red	1	PCS	5707292493224
33	89639-196	Cover/back Panda size 1,blue	1	PCS	5707292493330
33	89639-198	Cover,back size 1,green abc	1	PCS	5707292303165
34	895211-60S	Cover f/backsidecush.fut. Sz 1	1	SET	5707292496409
34	895211-70S	Cover f/backsidecush.fut. Sz 1	1	SET	5707292291899
34	895211-71S	Cover f/backsidecush.fut. Sz.1	1	SET	5707292291912
34	895211-72S	Cover f/backsidecush.fut. Sz.1	1	SET	5707292291936
34	895211-73S	Cover f/backside cush.fut.sz.1	1	SET	5707292355621
34	895211-78S	Cover f/backsidecush.fut.sz.1	1	SET	5707292291950
34	895211-79S	Cover f/backsidecush.fut. Sz.1	1	SET	5707292291974
35	896111-60S	Cover f/seatsidecush.fut.sz. 1	1	SET	5707292496447
35	896111-70S	Cover f/seatsidecush.fut.sz. 1	1	SET	5707292294357
35	896111-71S	Cover f/seatsidecush.fut.sz 1	1	SET	5707292294371
35	896111-72S	Cover f/seatsidecush.fut.sz. 1	1	SET	5707292294395
35	896111-73S	Cover seatside cush.fut. Sz.1	1	SET	5707292355645
35	896111-78S	Cover f/seatsidecush.fut.sz 1	1	SET	5707292294418
35	896111-79S	Cover f/seatsidecush.fut.sz. 1	1	SET	5707292294432
36	2506000	Ferrule fr20*10*1,0-2,0 in.bl.	1	PCS	5707292144195
37	5800775	Foam,seat Panda fut.sz.1,3 cm.	1	PCS	5707292183118
37	5700010	Glueing Panda 1,seatcushion	1	PCS	5707292178572
38	5800800	Foam,back Panda fut.sz.1, 3 cm	1	PCS	5707292183163
38	5800802	Foam back cushion, 5cm,	1	PCS	5707292183170
38	5700005	Glueing Panda 1,backcushion	1	PCS	5707292178527
39	5800830	Foam,backs. Right Panda fut.1	1	PCS	5707292183347
39	5800832	Foam,backs.right,5cm,pan.fut.1	1	PCS	5707292183361
40	5800831	Foam,backs. Left panda fut. 1	1	PCS	5707292183354
40	5800833	Foam,backs.left,5cm,pand.fut.1	1	PCS	5707292183378
41	5800860	Foam, seats. Right,panda fut.1	1	PCS	5707292183545
41	5800862	Foam,seats.right,5cm,pan.fut.1	1	PCS	5707292183569
42	5800861	Foam, seats. Left,panda fut. 1	1	PCS	5707292183552
42	5800863	Foam,seats.left,5cm,pand.fut.1	1	PCS	5707292183576
43	5700211	Glueing lumbar supp.panda 1+2	1	PCS	5707292180544
44	0631862	Insex uh. 5x12, FZB	2	PCS	5707292352293

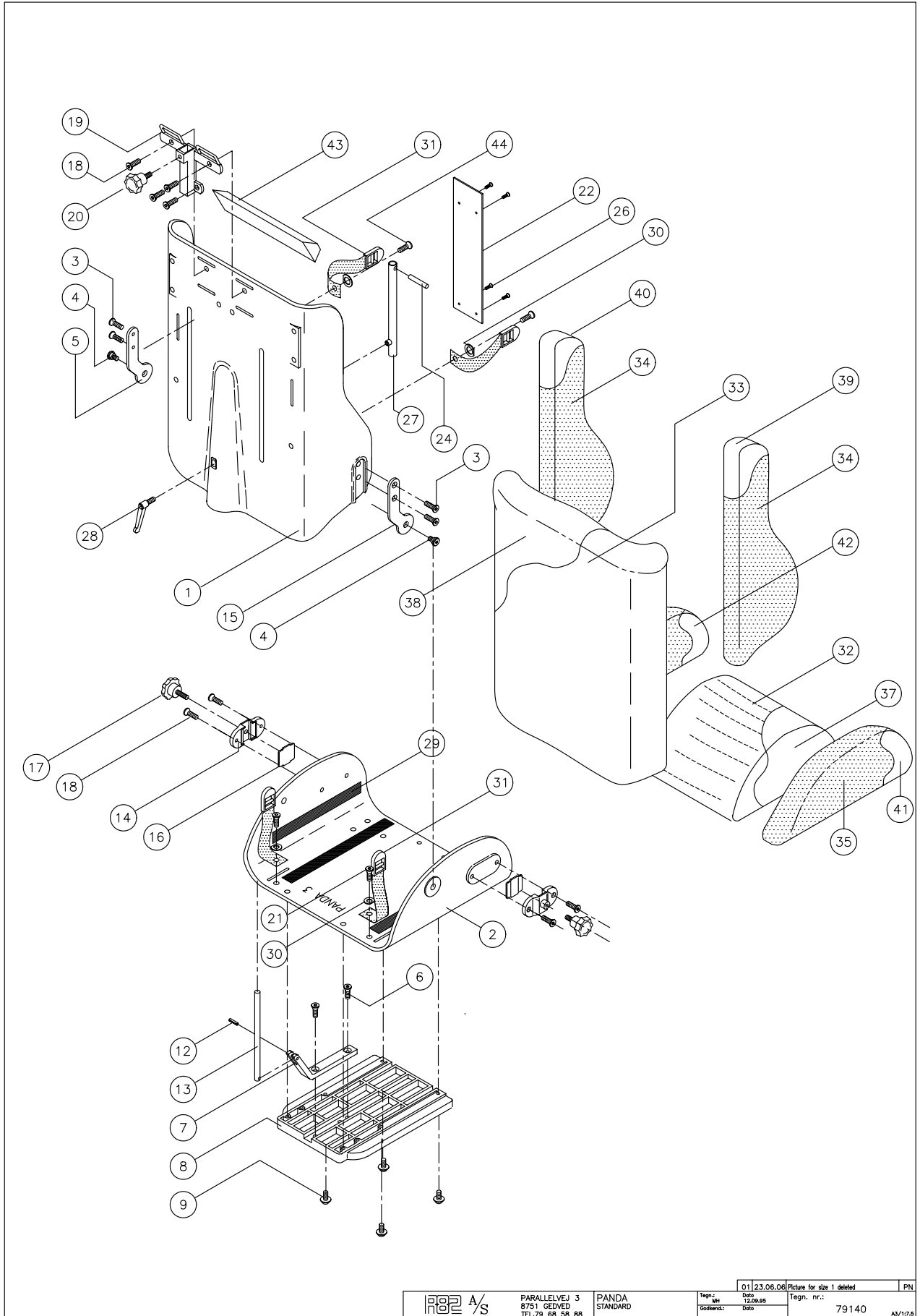


R82 A/S	PARALLELVEJ 3 8751 GEDVED TEL.79 68 58 88	PANDA STANDARD	Tegn. nr.:	01 23.06.06	Picture for size 1 deleted	PN
			Dato:	12.09.95	Tegn. nr.:	79140
			Gedvens.:	Dato:		A3/17.5

Panda Standard Size 2-4					
No.	Position	Description	Quantity	Unit of Measure	Ean code
82320-60	1	Plast back Panda 2, black	1,00	Pieces	5707292231246
82330-60	1	Plast back Panda 3, black	1,00	Pieces	5707292231321
82340-60	1	Plast back Panda 4, black	1,00	Pieces	5707292231406
82290-60	1	Plast back Panda 2½, black	1,00	Pieces	5707292231154
82325-60	2	Plast Seat Panda 2, black	1,00	Pieces	5707292231284
82335-60	2	Plast Seat Panda 3, black	1,00	Pieces	5707292231369
82345-60	2	Plast Seat Panda 4, black	1,00	Pieces	5707292231444
82295-60	2	Plast Seat Panda 2½, black	1,00	Pieces	5707292231192
0621600	3	Insex unders. M6*16, FZB	4,00	Pieces	5707292138965
6109002	4	Screw spec. M8, Panda 5	2,00	Pieces	5707292466754
82327	5	Side-mounting, left, Panda 2	1,00	Pieces	5707292123459
82337	5	Side-mounting, left, Panda 3	1,00	Pieces	5707292123527
82347	5	Side-mounting, left, Panda 4	1,00	Pieces	5707292123596
0621600	6	Insex screw counters. 6*16	2,00	Pieces	5707292138965
82328	7	Tip-mounting, Panda 2	1,00	Pieces	5707292123466
82338	7	Tip-mounting, Panda 3	1,00	Pieces	5707292123534
82348	7	Tip-mounting, Panda 4	1,00	Pieces	5707292123602
82298	7	Tip-mounting, Panda 2½	1,00	Pieces	5707292123190
82301-62	8	Panda pvc adaptor black	1,00	Pieces	5707292231239
0659571	9	Button head w/flange M6*25 FZB	4,00	Pieces	5707292355843
0589100	12	Stainl. Tightening sprig 5*20	1,00	Pieces	5707292137807
82323	13	Internal pipe, Panda 2	1,00	Pieces	5707292123428
82333	13	Internal pipe, Panda 3	1,00	Pieces	5707292123503
82343	13	Internal pipe, Panda 4	1,00	Pieces	5707292123572
82293	13	Internal pipe, Panda 2½	1,00	Pieces	5707292123176
82403	14	Pvc mounting f/armrest, Panda	2,00	Pieces	5707292123992
82326	15	Side-mounting, right, Panda 2	1,00	Pieces	5707292123442
82336	15	Side-mounting, right, Panda 3	1,00	Pieces	5707292123510
82346	15	Side-mounting, right, Panda 4	1,00	Pieces	5707292123589
82408	16	UNDERLAYING PLATE, PVC ARMREST	2,00	Pieces	5707292124043
3652501	17	R82 handle M6*16mm screw	2,00	Pieces	5707292413963
0611600	18	Insex screw counters. 6*12	4,00	Pieces	5707292138705
81138	19	Mounting for headrest w.strap	1,00	Pieces	5707292112637
3503001	20	R82 handle M8*15mm screw	1,00	Pieces	5707292413925
0611200	21	Insex screw counters. 6*12	2,00	Pieces	5707292138699
82321	22	Cover-plate, Panda 2	1,00	Pieces	5707292123404
82331	22	Cover-plate, Panda 3	1,00	Pieces	5707292123480
82341	22	Cover-plate, Panda 4	1,00	Pieces	5707292123558
82291	22	Cover-plate, Panda 2½	1,00	Pieces	5707292123152
6118805	24	Pin Ø8*44 f. Panda back, pvc	1,00	Pieces	5707292186690
1940000	26	Eco-syn, 3,5*10	4,00	Pieces	5707292142559
82322	27	External pipe, Panda 2	1,00	Pieces	5707292123411
82332	27	External pipe, Panda 3	1,00	Pieces	5707292123497
82342	27	External pipe, Panda 4	1,00	Pieces	5707292123565
82292	27	External pipe, Panda 2½	1,00	Pieces	5707292123169
3504025	28	Tightening handle 45mm M8*25	1,00	Pieces	5707292147103
4913035	29	30mm black ps14 with glue hook	0,85	Meter	5707292469755
1125000	30	Screenwasher FZB black 5/16*30	2,00	Pieces	5707292140470

5550260	31	Sewn strap holder.Pan.futura	2,00	Pieces	5707292178459
896121-65	32	Cover f.seat,futura sz.2, red	1,00	Pieces	5707292495945
896121-66	32	Cover,seat cush. Futura size 2, blue	1,00	Pieces	5707292497444
896131-65	32	Cover f.seat,futura sz.3, red	1,00	Pieces	5707292495969
896131-66	32	Cover f.seat,futura sz.3,blue	1,00	Pieces	5707292498250
896141-65	32	Cover for seatcushion,fut.sz 4, red	1,00	Pieces	5707292495976
896141-66	32	Cover for seatcushion,fut.sz 4, blue	1,00	Pieces	5707292498274
896161-65	32	Cover f/seatcushion,fut.sz 2½, red	1,00	Pieces	5707292495952
896161-66	32	Cover,seat cushion fut 2½, blue	1,00	Pieces	5707292497420
895221-65	33	Cover,backr.futura std.2, red	1,00	Pieces	5707292495846
895221-66	33	Cover,backr.futura std.2, blue	1,00	Pieces	5707292497406
895222-65	33	Cover,back s.2 std.+5, red	1,00	Pieces	5707292496683
895222-66	33	Cover.back cush.fut.std.sz 2, blue	1,00	Pieces	5707292497475
895231-65	33	Cover,backr.futura std.3, red	1,00	Pieces	5707292495860
895231-66	33	Cover,backr.futura std.3,blue	1,00	Pieces	5707292498243
895232-65	33	Cover,back s.3 std.+5, red	1,00	Pieces	5707292496690
895232-66	33	Cover,back cushion,size 3, blue	1,00	Pieces	5707292507006
895241-65	33	Cover,backr.futura std.4, red	1,00	Pieces	5707292495877
895241-66	33	Cover,backr.futura std.4,blue	1,00	Pieces	5707292498267
895242-65	33	Cover,back s.4 std.+5, red	1,00	Pieces	5707292496706
895242-66	33	Cover,back s.4 std.+5, blue	1,00	Pieces	5707292509079
895261-65	33	Cover,back,futura std.2½, red	1,00	Pieces	5707292495853
895261-66	33	Cover,back,futura std.2½, blue	1,00	Pieces	5707292498236
895262-65	33	Cover,back 2½ std.+5, red	1,00	Pieces	5707292496713
895262-66	33	Cover,back 2½ std.+5, blue	1,00	Pieces	5707292503152
895221-60S	34	Cover f/backsidecush.fut.sz 2, black	1,00	Set	5707292496416
895231-60S	34	Cover f/backsidecush.fut.sz. 3, black	1,00	Set	5707292496430
895241-60S	34	Cover,backcush,fut. Std.size 4, black	1,00	Set	5707292366139
895261-60S	34	Cover f/backsidecush.fut.sz 2½, black	1,00	Set	5707292496423
896121-60S	35	Cover f/seatsidecush.fut.sz. 2, black	1,00	Set	5707292496454
896131-60S	35	Cover f/seatsidecush.fut.sz 3, black	1,00	Set	5707292496485
896141-60S	35	Cover f/seatsidecush.fut.sz 4, black	1,00	Set	5707292366207
896161-60S	35	Cover f/seatsidecush.fut.sz 2½, black	1,00	Set	5707292496478
5800780	37	Foam, Seat 3 cm futura size 2	1,00	Pieces	5707292183125
5800785	37	Foam,seat 3 cm, futura size 3	1,00	Pieces	5707292183132
5800790	37	Foam, Seat 3 cm futura size 4	1,00	Pieces	5707292183149
5800795	37	Foam, Seat 3 cm futura size 2½	1,00	Pieces	5707292183156
5800805	38	Foam,back,3cm pan.fut. 2	1,00	Pieces	5707292183187
5800806	38	Foam,back,5cm pan.fut. 2	1,00	Pieces	5707292183194
5800810	38	Foam, back cushion, 3cm	1,00	Pieces	5707292183224
5800811	38	Foam,back,5cm pan.fut. 3	1,00	Pieces	5707292183231
5800815	38	Foam,backcushion,3cm pan.fut.4	1,00	Pieces	5707292183262
5800816	38	Foam,back,5cm pan.fut. 4	1,00	Pieces	5707292183279
5800820	38	Foam for back cushion 3cm	1,00	Pieces	5707292183309
5800821	38	Foam,back,5cm pan.fut. 2½	1,00	Pieces	5707292183316
5800835	39	Foam,back.right,3cm.pand.fut.2	1,00	Pieces	5707292183385
5800837	39	Foam,backs.right,5cm,pan.fut.2	1,00	Pieces	5707292183408
5800840	39	Foam,right back sdie 3 cm, fut	1,00	Pieces	5707292183422
5800842	39	Foam,backs.right,5cm,pan.fut.3	1,00	Pieces	5707292183446
5800845	39	Foam,backside 3 cm pan.fut.4	1,00	Pieces	5707292183460



5800847	39	Foam,backs,right,5cm,pan.fut.4	1,00	Pieces	5707292183484
5800850	39	Foam for right backside, 3cm	1,00	Pieces	5707292183507
5800852	39	Foam,backs.right,5cm,pan.fut2½	1,00	Pieces	5707292183521
5800836	40	Foam,back.left,3cm,pand.fut.2	1,00	Pieces	5707292183392
5800838	40	Foam,backs.left,5cm,pand.fut.2	1,00	Pieces	5707292183415
5800841	40	Foam, left back side 3 cm	1,00	Pieces	5707292183439
5800843	40	Foam,backs.left,5cm,pand.fut.3	1,00	Pieces	5707292183453
5800846	40	Foam,backside 3 cm pan.fut.4	1,00	Pieces	5707292183477
5800848	40	Foam,backs.left,5cm,pand.fut.4	1,00	Pieces	5707292183491
5800851	40	Foam for left backside, 3cm	1,00	Pieces	5707292183514
5800853	40	Foam,backs.left,5cm,pan.fut.2½	1,00	Pieces	5707292183538
5800865	41	Foam,seat side,right 3cm, sz 2	1,00	Pieces	5707292183583
5800867	41	Foam,seats.right,5cm,pan.fut.2	1,00	Pieces	5707292183606
5800870	41	Foam, right Seat side 3 cm	1,00	Pieces	5707292183620
5800872	41	Foam,seats.right,5cm,pan.fut.3	1,00	Pieces	5707292183644
5800875	41	Foam, right Seat side 3 cm,sz4	1,00	Pieces	5707292183668
5800877	41	Foam,seats.right,5cm,pan.fut.4	1,00	Pieces	5707292183682
5800880	41	Foam,seat side,right 3cm,sz 2½	1,00	Pieces	5707292183705
5800882	41	Foam,seats.right,5cm,pan.fut2½	1,00	Pieces	5707292183729
5800866	42	Foam, Seat side left, 3cm,sz 2	1,00	Pieces	5707292183590
5800868	42	Foam,seats.left,5cm,pand.fut.2	1,00	Pieces	5707292183613
5800871	42	Foam, left Seat side 3 cm	1,00	Pieces	5707292183637
5800873	42	Foam,seats.left,5cm,pand.fut.3	1,00	Pieces	5707292183651
5800876	42	Foam, Seat side left, 3 cm,sz4	1,00	Pieces	5707292183675
5800878	42	Foam,seats.left,5cm,pand.fut.4	1,00	Pieces	5707292183699
5800881	42	Foam,seat side,left, 3cm,sz 2½	1,00	Pieces	5707292183712
5800883	42	Foam,seats.left,5cm,pan.fut.2½	1,00	Pieces	5707292183736
5800890	43	Lombar support, upholstered	1,00	Pieces	5707292183774
5800891	43	Lombar support std 3 + act 4	1,00	Pieces	5707292183781
5800892	43	Lombar support pan.fut. 4	1,00	Pieces	5707292183798
0631862	44	Insex uh. 5*12, FZB	2,00	Pieces	5707292352293

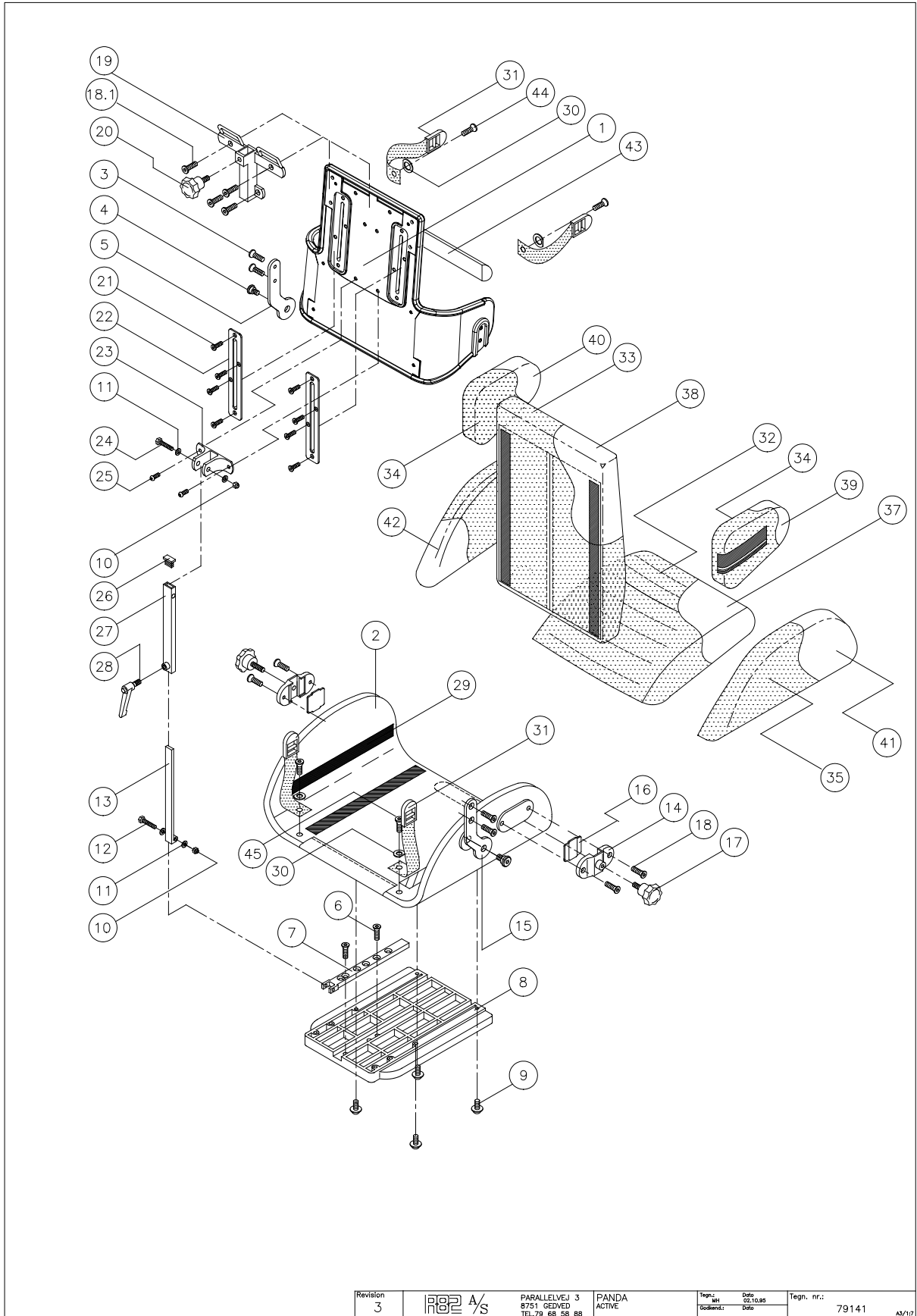

Panda Active Size 2-4					
No.	Position	Description	Quantity	Unit of Measure	Ean code
1	82270-66	Panda active plast. Back sz. 2	PCS	1	5707292230935
1	82280-66	Plast back Panda active 3,blue	PCS	1	5707292231079
1	82285-66	Panda active plast back	PCS	1	5707292231123
1	82275-66	Pvc back Panda active 2½,blue	PCS	1	5707292231000
1	82270-60	Back Panda active 2, black	PCS	1	5707292230911
1	82280-60	Plast back Panda active3,black	PCS	1	5707292231055
1	82285-60	Back Panda active 4, black	PCS	1	5707292231109
1	82275-60	Back Panda active 2½, black	PCS	1	5707292230980
2	82325-66	Plast Seat Panda 2, blue	PCS	1	5707292231307
2	82335-66	Plast seat panda fut.std. 3	PCS	1	5707292231383
2	82345-66	Plast Seat for Panda fut.std.4	PCS	1	5707292231468
2	82295-66	Seat for Panda 2½, plastic	PCS	1	5707292231215
2	82325-60	Plast Seat Panda 2, black	PCS	1	5707292231284
2	82345-60	Plast Seat Panda 4, black	PCS	1	5707292231444
2	82295-60	Plast Seat Panda 2½, black	PCS	1	5707292231192
3	0621600	Insex screw counters. 6*16	PCS	4	5707292138965
4	6109002	Screw spec. M8, Panda 5	PCS	2	5707292466754
5	82327	Side-mounting, left, Panda 2	PCS	1	5707292123459
5	82337	Side-mounting, left, Panda 3	PCS	1	5707292123527
5	82347	Side-mounting, left, Panda 4	PCS	1	5707292123596
6	0621600	Insex screw counters. 6*16	PCS	2	5707292138965
7	82350	Tilt mounting universal	PCS	1	5707292123626
7	82339	Tilt mounting Panda size 4	PCS	1	5707292123541
8	82301-62	Panda pvc adaptor black	PCS	1	5707292231239
9	0659571	Button head w/flange M6*25 FZB	PCS	4	5707292355843
10	1606500	Lock nut 6 mm	PCS	1	5707292141361
11	1103000	Fazet washer 6mm	PCS	2	5707292140289
12	1901500	Steel set screw 6*35mm	PCS	1	5707292141965
13	81711	Internal rod f.back tip, pa1,2	PCS	1	5707292118004
13	81695	Int. Rod f. Backtip Panda 3-4	PCS	1	5707292117861
14	82403	Pvc mounting f/armrest, Panda	PCS	2	5707292123992
15	82326	Side-mounting, right, Panda 2	PCS	1	5707292123442
15	82336	Side-mounting, right, Panda 3	PCS	1	5707292123510
15	82346	Side-mounting, right, Panda 4	PCS	1	5707292123589
16	82408	Underlying plate, pvc armrest	PCS	2	5707292124043
17	3652501	R82 handle M6*16mm screw	PCS	2	5707292413963
18	0611600	Insex screw counters. 6*12	PCS	4	5707292138705
18.1	0611000	Insex screw counters. 6*20	PCS	4	5707292138682
19	81138	Mounting for headrest w.strap	PCS	1	5707292112637
20	3503001	R82 handle M8x15mm screw	PCS	1	5707292413925
21	1940000	Eco-syn, 3,5*10	PCS	8	5707292142559
21	0621200	Insex screw 6*12mm	PCS	8	5707292138958
22	81918	Mounting f.sidesupport Impala	PCS	2	5707292119865
22	74201	Rail for side supports size 1	PCS	2	5707292449566
23	81712	Back mounting for Panda 1-4	PCS	1	5707292118011
24	1901800	Steel set screw 6*40mm	PCS	1	5707292141972
25	0645600	Button head screw 6*16 black	PCS	2	5707292139290
26	2506000	Ferrule fr20*10*1,0-2,0 in.bl.	PCS	1	5707292144195

27	81710	External pipe for back tip,pan	PCS	1	5707292117991
28	3502516	Tightening handle 45mm M8*16	PCS	1	5707292146915
29	4913035	30mm black ps14 with glue hook	MTR	0,85	5707292469755
30	1125000	Screenwasher FZB black 5/16*30	PCS	2	5707292140470
31	5550260	Sewn strap holder.Pan.futura	PCS	2	5707292178459
32	896131-65US	Cover seat cushion s3	PCS	1	5707292544377
32	896131-65	Cover f.seat,futura sz.3, red	PCS	1	5707292495969
32	896121-84	Cover, Seat cushion, size 2	PCS	1	5707292438331
32	896121-78	Cover f.seat,futura sz.2,g.abc	PCS	1	5707292294487
32	896121-73	Cover f. Seatside size 2 futur	PCS	1	5707292367044
32	896121-71	Cover f.seat,futura sz.2,b.abc	PCS	1	5707292294449
32	896121-66	Cover,seat cush. Futura size 2	PCS	1	5707292497444
32	896121-65	Cover f.seat,futura sz.2, red	PCS	1	5707292495945
32	896121-79	Cover f.seat,futura sz.2,g.hop	PCS	1	5707292294500
32	896121-72	Cover f.seat,futura sz.2,b.hop	PCS	1	5707292294463
32	896121-71US	Cover,backcushion,futura activ	PCS	1	5707292368973
32	896131-66	Cover f.seat,futura sz.3,blue	PCS	1	5707292498250
32	896131-71	Cover f.seat,futura sz.3,blue	PCS	1	5707292294630
32	896131-71US	Cover,seatcushion,futura activ	PCS	1	5707292294654
32	896131-72	Cover for seatcushion,fut.sz 3	PCS	1	5707292294661
32	896131-73	Cover p.futura Seat 3,red abc	PCS	1	5707292294685
32	896131-78	Cover for seatcushion,fut.sz 3	PCS	1	5707292294708
32	896131-79	Cover for seatcushion,fut.sz 3	PCS	1	5707292294722
32	896131-84	Cover, Seat cushion size 3	PCS	1	5707292368522
32	896141-65	Cover for seatcushion,fut.sz 4	PCS	1	5707292495976
32	896141-65US	Cover seat cushion size4	PCS	1	5707292550750
32	896141-66	Cover for seatcushion,fut.sz 4	PCS	1	5707292498274
32	896141-66US	Cover for seatcushion,fut.sz 4	PCS	1	5707292551207
32	896141-71	Cover for seatcushion,fut.sz 4	PCS	1	5707292295903
32	896141-71US	Cover,seatcushion,futura activ	PCS	1	5707292295927
32	896141-72	Cover for seatcushion,fut.sz 4	PCS	1	5707292295934
32	896141-78	Cover for seatcushion,fut.sz 4	PCS	1	5707292295958
32	896141-79	Cover for seatcushion,fut.sz 4	PCS	1	5707292295972
32	896161-84	Cover,seatsidecush.size 2½	PCS	1	5707292438379
32	896161-79	Cover f/seatcushion,fut.sz 2½	PCS	1	5707292297105
32	896161-78	Cover f/seatcushion,fut.sz 2½	PCS	1	5707292297082
32	896161-73	Cover for Seat cushion size 2½	PCS	1	5707292357007
32	896161-72	Cover f/ seatcushion,fut.sz 2½	PCS	1	5707292297068
32	896161-71US	Cover,seatchusion,futura activ	PCS	1	5707292297051
32	896161-71	Cover f/seatcushion,fut.sz 2½	PCS	1	5707292297037
32	896161-70	Cover seat cushion size2½	PCS	1	5707292367969
32	896161-66	Cover,seat cushion fut 2½	PCS	1	5707292497420
32	896161-65US	Cover f/seatcushion,fut.sz 2½	PCS	1	5707292541123
32	896161-65	Cover f/seatcushion,fut.sz 2½	PCS	1	5707292495952
32	896141-73	Cover for seatcushion,fut.sz 4	PCS	1	5707292361523
33	895131-65US	Cover,backcushion,futura activ	PCS	1	5707292544360
33	895131-65	Cover,backre.futura ac.3, red	PCS	1	5707292496287
33	895122-84	5cm cov.back cush.fut.act.sz.2	PCS	1	5707292598578
33	895122-78	5CM BETR.RYGHYNDE.FU.AKT.STR.2	PCS	1	5707292291028
33	895122-73	5CM BETR.RYGHYNDE.FU.AKT.STR.2	PCS	1	5707292367181

33	895122-71	5CM BETR.RYGHYNDE.FU.AKT.STR.2	PCS	1	5707292291004
33	895122-66	Cover, back cush. 5 cm, size 2	PCS	1	5707292561749
33	895122-65	Cover, back cush. 5 cm, size 2	PCS	1	5707292496799
33	895121-79	Cover,backre.futura ac.2,g.hop	PCS	1	5707292290977
33	895121-78	Cover,backre.futura ac.2,g.abc	PCS	1	5707292290953
33	895121-73	BETR.RYGHYNDE.FUTURA AKT.STR.2	PCS	1	5707292367136
33	895121-72	Cover,backre.futura ac.2,b.hop	PCS	1	5707292290939
33	895121-71US	Cover,backcushion,futura activ	PCS	1	5707292368966
33	895121-71	Cover,backre.futura ac.2,b.abc	PCS	1	5707292290915
33	895121-66	Cover,backre.futura ac.2,blue	PCS	1	5707292498076
33	895121-65	Cover,backre.futura ac.2, red	PCS	1	5707292496263
33	895131-66	Cover,backre.futura ac.3,blue	PCS	1	5707292498083
33	895131-71	Cover,backre.futura ac.3,b.abc	PCS	1	5707292291097
33	895131-71US	Cover,backcushion,futura activ	PCS	1	5707292291103
33	895131-72	Cover,backre.futura ac.3,b.hop	PCS	1	5707292291110
33	895131-73	Cover, back cushion size 3	PCS	1	5707292367167
33	895131-78	Cover,backre.futura ac.3,g.abc	PCS	1	5707292291127
33	895131-79	Cover,backre.futura ac.3,g.hop	PCS	1	5707292291134
33	895131-84	Cover, back cushion size 3	PCS	1	5707292368508
33	895132-65	Cover,backcush.+5cm,active 3	PCS	1	5707292496867
33	895132-66	Cover,backcush.+5cm,active 3	PCS	1	5707292502292
33	895132-71	Cover,backcush.+5cm,active 3	PCS	1	5707292291141
33	895132-72	Cover,backcush.+5cm,active 3	PCS	1	5707292291158
33	895132-73	Cover, back cushion +5cm, sz 3	PCS	1	5707292367204
33	895132-78	Cover,backcush.+5,active 3	PCS	1	5707292291165
33	895141-65	Cover,backre.futura ac.4, red	PCS	1	5707292496294
33	895141-65US	Betræk.ryghyn.futura akt.str 4	PCS	1	5707292550743
33	895141-66	Cover,backre.futura ac.4,blue	PCS	1	5707292498090
33	895141-66US	Cover,backre.futura ac.4,blue	PCS	1	5707292551184
33	895141-71	Cover,backre.futura ac.4,b.abc	PCS	1	5707292291301
33	895141-71US	Cover,backcushion,futura activ	PCS	1	5707292291325
33	895141-72	Cover,backre.futura ac.4,b.hop	PCS	1	5707292291332
33	895141-78	Cover,backre.futura ac.4,g.abc	PCS	1	5707292291363
33	895141-79	Cover,backre.futura ac.4,g.hop	PCS	1	5707292291387
33	895142-65	Cover,backcush.+5,act.4, red	PCS	1	5707292496874
33	895161-84	BETRÆK.RYGHY FUTURA AKT.STR.2½	PCS	1	5707292454164
33	895161-79	Cover,backr.futura ac.2½,g.hop	PCS	1	5707292291622
33	895161-78	Cover,backr.futura ac.2½,g.abc	PCS	1	5707292291615
33	895161-73	BETRÆK.RYGHY FUTURA AKT.STR.2½	PCS	1	5707292367150
33	895161-72	Cover,backr.futura ac.2½,b.hop	PCS	1	5707292291608
33	895161-71US	Cover,backcushion,futura activ	PCS	1	5707292291592
33	895161-71	Cover,backr.futura ac.2½,b.abc	PCS	1	5707292291585
33	895161-70	BETRÆK.RYGHY.FUTURA AKT.STR.2½	PCS	1	5707292367945
33	895161-66	Cover back cush fut. Act 2½	PCS	1	5707292497413
33	895161-65US	Cover,backr.futura ac.2½, red	PCS	1	5707292541116
33	895161-65	Cover,backr.futura ac.2½, red	PCS	1	5707292496270
33	895142-73	Cover backcush.+5,active 4	PCS	1	5707292365996
33	895142-72	Cover,backcush.+5,active 4	PCS	1	5707292291424
33	895142-71	Cover,backcush.+5,act.4,bl abc	PCS	1	5707292291417
33	895142-66	Cover,backcush.+5,act.4, blue	PCS	1	5707292502322

33	895162-65	Cover backcush.+5,active 2½	PCS	1	5707292496904
33	895162-66	Cover backcush.+5,active 2½	PCS	1	5707292527172
33	895162-71	Cover backcush.+5,active 2½	PCS	1	5707292291639
33	895162-72	Cover backcush.+5,active 2½	PCS	1	5707292291646
33	895162-73	BETRÆK.RYGH.5CM FUT.AKT.STR.2½	PCS	1	5707292367198
34	895121-60S	Cover for back side,act 2+2½+3	SET	1	5707292496546
34	895121-84S	Betr.rygside.fu.akt.str.2+2½+3	SET	1	5707292368515
34	895121-78S	Cover,back-side cush.green abc	SET	1	5707292290960
34	895121-73S	BETR.RYGSIDEHYND.FU.AKT.2+2½+3	SET	1	5707292367143
34	895121-71S	Cover for back side,act 2+2½+3	SET	1	5707292290922
34	895121-79S	Cover, backside cush,sz 2+2½+3	SET	1	5707292290984
34	895121-72S	Cover,backside cush,sz.2+2½+3	SET	1	5707292290946
34	895141-60S	Cover for backsidecushion sz.4	SET	1	5707292496553
34	895141-71S	Cover for backsidecushion sz.4	SET	1	5707292291318
34	895141-72S	Cover, backsidecushion size 4	SET	1	5707292291349
34	895141-78S	Cover,back-side cush.green abc	SET	1	5707292291370
34	895141-79S	Cover, backside cush.size 4	SET	1	5707292291394
34	895121-70S	BETR.RYGSIDE.FU.AKT.STR.2+2½+3	SET	1	5707292367952
34	895141-73S	Cover, backsidecushion size 4	SET	1	5707292291356
35	896131-60S	Cover f/seatsidecush.fut.sz 3	SET	1	5707292496485
35	896121-84S	Cover, seatsidecush, size 2	SET	1	5707292438348
35	896121-78S	Cover f/seatsidecush.fut.sz.2	SET	1	5707292294494
35	896121-73S	Cover, seatsidecush, size 2	SET	1	5707292367051
35	896121-71S	Cover f/seatsidecush.fut.sz. 2	SET	1	5707292294456
35	896121-60S	Cover f/seatsidecush.fut.sz. 2	SET	1	5707292496454
35	896121-79S	Cover f/seatsidecush.fut.sz 2	SET	1	5707292294517
35	896121-72S	Cover f/seatsidecush.fut.sz. 2	SET	1	5707292294470
35	896131-71S	Cover f/seatsidecush.fut.sz 3	SET	1	5707292294647
35	896131-72S	Cover f/seatsidecush.fut.sz 3	SET	1	5707292294678
35	896131-73S	Cover p.futura seats.3,red abc	SET	1	5707292294692
35	896131-78S	Cover f/seatsidecush.fut.sz. 3	SET	1	5707292294715
35	896131-79S	Cover f/seatsidecush.fut.sz 3	SET	1	5707292294739
35	896131-84S	Cover, seatsidecush, size 3	SET	1	5707292368539
35	896141-60S	Cover f/seatsidecush.fut.sz 4	SET	1	5707292366207
35	896141-71S	Cover f/seatsidecush.fut.sz 4	SET	1	5707292295910
35	896141-72S	Cover f/seatsidecush.fut.sz 4	SET	1	5707292295941
35	896141-78S	Cover f/seatsidecush.fut.sz 4	SET	1	5707292295965
35	896141-79S	Cover f/seatsidecush.fut.sz 4	SET	1	5707292295989
35	896161-84S	Cover,seatsidecush.size 2½	SET	1	5707292438409
35	896161-79S	Cover f/seatsidecush.fut.sz 2½	SET	1	5707292297112
35	896161-78S	Cover f/seatsidecush.fut.sz.2½	SET	1	5707292297099
35	896161-73S	Cover for seatside cush. Sz.2½	SET	1	5707292357014
35	896161-72S	Cover f/seatsidecush.fut.sz 2½	SET	1	5707292297075
35	896161-71S	Cover f/seatsidecush.fut.sz 2½	SET	1	5707292297044
35	896161-70S	Cover seat cushion size2½	SET	1	5707292367976
35	896161-60S	Cover f/seatsidecush.fut.sz 2½	SET	1	5707292496478
35	896141-73S	Cover f/seatsidecush.fut.sz 4	SET	1	5707292361547
37	5800030	Foam,seat-cushion 4cm. Panda 3	PCS	1	5707292181657
37	5800785	Foam,seat 3 cm, futura size 3	PCS	1	5707292183132
37	5800780	Foam, Seat 3 cm futura size 2	PCS	1	5707292183125



37	5800020	Foam,seatcushion 4cm. Panda 2	PCS	1	5707292181633
37	5800790	Foam, Seat 3 cm futura size 4	PCS	1	5707292183149
37	5800040	Foam,seat-cushion 4cm. Panda 4	PCS	1	5707292181671
37	5800795	Foam, Seat 3 cm futura size 2½	PCS	1	5707292183156
37	5800060	Foam seatside 4cm Panda 2½	PCS	1	5707292181718
38	5800812	Foam, back 3 cm futura size 3	PCS	2	5707292183248
38	5800809	Foam, back 5 cm futura size 2	PCS	1	5707292183217
38	5800807	Foam, back 3 cm futura size 2	PCS	1	5707292183200
38	5800814	Foam, back 5 cm futura size 3	PCS	1	5707292183255
38	5800817	Foam, back, futura active 4	PCS	1	5707292183286
38	5800819	Foam back5cm,futura act. 4	PCS	1	5707292183293
38	5800822	Foam, back 3 cm futura size 2½	PCS	1	5707292183323
38	5800824	Foam back5cm,futura act.2½	PCS	1	5707292183330
39	5800324	Foam backside 3 cm, p.activ 2	PCS	1	5707292182296
39	5800344	Foam back-side cush.3cm, activ	PCS	1	5707292182418
40	5800323	Foam,backside 3cm p.active 2	PCS	1	5707292182289
40	5800343	Foam back-side cush.3cm,activ	PCS	1	5707292182401
41	5800885	Foam seat side	PCS	4	5707292183743
41	5800870	Foam, right Seat side 3 cm	PCS	1	5707292183620
41	5800867	Foam,seats.right,5cm,pan.fut.2	PCS	1	5707292183606
41	5800865	Foam,seat side,right 3cm, sz 2	PCS	1	5707292183583
41	5800888	Foam seat side Pan.2,usa	PCS	4	5707292369048
41	5800872	Foam,seats.right,5cm,pan.fut.3	PCS	1	5707292183644
41	5800875	Foam, right Seat side 3 cm,sz4	PCS	1	5707292183668
41	5800886	Foam seat side	PCS	4	5707292183750
41	5800877	Foam,seats.right,5cm,pan.fut.4	PCS	1	5707292183682
41	5800880	Foam,seat side,right 3cm,sz 2½	PCS	1	5707292183705
41	5800887	Foam seat side Pan. 2½,usa	PCS	4	5707292183767
41	5800882	Foam,seats.right,5cm,pan.fut2½	PCS	1	5707292183729
42	5800871	Foam, left Seat side 3 cm	PCS	1	5707292183637
42	5800868	Foam,seats.left,5cm,pand.fut.2	PCS	1	5707292183613
42	5800866	Foam, Seat side left, 3cm,sz 2	PCS	1	5707292183590
42	5800873	Foam,seats.left,5cm,pand.fut.3	PCS	1	5707292183651
42	5800876	Foam, Seat side left, 3 cm,sz4	PCS	1	5707292183675
42	5800878	Foam,seats.left,5cm,pand.fut.4	PCS	1	5707292183699
42	5800881	Foam,seat side,left, 3cm,sz 2½	PCS	1	5707292183712
42	5800883	Foam,seats.left,5cm,pan.fut.2½	PCS	1	5707292183736
43	5800890	Lombar support, upholstered	PCS	1	5707292183774
43	5800893	Uphols. Lumbar support sz.2+2½	PCS	1	5707292183804
43	5800891	Lombar support std 3 + act 4	PCS	1	5707292183781
44	0631862	Insex uh. 5x12, FZB	PCS	2	5707292352293
45	0611200	Insex screw counters. 6*12	PCS	2	5707292138699