## Monteringsvejledning (DK)

Montering af bord med balje

## Monteringsveiledning (N)

Montering av bord med balje

## Monteringsanvisning (SV)

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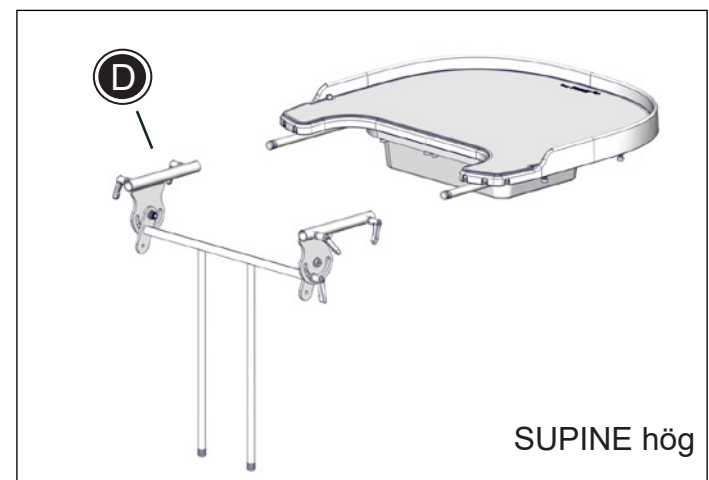
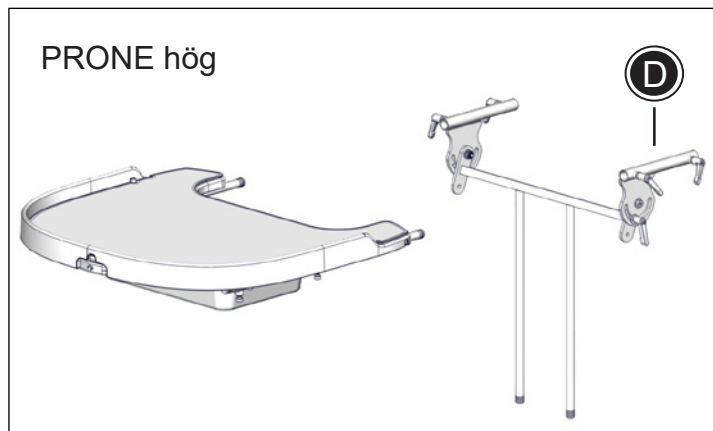
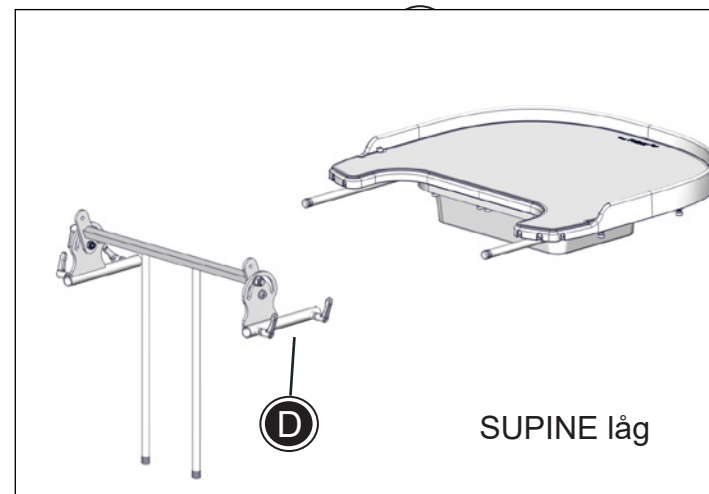
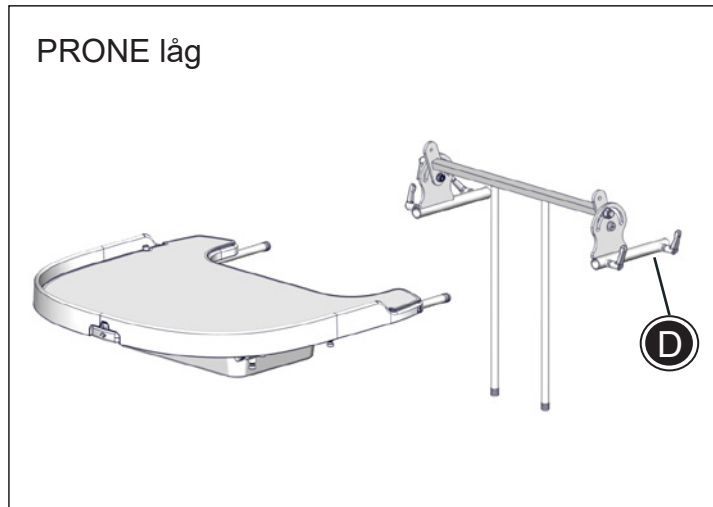
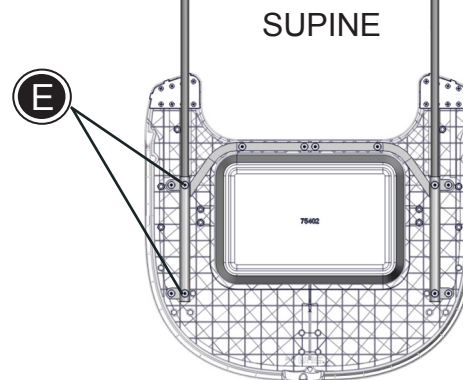
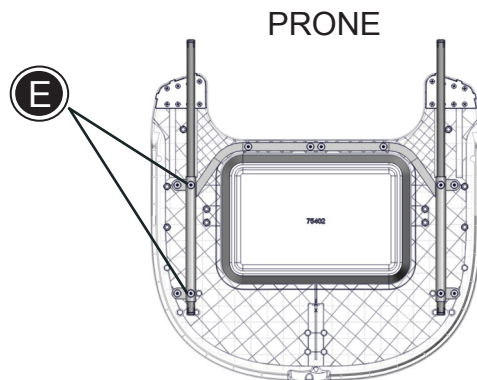
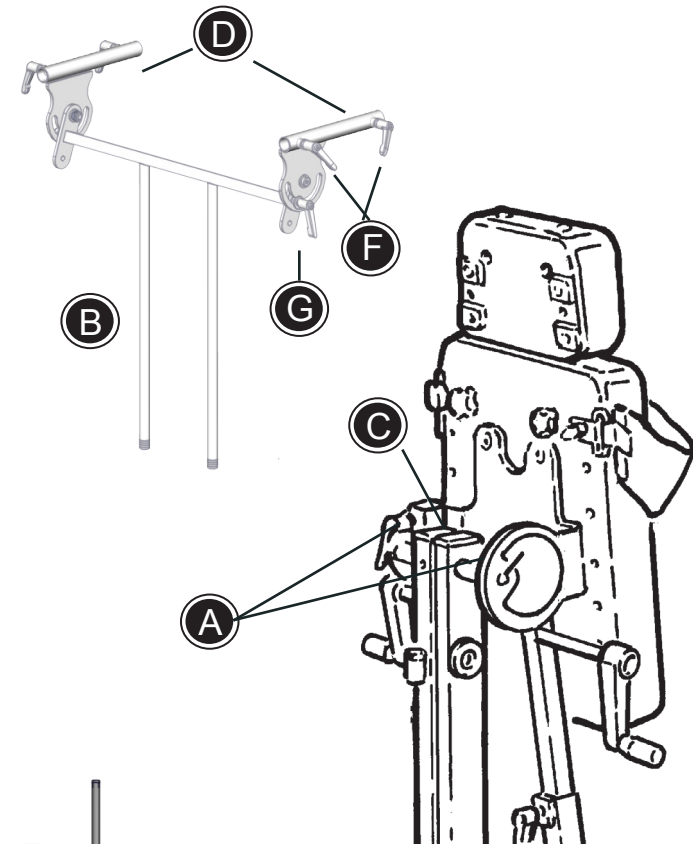
R82<sup>®</sup>

## Mounting tray with bowl - 1/3

The tray can be mounted in 4 positions - prone low, prone high, supine low and supine high.

At mounting use the enclosed 5 mm Allen key.

- \* The handles (A) are loosened completely, and the tray bracket (B) is placed in the two tubes (C) on the back of Gazelle PS. Fasten the handles (A) again. **Always** place the tray bracket in a position where the long ends on the horizontal tubes (D) turn towards the user.
- \* The bolts (E) are loosened in both sides, the tray bars are put in position for prone or supine mounting as shown on drawings below. The bolts (E) are fastened loosely.
- \* Handles (F) on both sides are loosened, the tray bars are mounted in the horizontal tubes and handles (F) are fastened again. The bolts (E) are fastened completely with Allen key when the desired position has been achieved.
- \* Loosen handles (A) to adjust the height of the tray. Loosen handle (G) to adjust the angle of the tray (the handle (G) should never be removed completely as the parts can be hard to assemble again).



# Mounting instruction

R82<sup>®</sup>

## Mounting tray with bowl - 2/3

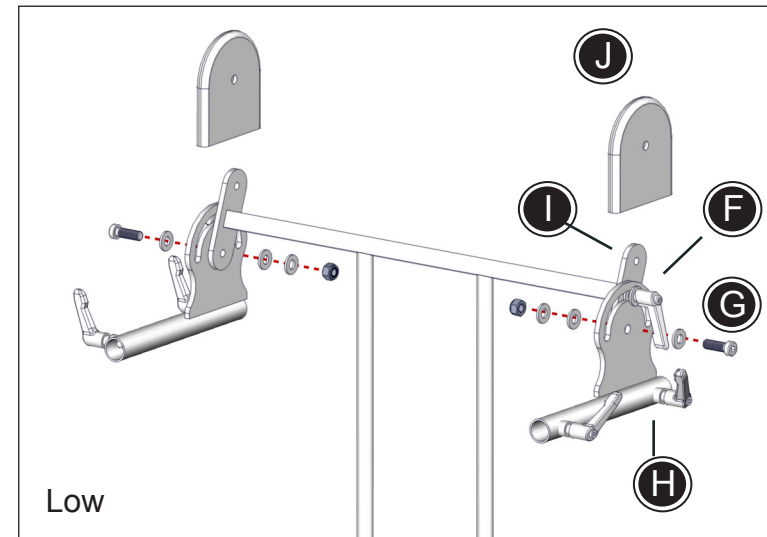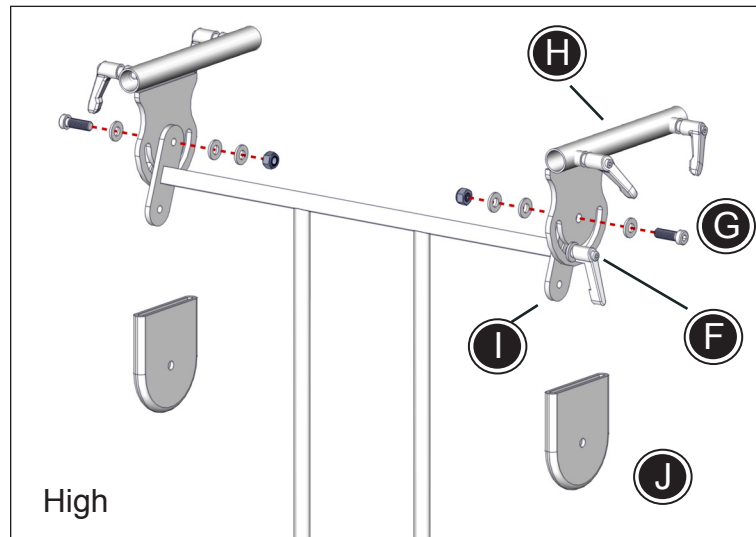- \* The tray can be adjusted in two positions - high and low. To do this remove bolt, washers and nut (G) in both sides, the handle (F) is loosened a little (is not to be removed completely), and the bracket (H) is turned 180 degrees. Bolt, washers and nut are mounted at the other end of the bracket (I).
- \* The bracket can be covered by the enclosed fabric hood (J).





# Mounting instruction

R82<sup>®</sup>

## Mounting tray with bowl - 3/3

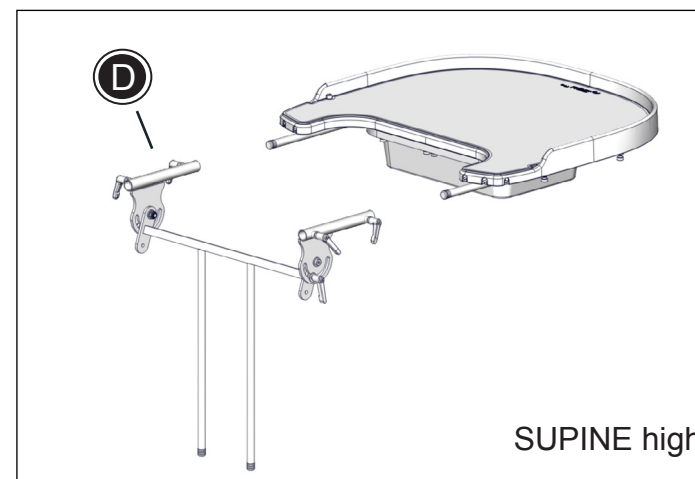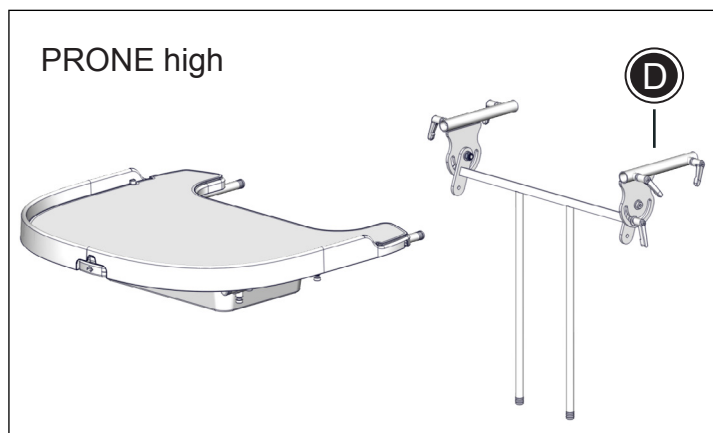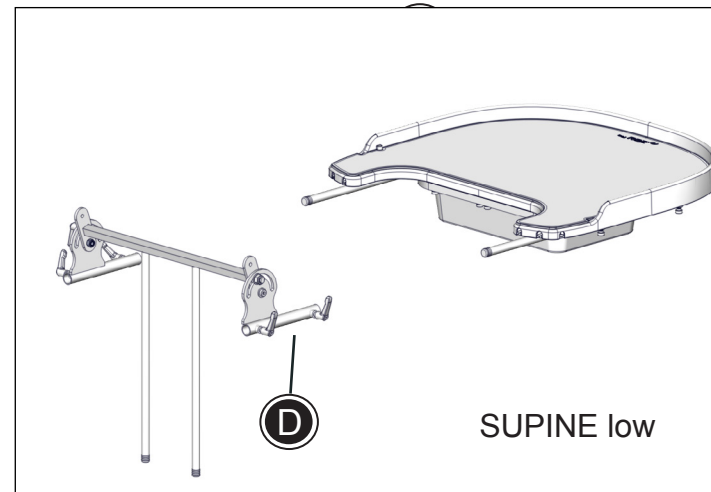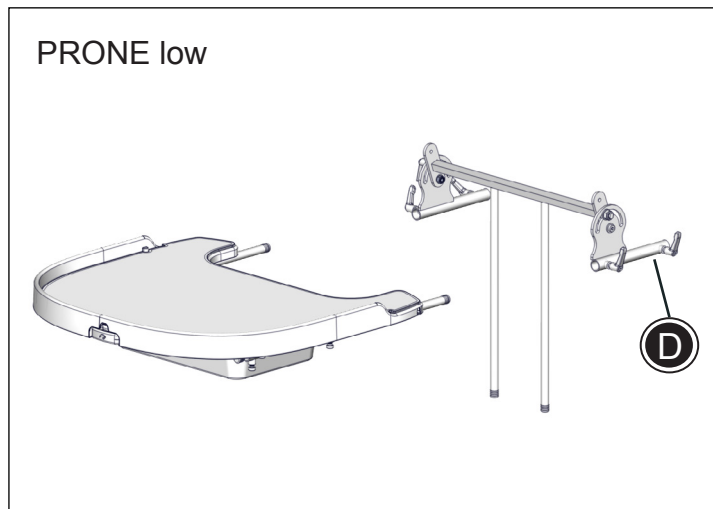Below please find the various possibilities for mounting of the tray. Please note that the horizontal tubes (D) **always** should be placed in a position where the long ends turn towards the user.

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