## **Monteringsvejledning (DK)**

Montering af R82 brystsele - polstret

## **Monteringsanvisning (SV)**

Montera R82 H-sele – polstrad

## **Monteringsveiledning (N)**

Montering av R82 brystsele - polstret

## **Mounting instruction (GB)**

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




Producer  
R82 A/S  
Parallevej 3  
DK-8751 Gedved

# Mounting instruction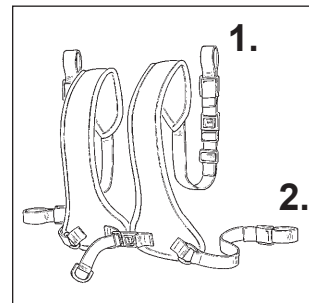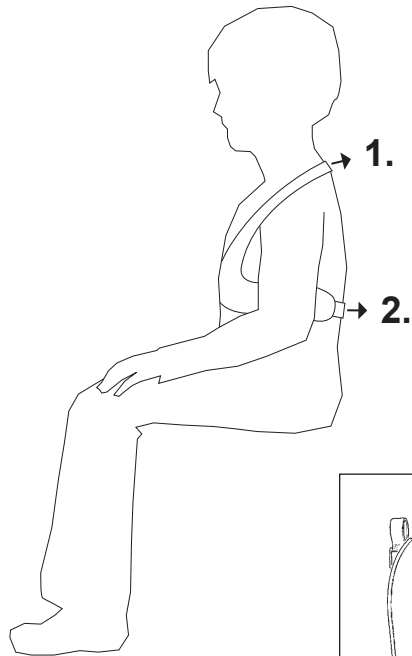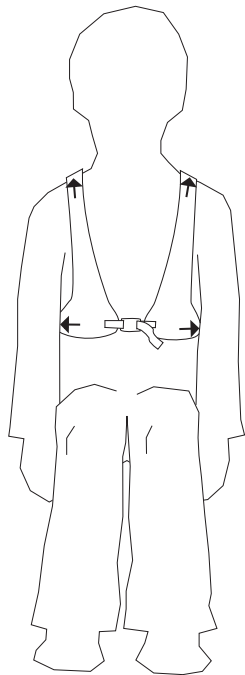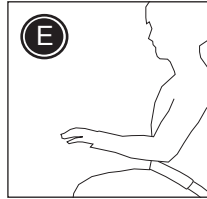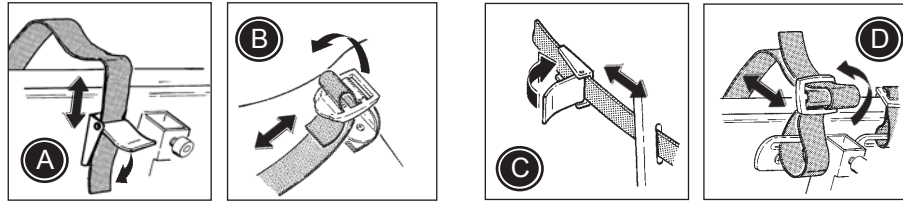

## How to mount the R82 Chest belt - upholstered

Use the fittings (A) and (B) on the chair to mount the chest belt. If no fittings are mounted, please use the enclosed fix locks and belt fittings to mount the chest belt as shown on picture (C) and (D). Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

- \* Always mount the shoulder straps (1) a little higher than the shoulder height of the user.
- \* Mount the lower straps (2) beneath the user's arms.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The chest belt is fitted correctly when two fingers can be put between the harness and the child's body. Check before every use.

 To obtain the best seating position and to prevent the user from sliding down and out of the chair, we recommend always to mount a hip belt (E) together with the chest belt.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.

 Carefully read the care label on the harness to maintain and wash the chest belt correctly.

 The latest version of the mounting instruction is always available online: [R82.org](http://R82.org)

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