---

## Monteringsvejledning (DK)

Montering af R82 abduktionssele

## Monteringsanvisning (SV)

Montering av R82 abduktionssele

## Monteringsveiledning (N)

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## Mounting instruction (GB)

How to mount the R82 Abduction belt

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
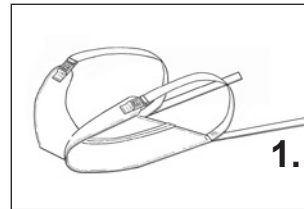
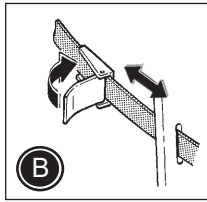
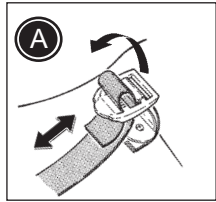
## How to mount the R82 Abduction belt

**R82**<sup>®</sup>  
by Etac

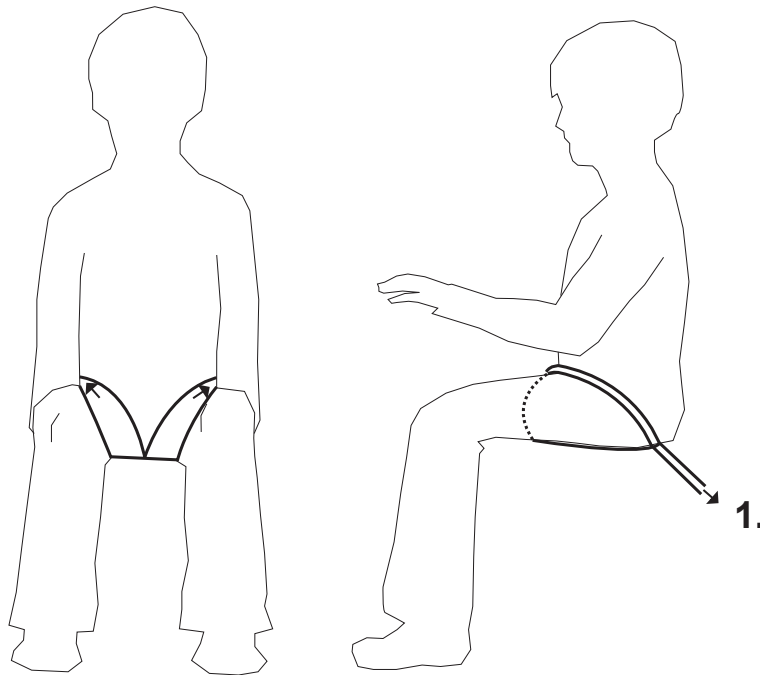Use the fittings (A) on the chair to mount the abduction belt. If no fittings are mounted, please use the enclosed fix locks and belt fittings (B) to mount the abduction belt as shown on picture. Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

- \* Mount the lower straps (1) in the back of the seat.
- \* The other straps is mounted at the front of the belt.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The abduction belt is fitted correctly when two fingers can be put between the harness and the child's body. Check before every use.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.





 Carefully read the care label on the harness to maintain and wash the abduction belt correctly.

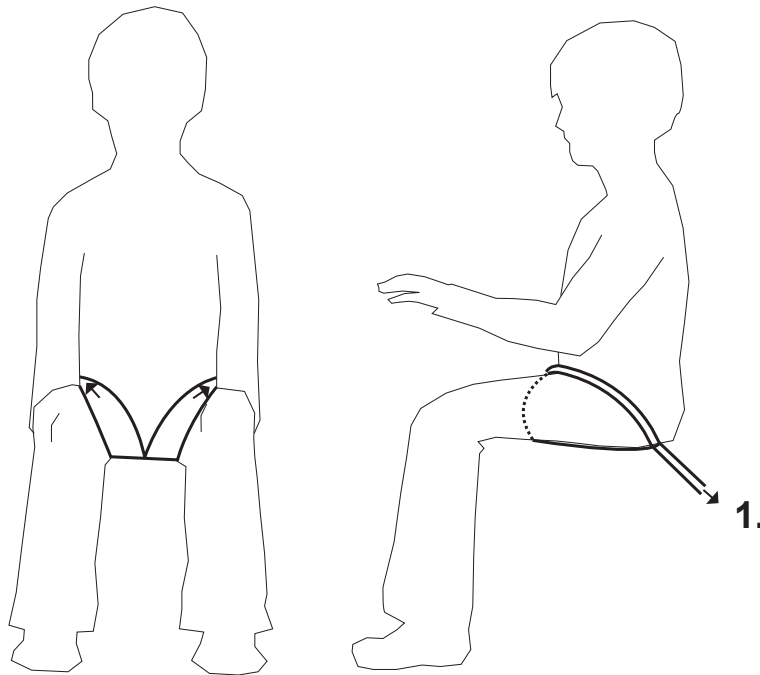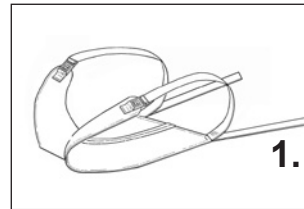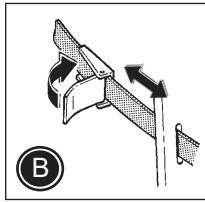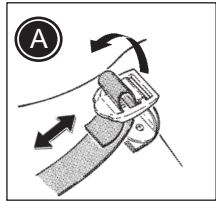 The latest version of the mounting instruction is always available online: [www.etac.com](http://www.etac.com)

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