



# Etac My-Loo

Raised toilet seat,  
with fixed mounting

 **etac**<sup>®</sup>

# Etac My-Loo

With a unique, patented arm support solution, a well-designed seat and quick, simple installation, My-Loo with fixed mounting offers safe support, a high level of comfort and ease of handling. The raised toilet seat is suitable for most toilets and sits securely in place.

## Developed to give independence with dignity

My-Loo enables users with reduced mobility or balance problems to manage toilet visits independently and safely. The raised seat and stable armrests make it easier to sit down and stand up from the toilet. An extra-large seat opening and generous hygiene recesses make personal hygiene easier.

## Available in five different versions

Etac My-Loo with fixed mounting is available in different designs, with or without armrests and in seat heights of 2, 6 and 10 cm. If necessary, armrests can be added later to a model without armrests.

My-Loo 2 cm is the perfect arm support solution for around the toilet if you want to avoid drilling into the walls or floor.

With its modern, discreet design, My-Loo blends in beautifully with most bathroom environments.



Etac My-Loo raised toilet seat with fixed mounting.		Item no.	Material
My-Loo with armrests, 20 mm		80301510	Armrest: Polypropylene
My-Loo with armrests, 60 mm		80301511	Seat/Lid: Polypropylene
My-Loo with armrests, 100 mm		80301512	Attachment: Stainless steel, Polypropylene, TPE
My-Loo, 60 mm		80301513	
My-Loo, 100 mm		80301514	

Weight	
My-Loo with armrests, 20 mm: 3.2 kg	My-Loo, 60 mm: 2.2 kg
My-Loo with armrests, 60 mm: 3.4 kg	My-Loo, 100 mm: 2.4 kg
My-Loo with armrests, 100 mm: 3.7 kg	

Maximum user weight
150 kg
Together with accessory Supporting legs, 80301603: 200 kg

Design
Myra Industriell Design AB

Maintenance

Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution. The product can be safely cleaned in a cabinet washer-disinfector at 85 °C for 3 minutes.

## Features



### Adjustable armrests

Using a unique, patented solution, the armrests can quickly and easily be adjusted in height. They are easily folded up and down with no pinching risk for the user. If needed My-Loo can be used with only one armrest.



### Simple, convenient installation

Etac My-Loo is installed quickly and easily using an innovative new solution. Practical, quick nuts make it possible to attach the raised toilet seat from above. The seat and armrests can then be clicked smoothly into their brackets.



### Spacious and practical

Generous recesses and an extra-large seat opening provide excellent accessibility. The raised toilet seat's design, with free space around the seat, provides plenty of room for all body shapes.



### Stable

My-Loo fixed sits securely in place and has a max user weight of 150 kg. For additional support and increased weight capacity, the accessory supporting legs can be added (max user weight 200 kg).



### Safe and comfortable

With stable, easy-grip armrests and a well-designed seat, My-Loo makes toilet visits safe and comfortable for the user. For extra comfort, a soft seat can be added.



### Easy to clean

The clean, smooth surfaces are easy to wipe with a soft cloth. Should separate cleaning be required, the seat and armrests are easy to remove from their brackets and equally simple to put back in place.



My-Loo with armrests  
2 cm



My-Loo with armrests  
6 cm



My-Loo with armrests  
10 cm




My-Loo  
6 cm



My-Loo  
10 cm

## Dimensions

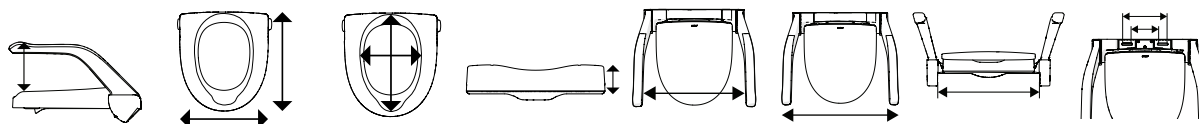

Model	Height armrests	Width/Length	Inner dimensions	Height	Width between armrests	Total width	Attachment width (max toilet width)	Dimensions between holes
80301510	215/245/285 mm	360/400 mm	215/285 mm	20 mm	480 mm	570 mm	400 mm	c/c 123-203 mm
80301511	175/205/245 mm	360/400 mm	230/360 mm	60 mm	480 mm	570 mm	400 mm	c/c 123-203 mm
80301512	135/165/205 mm	360/400 mm	230/360 mm	100 mm	480 mm	570 mm	400 mm	c/c 123-203 mm
80301513	-	360/400 mm	230/360 mm	60 mm	-	-	400 mm	c/c 123-203 mm
80301514	-	360/400 mm	230/360 mm	100 mm	-	-	400 mm	c/c 123-203 mm

## Accessories



### Soft seat

80301604  
Ergonomically shaped, soft and comfortable. Provides excellent pressure relief and comfortable support.  
Material: Polyurethane



### Top-mounted screw set

80301626  
For toilets without through mounting holes (blind fitting). Should only be used with the included supporting legs. Suitable for mounting holes Ø20 mm. Material: Aluminium, polyamide, stainless steel, EPDM



### Supporting legs

80301603  
For additional stability. Increases max. user weight to 200 kg. Foldable.  
Material: Aluminium, polyamide, polypropylene, TPE, stainless steel




### Toilet paper holder

80301601  
Attaches to the armrest.  
Material: Polyamide, stainless steel



### Storage basket

80301602  
Attaches to the armrest.  
Material: Polyamide, polyester/mesh, stainless steel

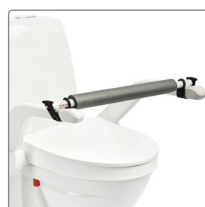

### Cross bar

80303034  
A secure support to hold on to. Attaches to the armrests. 60.5 cm  
Material: Aluminium, polyethylene



### Armrests

80301606  
Sold in pairs, right, left version.  
Material: Polypropylene

Etac is a world-leading developer and provider of ergonomic assistive devices and patient handling equipment. Our heart lies in the solutions that optimise quality of life for the individual, their family and caregivers.

For the latest news and continuously updated product information – visit: [www.etac.com](http://www.etac.com)