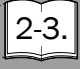
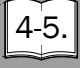
Etac My-Loo

Rekonditioneringsanvisning

Reconditioning instruction

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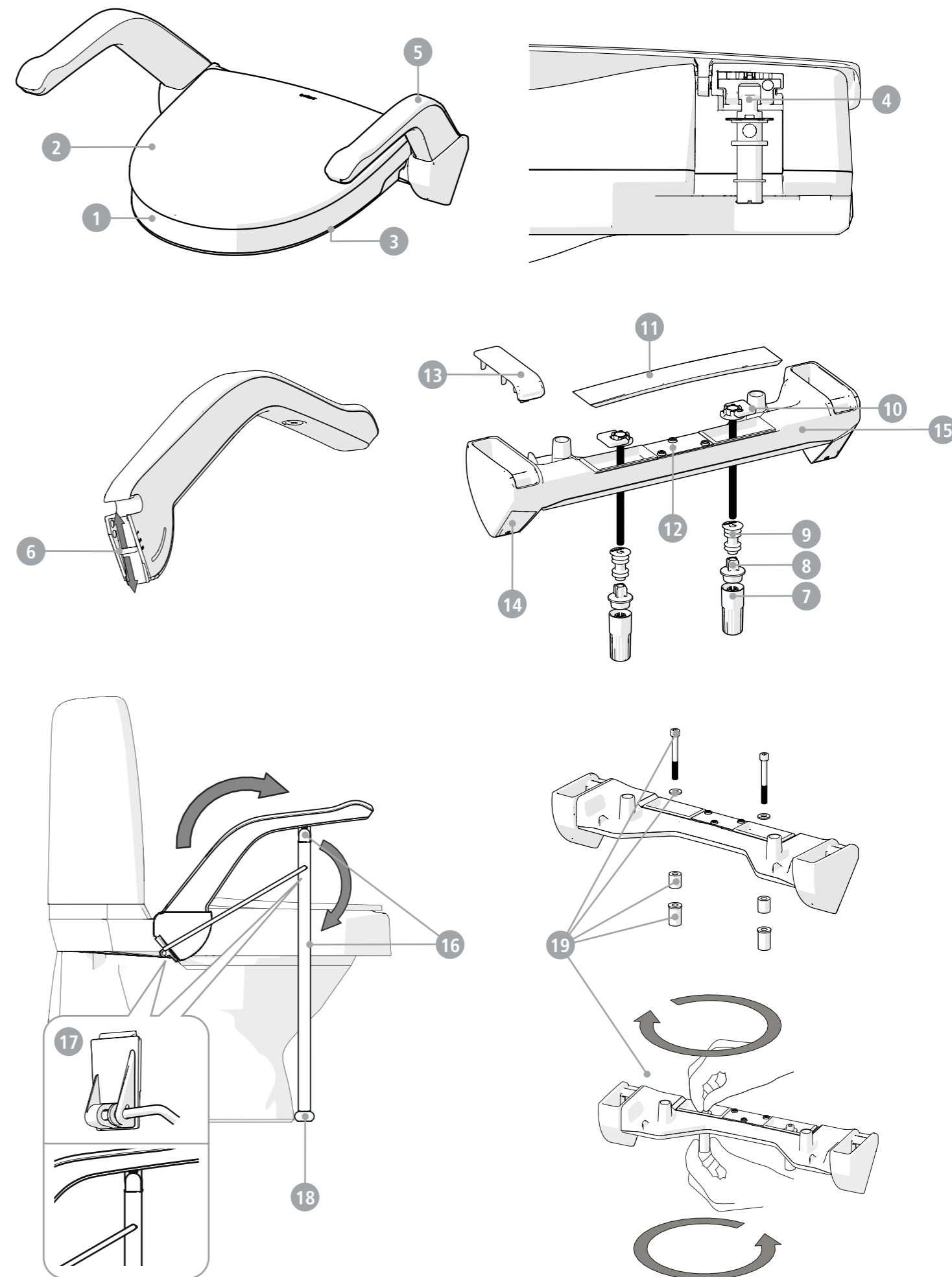
General

- Etac My-Loo may be further used.
- Original spare parts from Etac should always be used.
- Remember that the service life of the product is limited (see

manual). This is particularly important when prescribing the product to a user who is expected to use it for an extended period of time.

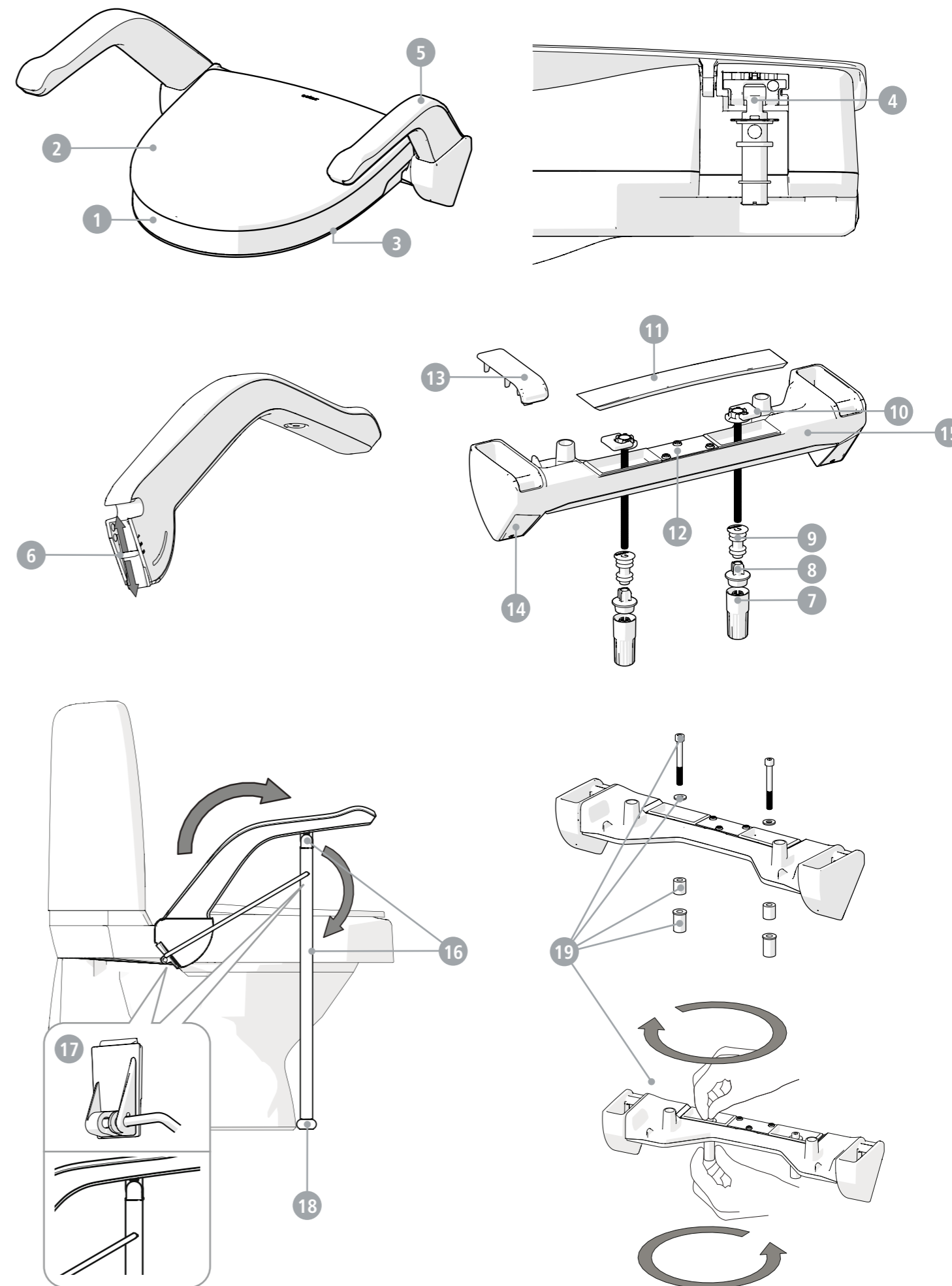
Washing

- Clean the product with normal nonabrasive cleaning agents (pH value of between 5-9). Rinse and dry. Disinfect the product with an alcohol-based disinfectant, 70%.

- The product can withstand cleaning in a sterilizer at 85 °C for 3 minutes.

①	Raiser
Check so that there are no cracks or damages.	
②	Lid
Check so that the lid is properly fixed and can be folded.	
Check so that there are no cracks or damages.	
③	Friction pieces
Check so that the friction pieces are properly fixed and that there are no damages.	
④	Raiser fixing
Check so that the raiser is fixed and can be folded up and down	
⑤	Arm supports
Check so that there are no cracks or damages.	
Check so that the arm support can be folded up and down	
⑥	Height control
Check so that there are no cracks or damages.	
Check so that the height control can be set in positions 1-3.	
⑦	Assembly nut
Check so that there are no cracks or damages.	
Check so that the assembly nut can be tightened.	
⑧	Sleeve
Check so that there are no cracks or damages.	
⑨	Bushing
Check so that there are no cracks or damages.	
⑩	Locking handle
Check so that there are no cracks or damages.	
Check so that the locking handle is fixed	
⑪	Cover
Check so that there are no cracks or damages.	
Check so that the cover can be clicked into place	

⑫	Cover holder
Check so that there are no cracks or damages.	
Check so that the cover can be clicked into place	
⑬	Cover, arm support holder
Check so that there are no cracks or damages.	
Check so that the cover can be clicked into place	
⑭	Holster cover
Check so that there are no cracks or damages.	
Check so that the cover is fixed	
⑮	Attachment
Check so that there are no cracks or damages.	
⑯	Support leg
Check so that the arm support can be folded	
Check so that the support leg attachment is ok and fixed	
⑰	Support leg, parallel rod
Check so that the parallel rod has no damages and is properly fixed	
Check so that the parallel rod attachment has no damages and is properly fixed	
⑱	Support leg, adjustable ferrule
Check so that the adjustable ferrule has no damages and can be adjusted	
⑲	Screw kit for top mounting
Check so that there are no cracks or damages in screws, washers, spacers and plugs.	
Check so that screw and plug can be mounted	

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