Star by Etac

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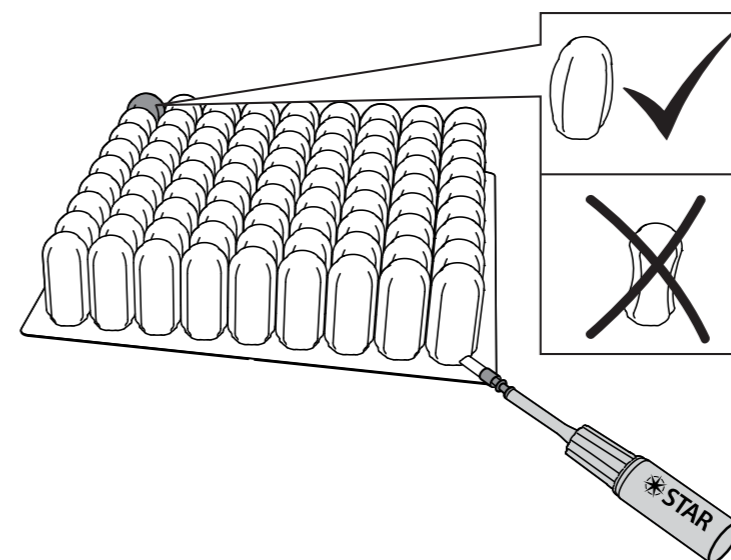
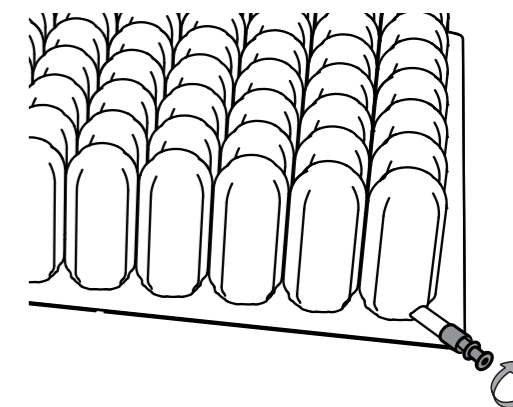
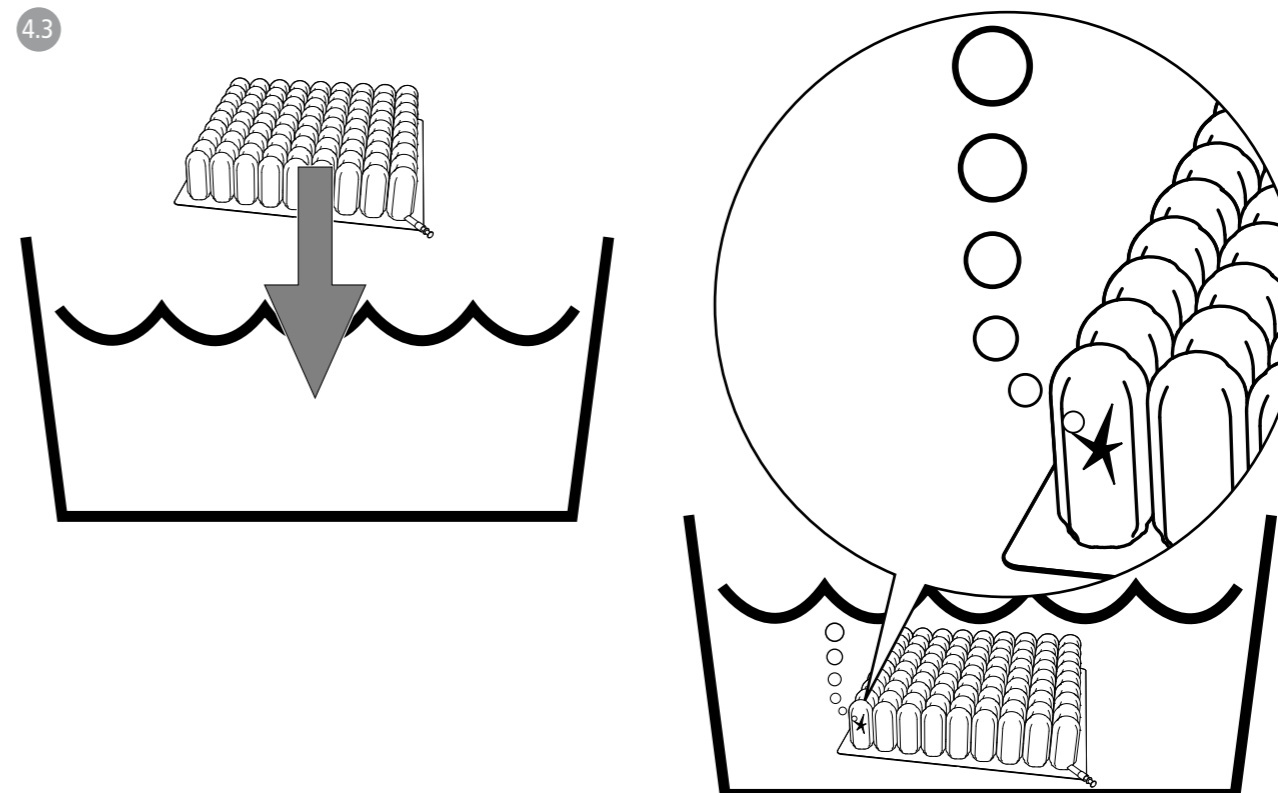
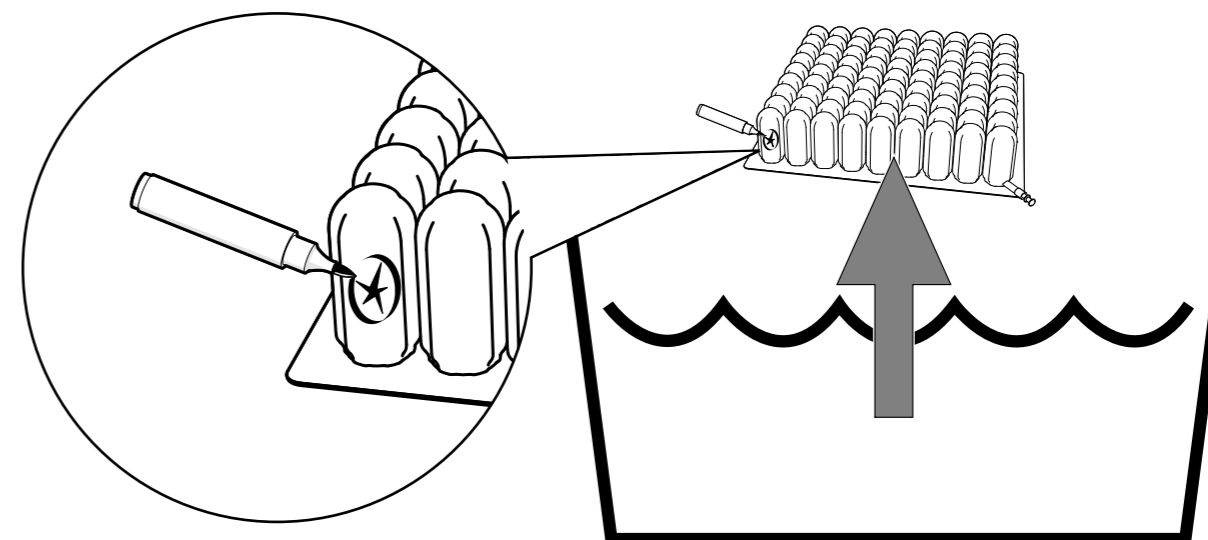
4.4

1. General

- StarLock, StabilAir and Standard Air can be re-used.
- Only official spare parts from Etac should be used.
- Remember that the service life of the product is limited (see

manual). This is particularly important when prescribing the product to a user who is expected to use it for an extended period of time.

2. Cleaning the cover

Machine washing:

Remove the cover, turn inside out and close the zip. Machine wash in warm water (maximum 60 °C for the standard cover, maximum 80 ° for the incontinence cover). Do not bleach. Rinse in cold water and tumble dry on low temperature or line dry.

Disinfection:

Standard Cover:

Standard cover can only be washed (see "Machine washing"). If disinfection is needed, we recommend to use incontinence cover.

Incontinence cover:

Use a 70% alcohol-based disinfectant solution or a Chlorine based solution, maximum 5,000 ppm. Rinse in cold water and tumble dry on low temperature or line dry.

3. Cleaning the cushion

Before cleaning or disinfecting the cushion, remove the cover and wash separately (see 'Cleaning the cover').

Handwash:

Inflate just until the cells contain air, then close the valve. Clean the cushion by hand with soap, bicarbonate or vinegar. Rinse in cold water. After rinsing, overinflate the cushion and leave to air dry for 24 hours, or dry in a drying cabinet on low heat.

Machine wash (StarLock and Standard Air only):

Inflate just until the cells contain air, then close the valve. Machine wash in 60 ° /140 ° F for maximum 10 min. Use a mild detergent. Rinse in cold water. After rinsing, overinflate the cushion and leave to air dry for 24 hours, or dry in a drying cabinet on low heat. N.B. The cushion must be filled with some air when washing in machine.

Note: Do not spin-dry or tumble dry. Can be dried in a drying cabinet on low heat.

StarLock - If there is water in the bulb pump after machine washing, remove the pump and drain the water. Mount the bulb pump again.

Mesh laundry bags that are big enough to loosely fit around the cushion, can be used if additional protection is desired during machine washing.

Disinfection:

Spot disinfection:

1. Wipe down the affected areas with a 70% alcohol-based disinfectant solution or a chlorine based solution, maximum 5000 ppm. Follow instructions of chemicals carefully.
2. Rinse thoroughly.

Complete product:

1. Wipe down with a 70% alcohol-based disinfectant solution or a chlorine based solution, maximum 5000 ppm. Follow instructions of chemicals carefully.
2. Rinse thoroughly.

Drying:

After rinsing, overinflate the cushion and leave to air dry for 24 hours, or dry in a drying cabinet on low heat.

4. Visual inspection (see pictures)

Visual Inspection:

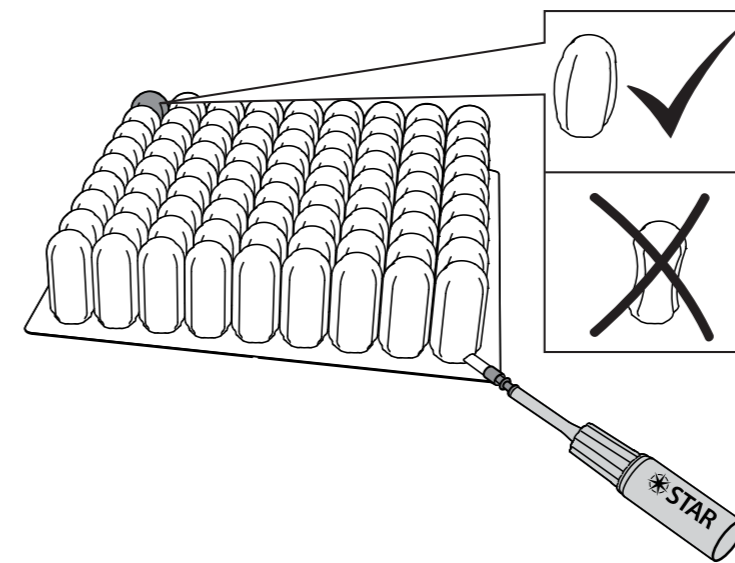
Inspect the cushion for indications of damage or puncture. If any air is lost over 24 hours, investigate the source of the leak. If the cushion has no signs of damage or leaks, then the cushion can be reboxed ready for reuse.

Check for leaks and damage:


- 4.1. Overinflate cushion.
- 4.2. Close valve.
- 4.3. Place cushion under water and watch for stream of air bubbles flowing from the cushion (this is the hole you need to patch).
- 4.4. Mark the damaged area with a permanent marker or a toothpick (punctures).

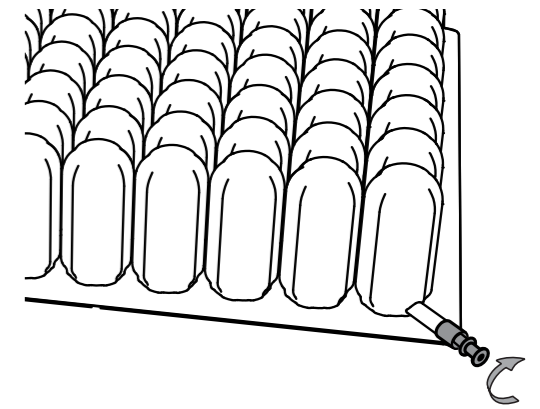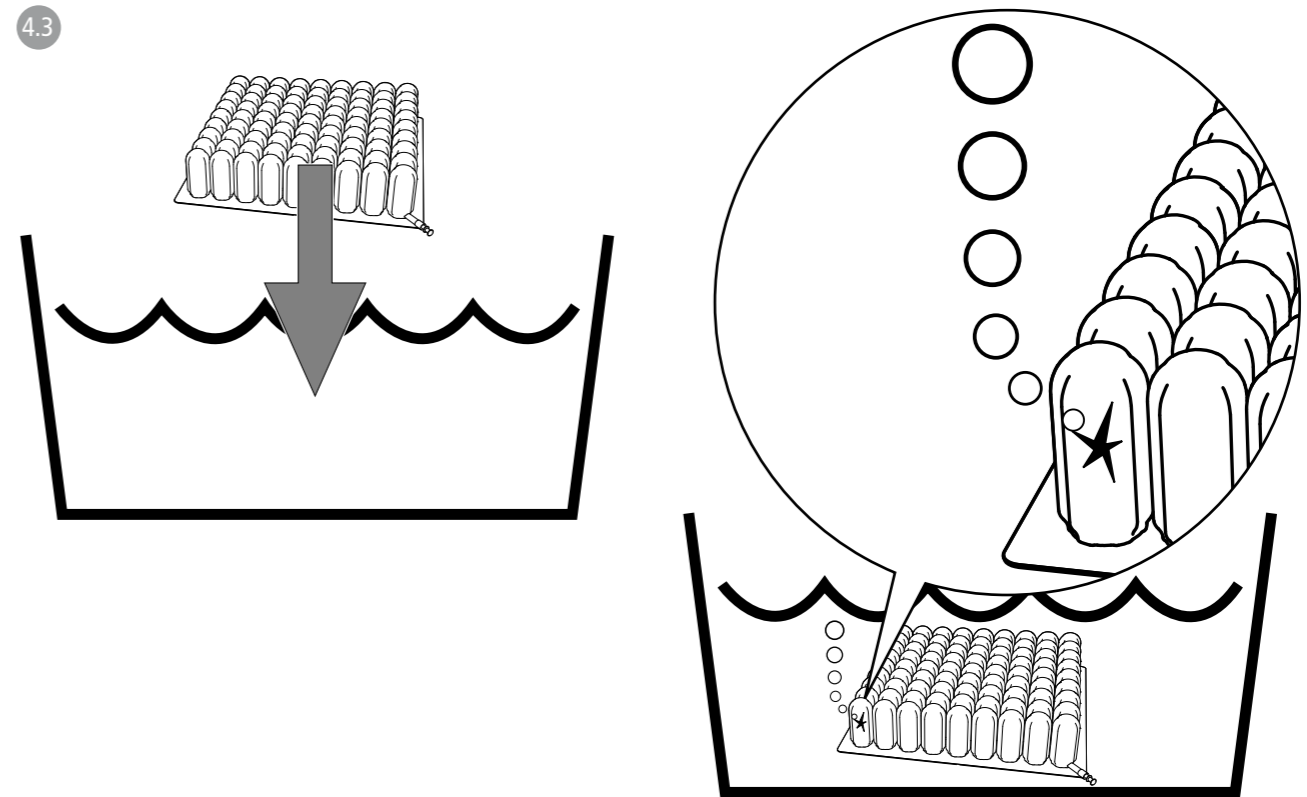
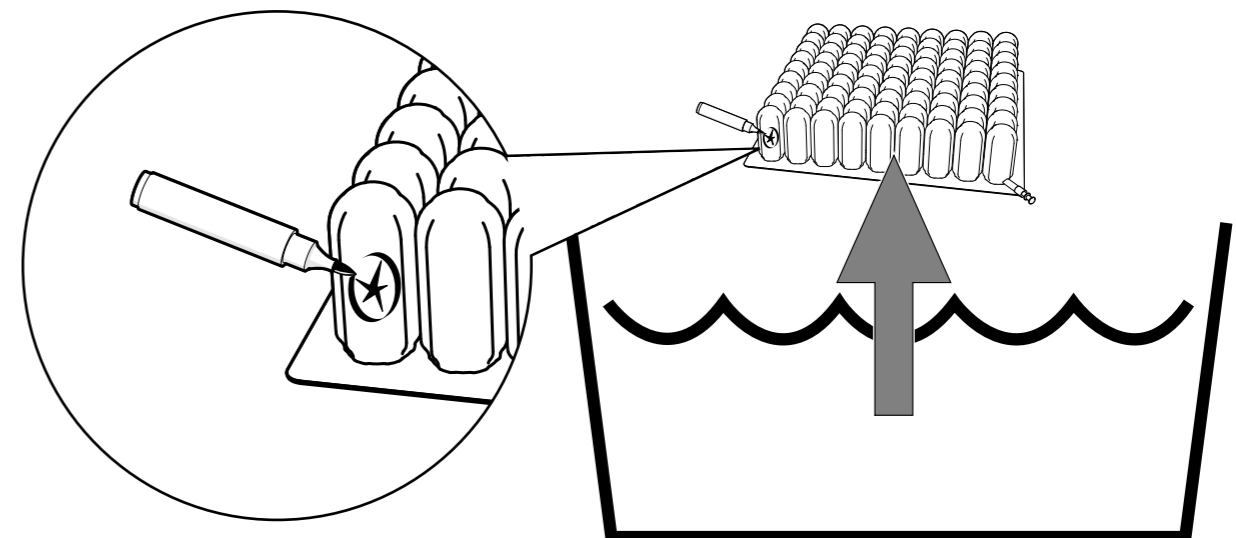
Depending on the type of damage, see repair instructions on www.etac.com for action.

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