

panda futura

size 5

Exploded views / Part lists


Rev. date (11.2018)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.



Contents


Standard size 5.....	3
Active size 5.....	6
Arm support, Back sides support, back extension and seat extension	8

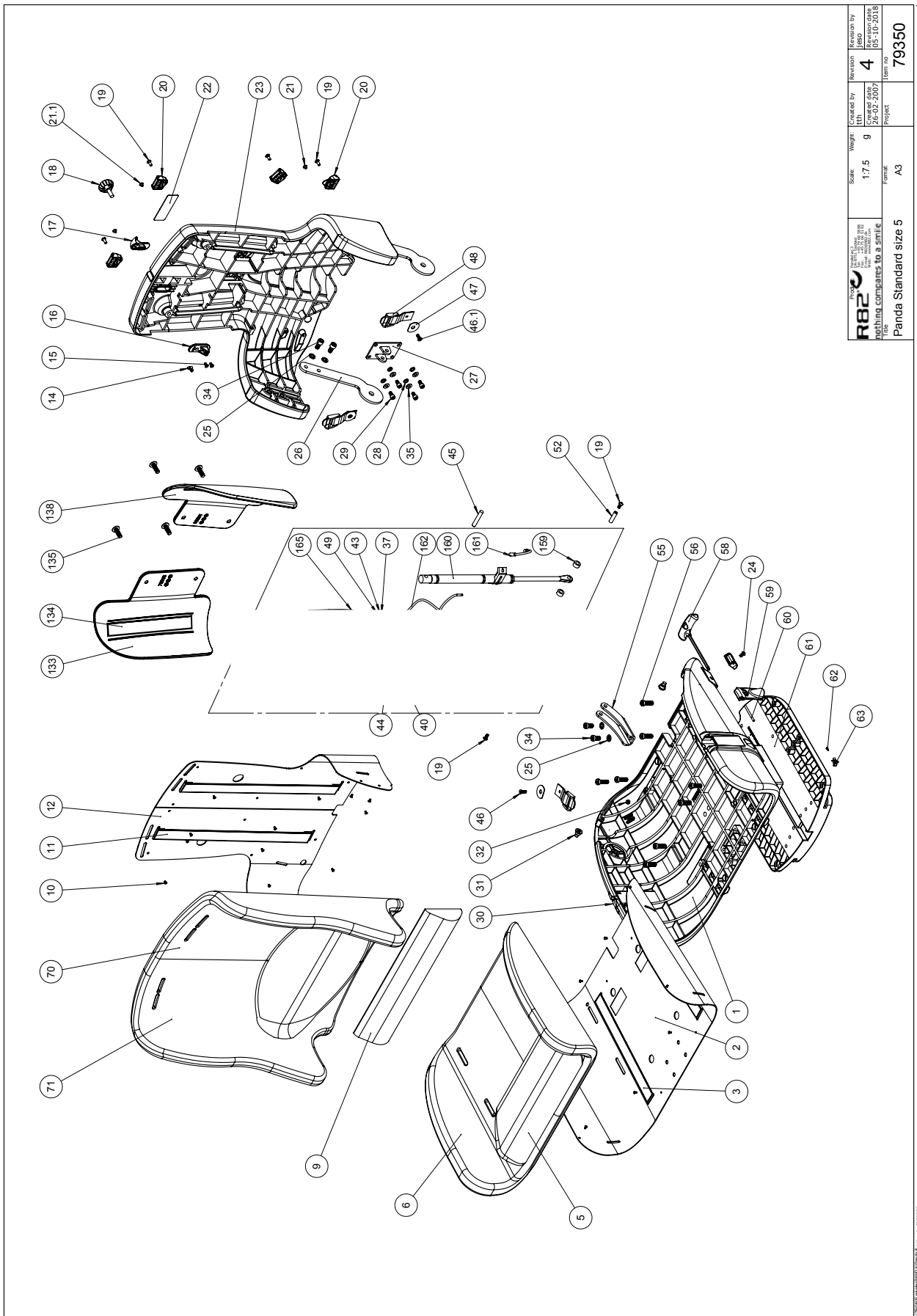

 R82 A/S Parallelvej 3 8751 Gedved Denmark	Scale: 1:7,5 Weight: 9 Form: A3	Created by: 26-02-2007 Lth: 26-02-2007 Rev: 05-10-2018	Revision by: 4 Item no: 79350
	Panda Standard size 5		Project:

Position	Art. No.	Description	Unit of measure	Quantity	Ean code
1	75505	Seat, Panda 5	PCS	1	5707292466648
2	75506	Cover plate Seat	PCS	1	5707292460288
3	4913040	30mm adhesive hook, black	PCS	1	5707292786517
3	4913035	30mm black ps14 with glue hook	PCS	1	5707292469755
5	5699990	Oplime Panda str.5,Sædehynde	PCS	1	5707292472472
5	89624-53	Foam Seat cushion 3 cm	PCS	1	5707292465283
6	89638-560	Cover/seat Panda 5, black	PCS	1	5707292472489
6	89638-585	Cover seat Panda 5	PCS	1	5707292472175
6	89638-591	Cover/seat Panda 5,blue abc	PCS	1	5707292461414
7	5699991	Oplime Panda 5,stand. Ryghynde	PCS	1	5707292472526
7	89626-53	Foam back 3 cm for size 5	PCS	1	5707292465290
8	89639-560	Cover for back, Panda 5	PCS	1	5707292472601
8	89639-585	Cover back panda std size5	PCS	1	5707292472199
8	89639-591	Cover/back Panda 5,blue abc	PCS	1	5707292461421
9	5700250	OPLIME LÆNDESTØTTE,PANTHER 1	PCS	1	5707292180902
10	1940010	Plastite 60, 2,9*6,5	PCS	22	5707292142573
11	4913040	30mm adhesive hook, black	PCS	1	5707292786517
11	4913035	30mm black ps14 with glue hook	PCS	1	5707292469755
12	75501	Cover plate back	PCS	1	5707292460370
14	0388010	Screwnippel 7*7mm	PCS	1	5707292136121
15	1940006	Eco syn 3,5*16	PCS	2	5707292474902
16	75514	Bund for udløserknap	PCS	1	5707292466655
17	75513	Handle for back adjustment	PCS	1	5707292466662
18	3501201	R82 handle M8*25mm screw	PCS	1	5707292413932
19	0640911	Button head M5x10 FZB tufflok	PCS	8	5707292428479
20	75306	Snap lock R82 25	PCS	8	5707292437624
21	1310542	Ferrule,black Ø8*Ø5mm	PCS	2	5707292472014
22+23	75500-1	Back Panda futura size 5	PCS	1	5707292607843
22	6153210	Cover plate, metal	PCS	1	5707292467027
23	75500-1	Back Panda futura size 5	PCS	1	5707292607843
24	0658016	Button head M5*16 stainless	PCS	2	5707292139399
25	1100500	Spring washer 8mm, stainless	PCS	6	5707292140142
26	75521	Mounting for seat/back	PCS	2	5707292460752
27	75526	Mounting for gas spring	PCS	1	5707292460783
28	1100800	Center spring washer 6mm	PCS	4	5707292140166
29	0602200	Insex screw 6*16	PCS	4	5707292138095
30	75516	Handle for armrest, right	PCS	1	5707292466686
31	6109002	Screw spec. M8, Panda 5	PCS	2	5707292466754
32	1606300	Lock nut 5 mm	PCS	2	5707292141354
34	0603530	Insex low head M8x16 stainless	PCS	6	5707292412492
35	1103000	Fazet washer 6mm	PCS	4	5707292140289
36	1310540	Covering pushbutton, black	PCS	2	5707292140692
37	1760005	Compensating nut 5mm	PCS	1	5707292141491
42	0371853	Strained wires 1,5mm 190 cm.	PCS	1	5707292135889
43	1102900	Fazet washer 5 mm	PCS	2	5707292140272
45	6118805	Pin Ø8*44 f. Panda back, pvc	PCS	1	5707292186690
46	0610516	Counter insex M5*16	PCS	4	5707292138644
47	1125000	Screenwasher FZB black 5/16*30	PCS	4	5707292140470
48	5550260	Sewn strap holder.Pan.futura	PCS	4	5707292178459

49	0380000	Cable tighter M5*23	PCS	1	5707292135933
51	1350000	Distancepart black 6*15*8,2 mm	PCS	2	5707292140876
52	75527	BØSNING Ø10*28-M6	PCS	1	5707292460790
55	75523	Bracket for back adjustment	PCS	1	5707292460745
56	0603535	Insex low head M8*25	PCS	8	5707292575319
58	75291	Hexagon key nv=5	PCS	1	5707292437655
59	75517	Handle for armrest, left	PCS	1	5707292466693
60	0529320	CYL.STIFT Ø4,0X50 MM.	PCS	2	5707292467201
61	75003	Baseplate 3	PCS	1	5707292458766
62	1810055	Platescrew st 2,9x6,5	PCS	1	5707292469519
63	0235080	Spring clip	PCS	1	5707292442901
131	89847-5UB	Foam cushions for backsides	SÆT	1	5707292465351
132	89840-571	Cover,backsidecush.act.5,blue	SÆT	1	5707292466938
133	75508	Back side, right, Panda 5	PCS	1	5707292466907
134	4913040	30mm adhesive hook, black	PCS	1	5707292786517
134	4913035	30mm black ps14 with glue hook	PCS	1	5707292469755
135	0662501	Button head w/flange 8*25	PCS	4	5707292403759
138	75509	Back side, left, Panda 5	PCS	1	5707292466914
159	1350010	Distancepart black 8*15*8,2mm	PCS	2	5707292140883
160	2207030	Springstopper l=403/268	PCS	1	5707292466082
161	0380060	Flexible v-brake pipe.	PCS	1	5707292425195
162	5002930	Spring stopper outer casing	PCS	1	5707292472038
165	0371853	Strained wires 1,5mm 190 cm.	PCS	1	5707292135889
	6016100	Panda 5 topinlay	PCS	1	5707292472410
	6016101	Panda 5 bottom inlay	PCS	1	5707292510518
	6016110	Panda 5 box 81	PCS	1	5707292475862
	5089100	Gasfjeder ryg mont. Panda 5	PCS	1	5707292472021

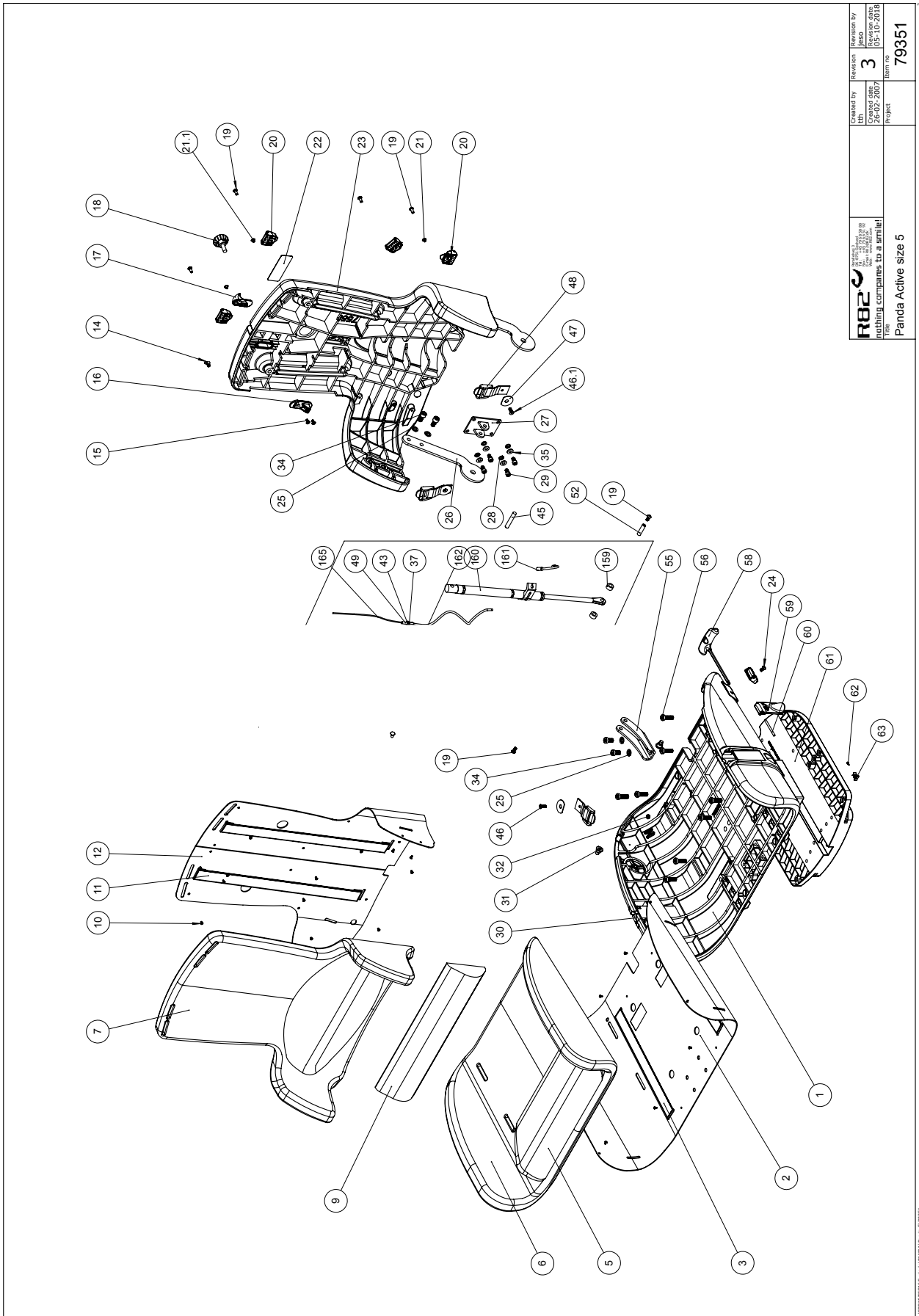


 R82 A/S Parallelvej 3 8751 Gedved Denmark	Checked by UH	Revision 3	Checked on by MBO
	Created date 26-02-2007	Revision date 05-10-2018	Item no. 79351
For Panda Active size 5		Project 79351	

Copyright reserved R82 A/S 2018

Position	Art. No.	Description	Unit of measure	Quantity	Ean code
1	75505	Seat, Panda 5	PCS	1	5707292466648
2	75506	Cover plate Seat	PCS	1	5707292460288
3	4913035	30mm black ps14 with glue hook	PCS	1	5707292469755
3	4913040	30mm adhesive hook, black	PCS	1	5707292786517
5	5699990	Glued Panda str.5,Seat-cushion	PCS	1	5707292472472
6	89638-560	Cover/seat Panda 5, black	PCS	1	5707292472489
6	89638-585	Cover seat Panda 5	PCS	1	5707292472175
7	5699992	Upholstered Panda 5, active	PCS	1	5707292474056
8	89640-560	Cover/back Panda active 5	PCS	1	5707292472496
8	89640-585	Cover back panda akt. sz. 5	PCS	1	5707292472212
9	5700250	Lumbar support,PANTHER 1	PCS	1	5707292180902
9	5700251	Lumbar support, Panther 2	PCS	1	5707292180919
10	1940010	Plastite 60, 2,9*6,5	PCS	22	5707292142573
11	4913035	30mm black ps14 with glue hook	PCS	1	5707292469755
11	4913040	30mm adhesive hook, black	PCS	1	5707292786517
12	75501	Cover plate back	PCS	1	5707292460370
14	0388010	Screwnippel 7*7mm	PCS	1	5707292136121
15	1940006	Eco syn 3,5*16	PCS	2	5707292474902
16	75514	Bottom for release button	PCS	1	5707292466655
17	75513	Handle for back adjustment	PCS	1	5707292466662
18	3501201	R82 handle M8*25mm screw	PCS	1	5707292413932
19	0640911	Button head M5x10 FZB tuflok	PCS	8	5707292428479
20	75306	Snap lock R82 25	PCS	8	5707292437624
21	1310542	Ferrule,black Ø8*Ø5mm	PCS	4	5707292472014
21.1	1310556	Ferrule, black ø8*5 mm	PCS	2	5707292592903
22	6153210	Cover plate, metal	PCS	1	5707292467027
23	75500-1	Back Panda futura size 5	PCS	1	5707292607843
24	0658016	Button head M5*16 stainless	PCS	2	5707292139399
25	1100500	Spring washer 8mm, stainless	PCS	6	5707292140142
26	75521	Mounting for seat/back	PCS	2	5707292460752
27	75526	Mounting for gas spring	PCS	1	5707292460783
28	1100800	Center spring washer 6mm	PCS	4	5707292140166
29	0602200	Insex screw 6*16	PCS	4	5707292138095
30	75516	Handle for armrest, right	PCS	1	5707292466686
31	6109002	Screw spec. M8, Panda 5	PCS	2	5707292466754
32	1606300	Lock nut 5 mm	PCS	2	5707292141354
34	0603530	Insex low head M8x16 stainless	PCS	6	5707292412492
35	1103000	Fazet washer 6mm	PCS	4	5707292140289
36	1310540	Covering pushbutton, black	PCS	2	5707292140692
37	1760005	Compensating nut 5mm	PCS	1	5707292141491
42	0371853	Strained wires 1,5mm 190 cm.	PCS	1	5707292135889
43	1102900	Fazet washer 5 mm	PCS	2	5707292140272
45	6118805	Pin Ø8*44 f. Panda back, pvc	PCS	1	5707292186690
46	0610516	Counter insex M5*16	PCS	4	5707292138644
47	1125000	Screenwasher FZB black 5/16*30	PCS	4	5707292140470
48	5550260	Sewn strap holder.Pan.futura	PCS	4	5707292178459
49	0380000	Cable tighter M5*23	PCS	1	5707292135933
51	1350000	Distancepart black 6*15*8,2 mm	PCS	2	5707292140876
52	75527	Bushing Ø10*28-M6	PCS	1	5707292460790

55	75523	Bracket for back adjustment	PCS	1	5707292460745
56	0603535	Insex low head M8*25	PCS	8	5707292575319
58	75291	Hexagon key nv=5	PCS	1	5707292437655
59	75517	Handle for armrest, left	PCS	1	5707292466693
60	0529320	CYL.STIFT Ø4,0X50 MM.	PCS	2	5707292467201
61	75003	Baseplate 3	PCS	1	5707292458766
62	1810055	Platescrew st 2,9x6,5	PCS	1	5707292469519
63	0235080	Spring clip	PCS	1	5707292442901
159	1350010	Distancepart black 8*15*8,2mm	PCS	2	5707292140883
160	2207030	Springstopper l=403/268	PCS	1	5707292466082
161	0380060	Flexible v-brake pipe.	PCS	1	5707292425195
162	5002930	Spring stopper outer casing	PCS	1	5707292472038
165	0371853	Strained wires 1,5mm 190 cm.	PCS	1	5707292135889
22+23	75500-1	Back Panda futura size 5	PCS	1	5707292607843
	5089100	Gasfjeder ryg mont. Panda 5	PCS	1	5707292472021
	6016100	Panda 5 topinlay	PCS	1	5707292472410
	6016101	Panda 5 bottom inlay	PCS	1	5707292510518
	6016110	Panda 5 box 81	PCS	1	5707292475862

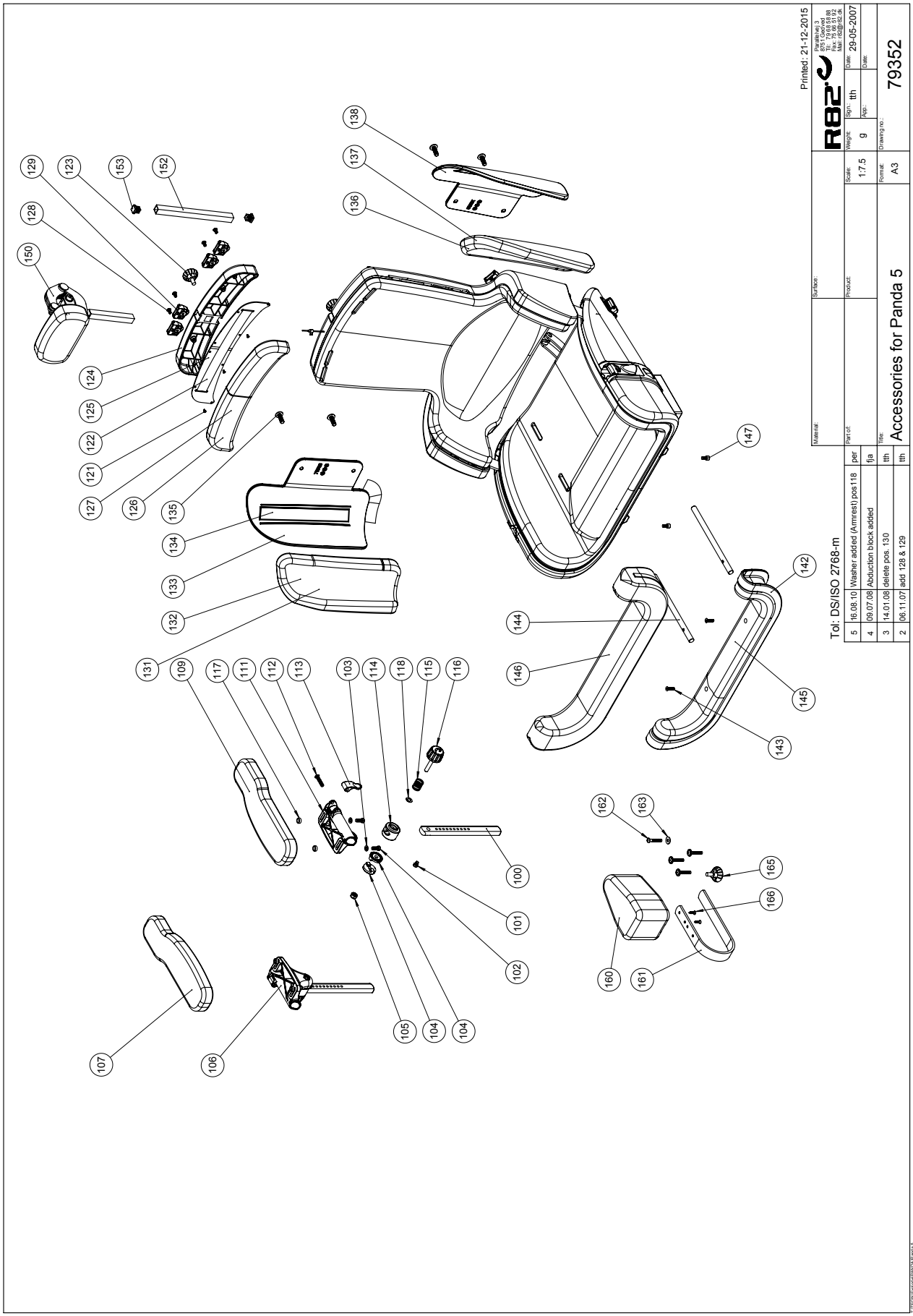

Printed: 21-12-2015

R82
R82 A/S
Sønderhedevej 3
8751 Gedved
DK
Tlf: +45 75 66 51 92
Fax: +45 75 66 51 92
Web: www.r82.org.dk

Material	Surface	Scale	Weight	ISO	1st	2nd	Date
		1:7.5	g				29-05-2007
Part of	Product	Reviz	Part	Reviz	Part	Reviz	Date
Title		Drawing no.					
Accessories for Panda 5		79352					

Tol: DS/ISO 2768-m

per	per	per
118	130	128 & 129
5	16.08.10	Washer added (Amrest) pos 118
4	09.07.08	Abduction block added
3	14.01.08	delete pos. 130
2	08.11.07	add 128 & 129

Position	Art. No.	Description	Unit of Measure	Quantity	Ean code
100	75538	Rod for arm rest +10 cm	PCS	2	5707292472540
100	75537	Rail for armrest size 1	PCS	2	5707292460325
101	0602479	Insex M6*8, stainless	PCS	2	5707292465412
102	0602207	Insex M6*18, stainless	PCS	4	5707292430380
103	1105060	Stainless washer 6mm	PCS	4	5707292140357
104	74063	Toothed disk o30*9	PCS	4	5707292377968
105	1603500	Steel nut 8mm, large pack	PCS	2	5707292141286
106	75392	Holder for armrest, right	PCS	1	5707292454775
107	6175163	Armrest pu. Wombat sz 4. Right	PCS	1	5707292443151
107	6175158	Armrest pad s.4,right,black	PCS	1	5707292189264
109	6175164	Armrest pu. Wombat sz 4. Left	PCS	1	5707292443168
109	6175160	Armrest pad,s.4,left,black	PCS	1	5707292189271
111	75391	Holder for armrest, left	PCS	1	5707292454768
112	0659580	Button head screw 6*30 mm	PCS	2	5707292139481
113	75395	Lock, armrest & table	PCS	2	5707292454799
114	75390	Bush for stand, armrest	PCS	2	5707292454751
115	0310113	Stainless spring 1,2*16indv.*20	PCS	2	5707292453143
116	3501635	R82 handle dia.,M8*37	PCS	2	5707292451910
117	1310539	Distance piece 12/8,4*5mm	PCS	4	5707292423474
118	1101050	Spring washer M8, wavy FZB	PCS	2	5707292458667
121	1940010	Plastite 60, 2,9*6,5	PCS	4	5707292142573
122	4913040	30mm adhesive hook, black	PCS	1	5707292786517
123	3503001	R82 handle M8x15mm screw	PCS	1	5707292413925
124	75502	Back extension	PCS	1	5707292466884
125	75503	Coverplate f. Backextension	PCS	1	5707292466891
128	0640911	Button head M5x10 FZB tuffok	PCS	2	5707292428479
129	75306	Snap lock R82 25	PCS	2	5707292437624
133	75508	Back side, right, Panda 5	PCS	1	5707292466907
134	4913040	30mm adhesive hook, black	PCS	1	5707292786517
135	0662501	Button head w/flange 8*25	PCS	4	5707292403759
138	75509	Back side, left, Panda 5	PCS	1	5707292466914
147	0602190	Insex screw 6*12	PCS	2	5707292138088
152	81155	Rod for backextension	PCS	1	5707292112781
153	2504000	Ferrule e15*1,0-2,0 int.black	PCS	2	5707292144102
160	6175114	Pommel size 4, black pur	PCS	1	5707292189134
161	74543	Brace f. Abduction block	PCS	1	5707292419064
162	5001001	Bag with screws for pommel	PCS	1	5707292154255
164	81626	Mounting f.pommel, Panda pvc	PCS	1	5707292117175
165	3503001	R82 handle M8x15mm screw	PCS	1	5707292413925
166	1811275	Cutting screw 4*30mm	PCS	2	5707292141743
153	2504000	FERRULE E15*1,0-2,0 INT.BLACK	PCS	2	5707292144102
160	6175114	POMMEL SIZE 4, BLACK PUR	PCS	1	5707292189134
161	74543	BRACE F. ABDUCTION BLOCK	PCS	1	5707292419064
162	0641600	BUTTON HEAD SCREW 6*16	pcs	4	5707292139269
164	81626	MOUNTING F.POMMEL, PANDA PVC	PCS	1	5707292117175
165	3503001	R82 HANDLE M8*15MM SCREW	PCS	1	5707292413925
166	1811275	CUTTING SCREW 4*30MM	PCS	2	5707292141743