


User manual

Swift/Smart/Edge/Easy




etac[®]

Creating Possibilities

78390 - 2023-12-14 - Version 19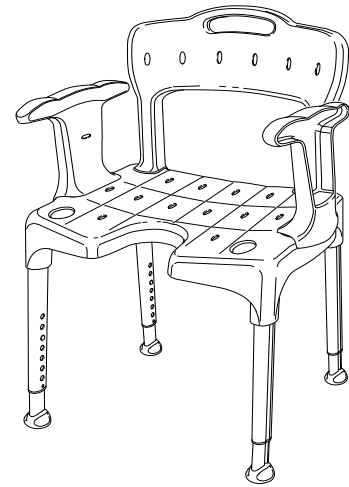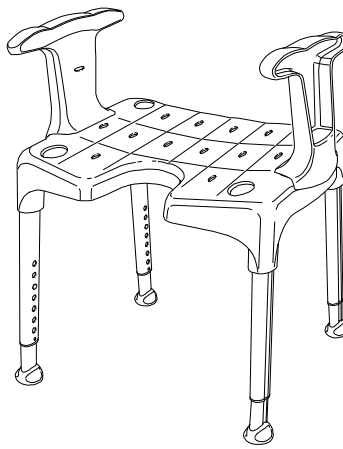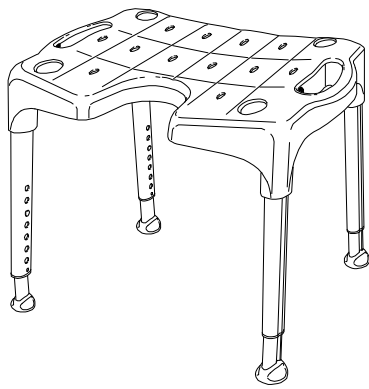

Etac Swift

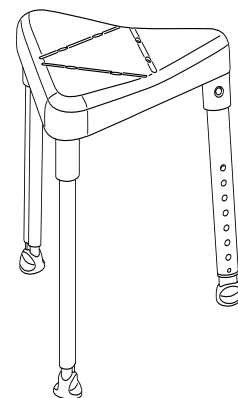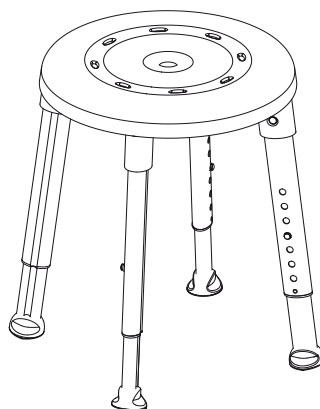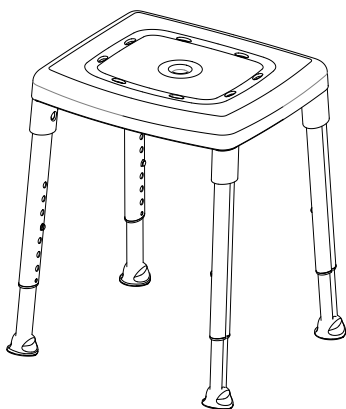Etac Smart

Etac Easy


Etac Edge

sv en da de it nl no bg cs el es et fi fr hr is lt ms pl pt sk ar


www.etac.com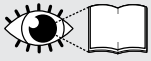

A

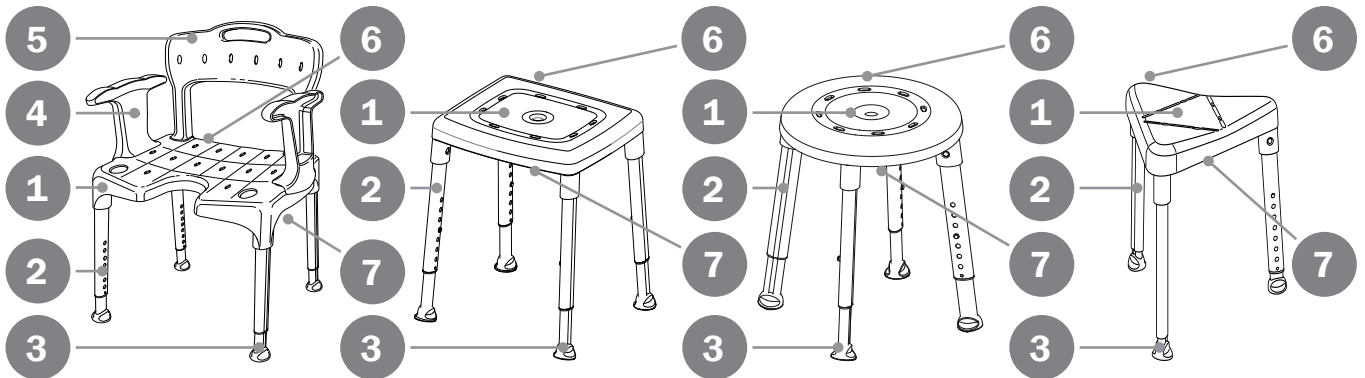B

1	2	3	4	5
6	7	8	9	10
11	12			

C

6

UDI Bar code GS1-128
(01)XXXXXXXXXXXXXXXX(11)YYMMDD(21)XXXXX

1 XXXXXXXX
XXXXXXX

2 **REF** XXXXXXXXXX
SN XXXXXX




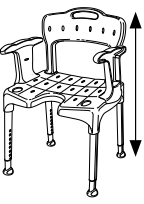
Etac Supply Center AB
Långgatan 12
SE-334 33 Anderstorp
YYYY-MM-DD

3

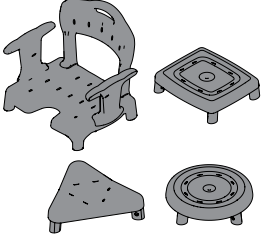
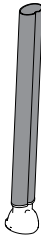
4

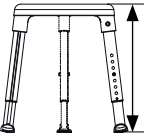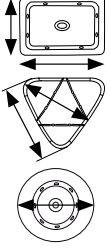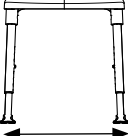5


D

							
Swift	420-570 mm 16½"-22½"	540 x 410 mm 21¼" x 16¼"	530-540/565-595 mm 20¾"-21¼" / 22¼"-23½"	3,1/3,9/4,7 kg 7/8/10 lbs	160 kg (*150 kg) 352 lbs	210 mm / 450 mm 8¼" / 17¾"	800-950 mm 31½"-37½"
Swift low	345-420 mm 13½"-16½"	540 x 410 mm 21¼" x 16¼"	520-530/555-565 mm 20¼"-20¾" / 21¾"-22¼"	2,9 kg/4,5 kg 6/10 lbs	160 kg 352 lbs	210 mm / 450 mm 8¼" / 17¾"	725-800 mm 28½"-31½"
Smart	420-570 mm 16½"-22½"	430 x 380 mm 21¼" x 16¼"	455-465 x 410-430 mm 18"-18¼" / 16¼"-17"	3,15 kg 7 lbs	200 kg (*150 kg) 440 lbs (*330 lbs)	-	-
Smart low	345-420 mm 13½"-16½"	430 x 380 mm 17" x 15"	455-460 x 410-415 mm 18" / 16¼"	2,98 kg 6 lbs	200 kg 440 lbs	-	-
Edge	420-570 mm 16½"-22½"	450 x 405 mm 17¾" x 16"	500-520 mm 19¾"-20½"	2,6 kg 6 lbs	130 kg 286 lbs	-	-
Edge low	345-420 mm 13½"-16½"	450 x 405 mm 17¾" x 16"	490-500 mm 19¼"-19¾"	2,4 kg 5 lbs	130 kg 286 lbs	-	-
Easy	420-570 mm 16½"-22½"	400 mm 16"	495-535 mm 19½"-21"	2,75 kg 6 lbs	200 kg (*150 kg) 440 lbs (*330 lbs)	-	-




*Item no. 81901070 / 81901080 / 81901090 / 81951070 / 81951080 / 81951090 / 81701480

					
Swift	380 mm 15"	448 mm 17¾"			
Swift low	380 mm 15"	448 mm 17¾"			
Smart	-	-			
Smart low	-	-			
Edge	-	-			
Edge low	-	-			
Easy	-	-			
			PP	TPE	Alu


E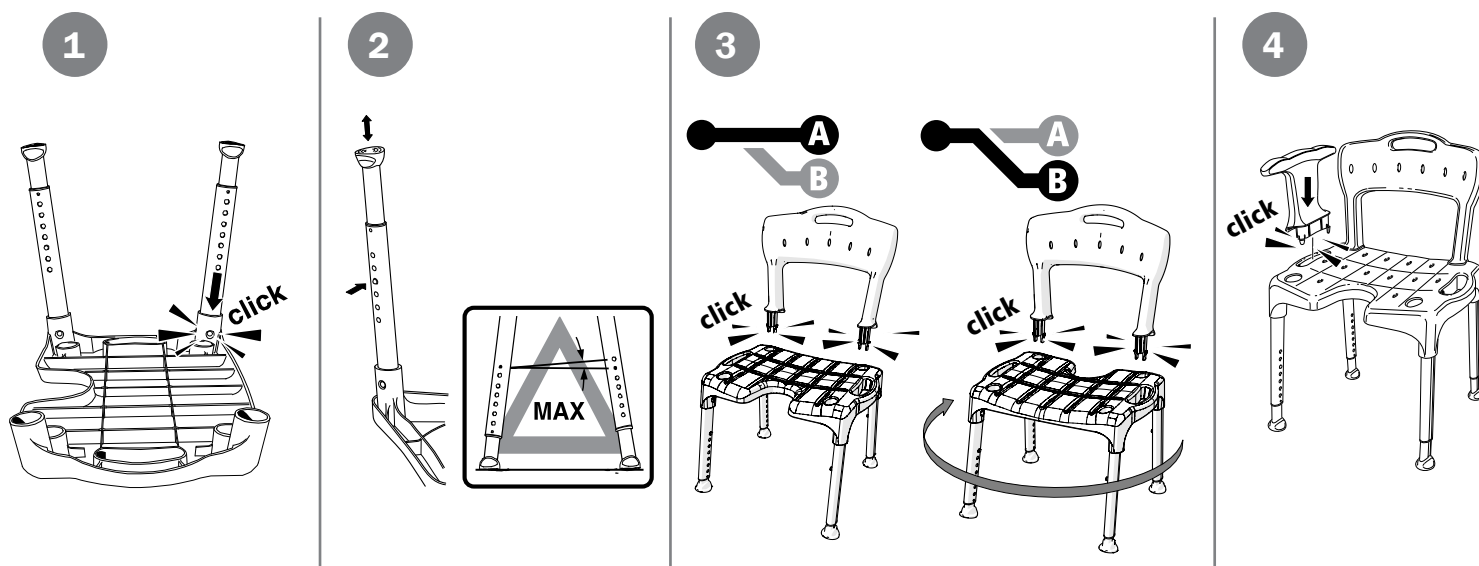

F

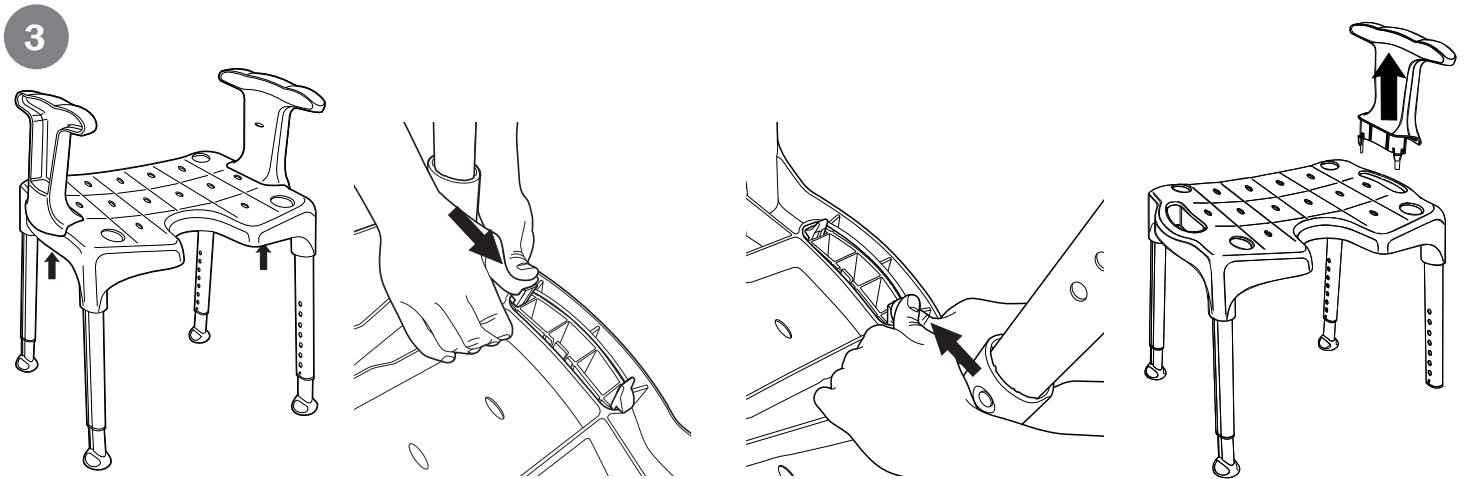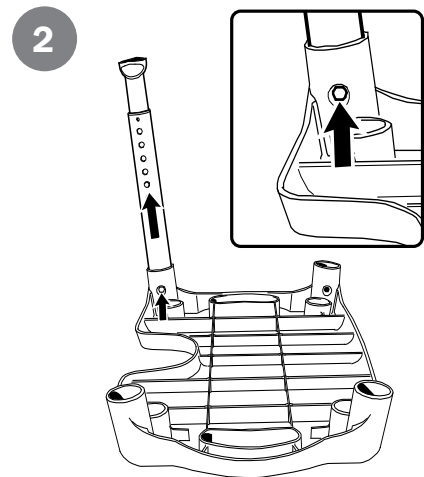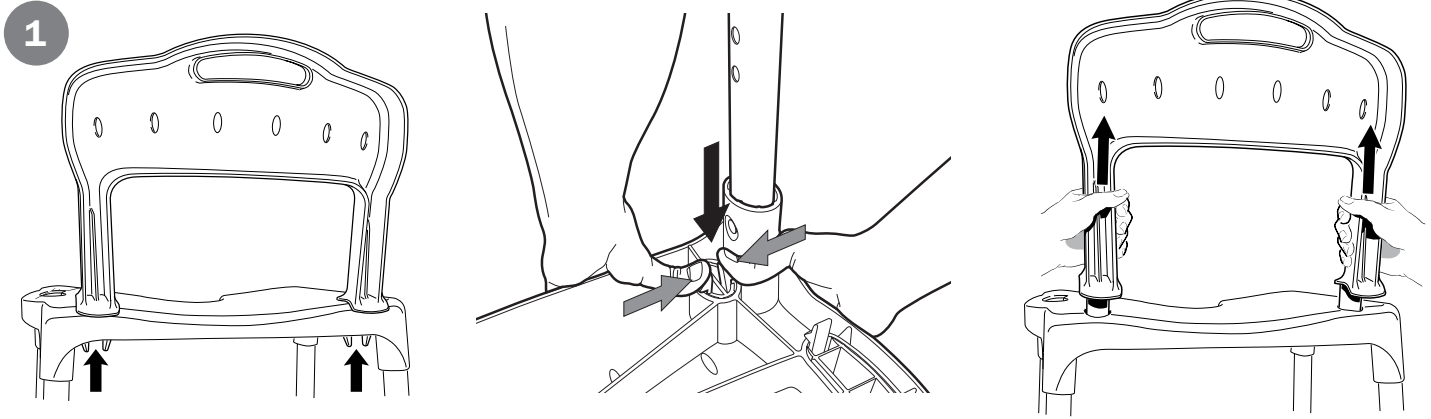G

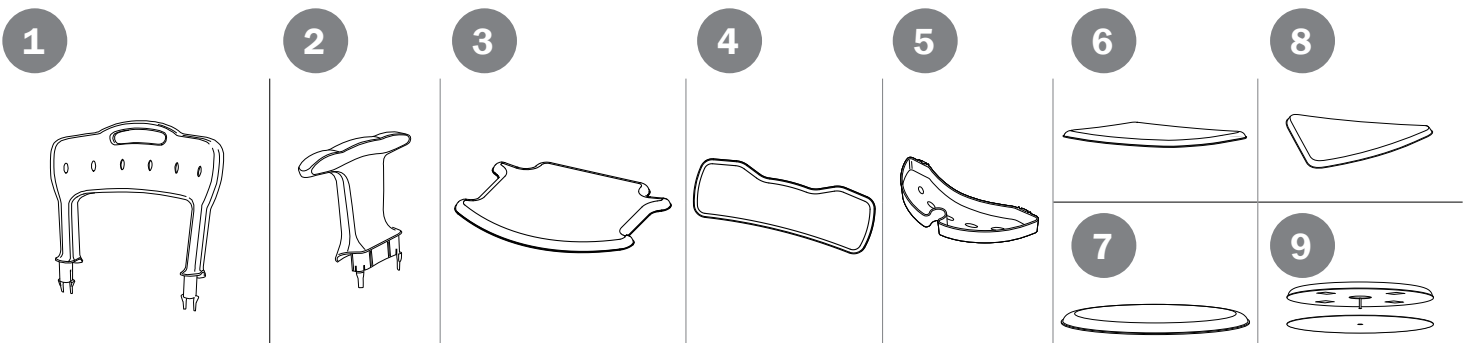sv en da de it nl no bg cs el es et fi fr hr is lt ms pl pt sk ar

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Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at www.etac.com. You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

In the manual, the user is the person sitting in the chair. The carer is the person helping the user.

Device description

Swift, Smart, Edge and Easy have a stable design with built-in flexibility that adapts to uneven floors. All devices have height-adjustable legs that are simple to attach. The legs are angled outwards to maximise stability regardless of height. All devices are equipped with ferrules that provide good grip, even on uneven and wet floors. The devices can be tilted forward to further facilitate standing up. There are different shapes of the seats and they are all designed with an anti-slip pattern, ensuring a safe and non-slip seat even when it is wet. The seats also have drainage holes that effectively remove excess water.

Intended purpose

Swift, Smart, Edge and Easy (hereafter also referred to as "the device" or "the product"), are medical devices intended for alleviation of or compensation for a functional impairment due to an injury or disability. The device is designed to facilitate personal hygiene and provide stability and support in the shower or the bathtub.

Intended user group

The user group for the device is based on the individual's functional ability and not on a specific diagnosis, health condition or age. It is intended for individuals with a height of 146 cm or more, or a mass of 40 kg or more. Secondary users of the device are caregivers providing assistance and clinicians/technicians who set up the device.

Intended environment

The device is intended for indoor use at home environments or institutions and is suitable for use in bathrooms, but not swimming pools or similar corrosive environments.

Intended application

The device is intended for short-term and long-term use and can be applied several times per day. The device is intended to be used in contact with intact skin. The device is intended for refurbishment and reuse.

Expected service life

The expected service life is 10 years. For complete information regarding the service life of the device, see www.etac.com.

Indications

Indication for use is disability, including, but not limited to, an individual with fluctuating balance issues, physiological and/or functional impairment, and/or a history of falls.

Contraindications

There are no known contraindications.

Warnings

Warnings describing a risk element for a specific action or setting of the device can be found in the relevant section.

 When moving sideways, do not lean on just one arm support.

 Do not stand on the device

 A defective device must not be used.

Declaration of conformity

The device meets the requirements of the Medical Device Regulation (EU) 2017/745.

The device is tested and meets the requirements of the EN ISO 10993-1, SS EN 12182.

Pre-purchase information

Pre-purchase information can be found at www.etac.com.

Adverse event

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Customization

is everything that goes beyond the manual's instructions and settings. Device specially adapted by the customer must not retain Etac's CE marking. Etac's warranty expires.

If there is any uncertainty regarding the execution of the customization, consult Etac.

Device coupled with another device may not retain either device's CE mark unless combination agreements exist. Contact Etac for current information.

Device description [Figure A](#)

1. Seat
2. Leg
3. Ferrule
4. Arm support
5. Back support
6. Product label
7. Item no label

Symbols, labels [Figure B](#)

Symbols in the manual and on the device:

1. Warning, precaution or limitation.
2. Useful advice and tips.
3. Material for recycling.
4. A height difference of more than one hole is not allowed for chair legs.
5. When moving sideways, do not lean on just one arm support.
6. Maximum user weight (see technical data)
- 7-8. Cleaning (see maintenance)
9. Read instructions for use
10. Showing option 1 of two possible options.
11. Showing option 2 of two possible options.
12. Product weight

Explanation item no label: [Figure C](#)

1. Product name
2. Product description
3. Serial number
4. Item number
5. Date of manufacture
6. Barcode acc. to GS1-128 GTIN-14 and serial number*

*The device's date of manufacture can be read from the barcode on the device. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

Technical data [Figure D](#)**Installation** [Figure E](#)

 A height difference of more than one hole is not allowed for chair legs.

Dismantling [Figure F](#)

1. Back support
2. Leg
3. Arm support

Maintenance

Clean the device with a solvent-free detergent which has a pH of 5-9, or with a 70% disinfectant solution. Can be decontaminated at max. 85 °C if so required.

Constituent materials are corrosion-resistant. 5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Storage

The device should be stored indoors in a dry place at a temperature above 5 °C. If the device has been stored for a long time (more than four months), its function must be checked by an expert before use.

Accessories [Figure G](#)

- Swift:** 1. Back support 2. Arm support
3. Soft seat **4.** Soft back **5.** Soap dish/
 Lid support/Shower head holder
6. Toilet paper holder
Smart/Edge/Easy: 7-9. Soft seat
Smart, Easy: 10. Turning plate

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Etac Ltd.
Promefa AG