
# 8 point Sling bar Balance bar **molift**<sup>®</sup>

by Etac

User Manual - Bedienungsanleitung - Brukermanual - Bruksanvisning - Käyttöopas - Manuale per l'utente -  
Manuel d'utilisation - Manual de usuario - Gebruikershandleiding - Brugervejledning

BM69399 Rev. B 2023-01-26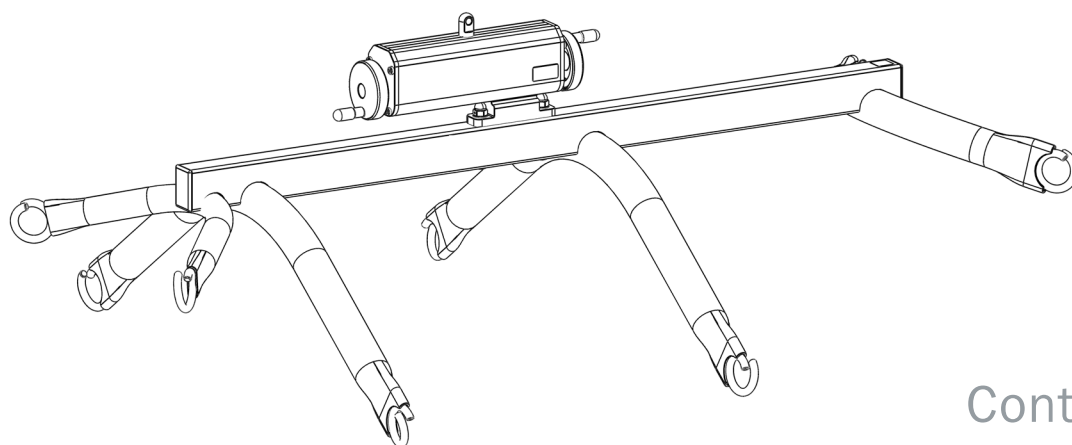

## Contents

User manual - English - 2
Gebrauchsanweisung - Deutsch - 12
Brukermanual - Norsk - 22
Bruksanvisning - Svenska - 32
Käyttöohje - Suomi - 42
Manual utente - Italiano - 52
Manual d`utilisation - Francais - 62
Manual de usuario - Español - 72
Handleiding - Nederlands - 82
Brugervejledning - Dansk - 92

## Introduction

**To fully benefit from the options offered by this product please read this user manual before use and save it for future reference.  
Ensure to follow user manuals for accessories.  
The latest version of the instructions is always available on [www.etac.com](http://www.etac.com) and can be printed in larger sizes.**

**Warnings and special attention are shown with a triangle symbol with an exclamation mark in it – follow these instructions carefully.**

Symbols used in this user manual	Explanation
	Examine, check
	Warning
	Lifting point
	Correct
	Incorrect

## General

**Before performing transfer of a user read the user manual for**

**RgoSling Fabric Stretcher  
RgoSling Repositioning Sheet  
Molift Air 200/350  
Molift Air 205/300  
Molift Air 500**

**To find the latest version of these user manuals go to [www.etac.com](http://www.etac.com).**

The 8 Point Sling bar Balance bar (8PSB) is intended to be used with ceiling hoist: Molift Air 200/350/500 or Air 205/300, and sling: RgoSling Repositioning Sheet and RgoSling Fabric Stretcher.

**Definition**

The person described as “the user” in this Instruction for Use is the person who is lifted in the product. “The carer” is the person who manoeuvres the product.

**Intended purpose**

To be used for lifting and adjusting the horizontal position of users, who require support to be transferred in lying position between two beds or similar

**Intended target group**

For users who are in need of support to be lifted and transferred in lying position within a rail system; e.g. persons with limited ability to move and who require body support during transfers. Not an exhaustive list.

**Intended environment**

For indoor use within a rail system in acute care, long term care, home care.

**Indications**

The device can be used for children, adults and elderly people with a need for support to be lifted and transferred between two lying surfaces within a rail system.

The target group for the device is based on individual health and mobility function, and not a specific diagnose or age.

**Contraindications**

There are no known contra indications regarding the hoist, but make sure the sling fits the user.

**Declaration of conformity**

	<p><b>The product, and its related accessories, described in this instruction for use (IFU), is in compliance with the regulation (EU) 2017/745 of 5. April 2017 – as a medical device, risk class I.</b></p> <p><b>The product is manufactured and tested according to EN/ISO 10535.</b></p> <p><b>Any serious incident that occurs in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.</b></p>
---	--

**Pre-sales and warranty**

See [www.etac.com](http://www.etac.com).

We recommend using Etac brand original spare parts, components, accessories and fittings.

**In case of an adverse event**

In case of an adverse event occurred during use of the device, this should be reported to Your distributor and the national competent authority in a timely manner. The distributor will forward information to Etac (manufacturer).


## Training requirements

**Never leave the user unattended.**

Only healthcare professionals and instructed carers may operate the 8PSB.

## Product description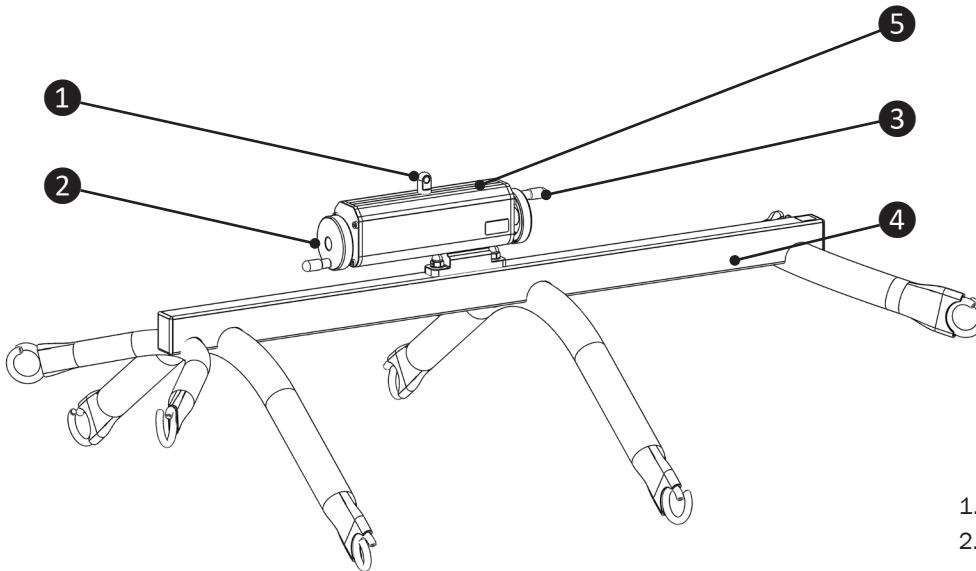

1. Bolt connection point
2. Handwheel, upper body side
3. Handwheel, lower body side
4. 8 Point Sling bar  
Balance bar (8PSB)
5. Balance bar housing

## Technical data



Safe working load: 350 kg / 772 lbs

Weight: 18 kg / 39.7 lbs

Material: Steel, aluminium, and plastic

Turning diameter (max.): 1640 mm

Outer dimensions (L x W x H): 1435 x 921 x 486 mm

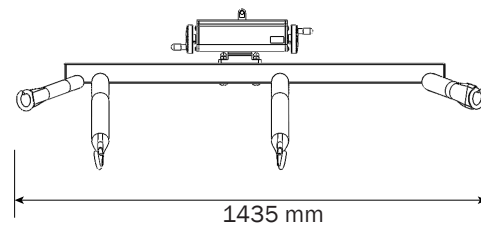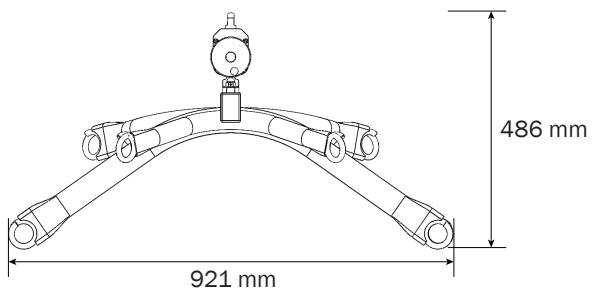
## Symbols and marking on product

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols, and instructions. Contact your distributor for instructions.

### Placement and explanation

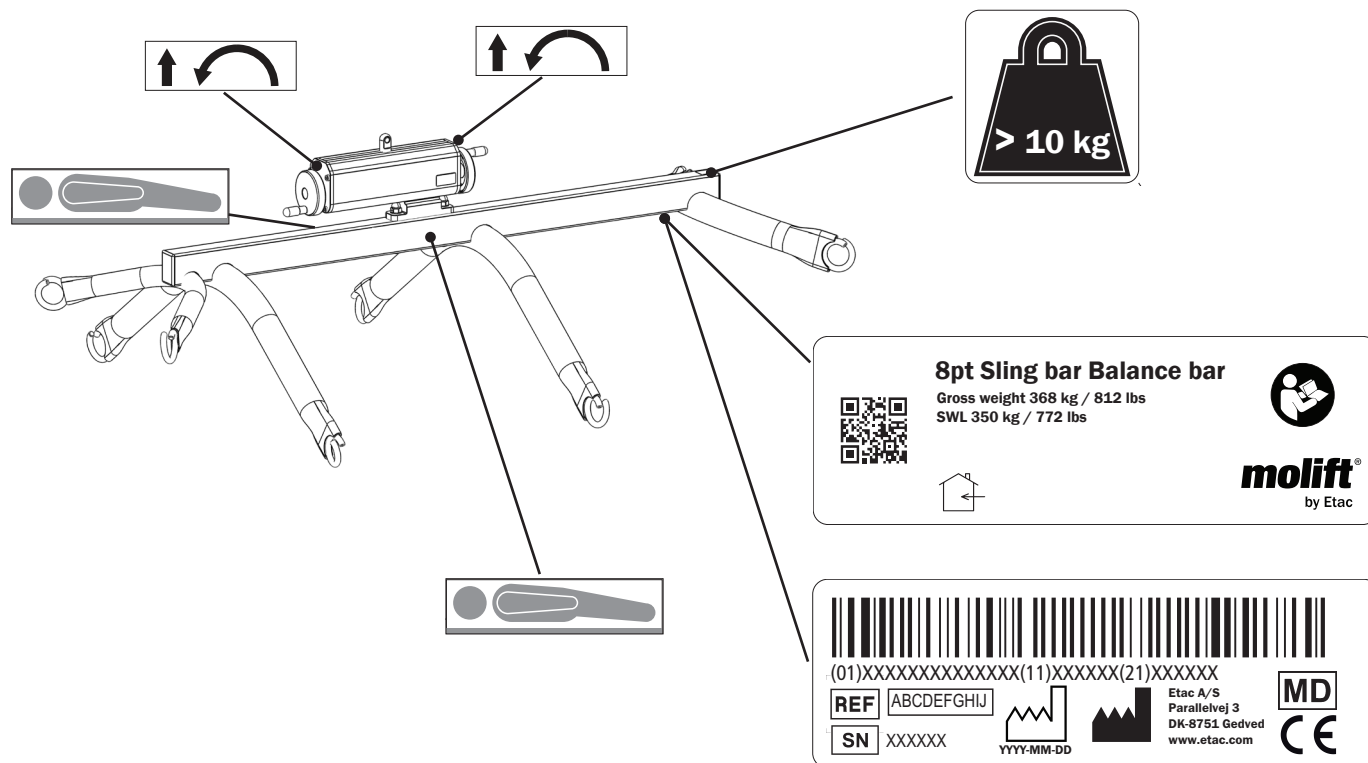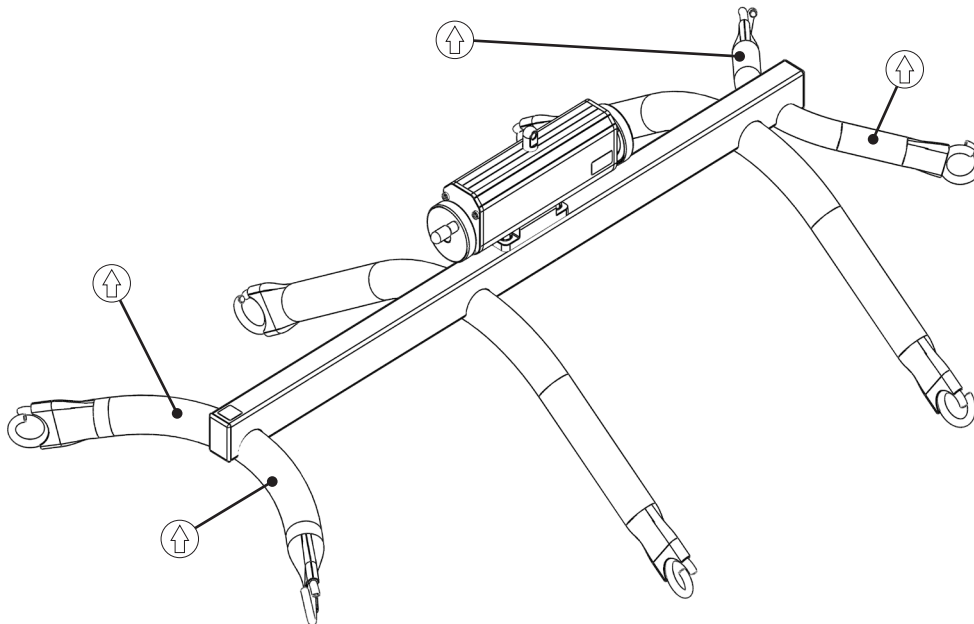
- |  |                            |  |               |  |                      |  |                           |
|--|----------------------------|--|---------------|--|----------------------|--|---------------------------|
|  | Item weight is above 10 kg |  | QR code       |  | Refer to user manual |  | Date of manufacture       |
|  | CE marked                  |  | Manufacturer  |  | Indoor use only      |  | Catalogue number          |
|  | Medical Device             |  | Serial number |  | User direction       |  | Rotation direction for up |

## Handling


### Transportation

8PSB is portable when not connected to hoist, as shown below. Note that the 8PSB weighs above 10 kg, and therefore must be transported by more than one person.

Areas, marked with  in illustration below, are suitable for handling during transport and lifting of 8PSB.

Recommended handling during transport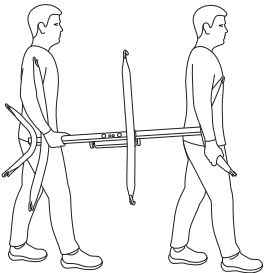

### Assembly

8PSB is delivered fully assembled.

### Disposal


When the product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. Please see recycle instruction at [www.etac.com](http://www.etac.com).

### Installation


**The device shall only be mounted on a rail system with a capacity of 350 kg or higher. If the device is mounted on a rail system with a lower lifting capacity, the lowest capacity also applies for the device.**

Place the 8PSB on a stable surface (e.g. floor).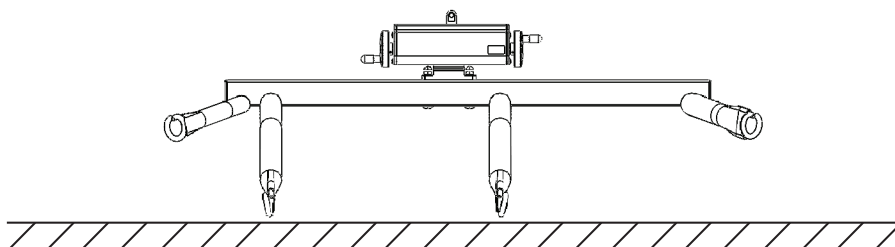


Place hoist above the 8PSB.

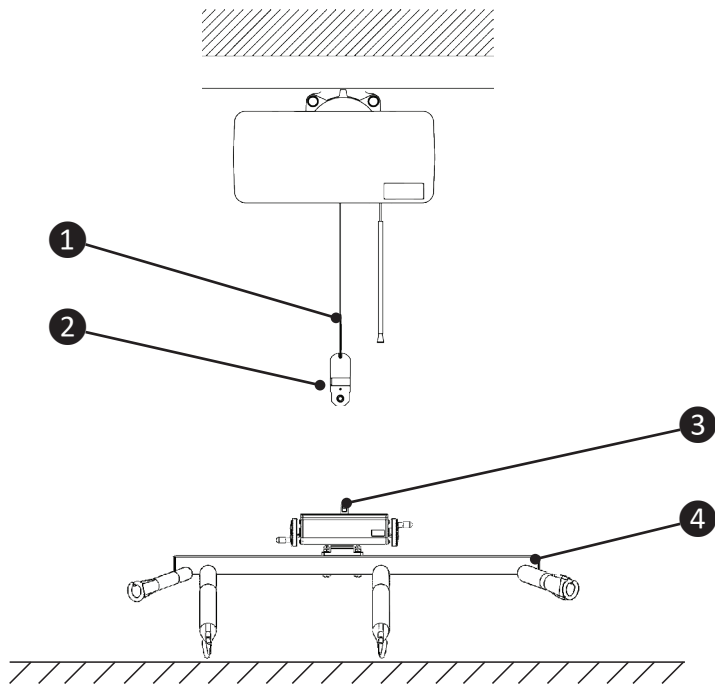
- 1. Belt
- 2. Belt connection point with quick release pin
- 3. Bolt connection point
- 4. 8 Point Sling bar Balance bar (8PSB)

Lower belt from ceiling hoist by using the ceiling hoist's hand control.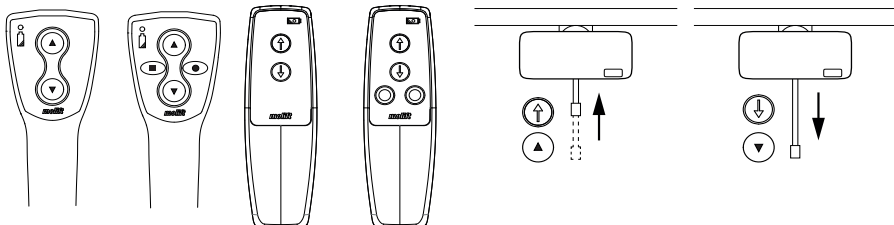


Ensure vertical alignment between hoist and 8PSB, so 8PSB will not swing when hoisted up from surface.

Attach belt connection point to the bolt connection point of the 8 Point Sling bar Balance bar using the quick release pin.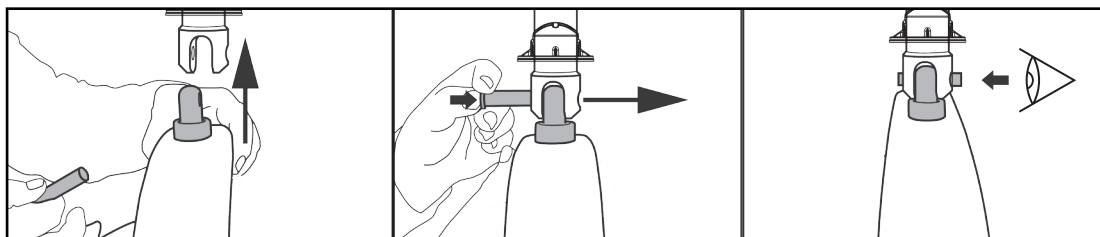

Ensure the 8 Point Sling bar Balance bar is properly fastened to the belt's connection point. Check that the quick release pin is fully inserted, and the two spheres at the end of the quick release pin are out, so the quick release pin can not retract.

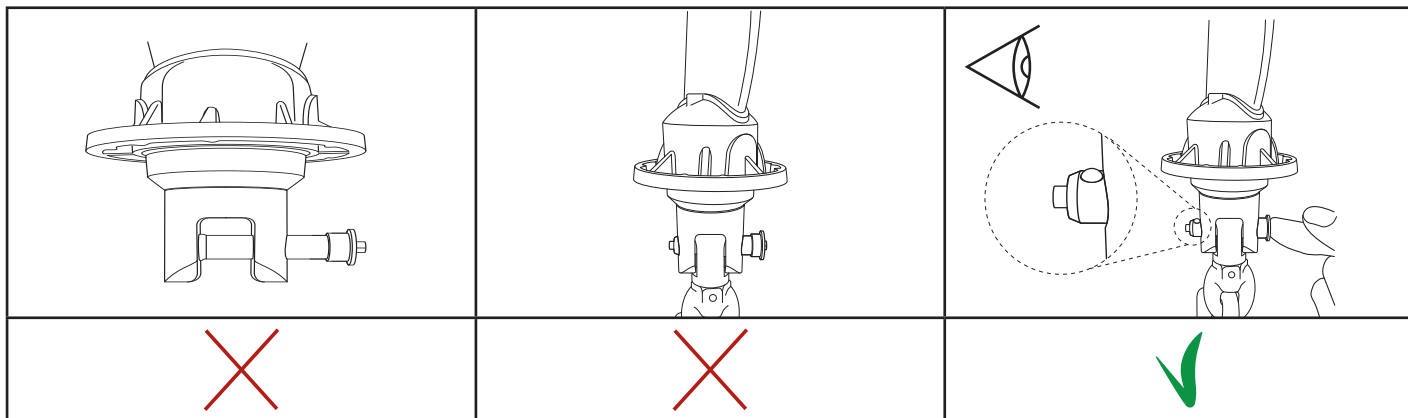Now 8PSB is ready for elevation.


**Risk of collision.**  
**8PSB can move sideways when elevated from ground.**  
**Be sure to keep 8PSB still while hoisting.**

Hoist 8PSB up to requested height by using the ceiling hoist's hand control.

For dismantling 8PSB follow the previous procedure in reverse order.

## Operation

See encircled area below for user's head area.

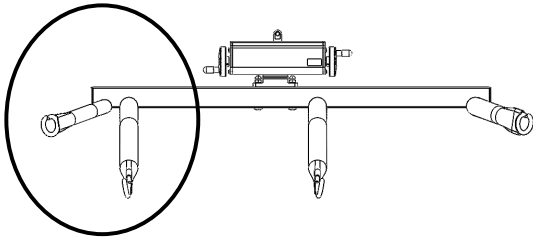**Do not use damaged or badly worn slings.**  
**Every sling loop must be in use and attached correctly to 8PSB while hoisting.**

**Keep hand control wire and emergency stop rod away from Balance bar housing.**

### Before hoisting the user

Place the user in the RgoSling Fabric Stretcher sling or RgoSling Repositioning sheet according to related user manual.

### Hoisting the user

Hoist the user in a still and smooth manner using the hand control to assess the horizontal position of the user.

**Under no circumstances should the user be angled with their head lower than their shoulders.**  
**Ensure most appropriate position for the user.**

**Keep 8PSB still at all times during transfer to avoid any unintended movement.**

If horizontal adjustment is required lower the user back to the surface.

**Do not attempt to adjust the tilt while the user is elevated.**


**Risk of pinching fingers when turning handwheel.**  
**Keep fingers away from bottom side of balance bar housing.**


**Risk of head injury.**  
**Be aware of any obstacle posing a risk of collision with user's head area during transfer.**

To raise the upper body, turn handwheel for upper body counterclockwise.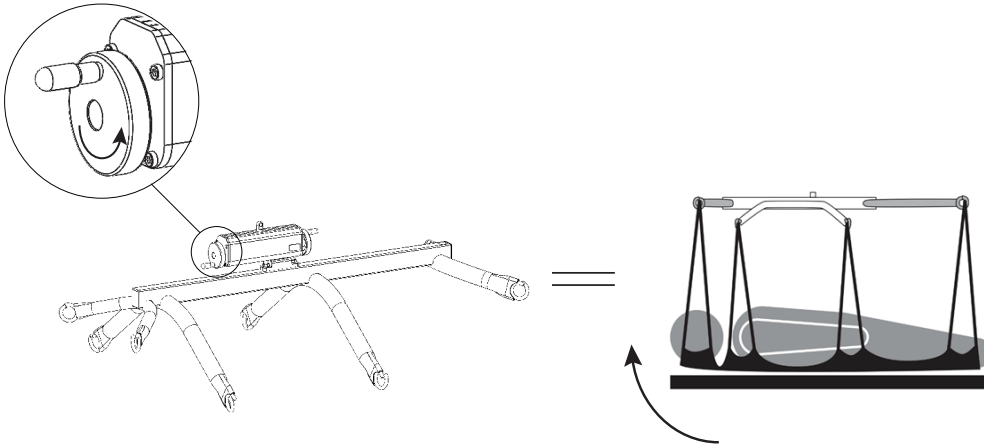

To lower the upper body, turn handwheel for upper body clockwise.

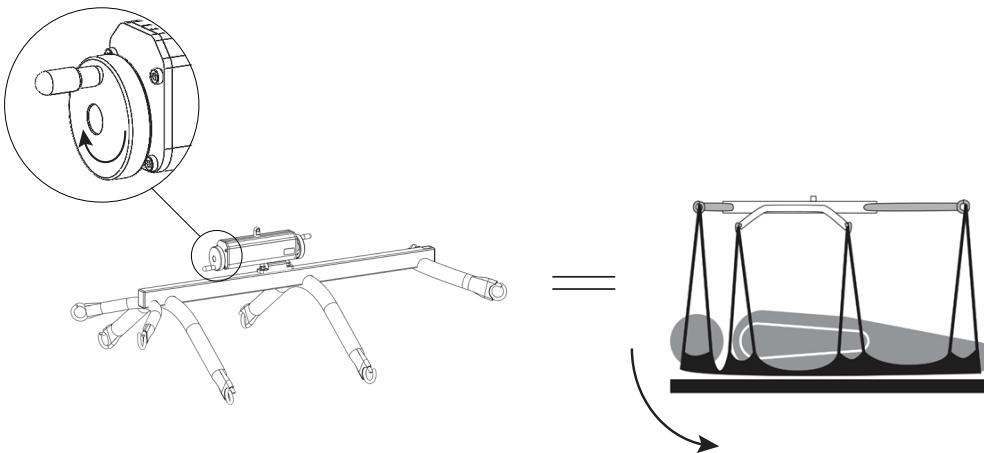## Maintenance and service

**Make sure cleaning does not damage or remove any labels.**

Clean on a regular basis. Clean surfaces with a damp cloth using an appropriate pH-neutral detergent. Do not use solvents or strong liquids, this may damage surfaces on the lifter. For disinfection when needed; Use isopropyl alcohol. Avoid abrasive cleaning products, and follow health and safety advice for the product.

### Service and periodic inspection

Service and periodic inspections is recommended to be carried out at planned intervals and by technical competent persons.

Specific periodic inspection checklist is available on [www.etac.com](http://www.etac.com).

## Troubleshooting

<b>Symptom</b>	<b>Possible cause / action</b>
The 8PSB does not slide smoothly inside of the aluminium profile, or the 8PSB sags to one side inside the aluminium bar.	The plastic rails, that the balance bar slides on are worn down and need replacement. Contact Your local Etac supplier for service and information.

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Etac A/S