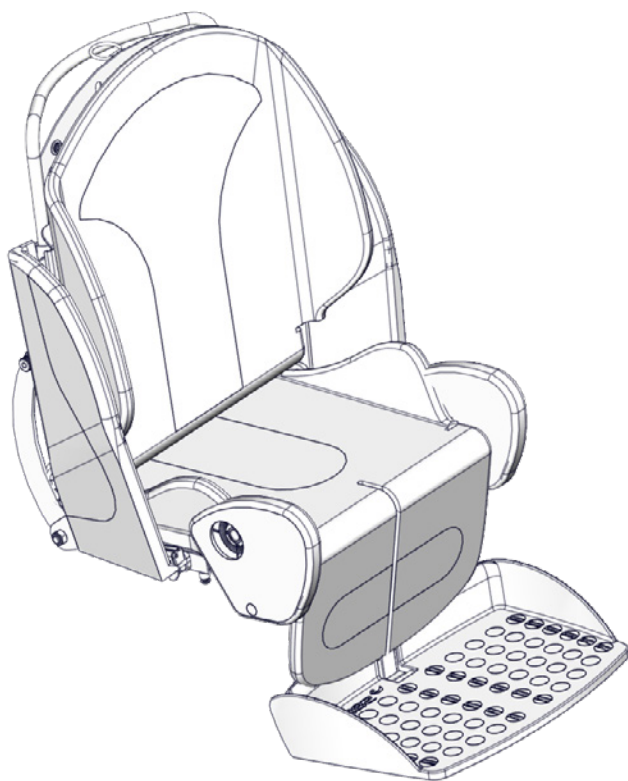
User manual

Stingray seat

R82[®]
by Etac

M1043 - 2023-05-03 - rev. 10


www.etac.com


	3	ENGLISH DEUTSCH	34 - 36 37 - 40
	4	NEDERLANDS DANSK	41 - 44 45 - 47
	5	NORSK SVENSKA	48 - 50 51 - 53
	6 - 7	SUOMI ESPAÑOL	54 - 56 57 - 60
	8	FRANÇAIS ITALIANO	61 - 63 64 - 66
	9	PORTUGUÊS PORTUGUÊS DO BRASIL	67 - 70 71 - 73
	10 - 11	РОССИЯ POLSKI	74 - 77 78 - 80
	12	ČESKY 中國	81 - 83 84 - 86
	13	日本 ΕΛΛΗΝΙΚΗ	87 - 91 92 - 95
	14 - 15		
	16		
	17		
	18		
	19		
	20		
	21 - 29		
mm (inch) kg (lb)	30		
	31-33		










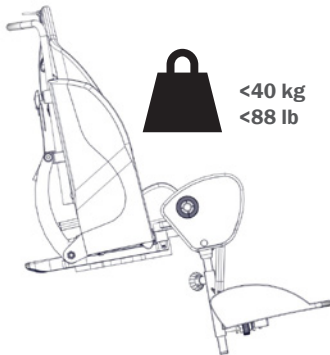
Stingray 1













Stingray 2

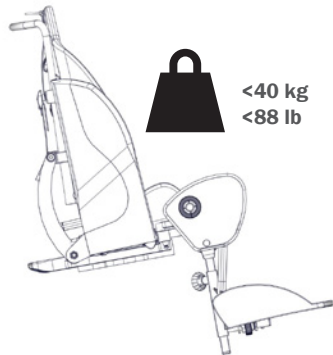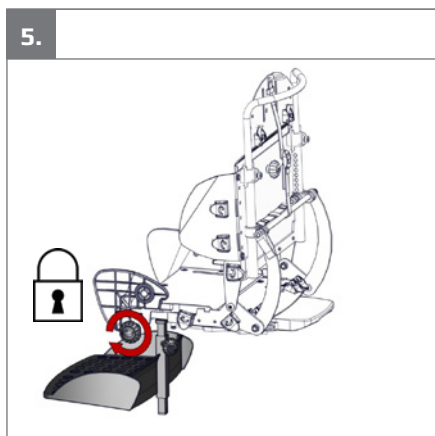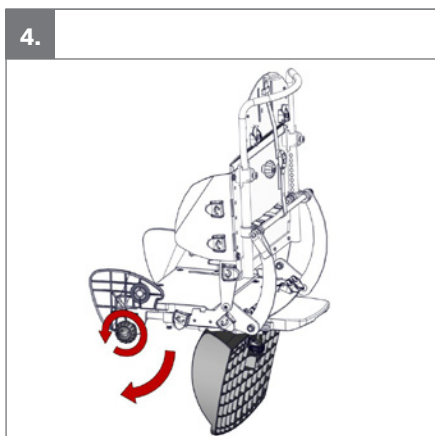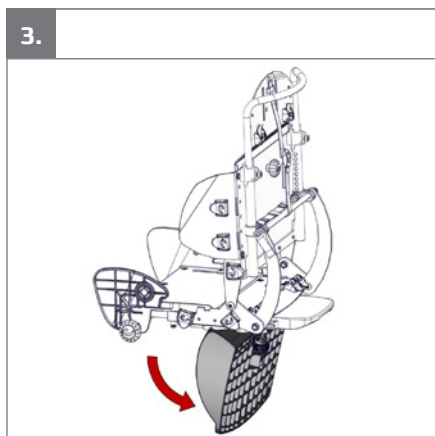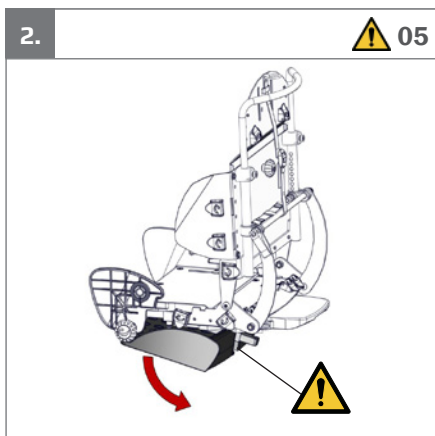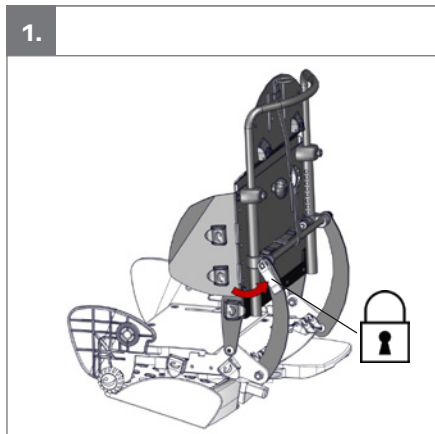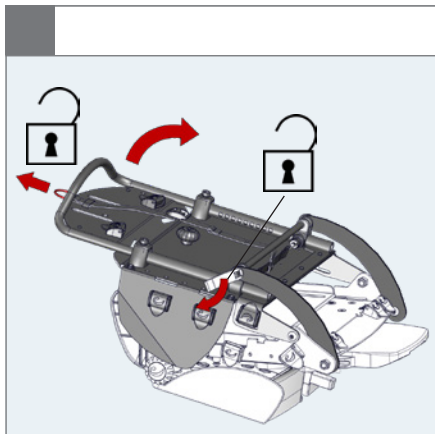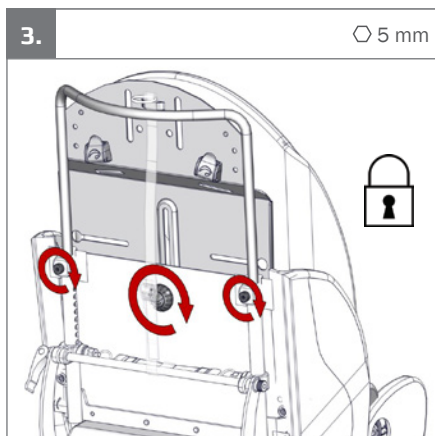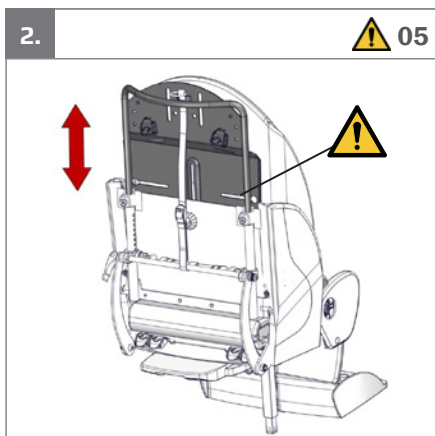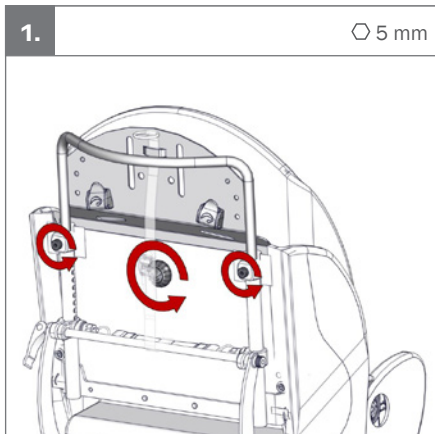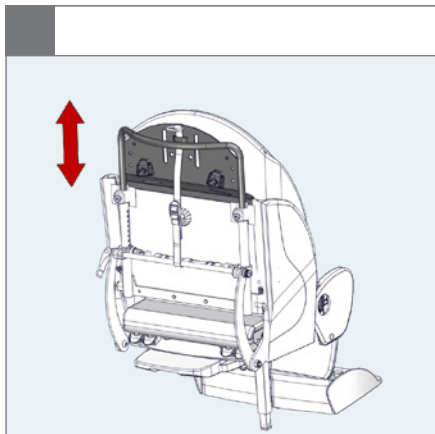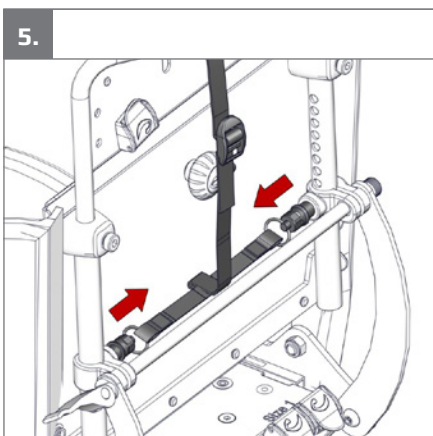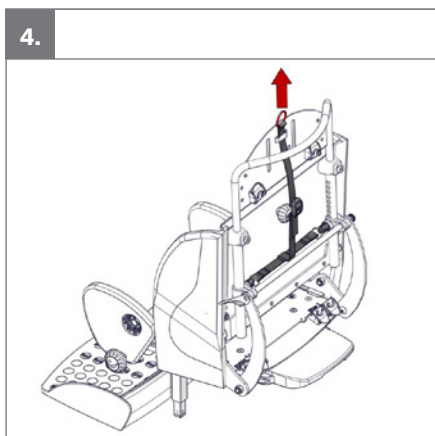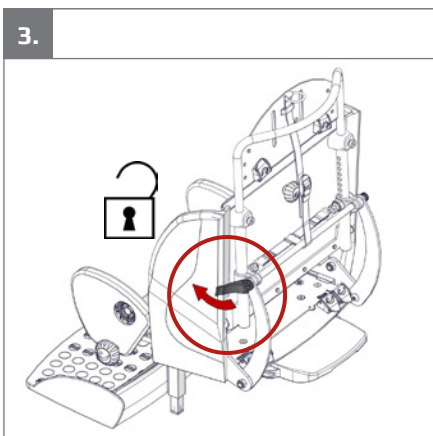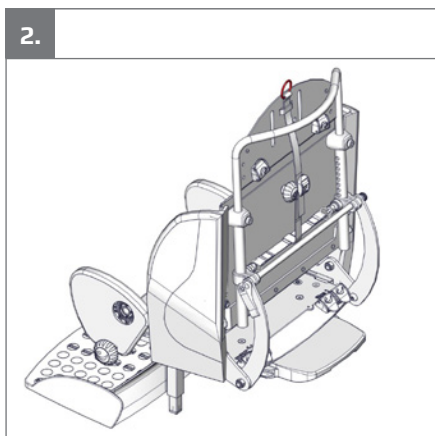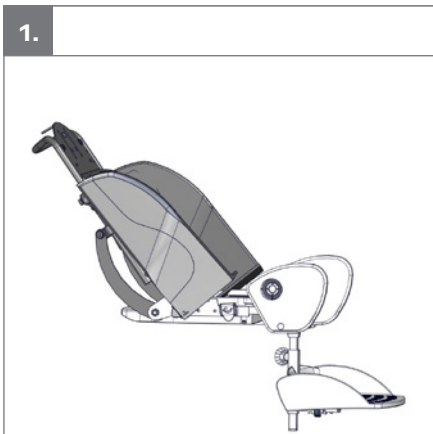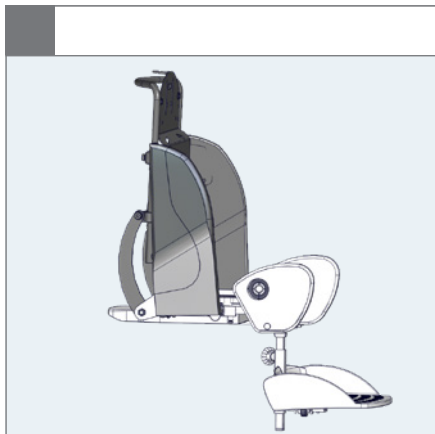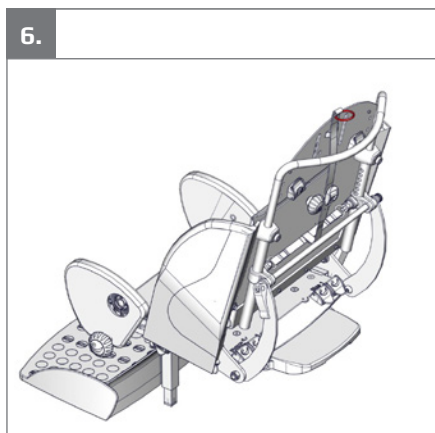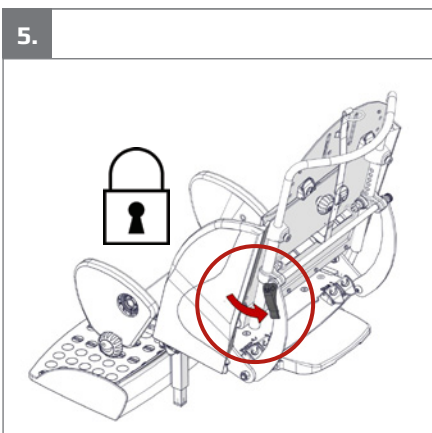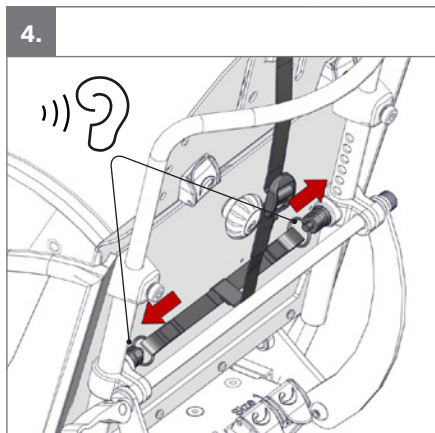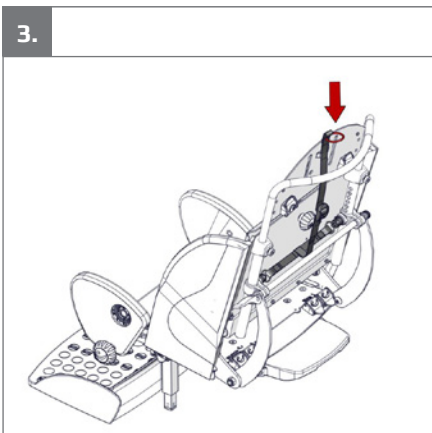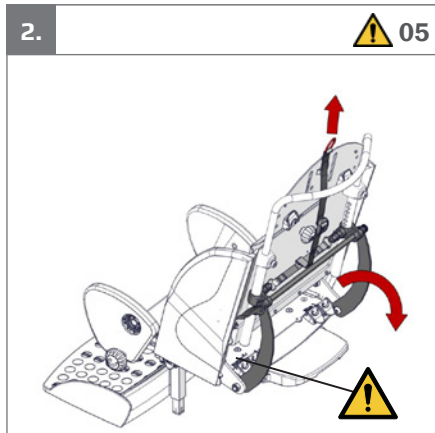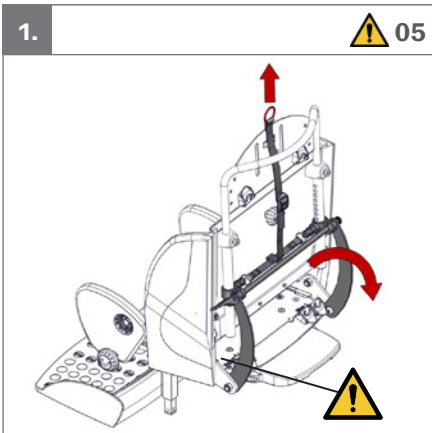
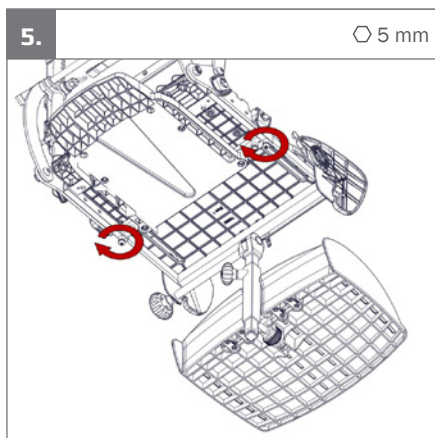
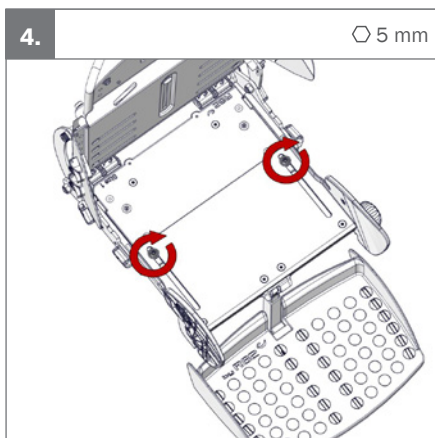
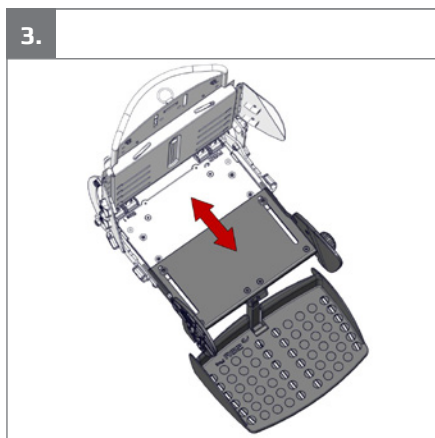
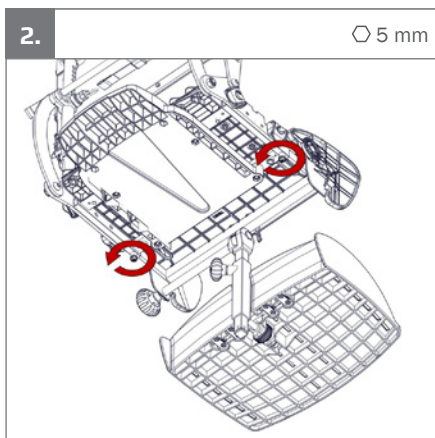
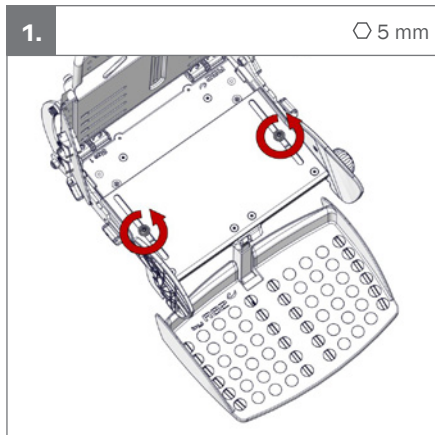
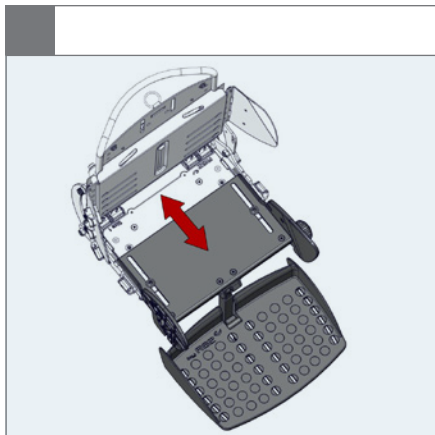
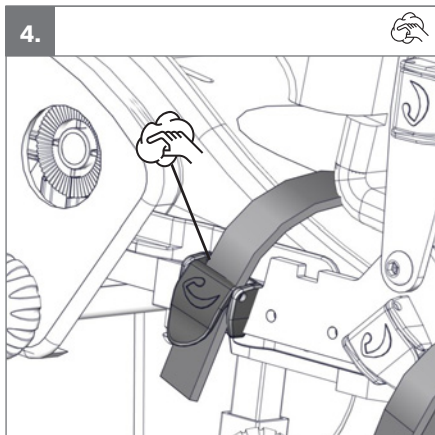
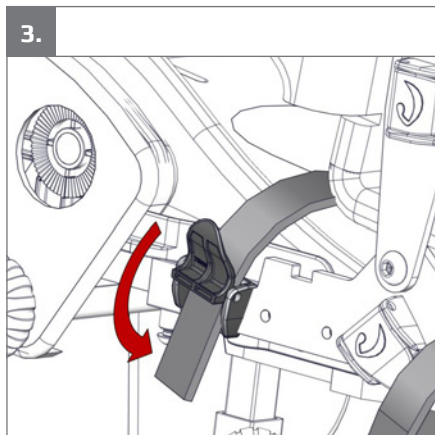
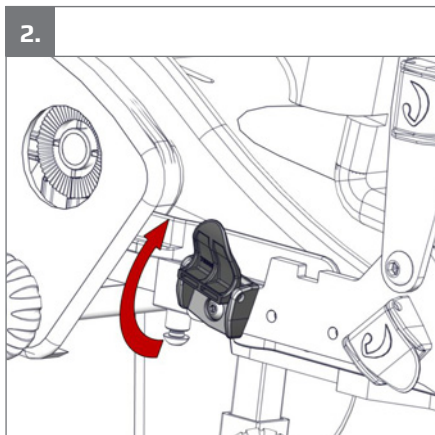
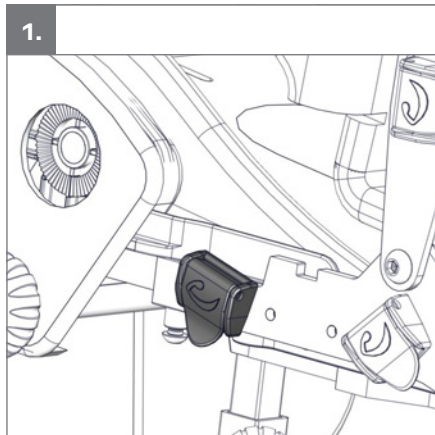
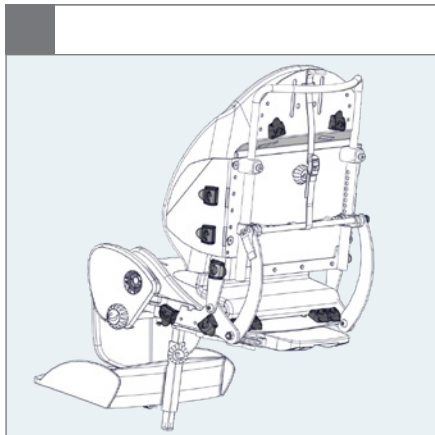
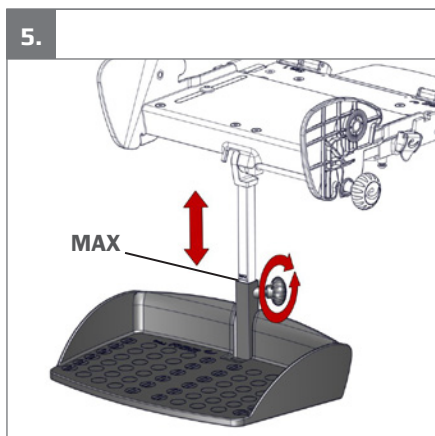
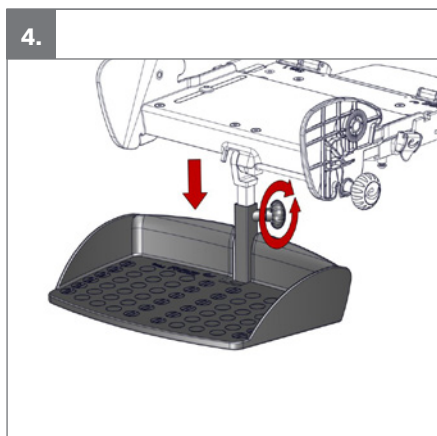
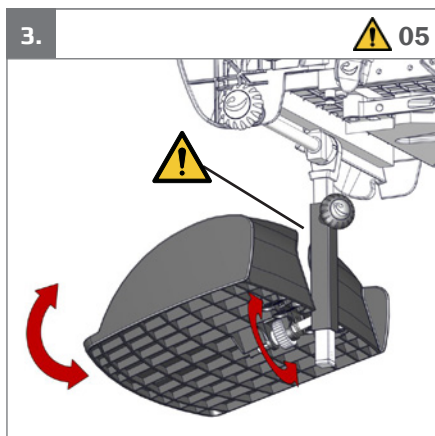
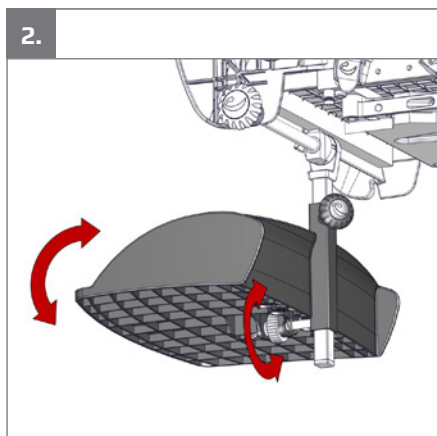
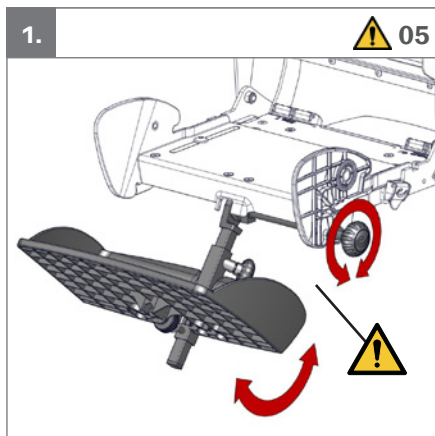
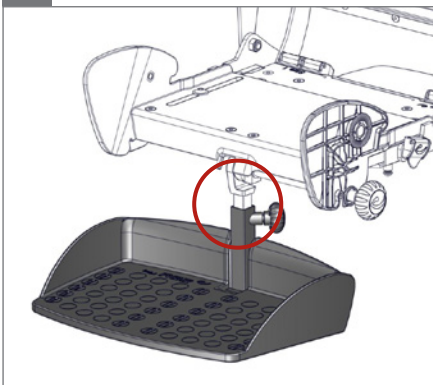
6.


7.

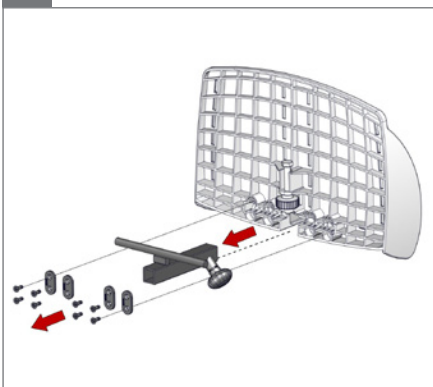
8.

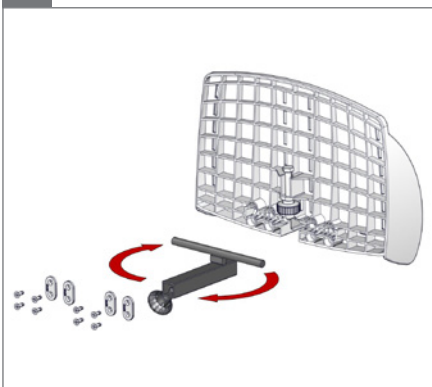
9.

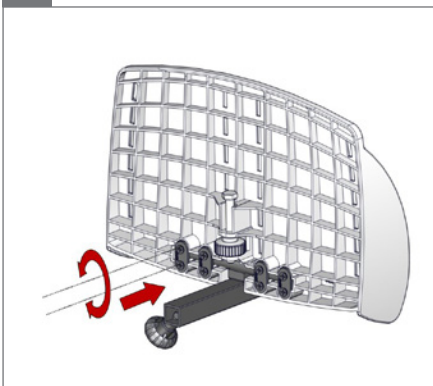10.

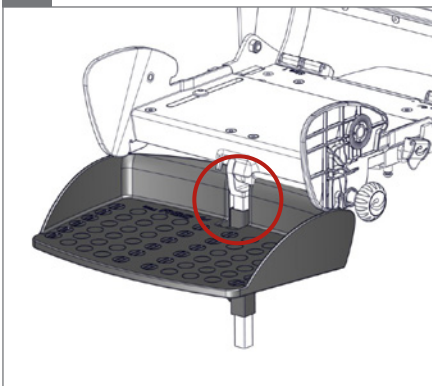
11.


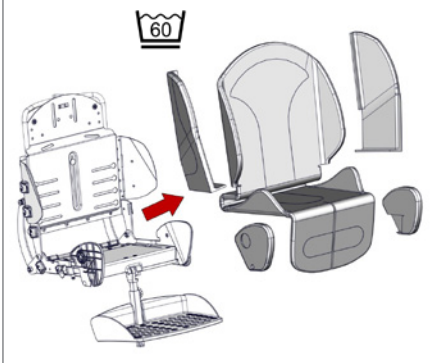
1.

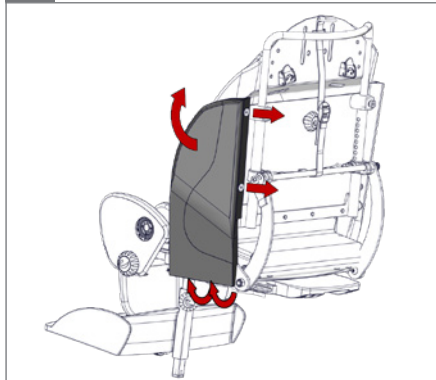
2.

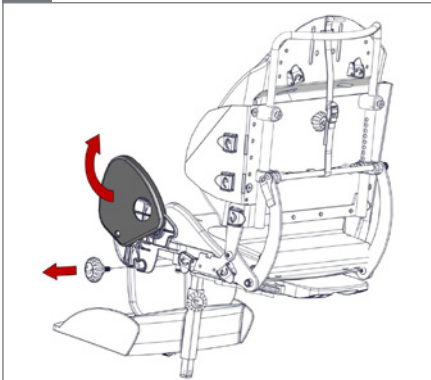
3.








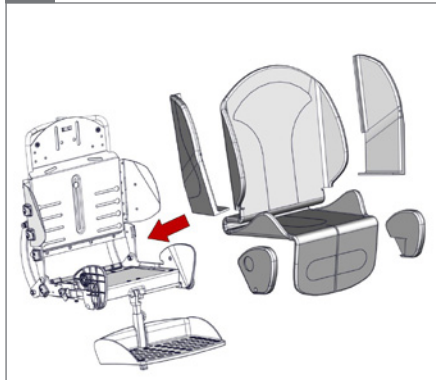
4.

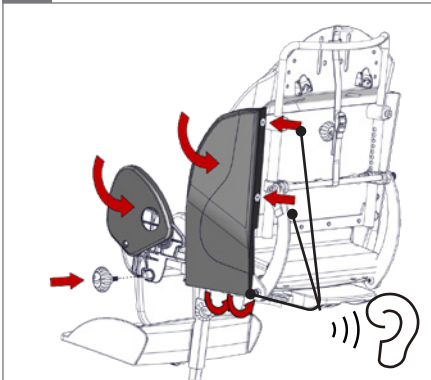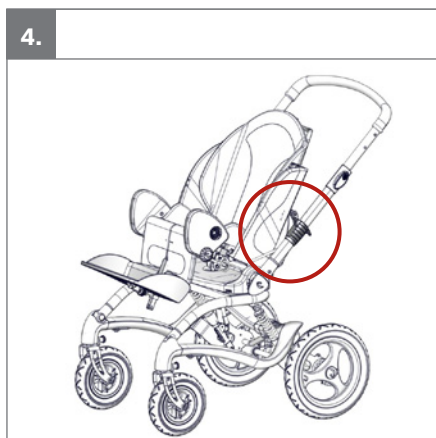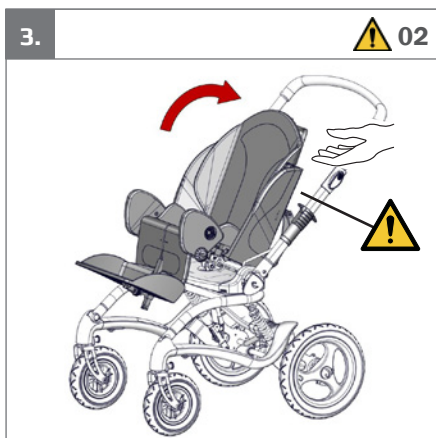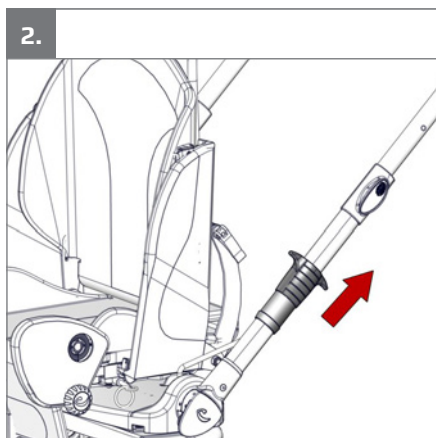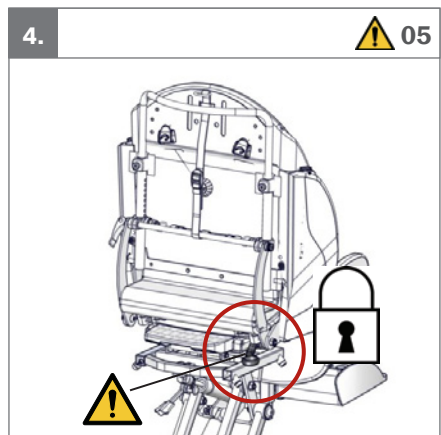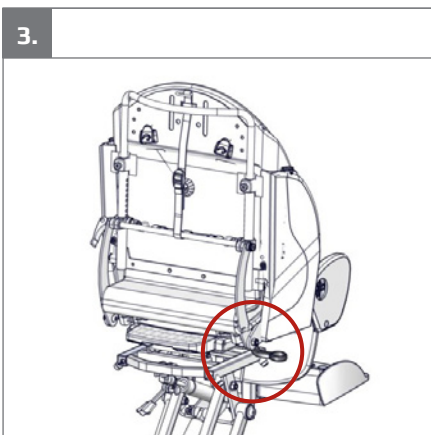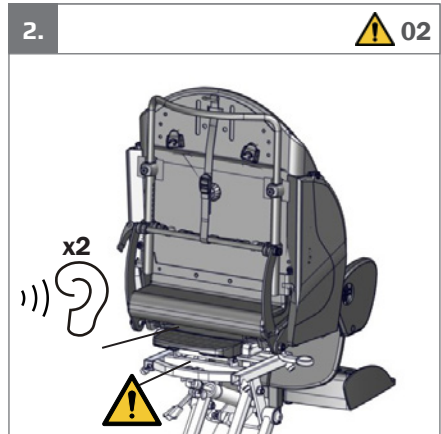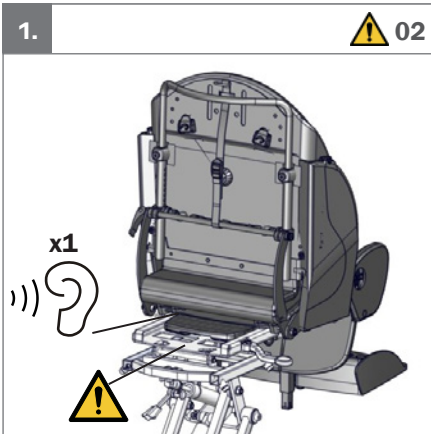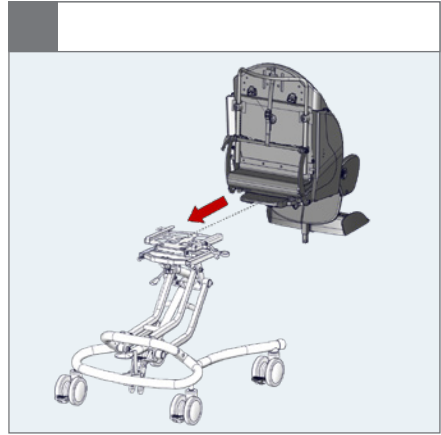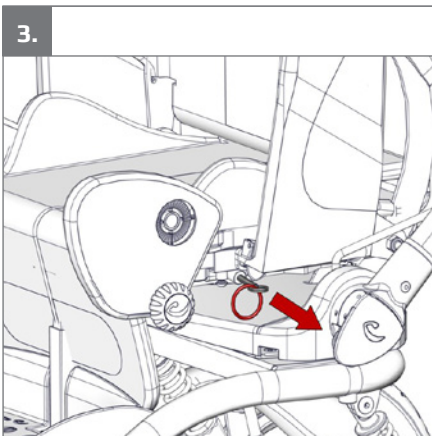
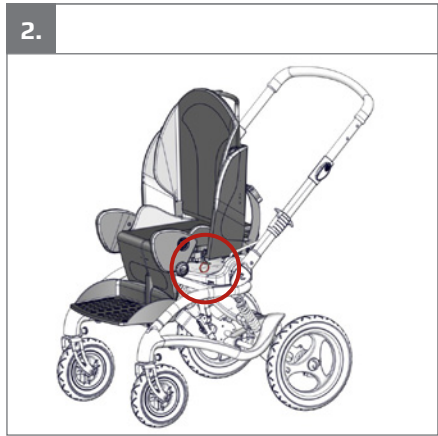
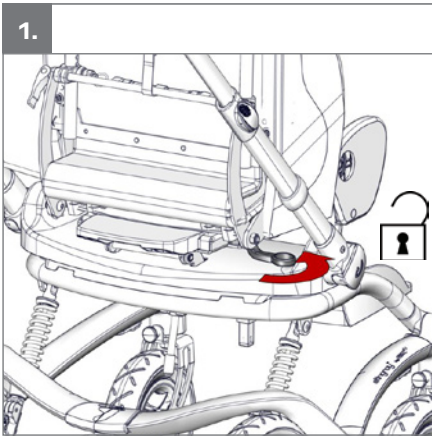
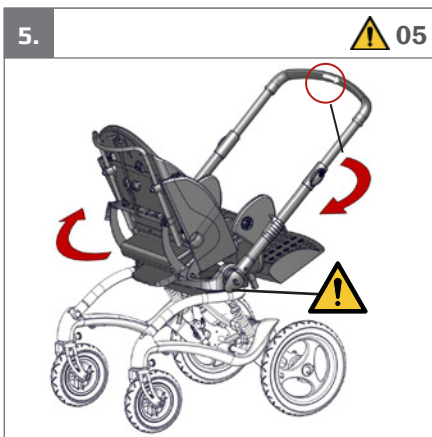
5.




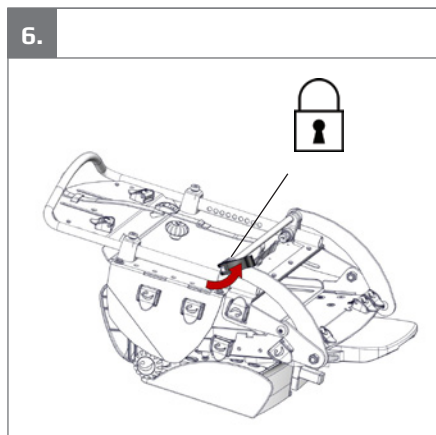
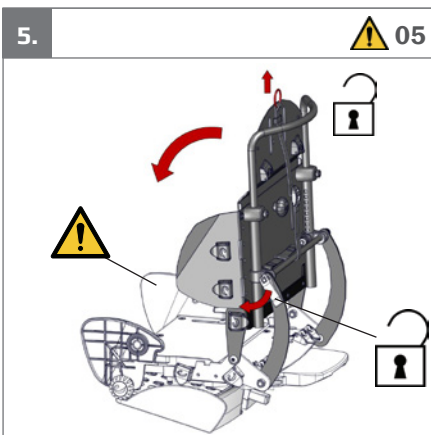
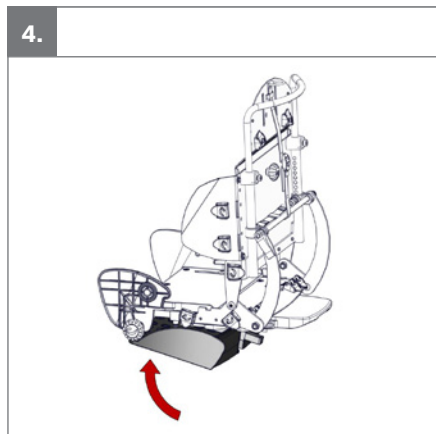
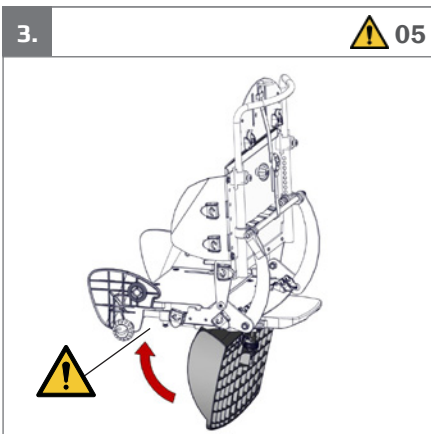
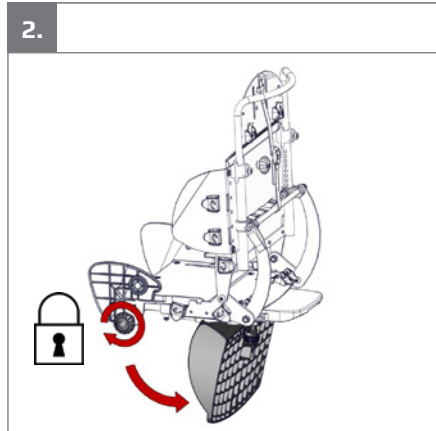
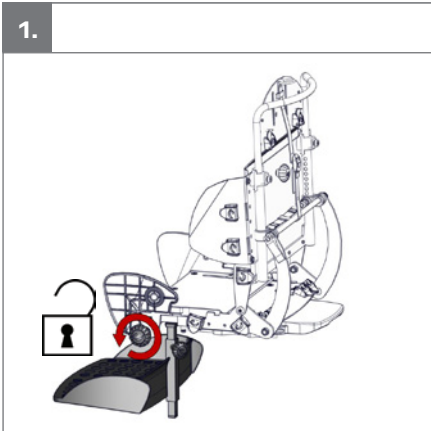
<40 kg
<88 lb

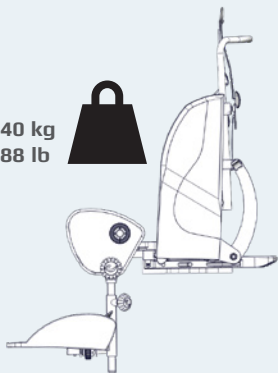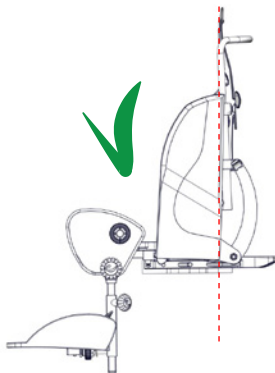
1.

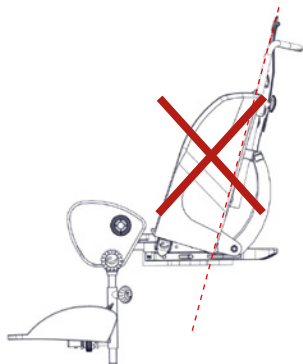
2.

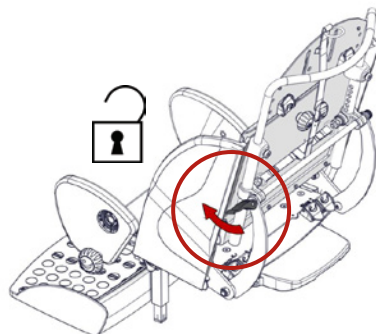
3.

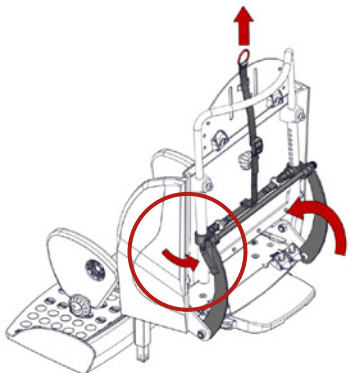
4.

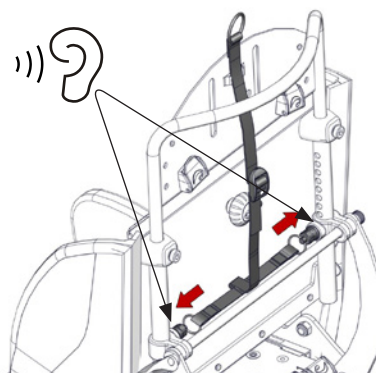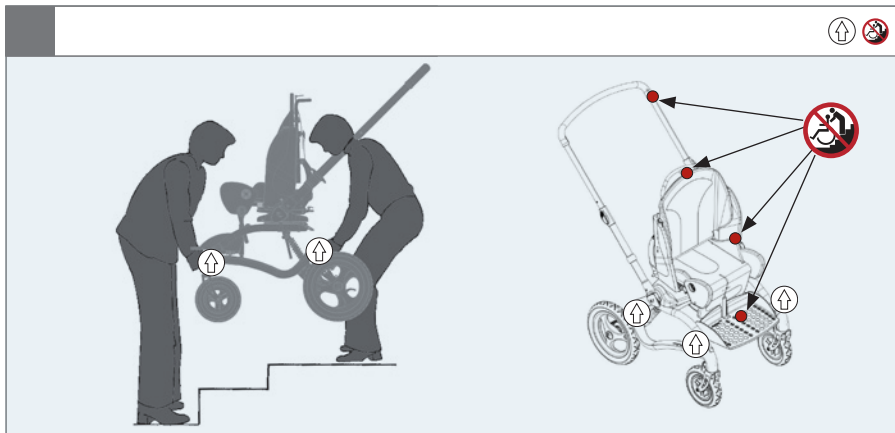5.

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

mm
(inch)
kg
(lb)

mm (inch)


	 1	 2
 Effective seat width	300 mm (11¾")	350 mm (13¾")
 Effective seat depth	180–300 mm (7–11¾")	250–400 mm (9¾–15½")
 Backrest height	410–550 mm (16–21½")	500–650 mm (19½–25¼")
 Backrest height at shoulders	270–410 mm (10¼–16¼")	360–510 mm (14¼–20")
 Back width	450 mm (17¾")	520 mm (20½")
 Effective back width	300 mm (11¾")	300 mm (11¾")
 Backrest angle	170°	170°
 Footrest to seat distance	140–450 mm (5½–17½")	140–450 mm (5½–17½")
 Leg to seat surface angle	(-30°)–(-30°)	(-30°)–(-30°)
 Folded width, seat	420 mm (16½")	475 mm (18½")
 Folded length, seat	575 mm (22½")	575 mm (22½")
 Folded height, seat	285 mm (11")	305 mm (12")
 Max user mass	40 kg (88 lb)	40 kg (88 lb)
 Max user mass at transportation	40 kg (88 lb)	40 kg (88 lb)
 Total mass	5.5 kg (12 lb)	6.5 kg (14.5 lb)
 Max rated load	47.5 kg (104.7 lb)	48.5 kg (106.9 lb)

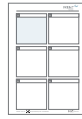
9996097170


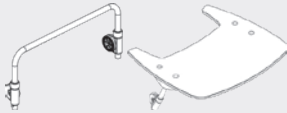
9996097171

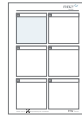9996097172

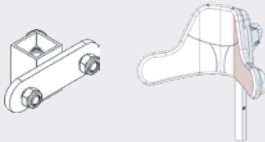
9996097173

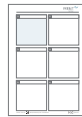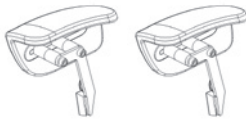9996097174

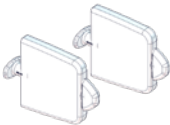
9996097175

9996097177

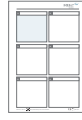
9996097179


9996097183


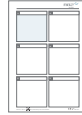
9996097184


9996097185

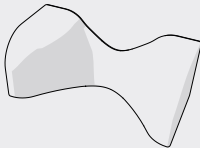
9996097227

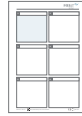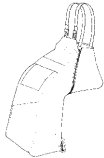
9996097608

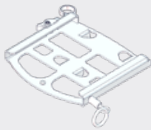
9996097734

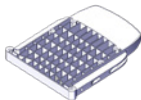9996097736

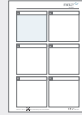9996097757

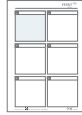9996097903

ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The Stingray has been designed for children who need good comfort and support. The seat depth and back height are easy to adjust, making it possible to expand the seat as when the child grows. The Stingray is suitable for users needing practical outdoor transportation, e.g. children with CP, GMFCS level 3-5. The Stingray comes in 2 sizes.

Enclosed tools:

- 3, 4, and 5 mm Allen key

CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12182
- EN 12183

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the

needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions. Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the “User Guide” carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Information about transportation can be found in the document “M1460 Transportation in Motor Vehicles” supplied with the product
- Information regarding frame and seat combinations including max. load in combination can be found at: www.etcac.com
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Ensure that moving parts does not squeeze or trap body parts when adjusting them

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- For transportation the product is suitable for children in the weight range of 22 kg to specified max user weight for transportation

Product

- Make sure the seat unit is securely mount-

ed in the frame interface. Check that the locking system work without obstructions from e.g. fabric or straps. Remove obstructions and activate the locking system

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Ensure the wheels are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1, EN 1021-2 and ISO 7176-16
- The product might lose its flame resistant characteristics if using other than original R82 cushions
- The product is approved for transportation in vehicles according to ISO 7176-19. More information can be found in the document “M1460 Transportation in Motor Vehicles” supplied with the product.

Even though the product complies with ISO-7176-19 then:

- Resistance to ignition can change with use, ageing and/or cleaning
- Precautions should be taken to avoid sources of ignition, including flames, cigarettes, etc..
- The environment (such as oxygen, alcohol, etc.) of the product occupant can increase the risk of ignitability

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded. The device may be reissued within its lifetime.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Seat back is adjustable and can be locked in position.
- Adjustment options for the foot rest.
- Product can be folded for storage or transportation.
- Nuts and bolts on the product are tightened.
- Check all fittings and buckles for signs of worn out parts.
- Product for cracks or signs for worn-out parts.
- All labels on the product are intact.
- Check that all wheels are moving freely and that all wheel locks/brakes are easily activated

Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to: Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use.

Carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Fabric and covers

The cover can be washed at max. 60° C using a mild detergent in a washing machine.

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Aluminium
- Plastic
- Steel
- Fabric

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

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Etac Ltd.
Gelbart AG
R82 A/S