User manual

Rabbit Up

R82[®]
by Etac

M1062 - 2023-12-12 - Rev. 8.0

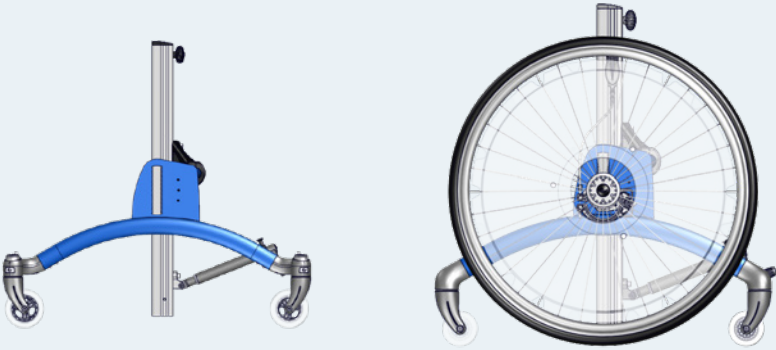
www.etac.com

	3	ENGLISH DEUTSCH	29 - 31 32 - 37
	4	NEDERLANDS DANSK	38 - 43 44 - 46
	5	NORSK SVENSKA	47 - 49 50 - 53
	6	SUOMI ESPAÑOL	54 - 56 57 - 62
	7	FRANÇAIS ITALIANO	63 - 68 69 - 73
	8	PORTUGUÊS PORTUGUÊS DO BRASIL	74 - 79 80 - 84
	9 - 10	РОССИЯ POLSKI	85 - 90 91 - 96
	11	ČESKY 中國	97 - 99 100 - 102
	12	日本 ΕΛΛΗΝΙΚΗ	103 - 108 109 - 114
	13		
	14 - 24		
 mm (inch) kg (lb)	25		
 mm (inch) kg (lb)	26		
	27 - 28		


1.


RABBIT 1
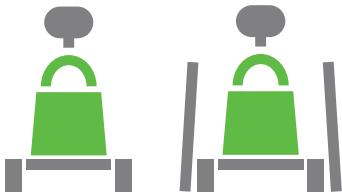
<40 kg
<88.1 lb

RABBIT 2
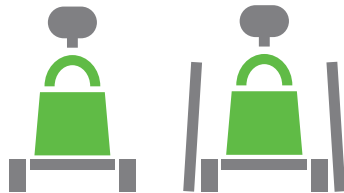
<50 kg
<110.2 lb

RABBIT 3

<60 kg
<132.2 lb


RABBIT 4

<70 kg
<154.3 lb

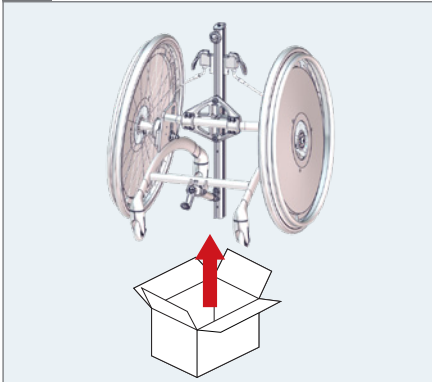
1a.


1b.

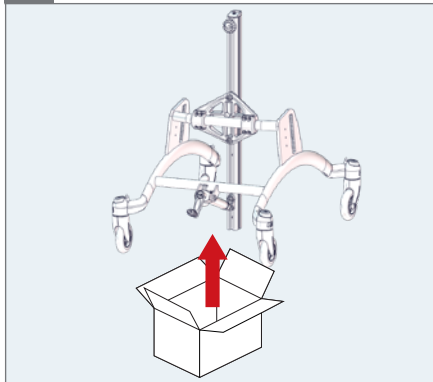
2a.








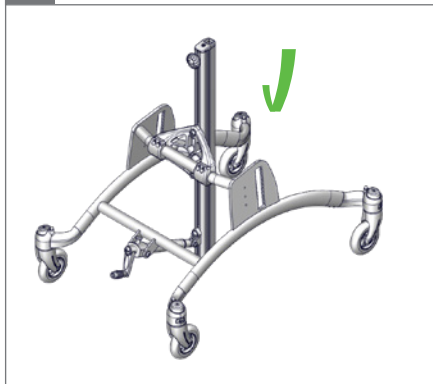
2b.












2c.

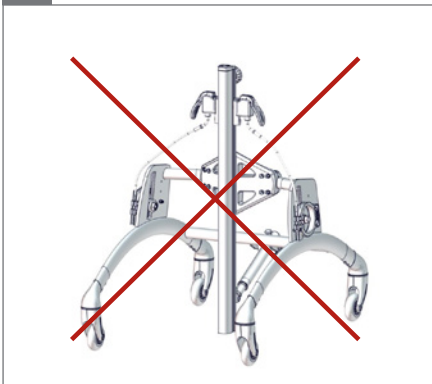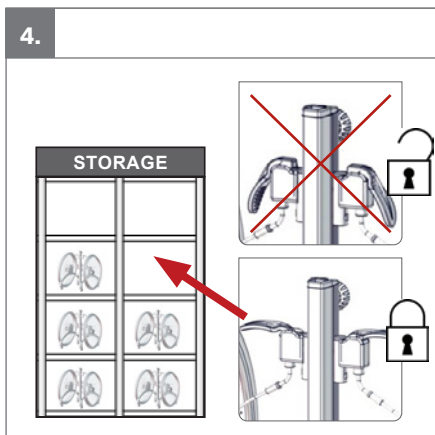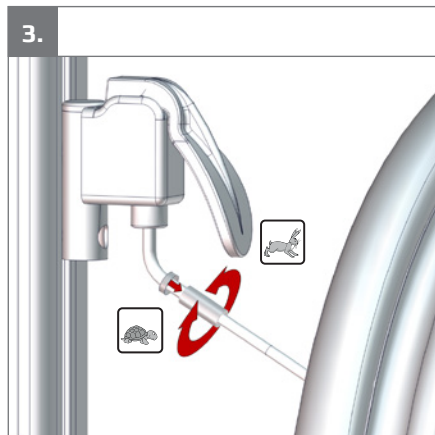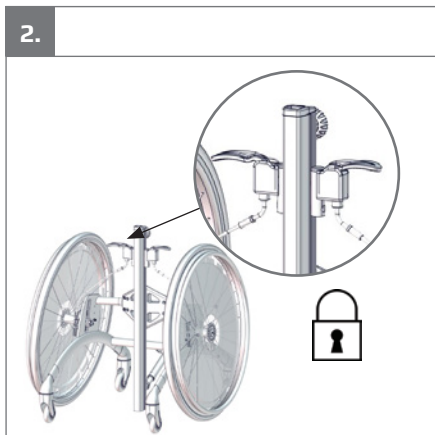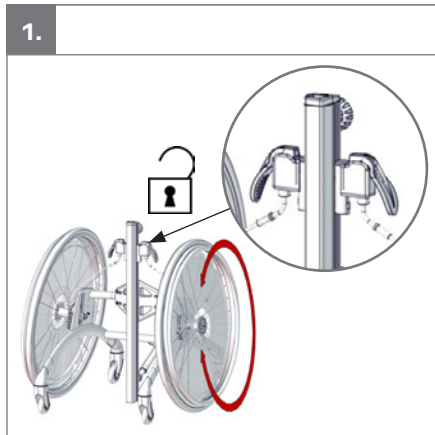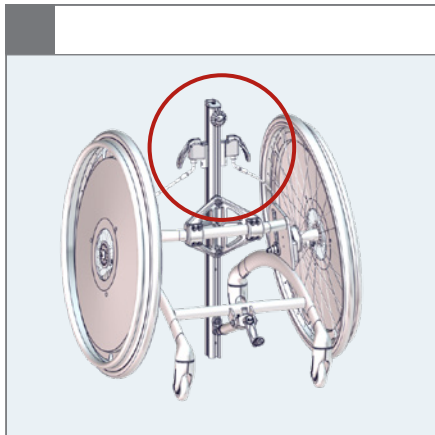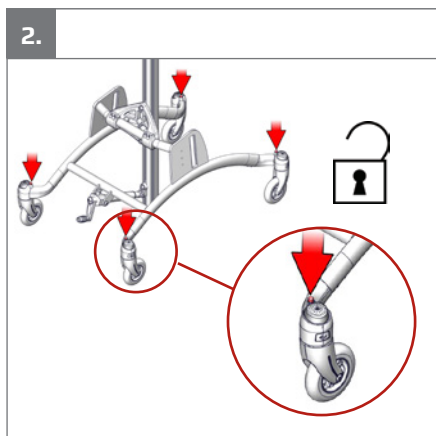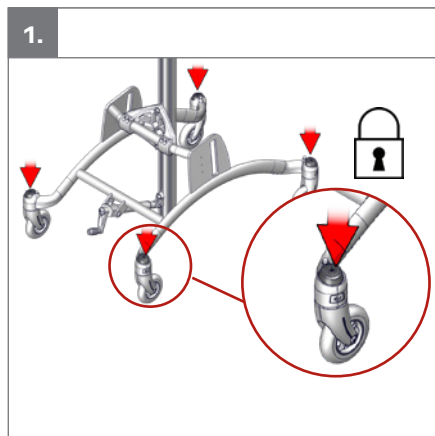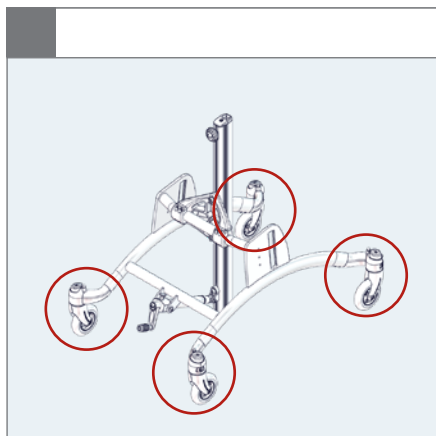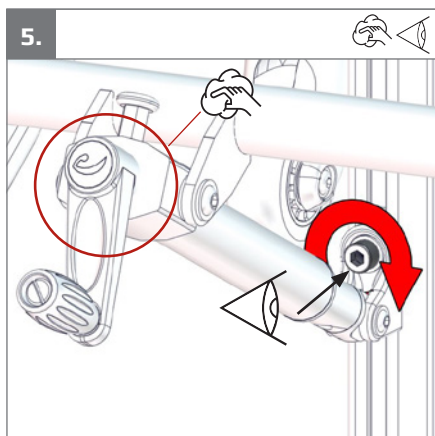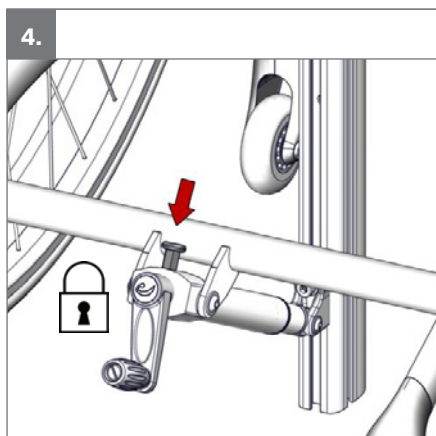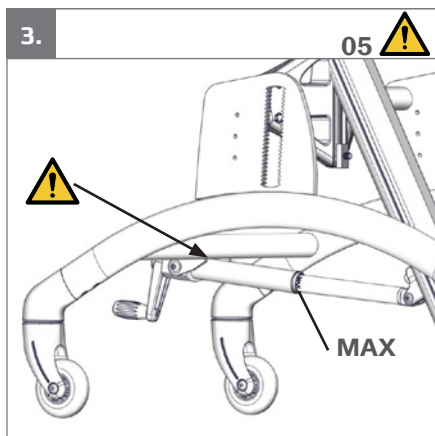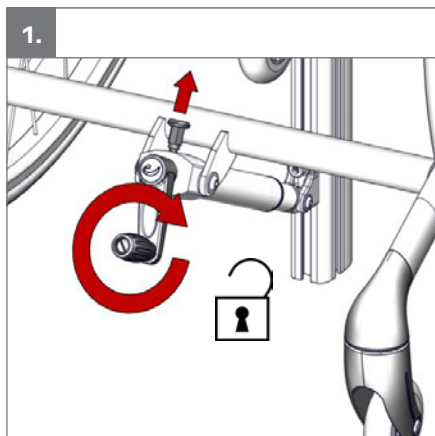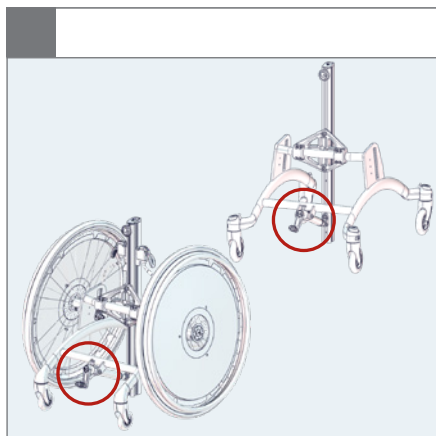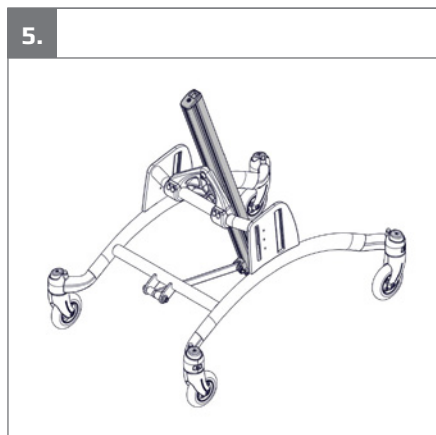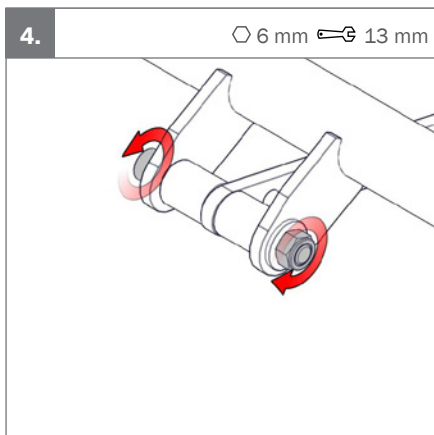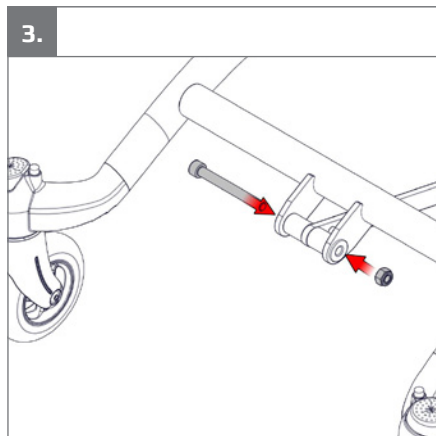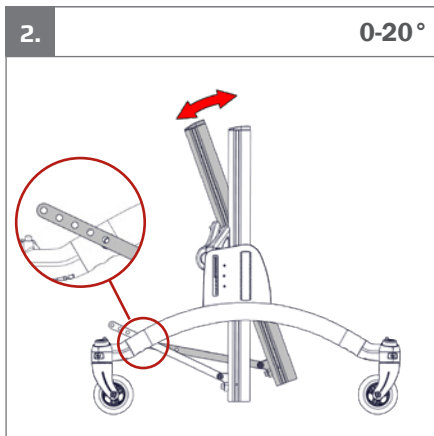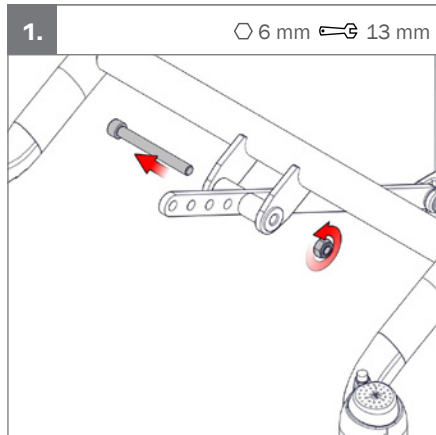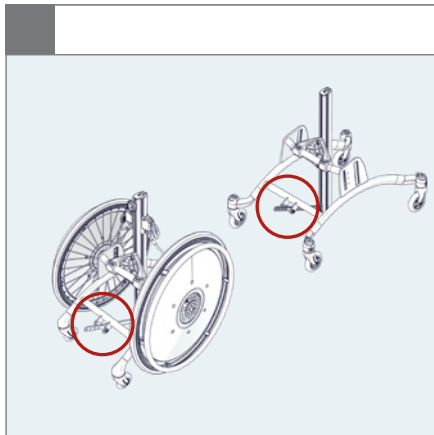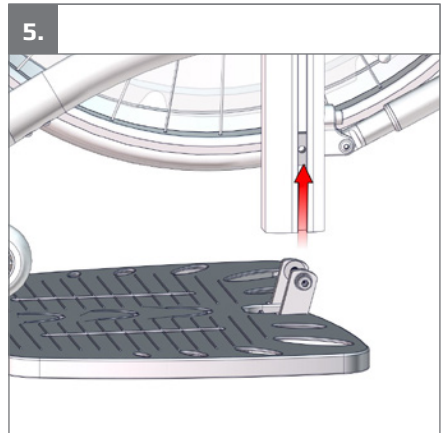
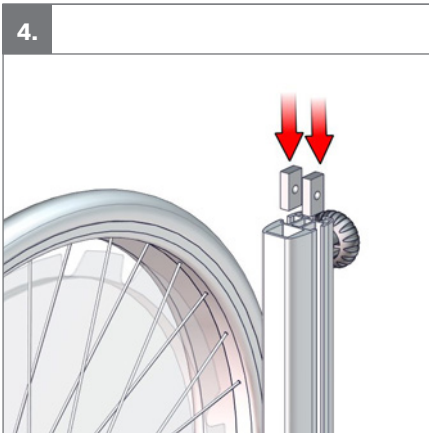
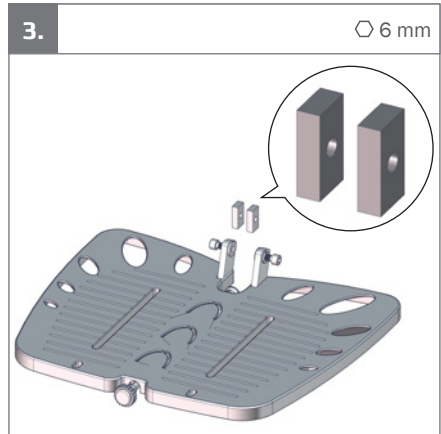
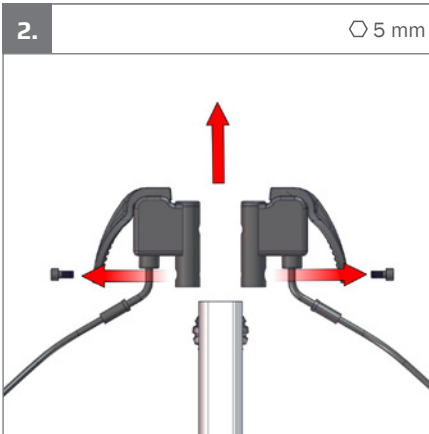
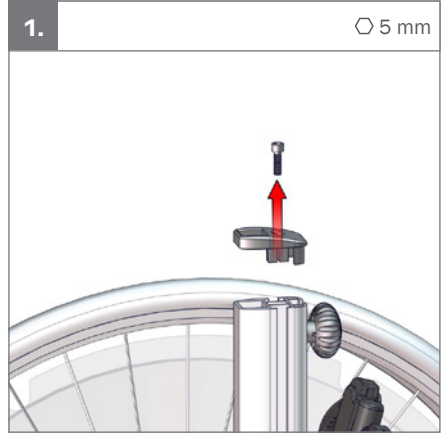
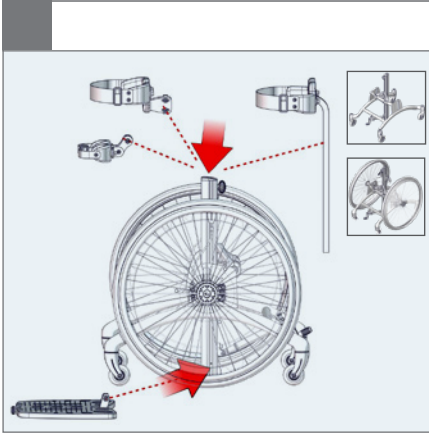
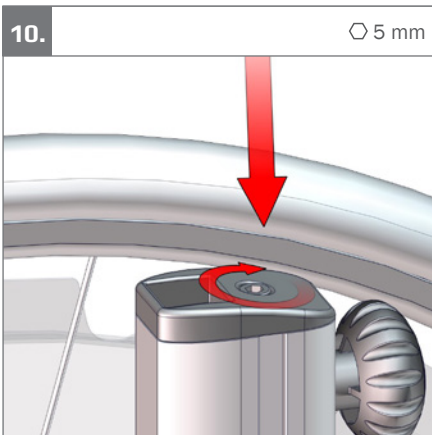
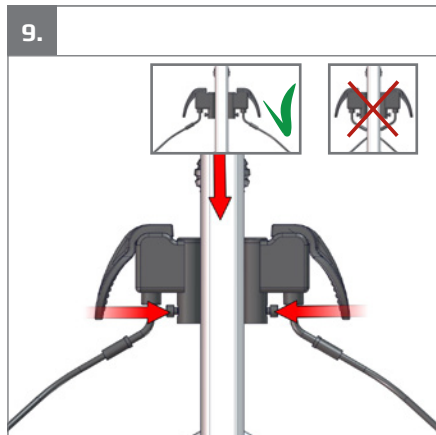
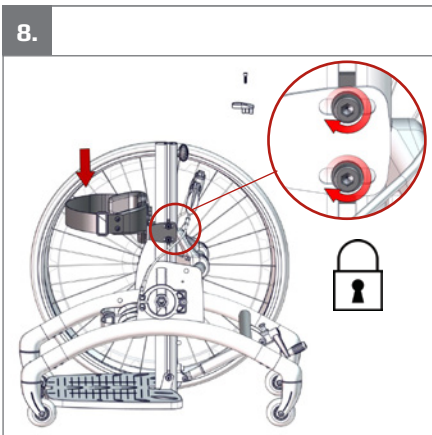
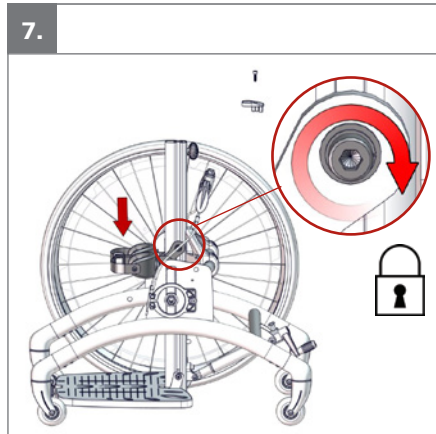
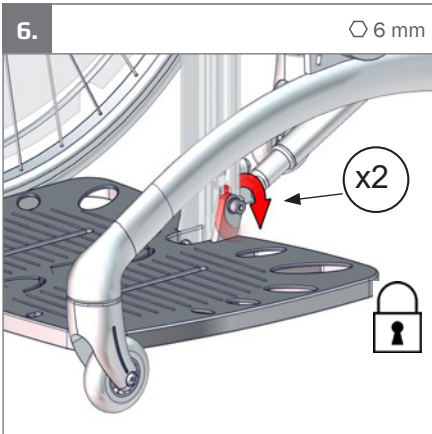
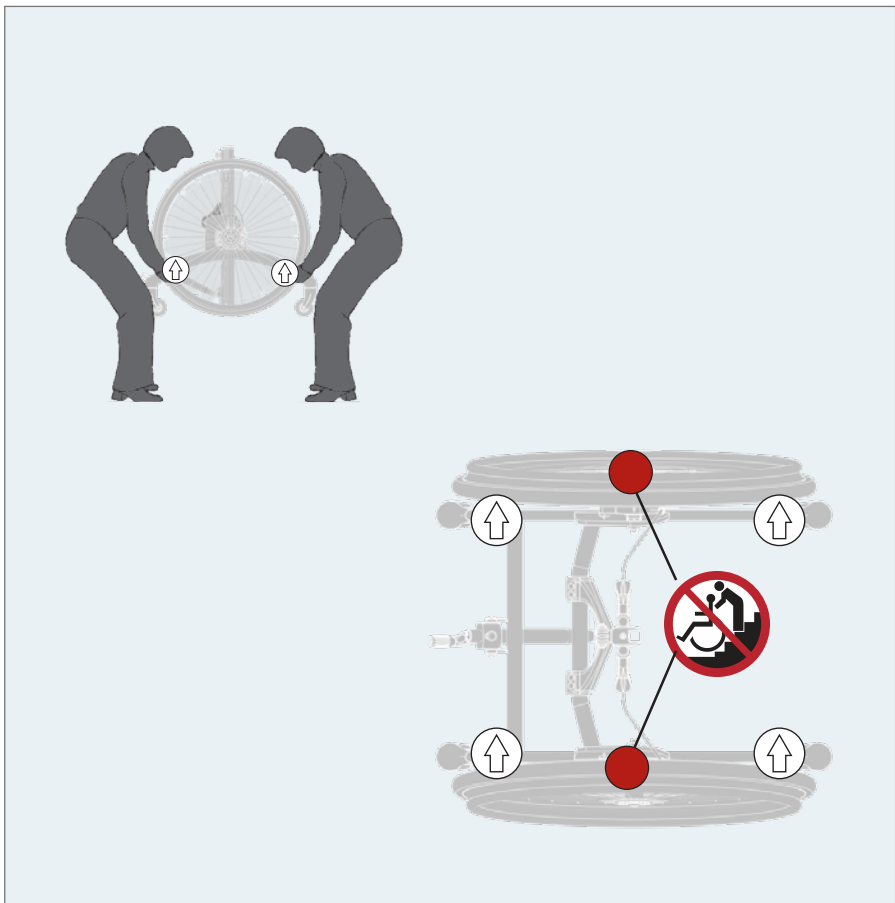
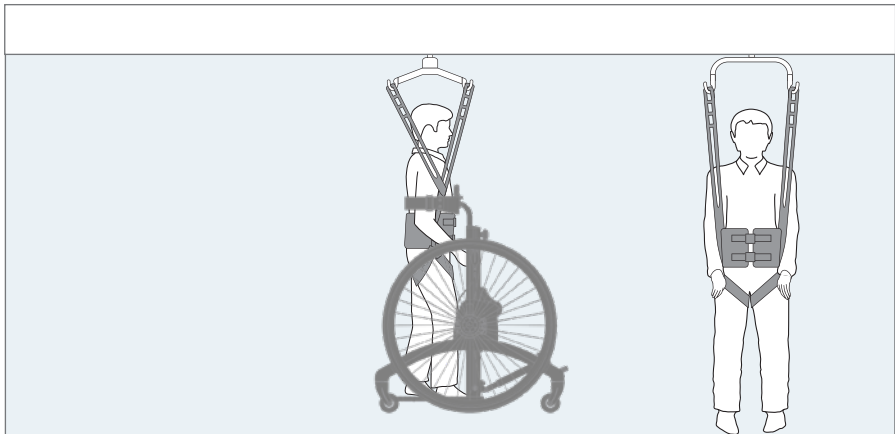
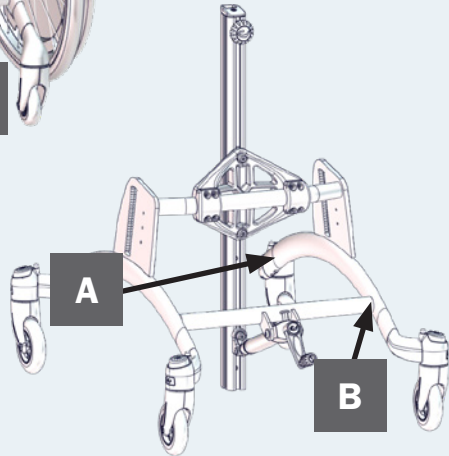
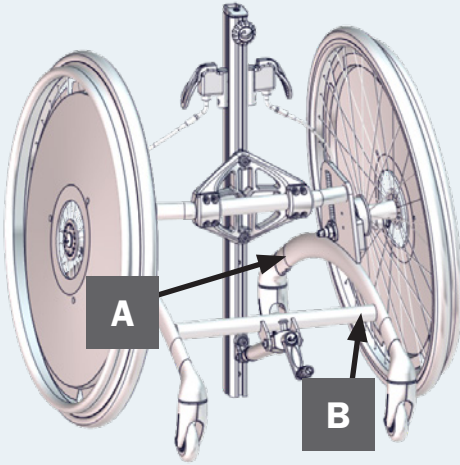
A

R82
by Etac

R82 A/S, Parallevej 3
DK-8751 Gedved
etac.com
Made in xxxxxxx

MD

CE

UK

YYYY-MM-DD

B

Posting group
Product name, size x

DMR ver.: xxxxx

Max. user mass: xxx kg

Max. rated load: xxx kg

Mass device: xxx kg

SR xxxxxxxxxxxx

REP xxxxxxxxxxxxxxxxxxxx

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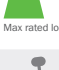
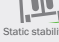
mm (inch) kg (lb)

	1	2	3	4
 Overall width	515 mm (20 $\frac{3}{4}$ ")	550 mm (21 $\frac{3}{4}$ ")	600 mm (23 $\frac{1}{2}$ ")	600 mm (23 $\frac{1}{2}$ ")
 Inside width	325 mm (12 $\frac{3}{4}$ ")	375 mm (14 $\frac{3}{4}$ ")	425 mm (16 $\frac{3}{4}$ ")	425 mm (16 $\frac{3}{4}$ ")
 Length	700 mm (27 $\frac{1}{2}$ ")	796 mm (31 $\frac{1}{4}$ ")	796 mm (31 $\frac{1}{4}$ ")	926 mm (36 $\frac{1}{2}$ ")
 Overall height	670–1000 mm (26 $\frac{1}{2}$ –39 $\frac{1}{4}$ ")	850–1180 mm (33 $\frac{1}{2}$ –46 $\frac{1}{2}$ ")	990–1310 mm (39–51 $\frac{1}{2}$ ")	1200–1530 mm (47 $\frac{1}{4}$ –60 $\frac{1}{4}$ ")
 User height, floor to axil	550–740 mm (21 $\frac{3}{4}$ –29 $\frac{1}{4}$ ")	750–1080 mm (29 $\frac{1}{2}$ –42 $\frac{1}{2}$ ")	830–1190 mm (33 $\frac{3}{4}$ –46 $\frac{3}{4}$ ")	960–1420 mm (37 $\frac{3}{4}$ –56")
 Length, center bar	500 mm (19 $\frac{3}{4}$ ")	700 mm (27 $\frac{1}{2}$ ")	800 mm (31 $\frac{1}{2}$ ")	1000 mm (39 $\frac{1}{4}$ ")
 Angle, center bar	0°–30°	0°–30°	0°–30°	0°–30°
 Foot plate angle	(15°) – (-10°)	(15°) – (-10°)	(15°) – (-10°)	(15°) – (-10°)
 Floor to foot plate	60 mm (2 $\frac{1}{4}$ ")	60 mm (2 $\frac{1}{4}$ ")	60 mm (2 $\frac{1}{4}$ ")	60 mm (2 $\frac{1}{4}$ ")
 Minimum Turning diameter	850 mm (33 $\frac{1}{2}$ ")	950 mm (37 $\frac{1}{2}$ ")	980 mm (38 $\frac{1}{2}$ ")	1085 mm (42 $\frac{3}{4}$ ")
 Mass device	9 kg (19.8 lb)	10.5 kg (23.1 lb)	11 kg (24.2 lb)	13.5 kg (29.3 lb)
 Max user mass	40 kg (88.1 lb)	50 kg (110.2 lb)	60 kg (132.2 lb)	70 kg (154.3 lb)
 Max rated load, no driving wheels	51 kg (112.4 lb)	62.5 kg (137.7 lb)	73 kg (160.9 lb)	85.5 kg (188.4 lb)
 Static stability downhill & uphill	10°	10°	10°	10°
 Static stability sideways	10°	10°	10°	10°

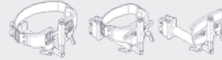
9996097635

9996097646


9996097636


9996097647

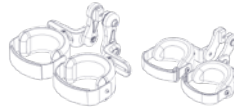9996097637

9996097248

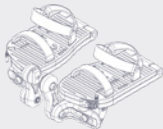
9996097638


9996097649

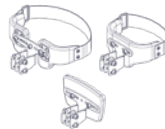9996097639

9996097651

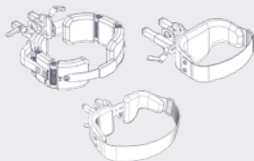
9996097642

9996097652

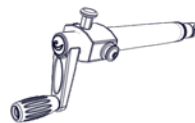9996097643

9996097656

9996097645

9996097655

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
9996097810

9996098152

9996098153

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and adults with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The device is a standing frame for children and adults with disabilities.

Indications

The device can be used by children and adults with disabilities, who experience limitations in their ability to maintain a standing position. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age. Rabbit Up mobile standing frame can also be used by children and adults with disabilities, who experience limitations in their ability to move around within the room or between rooms / outdoors.

The user is the person standing in the frame. The caregiver is the person operating the frame. Never leave the user unattended in this product. Ensure permanent supervision by a caregiver. The use of the device will be discontinued more times during the day where the user will be lying down or sitting in another assistive device or a conventional product.

Contra indications

There are no known contra indications.

Precautions

Special attention should be given to the considerations about the configuration of the product and the use of accessories for users with limitation in their:

- Sensory functions and the sensation of pain, consider extra checks of the pressure caused by the support provided by the product and its accessories.
- Movement function (e.g. experience involuntary movements).
- Ability to maintain a standing position, use the product with accessories (e.g. chest or back support).
- Ability to maintain the position of their head, consider to use accessories or to use a product that provide support to the positioning of the head.

Application areas

Rabbit Up (standing frame): For indoor use.

Rabbit Up (Mobile standing frame): For indoor and outdoor use (on even level and hard ground).

Enclosed tools:

- 5 mm Allen key
- 6 mm Allen key

CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12182

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions. Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user guide and who has been given the necessary instructions by a product advisor.
- Users who have difficulties reading the user guide should have someone read the user guide aloud to understand how to use the product.
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/ accessories.

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

User

- If the user is close to the max. load

limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it
- Ensure the wheels are fully-functional and secure before use
- When changing wheels or camber parts always refer to the mounting guide 9996097649 for correct placement and configuration. The mounting guide is always available at www.etac.com/ support.
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1 and EN 1021-2
- The product might lose its flame resistant characteristics if using other than original upholstery and/or accessories
- This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Reuse

This product and its accessories/components are suitable for reuse.

Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required
- Spoke guards is without cracks and secured to the wheel with the clips mounted on each guard
- Castors are moving freely and all wheel locks/brakes are easily activated
- The column angle is adjustable and it is possible to lock it in position
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Check all fittings and buckles for signs of worn out parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to: Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry

before use.

Carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Fabric and covers

The cover can be washed at max. 60°C using a mild detergent in a washing machine.

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Aluminium
- Steel
- Plastic
- Rubber
- Stainless steel
- Polyurethan
- Wood

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

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Etac Ltd.
Gelbart AG