## Strong Base Reconditioning

**Re-Use**

The Strong Base by R82 can be re-used. Please check the product's service life stated in the User guide. The latest version of the User Guide is always available at R82.org

**Cleaning****Washing**

Remove food particle accumulation and other excess dirt before washing. Pay special attention to wheels and wheel brakes.

Clean the frame with a pH neutral mix of warm water and non-abrasive cleaning agent or with a 70% disinfectant solution. Wipe and dry. A list of approved cleaning agents can be found at <http://www.r82.org/support/>

Do not clean any parts with a high pressure washer

**Ware-washing Process**

This product can be washed at 60° using a mild detergent in a washing machine designed for medical devices.

Remove all accessories before washing the main product and clean the accessories separately

Remove food particle accumulation and other excess dirt before washing. Pay special attention to wheels and wheel brakes.

Use the machine's drying function to dry the product

The ware-washing machine should be programmed at the following values:

- Water temperature: 60 °C
- Washing duration: 5 minutes
- Drying temperature: 50 °C
- Drying duration: 10 minutes
- Please allow the product to dry completely before reuse

**Check for defective or worn components**

Warped, cracked, rusty or deformed parts must be replaced. Worn and malfunctioning parts should be replaced. Minor damage to painted surfaces can be touched up with touch-up paint.

The CE mark must be removed when using other than original R82 spare parts and fittings. Original R82 parts have been tested together with the product for the intended purpose and have undergone statutory test procedures.

**Check**

Check and adjust the following functions as per their intended use.

All adjustments and settings must be performed in accordance with the User Guide. Please check that the:



- A. Test and check that the spindle for height adjustment works (only for products with a spindle for height adjustment)
- B. The holes in the frame intended for height adjustment. The thread for fasteners in each hole must be in good condition.
- C. Angling of the seat plane is working and it is possible to lock it into position
- D. The locking mechanisms on the castors work
- E. When prescribing the product, make sure the User Guide and that all labels are intact

Remember that the service life of the product is limited. This is particularly important when prescribing the product to a user who is expected to use it for an extended period

**Disposal**

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally friendly way.

- A. Spindle
- B. Height adjustment
- C. Seat angling
- D. Castors
- E. Identification labels, Serial number and manufacturer

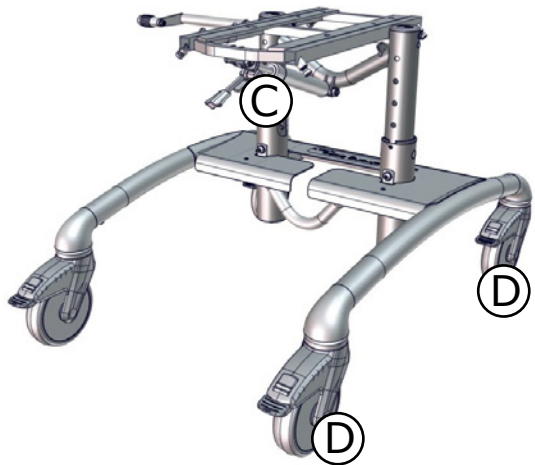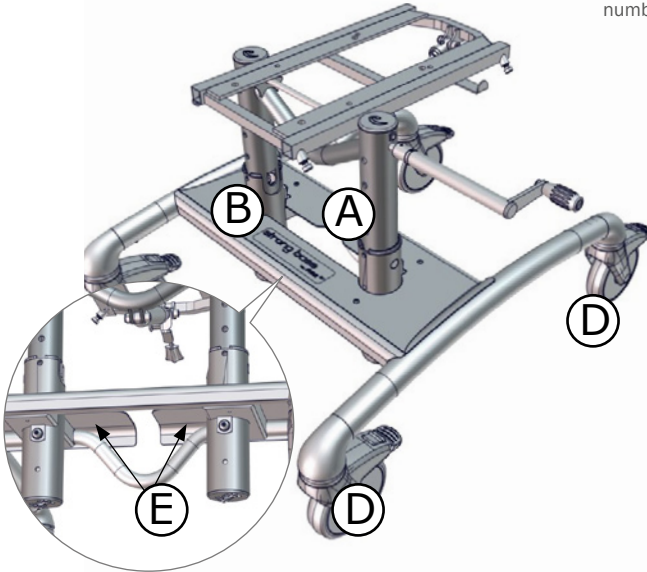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#### R82 GmbH
#### R82 Nederland B.V.
#### R82 au Pty Limited