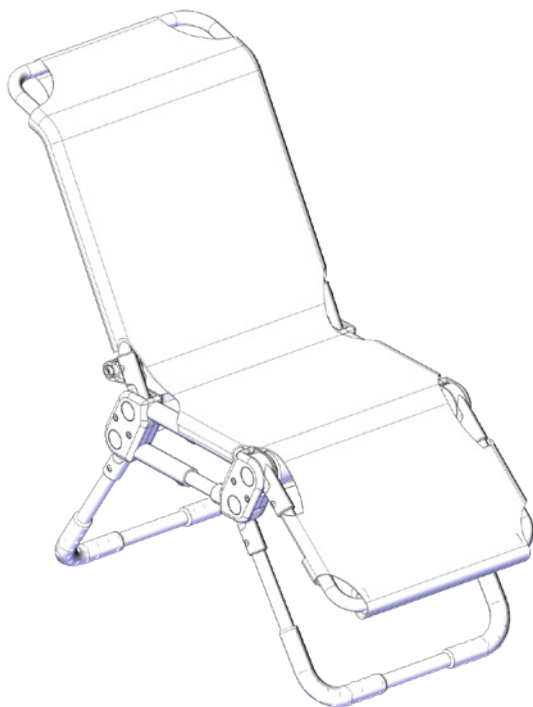
User manual

Manatee

R82[®]
by Etac


M1320 - 2023-05-01 - rev. 8.0


www.etac.com


	3	ENGLISH DEUTSCH	19 - 23 24 - 28
	4	NEDERLANDS DANSK	29 - 33 34 - 38
	5	NORSK SVENSKA	39 - 43 44 - 48
	6	SUOMI ESPAÑOL	49 - 53 54 - 58
	7	FRANÇAIS ITALIANO	59 - 63 64 - 68
	8	PORTUGUÊS PORTUGUÊS DO BRASIL	69 - 73 74 - 78
	9	РОССИЯ POLSKI	79 - 83 84 - 88
	10	ČESKY 中國	89 - 93 94 - 98
	11	日本 ΕΛΛΗΝΙΚΗ	99 - 104 105 - 109
	11	ÍSLENSKA	110 - 114
	12		
	13 - 15		
mm (inch) kg (lb)	16 - 17		
	18		

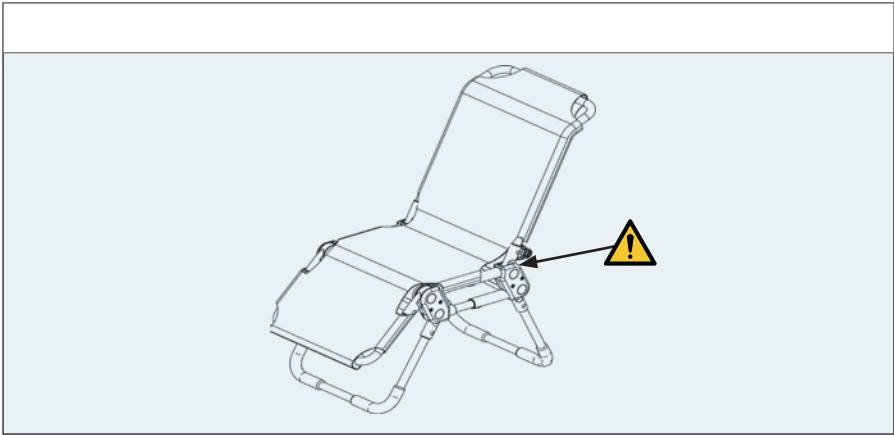
MANATEE 0

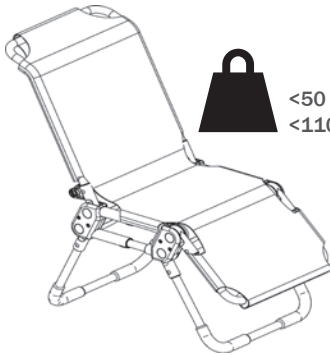
MANATEE 1












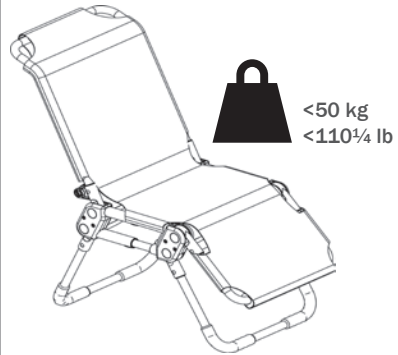
MANATEE 2












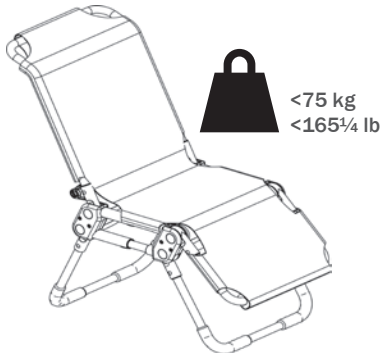
MANATEE 3

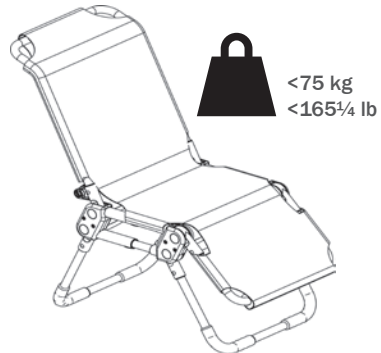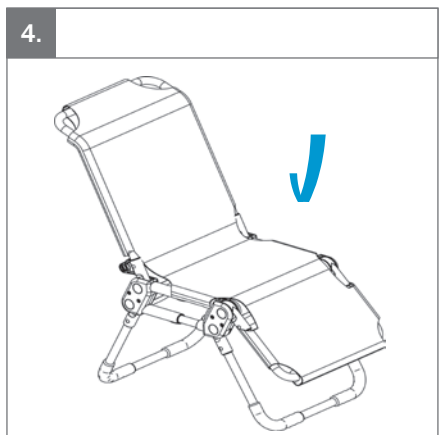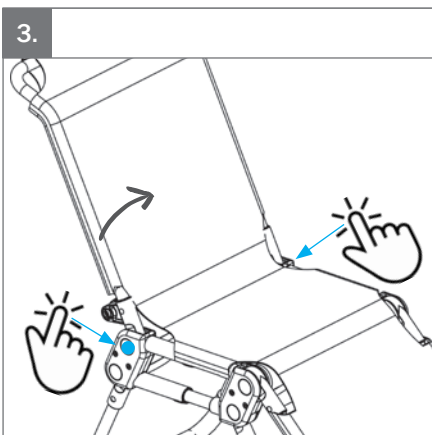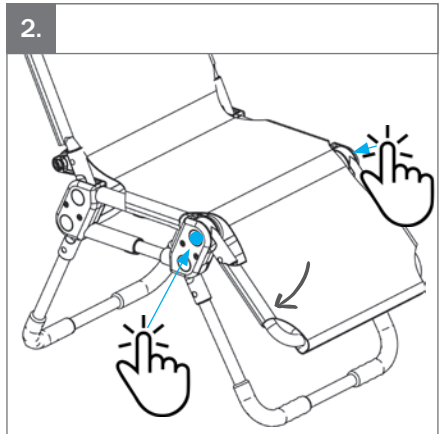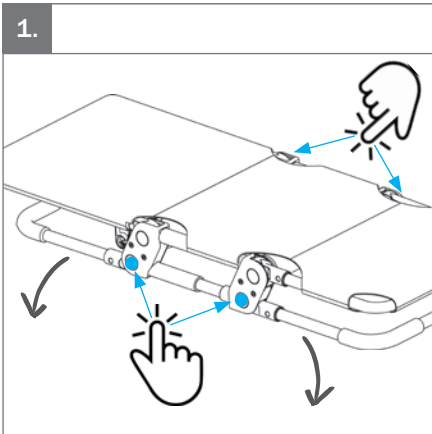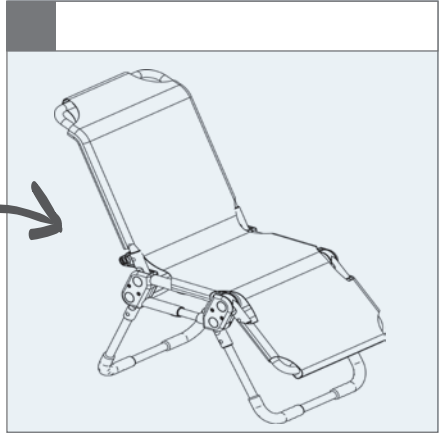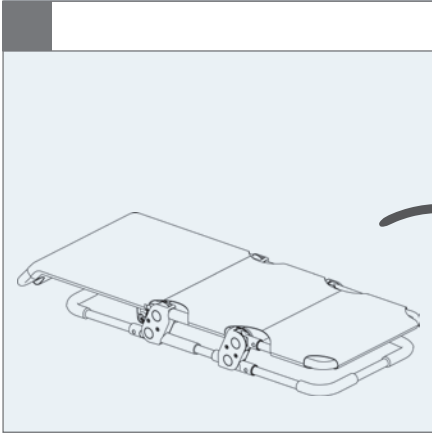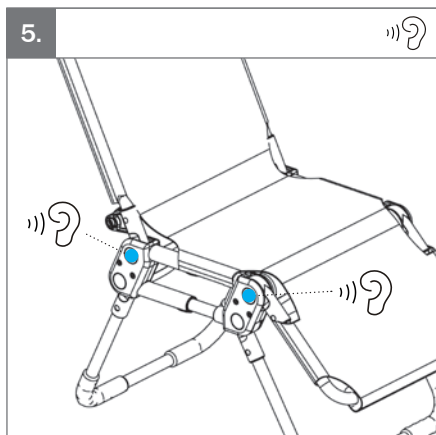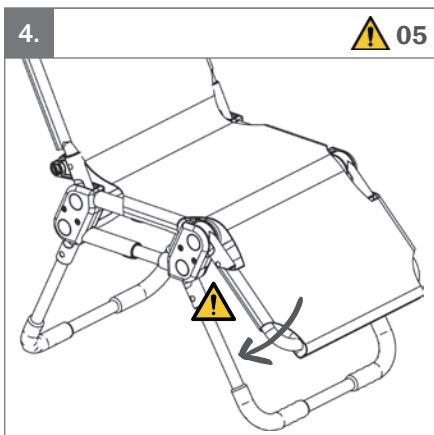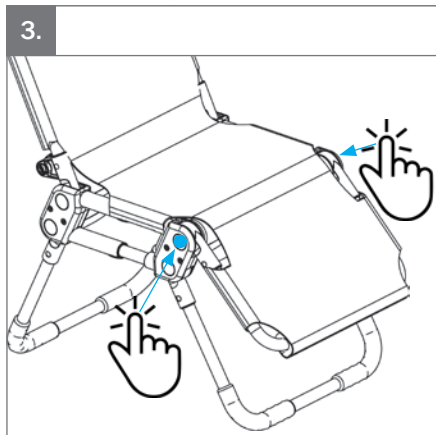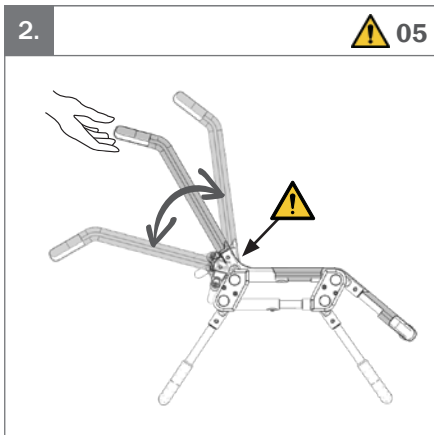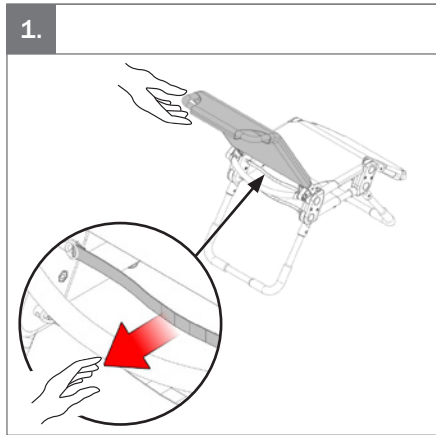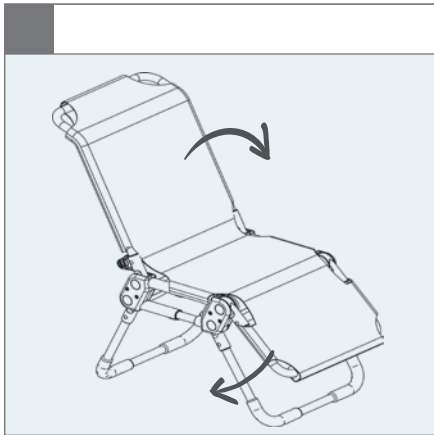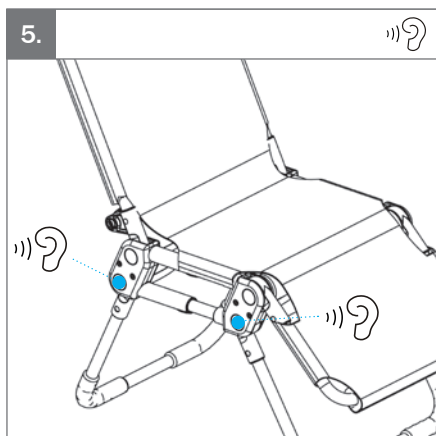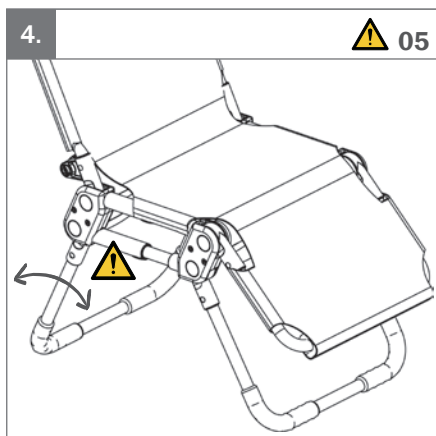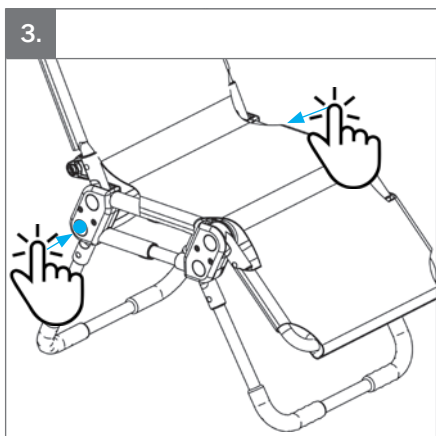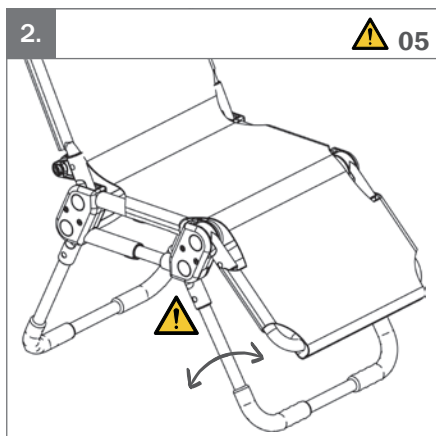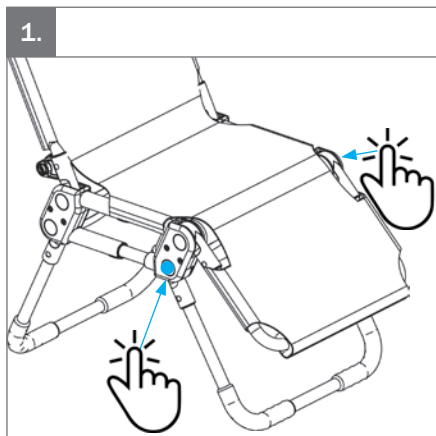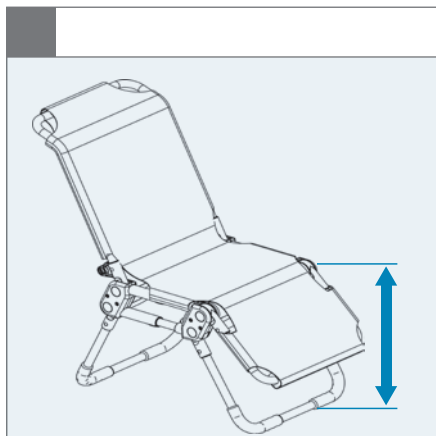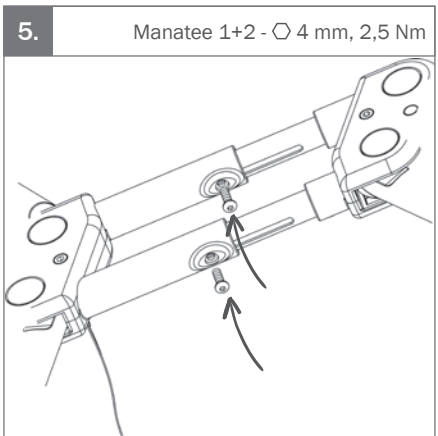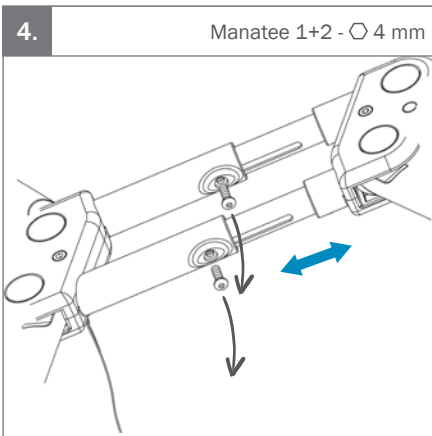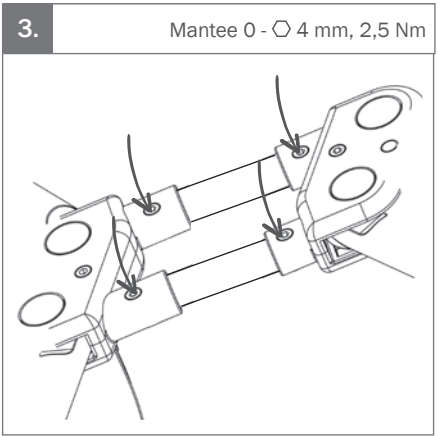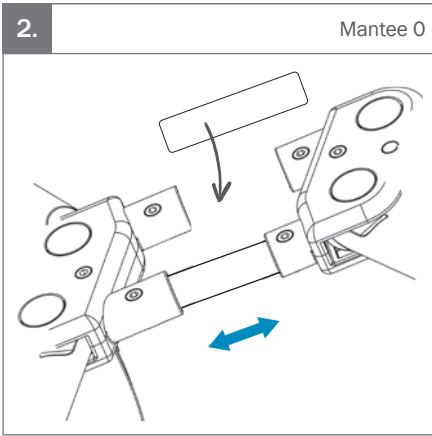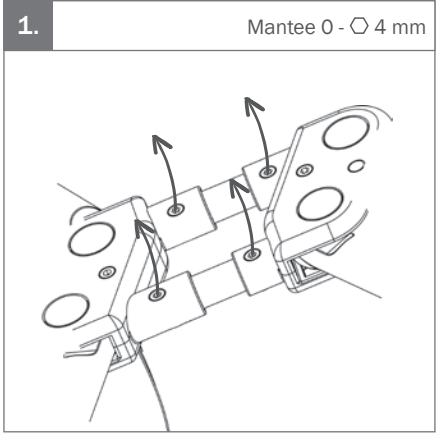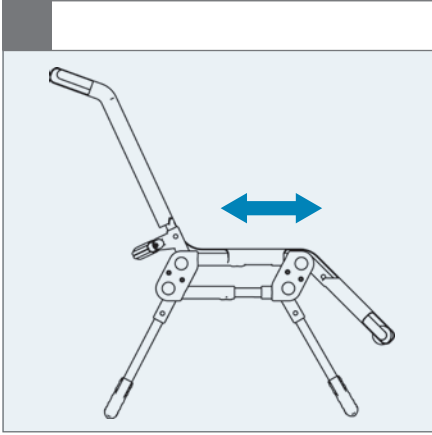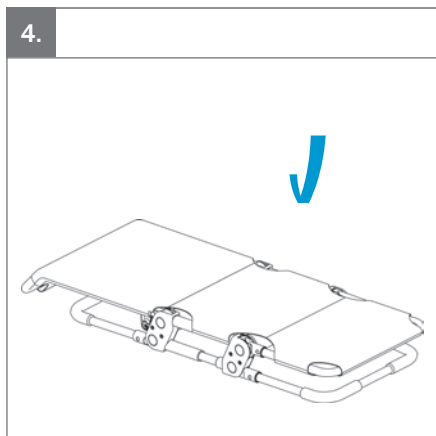
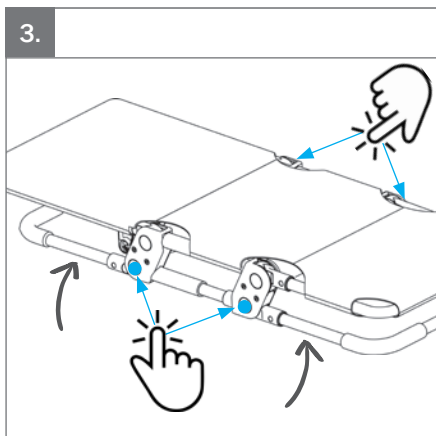
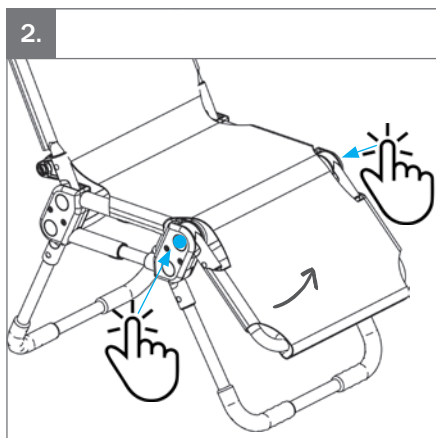
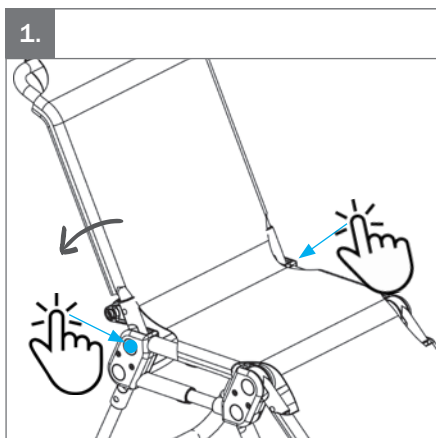
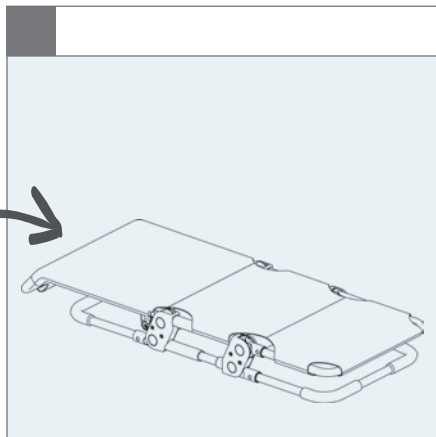
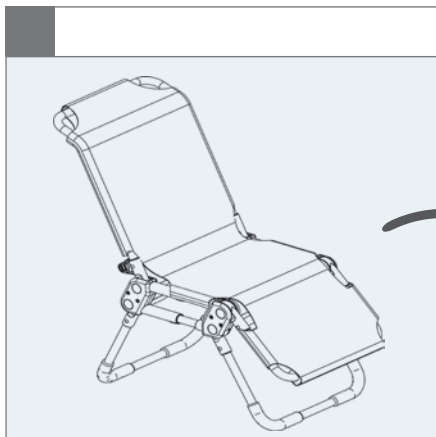
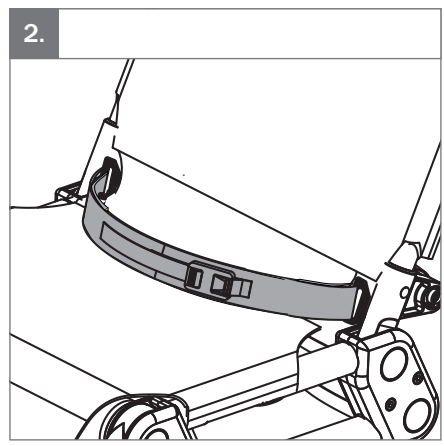
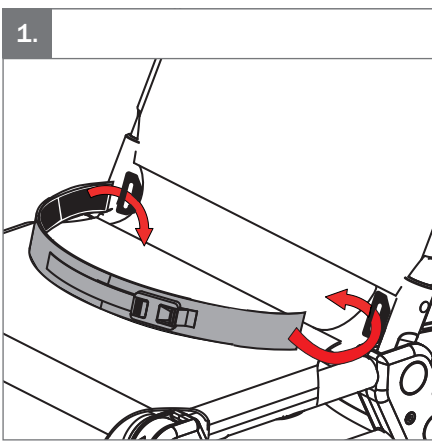
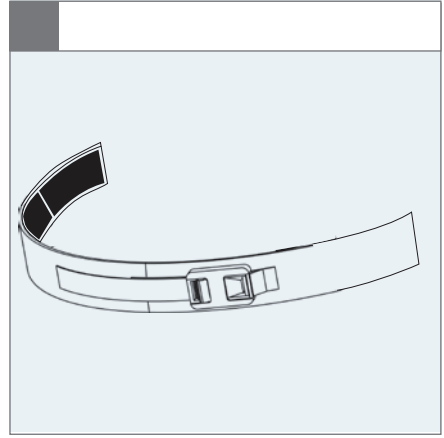
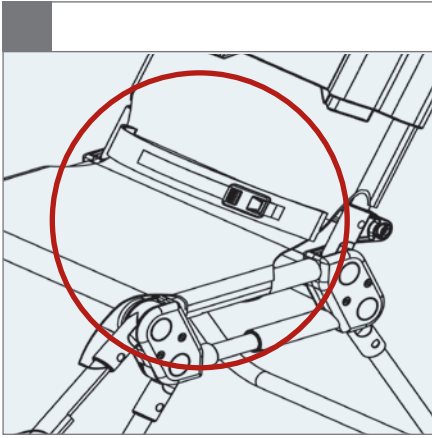
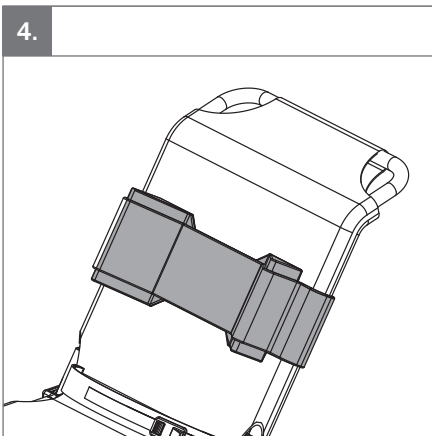
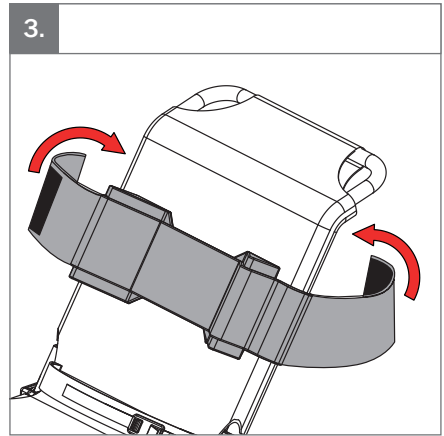
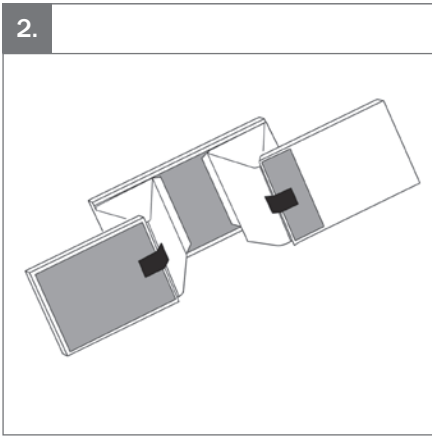
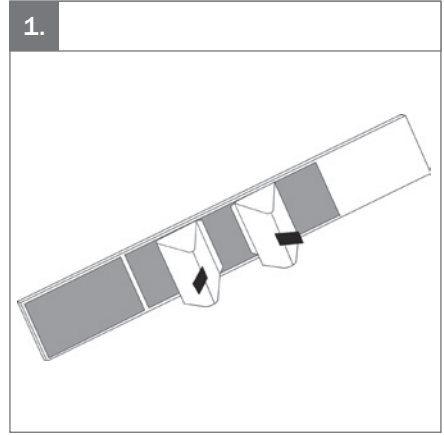
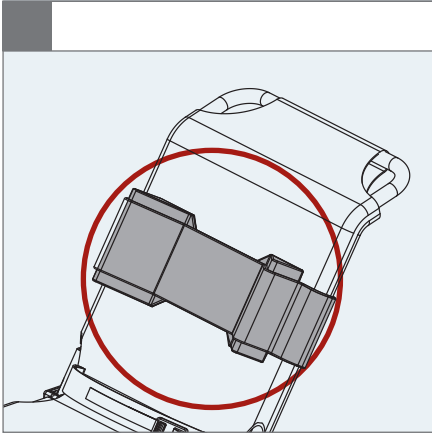
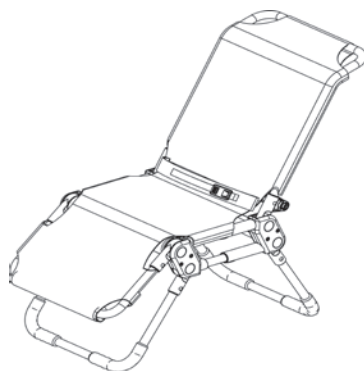
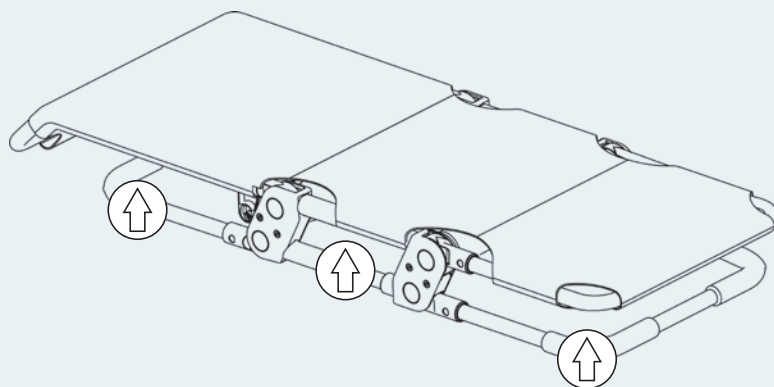
EN: For indoor use

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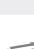
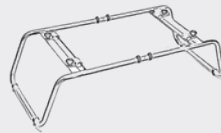
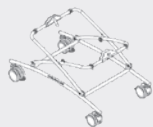
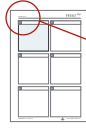
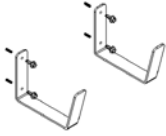
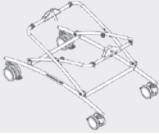
ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The Manatee is a bathing chair suitable for disabled children. The chair is designed to place the user in exactly the right sitting position. The depth of the seat can be adjusted, allowing the chair to grow with the child. The product is for indoor use in a bathroom environment.

The target group for this device is disabled children with cerebral palsy (CP). Also, children with muscular dystrophy, spina bifida, spinal cord injuries and neurological impairments can be subject to the device. The product is suitable for children with GMFCS level 3-5.

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12182
- ISO 17966

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

**Disposal**

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the user manual carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Be aware of any MAX mark indications before adjustment.
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use

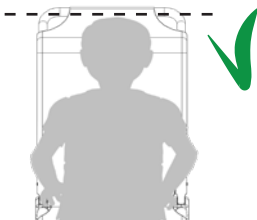
User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- The head of a user must always be within

the top of the frame

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Ensure the stability of the product is maintained before placing the user in it
- Please check on a regular basis that the zippers are safely closed.
- Check that all release buttons mesh into place after adjustment
- Ensure the Anti-slip rubber covers are functional
- Check on the cover, belts and accessories and make sure that the Velcro fastens securely
- The enclosed belts and accessories cannot be used as long-term supports but may only be used as an extra help when positioning the user.

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device.

Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded. The device may be reissued within its lifetime.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Backrest and leg support is adjustable and can be locked in position
 - Zipper(s) & Velcro
 - Chair is foldable/unfoldable
 - Nuts and bolts on the product are tightened
 - Product for cracks or signs for worn-out parts
 - All labels on the product are intact
- Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to:

Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Fabric and covers

The cover can be washed at max. 60°C using a mild detergent in a washing machine.

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Aluminium
- Rubber
- Plastic
- Polyester
- Stainless steel

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

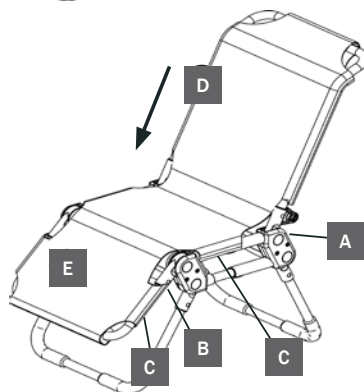
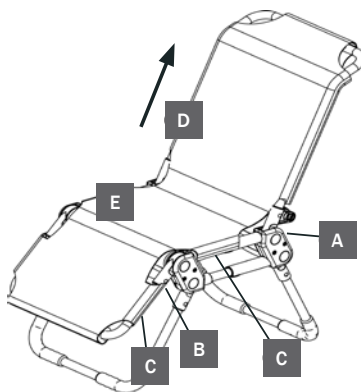
Dismounting and mounting the cover

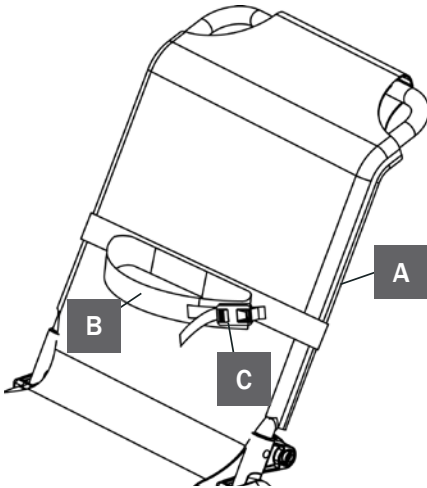
Follow the description below when the cover needs to be dismounted and washed:

1. Unzip the cover on the back (A) and underneath the seat (B)
2. Open the velcro underneath the chair (C) and the cover can be pulled up (D) and off the chair

Follow the description below to remount the cover on the chair:

1. Put the cover over the back and pull down (D). Make sure that the foam cushions in the cover are placed around the frame
2. Zip the cover on the back of the chair (A)
3. Mount the small cover piece with the zipper underneath the chair (B). Make sure that the velcro is pointing downwards
4. Tip up the leg support (E) and the velcro underneath the seat (C) and underneath the leg support (C) can be closed. Make sure that the foam cushions in the cover are placed around the frame
5. Tip down the leg support (E) and make sure the velcro is fastened correctly

Accessories

Body support

Please follow the description below to mount the body support the correct way:

1. Fasten the strap with velcro around the back (A)
2. Place the user in the chair. Mount the strap (B) around the body
3. Tighten the body support with velcro and the buckle (C)

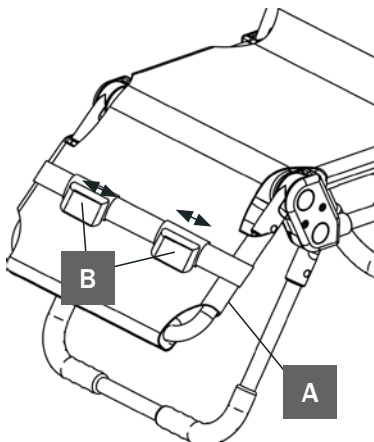
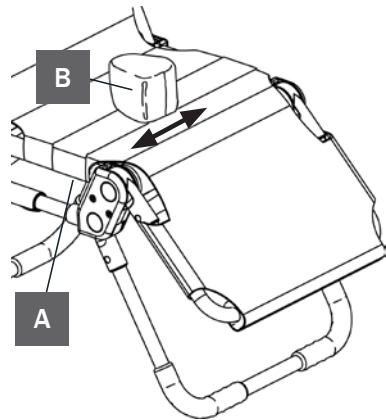
The body support may not be used as a long-term support. This support is only designed to make it easier to place the user in the chair.

Pommel with strap

Please follow the instructions below to mount the pommel and the strap the correct way:

1. Fasten the strap with velcro underneath the seat (A)
2. Adjust the pommel (B) sideways to gain the optimal position

The pommel with strap may not be used as a long-term support. This support is only designed to make it easier to place the user in the chair.

Straps for leg control

Please follow the description below to mount the straps the correct way:

1. Fasten the strap with velcro underneath the leg support (A)
2. Adjust the straps (B) sideways to gain the optimal position. Fasten with velcro

The straps may not be used as a long-term support. This support is only designed to make it easier to place the user in the chair.

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Etac Ltd.
Gelbart AG
R82 A/S