# Etac Swift Commode

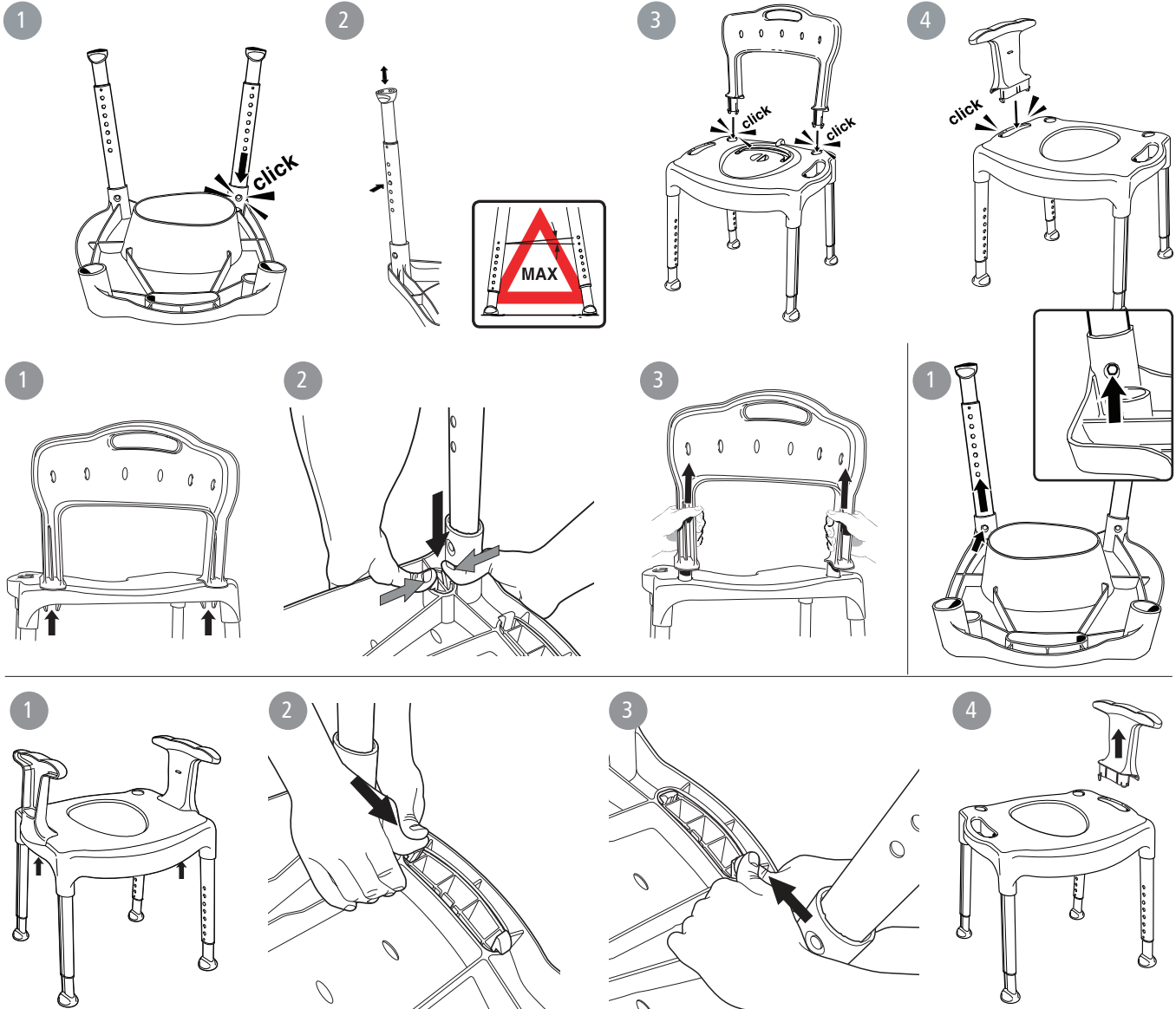
## Etac Swift freestanding

### Manual

etac®

783231 16-02-02

#### A

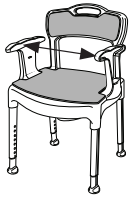
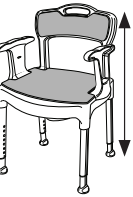
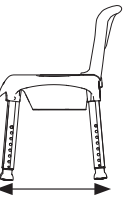
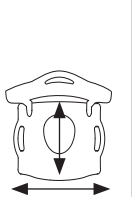
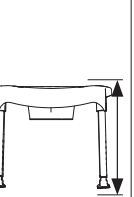
#### B

sv no da en de fi fr nl it es pt

CE

								
Etac Swift Commode	210 mm	450 mm	540 mm	800-950 mm	530-550 mm	540 x 410 mm	420-570 mm	5,4 kg
Etac Swift Freestanding	210 mm	450 mm	540 mm	800-950 mm	530-550 mm	540 x 410 mm	420-570 mm	3,9 kg
Etac Swift Commode low	210 mm	450 mm	540 mm	725-800 mm	520-530 mm	540 x 410 mm	345-420 mm	5,2 kg

SV

## Svenska

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## English

**Intended use**

The Swift commode is a comfortable and mobile toilet chair which makes it easier for users to go to the toilet at all times. The freestanding Swift toilet chair offers users stable seating at the correct height above the toilet seat.

**Description**

The products have a stable design with built-in flexibility which adapt to uneven floors. All products are fitted with height-adjustable legs that are simple to attach. A correct height setting and the use of armrests makes it easier for users to stand up. The leg ferrules offer good grip and the anti-slip pattern of the seat offer the important safety and stability properties that make the user feel safe. The Etac Swift Freestanding toilet seat raiser can be upgraded to a Swift Commode using accessories. Swift commode has been tested by a testing institute and complies with the requirements stated in EN ISO 10993-1 and SS EN 12182.

**Installation and use** ..... *Figure A*

**Maintenance and safety** ..... *Figure B*

**Warning:** A defective product must not be used.

**Warning:** A height difference of more than one hole is not allowed for chair legs.

**Warning:** When moving sideways, do not lean on just one armrest.

**Max user weight:** 130 kg.

**Cleaning:** Clean the product using a solvent-free cleaning agent with a pH value of 5-9 or with a 70% disinfectant solution.

**Decontamination:** Can be decontaminated at max. 85° C/185 °F for 3 minutes if required.

**Guarantee, service life**

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 10 years. For complete information regarding the service life of the product, see www.

etac.com.

**Technical data** ..... *Figure C Accessories*

1. Swift Commode pan including lid 2. Back support
3. Seat pad 4. Backrest pad
5. Soap dish/Lid support/Shower head holder

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R82 UK Limited.
Snug Seat, Inc.
Etac Supply Center AB