
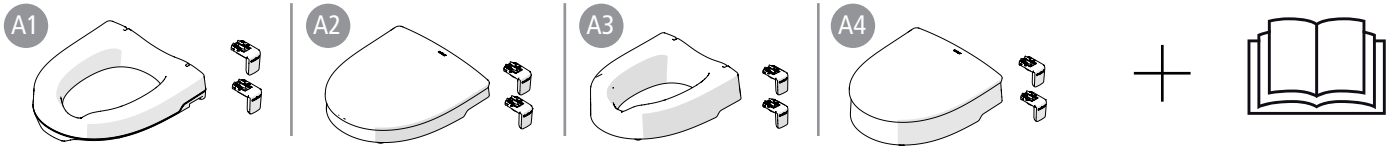
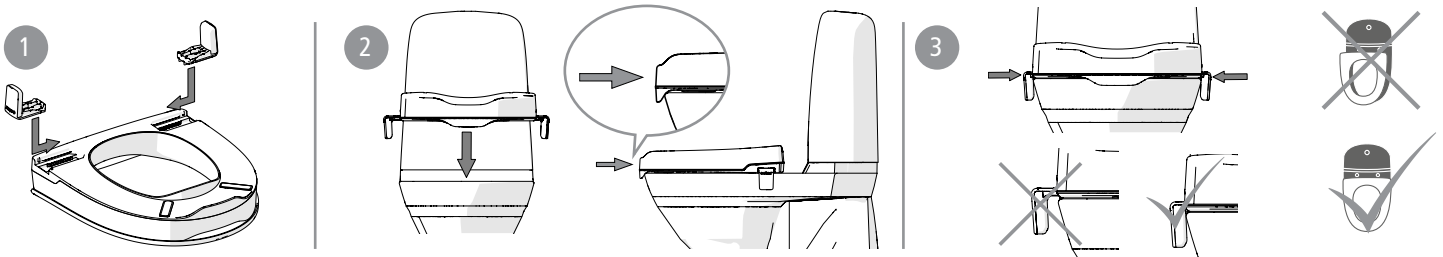
A

B

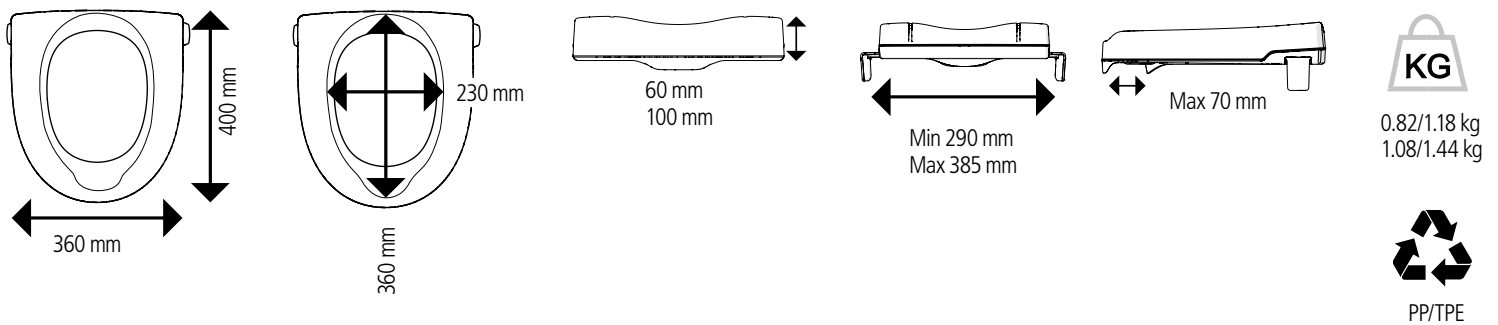C

D

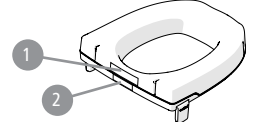
E

F

G

SV Etac My-Loo med kantstopp

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en Etac My-Loo with brackets

General:
Thank you for choosing an Etac product. To avoid injury during installation, handling and use, you should read this manual and keep it for future reference. You can also find it at www.etac.com, where you will also find other documentation, including prescriber information, pre-sale guides and conditioning instructions.

Intended use
The Etac My-Loo with brackets is a toilet seat raiser designed to facilitate toilet use by persons with restricted mobility. It is fitted directly onto the toilet, with the existing toilet seat lifted or removed. The Etac My-Loo is intended for use in home environments and similar.
Etac My-Loo may not be used in any

other way than above stated.
Contraindications:
A user with extensive balance problems should consider using a toilet seat raiser with arm support.
Etac My-Loo is CE marked in accordance with MDD 93/42 EEG.

 Do not stand on the product.

Delivery contents.....Figur A
Installation.....Figur B
1. Fit the brackets.
2. Place the toilet seat raiser on the toilet.
3. Press the brackets against the toilet to lock the toilet seat raiser. Make sure it is in the center of the toilet.

Technical data.....Figur C
The materials used are corrosion-resistant.

Maintenance.....Figur D
Cleaning and disinfection
1. Clean the product with normal non-abrasive cleaning agents (pH value of between 5-9). Rinse and dry. Disinfect the product with an alcohol-based disinfectant, 70%. The product is otherwise maintenance-free.

2. The product can withstand cleaning in a sterilizer at 85°C for 3 minutes. This applies for conditioning.
The materials used are resistant to all general disinfectants.

Safety.....Figur E
1. Maximum user weight 190 kg.
 2. Defective or incorrectly installed products must not be used.

Guarantee, service life
5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.
Service life 5 years. For complete information regarding the service life of the product, see www.etac.com.

Accessories.....Figur F
1. Lid
Label placement.....Figur G
1. Product label (warnings, CE-marking).
2. Item no label (serial no, production date).

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Etac UK Limited
R82 UK Limited.