Wombat klar til brug (DK)

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Wombat klar til bruk (N)

Wombat ready for use (GB)

Wombat einsatzbereit machen (D)

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Wombat - p íprava k použití (CZ)

Wombat klar til brug

R82[®]

Wombat Basic:

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Wombat ready for use




Wombat Basic:

When you unpack your Wombat, you will find the back of the chair packed separately. Begin by removing the frame from the box. Loosen handle (A) on the central bar. Release handle (B) under the right-hand side of the seat so the seat can be raised.

Pull out the footrest so that it is at a right angle to the bar. Loosen the tightening handle (C) for the footrest bar under the seat and adjust the footrest.

The back support is fitted on to the seat by turning the knob clockwise and pulling out the knob (D) behind the back support bar and attaching the back there.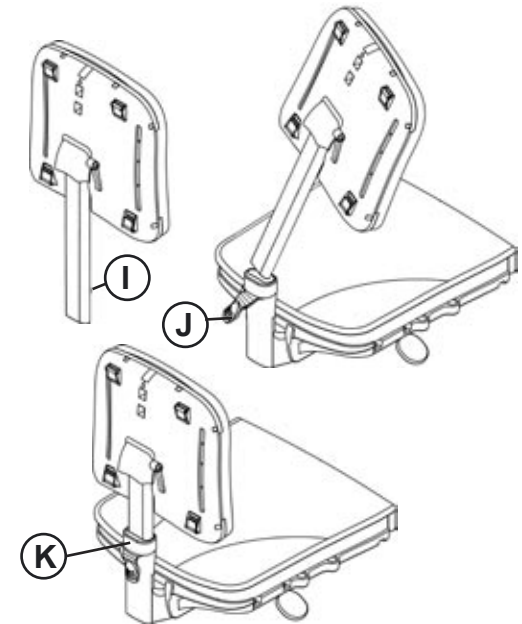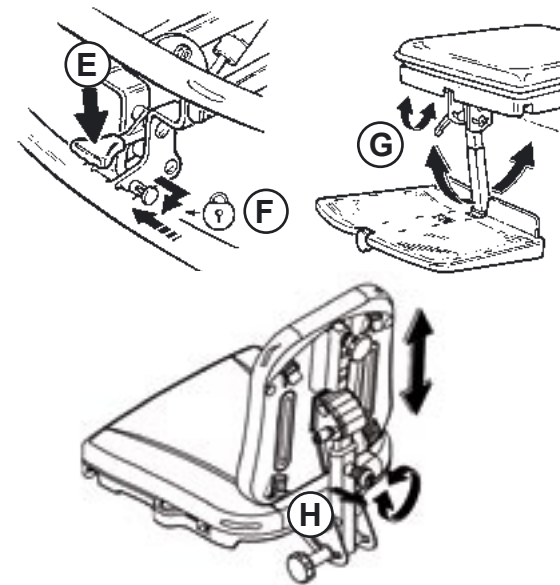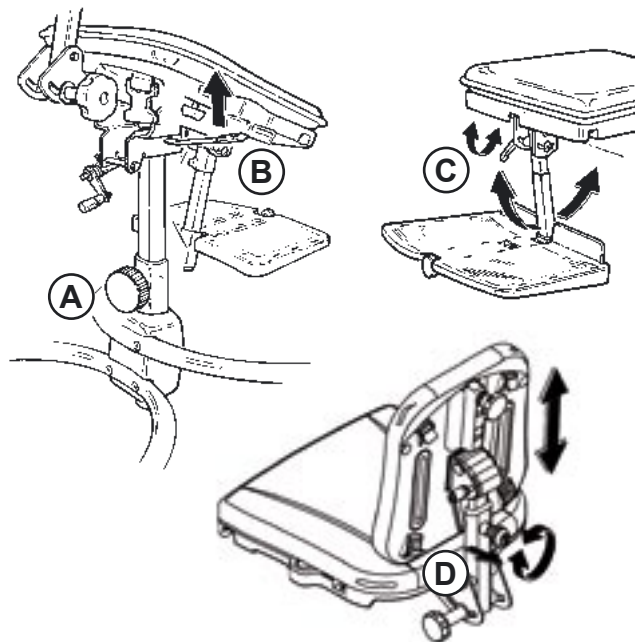

Wombat Upgrade:

When you unpack your Wombat, you will find the back of the chair packed separately. Begin by removing the frame from the box. Raise the seat unit by placing your foot on the pedal (E). Remember first to loosen the safety lock (F) to the right of the pedal.

Pull out the footrest so that it is at a right angle to the bar. Loosen the tightening handle (C) for the footrest bar under the seat and adjust the footrest.

Size 1+2: The back support is fitted on to the seat by turning the knob clockwise and pulling out the knob (D) behind the back support bar and attaching the back there.

Size 3: First remove the security nut (I) to be able to mount the back on size 3. Pull out the locking lever (J). Mount the back support bar in the fitting and tip into vertical. Depress the blue ring (K) to push down the bar in the fitting. Mount the security nut again. Adjust the height to the desired position. Resecure the locking lever.

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