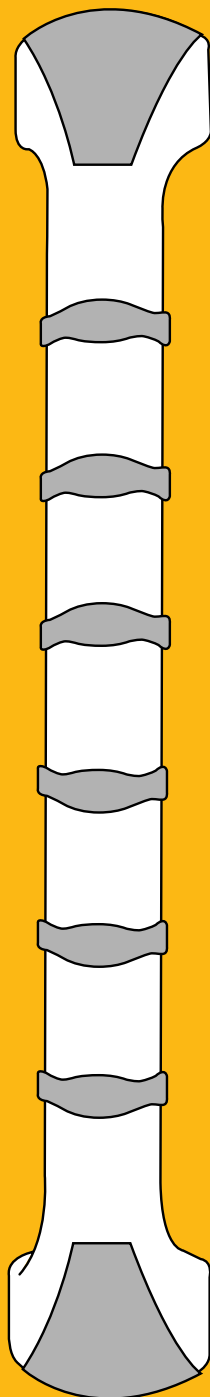
Handy

Grab Bar

Safety grab bar Ideal for all locations

Provides additional safety in the home

Special soft grip

- reduces risk of slipping
- easy to hold

Ribbed surface

- easy to grip
- never cold to touch

Attractive design

- superbly engineered
- suits all decor

Variety of sizes

- available in four sizes
to meet your needs

Hidden screws

- attractive finish
- no sharp edges
- fully insulated

Max User Weight

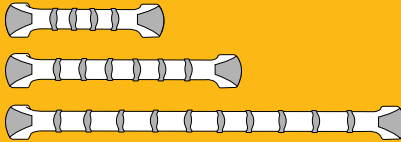
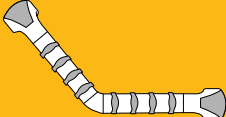
- 160kg / 350lbs
/ 25 stones

A Handy Grab Bar from Etac is an invaluable safety aid for everyone, and especially those who may be unsteady and would benefit from a little extra help.

The Handy Grab bar can serve the four basic safety functions.

- Pushing up
- Pulling up
- Steadying while sitting down
- Security while transferring position

The Range

Straight	Angled
	
	
<p>New attractive soft grip bar is comfortable and easy to hold. Ribbed surface provides additional support. Concealed screws remove any sharp edges. Can be positioned horizontally or vertically. Available in a range of sizes.</p>	<p>Attractive high quality grab bar features a unique soft ribbed grip providing comfort and additional support. Concealed screws remove any sharp edges. Angled design provides additional support when rising and lowering. Available in one size.</p>

Always ensure that all grab bars are securely fixed to the wall with suitable fixings. Consult a qualified craftsman if necessary.

See overleaf for fitting instructions, translated into 13 languages.

Etac, Box 203, SE-334 24 Anderstorp
Tel. +46/371-58 73 00, Fax. +46/371-58 73 90

www.etac.com

Made by Gordon Ellis & Co
www.gordonellis.com

 etac®

ENGLISH

1. Fitted correctly, straight rails should be installed either horizontally or vertically.
2. The rail may be fitted to most types of solid wall. If fitting to plasterboard ensure suitable fittings are used. If in doubt consult a tradesman.
3. Position the rail to a flat wall and mark the 4 screw positions.
4. To fix in position use no. 10 (5mm) x 50mm (3/16" x 2") roundhead woodscrews with suitable wall plugs. Depending upon application other fittings may be suitable.
5. Locate the end caps to the rail and push down to fit.
6. Before first use check the rail is securely fitted to the wall.
7. Clean using non-scratch detergent. Always ensure that all grab bars are securely fixed to the wall with suitable screws. Consult a qualified craftsman if necessary.

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