



Molift RgoSling

Built-in ergonomics for comfortable hoisting

 etac®

Molift RgoSling

Built-in ergonomy for comfortable hoisting



Maximum Comfort

All our slings are designed to distribute weight and pressure evenly for maximum comfort. As Molift is committed to comfort and safety we offer a wide range of various models and sizes. We recommend individual testing. We have a colour code indicating sling size to facilitate a quick and easy selection.

Anatomical shape makes the difference!

Molift RgoSling has an anatomical shape and a fitted cut with fixed padding both in the back and leg parts. The anatomic design allows for the natural curvature of the spine and stops the user from sliding. In combination with the balanced weight distribution, a safe and comfortable hoist is achieved.

Unique sliding loop function

The unique sliding loop function in the shoulder straps, balances the weight distribution in every stage of the hoist, making it very comfortable to hoist from a lying to a sitting position and vice versa. Thereby an external angle adjustment of the suspension is redundant.

Scandinavian Design

Molift slings are designed in Norway, applying the same high standards as it does to Molift hoists. All the slings have been approved according to quality standard EN ISO 10535. We recommend the use of 4-point suspension for comfortable hoisting, but the slings may also be used with a 2-point suspension.

Unique Features

The shoulder straps have the unique Molift sliding loops, to secure a comfortable and safe position for the patient. At the same time the clever design of the leg parts distributes the pressure for increased comfort. Sling application is made very easy due to the centrally placed pocket, together with the special folding technique developed by Molift.



Back pocket

The back pocket provides easy and comfortable application.



Leg support

The padded leg support with the unique placement of the strap, distributing the pressure on the thigh and thereby increasing the comfort.



Sliding loops

The sliding loops gives a balanced weight distribution.



Many options

The slings are available with HighBack and MediumBack.



Flip up labelling system

Clear and visible information for the everyday need. Maintenance information flipped up behind the main label.




Pocket in the back

Pocket in the back facilitates the placement of the sling.



Pocket on the leg support

The pocket provides easy and comfortable application of the leg support.

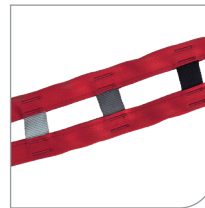

Color codes for easy fitting

The color-code system of the legstraps makes it easy to identify the different heights when attaching to the sling bar.



Hoisting straps

Several colour-coded loops in order to find the right fit.

Anti-slip

For greater safety, anti-slip protection is provided on the inside of RgoSling Amputee, RgoSling Toilet and RgoSling StandUp

Belt

Molift RgoSling Amputee and RgoSling Toilet has a belt for comfort and safety.

Leg support

Padded legsupport for the patients comfort. A pocket facilitates the assessment of the legrest.

Labelling

Flip up labelling system with clear and visible information for the everyday need. Maintenance information flipped up behind the main label.

Padding

Padding distributes the weight and provides a better comfort for the patient.

Guide pocket

Pocket that facilitates application, primarily in sitting position.



RgoSling MediumBack Padded.

Molift RgoSling MediumBack and HighBack

Built-in ergonomics for comfortable hoisting.

For homecare and institutional environments

Molift RgoSling MediumBack and Molift RgoSling HighBack are all-round slings that fit most users and hoisting situations in homecare and institutional environments.

Molift RgoSling

RgoSling MediumBack provides support for the body and the RgoSling HighBack provides support for the head as well. It allows the user to have a slightly inclined position which is suitable for users with impaired trunk and head stability. It is especially suitable for hoisting to and from a lying position and hoisting from the floor.

Molift RgoSling in Net-version

The RgoSling MediumBack and RgoSling HighBack are also available in a soft net mesh polyester material. The material breathes well and dries very quickly. The sling is especially suitable for shower and bathing situations. The mesh version has the same shape and features as the padded versions, such as; sliding loops, application pocket in the back and straps for increased weight distribution in the leg parts.



RgoSling HighBack Net.

Molift RgoSling MediumBack

Size	Padded	Net
XXS		1720300
XS	1720110	1720310
S	1720120	1720320
M	1720130	1720330
L	1720140	1720340
XL	1720150	1720350
XXL	1720160	1720360

Molift RgoSling HighBack

Size	Padded	Net
XXS		1720400
XS	1720210	1720410
S	1720220	1720420
M	1720230	1720430
L	1720240	1720440
XL	1720250	1720450
XXL	1720260	1720460

Recommended User Weight

XXS	12 – 17 kg (26–37 lbs)
XS	17 – 25 kg (37–55 lbs)
S	25 – 50 kg (55–110 lbs)
M	45 – 95 kg (99–210 lbs)
L	90 – 160 kg (198–353 lbs)
XL	160 – 240 kg (353–529 lbs)
XXL	230 – 300 kg (507–660 lbs)

Material


Polyester,
PE -cellfoam

Washing instruction



60–85° C
140–185° F

Colour code

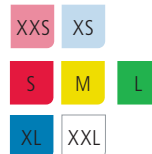

Molift RgoSling Toilet LowBack and HighBack

A flexible sling with optimal functions for toilet situations.



Meeting high demands when toileting

RgoSling Toilet provides an exceptional trunk support and is secured by double belt-buckles. The flexible leg supports facilitates individual fitting and can be positioned in different ways to accommodate the user's functions and needs.

Comfort and safety for the patient

Molift RgoSling Toilet has several alternative loops on both the shoulder and leg straps in order to adjust the sitting angle according to the user's needs, including an upright sitting position. The sling provides exceptional support in the sitting position. The belt-buckles can be adjusted along its length in order to get weight and pressure in optimal position whilst maintaining good stability and safety.



Available in HighBack and LowBack

Molift RgoSling Toilet HighBack provides additional support for users with reduced or no head stability.

Alternative application. Unique fixing of the leg support facilitates flexible and individual application

Size	LowBack	HighBack
XXS	1720500	1720600
XS	1720510	1720610
S	1720520	1720620
M	1720530	1720630
L	1720540	1720640
XL	1720550	1720650
XXL	1720560	1720660

Material

Polyester, PE-cellfoam

Recommended user weight

XXS	12–17 kg (26–37 lbs)
XS	17–25 kg (37–55 lbs)
S	25–50 kg (55–110 lbs)
M	45–95 kg (99–210 lbs)
L	90–160 kg (198–353 lbs)
XL	160–240 kg (353–529 lbs)
XXL	230–300 kg (507–660 lbs)

Colour code

XXS	XS	
S	M	L
XL	XXL	

Washing instruction



Molift RgoSling Amputee MediumBack and HighBack

Extra safety with the built-in waist belt.



Narrow opening for increased safety

Molift RgoSling Amputee provides the additional support necessary for hoisting and transferring users with impaired muscle tone or amputated limbs. The RgoSling Amputee has an anatomical cut and padding that provides exceptional support through the entire hoist and transfer situation. The leg parts are divided, and have extra length for application under both legs. Additional straps to avoid the legs from sliding out of position, and therefore helping the confidence of the patient.

Available in Highback and MediumBack

Molift RgoSling Amputee HighBack provides additional support for users with reduced trunk tone and head stability. The sling can be used for hoisting to and from horizontal and sitting positions.

Feature



Extra trunk support

The waist belt with double belt-buckles and anti-slip material on the inside, provides extra trunk support for patients with reduced muscle tone and stability.



Smaller opening

The opening around the buttocks is smaller than the standard version sling to prevent users from slipping down in the sling.

Size	MediumBack	HighBack
XXS	1721100	1721200
XS	1721110	1721210
S	1721120	1721220
M	1721130	1721230
L	1721140	1721240
XL	1721150	1721250
XXL	1721160	1721260

Material
Polyester,
PE-cellfoam

Recommended user weight

XXS	12 – 17 kg (26 – 37 lbs)
XS	17 – 25 kg (37 – 55 lbs)
S	25 – 50 kg (55 – 110 lbs)
M	45 – 95 kg (99 – 210 lbs)
L	90 – 160 kg (198 – 353 lbs)
XL	160 – 240 kg (353 – 529 lbs)
XXL	230 – 300 kg (507 – 660 lbs)

Colour code

XXS	XS	
S	M	L
XL	XXL	

Washing instruction



Molift RgoSling Comfort HighBack

Adapts to body shape.



Support for the entire back

Molift RgoSling Comfort HighBack is a full body support sling made of a mesh polyester material that breathes and dries quickly. It is particularly suitable when the sling is to be left underneath the user in a chair or wheelchair, and for users with muscular or skeletal pain. Molift RgoSling Comfort provides good support for the entire back. It is a high back model and puts the user into a more posteriorly tilted position. The shoulder straps have the unique sliding loop, which provides smooth transition from lying to sitting position and vice versa. Must be applied with the user in a lying position.

Size	Regular
S	1721420
M	1721430
L	1721440

Material
Polyester

Recommended user weight

S	25 – 50 kg (55–110 lbs)
M	45–95 kg (99 –210 lbs)
L	90 – 160 kg (198–353 lbs)

Colour code



Washing Instruction




Molift Accessories



Molift storage system

The Molift Sling Storage System is a useful product for storing and presenting your Molift Slings in a professional way! Item no. 3030700

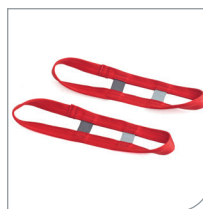

Molift RgoSling Extension loops

For loop extension 10, 20 or 30cm. Order two for a complete pair. Item no: 3013000

Molift RgoSling Ambulating Vest

For gait training, independent or with assistance.



Secure and comfortable gait training

Molift RgoSling Ambulating Vest is developed for safe and secure rehabilitation such as gait training, standing exercise as well as balance exercise.

Molift RgoSling Ambulating Vest has a double buckled wide padded belt that distributes pressure evenly over a larger surface. The padded groin straps limit the vest from sliding up and out of position.

Adopted features from professional climbing

Molift RgoSling Ambulating Vest has adopted features from professional climbing equipment to provide the best possible comfort and safety for the user. The self-tightening suspension loops prevent users from slipping or falling down.

The front buckle system provides increased possibilities for the user to more independently perform gait training or toileting, maybe even without assistance.

Accessory



Groin Straps

The groin straps are placed in an outward angle which minimizes risk of pinching when more weight is put on the vest.

Size	Item no.
XXS	1721300
XS	1721310
S	1721320
M	1721330
L	1721340
XL	1721350

Accessories

Groin Straps for Ambulating vest
Order two for a complete pair.

Size	Item no.
XXS-XS	1721700
S-M	1721720
L-XL	1721740

Material
Polyester,
PE-cellfoam

Recommended user weight

XXS	12–17 kg (26–37 lbs)
XS	17–25 kg (37–55 lbs)
S	25–50 kg (55–110 lbs)
M	45–95 kg (99–210 lbs)
L	90–160 kg (198–353 lbs)
XL	160–240 kg (353–529 lbs)

Washing Instruction



Colour code



A perfect match!

Molift Air, Molift RgoSlings and Molift Rail System create the perfect integrated solution for all situations. It combines the best from an installation, caregiver and patient perspective. With the high flexibility of the Molift Air and the wide range of Molift RgoSlings, this is a perfect match. Innovative and cost-effective for all situations! We put a lot of passion in developing every detail.

