

nandu

Exploded views / Part lists


Rev. date (12.2017)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.





Contents

Seat unit.....	3
Frame.....	4

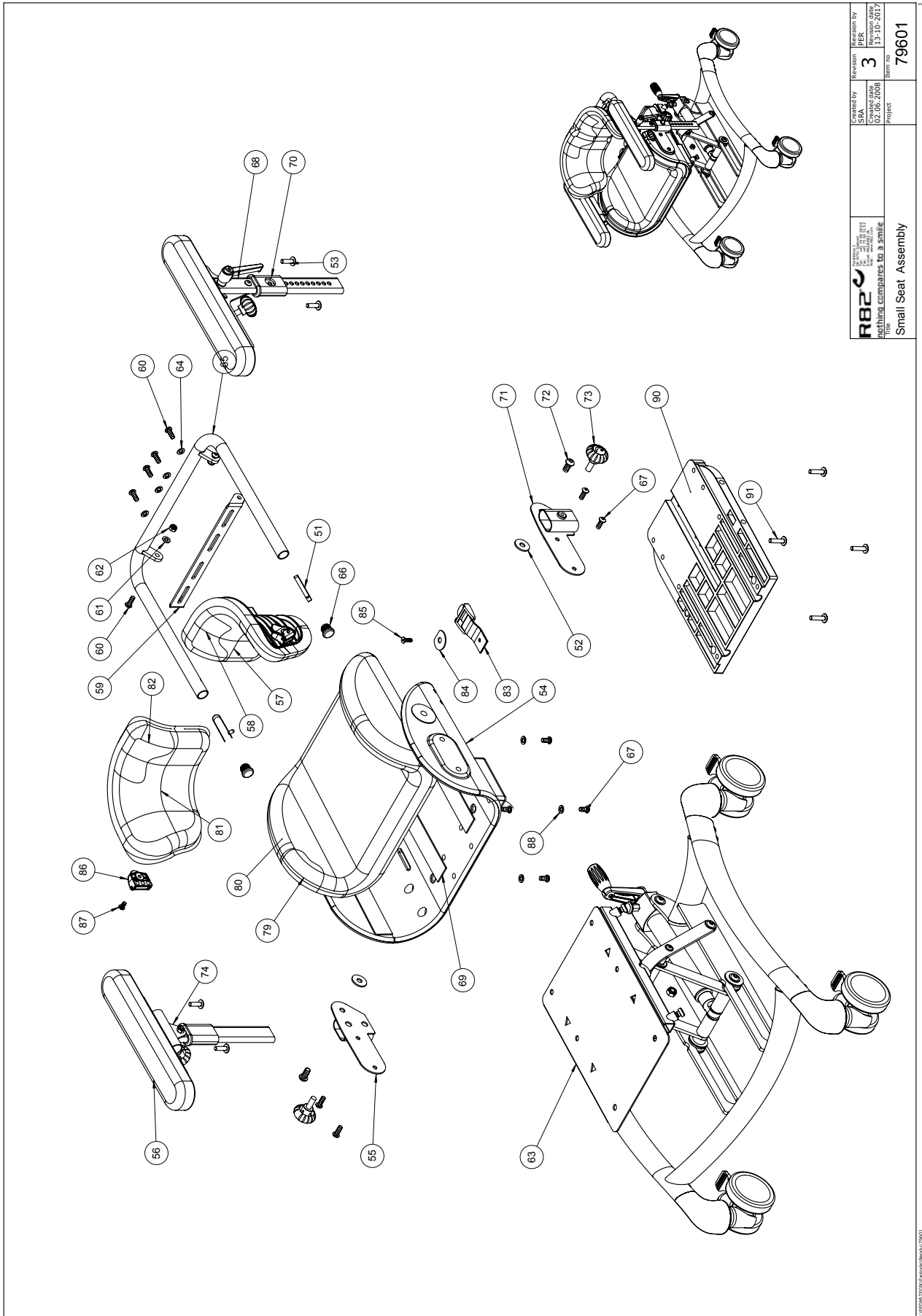

 R82	Checked by SKM	Revision 3	Checked by PER
	Created date 02.06.2008	Revision date 13.10.2017	Item no. 79601
Project Small Seat Assembly			

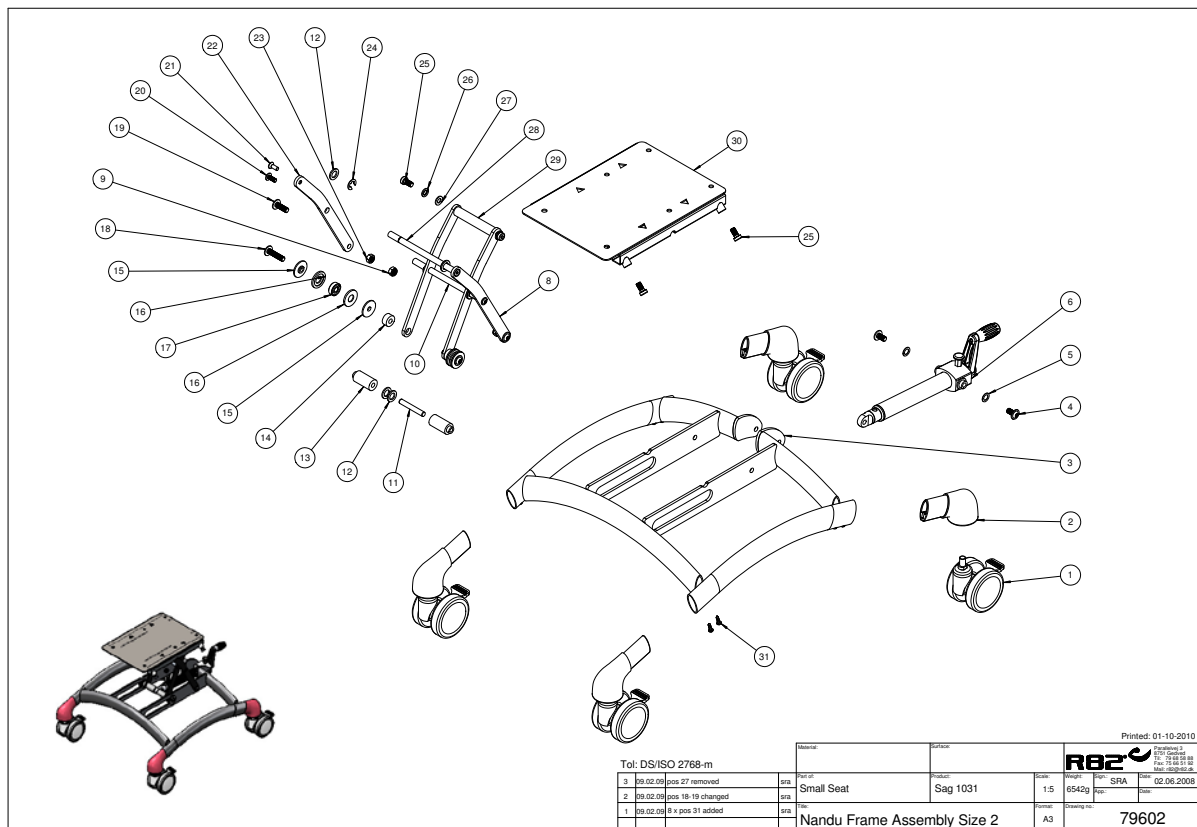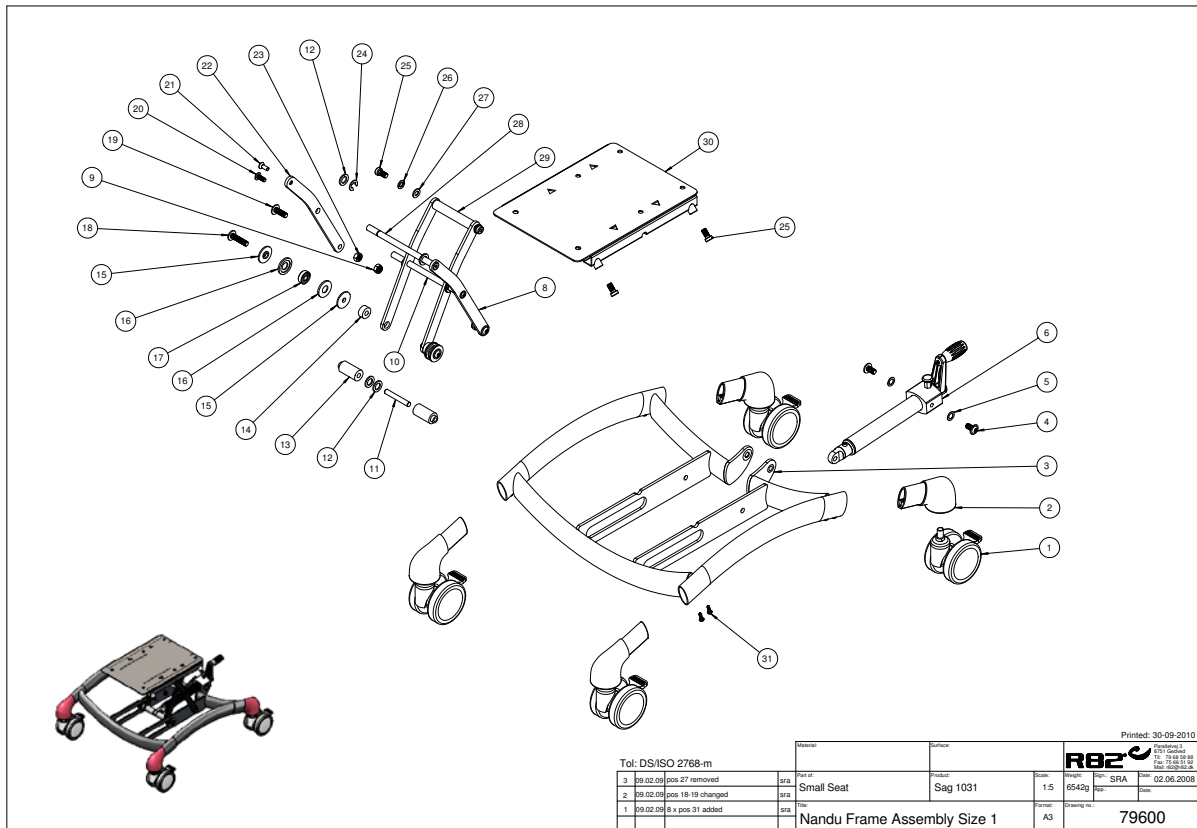

Pos. no.	Art. no.	Description	Unit of measure	Quantity	Ean code
1	3007500	75mm black wheel w/brake	PCS	4	5707292487834
2	73614	Holder for wheel size 1	PCS	4	5707292433275
3	77500-54	Small Seat frame size 1, grey	PCS	1	5707292487155
3	77501-54	Small Seat frame size 2, grey	PCS	1	5707292487667
4	0661501	Button head w/flange 8x16	PCS	2	5707292139566
5	1310640	Washer, nature Ø16*10*0,5mm	PCS	2	5707292140760
6	5086100	Spindel mounted toucan/rabbit	PCS	1	5707292420817
8	77516-54	Liftarm Left Side, Painted	PCS	1	5707292487353
9	1607000	Lock nut 8mm, large pack	PCS	2	5707292141378
10	77531	Connector Pin, Lift	PCS	1	5707292487803
11	0530010	Cycle pin Ø8*55mm	PCS	1	5707292513700
12	1310710	Nylon washer	PCS	4	5707292140807
13	77508	Bush	PCS	2	5707292487254
14	77510	Bush diam. 8,2/20mm l=9,5 mm	PCS	2	5707292487278
15	73751	Disk for Seat	PCS	4	5707292430823
16	77521	Disc for seat, pom	PCS	4	5707292501783
17	2985211	Ball bearing 608-2rsh	PCS	2	5707292432124
18	0661135	BUTTON HEAD 8*35 FZB	PCS	2	5707292139528
19	0661125	BUTTON HEAD 8*25 FZB	PCS	2	5707292499769
20	0659501	Button head w/flange M6*16 FZB	PCS	2	5707292356000
21	0621600	Insex screw counters. 6*16	PCS	2	5707292138965
22	77515-54	Liftarm Right Side, Painted	PCS	1	5707292487346
23	1603500	Steel nut 8mm, large pack	PCS	2	5707292141286
24	1145090	Locking disc 9mm,din6799,stain	PCS	4	5707292140609
25	0603530	Insex low head M8x16 stainless	PCS	6	5707292412492
26	1100495	Spring washer M8	PCS	2	5707292458674
27	1103200	Fazet washer 8mm	PCS	2	5707292140296
28	77511	Connector Pin	PCS	1	5707292487285
29	77519-54	Center liftarm, painted	PCS	1	5707292487599
30	77520-54	Seat Frame, painted	PCS	1	5707292487650
31	1940034	Eco-drill M4,2*22	PCS	8	5707292423115
51	6153500	Knob spring 0,5*6,5 mm.	PCS	2	5707292188571
52	1108000	Washer 8mm	PCS	2	5707292140388
53	0622800	Insex screw counters. 6*20	PCS	2	5707292138972
54	82325-60	Plast Seat Panda 2, black	PCS	1	5707292231284
54	82295-60	Plast Seat Panda 2½, black	PCS	1	5707292231192
54	82335-60	Plast Seat Panda 3, black	PCS	1	5707292231369
54	82345-60	Plast Seat Panda 4, black	PCS	1	5707292231444
55	77537	Bracket armsupport size2 right	PCS	1	5707292488053
55	77539	Bracket armsupport size2½ righ	PCS	1	5707292488077
55	77541	Bracket armsupport size 3 righ	PCS	1	5707292488091
55	77543	Bracket armsupport size 4 righ	PCS	1	5707292488114
56	77579	Pur Armrest	PCS	1	5707292517494
57+81	85639-165	Back cover, nandu sz. 1+2, red	SET	1	5707292491190
57+81	85639-265	Back cover, nandu sz.3+4, red	SET	1	5707292491213
58	5700590	Upholst. Back support,left,1+2	PCS	1	5707292489135
58	5700592	Upholst back support,left 3+4	PCS	1	5707292489166
59	77544	Bracket f. Back adjust	PCS	1	5707292487674
60	0641600	Button head screw 6*16	PCS	6	5707292139269

61	1103000	Fazet washer 6mm	PCS	2	5707292140289
62	1606500	Lock nut 6 mm	PCS	2	5707292141361
63	851011	Nandu frame size 1, silver	PCS	1	5707292490278
63	851012	Nandu frame size 2, silver	PCS	1	5707292490285
64	1101950	Washer Ø6,1-16*1 mm, FZB	PCS	4	5707292140210
65	77550	Brace support size 1	PCS	1	5707292487681
65	77551	Brace support size 2½	PCS	1	5707292487704
65	77552	Brace support size 3	PCS	1	5707292487711
65	77553	Brace support size 4	PCS	1	5707292487728
66	2502002	Ferrule d18*0,8-2,5 black	PCS	2	5707292433947
66	2502000	Ferrule d18*1,0-2,0 int.black	PCS	2	5707292144003
67	0641200	Button head screw 6*12	PCS	8	5707292139252
68	77586	Armrest, no pur, left mounted	PCS	1	5707292492104
69	4913035	30mm black ps14 with glue hook	PCS	1	5707292469755
70	82922	Bush for Armrest	PCS	1	5707292540539
71	77536	Bracket armsup. (size 2 left)	PCS	1	5707292488046
71	77538	Bracket armsupport size2½ left	PCS	1	5707292488060
71	77540	Bracket armsupport size 3 left	PCS	1	5707292488084
71	77542	Bracket armsupport size 4 left	PCS	1	5707292488107
72	0661300	button head with flange 8*12	PCS	2	5707292408068
73	3501401	R82 handle M6*20mm screw	PCS	1	5707292413901
74	77585	Armrest, no pur, right mounted	PCS	1	5707292492098
82	5700591	Upholst back support,right 1+2	PCS	1	5707292489159
82	5700593	Upholst back support,right 3+4	PCS	1	5707292489173
83	5550260	Sewn strap holder.Pan.futura	PCS	2	5707292178459
84	1125000	Screenwasher FZB black 5/16*30	PCS	2	5707292140470
85	0611200	Insex screw counters. 6*12	PCS	2	5707292138699
86	75306	Snap lock R82 25	PCS	2	5707292437624
87	0640911	Button head M5x10 FZB tuflok	PCS	2	5707292428479
88	1100800	Center spring washer 6mm	PCS	4	5707292140166
79+80	85628-165	Seat cushion, nandu size 1,red	PCS	1	5707292491923
79+80	85628-265	Seat cushion, nandu sz. 2,red	PCS	1	5707292491947
79+80	85628-365	Seat cushion, nandu sz. 3, red	PCS	1	5707292491961
79+80	85628-465	Seat cushion, nandu sz. 4,red	PCS	1	5707292491985
Module	5085101	Back support complete size 1	SÆT	1	5707292502643
Module	5077536	Bracket armsupport size 2 left	PCS	1	5707292532756
Module	5077537	Bracket armsupport size2 right	PCS	1	5707292532763
Module	5077586	Arm rest incl. cushion	PCS	1	5707292517500
Module	5077585	Armrest incl. cushion	PCS	1	5707292517456
Module	5085102	Back support complete size 2	SET	1	5707292502711
Module	5077539	Bracket armsupport size2½ righ	PCS	1	5707292532787
Module	5077538	Bracket armsupport size2½ left	PCS	1	5707292532770
Module	5085103	Back support complete size 3	SET	1	5707292502728
Module	5077541	Bracket armsupport size 3 righ	PCS	1	5707292532800
Module	5077540	Bracket armsupport size 3 left	PCS	1	5707292532794
Module	5085104	Back support complete size 4	SET	1	5707292502735
Module	5077543	Bracket armsupport size 4 righ	PCS	1	5707292532824
Module	5077542	Bracket armsupport size 4 left	PCS	1	5707292532817