

Periodic Inspection Checklist Molift Air

In accordance with ISO:10535 2006 Annex B.

Article no.:

.....

Serial no.:

.....

Production Year:

.....

Owner:

.....

Sling serial number:

.....

Sling Inspected separately and is not included in this inspection.

Situation of use:

- Home
- Institution (type of institution):
 - Hospital
 - Nursing home
 - Other

Service light indicator

<i>Service light</i>	<i>Mode</i>
No light	Power saving (Stand by)
Green	Ready for use
Yellow	Order service
Red	Perform service
Red + sound	Perform service immediately

Physical examination



Test FUNCTION and check for wear. All checkpoints must be checked of to approve lifter for further use.

- Perform 1 lifting cycle with SWL.
 - Brake shall not slip when loaded with SWL
 - Emergency stop
 - Electrical Emergency lowering
 - Handcontrol (buttons, strain relief, hook, battery indicator)
 - Suspension coupling (change of suspension, QR pin, rotation and swivel funktion)
 - Both Quick release couplings (connect to trolley)
 - Angle sensor (pull band sideways)
 - Charger/Battery
 - Accessories (t.ex: Scale, Propulsion, IRC):
-
-

Visual examination



Visual examination of load bearing structure to make sure there is no damage, cracks, frays or deformation. All checkpoints must be checked of to approve lifter for further use.

- Product labels
- Plastic covers
- Hand control
- Gearhouse
- Cables and other electronics.
- Lifting belt
- Suspension coupling
- Suspension (especially composite hooks)

Performed by

Periodic inspection should be performed by a person who is certified by Etac education.

Date/Place:

.....

Signature:

.....

Etac Molift Service ID (6 digits):

.....

- Approved without faults
- Approved after repair.
- Lifter is marked with "out of order" and sent for repair.
- Hoist is not eligible for repair and taken out of service.

If periodic inspection reveals any defect, wear or other damage that jeopardizes the safety of the patient it may not be used until the deficiency has been eliminated. Defects and damages should be reported back to the manufacturer for action in a non-conformity report (NCR).

- The owner is notified
- NCR report sent to Etac, molift@etac.com

Non conformity report can be obtained from www.molift.com or by request, molift@etac.com.