## ENGLISH

**General cleaning instructions**

All medical devices supplied by R82 should be cleaned on a regular basis to ensure hygiene and to secure that the functionality remains intact. Requirements for cleaning varies upon the environment in which the product is used and the usage of the product.

Cleaning of a medical device is wholly the responsibility of the owner of that device. Failure to clean a device may invalidate the device's warranty. Furthermore, failure to clean a device may compromise the clinical condition or safety of users and/or their caregivers.

Important! The guidelines given on this page should be seen as guidelines only. The instructions for cleaning in any R82 User Guide and/or reconditioning guide always overrules these guidelines.

**Cleaning by material type:**

Powder coated parts, Stainless steel, Plastics and PU: Wipe of any residue and dirt using a cloth wrung in a pH neutral mix of warm water and non-abrasive cleaning agent. Finish off with a dry lint free cloth to remove what remains of the mix from the surfaces. Disinfect parts when required, pay special attention to any surface that comes into contact with the user/caregiver.

Cushions and covers can be cleaned according to the washing label placed upon those. If no washing label is attached, then we do not recommend washing the item

Upholstery: Do not wash upholstery. Replace upholstery when required

**Ware-washing machine**

A wide selection of our products can be cleaned in a Ware-washing machine at either 60° or 85°C. Please refer to the product User Guide for detailed information.

**Cleaning agents:**

Use a mild non-abrasive cleaning agent with a PH value between 5-9. The cleaning agent must be used in a mix with water. The mix of the cleaning agent and water must be pH neutral. Do not use Acidic or Alkaline cleaning agents directly on any surface as they may damage the surface

Disinfectant: Surfaces can be disinfected with a 70% IPA solution

**Hygienic measures:**

**What:** All products from R82

**Who:** Distributors, owners, caregivers and authorized service staff

**When:** As required, scheduled maintenance, at repair or before reuse

**Measures:** We recommend wearing protective gloves when cleaning. Do not let water ingress into cables, bearings or frame parts. Any actuator should be retracted to minimum stroke distance before cleaning

**How:** First clean the surfaces then wipe the surfaces dry with a lint free cloth or towel. Subsequently an air dryer at low temperature can be used

**Disinfection:** It is recommended to wipe of any residue and dirt using a cloth wrung in a pH neutral mix of warm water and non-abrasive cleaning agent and let the surfaces dry before disinfection. Afterwards the surfaces can be disinfected with a 70% IPA solution

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