

# Product information

Concerning: Product care | Molift Air 200 / 350 / 500

## Firmware update – Air 200/350/500 version 1009

### Activating hoist after power save mode

Periodic issues with activating the hoist after power save mode is now corrected.

It is no longer possible for the user to encounter a situation, where the hoist requires the user to hold down a button for several seconds to wake it up.

New firmware has been implemented when operating the hoist and the periodic light flashing during charging, has been removed.

### Implementing status, from

Air 200, PCB p/n 2530112 – SN 440947

Air 350, PCB p/n 2540112 – SN 440944

### Technical information:

To improve user experience, the input latency from the hand control has been reduced.

An extra safety has been implemented in the firmware for when the hoist is operated with up/down movement. This safety is made with the PCB's relay in mind, which is why a new register has been added to Molift Service Tool. Live Data Register 123 is now observing the state of the relay. If the register reads 0, the relay is not shorting the motor. Opposite, if the register reads 1, the relay is then shorting the motor.

### Information sent to:

All Etac Sales companies

Export

2022-01-04