



# My First Steps

R82 Crocodile  
Starter Kit

**R82**<sup>®</sup>   
by Etac

# R82 Crocodile Starter Kit

We are adding a new accessory to the Crocodile size 1, which will make the Crocodile walker usable for the even smaller children. The Crocodile Starter Kit will give them the opportunity to be able to stand and walk with their peers. The Crocodile allows freedom of movement and facilitates walking in the upright position. The starter kit can be used as the only accessory or together with the other well-known accessories like back support, side supports and sling seat.

## Included in the starter kit



## My First Steps

The Crocodile starter kit – My First Steps makes it possible to retrofit the Crocodile size 1 to an even smaller size.

It allows the handles of the top frame to be positioned lower and closer to the child's body. This makes it perfect for even smaller children, allowing them to explore their surroundings and take the next step in their development.



## Two products in one

With the new starter kit, children will be able to start using the Crocodile walker from the time they are approximately 70 cm tall, depending on body stature. This will prolong the life time of the Crocodile size 1, making it usable for children from 70 cm up to 115 cm.



**Starter kit -  
My First Steps**

90-250 mm

**Size 1**

190-350 mm

### Width between handles

Compared to the standard top frame on the Crocodile size 1, the handles on the starter kit top frame are placed 10 cm closer to each other. This brings the handles closer to the body, providing the child with maximum stability.



**Starter kit -  
My First Steps**

380-700 mm

**Size 1**

420-715 mm

### Height from floor to handles

As with all of the Crocodiles, the top frame in starter kit is adjustable in height. The height adjustability lets the Crocodile grow with the child and when removing the starter kit and returning to a size 1, it will give the child a long period of joy in the walker.

## Promoting independence

The Crocodile encourages independent movement, exploration and discovery. It gives the child an opportunity to become more engaged and active within their environment.



Check out our how-to video to see how to mount the starter kit correct



Read more about the range of accessory options on our website...



## Measurements

		<b>Starter kit - My First Steps</b>	<b>Size 1</b>	<b>Size 2</b>	<b>Size 3</b>
	Width between handles	90-250 mm	190-350 mm	320-600 mm	330-640 mm
	Height from floor to handles	380-700 mm	420-715 mm	550-840 mm	750-1045 mm
	Max. user height	1150 mm	1150 mm	1500 mm	1800 mm
	Max. user mass	30 kg	30 kg	45 kg	80 kg
	Length	660 mm	720 mm	840 mm	870 mm
	Overall width	620 mm	620 mm	690 mm	690 mm
	Folded length	720 mm	720 mm	860 mm	1000 mm
	Folded width	620 mm	620 mm	690 mm	690 mm
	Folded height	310 mm	310 mm	270 mm	290 mm
	Minimum turning radius	1000 mm	1000 mm	1140 mm	1140 mm
	Mass device	5,5 kg	5,5 kg	6,5 kg	12 kg

Further measurements at our website

## Do you know our Crocodile walker?

The Crocodile is a walking aid that helps children and youngsters with disabilities, who need assistance in their daily walking. Placed behind the child, it allows freedom of movement and walking in an upright position.

The multi adjustable handles on the Crocodile enable an optimum position of the arms, creating the best possible working condition for the children.

