Monteringsvejledning (DK)

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
Mounting instruction

Mounting the power-unit on a High-low and High-low:x frame



Dismount the gasspring and the footpedal, to be able to mount the power unit on the High-low and High-low:x frame. Before dismounting, please make sure that the seat unit is being supported. Then follow the below:

1. Remove the screw to dismount the distance piece.
2. Remove the screw to dismount the footpedal.
3. Remove the screw to dismount the gasspring below.
4. Remove the screw above to dismount the gasspring completely.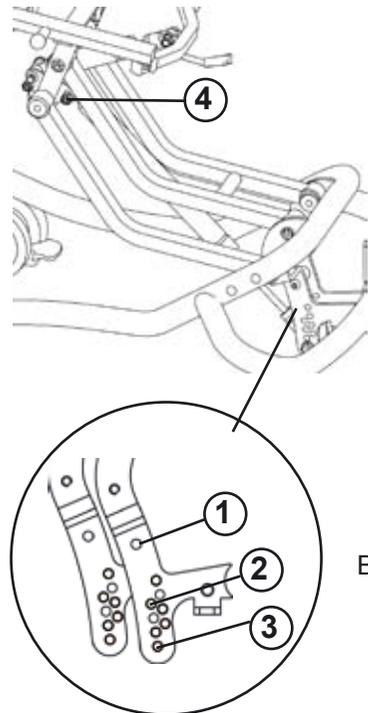

Mounting the power unit:

5. Mount the plate (A) on the batterie box. Mount the enclosed ferrule (B) in the fitting.
6. Mount the enclosed sleeve (C) in the hole in the actuator. The black sleeve must be removed before placing the new sleeve.
7. Mount the enclosed fitting (D) on the actuator. Make sure to turn the fitting the right way, according to out- and indoor frame. Mount the two ferrules in the fitting.
8. Mount, the fitting on the actuator, in the two holes (2) and (3) on the frame.
9. Mount the batterie box on the frame, in the hole (1) above.
10. Mount the plug from the actuator on the batterie box plate (E) with 2 screws. Then mount the remote control (F).
11. Mount the actuator on the frame at pos. (4).

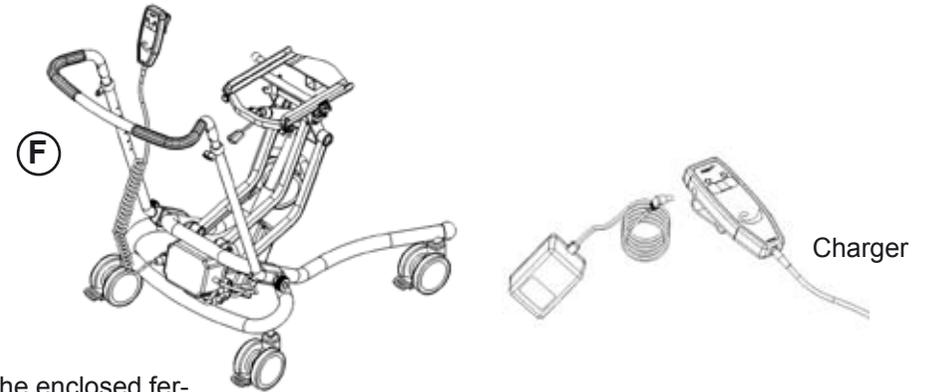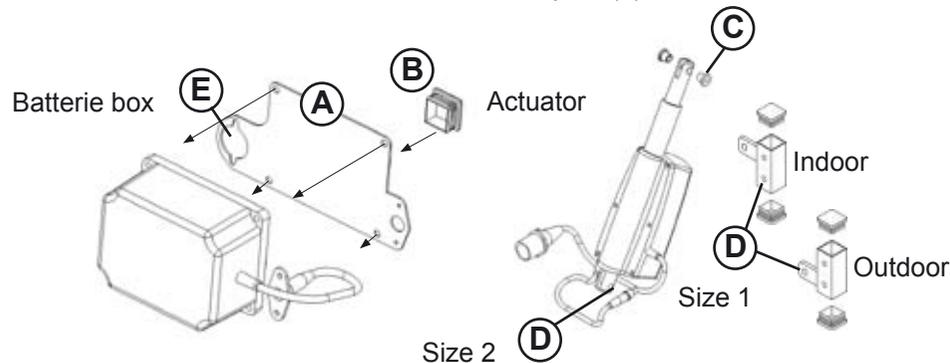Recharge the batteries:

For optimum lifetime of the actuator, connect it to the mains voltage 12 hours before use.

- * Connect the charger to the mains voltage
- * Mount the wire from the charger into the hand control - a small lamp will show
- * Charge the batterie until lamp turns off

To gain the optimal profit of the height adjustment function, the batteries must be recharged every night. The batteries can not be charged to much, but do not ever let them drain completely.

The batteries are to be replaced within a period of 4 years. New batteries must be of the same date and productionmark.