Monteringsvejledning (DK)

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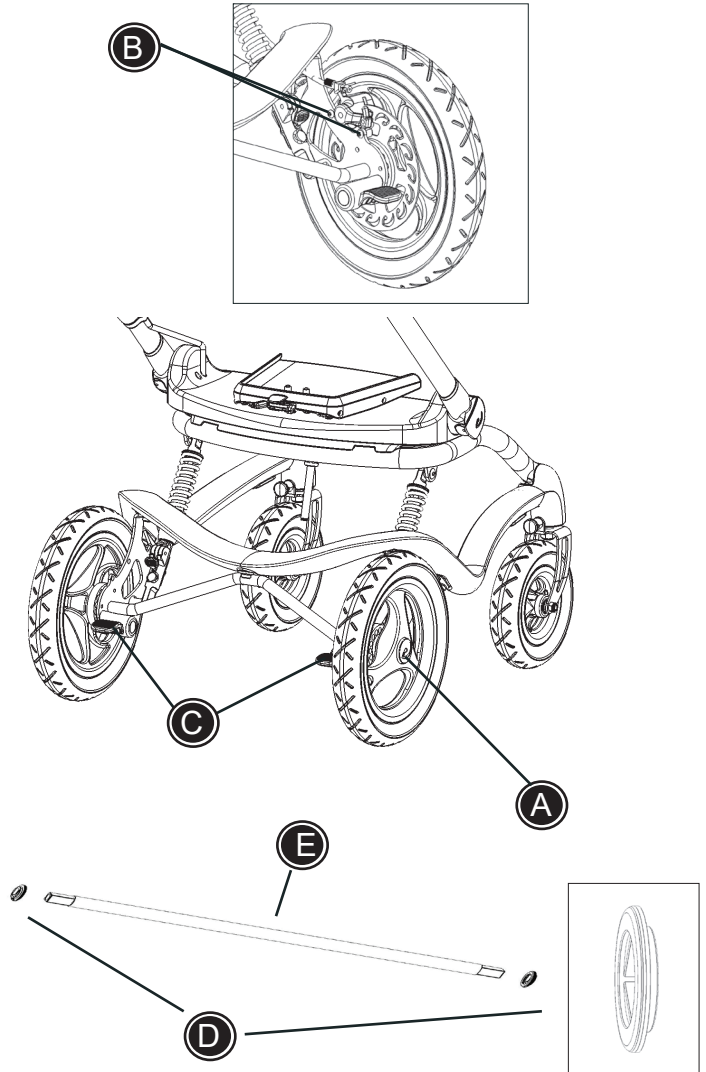
R82[®]

Mounting of kit for central brake

 Should only be performed by trained technician.

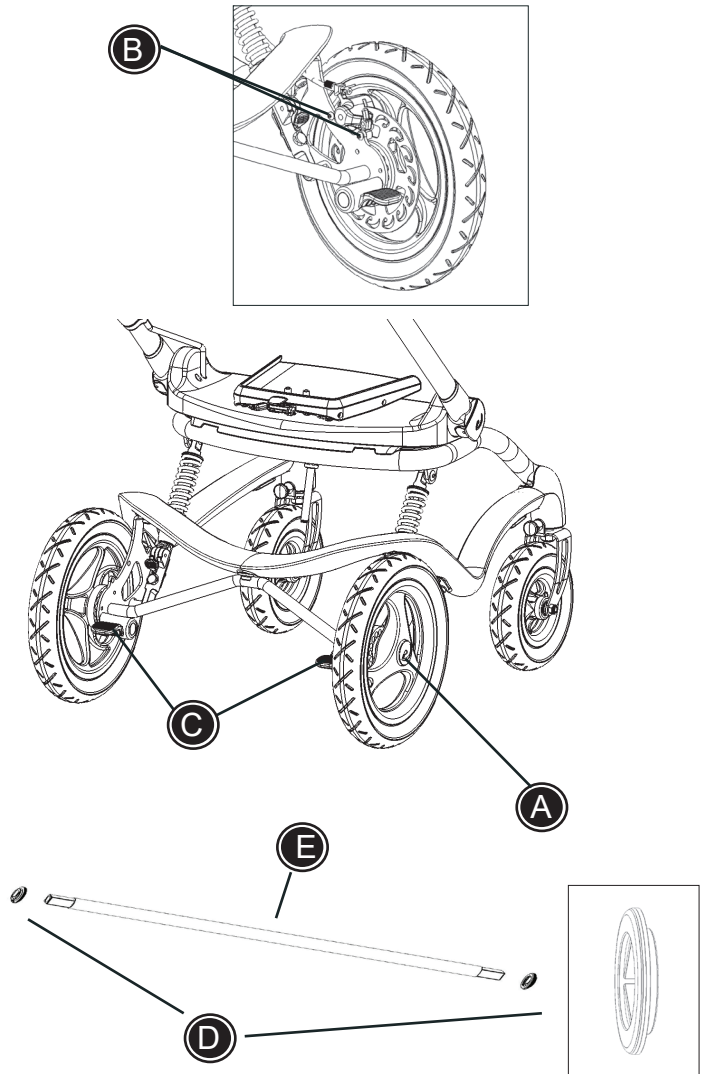
Mounting on Stingray with disc brakes:

1. Remove the safety cap (A) from both rear wheels.
2. Remove the two screws (B) at each side with a 5 mm Allen key and lift away the callipers.
3. Remove 12 mm nut (A) at both rear wheels and remove the rear wheels.
4. Turn the frame upside down.
5. Remove the two screws (C) that hold the brake at each side. Do not mix the two brakes up. Take the brake pedal with spring out of the housing for the brake and remove the bottom lid (this is not to be used anymore).
6. Put the rings (D) from the kit at each side of the bar (E) - the curvature of the rings (D) has to turn towards the center of the bar.
7. Assemble one of the housings for brake and brake pedals and screw it loosely to the frame. Put the springs from the housing for brake at each side of the bar (E) and place one of the ends in the housing for brake on the frame. Put the other housing for brake and brake pedal at the other side of the bar and push it in the place on the frame. The two housings for brake (C) are screwed to the frame again with the two screws at each side.
8. The wheels are put to the frame again together with the brake callipers (B) and the safety caps (A).

Mounting on Stingray without disc brakes:

1. The wheels are removed by means of the quick release system. Then follow items 4. to 7. above, and re-install the wheels.

 Any adjustments and addition of accessories must be placed and fitted correctly and checked before every use.

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