Monteringsvejledning (DK)

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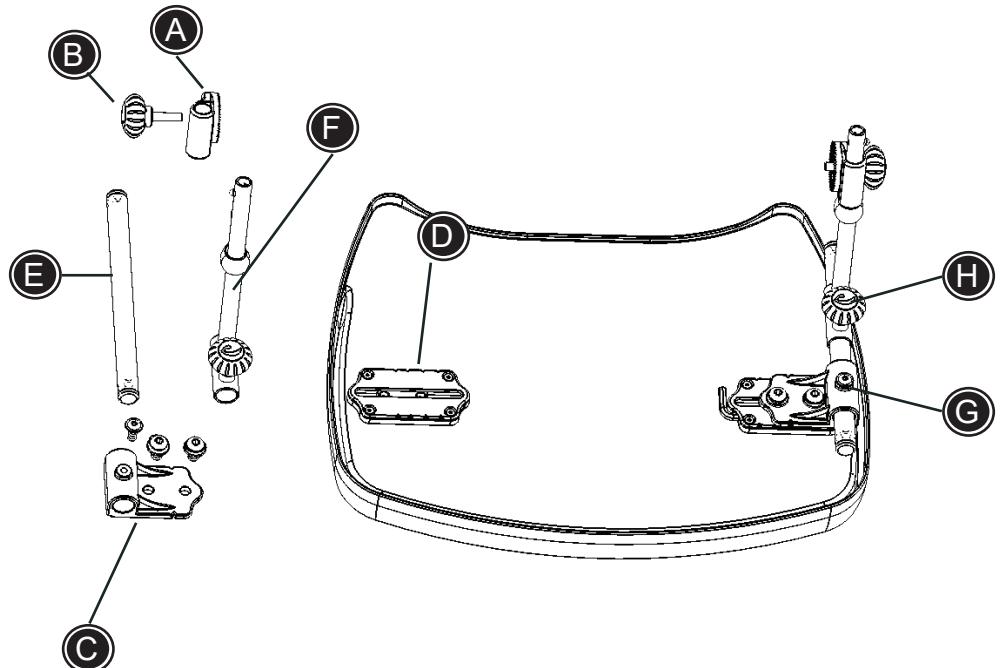
R82[®]

Small tray on Stingray

Follow the instructions below to assemble the tray (the enclosed 5 mm Allen key can be used on all bolts):


Mounting:

- * The holders (A) are mounted at each side of the seat with the finger screw (B) - remove first the protection cap in the fitting at the seat.
- * The fitting (C) is mounted in (D) with the enclosed bolts and washers. Please note the orientation of mounting bracket (C) as per the drawing. There are 4 positional markers in plate (D) and 1 on the mounting bracket (C). For Stingray Sz. 1 please line up the marker on bracket (C) with marker No. 4 (counting from the outside edge of the tray) on plate (D). For Stingray Sz. 2 use marker No. 2 on (D).
- * Assemble the tray rods (E) and (F) by means of the finger screw and mount them in the fitting (C) by means of the bolt (G).
- * The tray is mounted in the holders (A) at each side of the seat.

Adjustment:

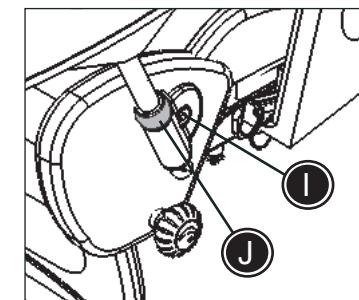- * Adjust the depth of the tray by either loosening the bolt (G) or the finger screw (H) under the tray.
- * Loosen the screws (I) at both sides to adjust the angle of the tray.
- * Move the screw (J) up or down to height adjust the tray.

! Any adjustments and addition of accessories must be placed and fitted correctly and checked before every use.

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