



Etac Prio

A comfort wheelchair
with unique innovations.

 **etac**[®]

Etac Prio – dynamic comfort

By changing seat and back angles and adjusting the elevating leg supports, you can vary body position and pressure points, which is important for seating comfort. But the Etac Prio achieves more than this through its dynamic features.

When the back is reclined, the dynamic pelvic support releases the tension, giving the user a relaxed reclined position. The head support follows the same dynamic principle: It moves slightly backwards when raising the wheelchair user to an upright position, which gives the user freedom to move the head.

Thanks to the Comfort Tilt Lock, there is no risk in unintentionally leaving the user in too upright a position as the tilt lock automatically stops the tilt at a pre set angle, ensuring a relaxed and safe position.



Innovations

Dynamic Pelvic Support

This unique function offers both stability in an upright position and comfort when reclined. This is how it works:



When the back support is reclined, the tension in the pelvic strap is automatically released, letting the wheelchair user sink back.



When the back support is brought to an upright position, the Dynamic Pelvic Support retightens the tension and gives a firm support.

Comfort Tilt Lock

When bringing the wheelchair user to an upright position, the Comfort Tilt Lock will stop the tilt in a comfortable, slightly reclined position. To adjust the tilt angle further upright, just press the tilt lever again and give a little push.


The Comfort Tilt Lock can be set at 0°, 4°, 8° or 12° tilt.



Dynamic Head Support



The Dynamic Head Support gives comfort in all positions. It moves slightly back when sitting upright, to avoid pushing the head forward. Super soft, it also allows the head to sink into the cushion.

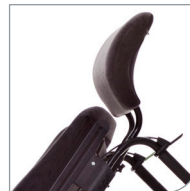

The smart head support attachment ensures the same height adjustment every time it is removed and reattached.

More smart improvements



Foot brake

Easy to manoeuvre for the attendant person, also equipped with a user brake lever. Available with 16" rear wheels.



Platform foot support

The platform foot support allows freedom to move the feet.



Seat depth adjustment

The seat depth is adjusted with the back support, which means that the foot support in relation to the seat is not affected.



Tilt lever

The tilt function is soft, controlled and safe. It is also possible for the attendant person to tilt standing beside the wheelchair.

Features and options



Seat cushion

Standard foam cushion. Cover: hygiene or plush.



Seat cushion comfort

Adjustable at rear with memory foam. Thigh troughs. Cover: hygiene or plush.



Arm support

Height and depth adjustable. Detachable.



Back support cushion basic

Padded, attached with zippers. Cover: hygiene or plush.



Back support cushion comfort

Padded, attached with zippers. Additional side and lumbar support. Cover: hygiene or plush.



Back support upholstery standard

With 5 adjustable Velcro straps and Dynamic Pelvic Support.



Back support upholstery comfort

With 5 adjustable Velcro straps and Dynamic Pelvic Support. Height adjustable pelvic support strap.



Bow handle

Height adjustable. Detachable.



Bow handle adjustable

Height and angle adjustable. Detachable.



Dynamic Head Support soft, narrow

Height adjustable, soft, width: 31 cm. Cover: hygiene or plush.



Dynamic Head Support soft, wide

Height adjustable, soft width: 37 cm. Cover: hygiene or plush.



Dynamic Head Support, soft comfort

Height adjustable, soft. Wide with wings for more comfort. Width: 41 cm. Cover: hygiene or plush.



Traditional head support

Including brace angled at 45°, 90° or swan neck. Adjustable in height, depth and angle. Order attachment separately, see accessories. Cover: Plush.



Foot brake

Only in combination with 16" wheels. Equipped with user brake lever, shown below.



Foot brake – user brake handle

The foot brake above is equipped with a user brake handle.



Brake

Adjustable for all drive wheel sizes and positions.



User brake with angled handle

Provides a larger gripping surface. Available in black or red.



User brake with extension handle

The brake lever can be angled forward to facilitate sideways transfers.



It is possible for the attendant person to tilt standing beside the wheelchair.



Push handle
Height adjustable and detachable.
With sliding stop.



Anti-tippers
Adjustable in height. Folds in under the seat.



Leg support, fixed angle
Swing-away and detachable.
With knee pad.



Heel strap
Adjustable in length.



Elevating leg support
Swing-away and detachable with flip-up calf support, adjustable in height and depth. Cover: hygiene.



Calf strap
Adjustable in length, detachable.



Foot support large
Flip-up. Adjustable in height and angle.



Platform foot support
Foldable and adjustable in height.
Detachable, with calf strap.



Foot support
Flip-up. Adjustable in height and angle.
Standard or long attachment tube.



Castor wheel
6" or 8".



Foot support extension
21 cm. Fits foot support above.



Rear wheel
16", 20", 22" or 24" rear wheel.
Solid tyres. Quick release hub.



Etac Prio can be equipped with attendant brakes, making the handling easier for the person who is pushing, which in turn enhances the safety for both the user and the carer.

Accessories



Attendant brakes
With sliding stop. 100 mm between handle grip and brake lever.



Padding for calf strap
Provides a softer support. Available in 20 cm (7¾"), 30 cm (11¾") and 40 cm (15¾") lengths.



Trunk support
Adjustable in height and depth. Detachable. Cover: Dartex.




Spoke guards
In grey or yellow print.



Tray
Transparent. Fits on arm support.




Tool kit
Contains four ring spanners, three Allen keys, one socket.

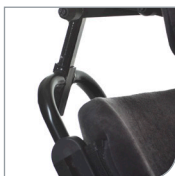

Attachment for traditional head support
Height-adjustable and removable. Available with straight or angled mounting.



Widening kit
Moves each arm support 1,25 cm outwards.

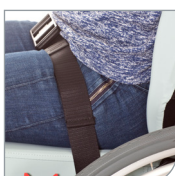

Positioning belt
Two-point fixation. Several attachment points on the frame.

Dimensions and weight



Seat width	Total width	Seat height 20", 22", 24"	Seat height 16"	Seat depth*	Transport width	Weight**	Transport weight***
40 cm short	59 cm	43.5-52.5 cm	45 or 47.5 cm	43-45.5-48 cm	56 cm	32.3 kg	21.7 kg
40 cm long	59 cm	43.5-52.5 cm	45 or 47.5 cm	49-51.5-54 cm	56 cm	32.6 kg	22.0 kg
45 cm short	64 cm	43.5-52.5 cm	45 or 47.5 cm	43-45.5-48 cm	61 cm	32.6 kg	22.0 kg
45 cm long	64 cm	43.5-52.5 cm	45 or 47.5 cm	49-51.5-54 cm	61 cm	32.9 kg	22.3 kg
50 cm short	69 cm	43.5-52.5 cm	45 or 47.5 cm	43-45.5-48 cm	66 cm	32.9 kg	22.3 kg
50 cm long	69 cm	43.5-52.5 cm	45 or 47.5 cm	49-51.5-54 cm	66 cm	33.2 kg	22.6 kg
55 cm short	74 cm	43.5-52.5 cm	45 or 47.5 cm	43-45.5-48 cm	71 cm	33.2 kg	22.6 kg
55 cm long	74 cm	43.5-52.5 cm	45 or 47.5 cm	49-51.5-54 cm	71 cm	33.5 kg	22.9 kg

* Measured from the middle of the back support cushion

** With arm supports, elevating leg supports, calf support, rear wheels, anti-tippers

*** Without arm supports, elevating leg supports, calf support, anti-tippers, rear wheels

Facts

Back support height	60 cm
Max. user weight	135 kg
Tilt angle	+3° - 20°
Back support adjustable	0° - 30°
Balance positions	Etac Prio with 20", 22" or 24" rear wheels has three alternative balance positions for each seat height.

Frame Colour



Pearl grey -94



Coral -96

Cushion cover



Dark grey,
plush textile



Lagoon green,
hygiene textile



Dark grey,
hygiene textile

Washing instructions

All covers are machine washable at 60° C.



The seat hygiene cover is machine washable at 95° C.
All hygiene covers are also wipeable with a wet cloth.



All covers are labeled with washing instructions.



40 years of standard-setting innovations

Swedish design and quality, with comfort and improvement in the front seat.

Since 1973 Etac has developed and manufactured products that improve the everyday life for people with disabilities. Since 1981 we have also been developing wheelchairs, with several innovations which have set new standards. Our focus on detailed improvement, together with robust design have become Etac's distinguishing hallmark. Our track record in functional, innovative and comfortable products includes a number of Swedish and international design awards.



red dot



Good Design
Award



Excellent
Swedish Design

Etac AB
Kista Science Tower
SE-164 51 Kista, Sweden
Tel +46 371 – 58 73 30
Fax +46 371 – 58 73 90
info@etac.se www.etac.com

etac[®]

Creating Possibilities