

Etac Socky stocking aids



Dressing should be easy

If pain or reduced mobility restricts your ability to put on socks or stockings, the Etac stocking aids will help. We have three models, with slightly different functions which are all easy to use.

For a perfect result, remember



Pull up the sock properly and over the hooks



Place the string behind the calf



Pull up with ease



Instruction video at *etac.com* How to use Etac Socky

Etac Socky stocking aid, Short

Socky short is the perfect stocking aid for both short and knee long socks including flight socks. It consists of a plastic base with two hooks under a pre-folded nylon fabric.

How to use Socky Short

The sock is simply pulled onto the Socky fabric until the top of the sock reaches over the hooks. Place the string behind the calf and pull. Socky short slides up around your foot and lower leg with ease, placing the socks perfectly.



Article Item. No Etac Socky, Short 80601001-2

Length & Weight Length 40 cm (15³/₄") Weight 94 g (3.3 oz) Material Fabric: Polyamide Plastic parts: Polypropylene

Etac Socky stocking aid, Long

Socky long is the ideal solution when hand strength and mobility is restricted or you need to reduce the friction between the sock and floor/carpet. It can be used for short as well as knee long socks. Socky long consists of a long plastic sheet placed under a pre-folded nylon fabric with a handle on top.



How to use Socky Long

The sock is simply placed around the Socky fabric and pulled to the top. The handle is used to pull Socky long up, while the plastic sheet slides on the floor and ensures that the sock stays in place.

Article Item. No Etac Socky, Long 80601001-2

Length & Weight Length 60 cm (23¹/₂") Weight 146 g (5.1 oz) **Material** Fabric: Polyamide Plastic parts: Polypropylene



Etac Socky stocking aid for support and compression stockings

This stocking aid is designed to suit different lengths and types of support and compression stockings. It consists of two long nylon fabric sections that connect with Velcro. Each part has a large handle to grip to pull the stocking over the foot and leg.

How to use Socky Long

Place the support stocking onto the Socky. The fabric parts divide when you pull up the stocking. It also facilitates a care person to assist.



Article Socky stocking aid for support and compression stockings

Item. No 80601003-2 Length & Weight Length 100 cm (39¼") Weight 90 g (3.2 oz)

Material Fabric: Polyamide

Etac AB Kista Science Tower SE-164 51 Kista, Sweden Tel +46 371 58 73 30 Fax +46 371 58 73 90 info@etac.se www.etac.com

