


Etac M100

Etac M100TR

Sparepart list

74994P 2019-09-23

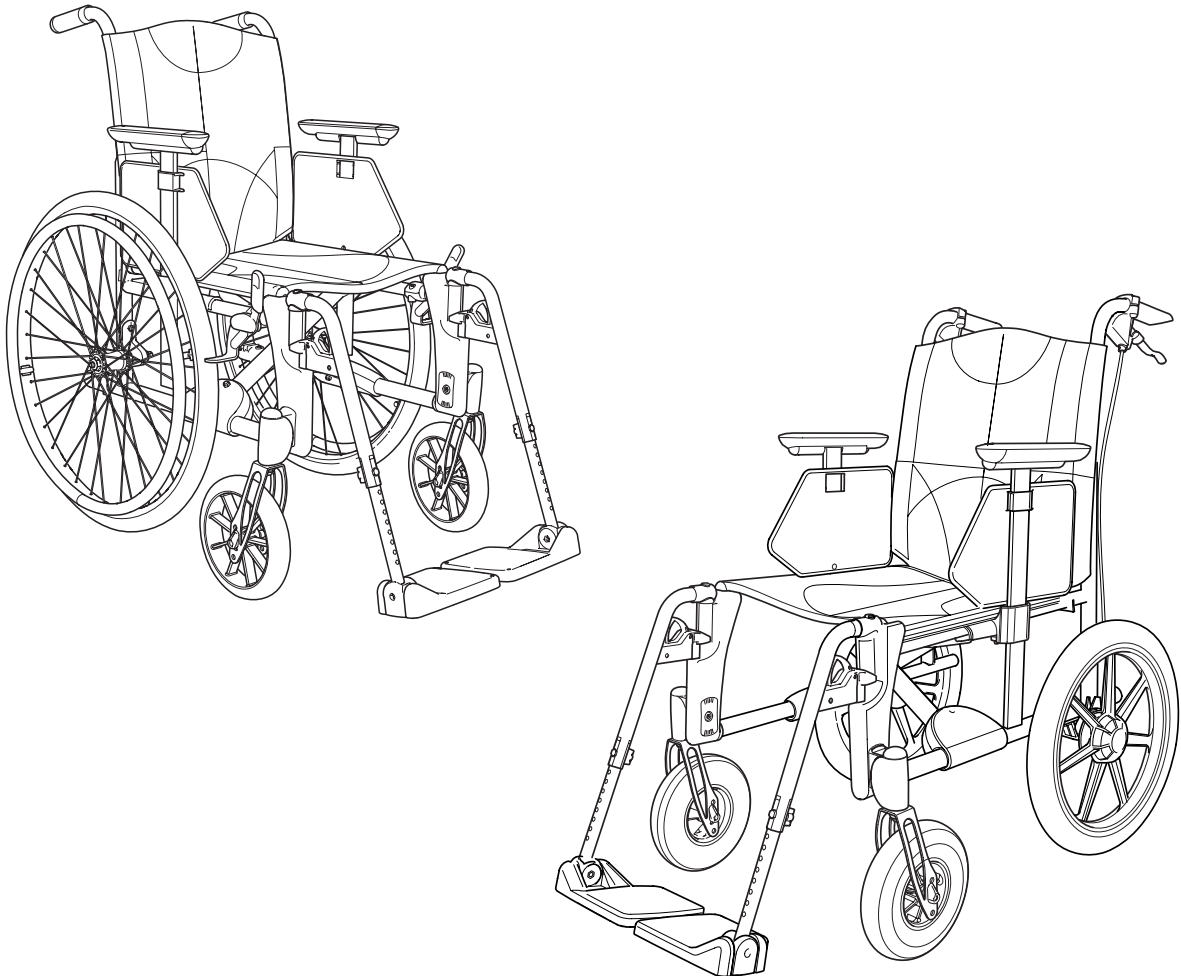

Index

Content	Page
Side frame	3
Push handles	4
Cross frame	5
Legrests	6
Footrests	7
Brakes	8
Rear wheels	9
Castors and front forks	10
Upholstery fixed	11
Upholstery adjustable	12
Armrests	13
Tray	14
Spoke guard	15
Headrest	16
Positionig belt	17
Cane holder	18
Calf strap	19
IV stand	20

Colour codes chassis:


Grey=42

Upholstery:

70=Black

60=Black cloth upholstery


Side frame

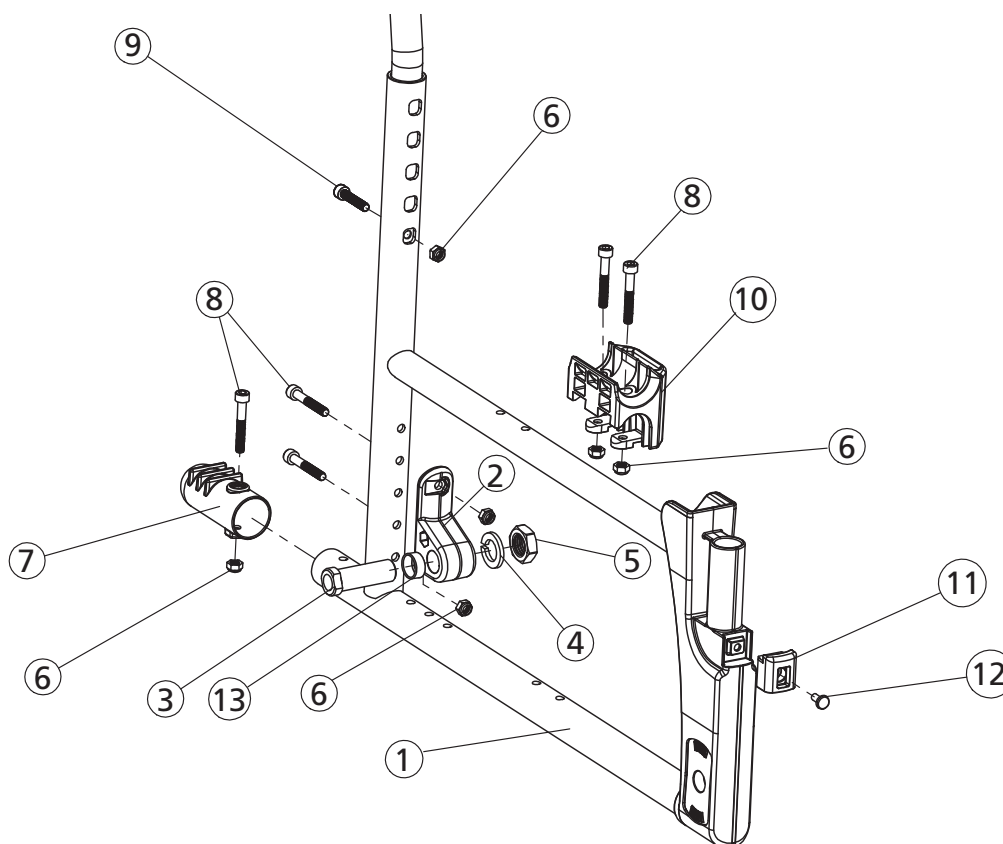

Pos.	Description	Item no.
1	Side frame M100 short.....	27060-xx
	Side frame M100 long.....	27210-xx
2	Attachment rear wheel M100.....	82273
3	Axle sleeve M100 D12.....	20024
4	Locking washer AZ 5/8	13039
5	Nut M16x 1 NV 24 B8	12052
6	Locking nut M6 Fzb.....	12017
7	Tilter M100 grey.....	50472
8	Screw M 6x 40 MC6S 10.9 fzb	11229
9	Screw M 6x 30 MC6S fzb	11019
10	Attachment armrest M100	50859
11	Locking knob side frame part	50845
12	Screw DG 50x 14 WN 1552.....	11528
13	Spacer rear wheel low Cross 5.....	82588
14	Touch up paint	80517-xx


Colour codes chassis:
Grey=42

Push handles

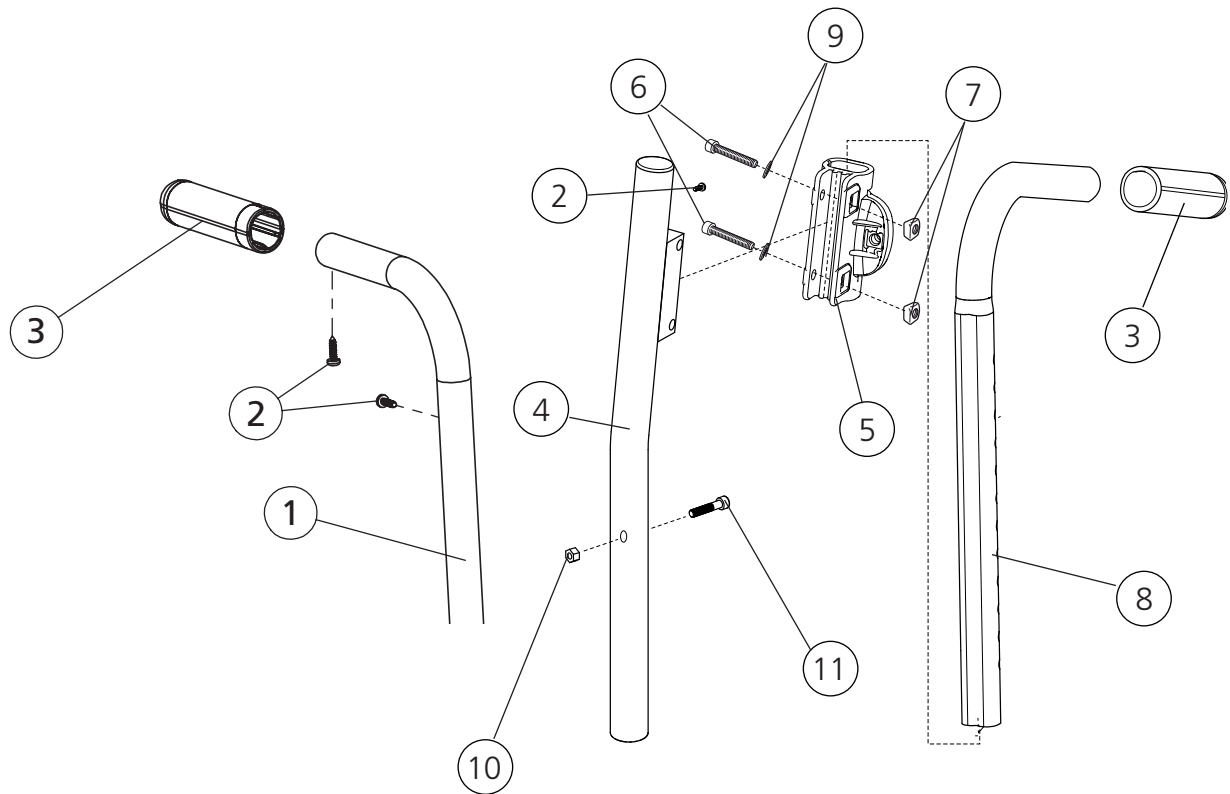


Pos.	Description	Item no.
1	Backrest tube M100 upper	27067-42
2	Screw B 8x16 RTS Fzb st 4.2	11041
3	Cover for handle M100	82221
4	Attachment tube push handle M100	27068-01
5	Attachment push handles height adj. compl. ..	25676
6	Screw M6x30 MC6S	11398
7	Square nut high M6	12058
8	Push handles height adjustable	27298
9	Washer BRB M6	13030
10	Locking nut M 6 fzb	12017
11	Screw M 6x 30 MC6S fzb	11019
2,3,8	Push handle height adj Cross std	27338-01

Cross frame

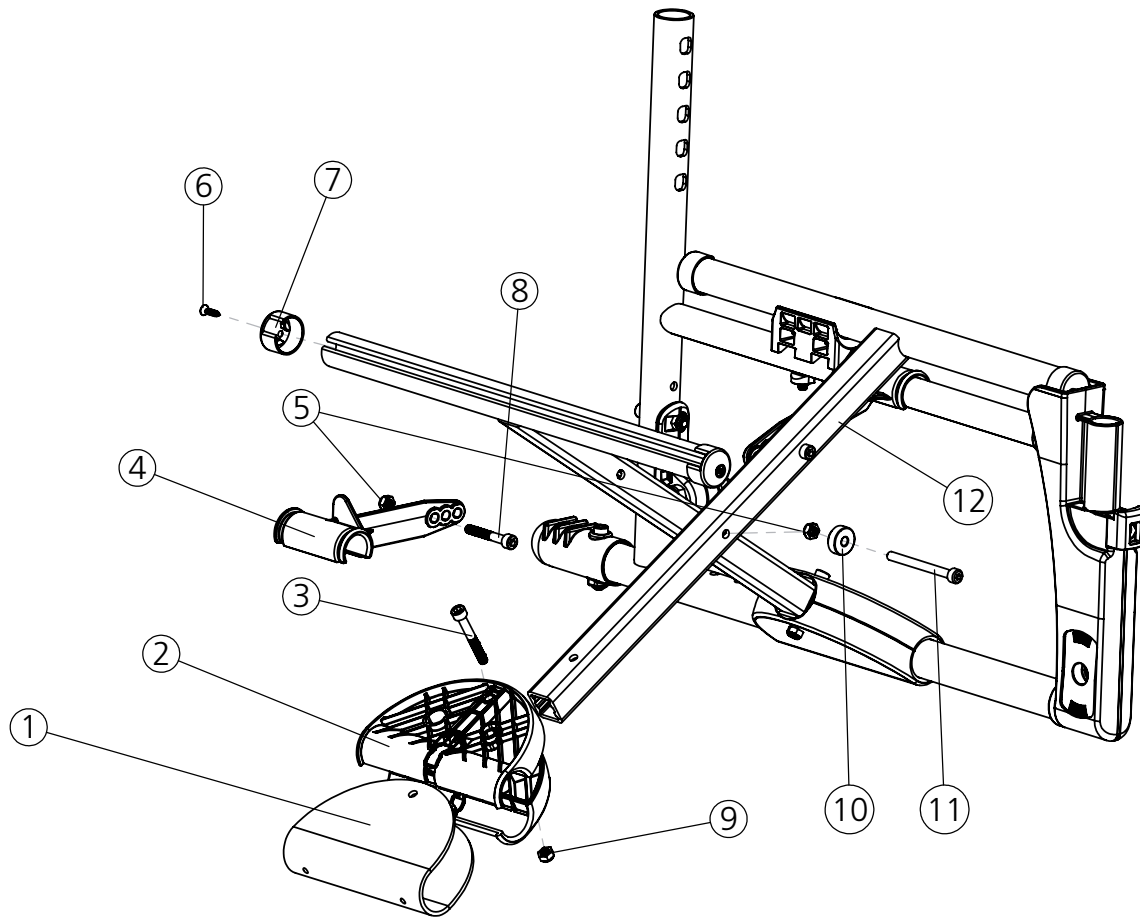


Pos.	Description	Item no.	Pos.	Description	Item no.
1	Attachment plate cross frame.....	82223	12	Cross frame M100 short 45 cm	27095-42
2	Attachment cross frame M100	50477		Cross frame M100 short 47.5 cm	27096-42
3	Screw M 6x 45 MC6S 10.9 black.	11231		Cross frame M100 short 50 cm	27097-42
4	Plate cross frame M100			Cross frame M100 long 37.5 cm right	27115-42-1
	37.5-42.5 cm	50474		Cross frame M100 long 37.5 cm left	27115-42-2
	Plate cross frame M100 45-50 cm	50475		Cross frame M100 long 40 cm right	27116-42-1
5	Locking nut M6 Fzb.....	12017		Cross frame M100 long 40 cm left..	27116-42-2
6	Screw B 8x 16 FXS	11401		Cross frame M100 long 42.5 cm right	27117-42-1
7	Plug seat tube Cross/M100	50406		Cross frame M100 long 42.5 cm left	27117-42-2
8	Screw M 6x 40 MC6S fzb	11229		Cross frame M100 long 45 cm right	27118-42-1
9	Locking nut M 6	12035		Cross frame M100 long 45 cm left..	27118-42-2
10	Spacer plastic cross frame	50483		Cross frame M100 long 47.5 cm right	27119-42-1
11	Screw M 6x 65 MC6S 10.9 fzb ...	11236		Cross frame M100 long 47.5 cm left	27119-42-2
12	Cross frame M100 short 37.5.....	27092-42		Cross frame M100 long 50 cm right	27120-42-1
	Cross frame M100 short 40.....	27093-42		Cross frame M100 long 50 cm left..	27120-42-2
	Cross frame M100 short 42.5.....	27094-42			

Legrests

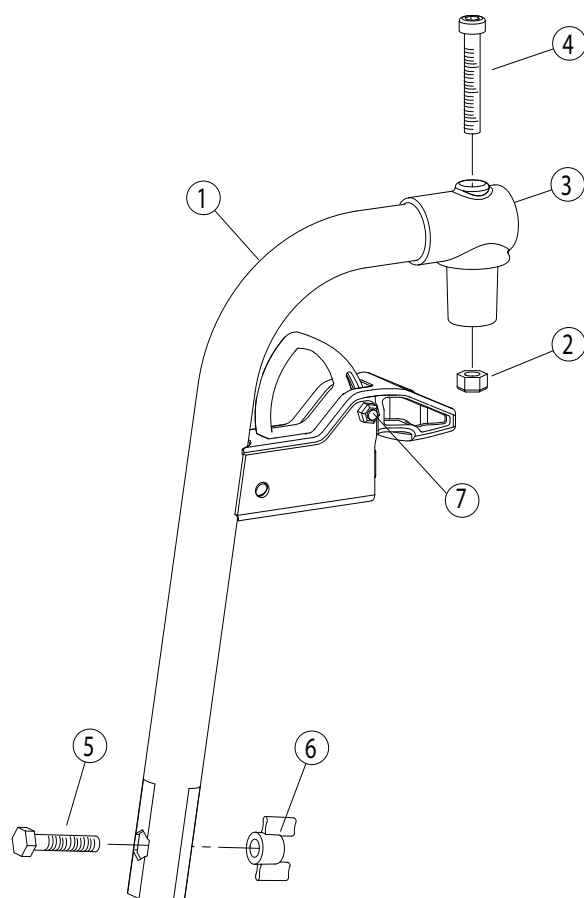


Pos.	Description	Item no.
1-7	Legrest Next/M100 wide angle	26120-42
1-7	Legrest Next narrow angle	26121-42
2	Locking nut M 6	12035
3	Pin legrest Next/M100	50854
4	Screw M6x 35 MC6S	11533
5	Screw M6x 30 M6S	11555
6	Wing nut WN 314.1	80847
7	Locking nut M4 domed A2.....	12000

Footrests

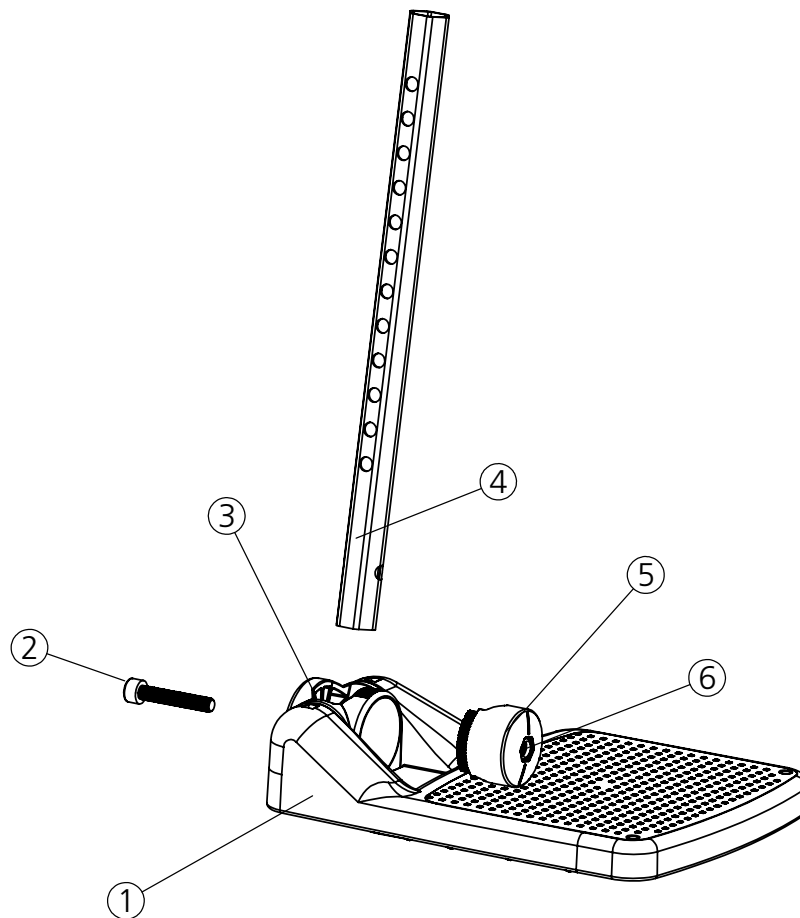


Pos.	Description	Item no.	Pos.	Description	Item no.
1-6	Footrest M100 std 37,5 cm right	82341-1	1-6	Footrest M100 long 37,5 cm right	82352-1
	Footrest M100 std 37,5 cm left..	82341-2		Footrest M100 long 37,5 cm left	82352-2
	Footrest M100 std 40 cm right...	82342-1		Footrest M100 long 40 cm right	82353-1
	Footrest M100 std 40 cm left.....	82342-2		Footrest M100 long 40 cm left..	82353-2
	Footrest M100 std 42,5 cm right	82343-1		Footrest M100 long 42,5 cm right	82354-1
	Footrest M100 std 42,5 cm left..	82343-2		Footrest M100 long 42,5 cm left	82354-2
	Footrest M100 std 45 cm right...	82344-1		Footrest M100 long 45 cm right	82355-1
	Footrest M100 std 45 cm left.....	82344-2		Footrest M100 long 45 cm left..	82355-2
	Footrest M100 std 47,5 cm right	82345-1		Footrest M100 long 47,5 cm right	82356-1
	Footrest M100 std 47,5 cm left..	82345-2		Footrest M100 long 47,5 cm left	82356-2
	Footrest M100 std 50 cm right...	82346-1		Footrest M100 long 50 cm right	82357-1
	Footrest M100 std 50 cm left.....	82346-2		Footrest M100 long 50 cm left..	82357-2
			2	Screw M 6x 35 MC6S	11533
			3	Attachment footrest M100	82276
			4	Attachment tube M100 std.....	82317
				Attachment tube M100 long	82318
			5	Angle plate footrest M100.....	82277
			6	Locking nut M6 Fzb	12017
			7	Heel strap footrest	24458

Brakes

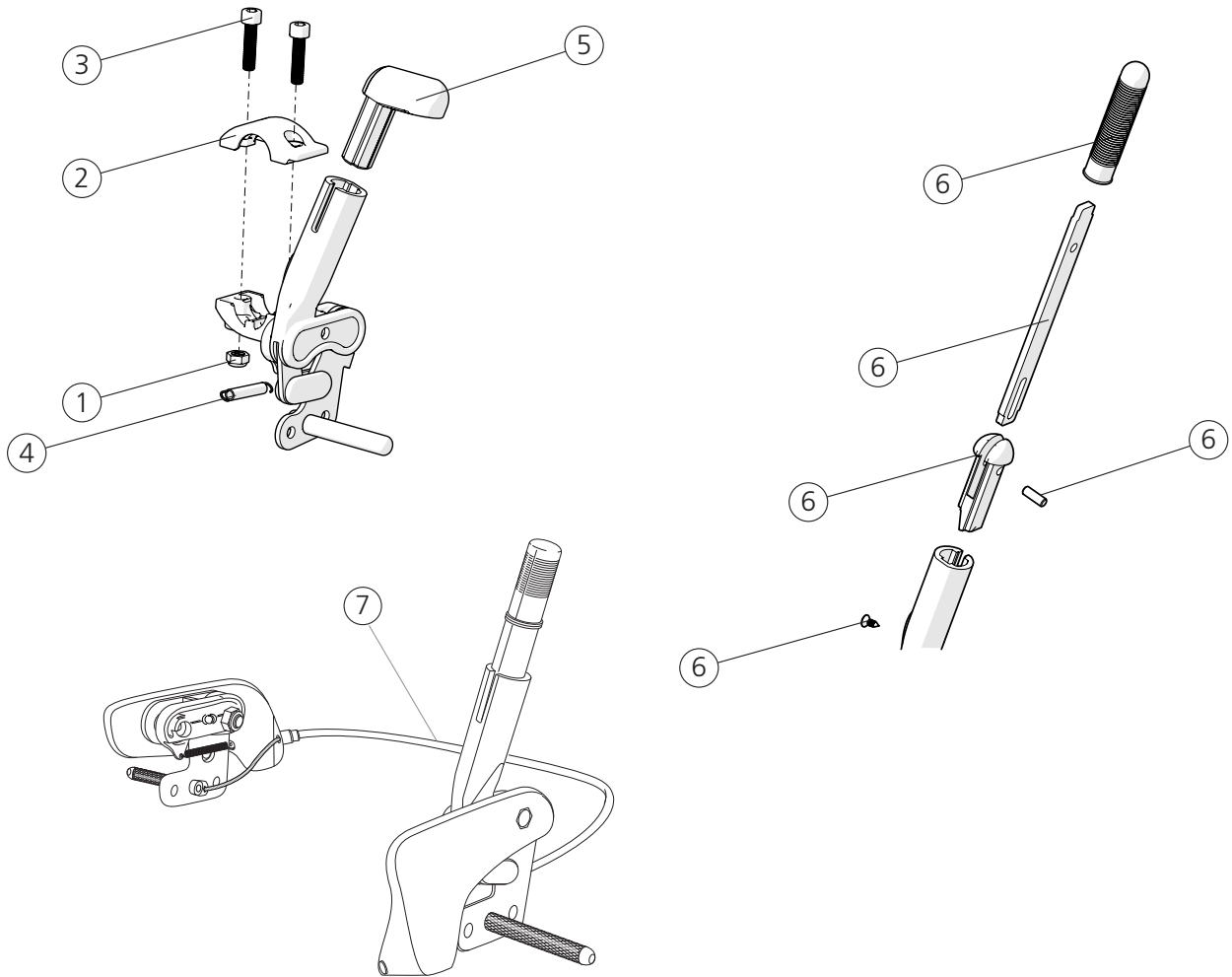


Pos.	Description	Item no
1-5	Brake with angled red handle right	28446-1
	Brake with angled red handle left	28446-2
	Brake with angled black handle right	28444-1
	Brake with angled black handle left	28444-2
1	Locking nut M6 DIN 985-6 zink plated.....	12017
2	Clamp brake	50479
3	Screw M6x 25 MC65 8.8 zink plated	11033
4	Spring brake	80796
5	Plug for angled brake red 1 pce	51012
	Brake handle angled compl kit red	27574
	Brake handle angled compl kit black	25537
6	Brake lever extension	25515
1-4, 6	Brake M100/Crissy with extended handle foldable right	28455-1
	Brake M100/Crissy with extended handle foldable left	28455-2
7	Single hand brake compl M100/Crissy right	28456-1
	Single hand brake compl M100/Crissy left.....	28456-2

Rear wheels

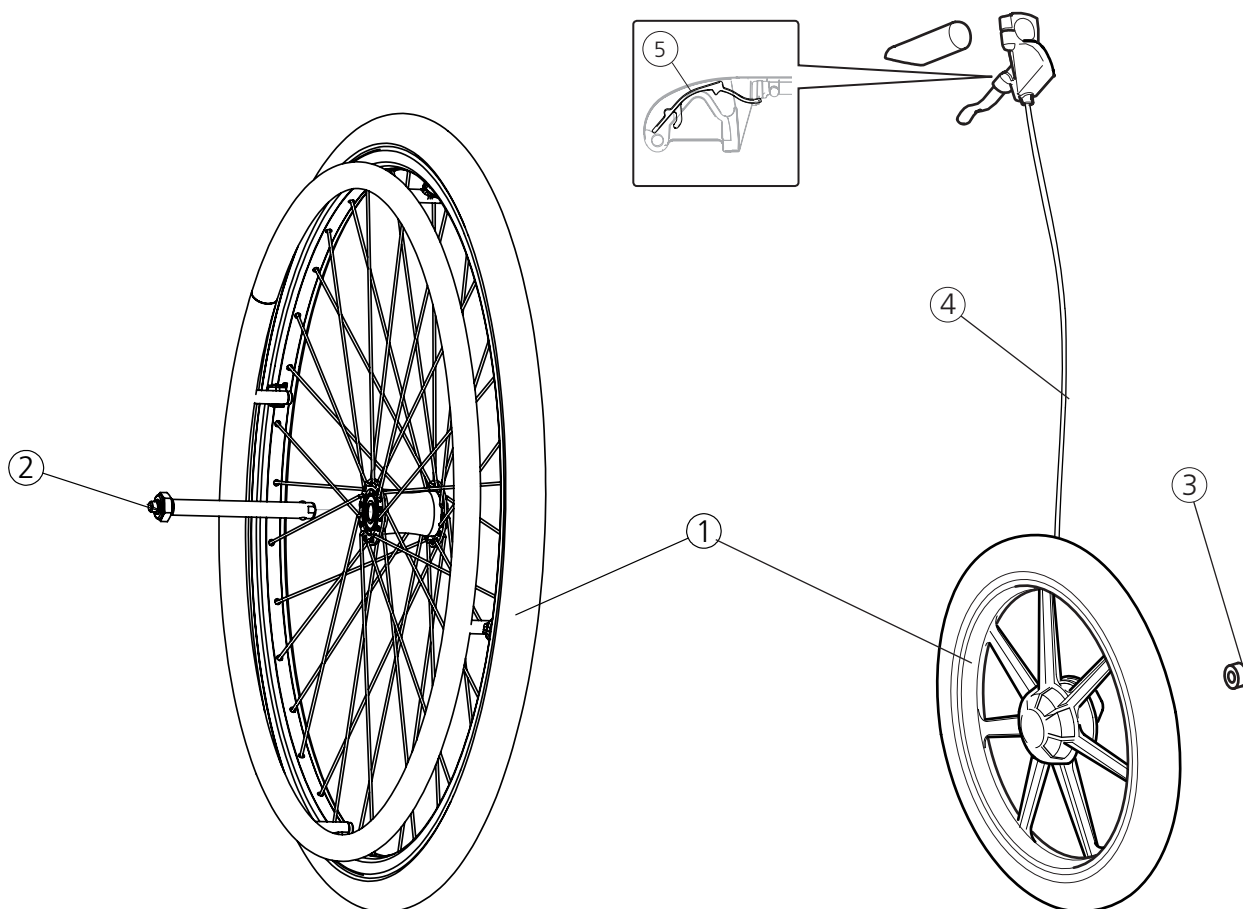


Pos.	Description	Item no.
1	Rear wheel 20 solid/alu M100	82486
	Rear wheel 22 solid/alu M100	82485
	Rear wheel 24 solid/alu M100	82484
	Rear wheel 20 drum brake solid/alu	81337
	Rear wheel 22 drum brake solid/alu	81339
	Rear wheel 24 drum brake solid/alu	81346
	Wheel 16" solid right.....	83283
	Wheel 16" solid left.....	83284
	Rear wheel 24 C5 solid excl axle*	27567
	<i>*Spacer 82588 is required to mount this wheel on M100. see Pos.13 section chassis on p. 3</i>	
2	QR axle for M100 rear wheel steel hub.	82483
	Axle drum brake Cr/Tw/M100 138mm QR	81325
3	Mutter M12	12078
4	Attachment kit drum brake rear wheel	
	20" 22" 24" hight adj push handles....	27359
	Attachment kit drum brake rear wheel	
	20" 22" 24" fixed push handles	27360
	Attachment kit drum brake	
	wheel 16" hight adj push handles.....	27361
	Attachment kit drum brake	
	wheel 16" fixed push handles.....	27362
5	Wire lock brake handle 100 mm 10 pcs	82854
	Wire lock brake handle 75 mm 10 pcs .	82855

Castors and front forks

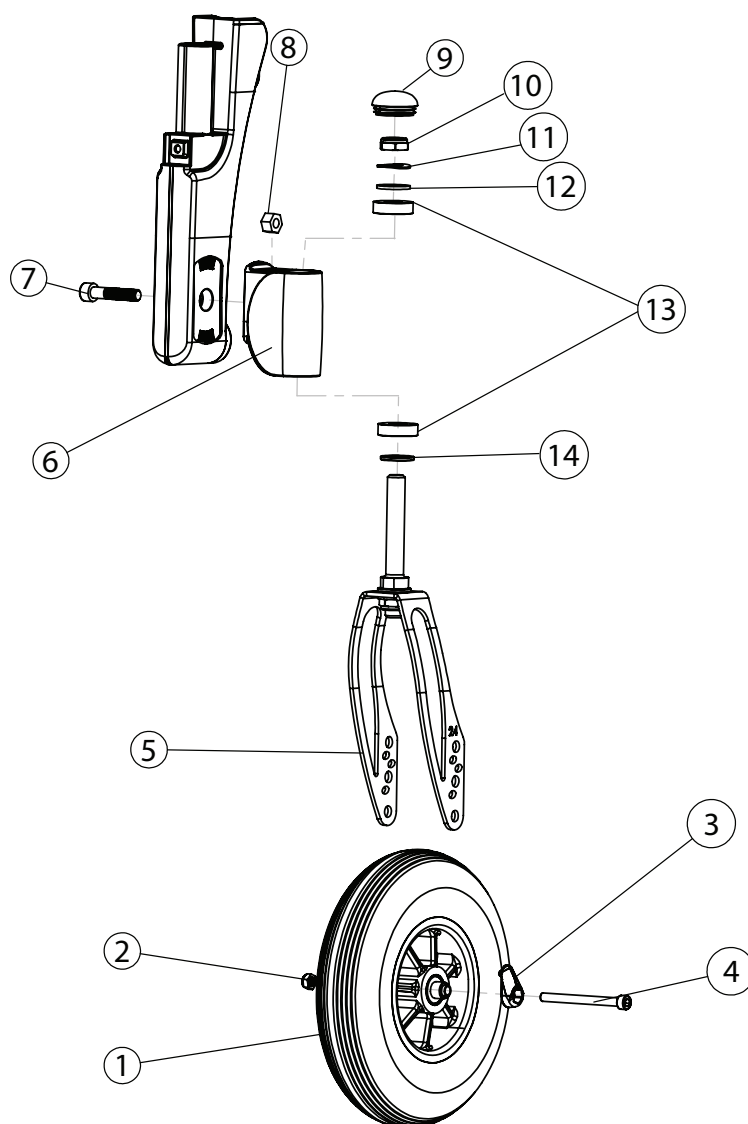


Pos.	Description	Item no.	Pos.	Description	Item no.
1-4	Castor 8 200x35 solid soft	27864	6	Front fork housing M100 black	83029
	Castor 8 200x50 solid soft	27868	7	Screw M 8x 1x40 MC6S 12.9 black...	11305
	Castor 6 150x30 solid soft	27863	8	Locking nut M 8x1 DIN 985	12051
	Castor 5 125x30 solid hard	27865	9	Plug front fork housing M100	51136
2	Locking nut M6	12035	10	Locking nut 1/2 UNF low black	12037
3	Cover axle wheel PA 6	51135	11	Washer 13x24x1.2 DIN 137B fzb	13040
3	Cover axle wheel PA 6 (50 pcs)	27954	12	Washer BRB 13.5 black	13031
4	Screw M 6x 75 MC6S 12.9 black	11338	13	Ball bearing 1/2 EE4 2RS	80244
5	Front fork M100 std black	27126-01	14	Washer BRB 13.5 black, chamfered...	13032
	Front fork M100 short	27125-01			

Upholstery fixed

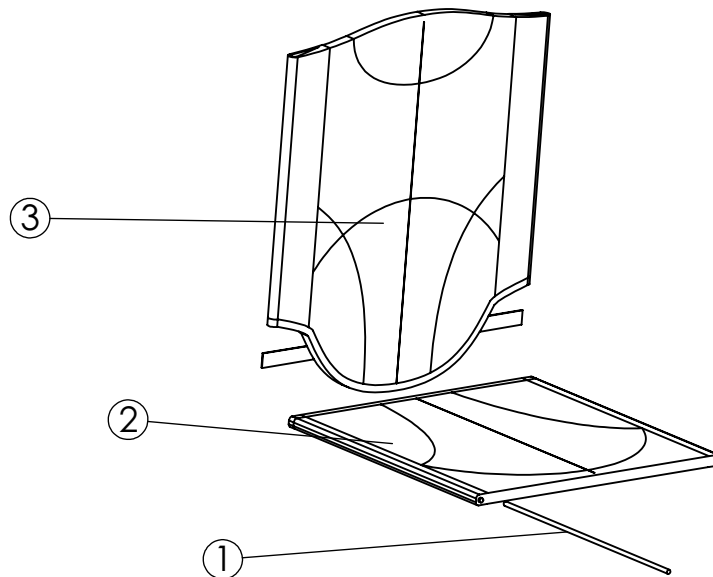


Pos.	Description	Item no.	Pos.	Description	Item no.
1	Tension rod L381 short	80617	3	Upholstery back M100 fixed 37.5 cm	62523-70
	Tension rod L446 long	80618		40 cm	62524-70
2	Upholstery seat M100 short			42.5 cm	62525-70
	37.5 cm	62501-70		45 cm	62526-70
	40 cm	62502-70		47.5 cm	62527-70
	42.5 cm	62503-70		50 cm	62528-70
	45 cm	62504-70		Back upholstery M100 fixed height adjustable	
	47.5 cm	62505-70		push handles	37.5 cm 62534-70
	50 cm	62506-70			40 cm 62535-70
	Upholstery seat M100 long				42.5 cm 62536-70
	37.5 cm	62512-70			45 cm 62537-70
	40 cm	62513-70			47.5 cm 62538-70
	42.5 cm	62514-70			50 cm 62539-70
	45 cm	62515-70			
	47.5 cm	62516-70			
	50 cm	62517-70			

Upholstery adjustable

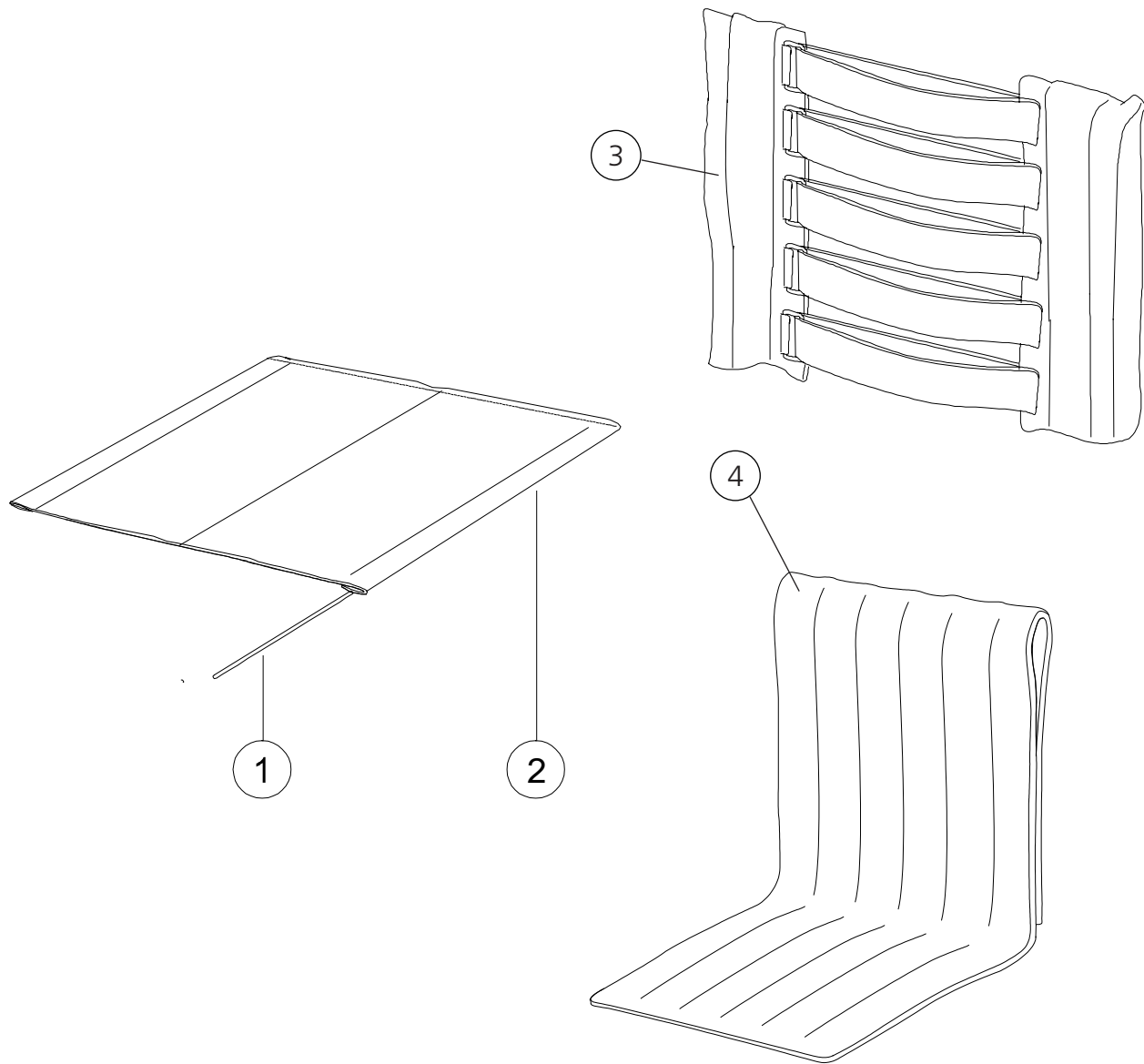


Pos.	Description	Item no.	Pos.	Description	Item no.
1	Tension rod L381 short	80617	3	Upholstery back M100/Twin adj	
	Tension rod L446 long	80618		35-37,5 cm	62364-70
2	Upholstery seat M100 short			40-42,5 cm	62365-70
	37.5 cm	62490-70		45 cm	62366-70
	40 cm	62361-70		47,5-50 cm	62367-70
	42.5 cm	62380-70		Upholstery back M100/Twin adj, adj push	
	45 cm	62362-70		handles	35-37,5 cm 62395-60
	47.5 cm	62491-70			40-42,5 cm 62396-60
	50 cm	62363-70			45 cm 62397-60
	Upholstery seat M100 long				47,5-50 cm 62398-60
	37.5 cm	62545-70	4	Cover back M100/Twin	
	40 cm	62546-70		37,5 cm	62454-70
	42.5 cm	62547-70		40 cm	62369-70
	45 cm	62548-70		42,5/45 cm	62370-70
	47.5 cm	62549-70		47,5/50 cm	62371-70
	50 cm	62550-70			

Armrests

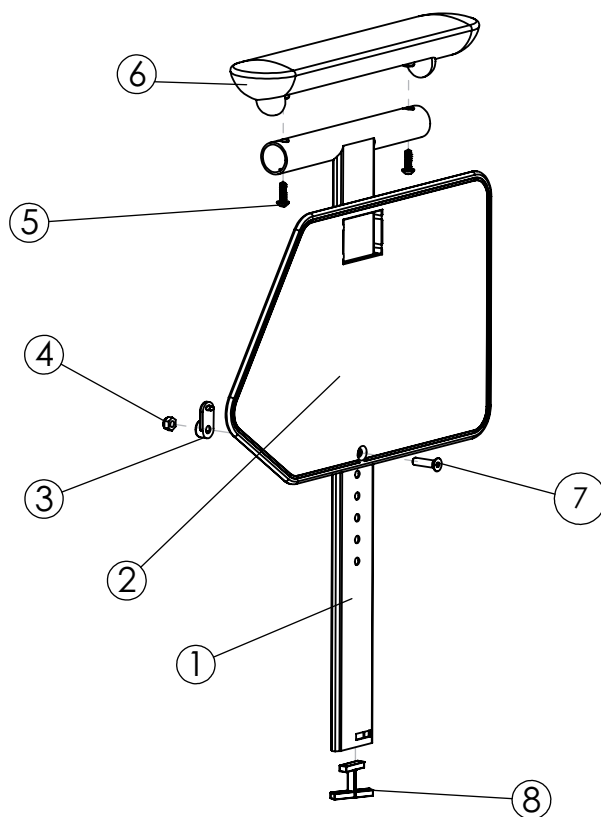


Pos.	Description	Item no.
1-7	Armrest M100 short right	27230-1
	Armrest M100 short left.....	27230-2
	Armrest M100 long right	27231-1
	Armrest M100 long left.....	27231-2
1	Armrest profile M100 short.....	82296
	Armrest profile M100 long right	82297-1
	Armrest profile M100 long left	82297-2
2	Sideguard Cross Cross/M100 right	50316
	Sideguard Cross Cross/M100 left	50317
3	Nut pocket armrest	50916
4	Locking nut M 5	12036
5	Screw B 8x19 RXS/A2 ph st 4	11064
6	Armrest pad M100 short right	82294-1
	Armrest pad M100 short left	82294-2
	Armrest pad M100 long right	82295-1
	Armrest pad M100 long left	82295-2
7	Screw M 5x 20 MF6S black	11526
8	Lock armrest M100	50915

Tray

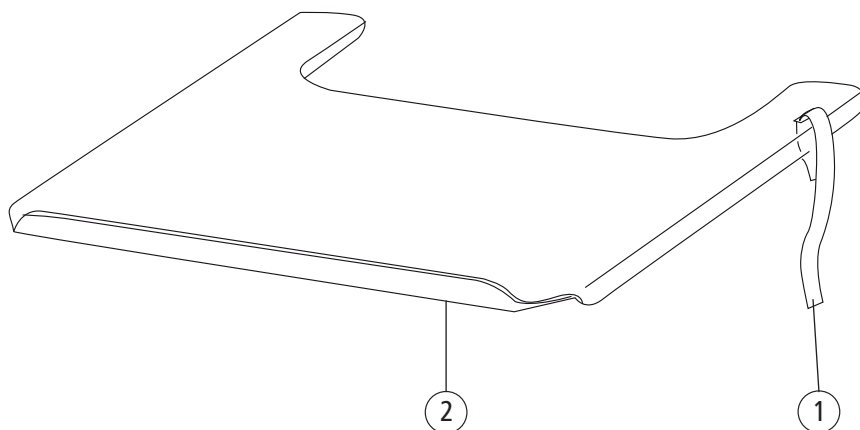


Pos	Description	Item No.
1	Strap for tray 27.5-50 cm (11-20")	62103
1-2	Tray 47.5 cm compl for wheelchair 37.5 cm (15")	25187
	Tray 50 cm compl for wheelchair 40 cm (16").....	24418
	Tray 52.5 cm compl for wheelchair 42.5 cm (17")	25188
	Tray 55 cm compl for wheelchair 45 cm (18").....	24419
	Tray 57.5 cm compl for wheelchair 47.5 cm (19")	25731
	Tray 60 cm compl for wheelchair 50 cm (20").....	24601

Spokeguard

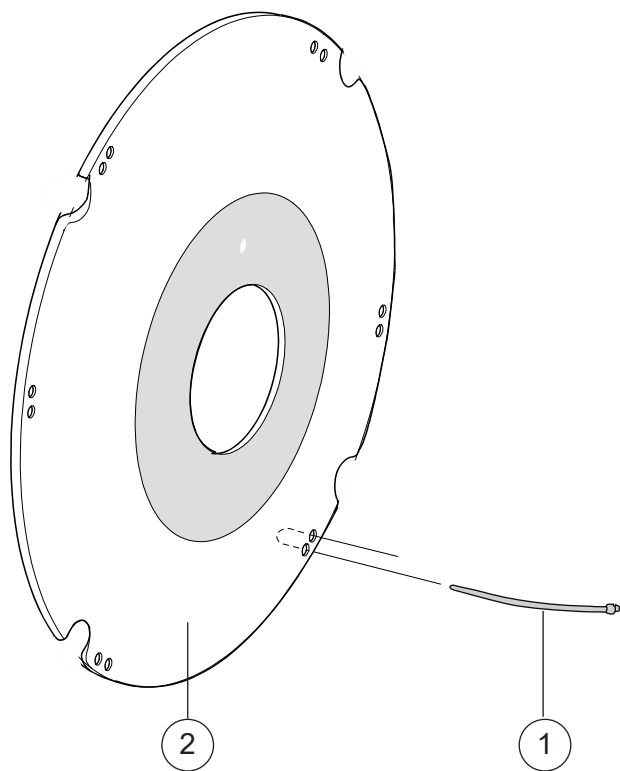


Pos	Description	Item No
1	Attachment kit spokeguard	26276
1-2	Spoke guard 20" with logo grey.....	25502-28
	Spoke guard 20" with logo yellow.....	25502-29
	Spoke guard 22" with logo grey.....	25503-28
	Spoke guard 22" with logo yellow.....	25503-29
	Spoke guard 24" with logo grey.....	25504-28
	Spoke guard 24" with logo yellow.....	25504-29

Headrest

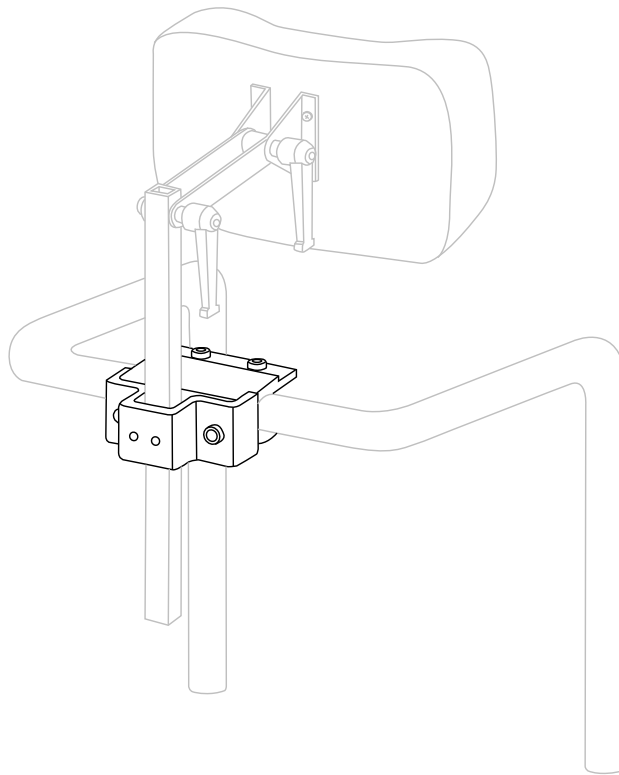


Pos	Description	Item No.
1	Headrest mod 1 Dartex 26x14 cm incl tube with two joints excl attachment	24462-63
	Headrest mod 1 dark grey 26x14 cm incl tube with two joints excl attachment	24462-60
2	Headrest bracket Cross back 2 compl	25795

Positioning belt

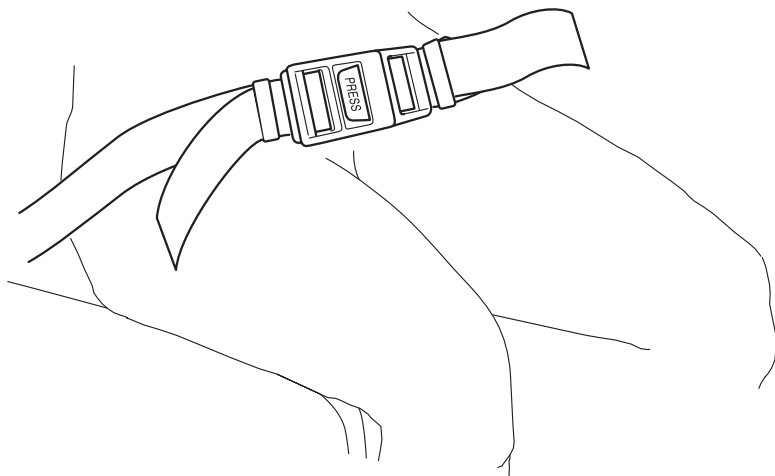


Pos	Description	Item No.
1	Positioning belt	80209440

Cane holder

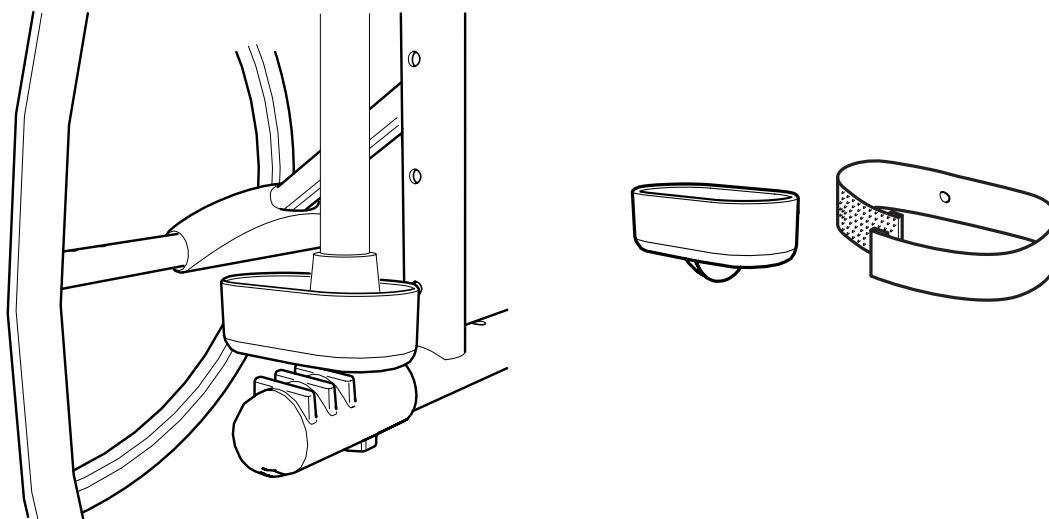


Pos	Benämning	Art Nr
1	Cane holder M100 compl.....	27235

Vadband

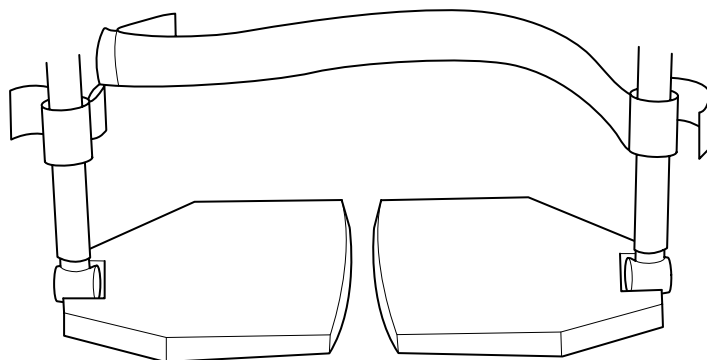

Pos	Benämning	Art Nr
1	Calf strap Cross/Next 35/37.5 cm (14/15") compl	25356
	Calf strap Cross/Next 40/42.5 cm (16/17") compl	25357
	Calf strap Cross/Next 45/47.5 cm (18/19") compl	25358
	Calf strap Cross/Next 50/52.5 cm (20/21") compl	25359

IV stand

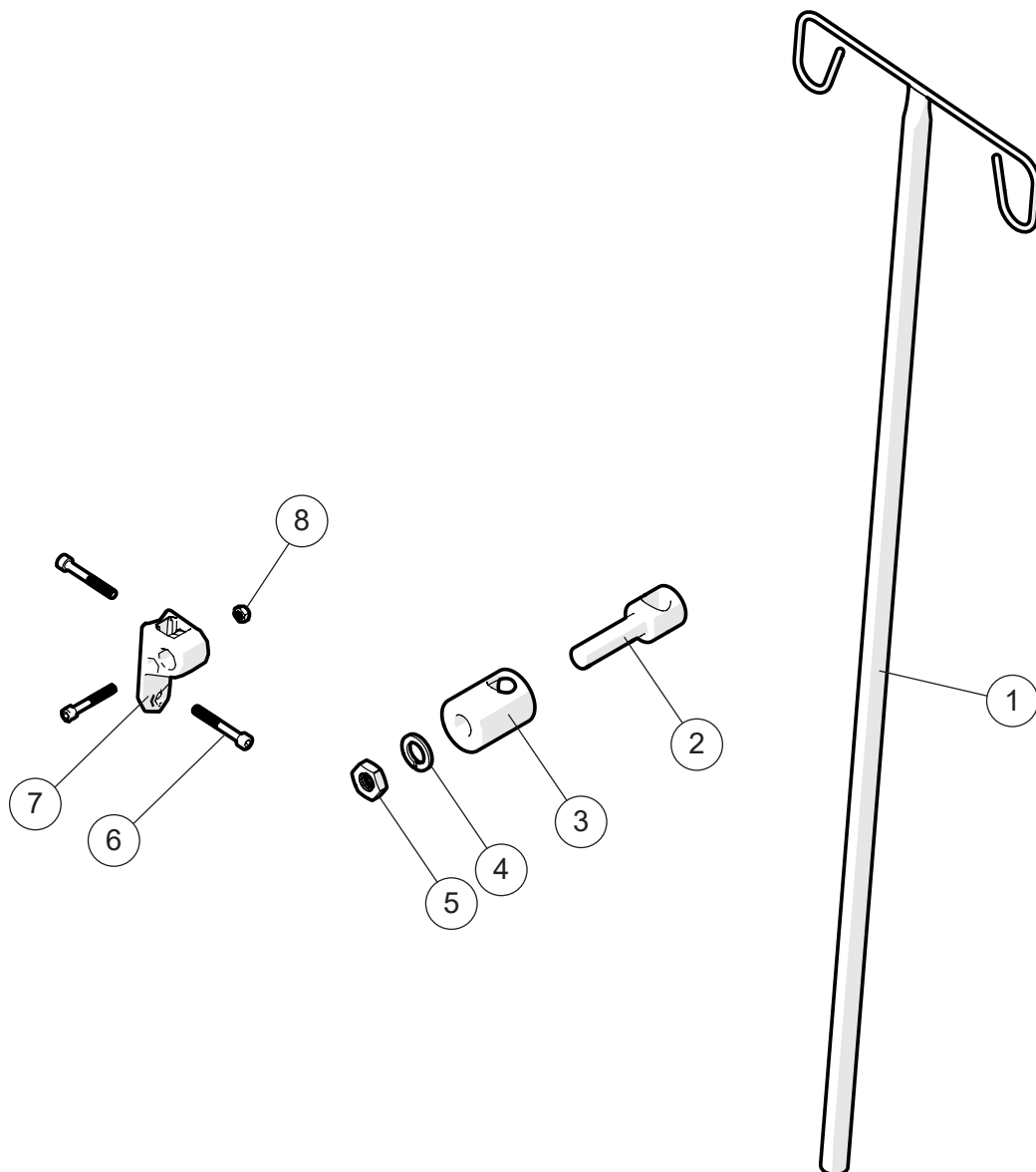

Pos	Benämning	Art Nr
1-7	IV stand M100 compl	28324
1	IV stand	83053
2-7	Attachment IV stand	83145
2	Locking inner	
3	Locking outer	
4	Locking washer M16	
5	Nut M16.....	
6	Screw MC6S M6x45	
7	Attachment rear wheel	