Etac Fresh

Manual

etac®



781871 21-09-01

sv no da en de fi fr nl it es pt


CE

A







8160 0011
8160 0021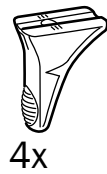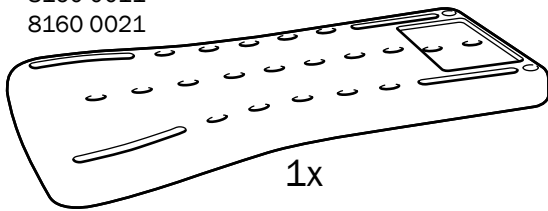




8160 0014
8160 0024

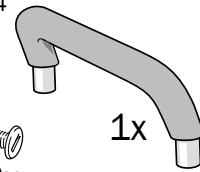
B

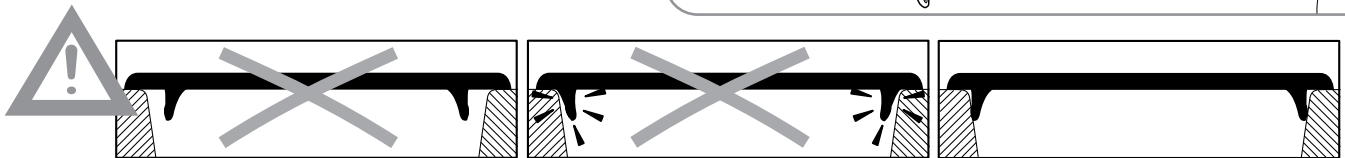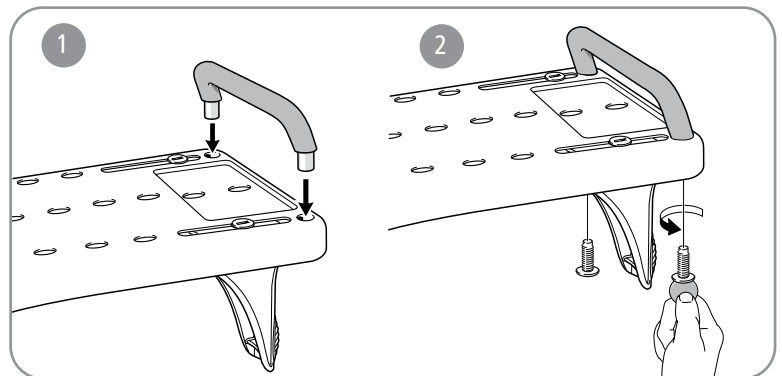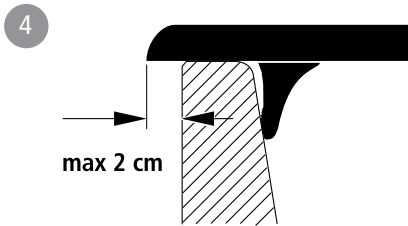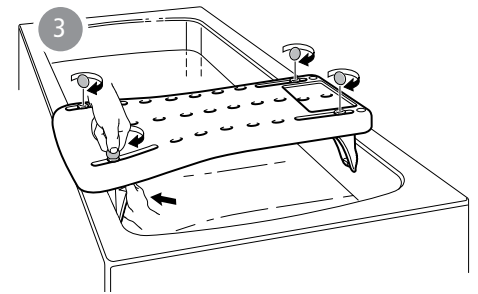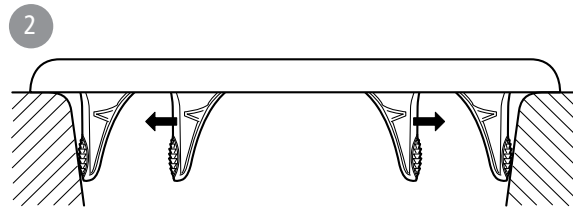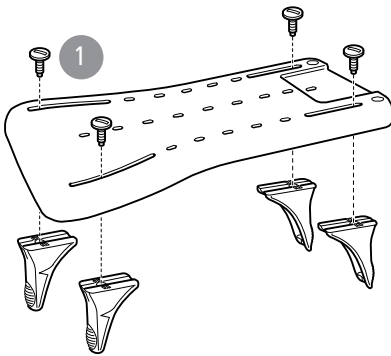
C

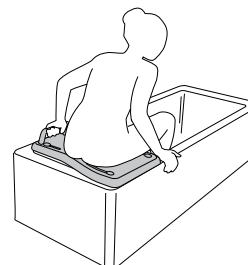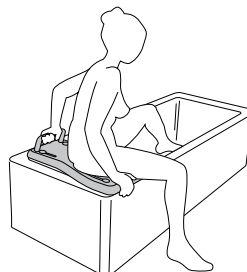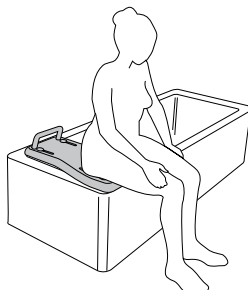D

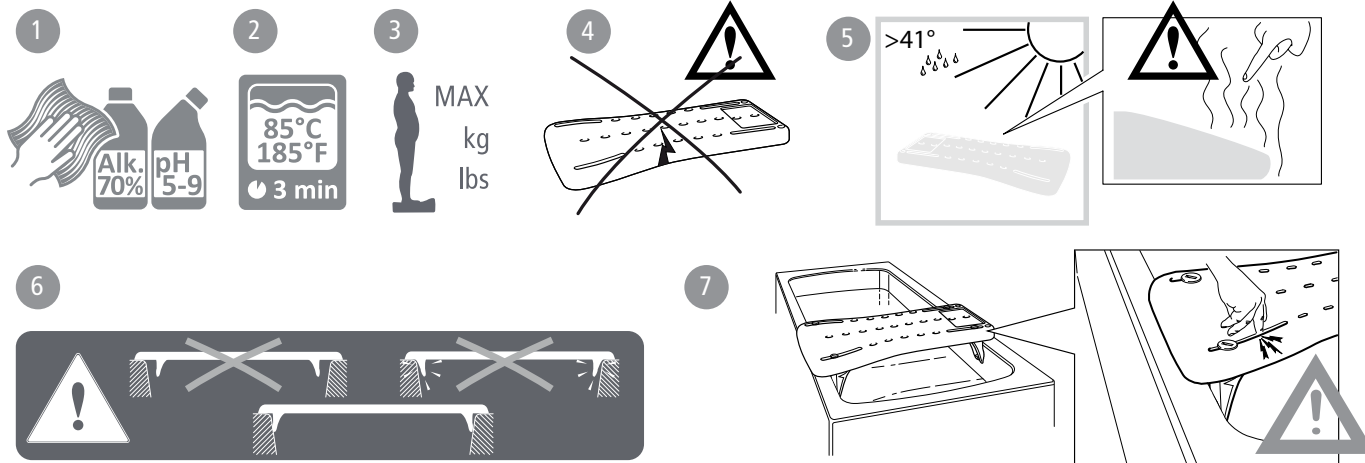E

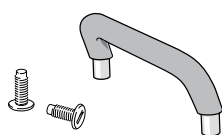SV

Svenska

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en English

Intended use

Plastic bath board, with non-slip pattern and sturdy brackets to make washing safe and easy. The extra wide seating surface on one side makes it easier to get into and out of the bath tub. Easy to clean. Etac Fresh is intended for use in home environments and in institutions, e.g. hospitals and nursing homes. The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

Delivery contents *Figure A*

Installation *Figure B*

Usage *Figure C*

Maintenance and safety *Figure D*

1. Clean/disinfect the product with normal non-abrasive cleaning agents with a pH value of between 5-9, or with 70% disinfectant solution. Rinse and dry. The product is otherwise maintenance-free.

2. The product can withstand cleaning in a sterilizer at 85°C for 3 minutes. This applies for reconditioning.

3. Maximum user weight 150 kg.

4. A defective product must not be used.

5. Heated product can cause burns.

6. Check before each use that the product is correctly assembled.

7. Risk of crushing!

The materials used are corrosion-resistant.

The materials used are resistant to all general disinfectants.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Guarantee, service life

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 10 years. For complete information regarding the service life of the product, see www.etac.com.

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Etac Supply Center AB