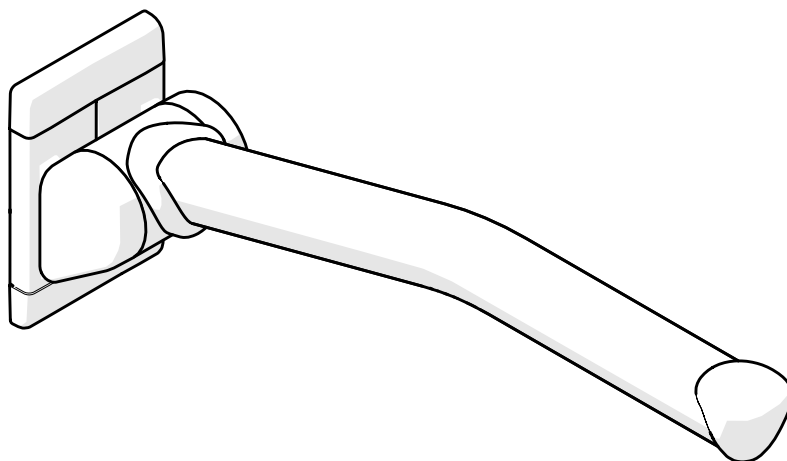
# Etac Rex

## Manual

etac®

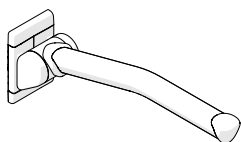
78570.4 19-02-27


sv no da en de fi fr nl it es pt

A


Etac Rex 60 cm


Etac Rex 70 cm



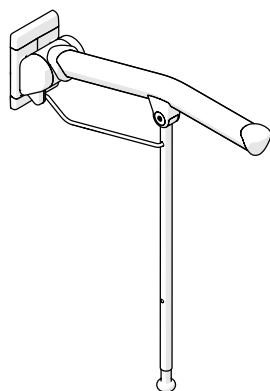
Etac Rex 70 cm

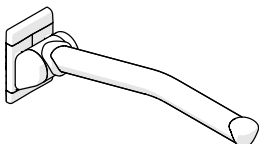
Etac Rex 85 cm


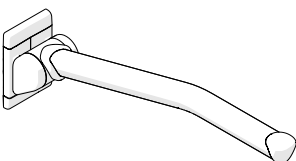
Etac Rex 85 cm

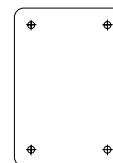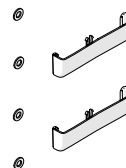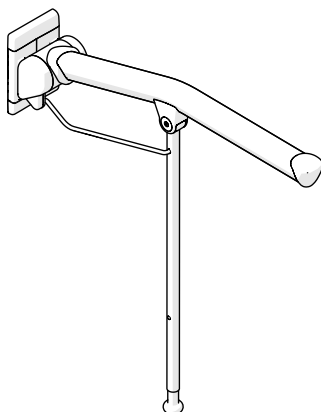**B**

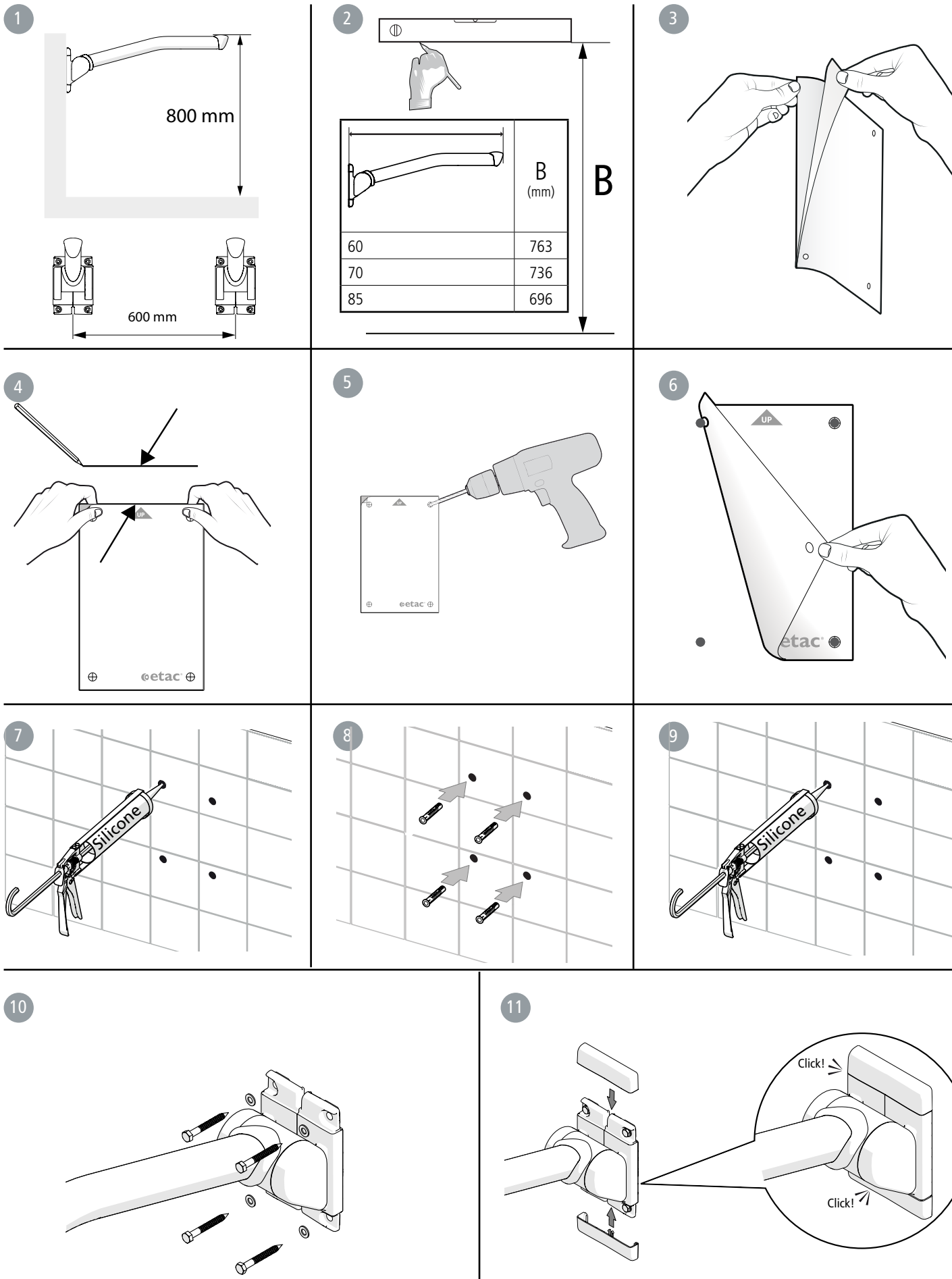12

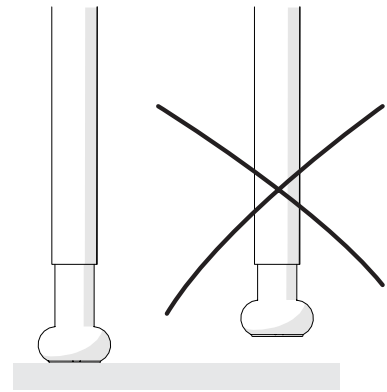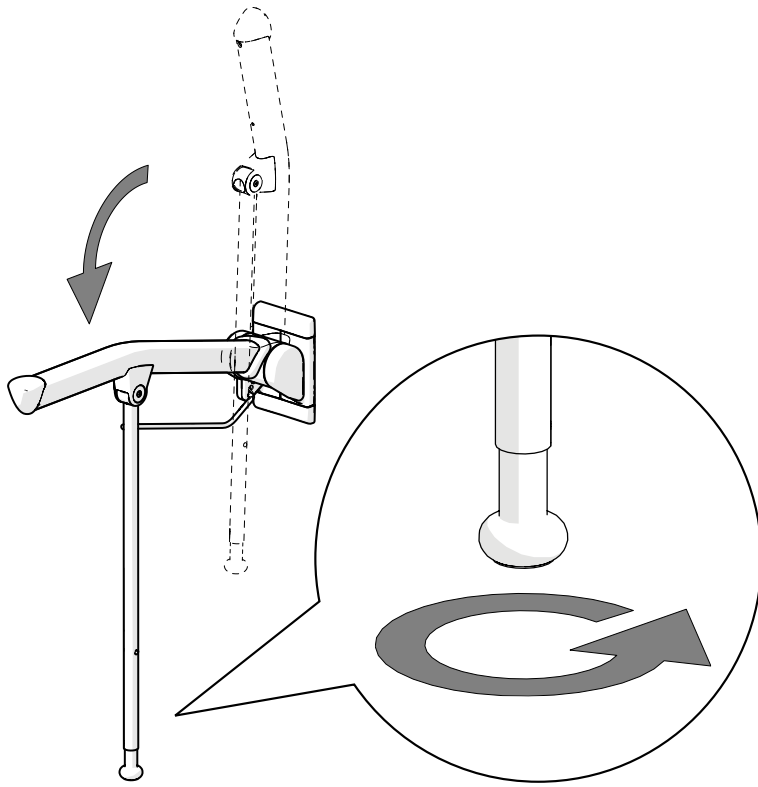C

1

2

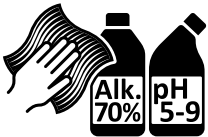
3

D

1



2


3

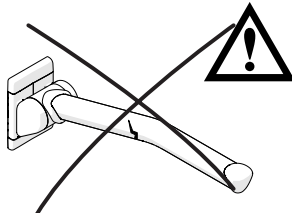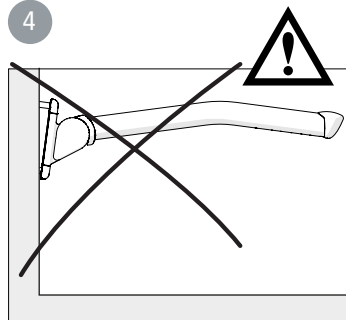
4


5

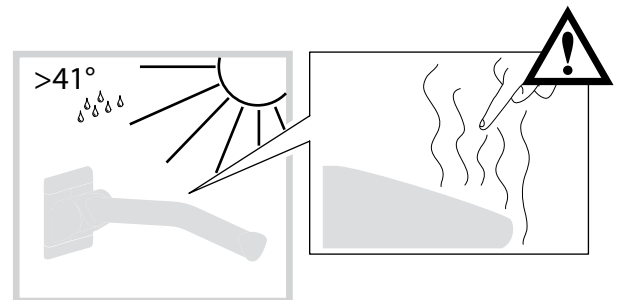E

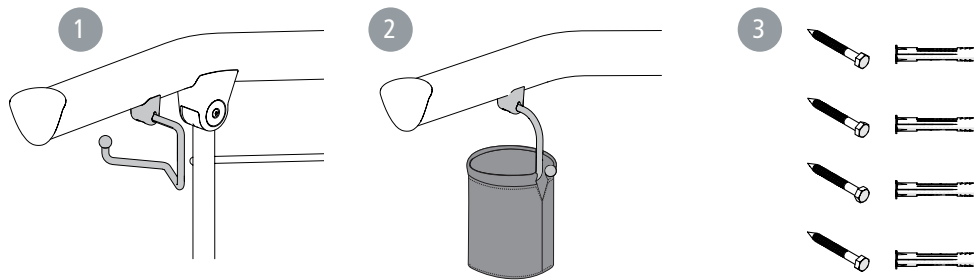F

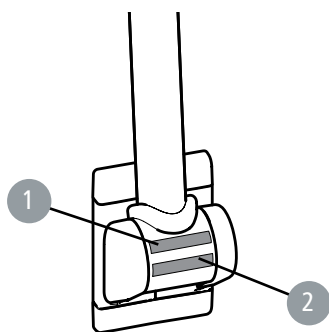G

600/700/850 mm (23.6"/27.6"/33.5")	670/767/911 mm (26.4"/30.2"/35.9")	155 mm (6.1")	155 mm (6.1")	130 mm (5.1")	155x95 mm (6.1"x3.7")	193x130 mm (7.6"x5.1")	800 +70 -30 mm (31.5" +2.76" -1.18")	52x55 mm (2.05"x2.17")

max kg lbs	max kg lbs
*135 kg/297 lbs	150 kg/330 lbs

\*85 cm=100 kg/220 lbs

Etac Rex 60 cm	Etac Rex 70 cm	Etac Rex 85 cm	Etac Rex 70 cm	Etac Rex 85 cm		
2.7 kg 6 lbs	2.9 kg 6.4 lbs	3.3 kg 7.3 lbs	3,6 kg 7.9 lbs	4,0 kg 8.8 lbs	Alu/ABS/PUR/PP/SS	PA6GF/TPE/SS

SV

## Svenska

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**English**

**General, intended use**

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at [www.etac.com](http://www.etac.com). You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

The Etac Rex is a wall-mounted toilet arm support designed as a standing support for toilet use by persons with restricted mobility. The Etac Rex toilet arm support can be folded up when not in use. The Etac Rex toilet arm support with support legs is recommended where an arm support for user weights above 135 kg is required (max 150 kg).

Etac Rex is not designed to be used in any other way than described above.

Etac Rex has been tested and complies with the requirements set out in EN ISO 10993-1, SS EN 12182.

The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

There are no known contraindications.

**Delivery contents** ..... [Figure A](#)

**Installation options**

May be mounted on walls with sufficient load-bearing capacity only. Please ensure that suitable fixtures are used. For mounting in concrete, plywood or wooden joists, we recommend the screw set accessory.

Mounting on walls with insufficient load-bearing capacity or with unsuitable fixtures can affect user safety. Mounting of the product and assessment of the wall's load-bearing capacity must be carried out by a qualified person.

**Installation** ..... [Figure B](#)

**Usage** ..... [Figure C](#)

**Maintenance and safety** ..... [Figure D](#)

1. Clean/disinfect the product with normal non-abrasive cleaning agents with a pH value of between 5-9, or with 70% disinfectant solution. Rinse and dry. The product is otherwise maintenance-free.
2. The product can withstand cleaning in a sterilizer at 85°C for 3 minutes. This applies for

reconditioning.

3. Maximum user weight (see technical data).
4. Check that the bolts are securely tightened. Defective or incorrectly installed products must not be used. In case of damage, contact your dealer.
5. Heated product can cause burns. The materials used are corrosion-resistant. The materials used are resistant to all general disinfectants.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

**Guarantee, service life**

5-year guarantee against material and manufacturing defects. For terms and conditions, see [www.etac.com](http://www.etac.com).

Service life 10 years. For complete information regarding the service life of the product, see [www.etac.com](http://www.etac.com).

The product's date of manufacture can be read from the barcode on the product. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

**Storage**

The product should be stored indoors in a dry place at a temperature above 5°C. If the product has been stored for a long time (more than four months), its function must be checked by an expert before use.

**Accessories** ..... [Figure E](#)

1. Toilet paper holder 2. Storage basket 3. Screw set

**Label placement** ..... [Figure F](#)

1. Product label (warnings, CE-marking)
2. Item no label (serial no, production date)

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Etac Supply Center AB