Instruction for use

IMM1037_IFU.Version6.2018-12-17

HipTurner

immedia
a part of Etac

Index

Illustrations.....	3
User manual – English	4
Bruksanvisning – Svenska.....	9
Bruksanvisning – Norsk	14
Brugervejledning – Dansk.....	19
Käyttöohjeet – Suomi.....	24
Gebrauchsanweisung – Deutsch ...	29
Gebruiksaanwijzing – Nederlands ...	34
Instruction d'utilisation – Français ...	39
Istruzioni per l'uso – Italiano	44
Instrucciones de uso – Español	49

See www.etac.com

Illustrations

ILL.

GB

SV

NO

DA

FI

DE

NL

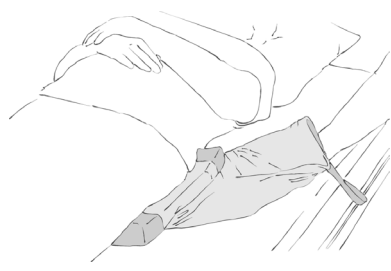
FR

IT

ES


1


2

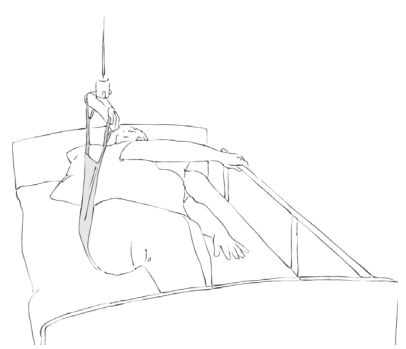3

4

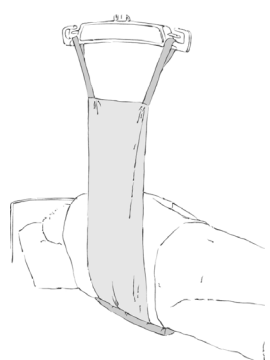5

General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, please read this manual carefully.

The person described as “the user” in this “Instruction for Use” is the person who is lying or sitting on the product. The carers are the people who manoeuvre the product.

This symbol appears alongside the text in the manual. It draws the reader’s attention to points at which there may be a risk to the health and safety of the user or carer.

These products comply with the standards applicable for Class 1 products in the European Council Directive MDD 93/42/EEC on medical devices.

At Etac we strive to continually improve our products and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations

and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

For further information on the Etac transfer range, see www.etac.com.

The product can be scrapped in accordance with national regulations.

Symbols in general

Washing

pH level

Read the user manual

User mass limit = maximum rated load

Do not bleach

Wipe off

Warning

The product can be scrapped according to national regulations

Tumbledry

Never leave on the floor

Production year, month and day
YYY-MM-DD

Manufacturer
Class 1 - European Council Directive MDD 93/42/EEC on medical devices

Iron

Risk of sliding down

Batch no./Lot no.

Do not dryclean

Keep out of the sun

Intended use

HipTurner is used to position a bedpan easily and comfortably under a user. It is especially useful when the user is lying on a pressure-relieving mattress, where positioning a bedpan is more

difficult. This device reduces the strain on carers and minimises the risk of bedpan spills.

Intended environment

Acute care, Long-term care, Home care

Practical handling

Positioning

HipTurner is retained in its pillow, so that the nylon side is facing away from the carer.

Position HipTurner pillow very close to the user's hip (Illustration 1) and feed the nylon piece in under the small of the user's back. (Illustration 2). The carer on the opposite side grasps the

functional handles and pulls the nylon piece taut (Illustration 3) and smooths it out under the user's body.

Attach the functional handles to the hoist and prepare the user for turning (Illustration 4) (If possible, the user should grip the guard rail and bend his/her legs).

Use

When the hoist is raised, the user is turned onto the fixed pillow and is therefore raised a couple of centimetres from the sheet, so that the bedpan can be positioned centrally under the user's rear (Illustration 5).

The hoist is then lowered and the user is now lying on the bedpan. Raise the bed headboard while the user is using the bedpan. Then lower the bed headboard and attach the functional handles to the hoist.

Before lifting, one carer grasps the handle of the bedpan so that it doesn't tip over when the hoist is raised.

The user is turned again on HipTurner's fixed pillow and the bedpan can now be removed.

While the user is still in the side position, lower body hygiene can be performed.

Then lower the hoist and feed the nylon piece under the small of the user's back and remove.

Notice

Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling. Etac offers advice and training for carers. For further information, contact Etac.

Always use the correct manual handling techniques

Encourage the user to assist where possible.

To ensure that the user feels safe and that every manual handling is smooth, always plan the manual handling in advance.

There is a risk that the user may slide off. Never leave the user alone on the edge of the bed.

Never leave the product on the floor.

Always conduct a risk assessment, and ensure that the assistive product can be used for the individual user and, in combination with other devices, that it is safe for the user and carers.

It is recommended that the guard rail is in place when the user is left on the glide system, unless a risk assessment has found that the user can be safely left on the glide system without the need of guard rails.

Always remove the glide system when you have completed the job.

Do not use the product as a sling.

If in any doubt - please contact Etac.

Material and cleaning

Material:

HipTurner:

Glide piece: Nylon

Handles: Polyester

Cover on raised part: Non-slip

Foam: Polyethylene

Zipper: Polyester.

The product should be checked regularly, preferably each time it is used, and especially after washing.

Check that there is no damage to seams or fabric.

Never try to repair a product yourself.
Never use a defective product.

Washing instructions:

Do not use fabric softener – as this will reduce the glide effect. This product can be cleaned with a solvent-free cleaning agent with a pH level between 5 and 9, or a 70% disinfection solution.

Products and accessories

Article no.	Product Description	Size (mm)
IM462	HipTurner (pad W250xL800 mm)	W400xL200xH(45)50

ILL.

GB

SV

NO

DA

FI

DE

NL

FR

IT

ES

Service information

Inspection

Thank you for choosing an Etac product. If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly. Before taking a product into service always conduct a risk assessment according to national and/or local legislation.

First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description.
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from www.etac.com or contact Etac Customer Service or your local distributor.

- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

Periodic Inspection:

- Make sure that your Manual transfer system is always in perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.

Visual inspection/Check the product:

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.

Stability test:

Try to bend the product and verify that it feels firm, stable and solid.

Mechanical load or stress/Test handles:

Pull hard on the handles in opposite directions and check the material, seams and stitching.

Test buckles:

Lock the buckle and pull in opposite directions, check the buckle and the stitching.

Test low/high friction:

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.

Low friction – slides effortlessly

High friction – no sliding/movement

Check

Manual transfer aids with handles, straps and buckles:

Manual transfer aids for sitting and lying transfer:

Manual transfer aids with high or low friction:

The product can be scrapped in accordance with national regulations

Read the User Manual

If in doubt, please contact Etac Customer Service or local distributor for more information and guidance.

For more information about training and the use of these products, contact the Etac transfer experts – www.etac.com.

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Etac AB
Etac Immedia A/S