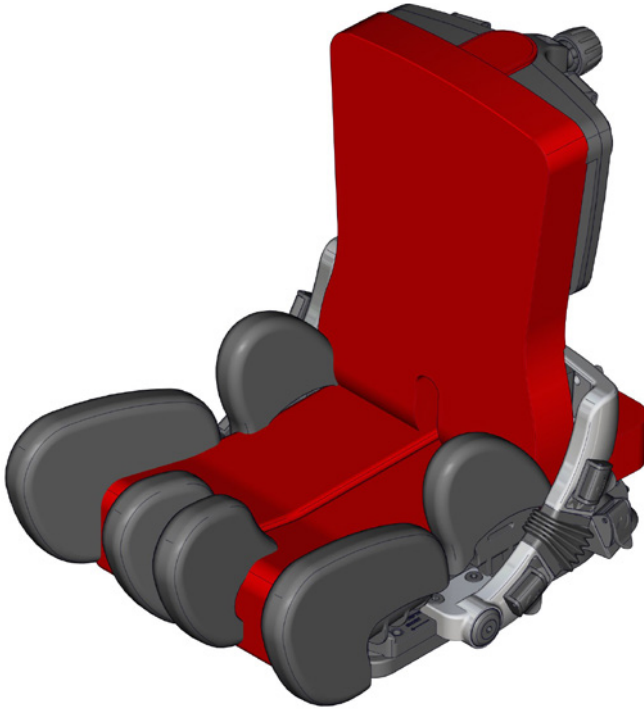
460 mm (18")

max 80 kg (176.3 lb)

x:panda User Manual

EN 12182


EN 12183

2023-01-rev.9.0


	4	ENGLISH DEUTSCH	26 - 29 30 - 35
	5	NEDERLANDS DANSK	36 - 41 42 - 45
	6	NORSK SVENSKA	46 - 49 50 - 54
	7-8	SUOMI ESPAÑOL	55 - 58 59 - 64
	9 - 12	FRANÇAIS ITALIANO	65 - 69 70 - 74
	13	PORTUGUÊS PORTUGUÊS DO BRASIL	75 - 79 80 - 84
	14	РОССИЯ POLSKI	85 - 90 91 - 96
	15 - 16	ČESKY 中國	97 - 101 102 - 105
	17	日本 ΕΛΛΗΝΙΚΗ	106 - 111 112 - 117
	18	ÍSLENSKA	118 - 121
	18		
	19 - 22		
mm (inch) kg (lb)	23		
	24 - 25		

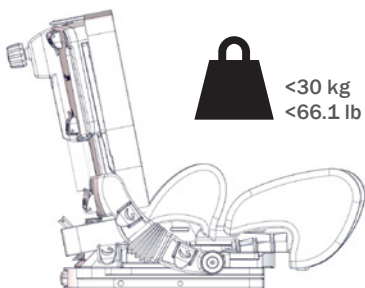
x:panda size 1


x:panda size 2

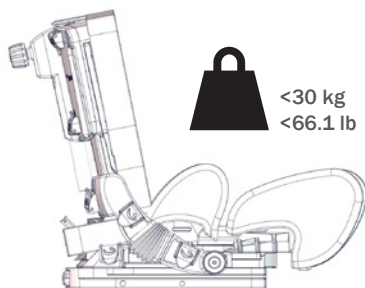x:panda size 3

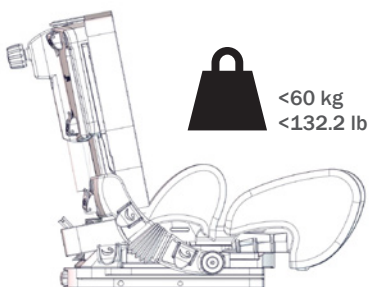
x:panda size 4 ⚠ 03


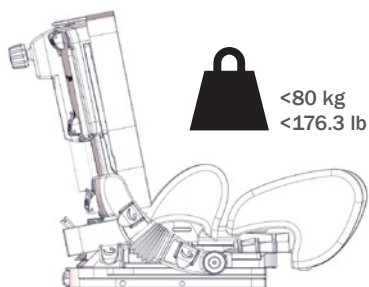
A.

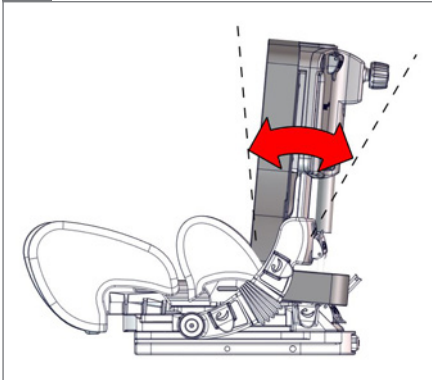
B.

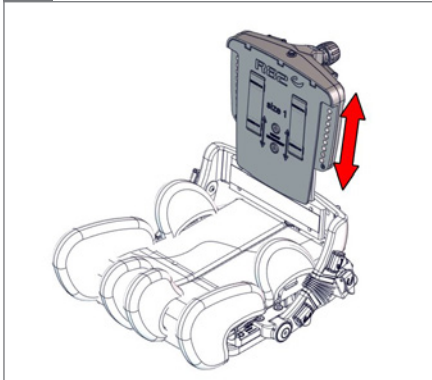
C.

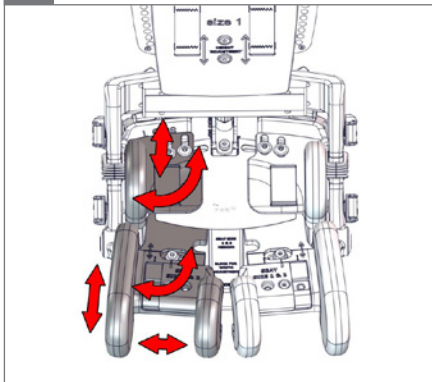D.

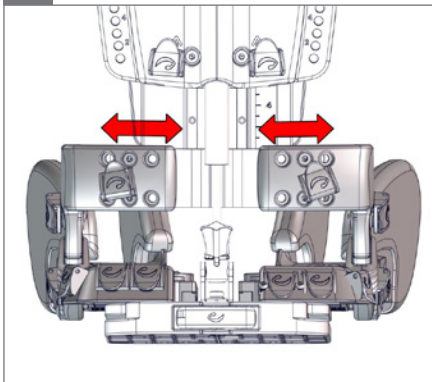
E.


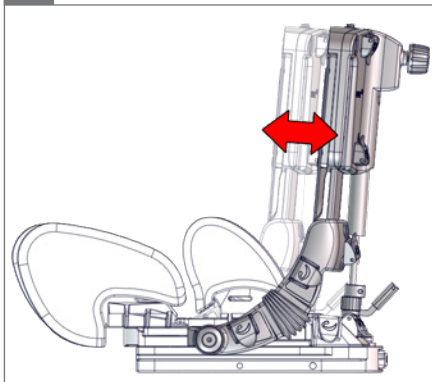
A.

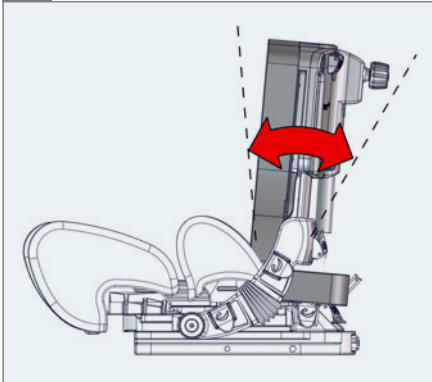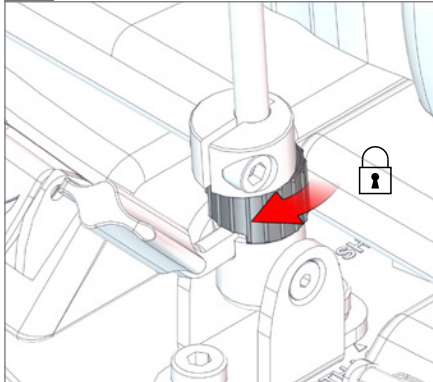1.

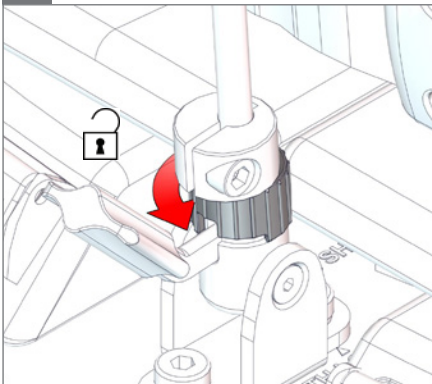
2.


3.

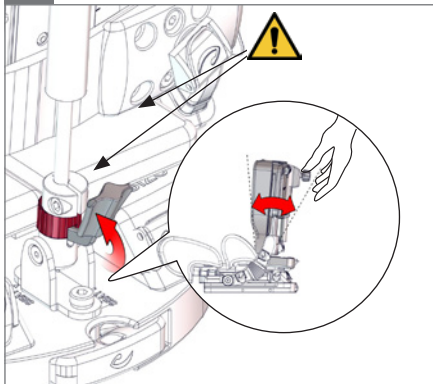! 05

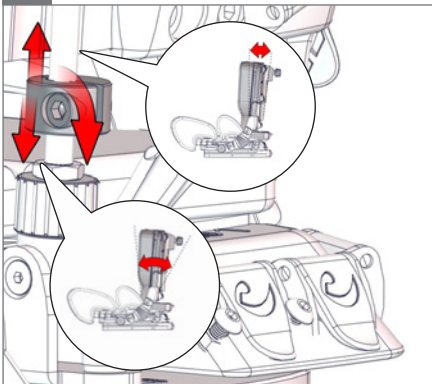5.


W

! 05

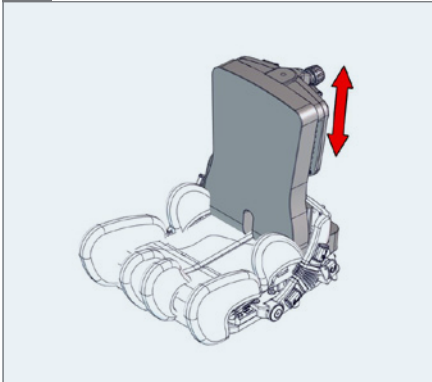
B.


1.

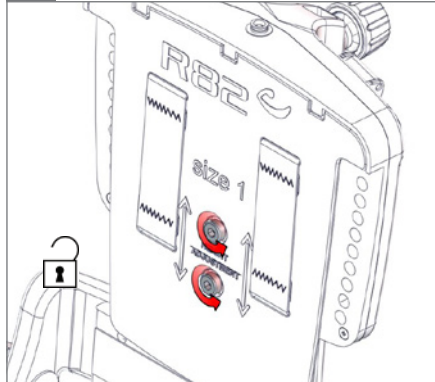○ 5 mm


2.

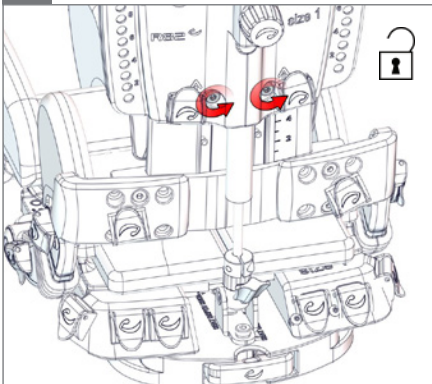○ 5 mm

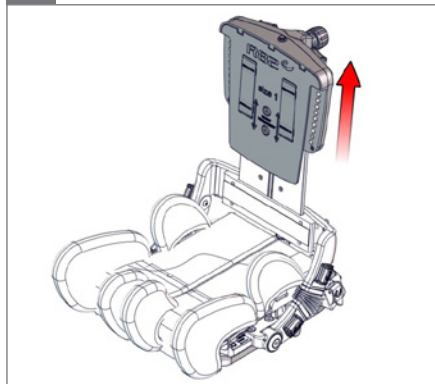
3.


4.

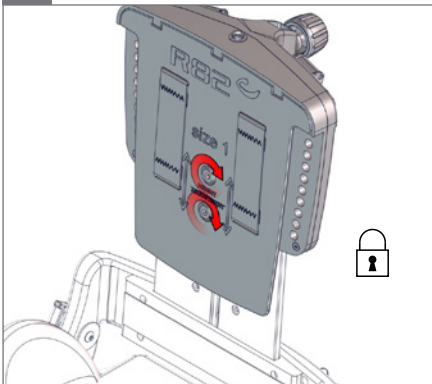○ 5 mm


5.

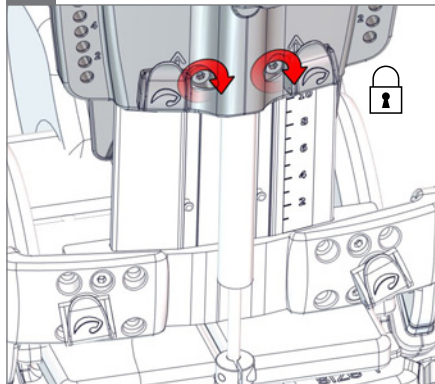○ 5 mm


c.

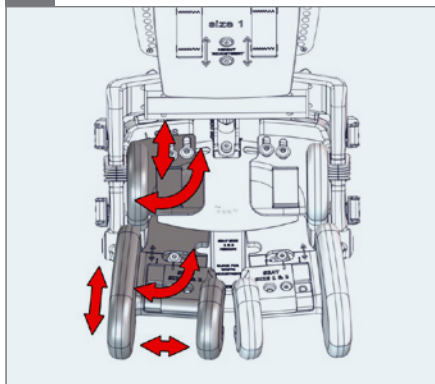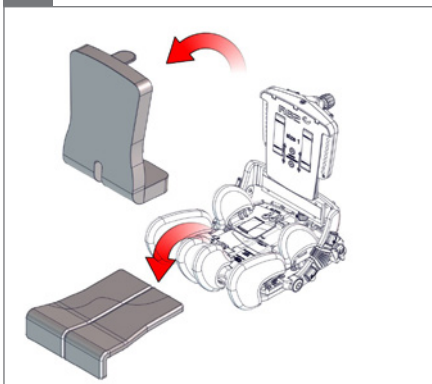
1.


2.

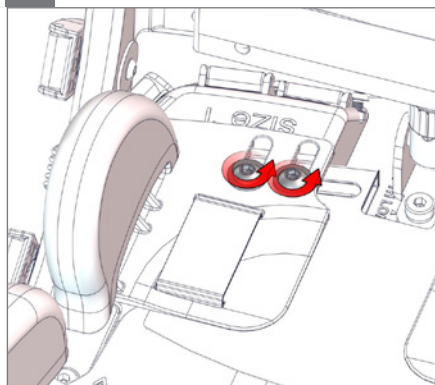○ 5 mm

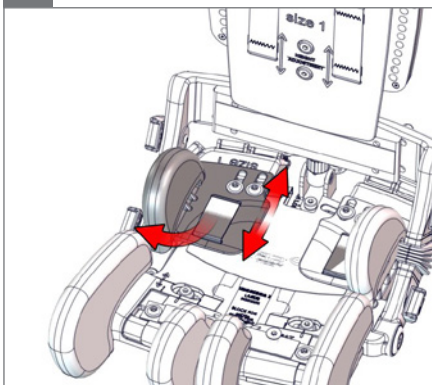
3.


4.

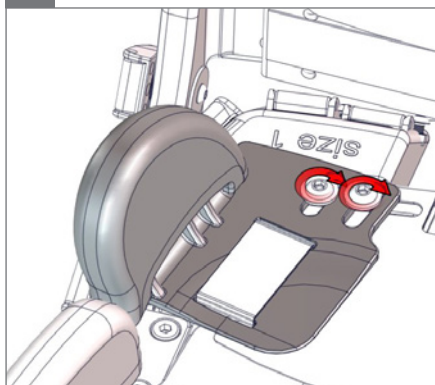○ 5 mm


5.

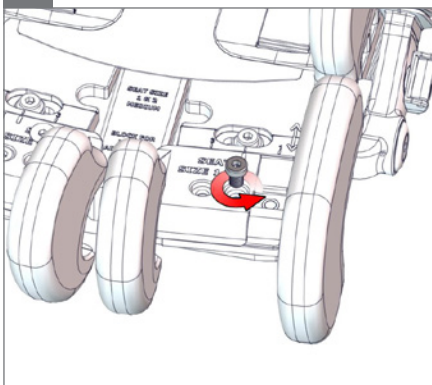○ 5 mm

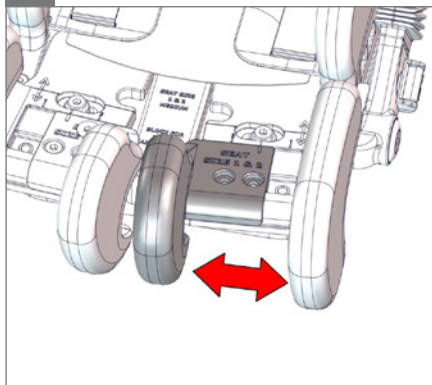
6.


7.

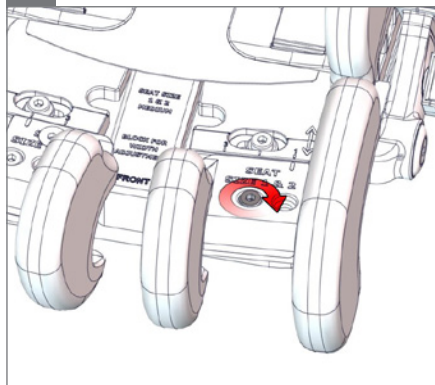○ 5 mm


8.

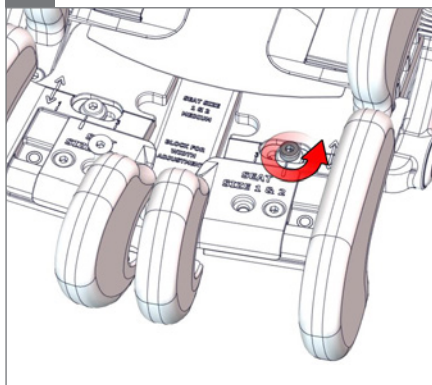
○ 5 mm

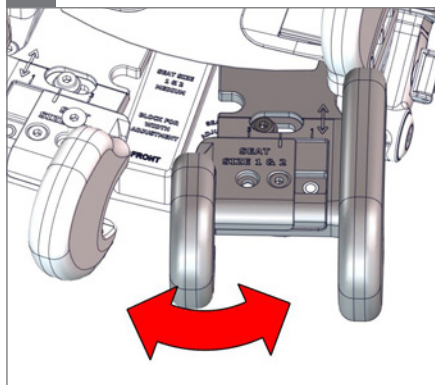9.

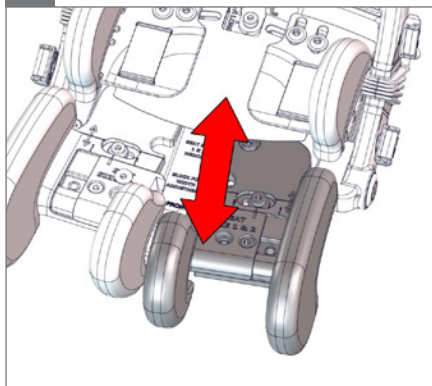
10.


11.

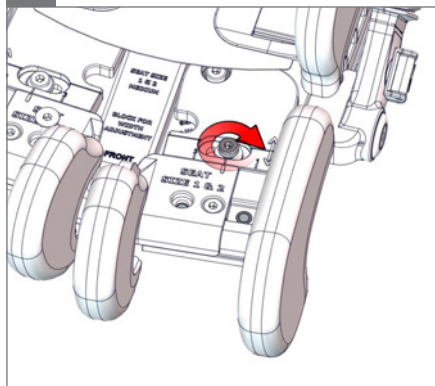○ 5 mm



D.

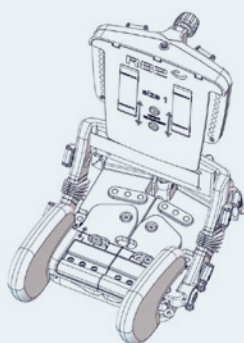Small

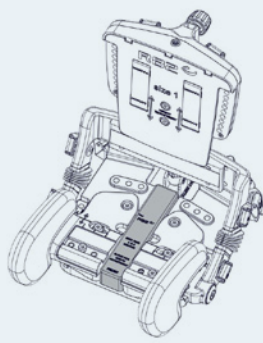
Medium

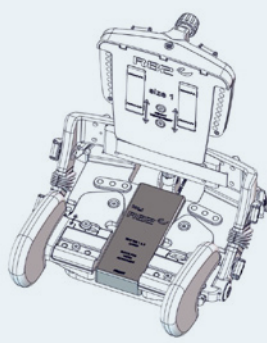Large


1.

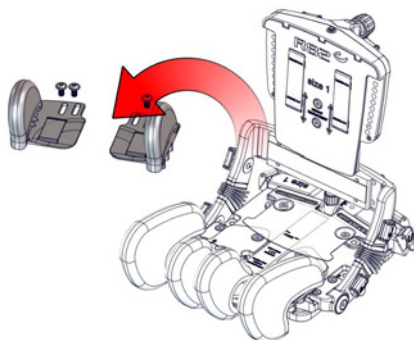○ 5 mm

2.

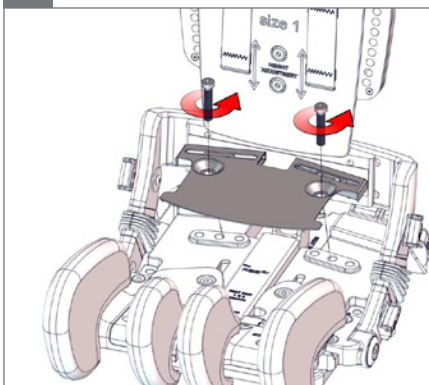
○ 5 mm


3.

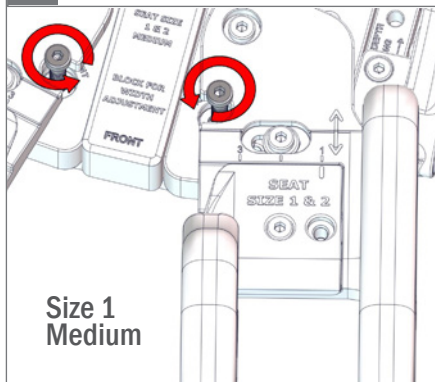○ 5 mm

4.

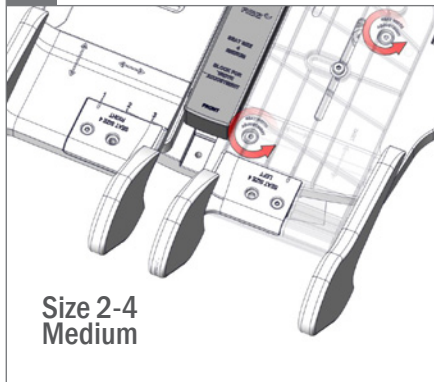
○ 5 mm

5.

○ 5 mm

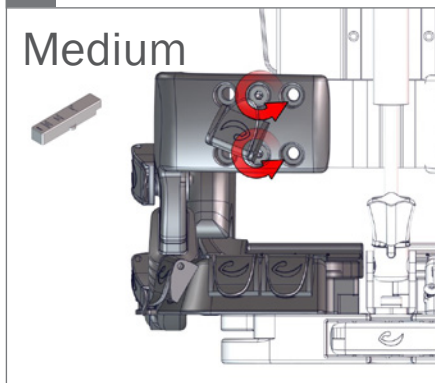
Medium


6.

○ 5 mm

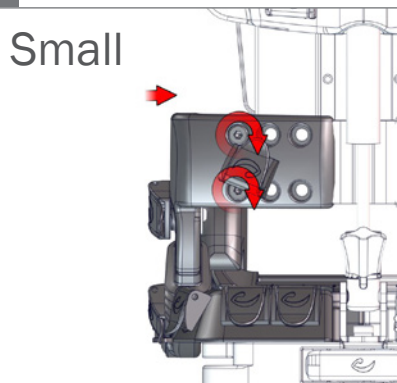Small

Large

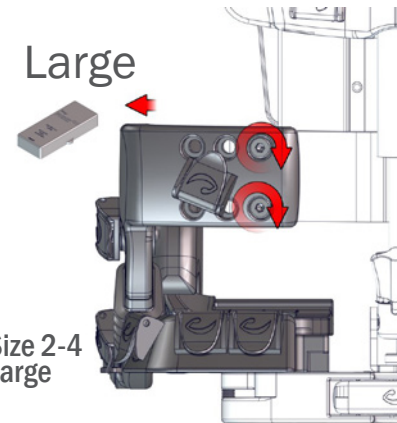
7.


⊕ 5 mm

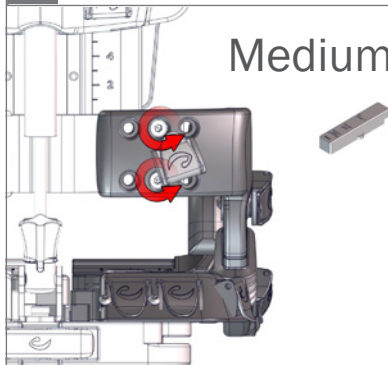Medium

8.

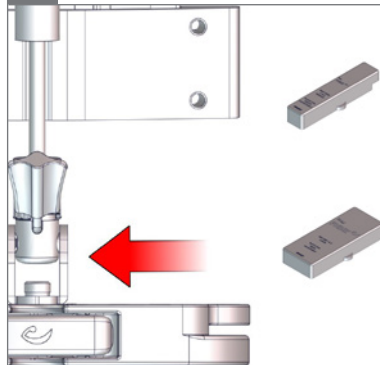
⊕ 5 mm


9.

⊕ 5 mm

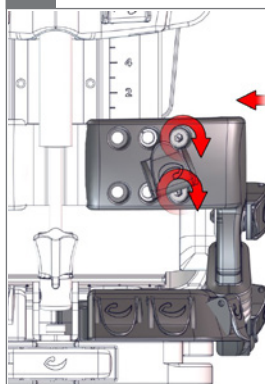Small


Large

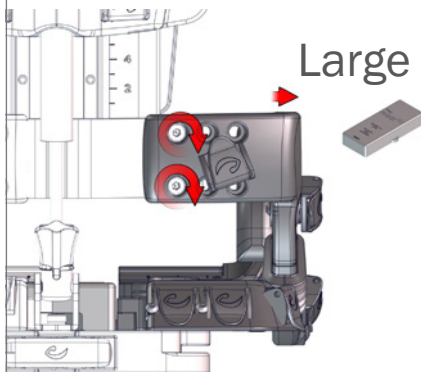10.

⊕ 5 mm

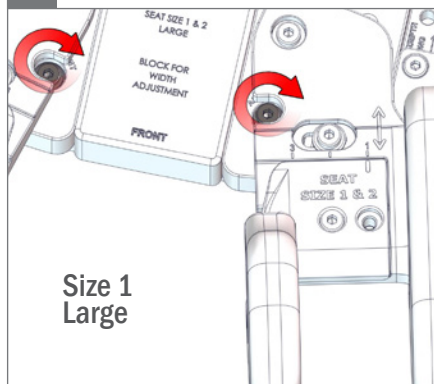Size 1
Large

11.

⊕ 5 mm


Size 2-4
Large


12.

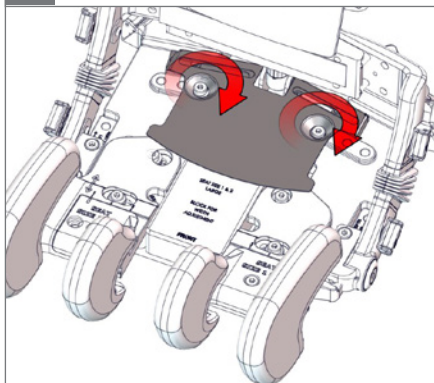⌀ 5 mm


13.

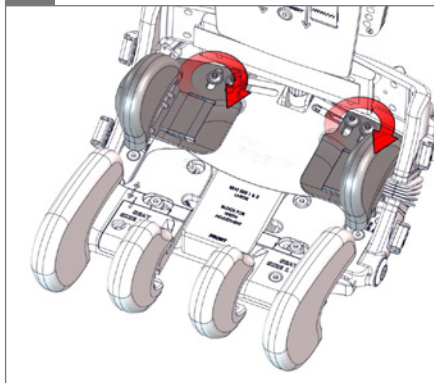⌀ 5 mm

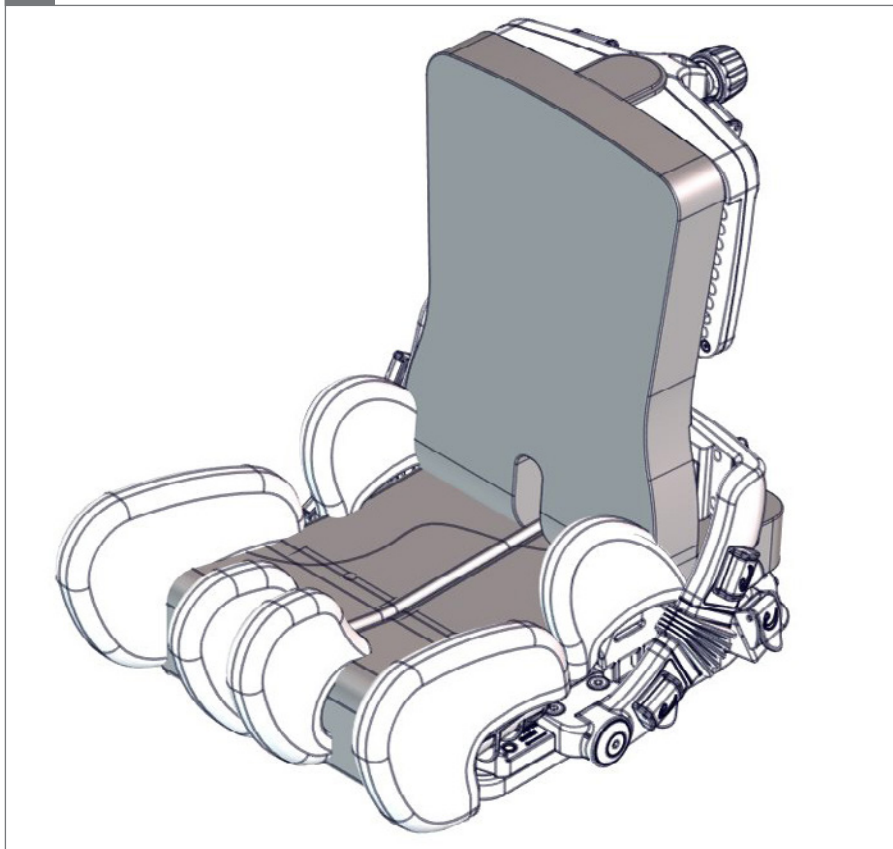
14.


E.

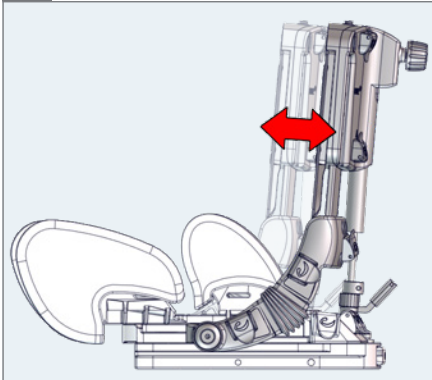
1.

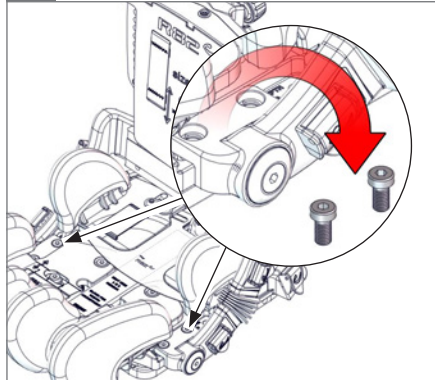⌀ 5 mm


2.

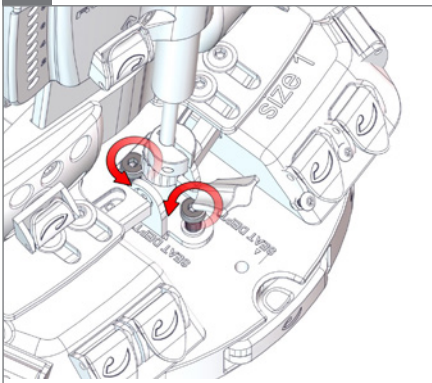⌀ 5 mm

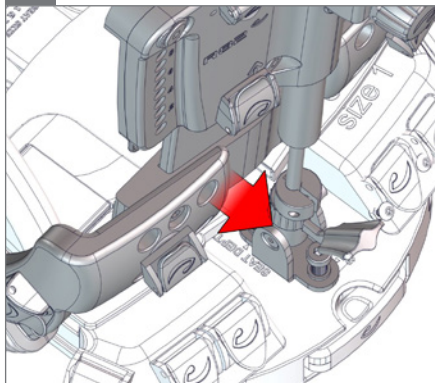
3.


4.

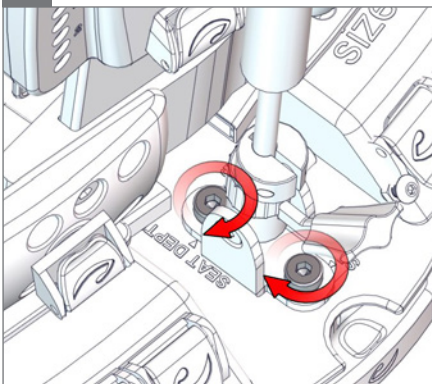⌀ 5 mm


5.

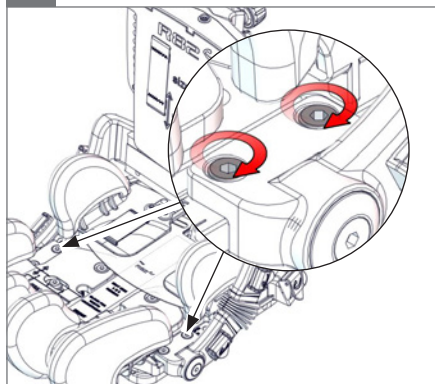
⌀ 5 mm

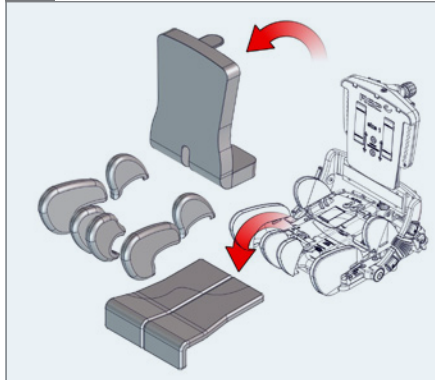
1.

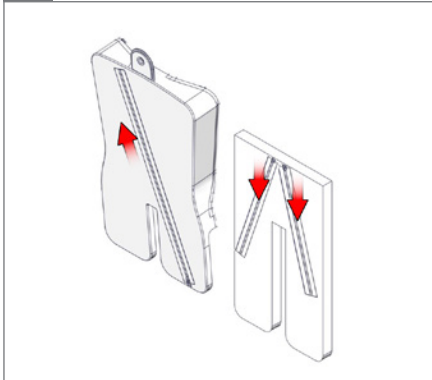2.

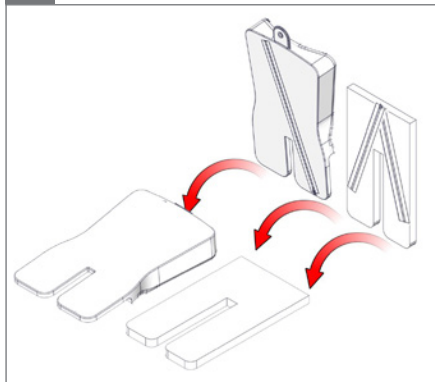
3.


4.

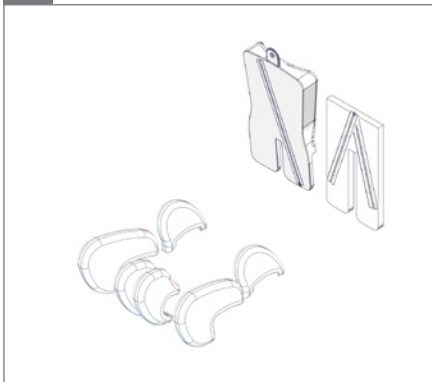60

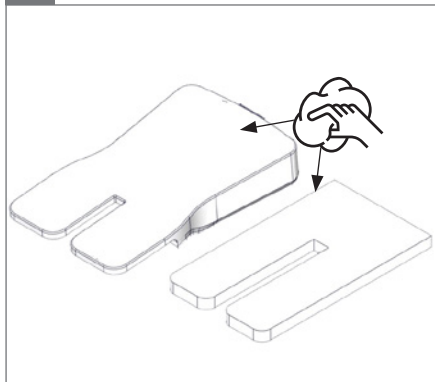
5.


F.

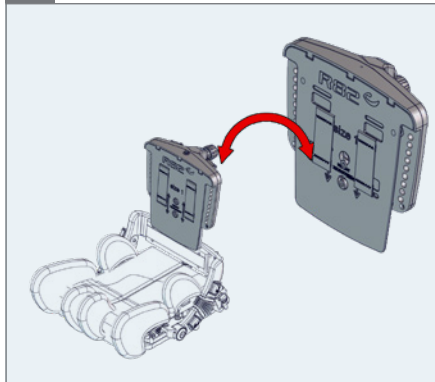Seat/Back	Size 1	Size 2	Size 3	Size 4
Size 1	√	√	-	-
Size 2	√	√	√	-
Size 3	-	√	√	√
Size 4	-	-	√	√


1.

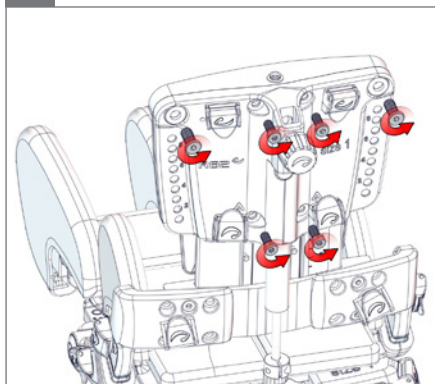○ 5 mm

2.

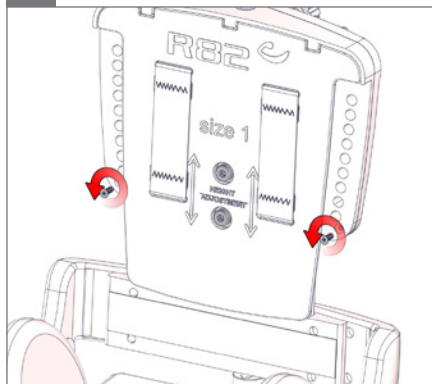
⊕


3.

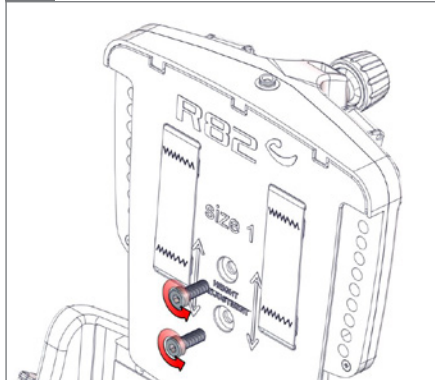
⌀ 5 mm

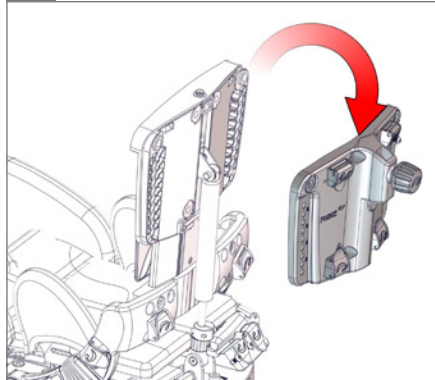
4.



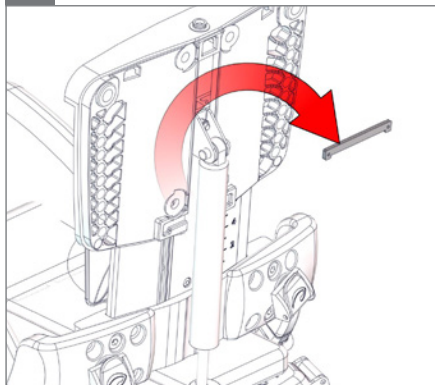
5.

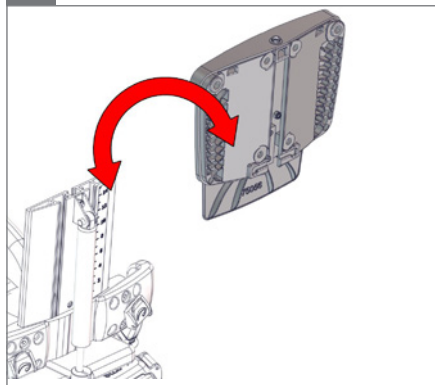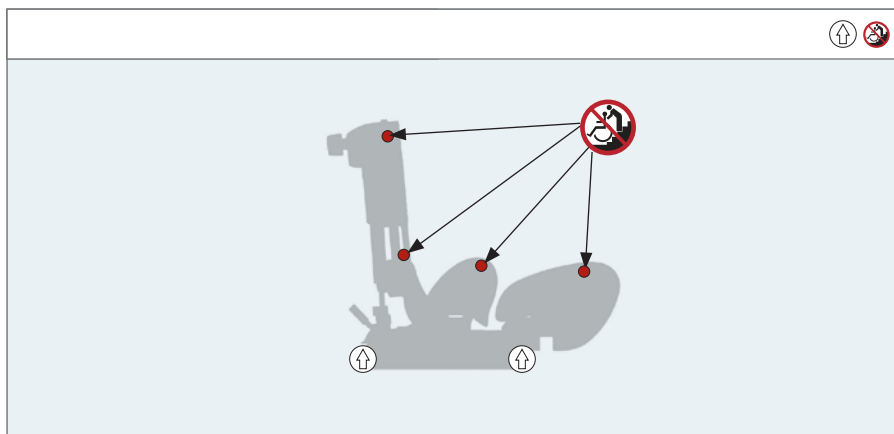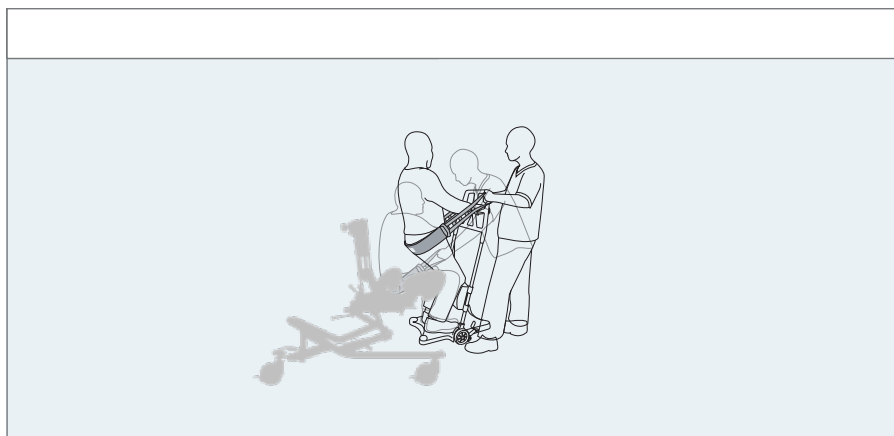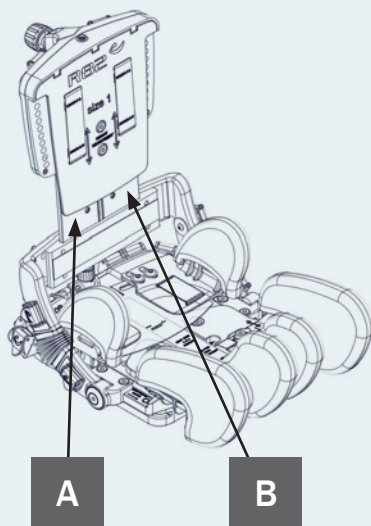6.

A

R82[®]
by Etac
R82 A/S Parallelvej 3
DK-8751 Gedved
etac.com
Made in XXXXXXXX YYYY-MM-DD

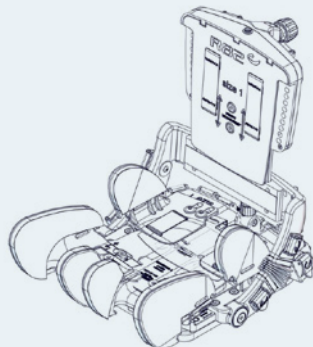
B

Product: xxxx - Size x
Art. no.: xxxxxxxx Max user mass: xxx kg
XXXXXXX Max user mass: xxx kg
DMR ver.: XXXX Mass device: xxx kg
Max rated load: xxx kg

(xx)xxxxxxxxxxxxxxxxxx(pj)xxxxxxxx(pj)xxxxxxxx

EN: For indoor and outdoor use

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mm
(inch)
kg
(lb)

mm (inch) kg (lb)

	1	2	3	4
 Effective seat width	160, 200, 240 mm (6¼", 7¾", 9½")	200, 240, 280 mm (7¾", 9½", 11")	260, 310, 360 mm (10¼", 12¼", 14¼")	340, 400, 460 mm (13½", 15¾", 18")
 Width, thighs	70, 90, 110 mm (2¾", 3½", 4¼")	100, 120, 140 mm (4", 4¾", 5½")	120, 145, 170 mm (4¾", 5¾", 6¾")	130, 190, 250 mm (5", 7½", 9¾")
 Effective seat depth	160–260 mm (6¼"–10¼")	220–330 mm (8½"–13")	260–390 mm (10¼"–15¼")	390–530 mm (15¼"–20¾")
 Backrest height	270–370 mm (10½"–14½")	320–420 mm (12½"–16½")	350–470 mm (13¾"–18½")	500–710 mm (19¾"–28")
 Back width, top of backrest	230 mm (9")	280 mm (11")	330 mm (13")	350+50 mm (13¾"+2")
 Overall width, both legs in max abduction	310, 350, 390 mm (12¼", 13¾", 15¼")	380, 420, 460 mm (15", 16½", 18")	470, 520, 570 mm (18½", 20½", 22½")	590, 650, 710 mm (23¼", 25½", 28")
 Armrest to seat distance	100–165 mm (4"–6½")	100–165 mm (4"–6½")	100–165 mm (4"–6½")	100–165 mm (4"–6½")
 Front location of armrest structure	140 mm (5½")	140 mm (5½")	180 mm (7")	350 mm (13")
 Backrest angle	(-5°)–(30°)	(-5°)–(30°)	(-5°)–(30°)	(-5°)–(30°)
 Abduction angle, each leg (a1) Size 1 in the smallest position	0°–15° 0°–8°	0°–15°	0°–15°	0°–15°
 Maximum user mass	30 kg (66.1 lb)	30 kg (66.1 lb)	60 kg (132.2 lb)	80 kg (176.3 lb)
 Maximum user mass at transportation	30 kg (66.1 lb)	30 kg (66.1 lb)	60 kg (132.2 lb)	75 kg (165.3 lb)
 Mass device*	6.5 kg (14.3 lb)	8.5 kg (18.7 lb)	11 kg (24.3 lb)	13 kg (28.5 lb)
 Max rated load	38.5 kg (84.7 lb)	40.5 kg (89.2 lb)	73 kg (160.9 lb)	95 kg (209.4 lb)
Minimum User height	650 mm (25½")	850 mm (33½")	1000 mm (39½")	1390 mm (54¾")
Maximum User height	1050 mm (41¾")	1280 mm (50½")	1430 mm (56¼")	1800 mm (70¾")

*seat only (without accessories)

9996097751

9996097752

9996097701

9996097711

9996097702

9996097712

9996097750

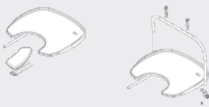
9996097714

9996097705


9996097717

9996097706

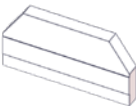9996097719

9996097707

9996097720

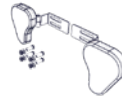
9996097708

9996097721

9996097722

9996097741

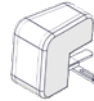
9996097724

9996097901


9996097740

9996097733

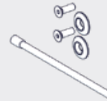9996097726

9996097760

9996097700

9996097675

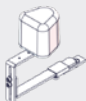
9996097728

9996098102

9996097739

9996097727

ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The device is a modular seating system for children and adults with disabilities.

Indications

The device can be used by children and adults with disabilities, who lack the ability to maintain a seated position. The target group for the device is based on functional ability and not a specific health condition or age.

The user is the person sitting in the device.

The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

The use of the device will be discontinued more times during the day where the user will be lying down, standing or sitting in another assistive device or a conventional product.

For transportation the device is suitable for children and adults in the weight range of 22 kg to specified max user weight for transportation.

Contra indications

There are no known contra indications

Precautions

Special attention should be given to the considerations about the configuration of the product and the use of accessories for users with limitation in their:

- Sensory functions and the sensation of pain consider extra checks of the pressure caused by the support provided by the device and its accessories.
- Movement function (e.g. experience involuntary movements), consider using a device with a dynamic backrest.
- Ability to maintain a seated position, use the product with accessories (e.g. side supports, vests, belts or arm supports and a tray).
- Ability to maintain the position of their head, consider to use accessories or to use a product that provide support to the positioning of the head.

Application areas

The product is for indoor and outdoor use

Enclosed tools:

- 5 mm Allen key

CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12182
- EN 12183

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

For information about product safety notices and product recalls, local dealer information are available at the website etac.com

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user.
- This product contains small parts which could present a choking hazard if removed from their intended placing.
- Always use the correct lifting techniques and aids for this purpose.
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver.
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier.
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible.
- Keep a hand on the handle on the back of the chair when adjusting the back angle.
- Information about transportation can be found in the document "M1460 Transportation in Motor Vehicles" supplied with the product.
- Information regarding frame and seat combinations including max. load in combination can be found at: etac.com.
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user guide.
- Ensure that moving parts does not squeeze or trap body parts when adjusting them.
- Ensure that body parts are not trapped in openings for mounting of supports/ accessories.

Environment

- Observe the surface temperature on the product, before placing a user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use.

User

- If the user is close to the max. load

limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered.

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children.
- Ensure the stability of the product is maintained before placing the user in it
- Use the fixlocks mounted on the product to fasten and adjust all vests and belts. Note the prepared slots in the hip and leg supports intended for belt guidance.
- Position the back vertical and tighten the bushing up against the gaspring before transportation.
- Inspect the product and all of its accessories and replace any worn out parts before use.
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring.
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1, EN 1021-2 and ISO 7176-16.
- The product might lose its flame resistant characteristics if using other than original R82 cushions.
- The device conforms to ISO 7176-8, covering requirements and test methods for static, impact and fatigue strengths.
- Unoccupied device is suitable for land and air transport.
- Removable parts without use of tools: head support and arm rest. The removal will have adverse effect on the device.
- The product is approved for transportation in vehicles according to ISO 7176-19. More information can be found in the document "M1460 Transportation in Motor Vehicles" supplied with the product.
- In accordance with ISO 7176-19 Annex D, the product is evaluated regarding the ability to accommodate vehicle-anchored pelvic and shoulder belts. The device receives an overall score of "Good".

- Note that size 4 has another max user mass in transportation which is lower than the max user mass.

Accessories

Head Supports

- When adjusting the head support, be careful not to get fingers caught between back of the chair and the head support fitting.

Arm rest

- Be careful not to get fingers caught between the chair and the armpad.
- Do not carry the product by the armrests.

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded. The device may be reissued within its lifetime.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Adjustment options for back, hip and legs
- Gas-spring for back recline and the security lock
- Seat back is adjustable and can be locked in position
- Nuts and bolts on the product are tightened
- Check all fittings and buckles for signs of worn out parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to: Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Fabric and covers

The cover can be washed at max. 60°C using a mild detergent in a washing machine.

Separate foam and cover before wash.

Foam

The foam inserts must only be washed by hand. Use warm water and a mild detergent/soap without chlorine and let all parts dry before use.

During and after washing the foam, do not bend, squeeze or wring it, as it may negatively affect the shape.

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Aluminium
- Rubber
- Plastic
- Steel
- Fabric
- Foam

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized.

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© 2013 R82 A/S All rights reserved The R82 logo and the icons are registered trademarks of R82 A/S