690 mm (27¼")

<100 kg (220.4 lb)


Multi Frame User Manual


EN 12183

2023-02 rev.9.0

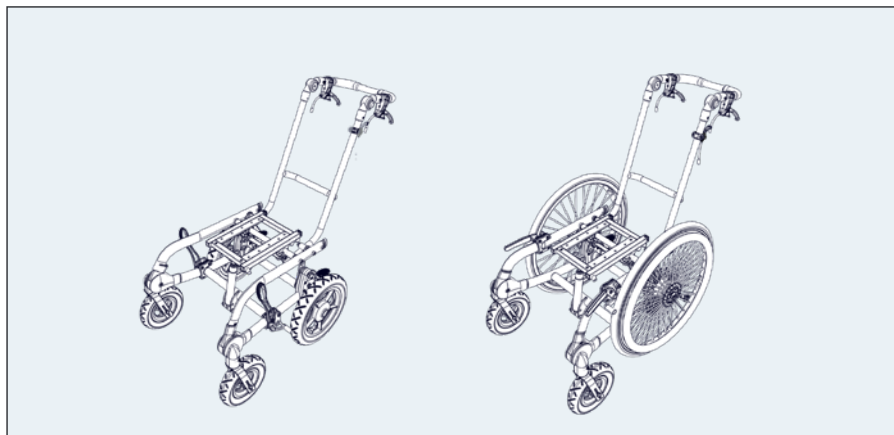	3	ENGLISH DEUTSCH	36-39 40-43
	4 - 5	NEDERLANDS DANSK	44-47 48-51
	6	NORSK SVENSKA	52-55 56-59
	7 - 10	SUOMI ESPAÑOL	60-63 64-67
	11	FRANÇAIS ITALIANO	68-71 72-75
	12 - 13	PORTUGUÊS PORTUGUÊS DO BRASIL	76-79 80-83
	14 - 17	РОССИЯ POLSKI	84-87 88-91
	18	ČESKY 中國	92-95 96-99
	19 - 22	日本 ΕΛΛΗΝΙΚΗ	100-103 104-107
	23 - 24		
	25		
	26		
	27		
	28		
	28		
	29 - 32		
mm (inch) kg (lb)	33 - 34		
	35		


SZ 1

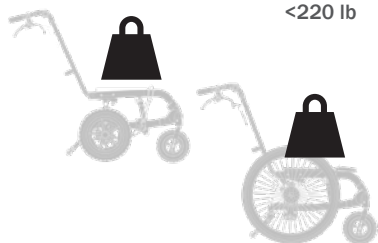
<100 kg
<220 lb

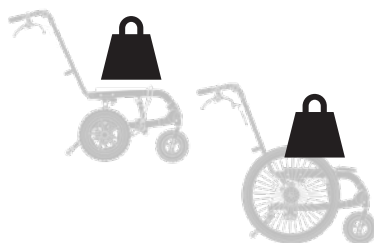SZ 2


<100 kg
<220 lb

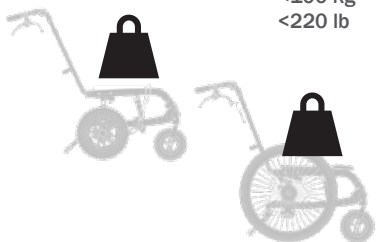
SZ 3


<100 kg
<220 lb

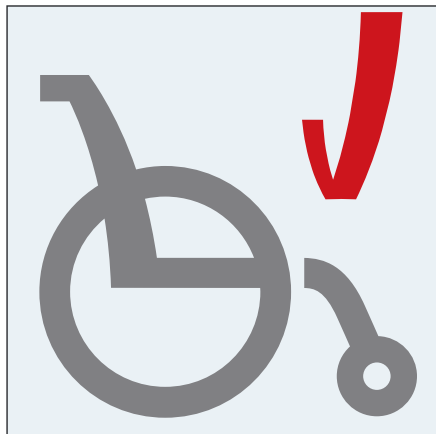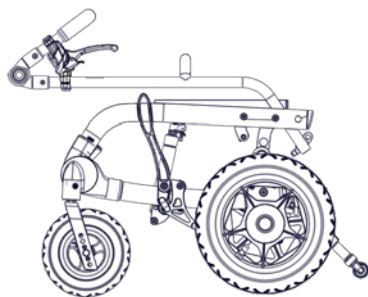
1.

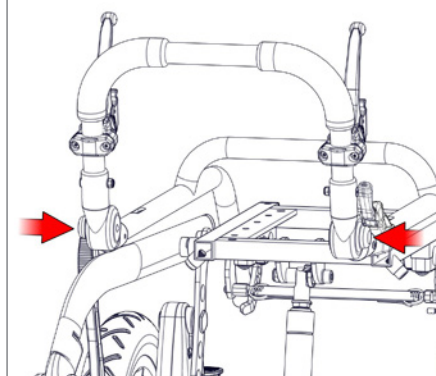
2.

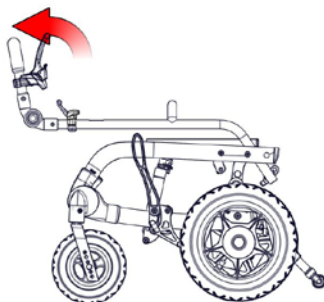
3.

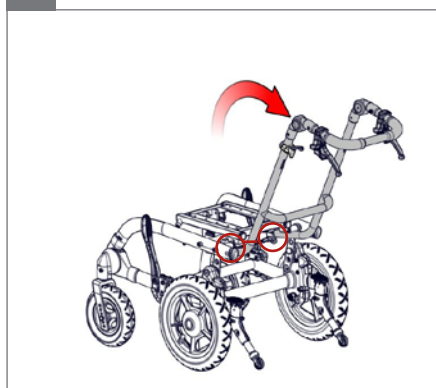4.

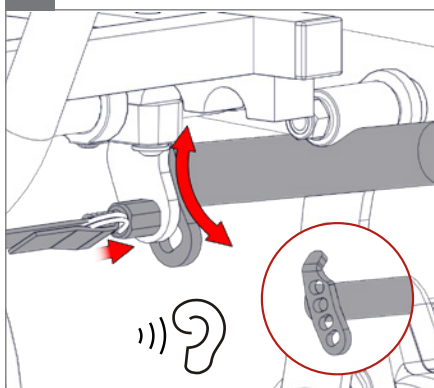
5.


5.

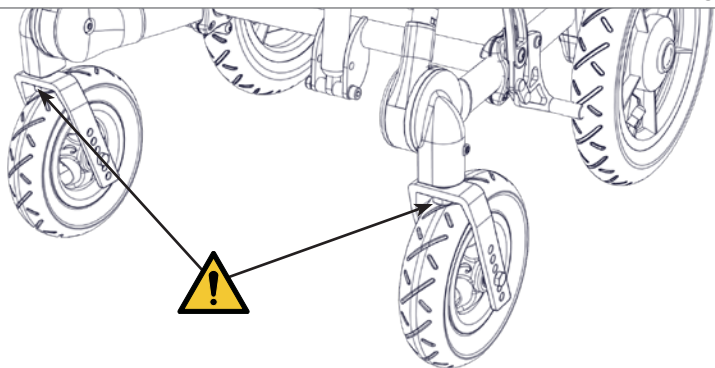
 05


6.

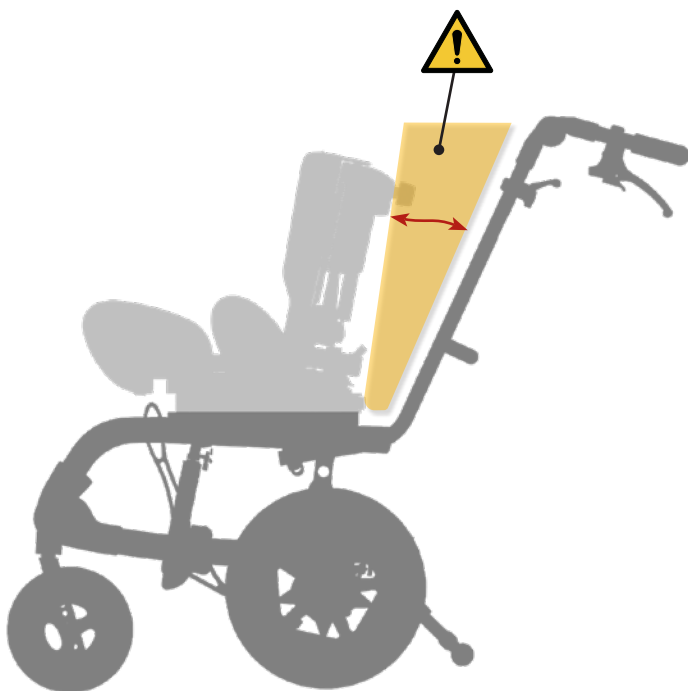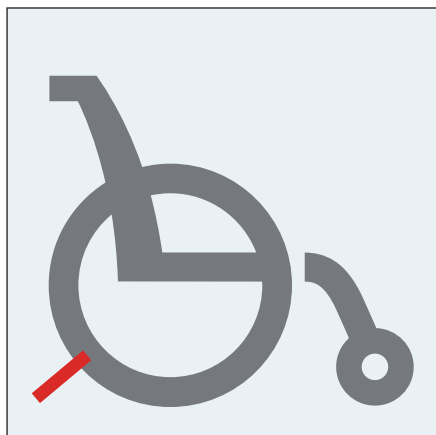 05

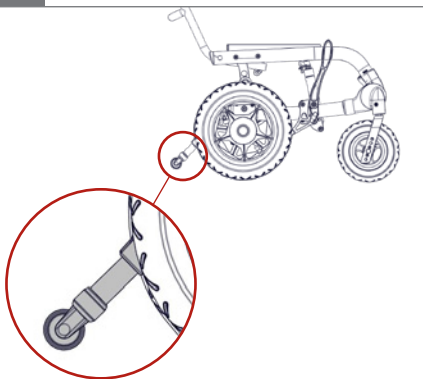
1.


2.

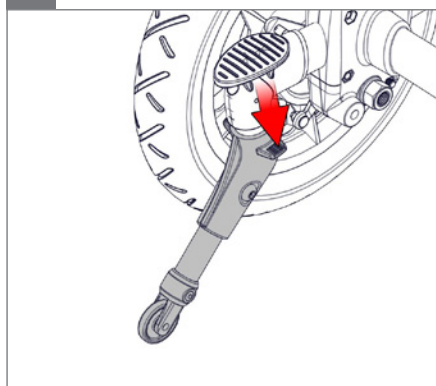
○ 3 mm

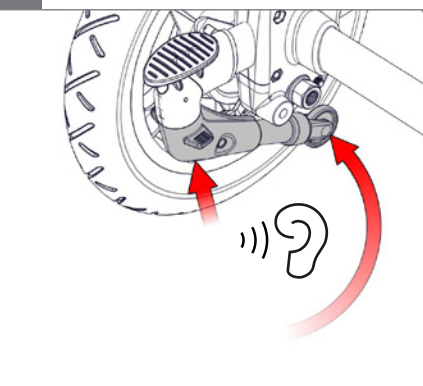
3.








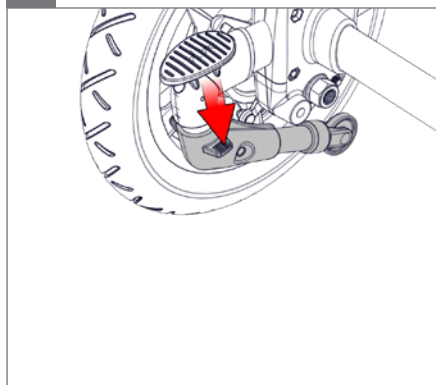
4.

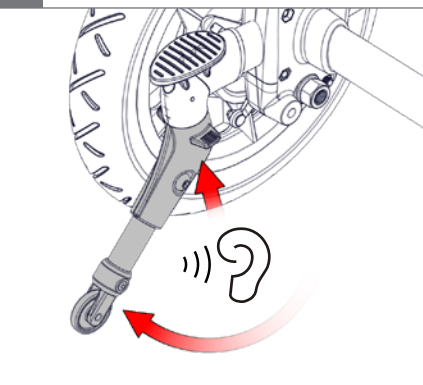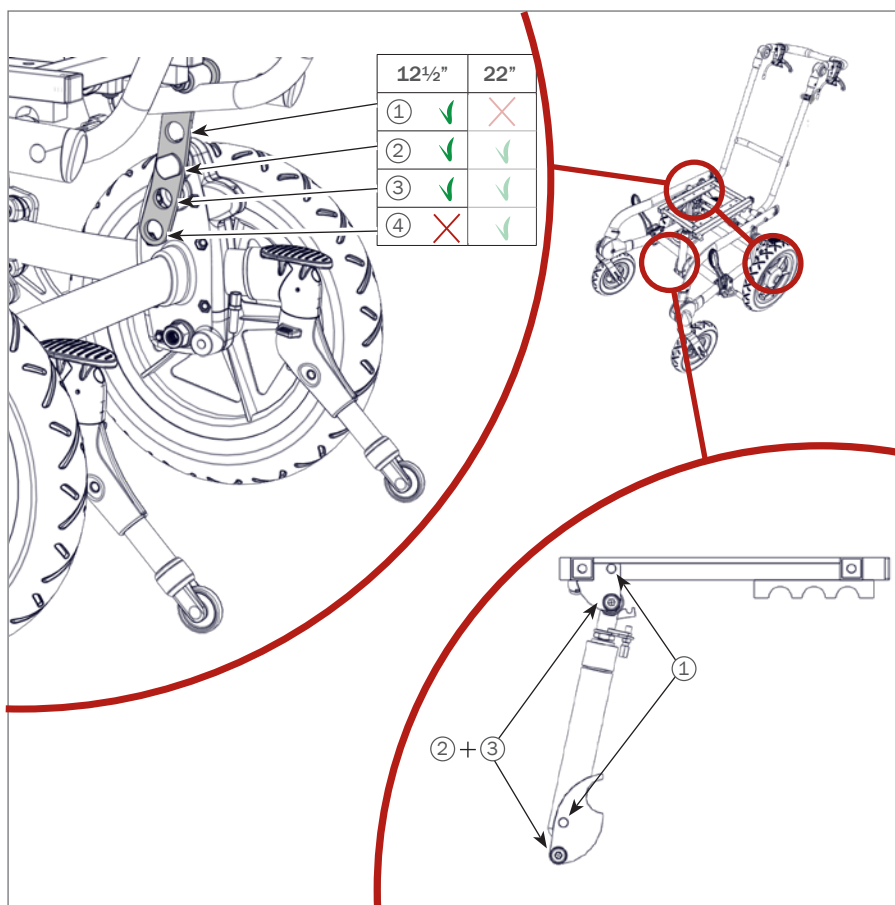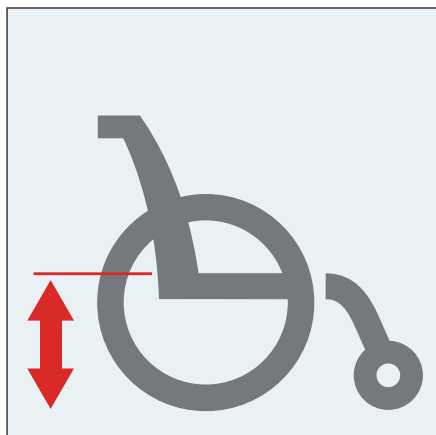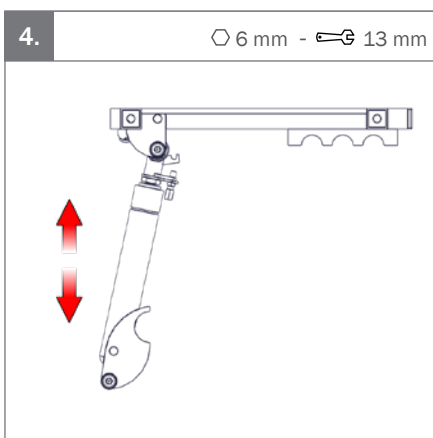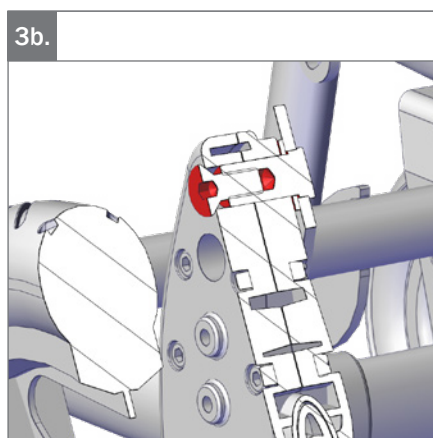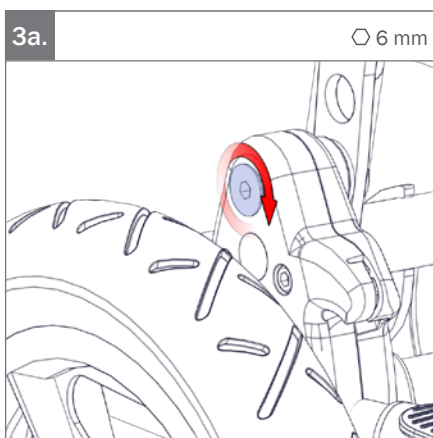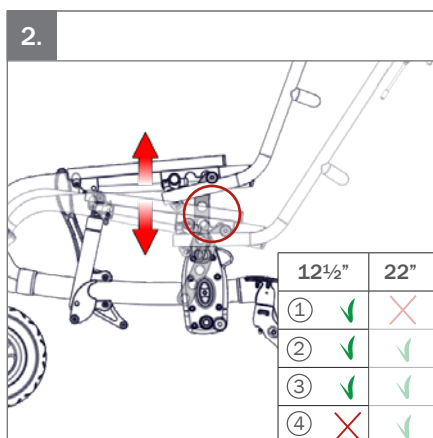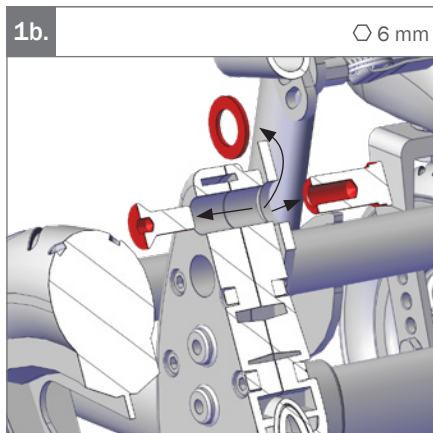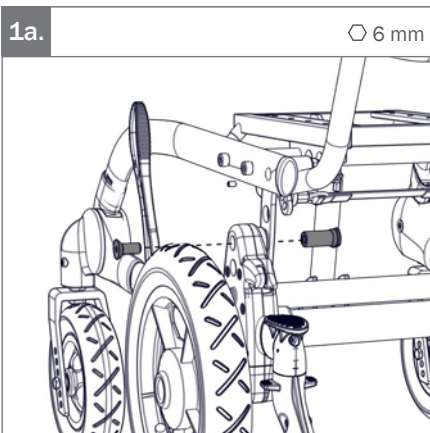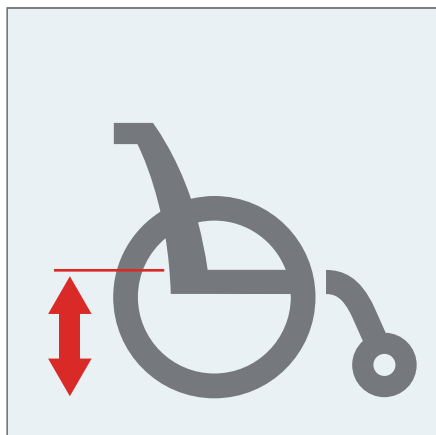
5.


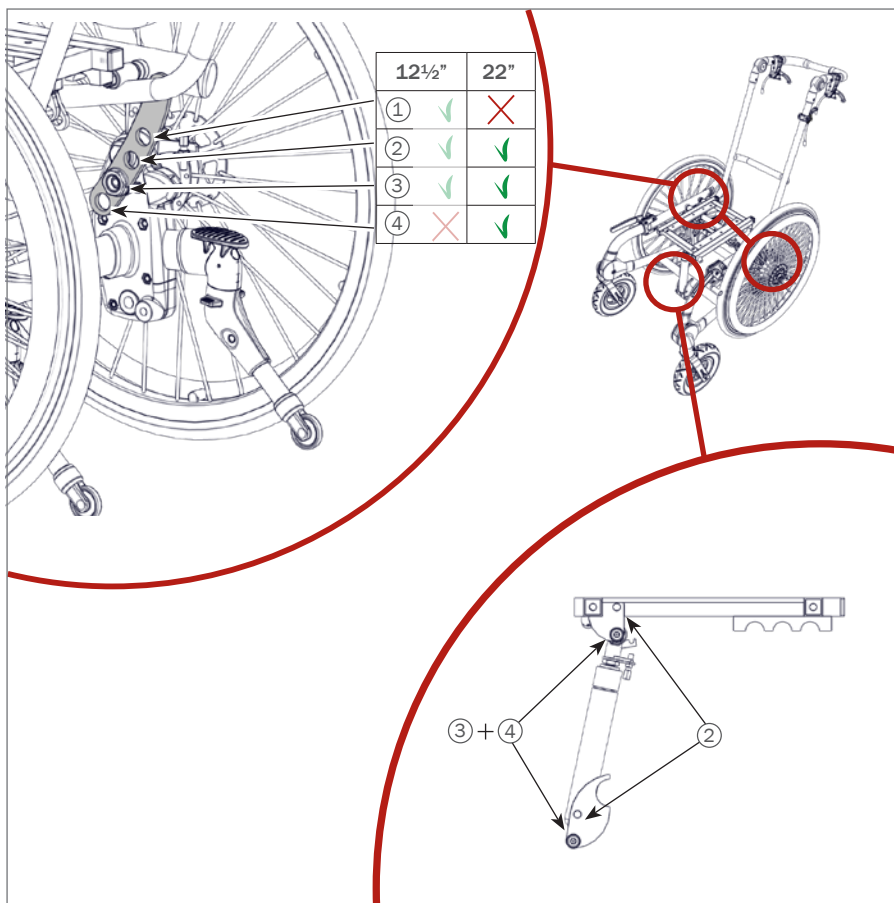
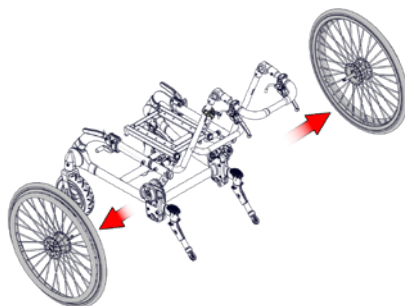
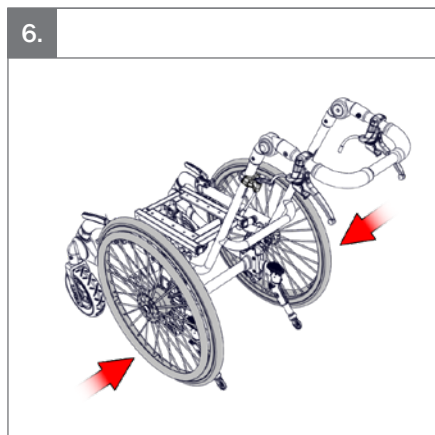
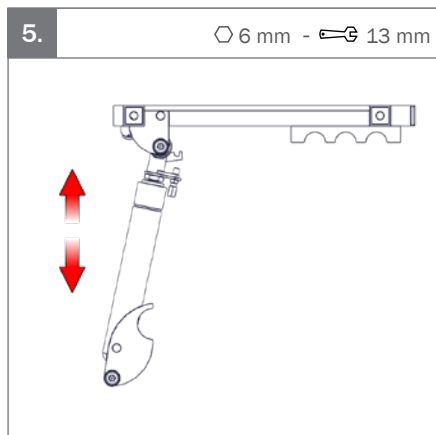
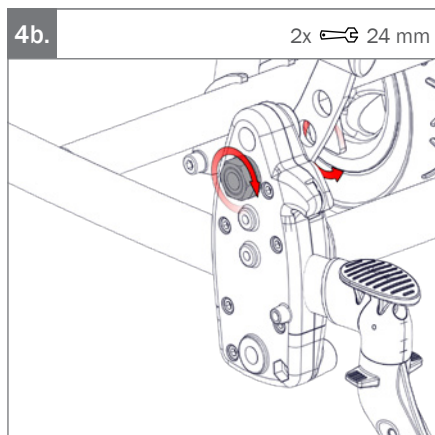
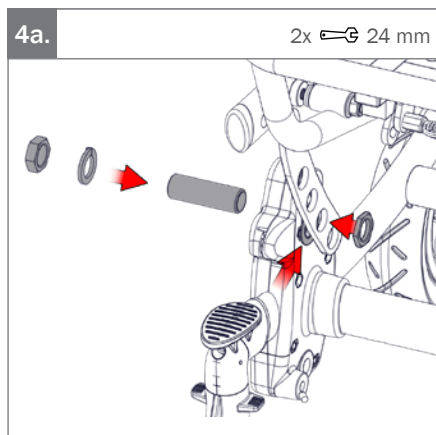
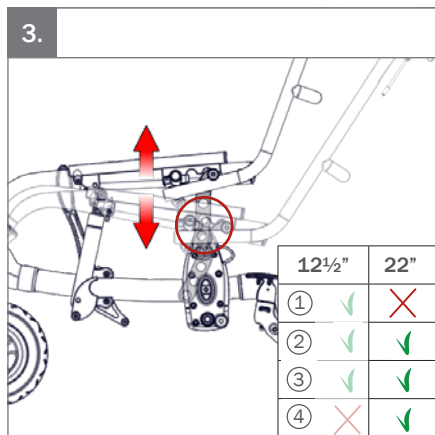
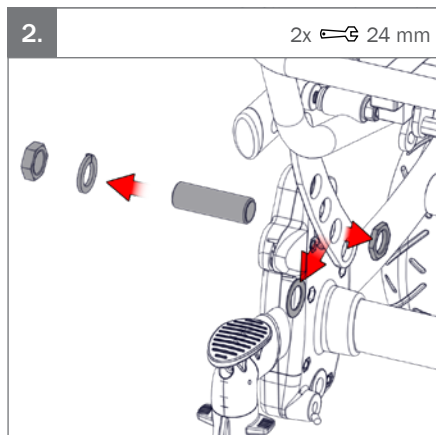
1.


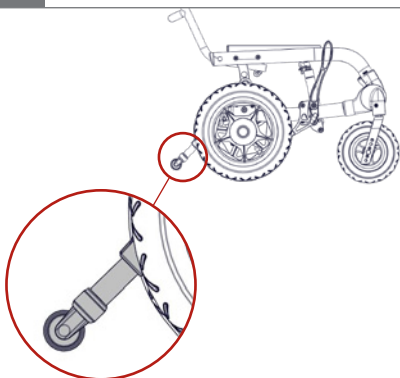
1.


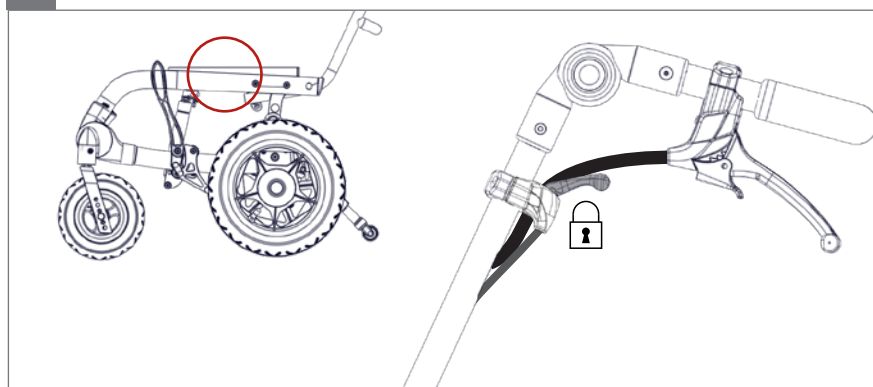
2.

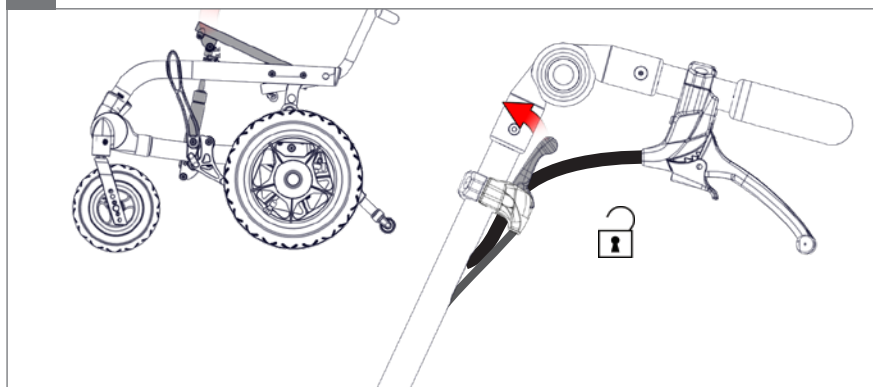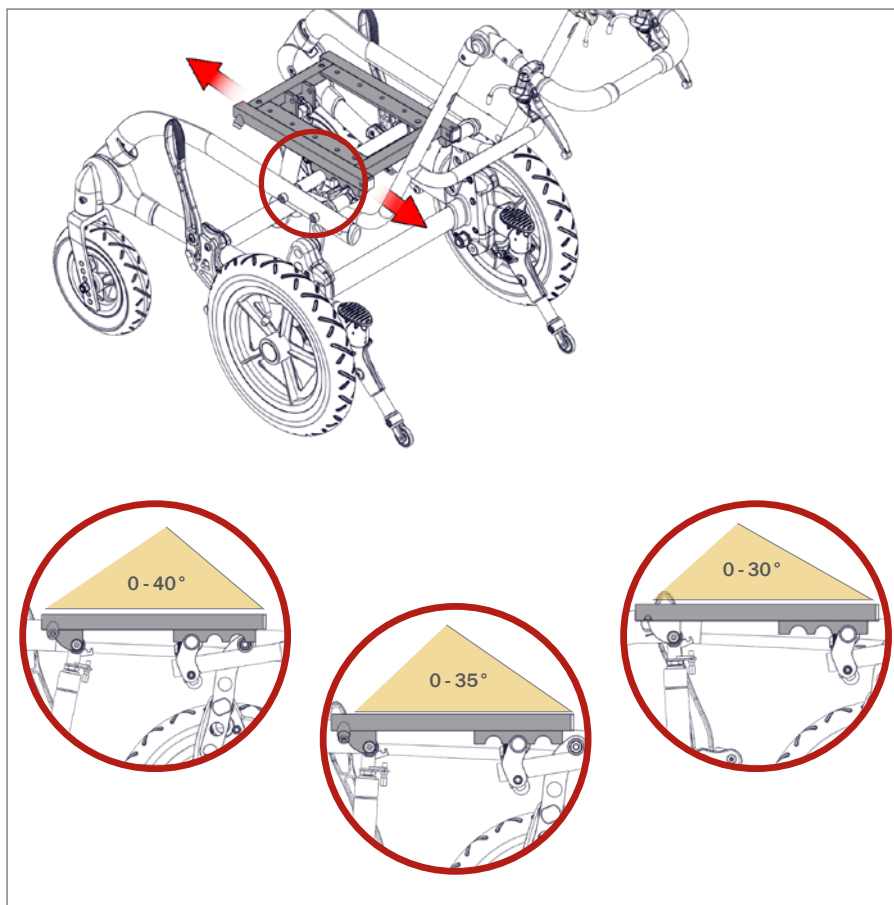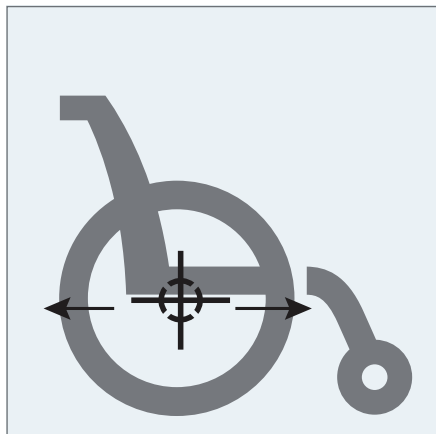3.



1.

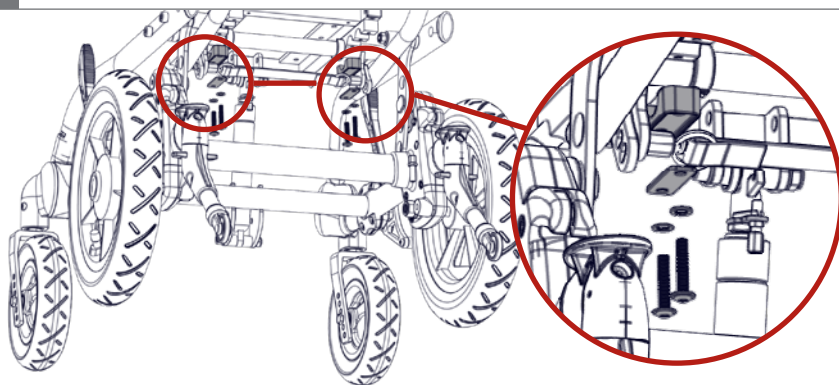
⌀ 4 mm

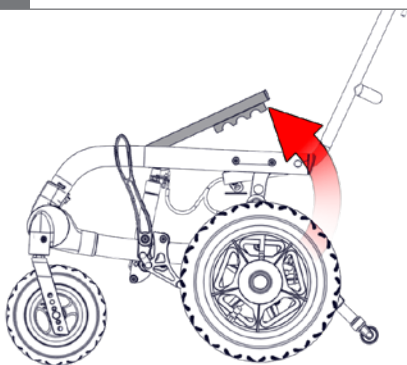
2.



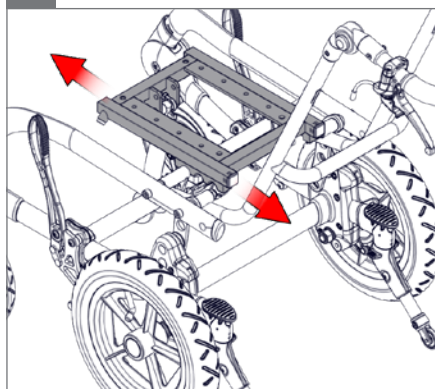
3.

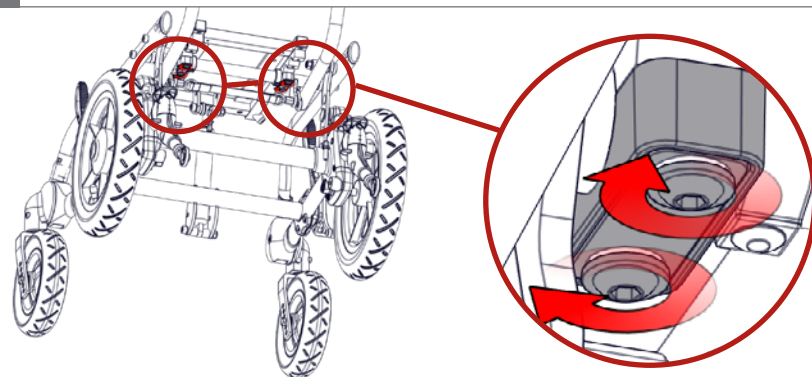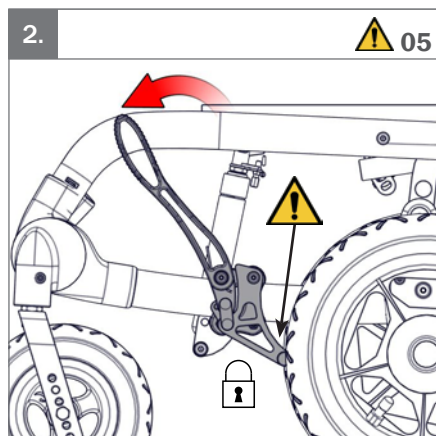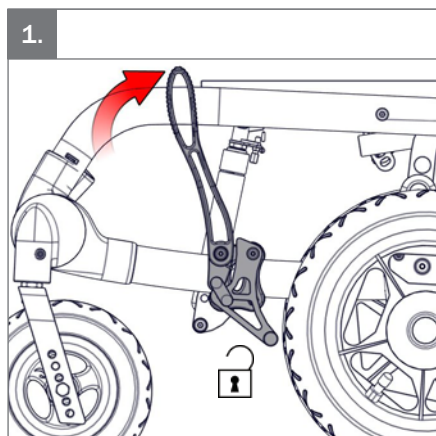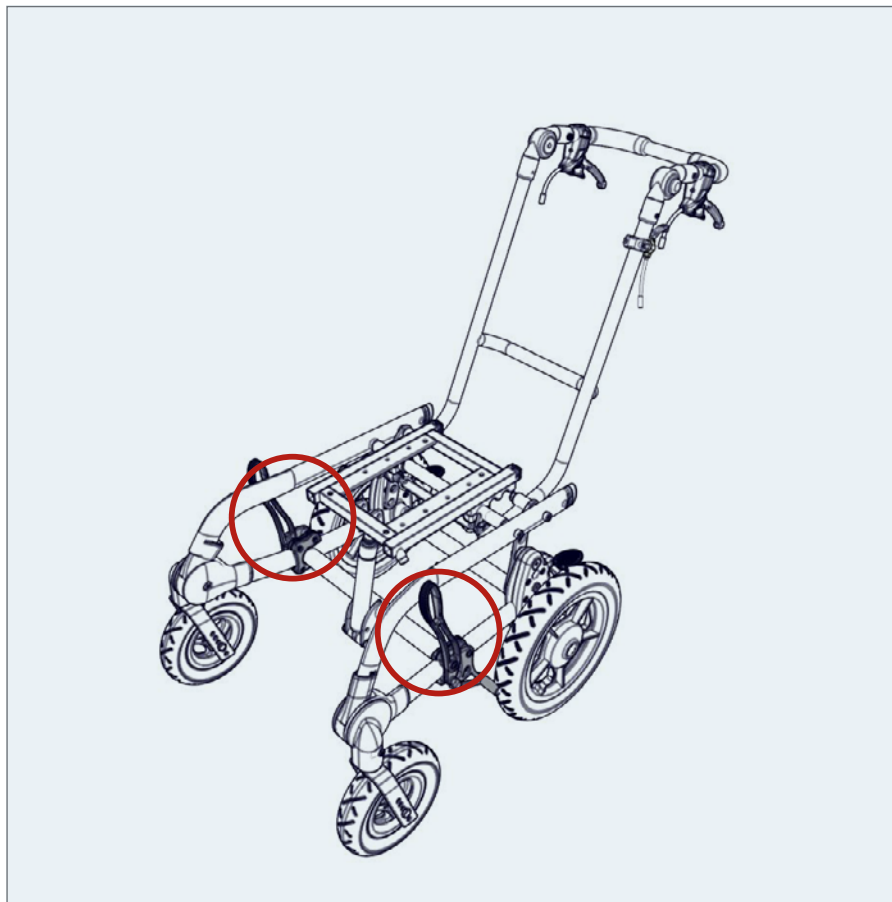
4.


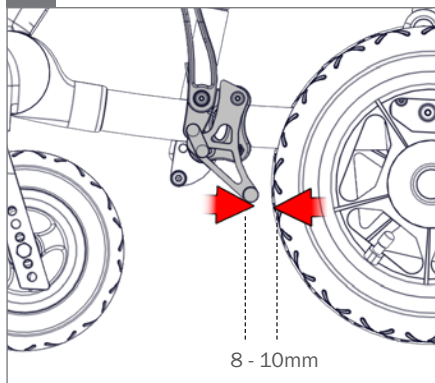
3.

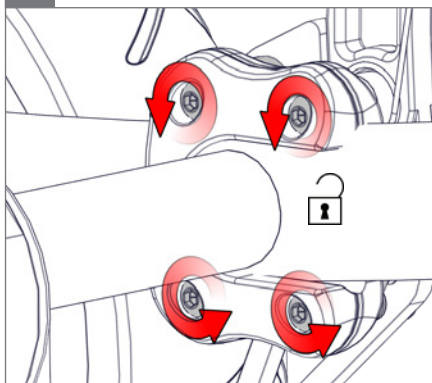4.

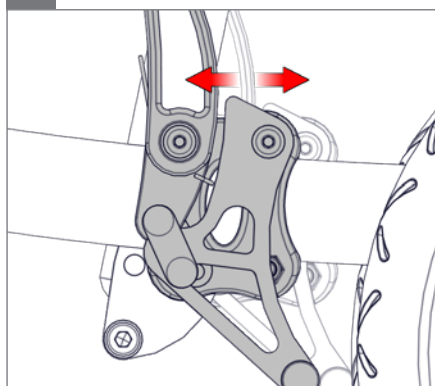
5.



6.

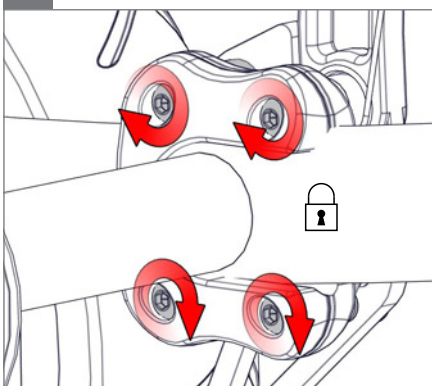○ 5 mm

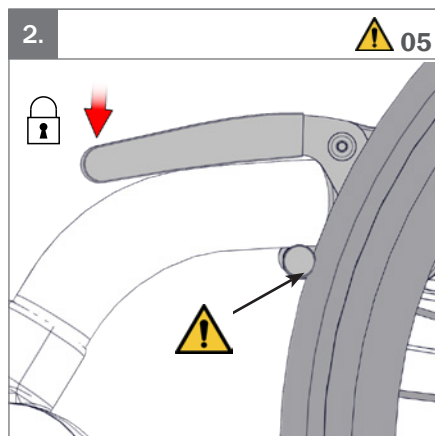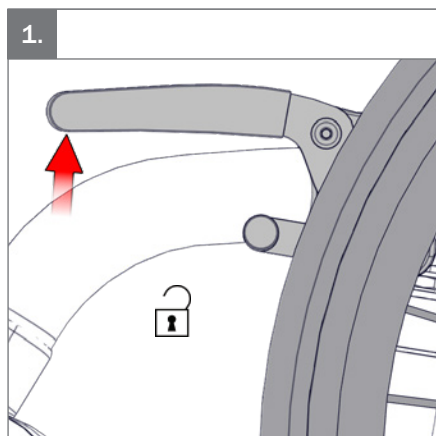
7.


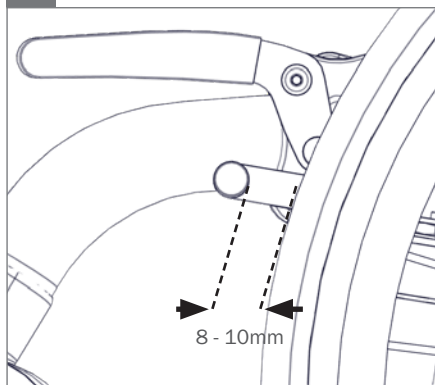
3.

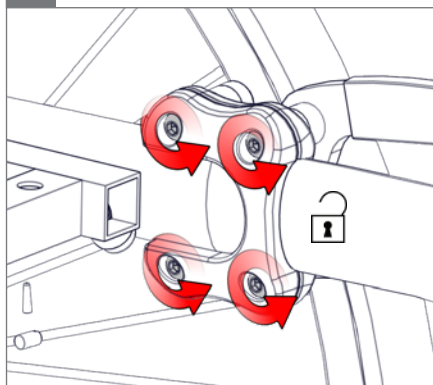4.


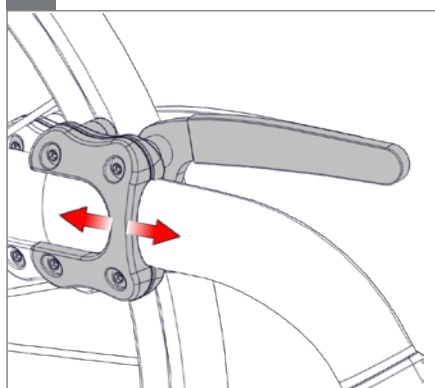
5.


6.

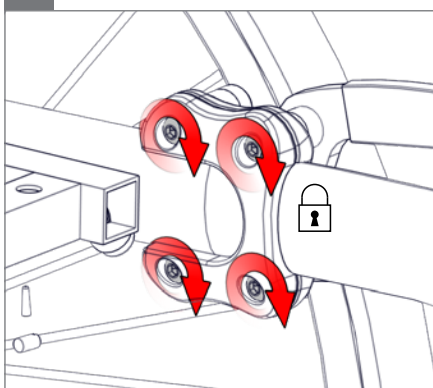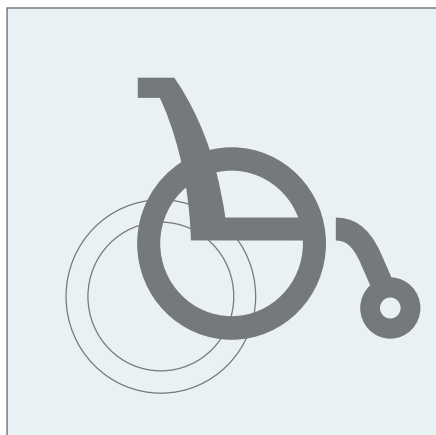○ 5 mm


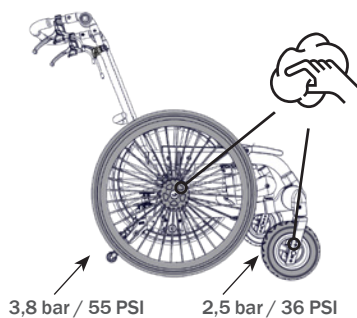
1.





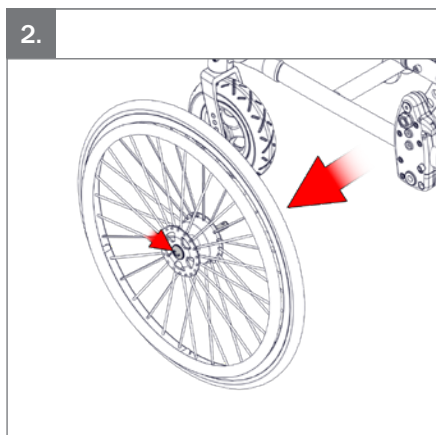
2.

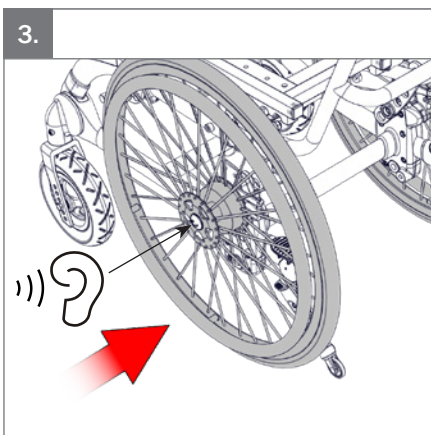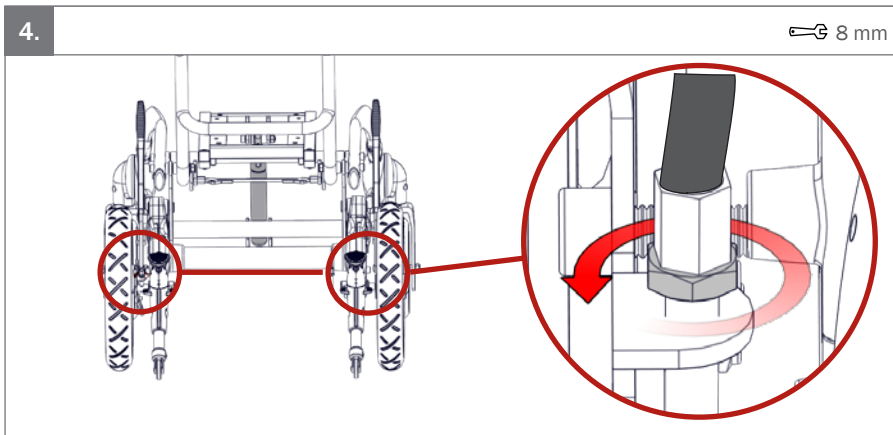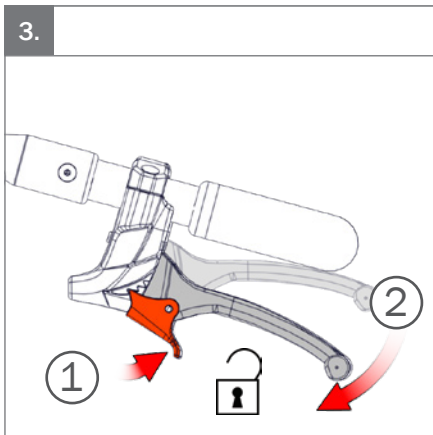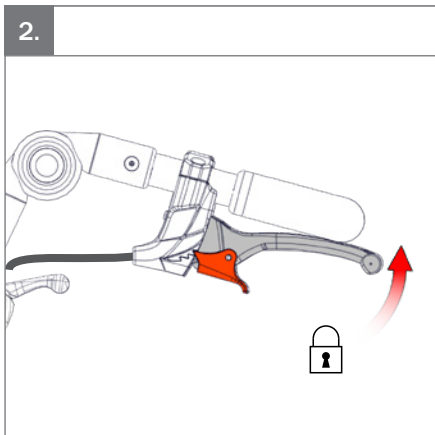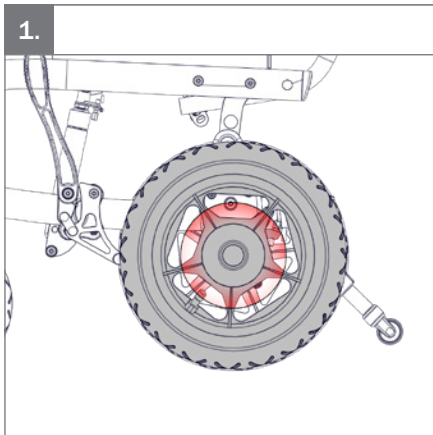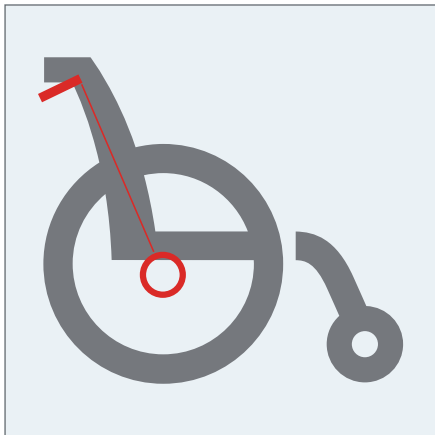
3.

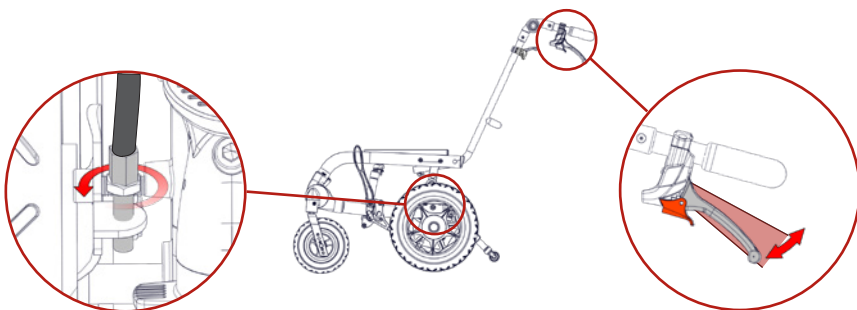
5.





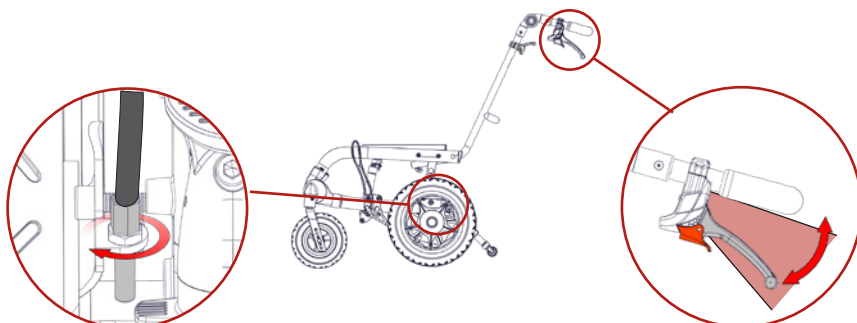
6.

7.

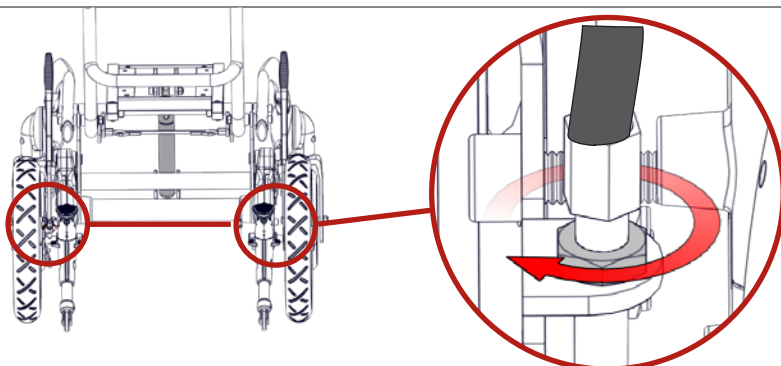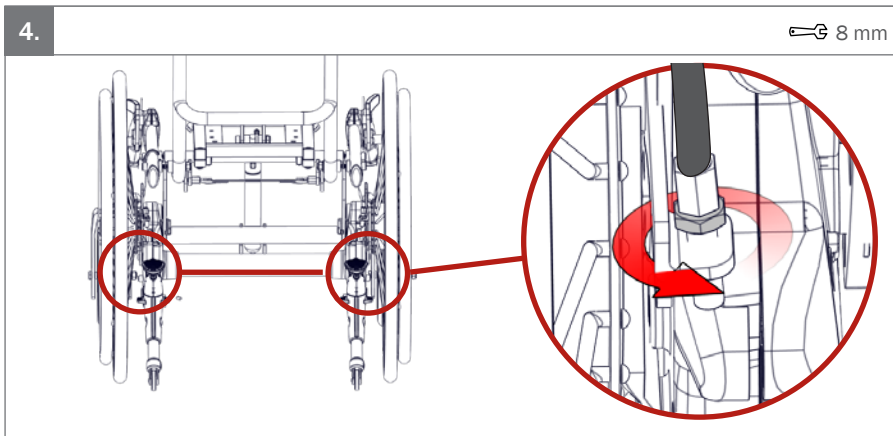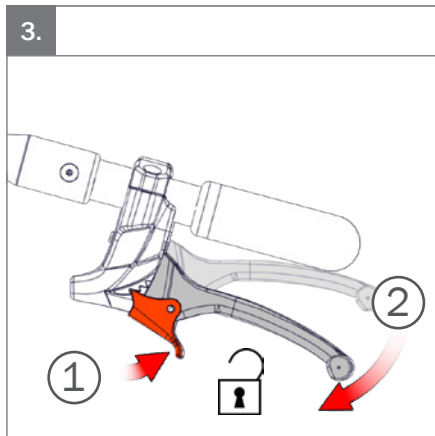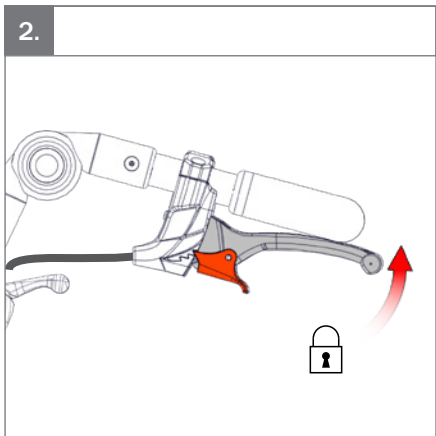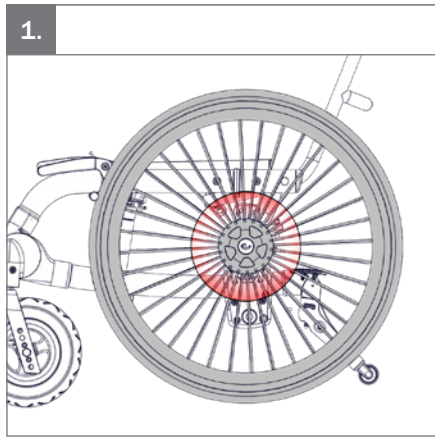
8 mm

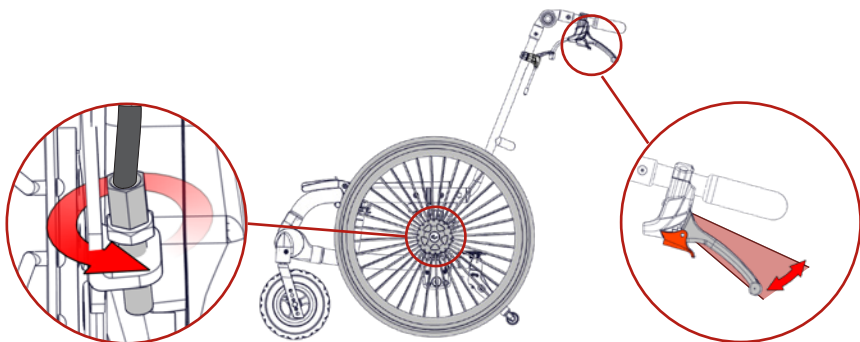
5.






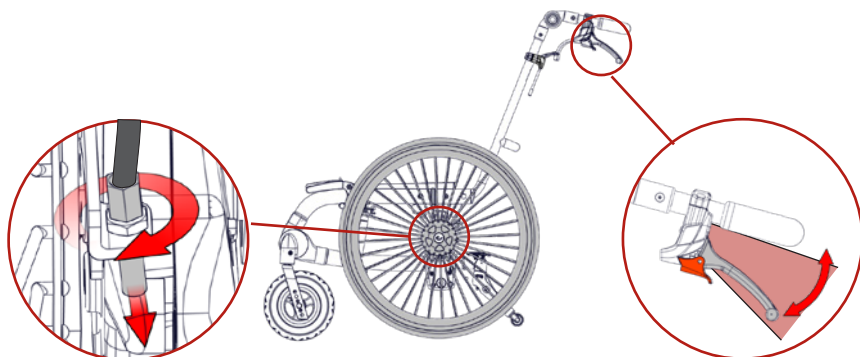
6.

7.

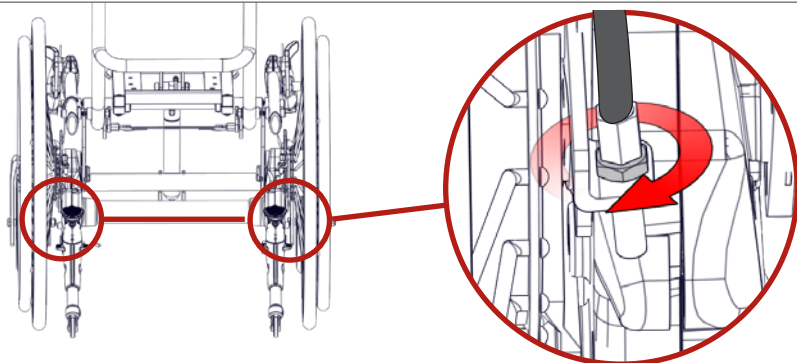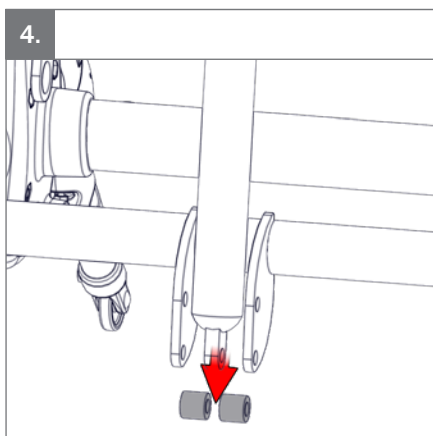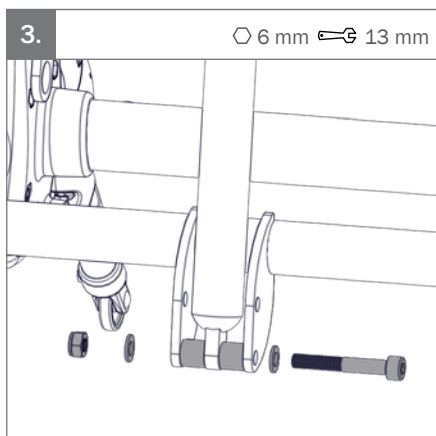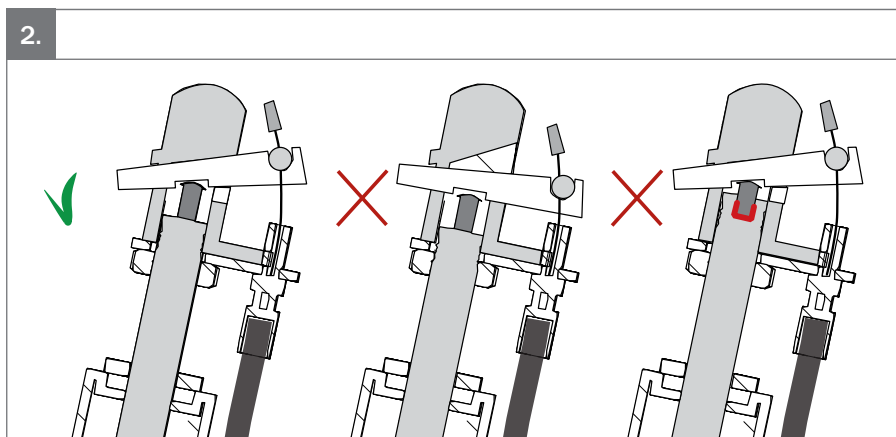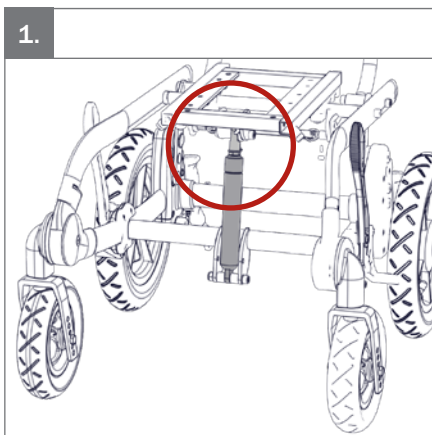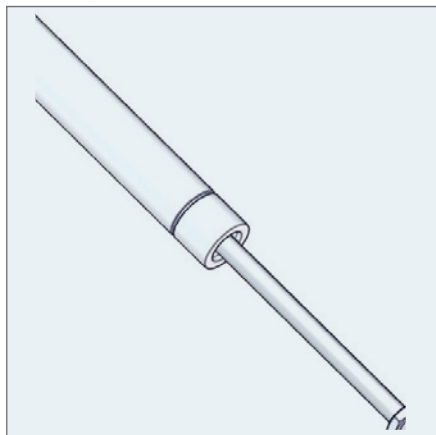8 mm



5.

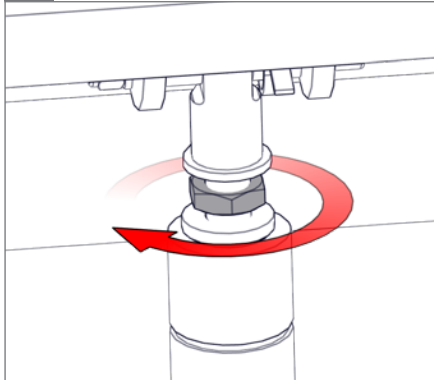
17 mm

6a.

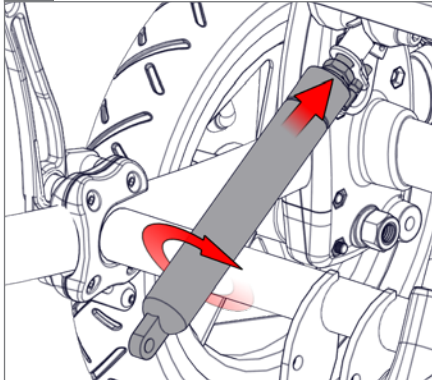
6b.

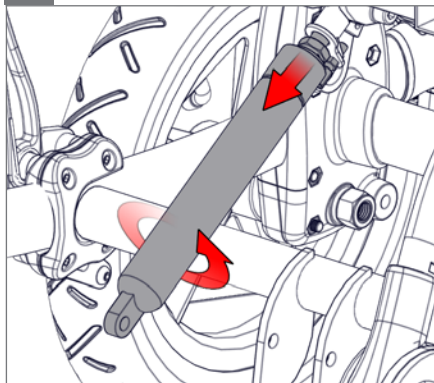
7.

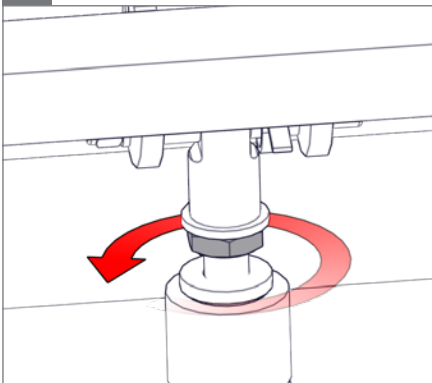17 mm


8.

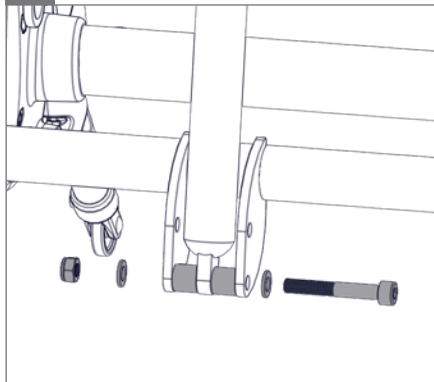6 mm 13 mm

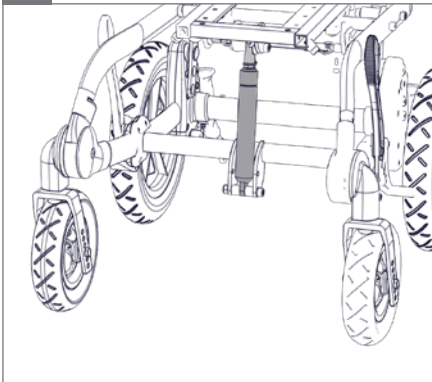
9.


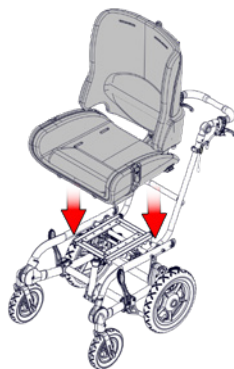
1. Panda Futura 5 & x:panda sz 3 & 4

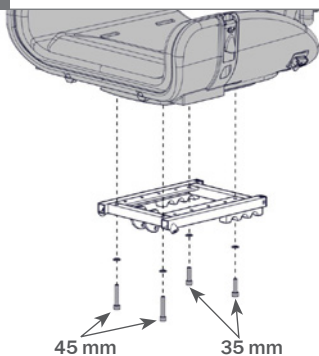2.



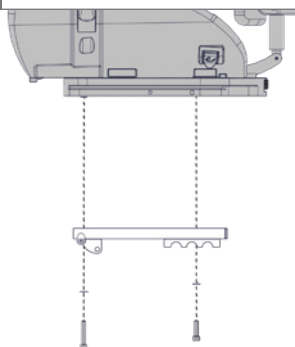
3.


4.

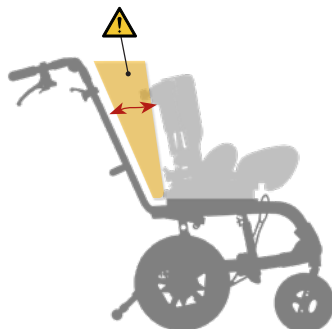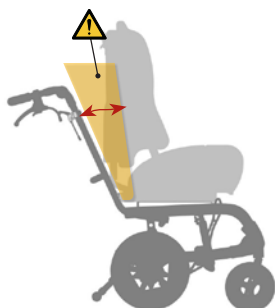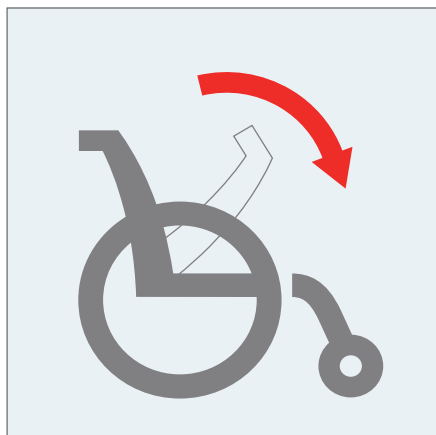 05

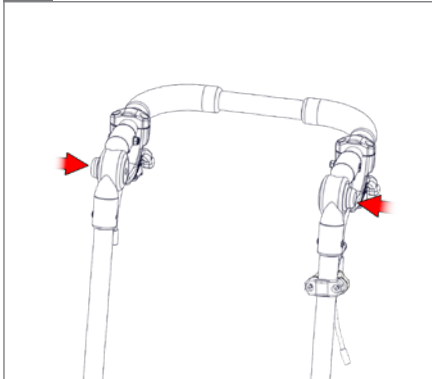
1.


2.

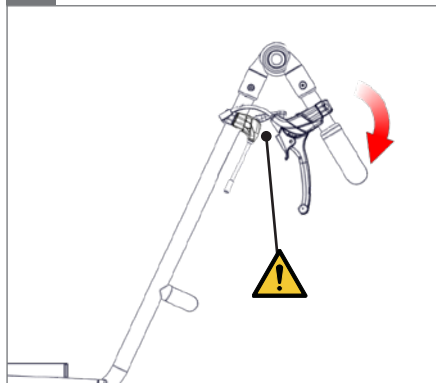⚠ 05




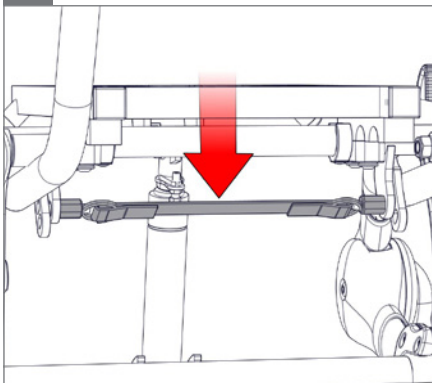
3.



4.

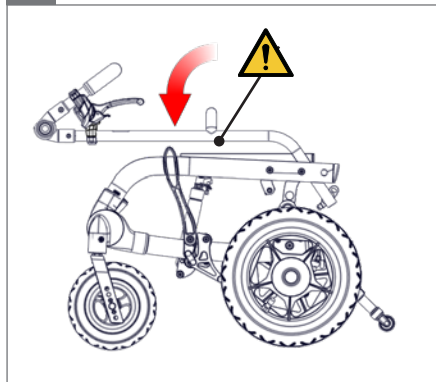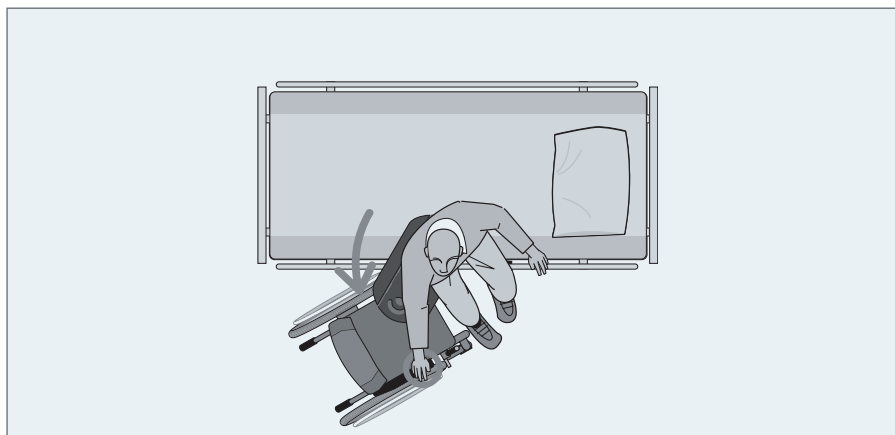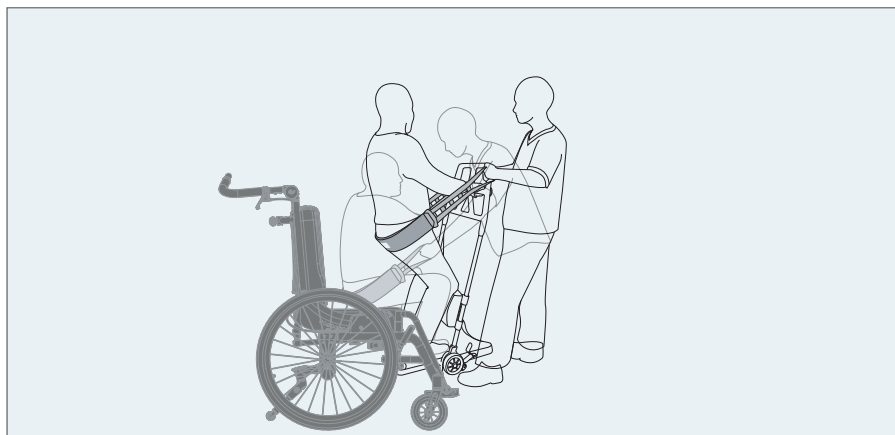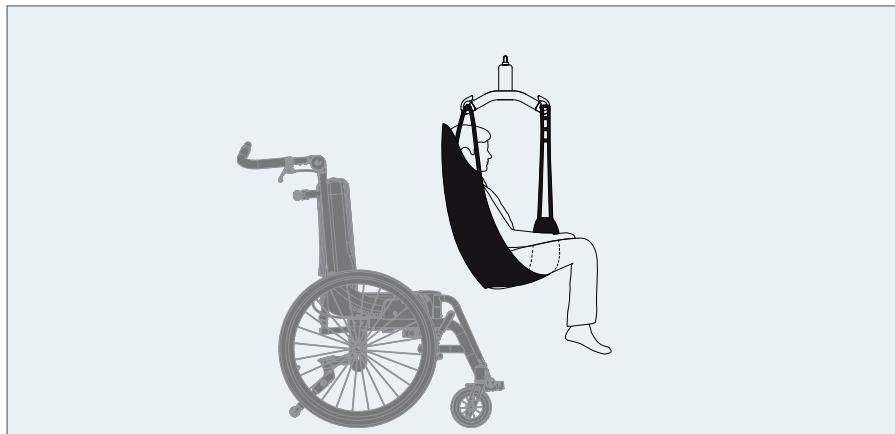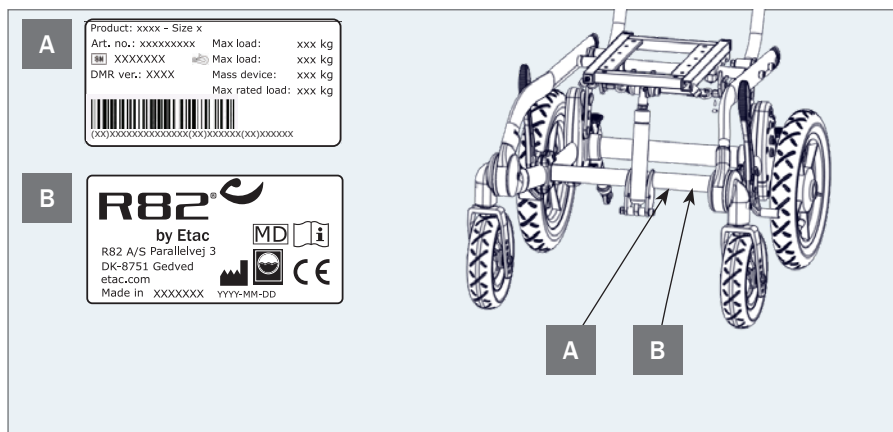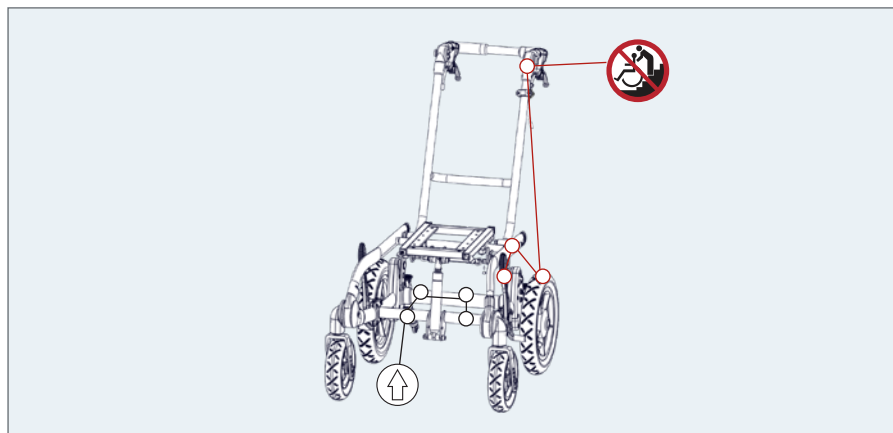⚠ 05

EN: For indoor and outdoor use

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

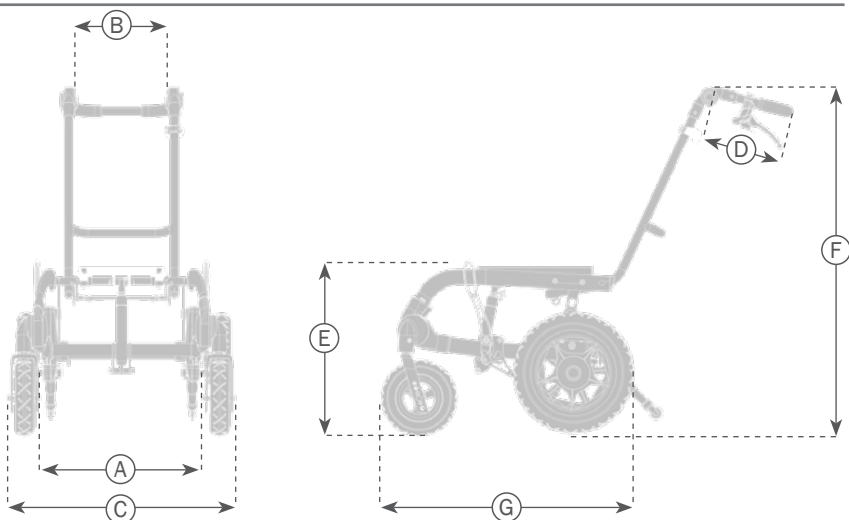
ES: Uso incorrecto

BR: Uso incorreto

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

mm (inch)	1	2	3
 Inside width (A)	340 mm (13½")	390 mm (15¼")	440 mm (17¼")
 Seat surface height at front edge (E) (top/mid/bottom position)	395 / 425/ 455 mm (15½ / 16¾ / 18")	395 / 425/ 455 mm (15½ / 16¾ / 18")	395 / 425/ 455 mm (15½ / 16¾ / 18")
 Back width inside (B)	245 mm (9¾")	295 mm (11½")	345 mm (13½")
 Overall width with 12½" wheels (C)	590 mm (23¼")	640 mm (25¼")	690 mm (27¼")
 Length with 12½" wheels (G)	640 mm (25¼")	640 mm (25¼")	690 mm (27¼")
 Overall height push brace 15° (F)	915 mm (36")	915 mm (36")	915 mm (36")
 Overall height push brace 30° (F)	850 mm (33½")	850 mm (33½")	850 mm (33½")
Hand brace length (D)	225 mm (8¾")	225 mm (8¾")	225 mm (8¾")
 Seat plane angle	0-30° / 35° / 40°	0-30° / 35° / 40°	0-30° / 35° / 40°
 Folded height without back wheels (mid position)	400 mm (15¾")	400 mm (15¾")	400 mm (15¾")
 Folded height with 12½" wheels (mid position)	470 mm (18½")	470 mm (18½")	470 mm (18½")
 Folded length with 12½" wheels	730 mm (28¾")	730 mm (28¾")	730 mm (28¾")
 Footrest to adaptorplate	50 - 340 mm (2-13½")	50 - 340 mm (2-13½")	50 - 340 mm (2-13½")
 Max load	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)
 Max load at transportation	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)
 Mass device with 12½" air wheels and solid front wheels	13 kg (28.6 lb)	13 kg (28.6 lb)	13 kg (28.6 lb)
 Max rated load	115 kg (253.5 lb)	115 kg (253.5 lb)	115 kg (253.5 lb)

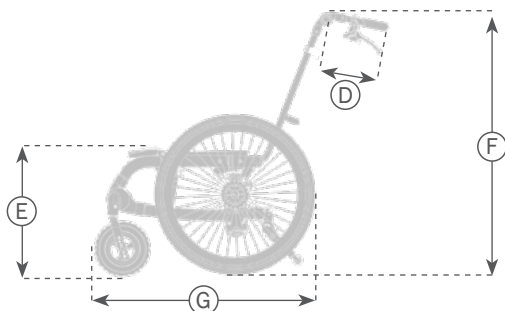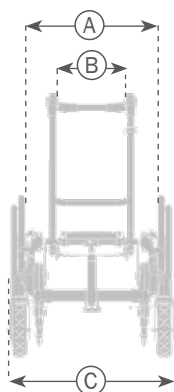
mm (inch)	1	2	3
 Inside width (A)	340 mm (13½")	390 mm (15¼")	440 mm (17¼")
 Seat surface height at front edge (E) (top/mid/bottom position)	395 / 425/ 455 mm (15½ / 16¾ / 18")	395 / 425/ 455 mm (15½ / 16¾ / 18")	395 / 425/ 455 mm (15½ / 16¾ / 18")
 Back width inside (B)	245 mm (9¾")	295 mm (11½")	345 mm (13½")
 Overall width with 22" wheels (C)	590 mm (23¾")	640 mm (25¼")	690 mm (27¼")
 Width between 22" wheels with drum brake (A)	430 mm (17")	480 mm (19")	530 mm (20¾")
 Width between 22" wheels without drum brake	440 mm (17¼")	490 mm (19¼")	540 mm (21¼")
 Length with 22" wheels (G)	750 mm (29½")	750 mm (29½")	800 mm (31½")
 Overall height push brace 15° (F)	915 mm (36")	915 mm (36")	915 mm (36")
 Overall height push brace 30° (F)	850 mm (33½")	850 mm (33½")	850 mm (33½")
 Hand brace length (D)	225 mm (8¾")	225 mm (8¾")	225 mm (8¾")
 Seat plane angle	0-30° / 35° / 40°	0-30° / 35° / 40°	0-30° / 35° / 40°
 Folded height without back wheels (mid position)	400 mm (15¾")	400 mm (15¾")	400 mm (15¾")
 Folded height with 22" back wheel (mid position)	560 mm (22")	560 mm (22")	560 mm (22")
 Folded length with 22" wheels	850 mm (33½")	850 mm (33½")	850 mm (33½")
 Handrim diameter	480 mm (19")	480 mm (19")	480 mm (19")
 Footrest to adaptorplate	50 - 340 mm (2-13½")	50 - 340 mm (2-13½")	50 - 340 mm (2-13½")
Max load	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)
Max load at transportation	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)
Mass device with 22" air wheels and air front wheels	14 kg (30.8 lb)	14 kg (30.8 lb)	14 kg (30.8 lb)
Max rated load	116 kg (255.7 lb)	116 kg (255.7 lb)	116 kg (255.7 lb)

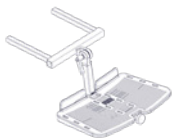9996097118

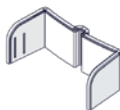
9996097338

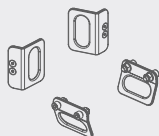9996097135

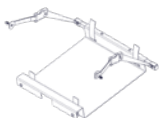9996097339

9996097165

N/A

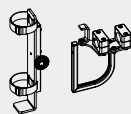
9996097714

N/A

9996097138

N/A

9996097115

N/A

9996097114

N/A

9996097755

N/A

ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this user manual before use and save it for future reference.

Intended purpose

The device is a frame for seating systems for children and adults with disabilities.

Indications

The device is used together with a seating system (i.e. a modular seating system like R82 x:panda or R82 Panda Futura). It can be used by children and adults with disabilities, who lack the ability to maintain a seated position and move around (Within the room or between rooms / outdoors) without an assistive device. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age.

The user is the person sitting in the seat used with the frame. The caregiver is the person operating the frame. Never leave the user unattended in this product. Ensure permanent supervision by a caregiver. The use of the device will be discontinued more times during the day where the user will be lying down, standing or sitting in another assistive device or a conventional product.

For transportation the device is suitable for children and adults in the weight range of 22 kg to specified max user weight for transportation.

Contra indications

There are no known contra indications

Precautions

There are no known precautions

Application areas

The product is for indoor and outdoor use

Enclosed tools:

- 4, 5 and 6 mm Allen key
- 10 and 13 mm spanner

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12183

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Alignment of user weight/mass of the combined seat and base, approval for transportation in vehicles of both products (ISO16840-4 for seat

and ISO7176-19 for base)

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- Be aware of any MAX mark indications before adjustment
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/accessories.
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Always use the product in a moderate pace. Do not run with the product
- Information regarding frame and seat combinations including max. load in combination can be found at: etac.com
- Information about transportation can be found in the document "M1460 Transportation in Motor Vehicles" supplied with the product
- Information about roadability can be found in the document "Roadability" supplied with the product
- Activate the anti-tips before angling the seat and back
- Deactivate the anti-tips when forcing kerbs higher than 100 mm
- We recommend you notify the user before angling the seat and back
- Ensure the wheels are fully-functional and secure before use
- The correct tyre pressure is printed on the tyre casing and listed in the section for tyre technical data.
- In case of a tyre puncture do not use the product until remedied
- Unoccupied device is suitable for land and air transport
- Attaching additional loads can adversely affect stability. Position of points intended to carry additional loads are specified in the relevant Mounting Guide referred to under section for accessories
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Inspect the product and all of its accessories and replace any worn out parts before use
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1, EN 1021-2 and ISO 7176-16
- The product might lose its flame resistant characteristics if using other than original R82: Cushions
- The product is approved for transportation in vehicles according to ISO 7176-19. More information can be found in the document "M1460 Transportation in Motor Vehicles" supplied with the product.
- In accordance with ISO 7176-19 Annex D, the product is evaluated regarding the ability to accommodate vehicle-anchored pelvic and shoulder belts. The device receives an overall score of "Good"

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Removable parts without use of tool: wheels and foot support. The removal will have adverse effect on the device
- Ensure the stability of the product is maintained before placing the user in it
- Activate the brakes before placing the user into the product

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device.

Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded. The device may be reissued within its lifetime.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
 - If a seat adapter is mounted, check that the red handle for locking works and that the safety lock "clicks into position when placing a seat into the adapter
 - Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required
 - Quick release wheels are removable/replaceable
 - Anti-tips can be fixed in working and resting position
 - Tire pressure
 - Spoke guards is without cracks and secured to the wheel with the clips mounted on each guard
 - The Pushbrace can be folded over the frame
 - Before storage: Always deactivate wheel locks/brakes
 - Nuts and bolts on the product are tightened
 - Oil the swing-away parts
 - Product for cracks or signs for worn-out parts
 - All labels on the product are intact
- Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to:

Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use.

Carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Aluminium
- Plastic
- Rubber
- Steel

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

Safety product

Due to variant and settings the wheelchair may become larger than the recommended dimensions.

Compliance with recommended overall width and overall length ensures unhindered use of emergency escape routes. (See table 1)

Table 1

Recommended dimensions	Ready for use (mm)
Overall length – maximum	1200
Overall width – maximum	700
Overall height – maximum	1200
Turning diameter – maximum	1000
Pivot width – maximum	1300
Ground clearance - minimum	30

Product safety

Due to variant and settings the wheelchair may not meet the limits for wheelchairs transportable by train. Measures can be taken to verify whether the specific product in question meets the limits. (See table 2)

Table 2

Wheelchairs transportable by train - limits	
Overall length – maximum	1200 mm + 50 mm for feet
Overall width – maximum	700 mm + 50 mm on each side for hands
Overall height – maximum	1375 mm (including 95th percentile male occupant)
Turning diameter – maximum	1500 mm
Smallest wheels	Accommodate a gap of 75 mm horizontal and 50 mm vertical
Weight – maximum	200 kg for manual wheelchair and occupant (including baggage)
Obstacle height that can be overcome – maximum	50 mm
Ground clearance - minimum	60 mm under foot rest with upward slope angle of 10° on top for going forward
Safe slope on which the wheelchair remain stable - maximum	6° dynamic stability in all directions 9° static stability in all directions, including with brake applied

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R82 A/S