

Flamingo High-low (bath frame)

User information appropriate for use by people with visual, reading or cognitive disability	The latest version of all instructions is always available on the R82 website support.R82.org and can be printed in larger sizes														
Information available in pictogram	The instructions are made as pictograms														
Intended use and intended environment including standard options	The High-low Bath Frame is a toilet/bathing frame with user friendly angle and height adjustment. The High-low Bath Frame is designed for Flamingo seats and can be placed over the toilet or used freestanding with a pan. The frame comes in four sizes and is only for indoor use but can in an emergency be taken outside.														
Maintenance instructions	<p>Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.</p> <table border="1" data-bbox="593 855 1347 1753"> <thead> <tr> <th>Frequency</th> <th>Description</th> </tr> </thead> <tbody> <tr> <td>1st day</td> <td> <ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Store the enclosed tools on/at the product </td> </tr> <tr> <td>After every use</td> <td> <ul style="list-style-type: none"> - Wipe the product dry </td> </tr> <tr> <td>Daily</td> <td> <ul style="list-style-type: none"> - Disinfect the product. please check the disinfection section in this User Guide for more information - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fitted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any MAX mark indications before adjustment - Check that all castors are moving freely and that all wheel locks/brakes are easily activated </td> </tr> <tr> <td>Weekly</td> <td> <ul style="list-style-type: none"> - Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use </td> </tr> <tr> <td>Monthly</td> <td> <ul style="list-style-type: none"> - All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures - Oil the swing-away parts. We recommend you use a professional lubricant system </td> </tr> <tr> <td>Every 6 months</td> <td> <ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty </td> </tr> </tbody> </table>	Frequency	Description	1 st day	<ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Store the enclosed tools on/at the product 	After every use	<ul style="list-style-type: none"> - Wipe the product dry 	Daily	<ul style="list-style-type: none"> - Disinfect the product. please check the disinfection section in this User Guide for more information - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fitted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any MAX mark indications before adjustment - Check that all castors are moving freely and that all wheel locks/brakes are easily activated 	Weekly	<ul style="list-style-type: none"> - Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use 	Monthly	<ul style="list-style-type: none"> - All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures - Oil the swing-away parts. We recommend you use a professional lubricant system 	Every 6 months	<ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty
Frequency	Description														
1 st day	<ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Store the enclosed tools on/at the product 														
After every use	<ul style="list-style-type: none"> - Wipe the product dry 														
Daily	<ul style="list-style-type: none"> - Disinfect the product. please check the disinfection section in this User Guide for more information - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fitted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any MAX mark indications before adjustment - Check that all castors are moving freely and that all wheel locks/brakes are easily activated 														
Weekly	<ul style="list-style-type: none"> - Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use 														
Monthly	<ul style="list-style-type: none"> - All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures - Oil the swing-away parts. We recommend you use a professional lubricant system 														
Every 6 months	<ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty 														

Cleaning description

This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

Before washing

Flamingo High-low (bath frame)

Before washing remove the following items and clean them separately:
Accessories and Seat

Disinfection description	The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.
Dimensions (Incl. max load)	http://www.R82.org - Select product or click here
Mass	N/A
Used in combination with other products	N/A
Warning about dangerous combinations	The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.
List of accessories, detachable parts and materials	http://www.R82.org - Select product or click here
Programmable controller information	N/A
Operator control adjustments	Manual
Folding and dismantling	N/A
Transport instructions (unoccupied)	Land and/or air
Sound power level	N/A
Turning diameter	NA
Forces on the wall	N/A
Electromagnetic environment warning	N/A
Electromagnetic influence warning	N/A
Life-time	The lifetime of the product in normal use is 5 years